















and effect-efficacy

This is the dental instrument with s a one file glide path solution. Made with NiTi Alloy and uniquely shaped with a variable taper.

Instructions for use

According to its intended use, choose the most appropriate Perfect PX One and use it for glide path root canal.

Precautions for use

4) Irrigate.

1) Prepare straightline access to canal orifice. 2) Explore the canal up to a 010 K-file. 3) Establish working length with an apex locator. alone or in combination with a radiograph.Confirm patency with a 010 Kfile and verify a smooth, reproducible glide path.

5) Use PX One in one or more passes until the full working length is reached.

6) Irrigate the expanded glide path. 7) Reconfirm the working length with a 010 K-file before shaping the canal with a rotary orreciprocating file (such as MTF PLUS GOLD Rotary File).



21,25,31mm



























