

IBP™


Implant Bite Post



The **Implant Bite Post** has recently been introduced to implant dentistry. This important innovation will serve as surrogate preparations, allowing the clinician to capture accurate occlusal records every time. **These Implant Bite Posts** are available in most platforms and come in two different lengths.



The upper left image shows posts placed in the mouth with light finger pressure only. The lower right image shows a bite being taken. The bite posts are sent to the lab with the complete case.

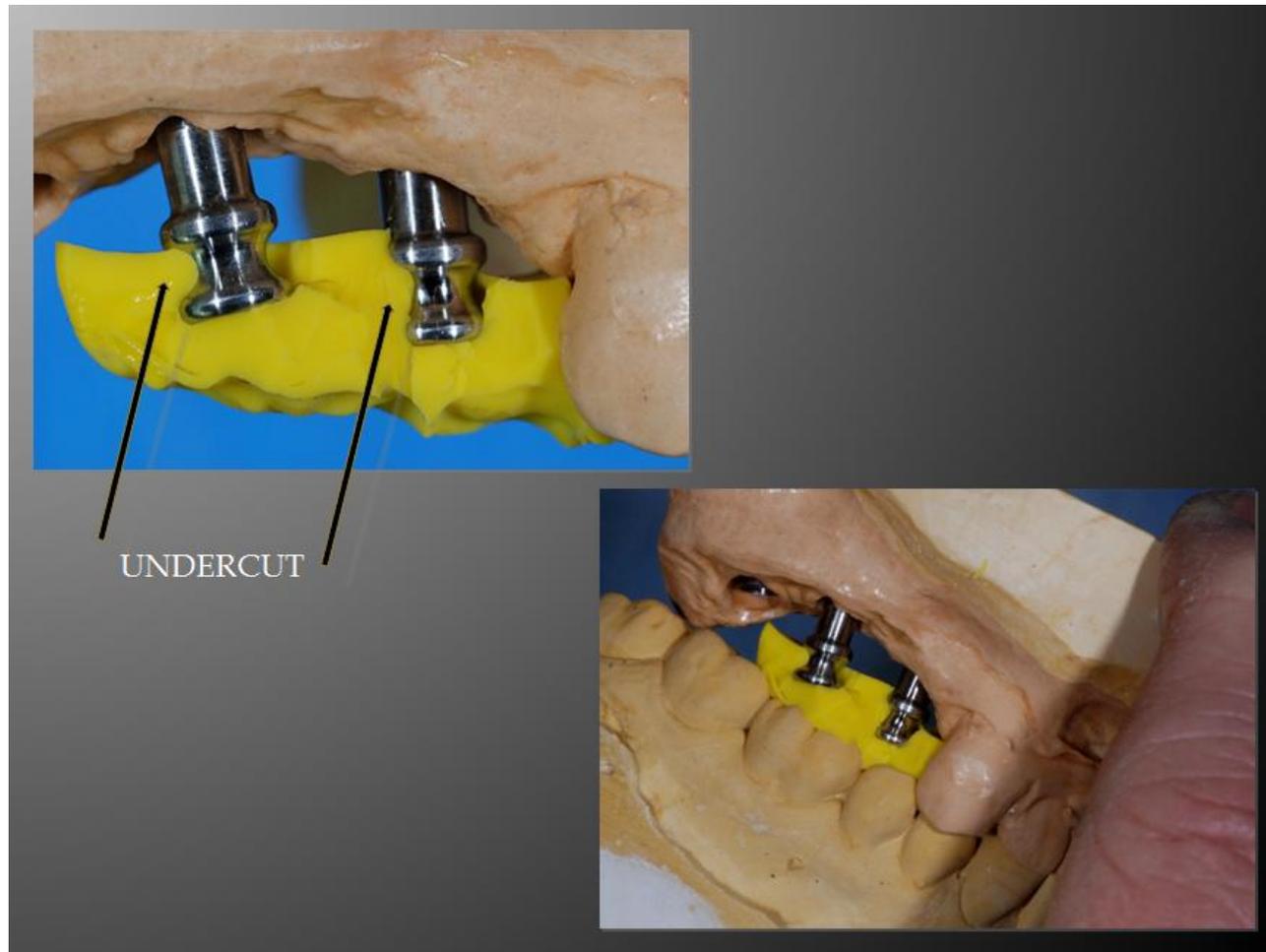

The upper left image shows the **Implant Bite Posts** placed on the model with light finger pressure only. The bite was trimmed to remove tissue contact and to allow facial view. The lower right image shows the models placed together. They are checked for accuracy and the casts are then mounted.



Here the case was seated and delivered to the patient with minimal occlusal adjustments.