



Blue Sky Plan®

USER MANUAL

 mdi Europa GmbH
Langenhagener Straße 71 D-30855 Langenhagen, Germany
 0297

Contents

1 Introduction	6
1.1 Indications For Use	6
1.2 Warnings	7
2 Installing Blue Sky Plan	9
2.1 Installation process	9
2.2 Hardware requirements	11
2.3 Operating system	11
3 Blue Sky Launcher	12
3.1 Modules and planning types	12
3.2 Project Setup	13
3.3 Project Settings	14
3.4 Data Import	17
3.5 Catalog management	18
3.6 Project management	19
4 Licenses	21
4.1 The main characteristics of licenses	21
4.2 License registration	21
4.3 BlueSkyBio Digital account options	24
4.4 Licenses in Preferences	24
4.5 Shared Licenses	25
4.6 Purchasing export	28
5 Loading the patient's CT or CBCT scan into Blue Sky Plan	31
5.1 Selecting the DICOM images location	31
5.2 Selecting the series	32
5.3 Enhancing performance	32
5.4 Scan alignment and defining the range of slices	32
5.5 Import additional surfaces	33
5.6 Positioning the panoramic curve	34
6 Software interface	36
6.1 Screens and views	36
6.2 Multi-planer screen	44
6.3 3D Mouse Controls	46
7 CT Dicom Viewer	50
8 Placing implants and pins	53
8.1 Adding an implant or pin	53
8.2 Positioning and manipulating the implant	62
8.3 Further operations on the implant	62
8.4 Side panel	64
8.5 Dialog with the parameters of the selected implant	64
8.6 Validating implants	65
8.7 Ordering implants	66
9 Tools	67
9.1 Toolbar	67
9.2 Opening a project	68

9.3 Saving a project	68
9.4 Closing a project	68
9.5 Saving images in JPG format	68
9.6 Saving screenshots and generating a Drill Report	69
9.7 Modifying window and level	71
9.8 Zooming.....	72
9.9 Moving the image.....	72
9.10 Moving objects.....	72
9.11 Measurements and overlays	73
9.12 Linear measurement.....	74
9.13 Angular measurement.....	74
9.14 Density measurement.....	74
9.15 Measurement correction and number position	75
9.16 Deleting a measurement.....	75
9.17 Overlays	75
9.18 Panoramic	75
9.19 Show Dimensions	78
10 Importing the Scan Appliance	80
10.1 Gutta percha markers coincide	81
10.2 Gutta percha markers do not coincide	82
11 STL model import (planning with CT scan)	84
12 Planning without CT scan	87
13 STL model adjustment.....	91
14 Scan Body Registration.....	92
15 Virtual teeth	95
15.1 Inserting crowns.....	95
15.2 Positioning the crown and manipulating it	96
15.3 Adding the AI crown	97
15.4 Aligning an implant and further operations	98
15.5 Context menu	99
16 Tooth and jaw segmentation	101
16.1 Automatic Tooth Segmentation	101
16.2 Virtual Tooth Segmentation	105
16.3 Automatic Jaw Segmentation.....	106
16.4 Airways	109
17 Surgical guides	111
17.1 Guide fabrication based on scan appliance	111
17.2 Guide fabrication based on scanned surface models.....	113
17.3 Removing undercuts	116
17.4 Brush tools.....	117
17.5 Text tool	118
18 Endodontic Mode	120
19 Data export.....	123
20 External softwares	124
20.1 Medit import.....	124
21 Articulation pins.....	126
22 Coordinates export	128
23 Boolean Operations	130
23.1 Union	130

23.2 Difference	131
23.3 Intersection	131
24 Blue Sky Plan Wizard.....	133
24.1 Load DICOM data	134
24.2 Adding a mandibular canal.....	136
24.3 Model upload and preparation	138
24.4 Model aligning.....	138
24.5 Crowns and implants	140
24.6 Surgical Guide	144
24.7 Saving and ordering.....	146
25 Orthodontics Module	147
25.1 Loading the Input STL Model	148
25.2 Buccal Bite Registration	149
25.3 Mandible articulation	150
25.4 Annotating Teeth and Jaw Alignment	151
25.5 Two Jaw Functionality	154
25.6 Teeth Segmentation	154
25.7 Model Trimming	156
25.8 Teeth Moving	157
25.9 Generating Intermediate Positions	163
25.10 Orthodontic Aligners	165
25.11 Indirect Bonding Tray - Align Brackets to Teeth.....	170
25.12 Indirect Bonding Tray - Align Brackets to Wire	172
25.13 Generating and Exporting STL Models	173
25.14 Show Dimensions	177
25.15 Quick Instructions for Using the Computerised Orthodontic Bracket	
System.....	178
26 Denture module.....	181
26.1 Loading the patient's STL model	181
26.2 Adding virtual teeth.....	181
26.3 Mark visible teeth for teeth chain	182
26.4 Clear Teeth Chain	182
26.5 Show / Hide teeth chain	182
26.6 Move Entire Teeth Chain.....	183
26.7 Manipulate model.....	183
26.8 Create denture	183
26.9 Step 1 – Undercuts and insert direction	184
26.10 Step 2 – Select denture base	185
26.11 Step 3 – Shape gingiva	185
26.12 Step 4 – Denture.....	186
26.13 Export	187
26.14 Teeth Surfaces panel	187
26.15 Opening saved project	188
26.16 Screenshots.....	190
27 Crown and Bridge module.....	191
27.1 Loading the patient's STL model	191
27.2 Adding virtual teeth.....	191
27.3 Restoration Design panel.....	192
27.4 Restoration type selection – Crown on a Tibase	192
27.5 Restoration type selection – Conventional crown	196

27.6 Create Bridge.....	201
27.7 Teeth Surfaces panel	203
27.8 Teeth Edit panel	203
27.9 Parts panel	204
27.10 Screenshots panel	205
27.11 CAM Export.....	206
27.12 Crown and Bridge Preferences	207
28 Splint Module	208
28.1 Loading the patient's STL model	209
28.2 Align to Grid	209
28.3 Buccal Bite Registration	210
28.4 Jaw Articulation	211
28.5 Remove Undercuts	211
28.6 Add Anterior Deprogrammers	213
28.7 Draw Splint Curve	215
28.8 Set Raised Areas	216
28.9 Edit Splint Model	217
28.10 Export and Upload to BioBigBox	218
28.11 Keyboard Shortcuts.....	219
28.12 Manipulation Shortcuts	219
28.13 Visibility Panel.....	219
28.14 Editing Mode	221
28.15 Edit Model.....	221
28.16 Notes and Measurements	222
28.17 Create a screenshot	223
28.18 Jaw Movement.....	223
28.19 Show dimensions.....	223
29 Preferences	225
29.1 General	225
29.2 Customization	226
29.3 Project	226
29.4 Shortcuts	226
29.5 Advanced	226
29.6 Surgical Guide	227
29.7 BSB Direct Cut Drills	227
29.8 Orthodontics and Tooth Movement Limits	227
29.9 Crown and Bridge	227
30 Automatic updates	228
31 Footnotes	230
31.1 Blue Sky Bio Aligners Prescribed Instruction For Use	230
31.2 Computerised Orthodontic Bracket System Instructions for Use	231
31.3 Blue Sky Bio Aligner G - Patient Instruction	232
31.4 Blue Sky Bio Aligner G - User Instruction	234


1 Introduction

Blue Sky Plan® is Windows computer software intended for 3D virtual surgical implant placement, giving clinicians a more accurate way to treat patients. Blue Sky Plan® is precise and can help the clinician determine the ideal position for implant placement for the best esthetic and functional results. Blue Sky Plan® can be used for diagnostic purposes and can aid in the fabrication of surgical guides for computer-guided surgery. Using Blue Sky Plan® is the way to give your patients the best care available.

Blue Sky Plan converts a Computer Tomography (CT) or Cone Beam Computer Tomography scan into a format that allows the dentist to assess the anatomic topography of the maxilla and mandible as well as the location of important structures. The patient's CT or CBCT images will be stored in DICOM format from an Imaging Center. The software converts these images and makes it possible to view the patient's anatomic structures. These data can be used for accurate pre-surgical treatment planning.

The dentist can use the program to:

- carry out pre-surgical planning
- help fabricate an accurate surgical guide
- decrease the risks of surgical complications
- create realistic simulations of the treatment plan
- assist in selecting the type and size of implants
- assess results of prior bone grafting procedures
- store the patient data
- allow visual communication with patients and the implant team
- demonstrate knowledge, credibility, and safety
- help provide advanced patient care

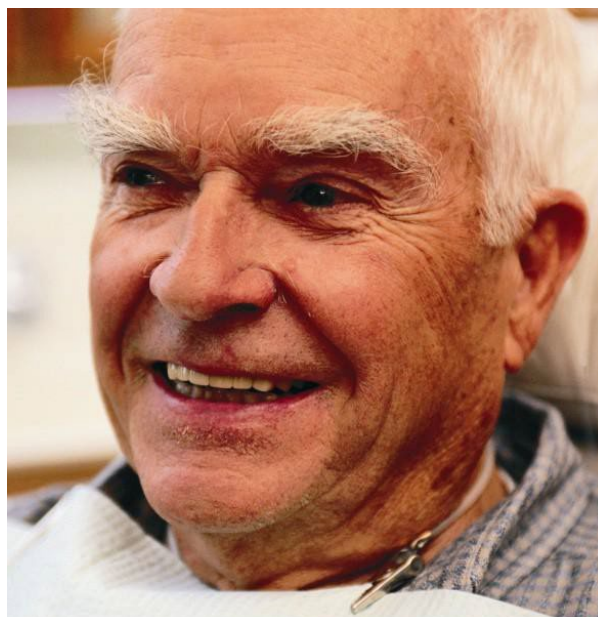

Blue Sky Plan® software can be downloaded from <http://www.blueskybio.com>. A user license must be obtained for each installation. During the installation process the software will prompt the user to request a license via the internet.

1.1 Indications For Use

Blue Sky Plan is intended to be used as conversion software for Computed Tomography (CT) generated DICOM images into a format that allows a dentist to assess the anatomic topography of the maxilla and mandible as well as location of important structures. It allows the information to be used for pre-surgical treatment planning of dental implant procedures.

Blue Sky Bio Aligners are a series of clear, lightweight, plastic appliances indicated for the treatment of tooth malocclusions in patients with permanent dentition (i.e., all second molars). Utilizing a series of incremental tooth movements, it sequentially positions teeth by way of continuous gentle force.

Blue Sky Bio Computerized Bracket System is a software system intended for use as an aid in orthodontic treatment planning to correct Malocclusions in Orthodontic Patients. For use by dental professionals trained in orthodontic treatment, including radiographic analyses and treatment planning. Blue Sky Bio Computerized Orthodontic Bracket System is intended for use with commercially

available brackets currently used in standard orthodontic treatment. The end product is an indirect bonding tray for use by the Dental professional to place multiple brackets at the same time.

The denture module is intended to be used to design a denture and export the entire denture or selected parts of the denture from the software for 3D printing

The crown and bridge functionality allows a dental professional to design one or more new teeth in the Blue Sky Plan that will attach to either a damaged tooth or a titanium based abutment.

1.2 Warnings

The software does not contain any functionality verifying if real-life implementation of the virtual treatment plan might result in damage to the patient. Any virtual treatment plan should be evaluated by a trained licensed professional confirming that the virtual treatment can be implemented in real life and will not cause harm to the patient.


If the segmentation function of the software is used, the segmentation has to be carefully evaluated to ensure that it matches the patient's anatomy and corrected if necessary.

It is important to verify that the imported and exported data sets are of adequate quality and match the patient's anatomical structures.


Appliances manufactured based on the virtual treatment plan should be made only from biocompatible materials that have suitable mechanical properties and are in compliance with local regulatory bodies. Appliances should be marked with patient information to avoid errors when delivering the appliance. Poor quality radiographic or optical images, inadequate training of user or inadequate hardware can affect the image results.

The DICOM data can be used to do the following:


1) make a scale 3D model of the jaw with the planned implant according to the illustrations below:

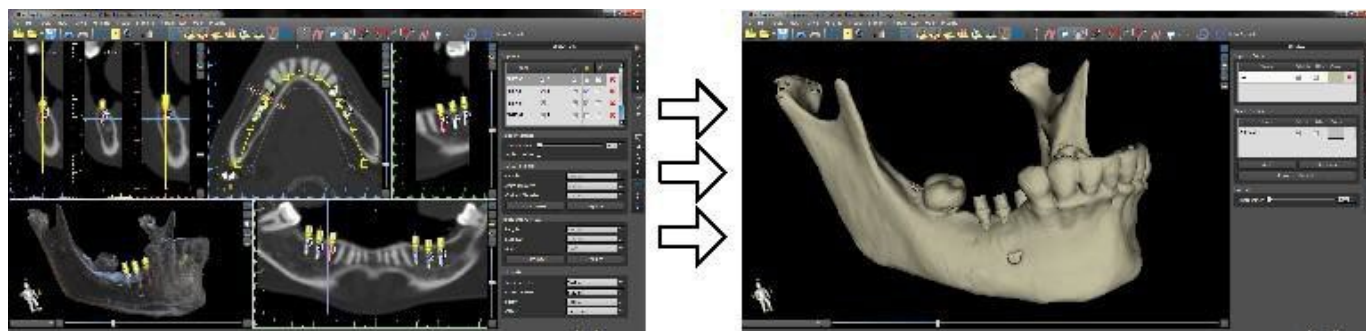

2) fabricate a surgical guide supported on bone

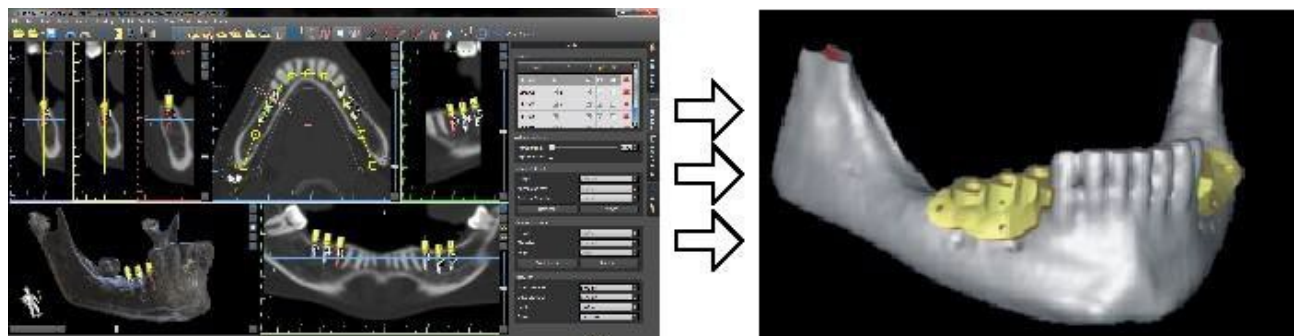

3) fabricate a surgical guide supported on the gingiva

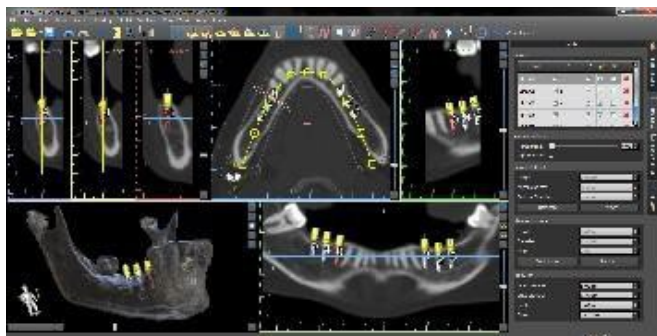


*The files with three-dimensional positioning of implants and the file of the three-dimensional guide can only be generated in Blue Sky Plan®.

2 Installing Blue Sky Plan


Blue Sky Plan® software supports Windows operating system. Installation packages are available for the systems indicated.

2.1 Installation process


After downloading the Blue Sky Plan® software to your hard drive, simply execute the file and follow the steps to proceed:

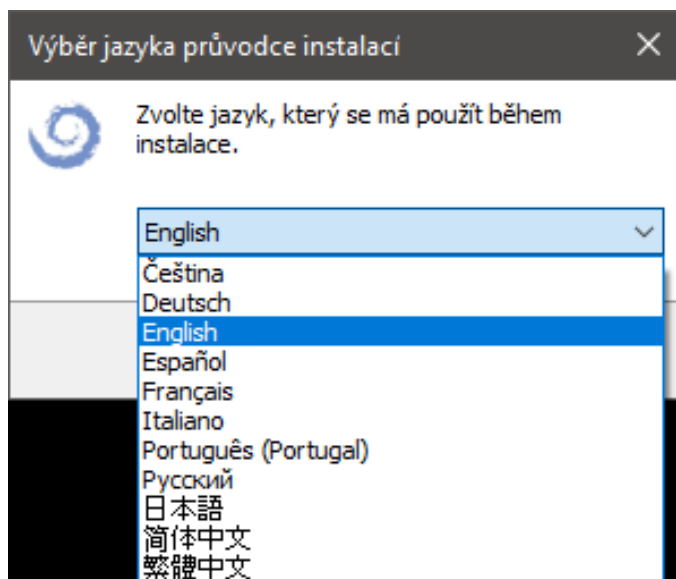


Choose the language and click "OK".

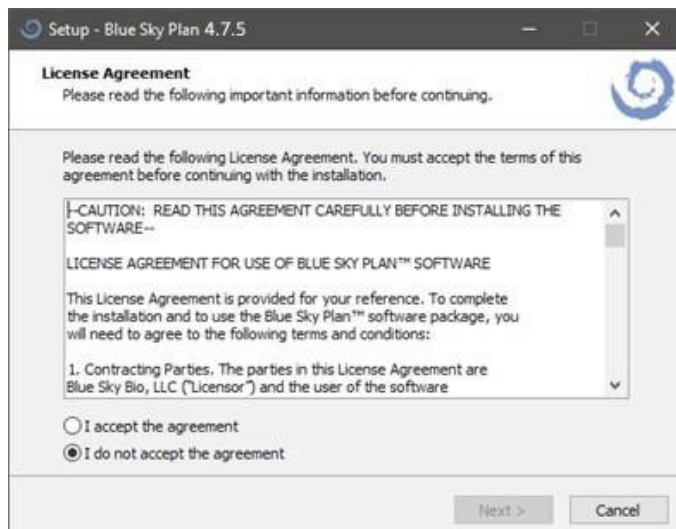


Carefully read the license agreement, select one of the options, and click "Next".

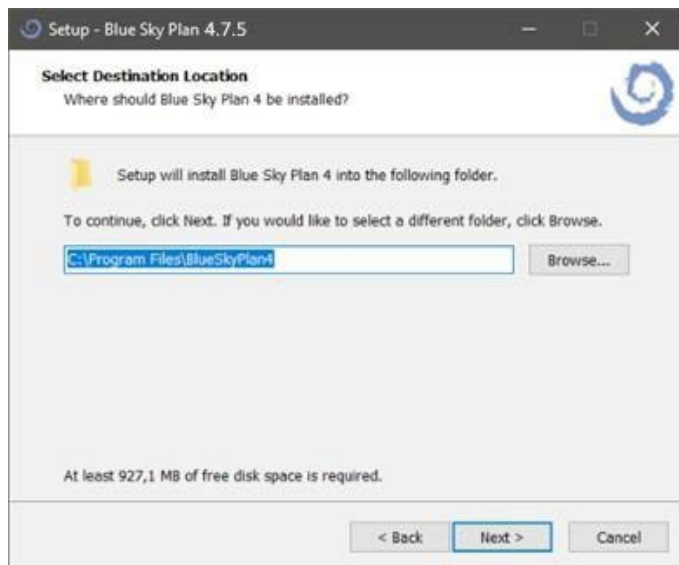

Click "Next".

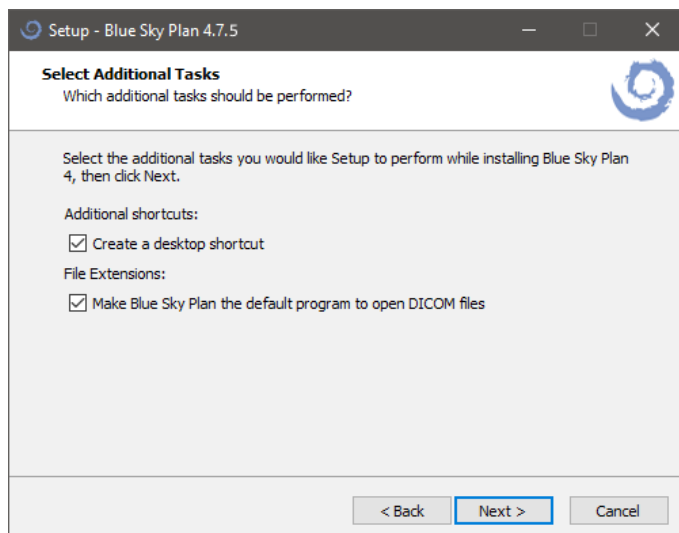

Click "Next".

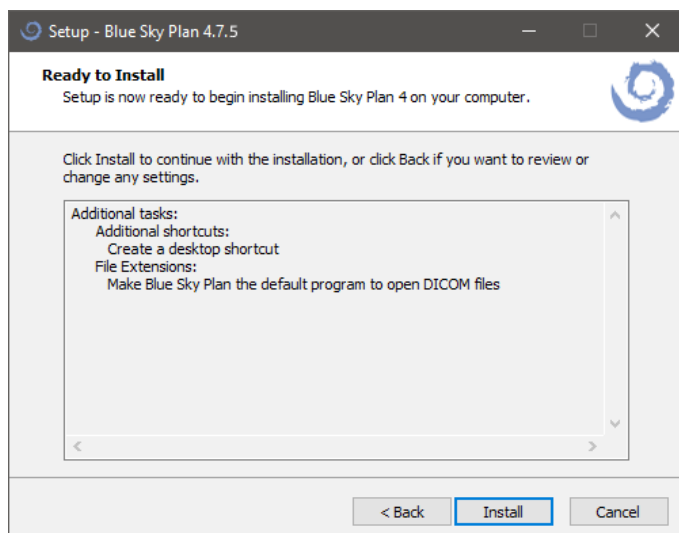

Click "Install".



Click "Finish".

2.2 Hardware requirements

Minimum requirements:

- OS: Windows 10 Home/Professional 64 bit
- Processor: Quad Core Intel i7 or comparable
- RAM: At least 16 GB
- Video card: AMD Radeon RX 470, NVIDIA GeForce GTX 1050 Ti
- Monitor: resolution at least 1850p
- Hard Disk: 5 GB of free space

2.3 Operating system

Minimum requirements:

- Windows 10 Home/Professional 64 bit

3 Blue Sky Launcher

The Blue Sky Launcher is a multifunctional tool that provides access to all features of the Blue Sky Plan® application. It allows users to launch the specific treatment planning modules, organized in hexagonal tiles (1), and provides access to processes linked to the BlueSkyBio Digital account (2)(see chapter 4). In addition, users can manage catalog packages (3)(e.g., implants, teeth) and open existing projects (4). The Blue Sky Launcher serves as a "central hub", effectively integrating all key functionalities into one intuitive interface.


The treatment planning options are organized into distinct modules (Orthodontics, Implant Planning and Surgical Guide, etc.), which can be further divided into planning types (Aligners, Surgical Guide, etc.). Each module and planning type has its own specific requirements regarding the data that needs to be imported.



3.1 Modules and planning types


Planning a new treatment begins by creating a new project. To start, the user selects a specific module (1) and, if available, a specific type of planning (2). Below is a list of all the modules and their planning types:

- **Easy CT DICOM Viewer** - see chapter 7
- **Orthodontics** - see chapter 25
 - **Aligners**
 - **Indirect Bonding Tray (IBT) Align Brackets to Teeth**
 - **Indirect Bonding Tray (IBT) Align Brackets to Wire**
- **Splint** - see chapter 28
 - **Incisal Splint (AMPSA)**
 - **Occlusal Bite Splint**
- **Implant Planning and Surgical Guides**
 - **One or Two Implant Case** - see chapter 24
 - **Surgical Guide** - see chapter 17
- **Denture** - see chapter 26
- **Cephalometric analysis**
 - **Cephalometric analysis from an Image**
 - **Cephalometric analysis from a CT**
- **Crown and Bridge** - see chapter 27
 - **Conventional Crown**
 - **Crown on Ti-base**
 - **Bridge**
- **Endodontics** - see chapter 18
- **Model Master** - see chapter 13
 - **Model Master (CT)**
 - **Model Master (STL Only)**

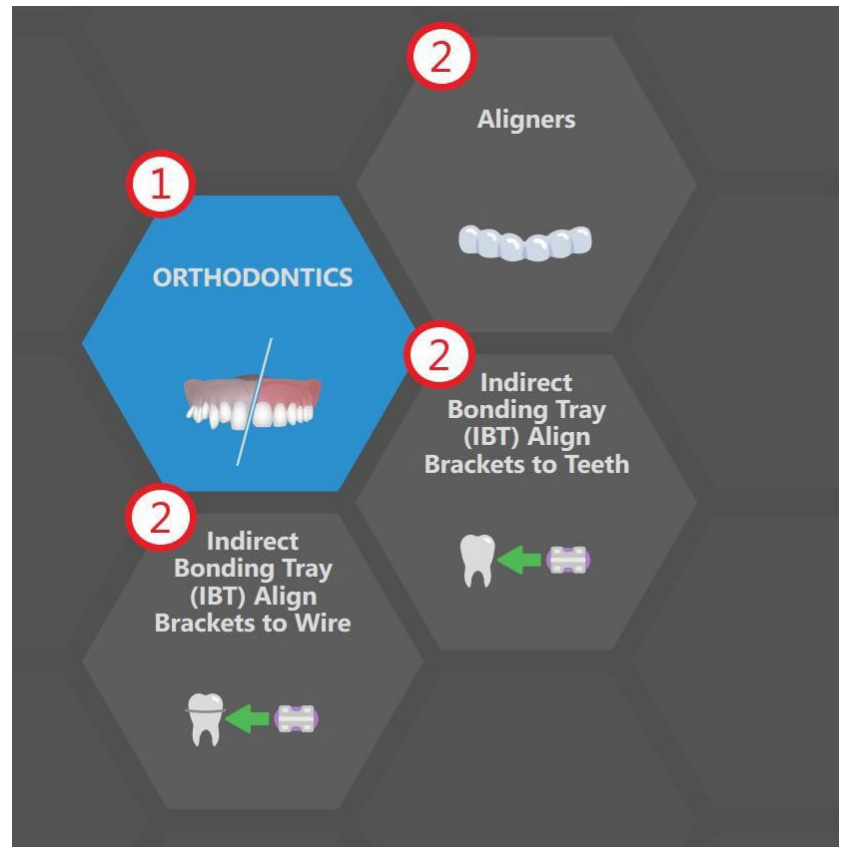


3.2 Project Setup

After selecting the module and planning type, the user enters the Project Setup flow, which consists of two interconnected steps. In the Project Settings (1) the user has to create or select the patient, treatment, and select the planned jaw. In Data Import (2) the user has to import the relevant data. When all these required steps are done, it is possible to start the actual planning that has been selected. Navigation between steps is provided by two blue arrows, Back (3) and Next (4). Returning to the previous step (Back) is always allowed, while moving forward (Next) is only allowed if all the requirements for the current step have been fulfilled.

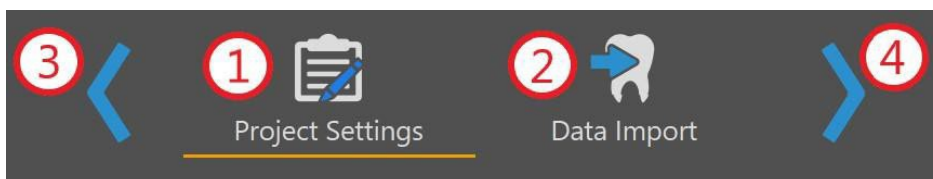

3.3 Project Settings

The first step of the flow consists of: creating / selecting the patient (1), creating / selecting the treatment (2), and, if available, choosing the type of jaw (3). Patient and treatment management enables integration with online services, such as BioBigBox, LabPronto and iSmile.

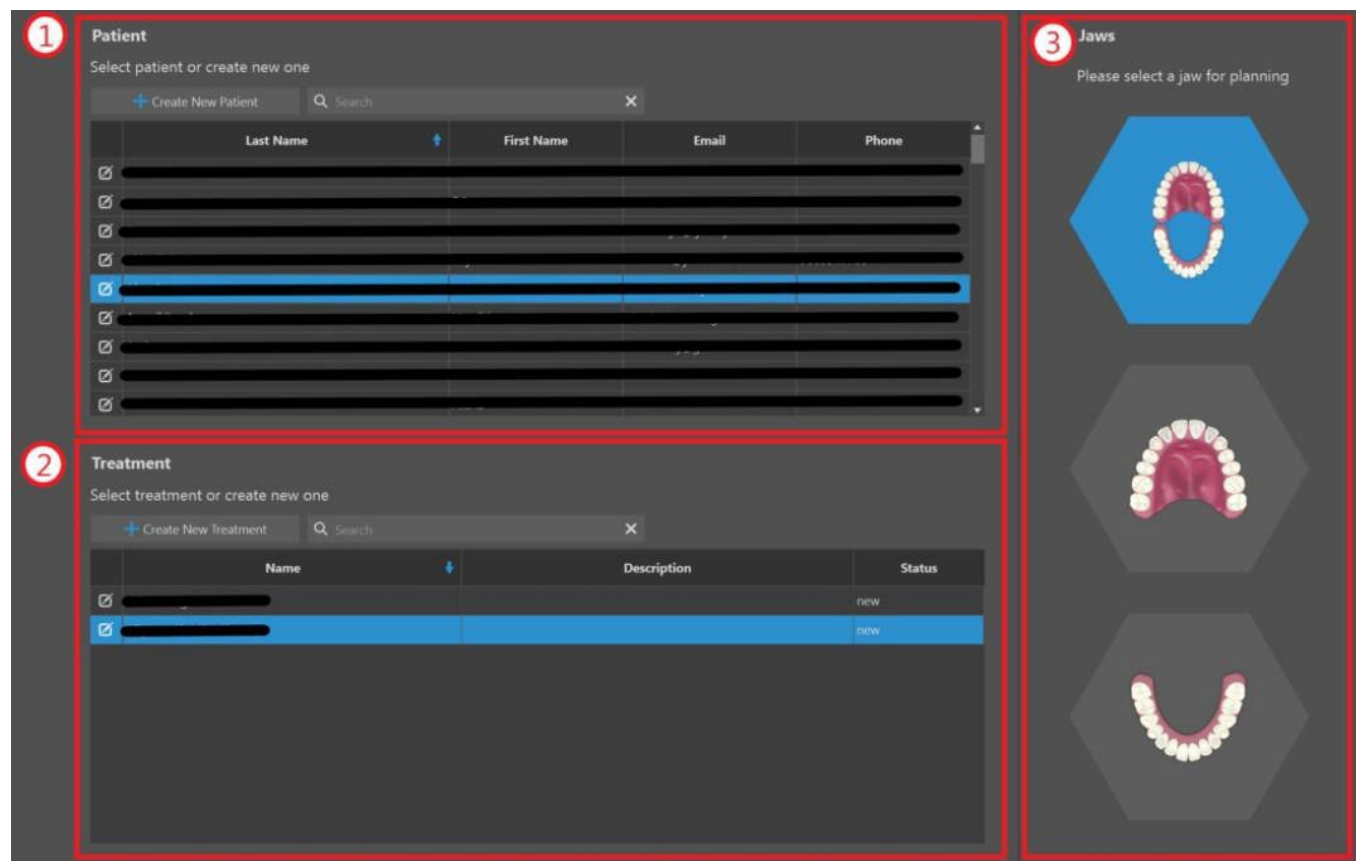

3.3.1 Creating and editing patients

At the beginning, the new "generic patient" is automatically created. The user can also create the patient manually by clicking the "Create New Patient" button - the "Add Patient" dialog will open.

Creating a patient requires at least the "First Name" and "Last Name" fields to be filled. Other fields are optional, though "Email" needs to be added in a standard email format. Click "Create" to add the patient, or "Cancel" to close the dialog.

Any patient information can be edited later. To do so, click the "Edit patient information" icon on the left side of the "Last Name" cell. Adjust any field as needed and click the "Update" button to save the changes. The "Remove" button deletes the patient entirely, and the "Cancel" button closes the dialog.

3.3.2 Creating and editing treatments

After creating a patient, the new treatment is automatically created. The user can also create the treatment manually by clicking the "Create New Treatment" button - the "Add Treatment" dialog will open.

Dialog box titled "Add Treatment - Blue Sky Plan".

Fields:

- Name * (required): Aligners
- Description: (empty text area)
- Status: new

Buttons: + Create, X Cancel

Note: Fields marked with * are required

The "Name" field is required for creating a treatment, other fields are optional. Click "Create" to add the treatment, or "Cancel" to close the dialog.

Section titled "Treatment".

Text: Select treatment or create new one

Buttons: + Create New Treatment, Search

	Name	
	[Redacted]	
	[Redacted]	

As with patients, treatments can also be edited. Click the "Edit Treatment Information" icon on the left side of the "Name" cell. The "Update" button saves the changes, the "Remove" button deletes the treatment, and the "Cancel" button closes the dialog.

3.3.3 Sorting columns and search bar


To enhance the organization of patients and treatments, all columns in the tables can be sorted by various criteria. Any patient and treatment can be easily found using the search bar.

3.3.4 Jaw selection

Some modules, like Orthodontics, require a planned jaw - maxilla, mandible or both jaws. Clicking the planned jaw type enables the user to continue to the Data Import step.


3.4 Data Import

The second step involves importing the data that is needed for the next step of planning. Depending on the selected planning type, it will be necessary to import CT data or an STL model (or an image in the case of Cephalometric Analysis from Image). When the user selects and confirms the desired data in the window dialog that opens upon entering Data Import, the data is automatically categorized based upon its titles. It is also possible to import data via the buttons "Import Model(s)" and "Import CT Scan" (or "Import Image(s)" in the case of Cephalometric Analysis from Image)(1) or via the Import Item ("+") button (2). Individual data are represented by a single item. The selected item is displayed in the preview (3).

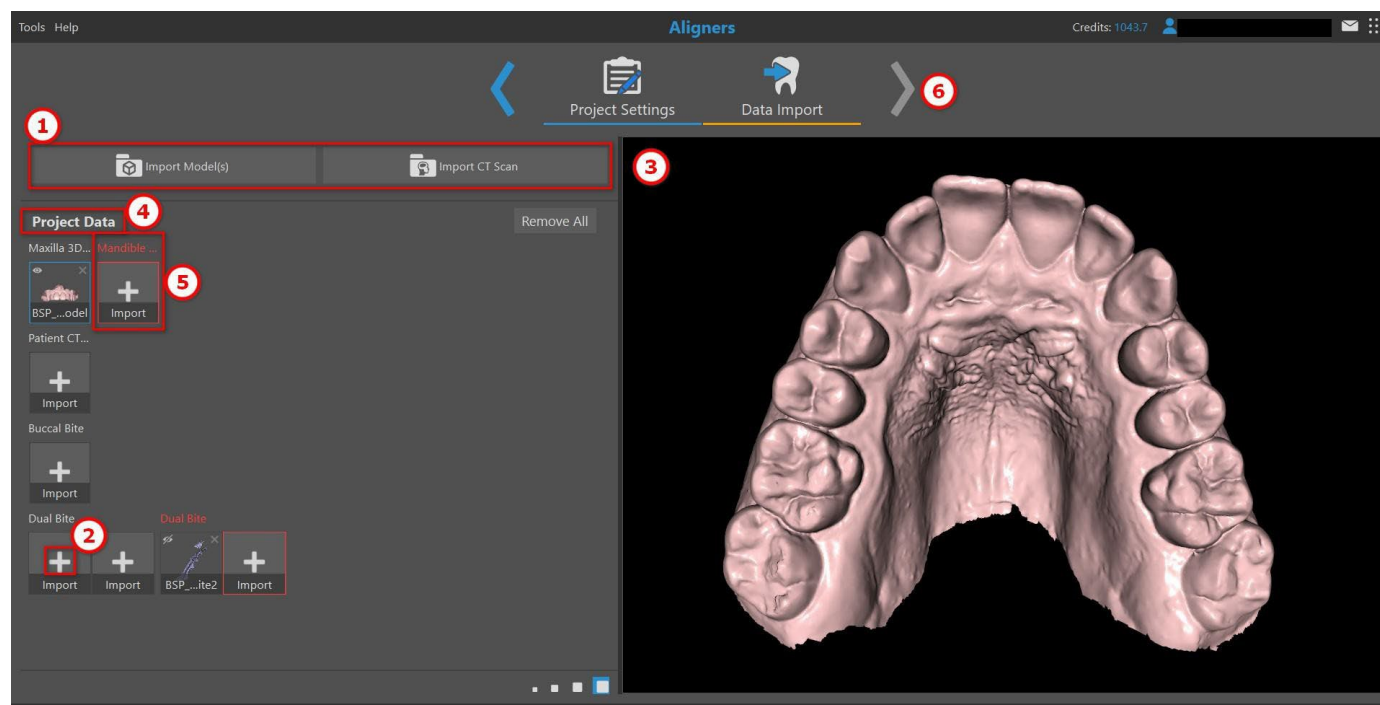

3.4.1 Data management


Project Data (4) contains mandatory and optional data. Mandatory data, outlined with a red border until filled, are required for further planning (5). On the other hand, data that were automatically classified as "Uncategorized" should be moved to the Project Data, if possible, because they might not be included in further planning (depending on the planning type selected and the type of data).

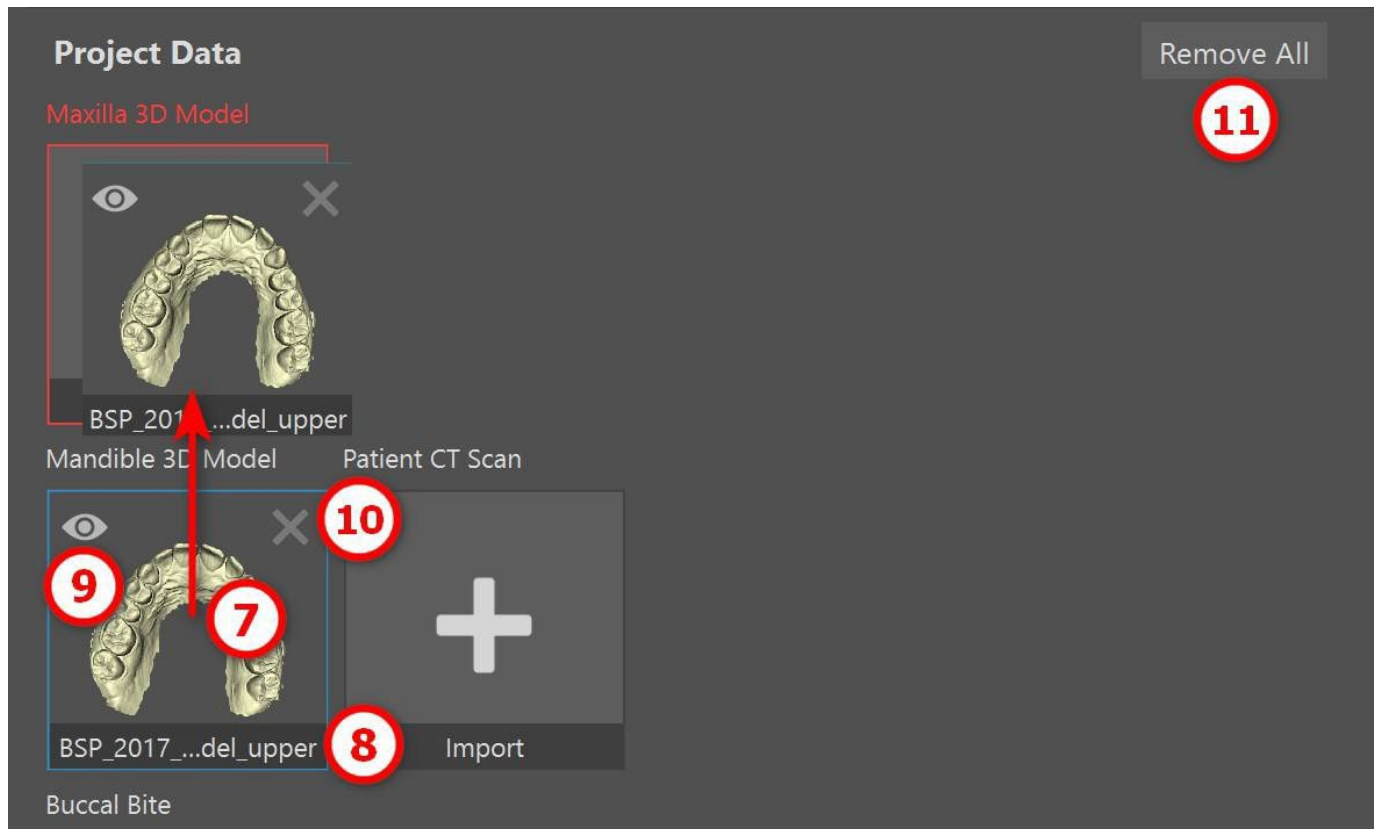

Data can be moved between categories via the drag-and-drop function (7), renamed by clicking on the file name (8), shown or hidden in the preview (9), and removed either one by one using the cross (10) or in bulk by clicking the button "Remove All" (11). Once all requirements are met, it is possible to start planning by clicking the "Next" arrow (6).


3.5 Catalog management

Blue Sky Plan® allows the user to easily manage various catalog packages, including implants, teeth, brackets, and buttons (1). The catalog overview provides information about the package name (2), its size (3) and the package status (4)(installed or downloadable). Packages can be downloaded one by one (5), or en masse (6). In either case, the user can easily track the download progress at all times (7). Each package, except for those installed by default, can be uninstalled (8). If necessary, a package can be imported manually (9) by selecting the appropriate file in the window dialog.

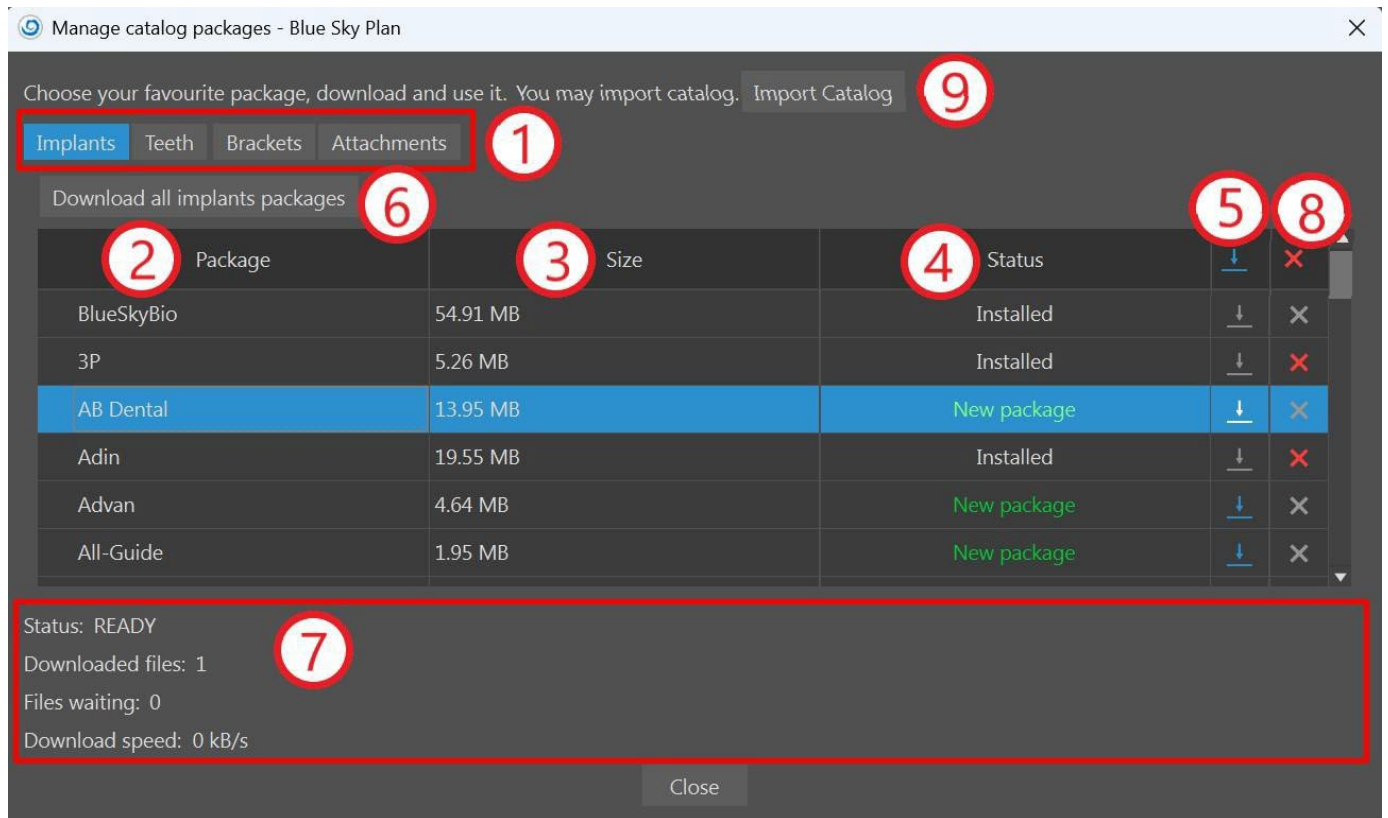


3.6 Project management

Blue Sky Launcher allows the user to access and manage existing projects saved in any module. Each project provides key details, such as the patient's name, last update, project name and planning type (1). The pin feature (2) makes projects easily accessible and displays them on the leftmost side.

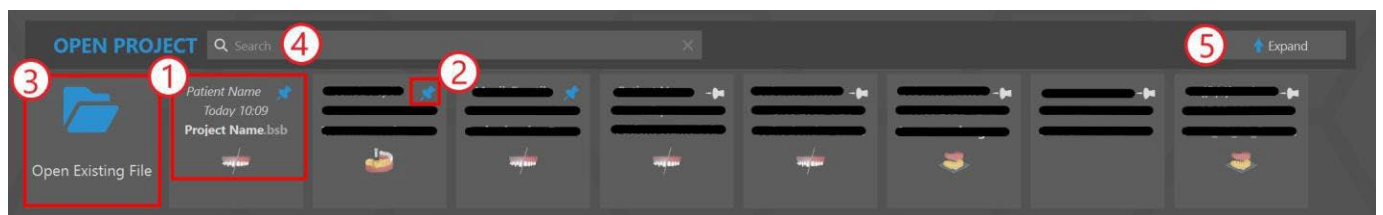

To open an existing project, the user can click the "Open Existing File" button (3) and search for it through a pop-up file dialog. If the user wants to locate a project more specifically, they can use the search bar function (4). Clicking the "Expand" button (5) will reveal additional projects with more detailed information, and a preview (6). With the filter function (7), the user can narrow the list of projects based on criteria such as the planning type, whether the project is pinned, or whether it is saved on a cloud or locally. The "Clear filters" button resets all the applied filters (8). Finally, the project section can be returned to its original size by clicking the "Collapse" button (9).

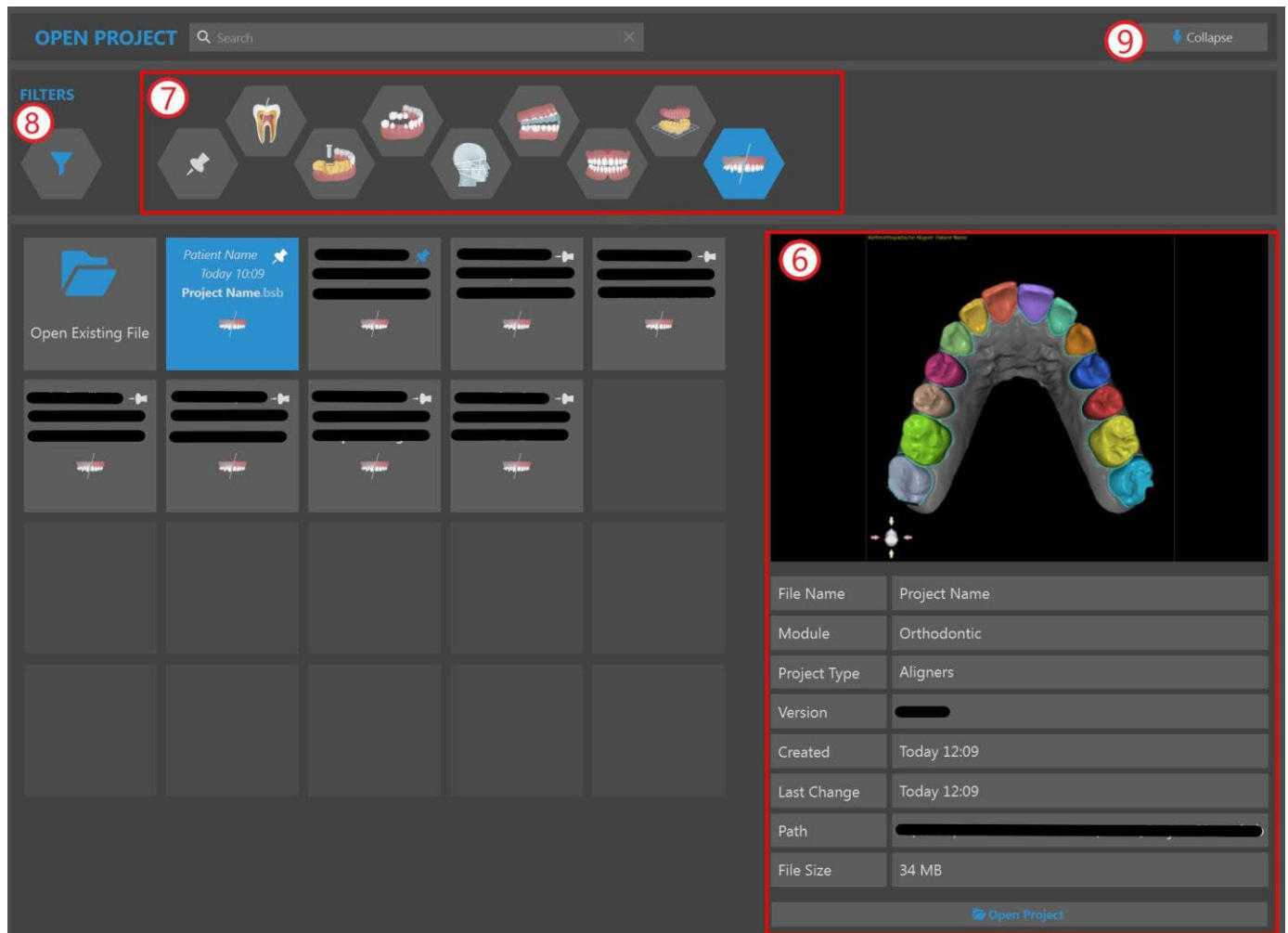

To open a project, the user can either left-click on it and select the "Open Project" button in the pop-up dialog or, if the project section is expanded, double-click the project or use the "Open Project" button located on the right side below the project details.

4 Licenses


The license system is based on the BlueSkyBio Digital account (hereafter the "BSB account") environment, where one unique license is generated for each user. The whole process of license creation is done automatically when the user launches the current version of theBlue Sky Plan®.

4.1 The main characteristics of licenses

- The credits from one user's account can be used on multiple machines. Exports on the given license are shared between those multiple machines.
- The user can share their license with another user as soon as the owner of the license allows sharing (through email approval).

4.2 License registration

The process of license registration is very simple and intuitive. On the first launch of Blue Sky Plan®, the user is redirected to the BSB account login page in the browser.

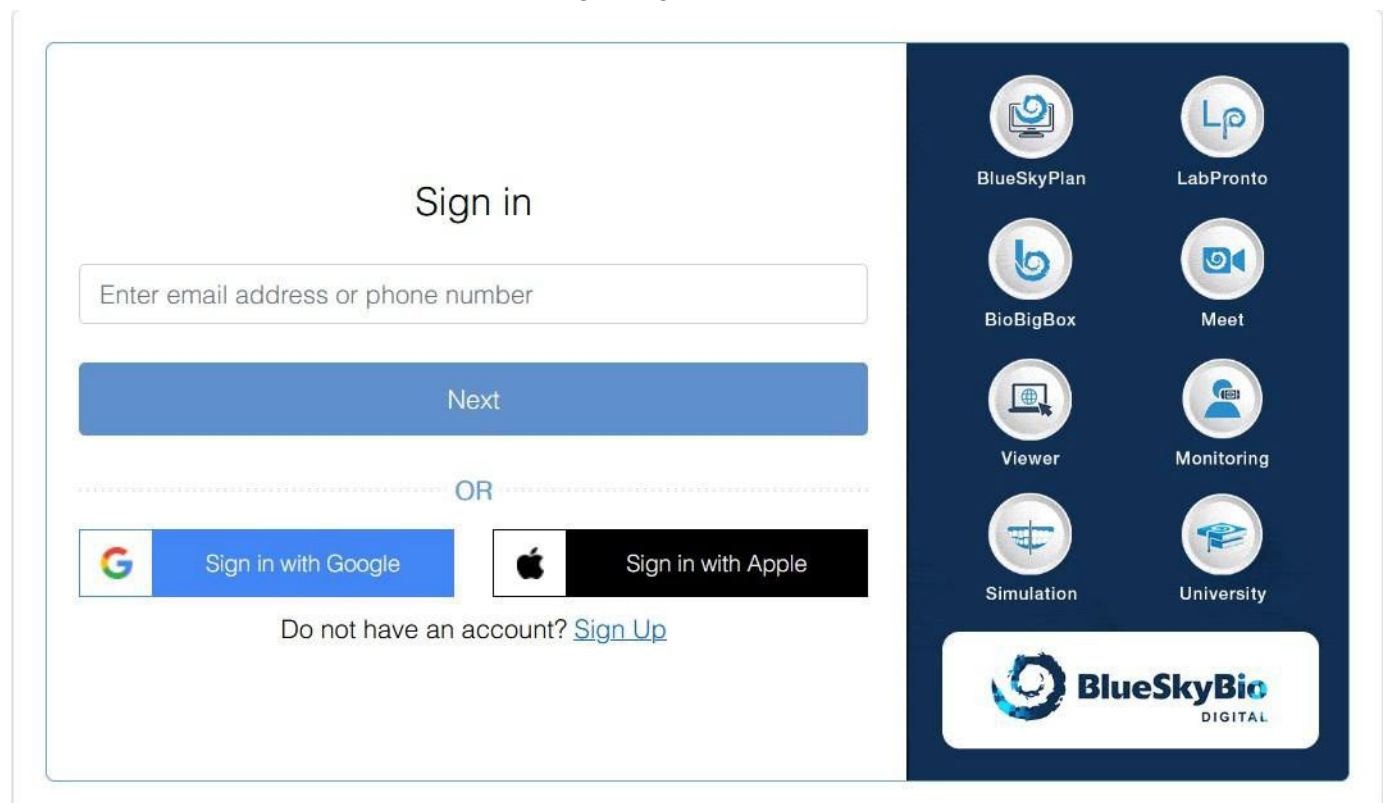

Sign in

Enter email address or phone number

Next

OR

Sign in with Google

Sign in with Apple

Do not have an account? [Sign Up](#)

BlueSkyPlan

LabPronto

BioBigBox

Meet

Viewer

Monitoring


Simulation


University

BlueSkyBio DIGITAL


If you don't yet have a BSB account, click the "Sign Up" button and fill in all the necessary fields. It's also possible to create the BSB account and log in using a Google or Apple account.


Create your Blue Sky Digital Account

 Continue with Google

 Continue with Apple

OR






Password must be at least eight characters in length. It must contain at least one uppercase, lowercase, number and a special character.

Select


▼


☐ I am human



hCaptcha
Privacy - Terms


[Sign In Instead](#)


Next


BlueSkyPlan


LabPronto


BioBigBox


Meet

Viewer

Monitoring

Simulation

University



After creating the BSB account, a verification code is sent to the registered phone number.

Sign in


Please type the verification code sent to +


☐ Remember me


Resend in 40s.


[Additional Sign In Options](#)


Login


BlueSkyPlan


LabPronto


BioBigBox


Meet

Viewer

Monitoring

Simulation

University



Once the verification code is entered, the registration process is completed by confirming the email sent to the user's email address.



Welcome to Blue Sky Bio Digital Solutions!

Please confirm your account by pressing the button below.

Confirm




LabPronto.com

biobigbox.com



Copyright 2024 Blue Sky Bio. All Rights Reserved.


Completing the registration process enables the user to access the Blue Sky Plan® application. The current registered user is displayed in the top-right corner of the Blue Sky Launcher.

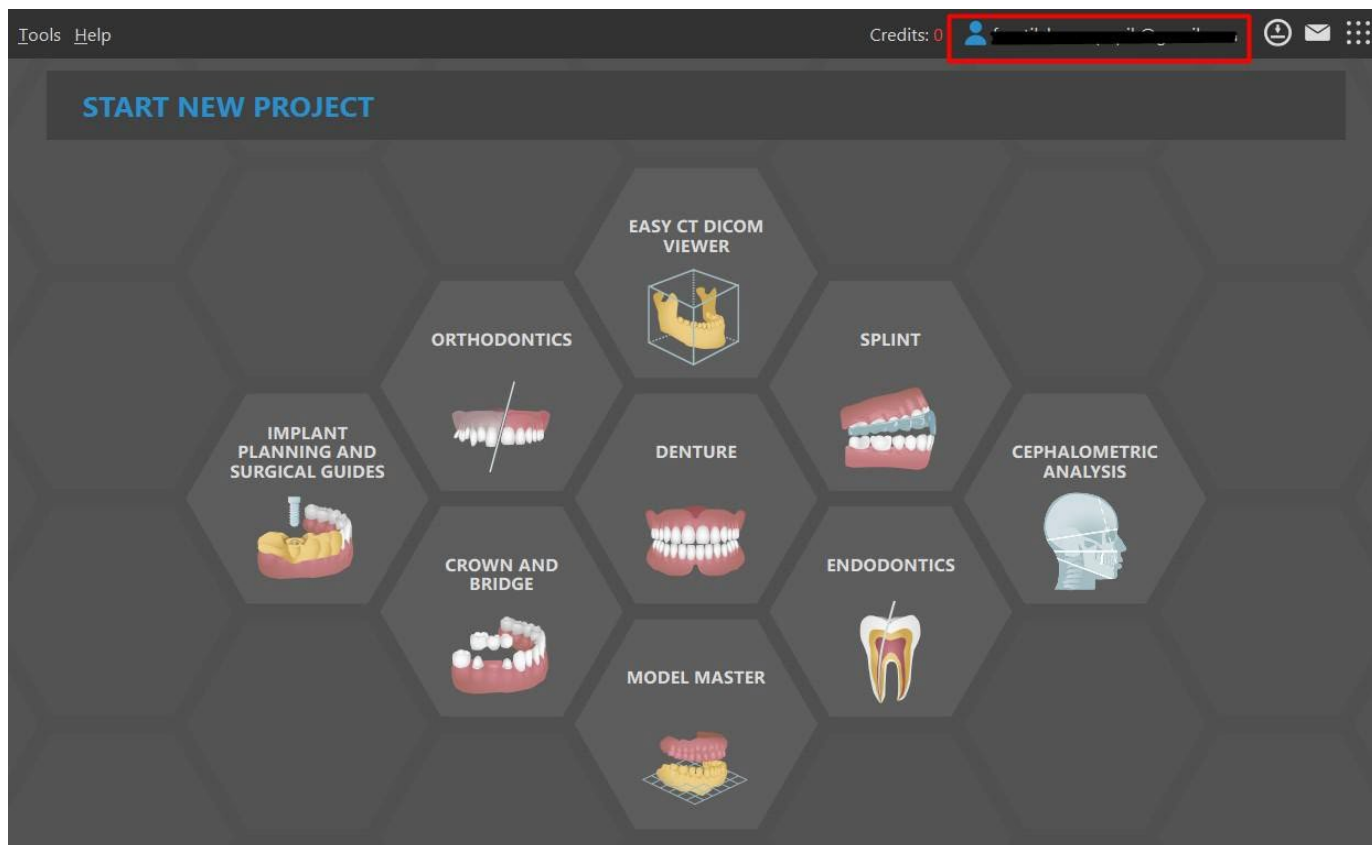

If the user is logged out from Blue Sky Plan®, the application will automatically redirect them to the BSB account login page, where they can log in with their email and password (or, alternatively, by the verification code sent to their phone number). After logging in to the BSB account, the user is also logged in to the BSB application.

4.3 BlueSkyBio Digital account options

In the top-right corner of the Blue Sky Launcher, the user may choose from different options linked to their BSB account and online services in general.

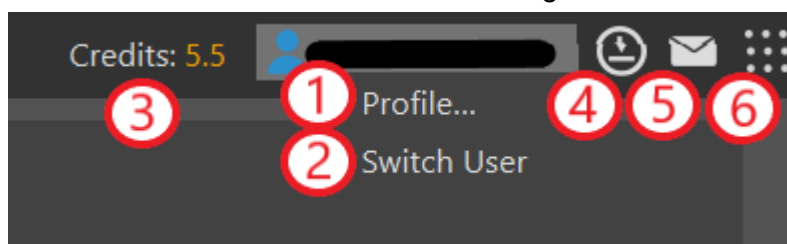


To check and manage the BSB account profile in the browser, open the context menu by left-clicking on the user's account and select "Profile..." (1). If necessary, the user can log out of the Blue Sky Plan® and allow another user to log in by clicking the "Switch User" (2), which will automatically open the BSB account login page in the browser.

In addition, the user can purchase credits (3), check if the application is updated (4), read notifications (5) and join any online service offered by Blue Sky Bio (6).

4.4 Licenses in Preferences

The user can check their license in *Tools > Preferences > BlueSkyBio Account*.

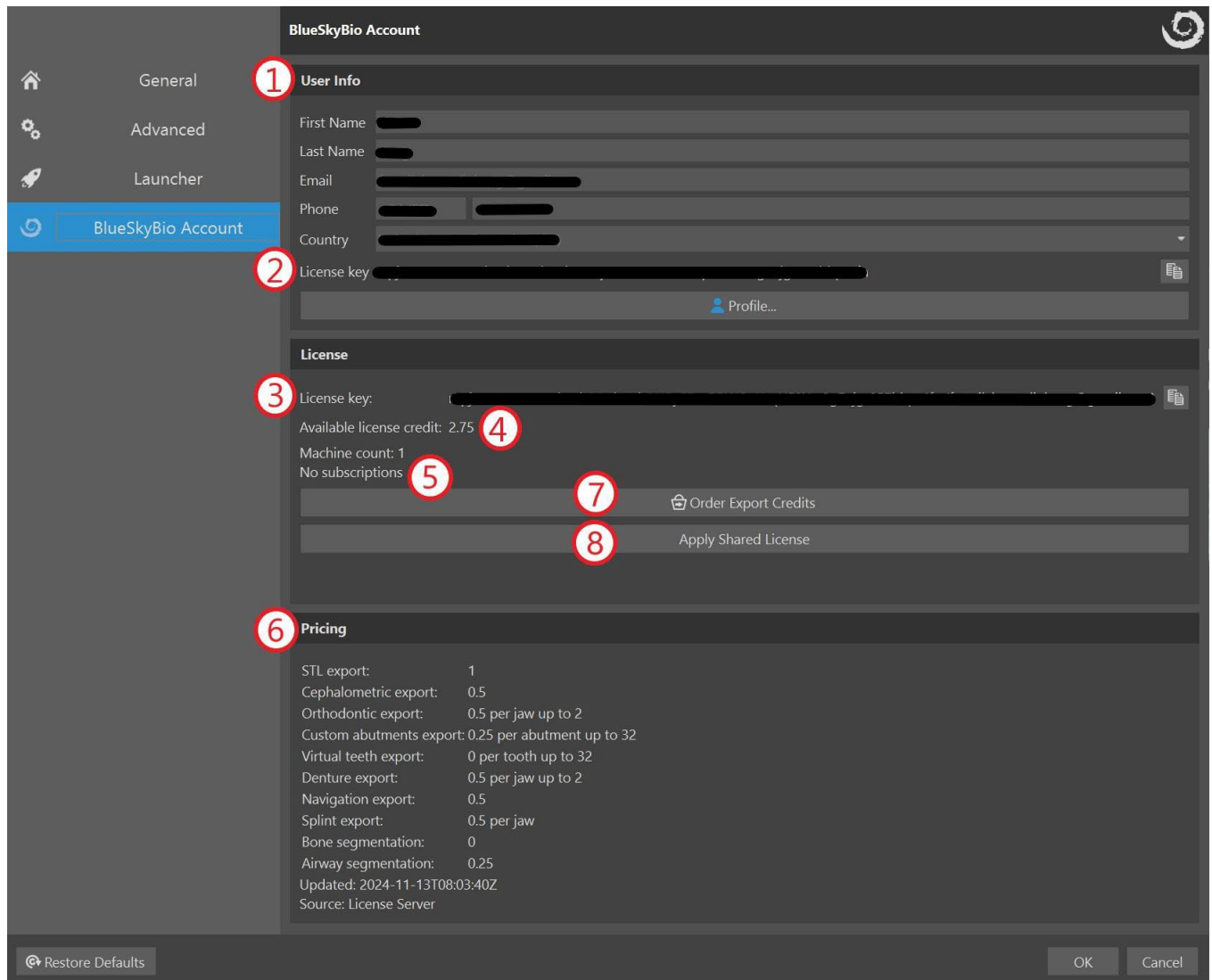


In "User Info" (1), the user can check the mandatory data, registered with their license. In this section, the user can also view their own (private) license key (2), whereas in the "License" section the user can view the license being used at the moment (3). This license can be their own (private) or a license shared with another user. If it is shared license, both license keys (2)(3) will differ. "Available license credit" (4) shows the remaining credit on the currently used license. The user also has an overview of all the active subscriptions (5). Pricing (6) shows the cost of different types of exports. The user can purchase credit or subscriptions by clicking the "Order Export Credits" button (7). The "Apply Shared License" or "Disconnect Shared License" button (8) switches between a private license and a shared license.


4.5 Shared Licenses

Users are able to use not only their own (private) licenses but also the licenses of other users. Such licenses are called "shared licenses".

In *Tools > Preferences > BlueSkyBio Account*, click on „Apply Shared License“.

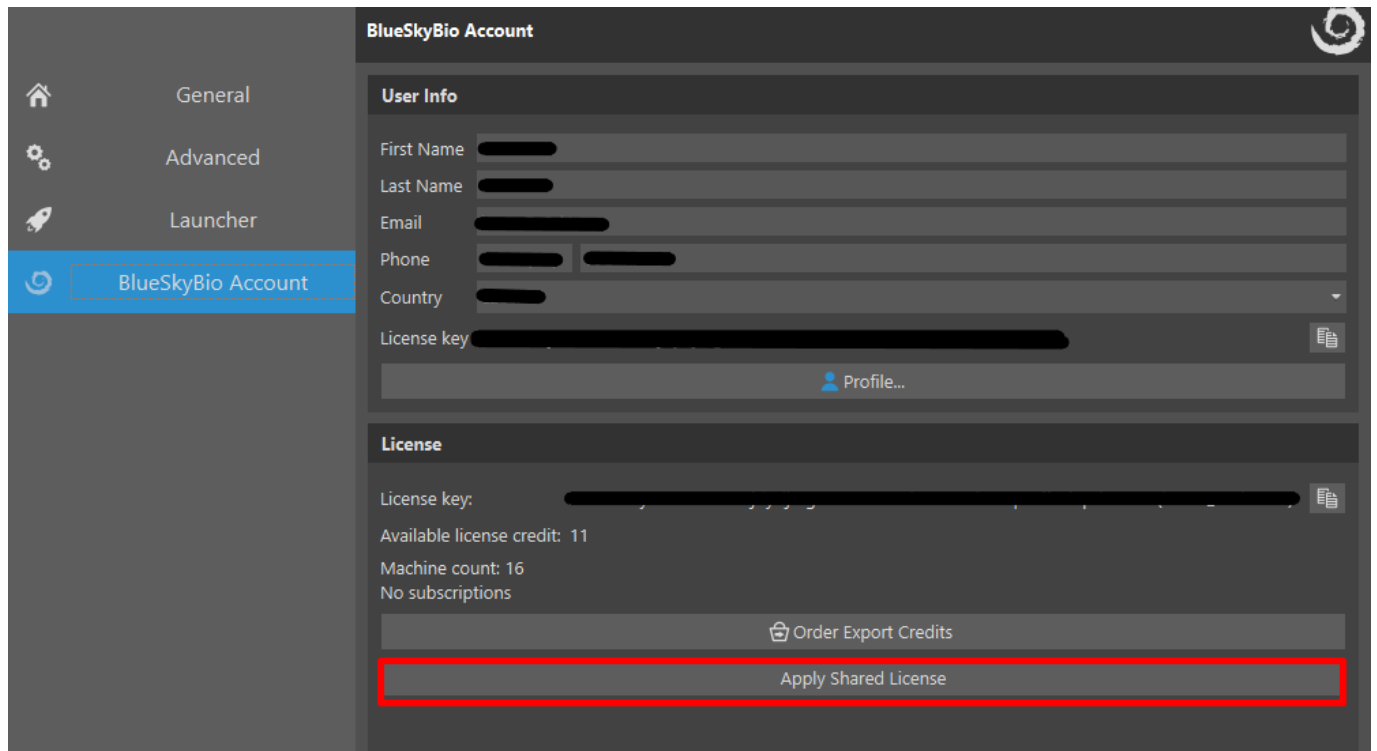

Enter the other user's license key – you are now informed that the owner's approval is needed. Note, that both users must share the same country (*Country Code*), otherwise the sharing license won't be possible.

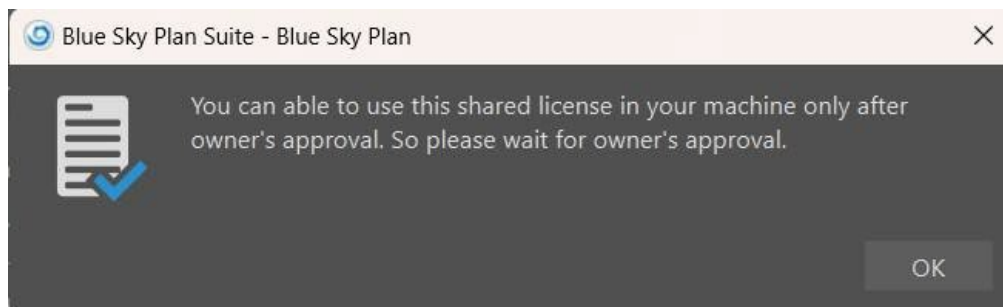

Meanwhile, the owner of the license receives an email with the license request.



BlueSkyPlan.com

Powered by BlueSkyBio.com

License Request!

Dear **Josef**,

New computer "xxxx-xxxx-xxxx [REDACTED]" is requesting your approval to use your existing license. If it is you or someone else with whom you shared your license then please approve it.

Name: [REDACTED]

Email: [REDACTED]

Approve

Sincerely,
The BlueSkyBio Digital Team.

Approving this email will allow you to use the owner's license on a computer with a unique machine code.



BlueSkyPlan.com

[Purchase Exports](#)

[Manage Credit Cards](#)

[Login](#)


[Signup](#)




[Czech Republic](#)

Request for sharing your license on computer xxxx-xxxx-xxxx [REDACTED] is approved.

When the owner of the license approves the request, the user who sent the request can click on "Apply Shared License" again and confirm the shared license key. The shared key is successfully applied and ready to use.

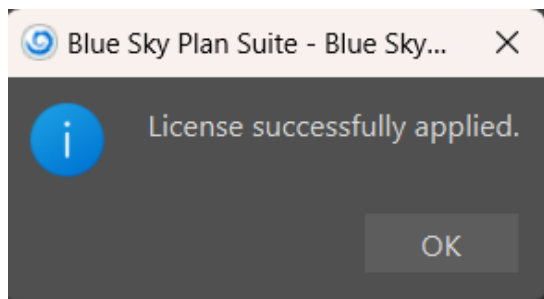

From now on, all credits will be deducted from the shared license. The user can switch back to their private license at any time by clicking “Disconnect Shared License.”

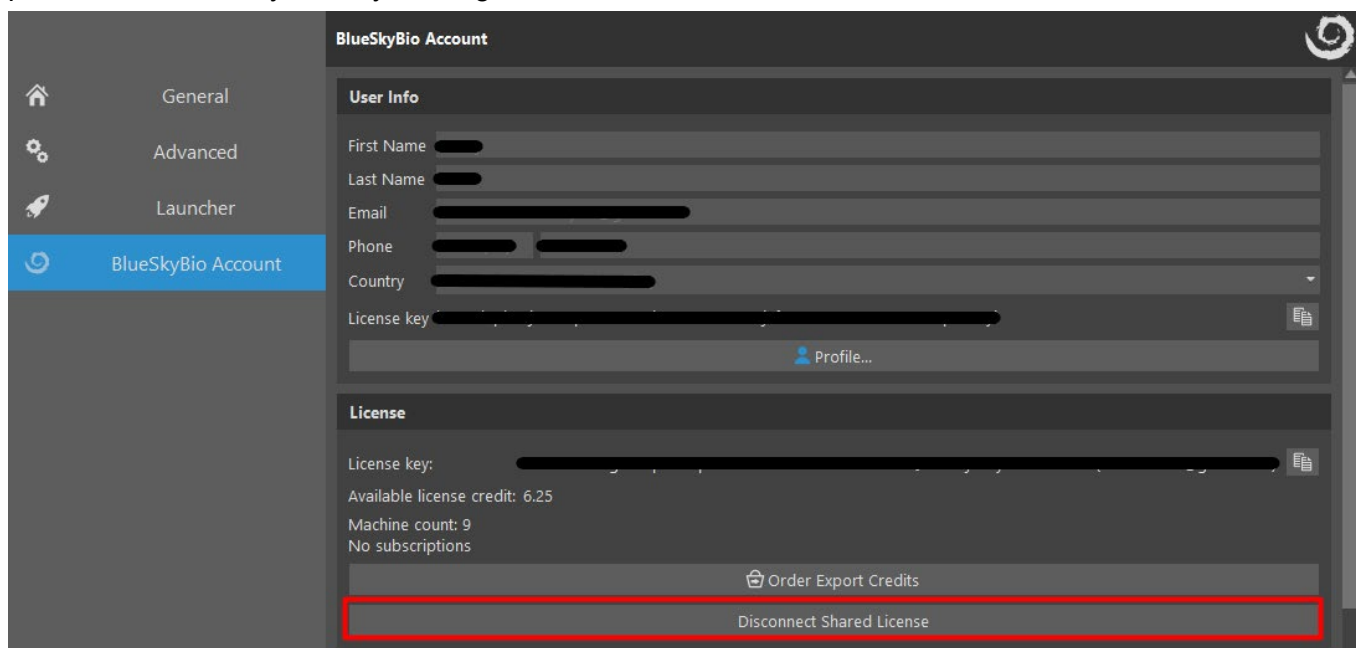

4.6 Purchasing export

If the user is logged into the Blue Sky Plan®, it allows them to purchase the credits or subscriptions required for various types of exports.

The user can buy credits either by clicking "Credits" in the top-right corner of the Blue Sky Launcher, or by navigating to *Tools > Preferences > BlueSkyBio Account* and click “Order Export Credits”.

The user is redirected to the "Purchase Exports" page, where they can select the desired package type and proceed to the checkout.

Select Package Type

	Package Type	Export Credits	Free Bonus Credits	Price
<input checked="" type="radio"/>	STLE5	5	1	
<input type="radio"/>	STLE10	10	2	
<input type="radio"/>	STLE50	50	3	
<input type="radio"/>	STLE100	100	5	
<input type="radio"/>	STLE200	200	10	
<input type="radio"/>	STLE300	300	15	
<input type="radio"/>	STLE500	500	25	
<input type="radio"/>	STLE1000	950	50	

Proceed to Checkout

The user can choose from various payment methods. After the ordering process is complete, the export credits are added to the user's license.


5 Loading the patient's CT or CBCT scan into Blue Sky Plan

Blue Sky Plan® can load uncompressed DICOM images (exported by all CT scanners) into the software. Blue Sky Plan is “ self processing ” allowing the user to directly import the DICOM images in a few easy steps. A complete scanning protocol is available via the Blue Sky Bio website and can be sent to the scan site.

5.1 Selecting the DICOM images location

To import the patient's CT or CBCT images, follow these steps:

1. Select planning type "Surgical guides" and then "Import Patient CT Scan".
2. A new window will appear.

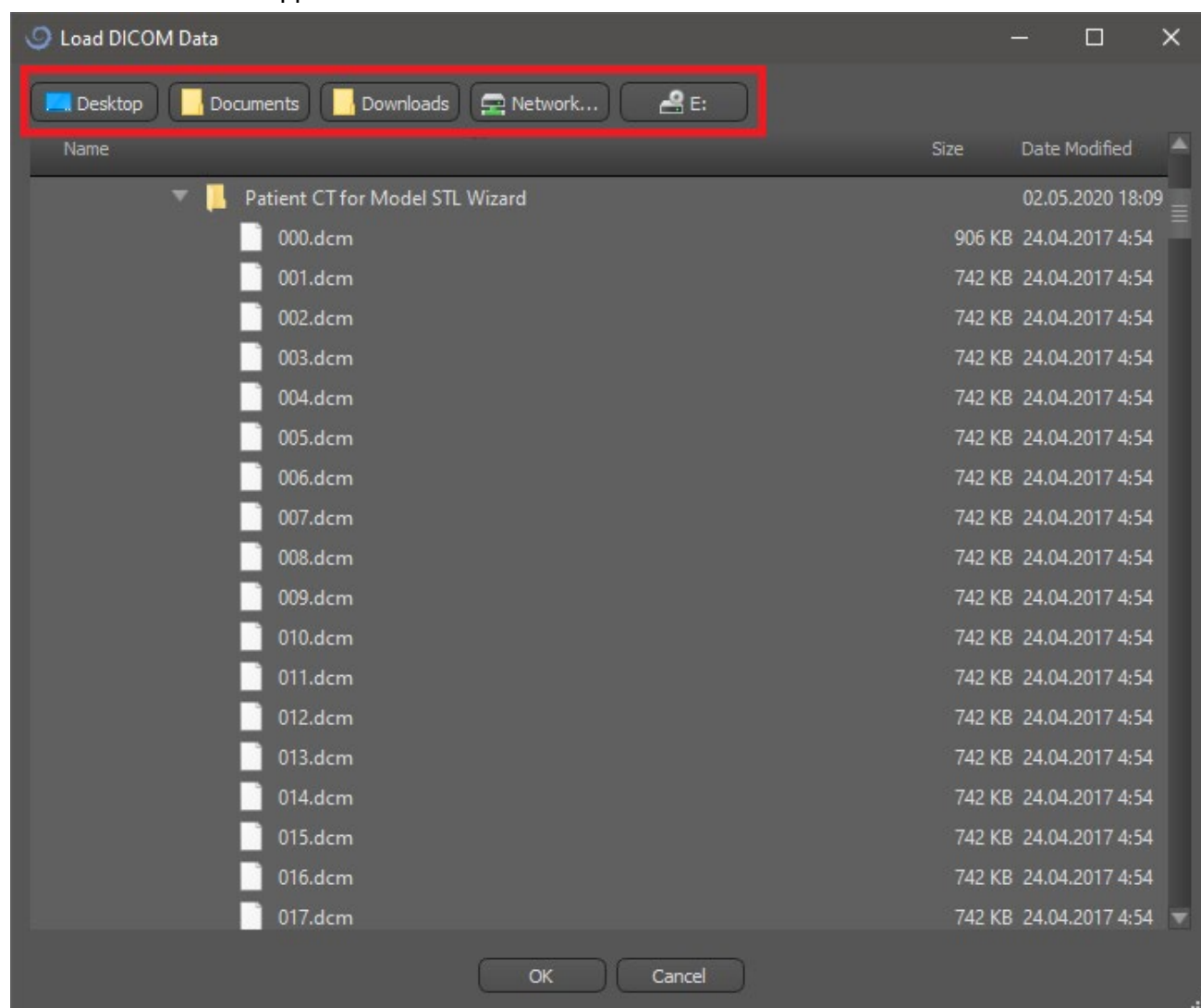


Note that there are several shortcut buttons that appear on the top of the window. These buttons allow quick navigation to the folder containing the DICOM images.

Select the directory that contains the DICOM images (you can also load a zipped file with DICOM images) and click OK. Images must be stored in DICOM format from your Imaging Center in order to load them into Blue Sky Plan®.

5.2 Selecting the series

A list of available series that can be loaded will appear (1). Ideally each series should be saved in a separate folder, and only one series will appear.

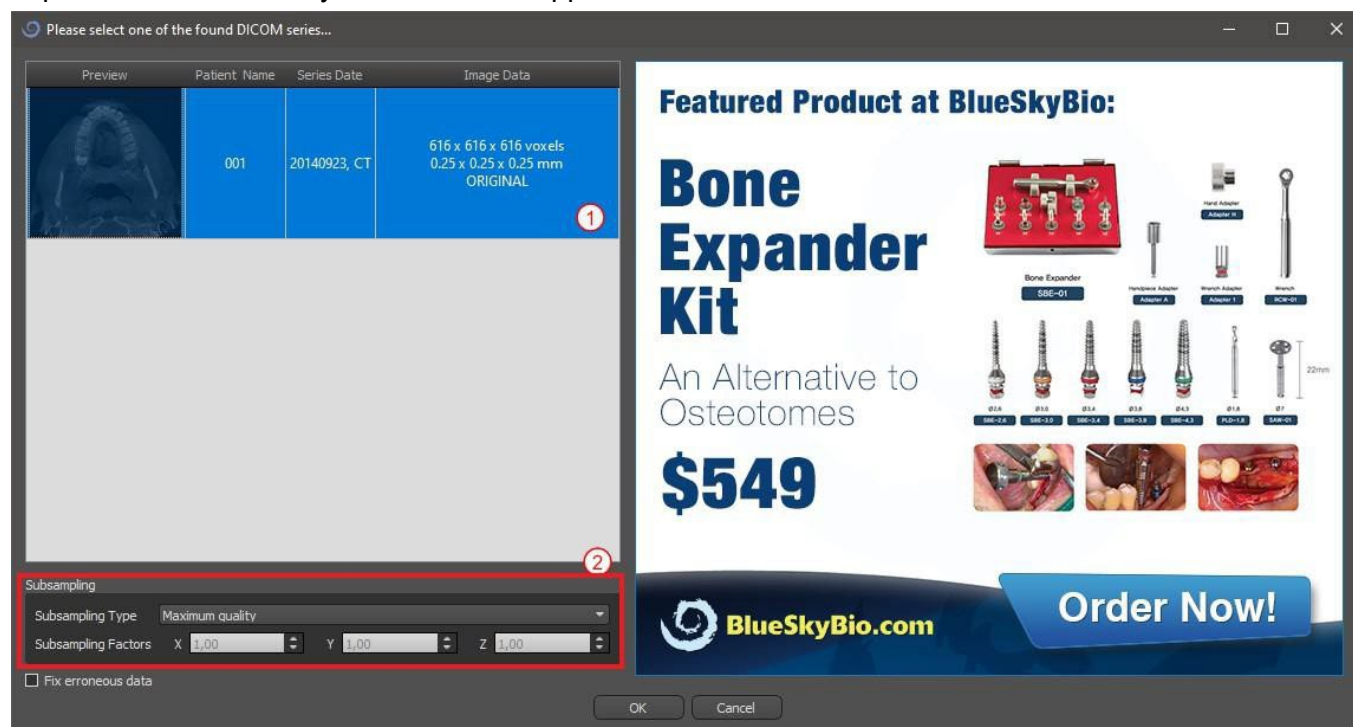


5.3 Enhancing performance

In the case that the computer does not have sufficient system or video memory, and possibly also if the video card is not sufficiently powerful, the program makes it possible to reduce (downsample) the input data; the user may set this manually for each axis or choose one of the available settings (2).

The software will automatically compensate for the “skipped” slices, and the quality of the treatment plan and Surgical Template will not be compromised.

5.4 Scan alignment and defining the range of slices

In this step you align volumetric data and delimit the area of interest in the data. It is possible to crop the data in all directions by dragging the yellow lines (1) or by directly setting the dimension of the slice for the individual axes (2). The data display mode can be changed by selecting “Imaging Mode” (3).

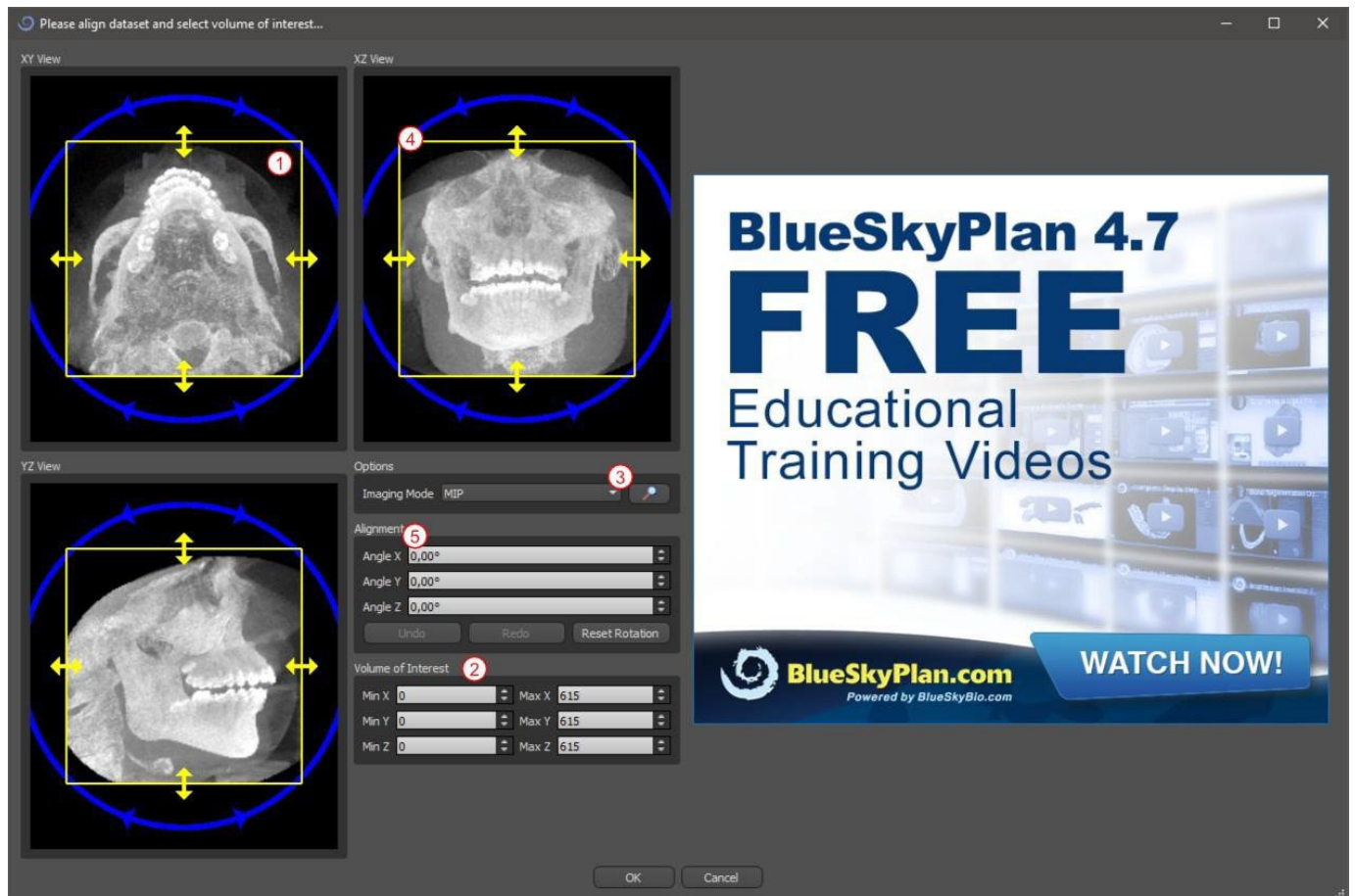


The data can be aligned using the blue circles (4); it is also possible to enter values for rotation around the individual coordinate axes manually (5). The transformation performed then applies correctly for importing data.

Given the significant memory and performance requirements for visualizing 3D medical data, it is advisable to crop the input data to the actual area of interest; this allows the program to run much more smoothly.


5.5 Import additional surfaces

After loading CT data, the user can import additional surfaces. After choosing one, select the directory that contains required model or scan.

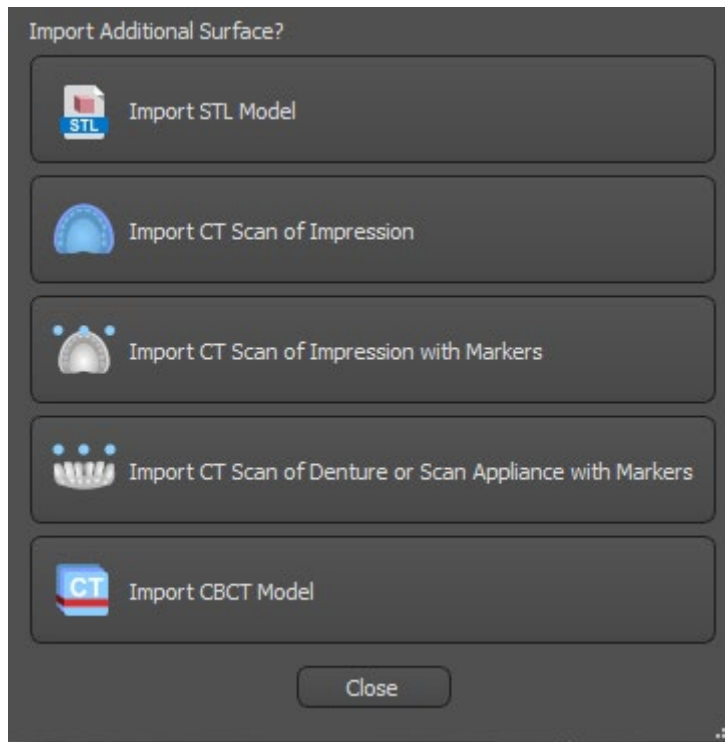

5.6 Positioning the panoramic curve

The panoramic curve (the middle of the three, alternatively five, curves which appear in the axial view) is generated automatically after importing CT data. It is possible to manually adjust it.

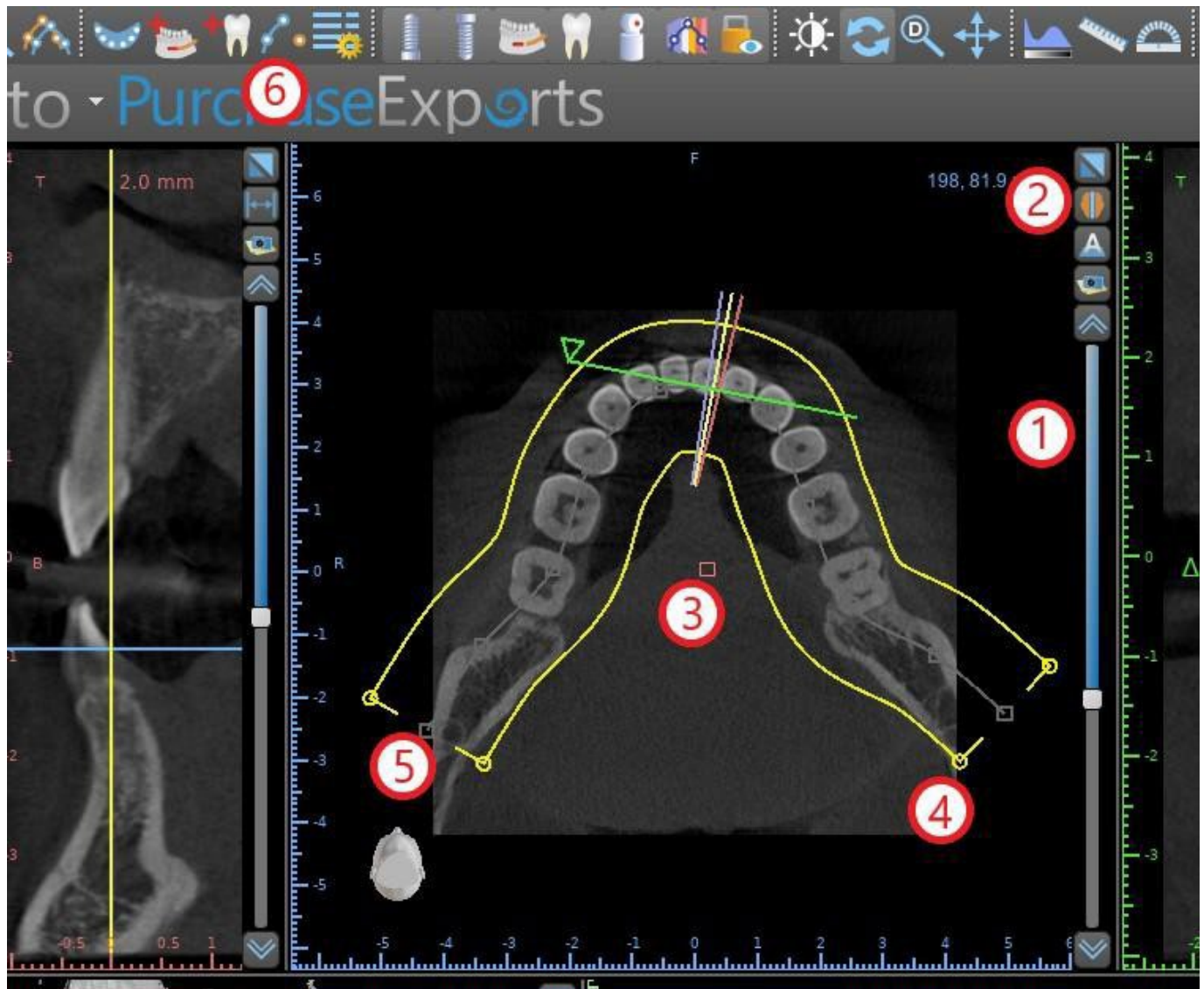


This can be done by using the slider (1) to identify an axial slice which shows the complete arch/ridge of the patient's jaw. If necessary, it is possible to switch the display mode of the axial slice to an X-ray display by clicking button (2).


The curve as a whole can be shifted by dragging red handle (3), and the width of the curve can be adjusted by dragging any yellow handle (4). Dragging the yellow handle within the curve decreases/increases the curve. By dragging any of the grey handles (5) along the curve user can align the curve in position as desired.

Alternatively the entire curve can be redrawn by pressing the "Draw dental curve" button (6). The user can left-click periodically as the mouse moves around the arch (from left to right), placing dots. The software will connect the dots to draw the panoramic curve. When the curve is complete, simply double click, or click on the "Draw dental curve" button (3) again, or press "Esc".

6 Software interface

Blue Sky Plan® allows seamless interaction among the various windows displaying the axial, sagittal, coronal, cross-sectional, tangential, panoramic, and 3D images.

Depending on the intended use, it is possible to select the active windows for multi-planar functions (axial, sagittal, coronal, and 3D) or implant functions (cross-sectional, tangential, axial, panoramic, and 3D). User can switch between different perspectives (in menu bar "View > Perspectives" (1), alternatively by clicking on the button  in icon toolbar (2)). Within any perspective, it is possible to add any view window (in menu bar "View > Views" (3)).

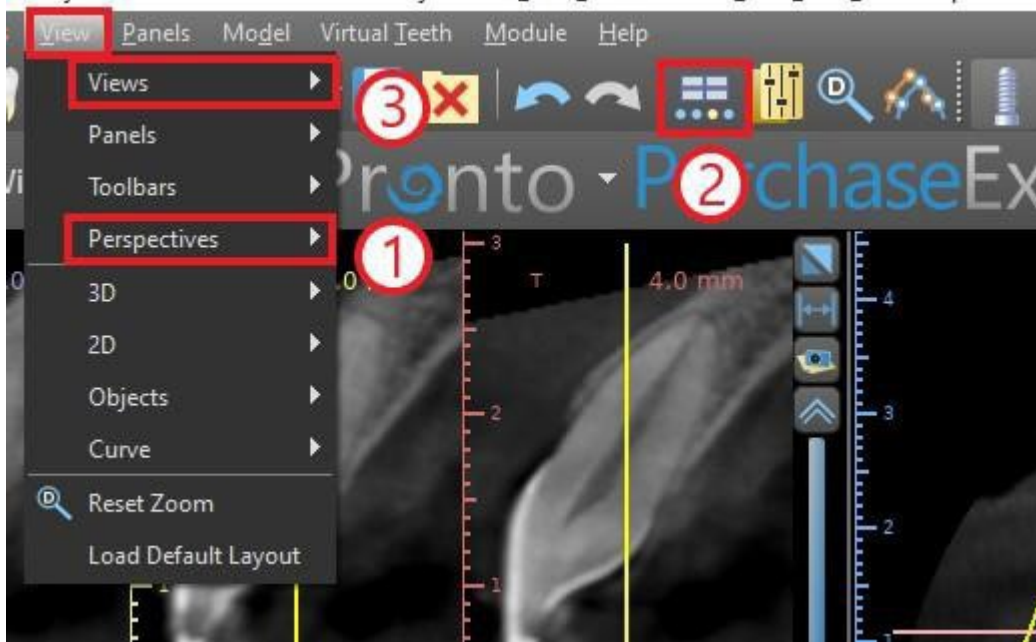


The following information and tools can be viewed in most of active windows:

- individual colors for each cross section of images,
- indicators showing the current position of the images,
- label of the image
- rulers

6.1 Screens and views


The implant screen allows visualization of the cross-sectional, tangential, axial, panoramic, and 3D images. All implant planning tools can be accessed through the menu bar (1) below. The icon toolbar and the panoramic image toolbar (2 and 3, respectively, in the following illustration) show the most-used tools.

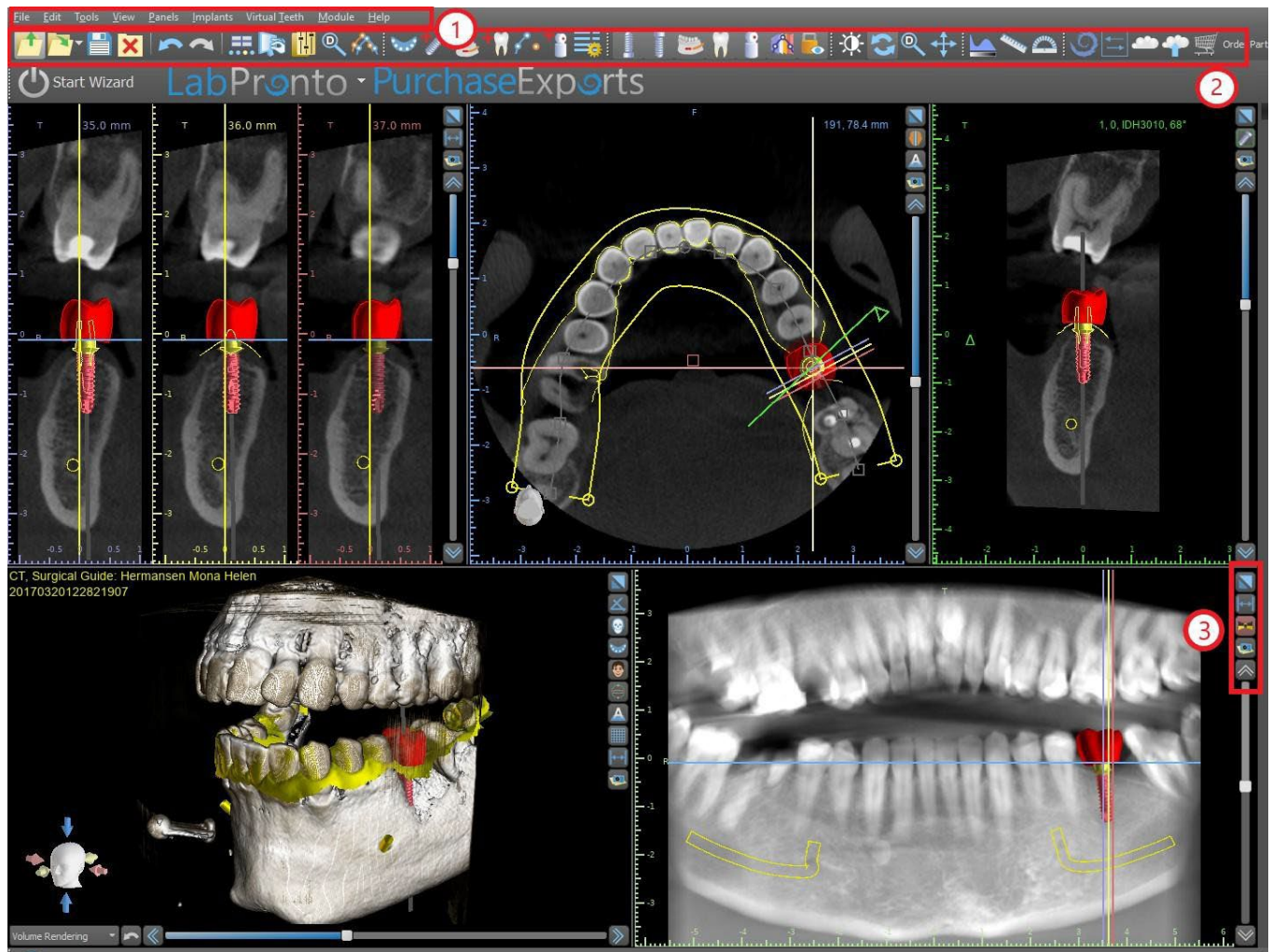

6.1.1 Axial window

The axial images are horizontal slices of the volume and can be seen in the middle top (or in the following illustration) view in the application (1). They are marked in blue. All the other images are generated from the sequence of axial images.


By moving the axial slider (2), it is possible to view the whole sequence of axial images. The indicator line of the axial slice appears in the cross sectional, panoramic, and 3D images (3).

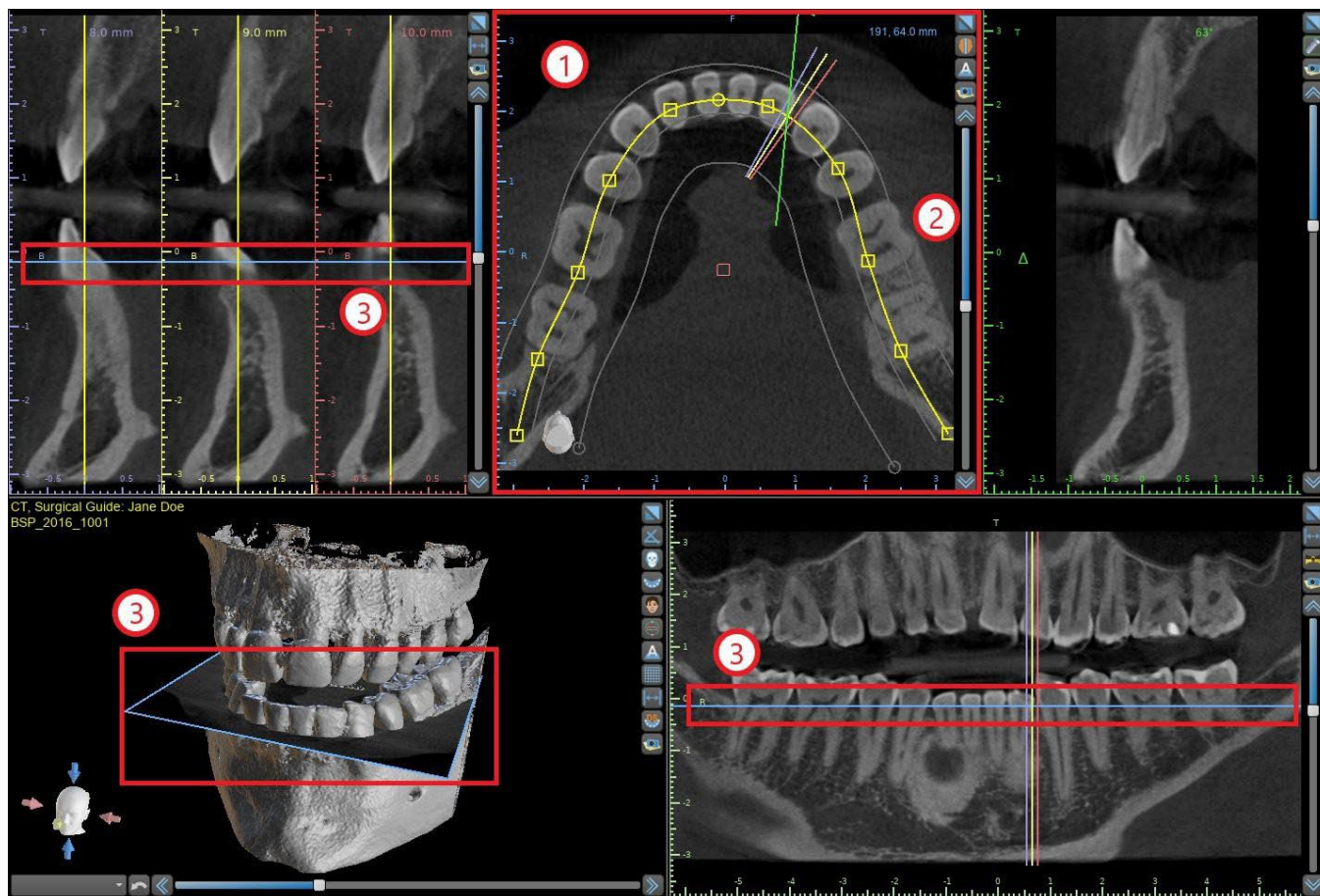

6.1.2 Cross-sectional window

The cross-sections view displays vertical slices going around the patient's ridge. As the user moves through the slices using the cross-sectional slider (1), the corresponding multi-color lines in the axial and panoramic views (2 and 3) will move accordingly, identifying the location of the five (or three) images being displayed in the cross-sectional views.

These images are used to evaluate the bone height and thickness in planning. When implants are placed, they are often placed into one of the images in the cross-sectional view.


The distance separating the images can be modified by clicking on the  icon (4), and a new value can be selected from the drop-down menu. By default the distance between images is set to 1mm.



6.1.3 Panoramic window


The panoramic image is located in the bottom right-hand window of the application (1) and is marked with light green.

Thanks to the option of rotating the plane passing through the implant, it is possible to easily check and correct the tilt of the implant's position when viewing from any angle.

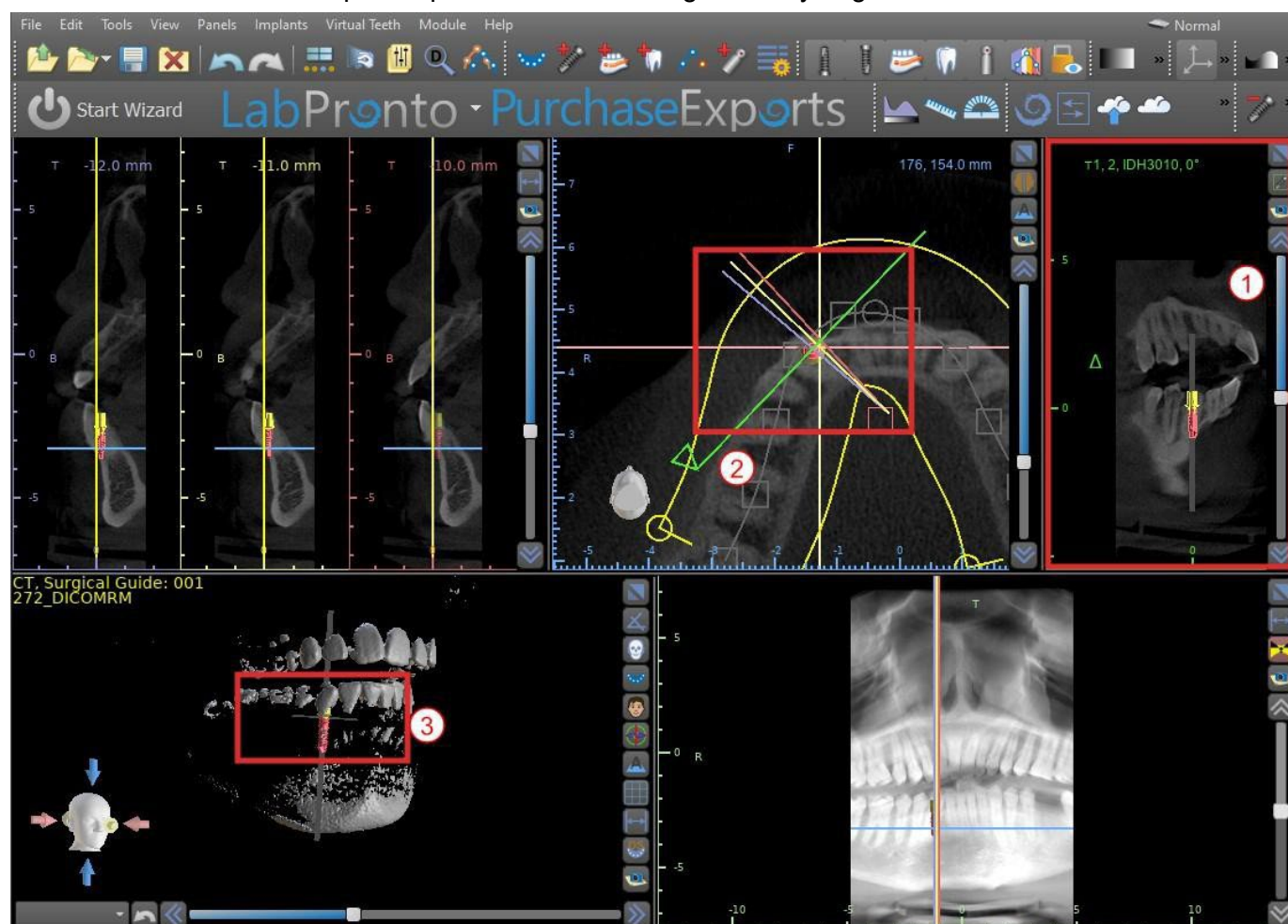

6.1.5 3D View

The 3D image is located in the bottom left-hand view of the application (1). To rotate the 3D image, click and hold the left mouse button in the model, to zoom, click and hold the right mouse button.

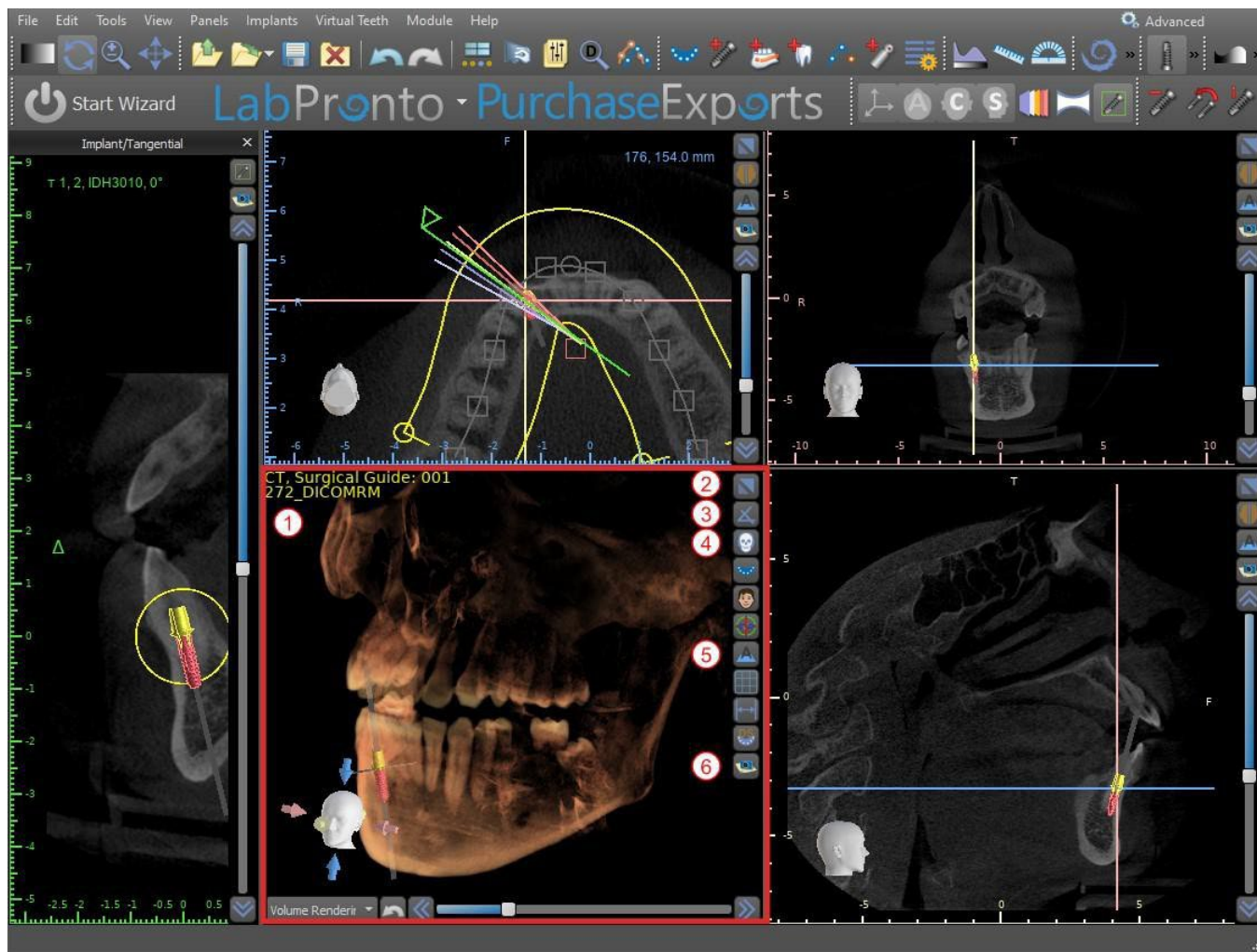

6.1.6 3D Shortcut buttons

The 3D view has several valuable shortcut buttons on the right side of the view.

The first button (2) maximizes the window.

The second button (3) allows the user to change the positioning of the view. The user can click on the button to open a fly out menu. The orientation of the patient's anatomy will adjust according to the option chosen.

The third button (4) allows the user to take advantage of different volume rendering options. It is possible to set the rendering type and quality using the contextual menu options or the Volume Rendering panel, which can be accessed from the menu under the "Panels" option.

The "Presets" option opens a selection of different 3D volume rendering presets with different coloring schemes.

In "Surfaces" mode, differential surfaces ("CT surfaces") are depicted (1) that are defined in the "Surfaces" panel, which can be accessed from the main menu under the "Panels" option or by double-clicking in the 3D window.

The user can assign a name to each density surface and set threshold values, the color, or the transparency.

In the "STL Surfaces" table located on the same panel (2), users can find created or imported models for which it is possible to set the color, visibility, and transparency.

The visibility of STL models, unlike that of CT surfaces, is not affected by the chosen volume rendering mode.

The fourth button (5) turns on the visibility of the slices in the 3D window.

The fifth button (6) inserts a current snapshot of the 3D window into the "Screenshots" panel.

6.1.7 Surface culling (3D cut)

A 3D visualization can be cropped with the Surface Culling function (3). In “Surfaces” mode, each CT surface is cropped independently of the others. The first non-deletable surface shares its cropping area with the other visualization modes (Shaded Bones, MIP, X-Ray, etc.).

Cropping serves to:

- remove noise
- remove superfluous anatomy
- isolate anatomy


The cropping tools are in the lower part of the “Surfaces” panel (3). It is possible to crop data with the “Cut” button, to delete a previously performed crop with the “Clear” button, and to isolate the selected anatomic portion. It is also possible to invert the current cropping.

After the user clicks the “Cut” button, the application switches to drawing mode. At this point, by clicking on the left mouse button in the 3D window and dragging the cursor, it is possible to draw an area which is cut out of the selected surface after when the mouse button is released. Drawing can also be invoked at any point by pressing the “Shift” key and holding it down during the whole drawing period.

If we want to isolate a particular part of the anatomy, we must first ensure that it is not connected to any other parts. We can ensure this using the cropping function described above. We can isolate anatomy in this way with the “Isolate” button and by clicking on it in the 3D window. All parts of the anatomy are then removed except for the selected part.

As soon as we have a cropped part of the anatomy that we want to display, it is possible to invert the cropping with the Invert check box.

Any cropping actions can be reversed using the “Undo” function.

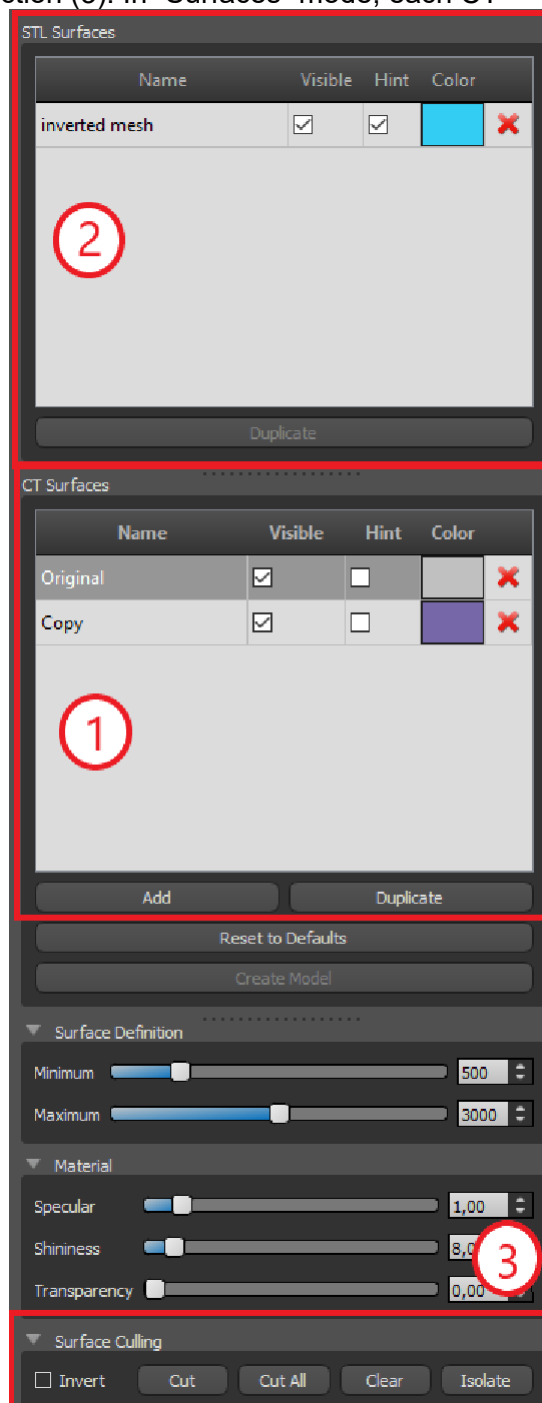


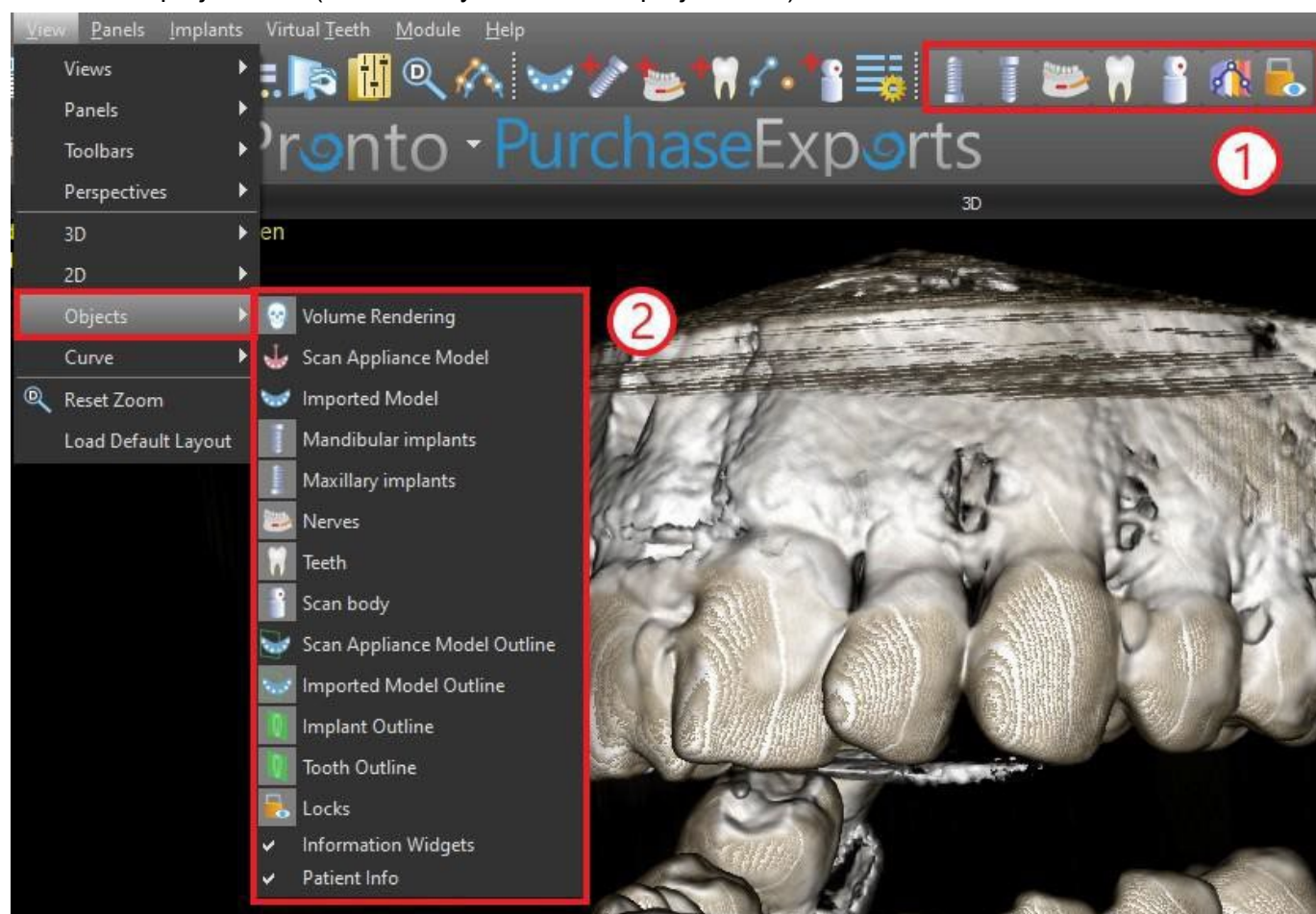
6.1.8 Indicators

The indicators are reference lines that indicate the positioning between the images shown on the screen.

Usually the indicators are already enabled in the windows of 2D images. To disable them, click “View > 2D” in the menu and choose the indicator to be disabled.


It is also possible to enable the indicators on the 3D images. Click “View > 3D” in the menu and choose the projection type to be enabled on the 3D image.

To change the visibility of objects, such as implants or teeth, it is possible to use the associated panels, the buttons in the icon bar (1), or the “View > Objects” option in the menu bar (2). After switching the visibility of the dental curve and slice indicators in the 2D views, it is now possible to use the “All projections” (alternatively the "Hide all projections") icon  in the icon bar.



6.2 Multi-planer screen


To access the multi-planar screen, click the “View > Perspectives > MPR” menu option. The multi-planar screen makes it possible to view coronal, axial, sagittal, and 3D images. This screen is generally used for applications such as oral and maxillofacial lesions and defects, removal of impacted teeth, and treatment of temporomandibular joint pathology.

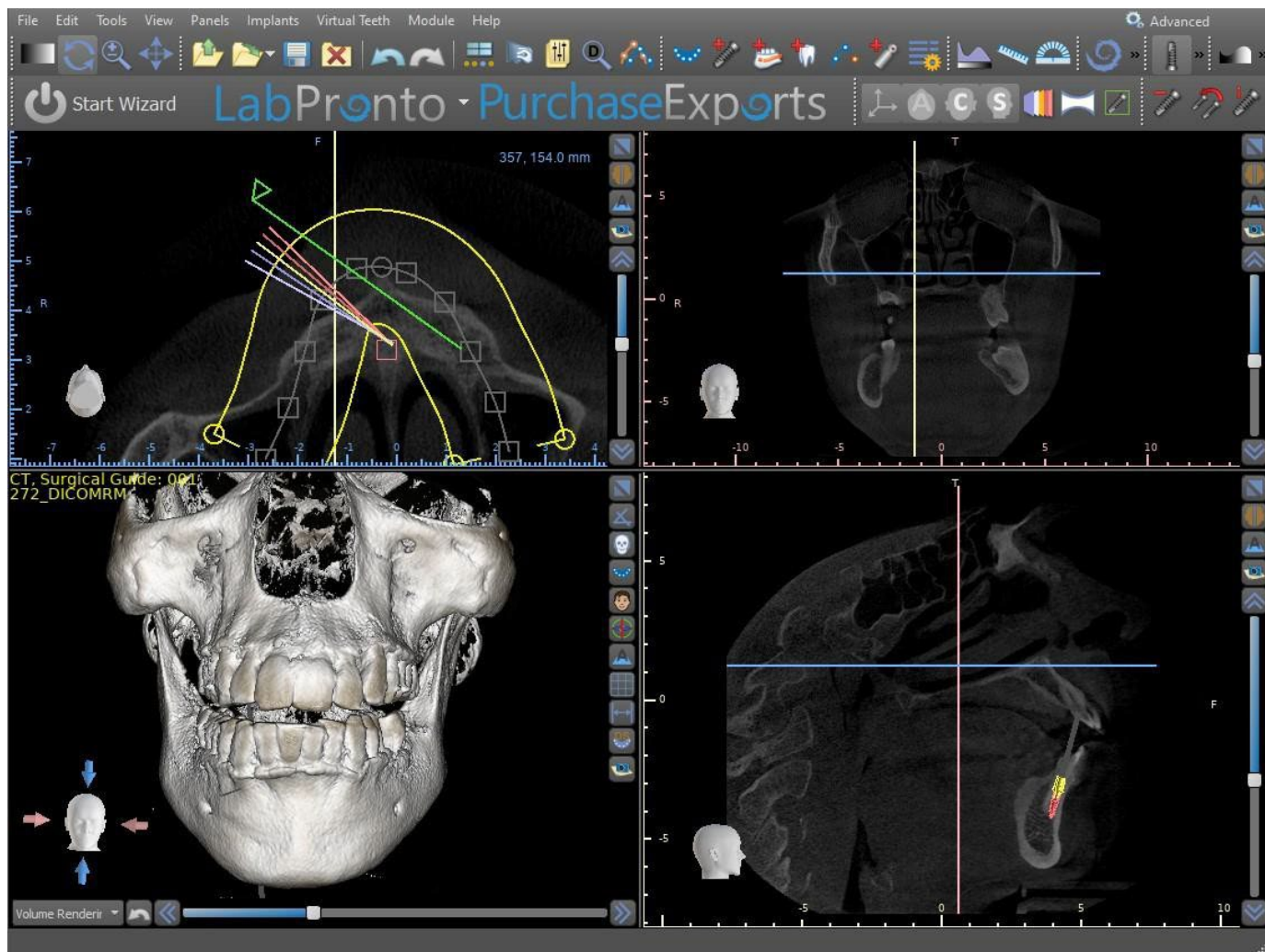

To locate the desired image, move the slide bar located next to the windows. Indicator lines show the exact image position of one window in relation to the images in the other windows.

6.2.1 Sagittal window

The sagittal image is a vertical plane that divides the skull into a right and left side. It is located in the bottom left window of the software (1) and is marked in yellow. When the user moves the scrollbar of the sagittal window (2), the entire sequence of images in this group will be shown. The indicator line of the sagittal slice appears in the coronal (3), axial (4), and 3D images.


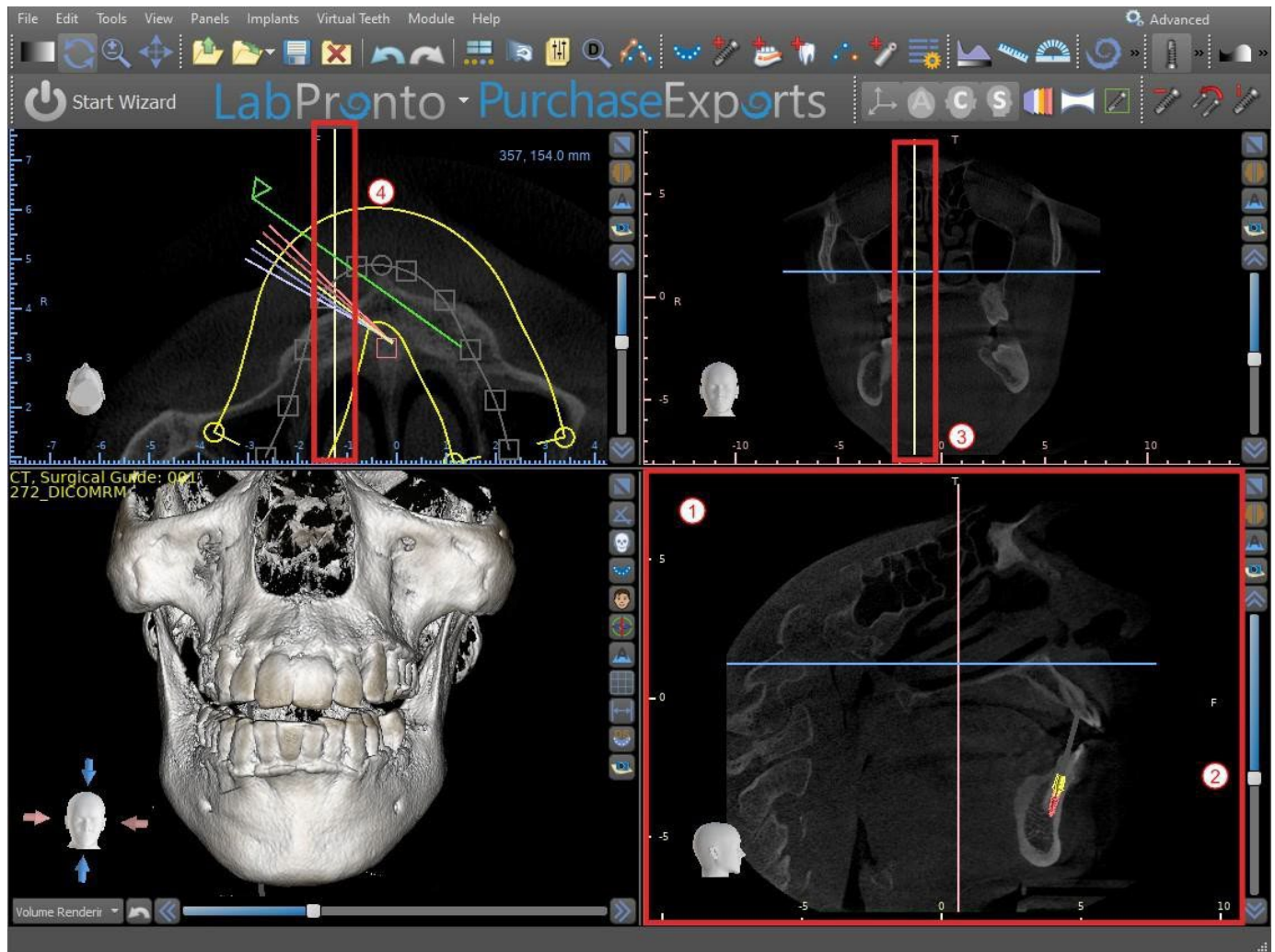


6.2.2 Coronal window


The coronal image is a vertical plane that divides the skull into a front and back part. This image is located in the top right-hand window of the application and is marked in pink. When the user moves the scrollbar in the coronal window, the entire sequence of images in this group will be shown. The indicator line of the coronal slice appears in the axial, sagittal, and 3D images.

6.3 3D Mouse Controls


BSP allows the user to use a 3D mouse. In order to use it, please download the software from this web page <https://3dconnexion.com/drivers/>.

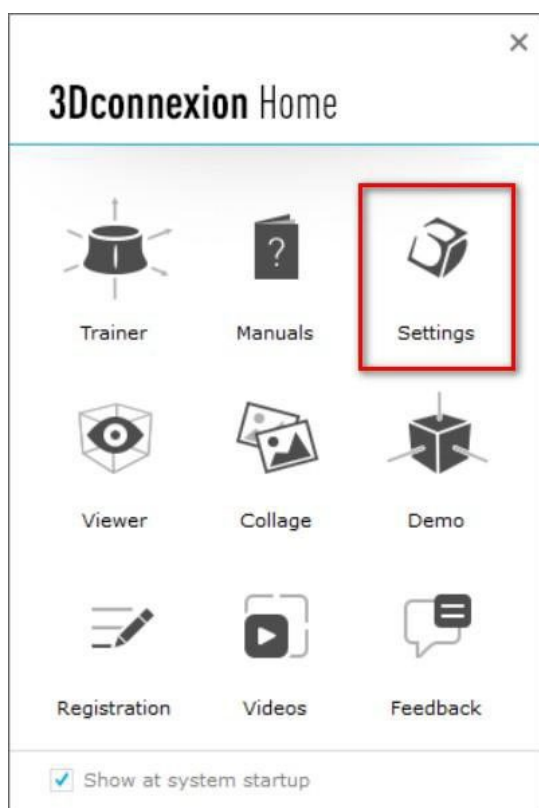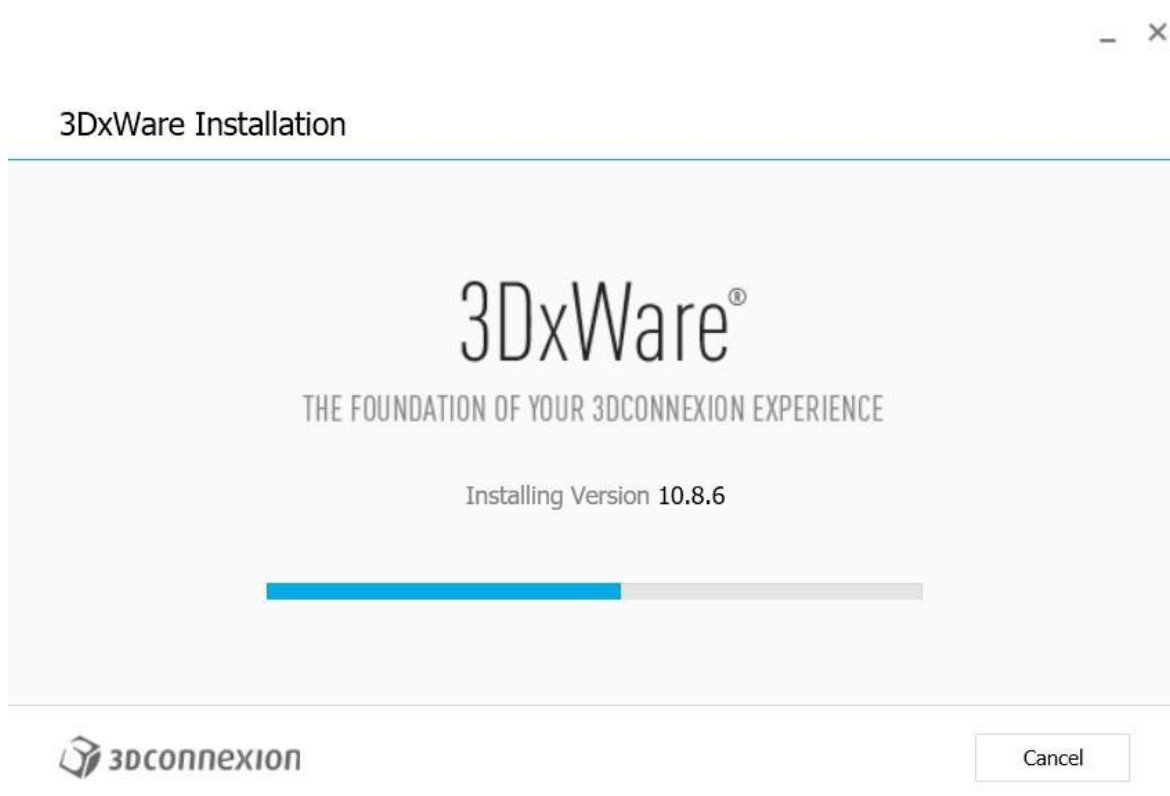


After installation, open the 3Dconnexion Settings and click on **Buttons** (2) to open the buttons menu. Make sure you have set **Blue Sky Plan** (1) for 3D mouse settings.

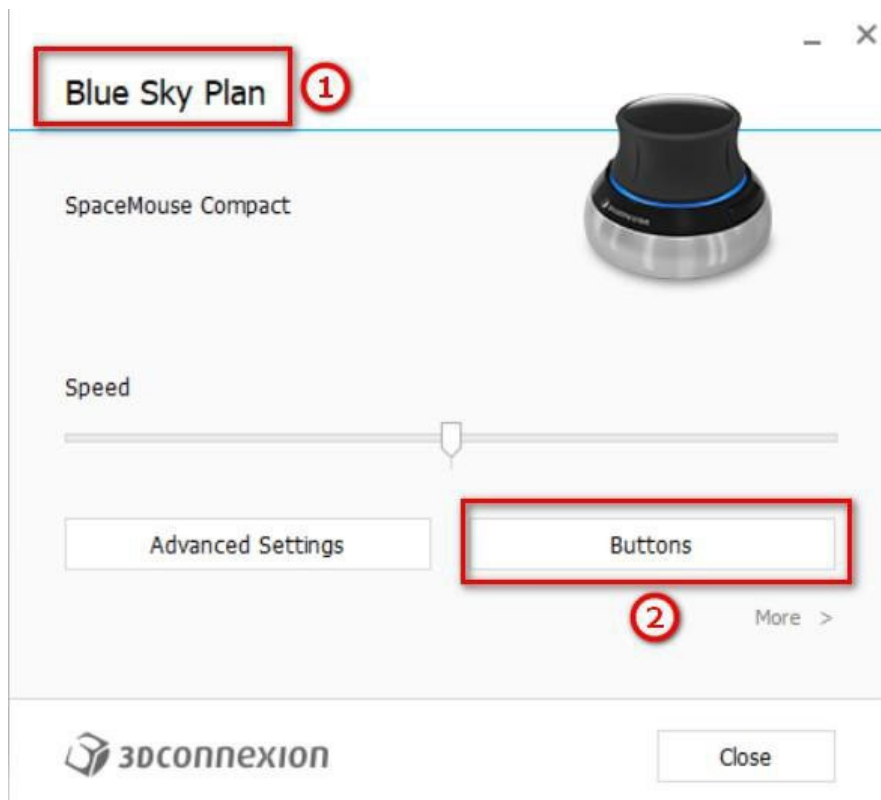


In the buttons menu for Blue Sky Plan, set the left and right mouse buttons (3) for the controls. Blue Sky Plan allows the user to set the controls (4) for: File, Objects, Orthodontics, Panels and Perspectives.

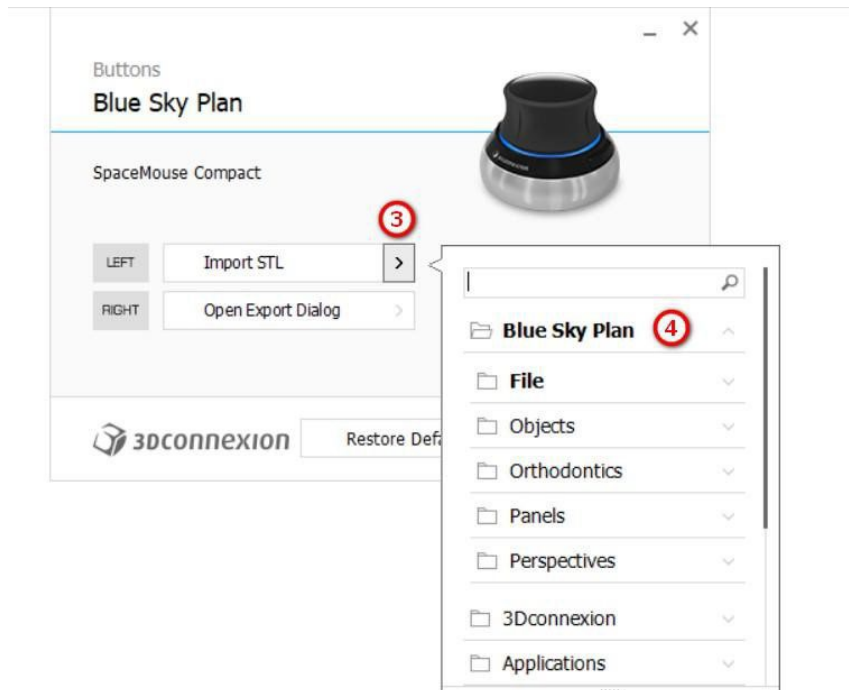

The user is able to set basic functions such as undo/redo, copy/paste or camera controls. You can also set BSP functions which control the program's UI.

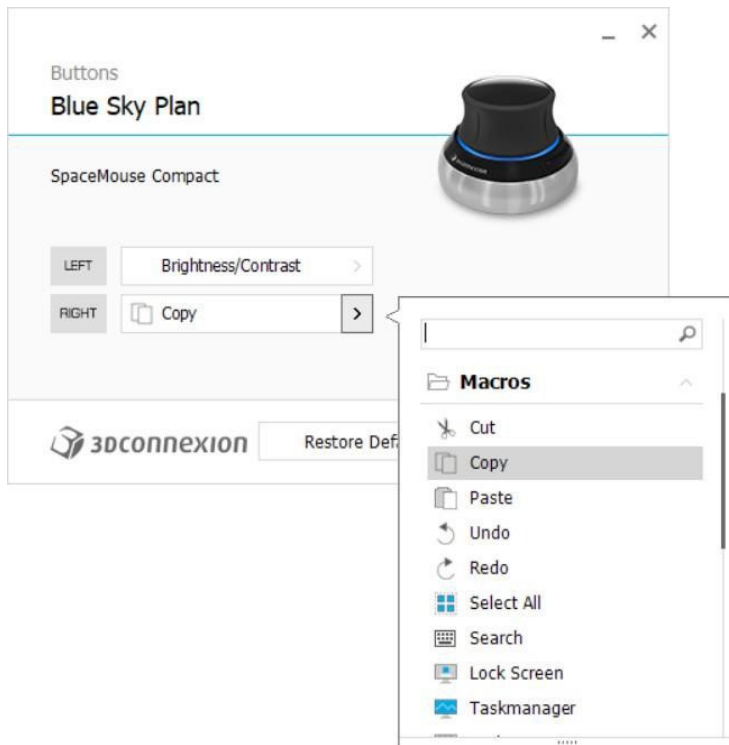

The controls (5) for the 3D mouse for Blue Sky Plan have now been set.

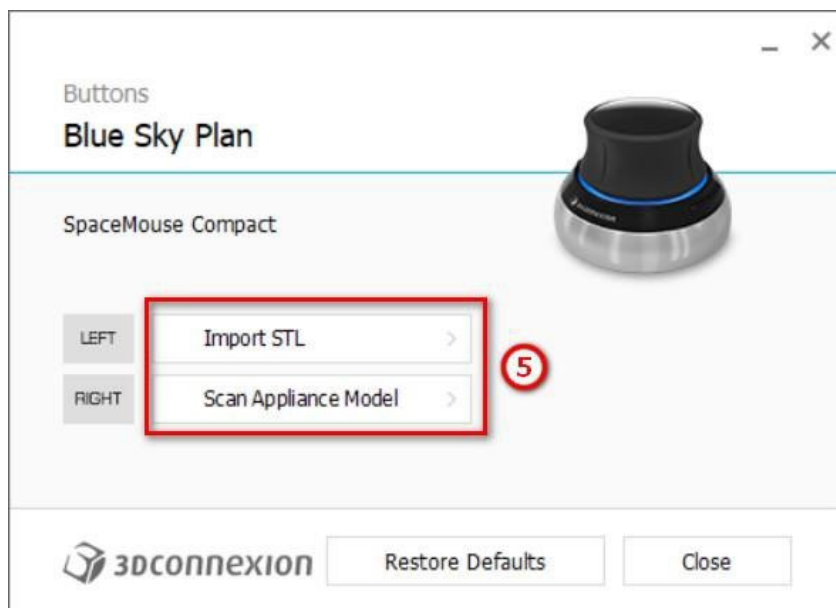

All options can be restored by clicking on the "Restore Defaults" button. For more advanced settings, click on the "Advanced Settings" button.

7 CT Dicom Viewer


CT DICOM Viewer simplifies working with CT Data, especially for reviewing CT scans. To start a new project, click on the CT DICOM Viewer button (1) and choose a file containing DICOM data (2). The next steps involve selecting one of the DICOM series (3) and defining the volume of interest and imaging mode (4).



CT DICOM Viewer can be set as the “shell open module” in Preferences -> Advanced (i.e. the default module in which data is opened “when double-clicking”).


The CT DICOM Viewer is based on the Model Editing module, the main differences between the two are:

- Projects saved in the CT DICOM Viewer can be opened in the Model Editing module
- UI changes done in the CT DICOM Viewer are not saved (toolbars, panels...)
- The “Import Additional Surface” dialog doesn’t appear in the CT DICOM Viewer
- Only the Main Toolbar, Measurements Toolbar, and Messages Toolbar are activated by default
- Brightness/Contrast, Ortho Slices, Volume Rendering, Surfaces, Notes List, Nerve List, and Screenshots panels are available by default

To work with additional tools (chapter 7), select the Model Editing or Surgical Guides (chapter 15) modules from the Blue Sky Launcher or switch to the Normal/Advanced module.

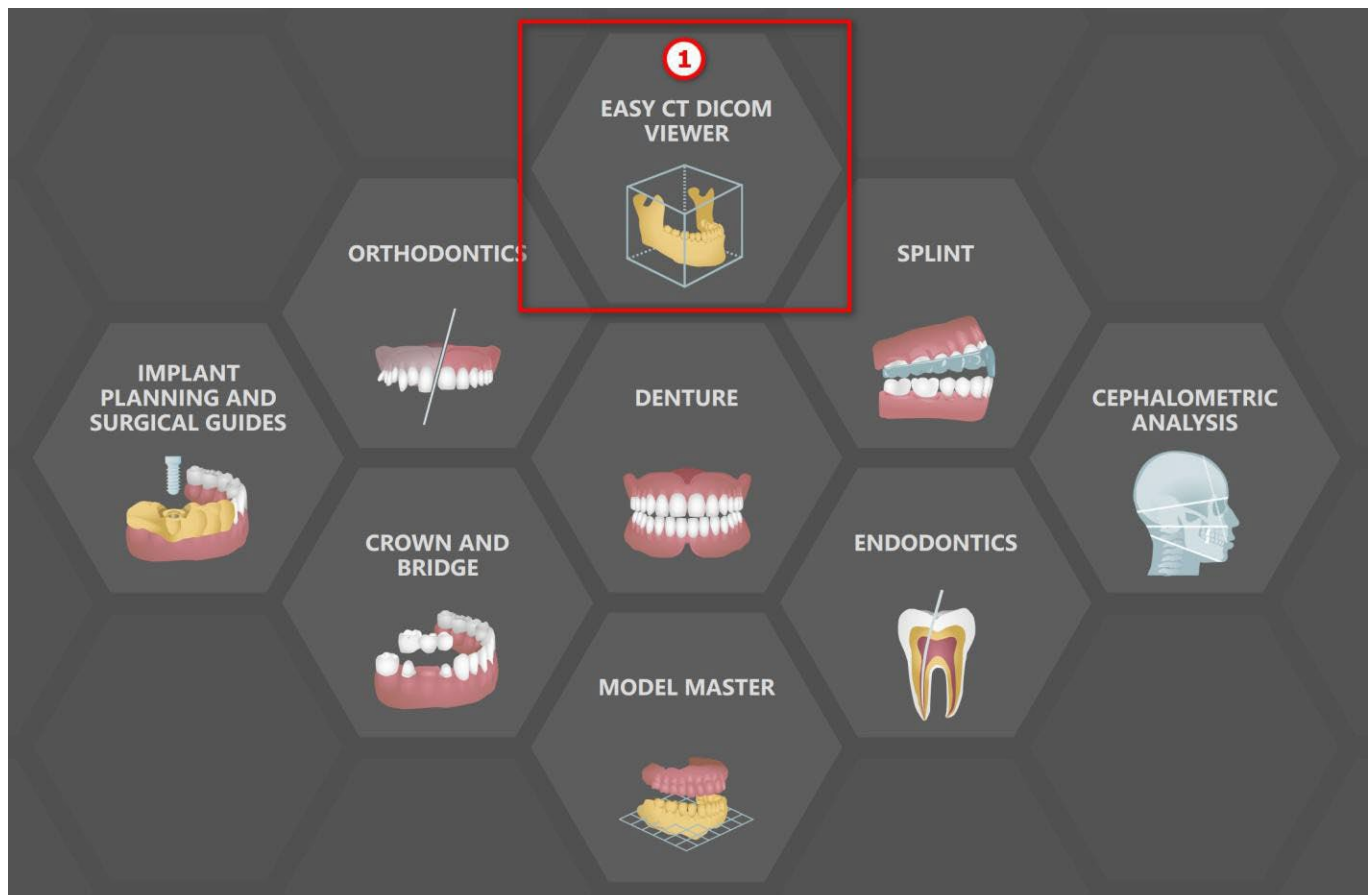

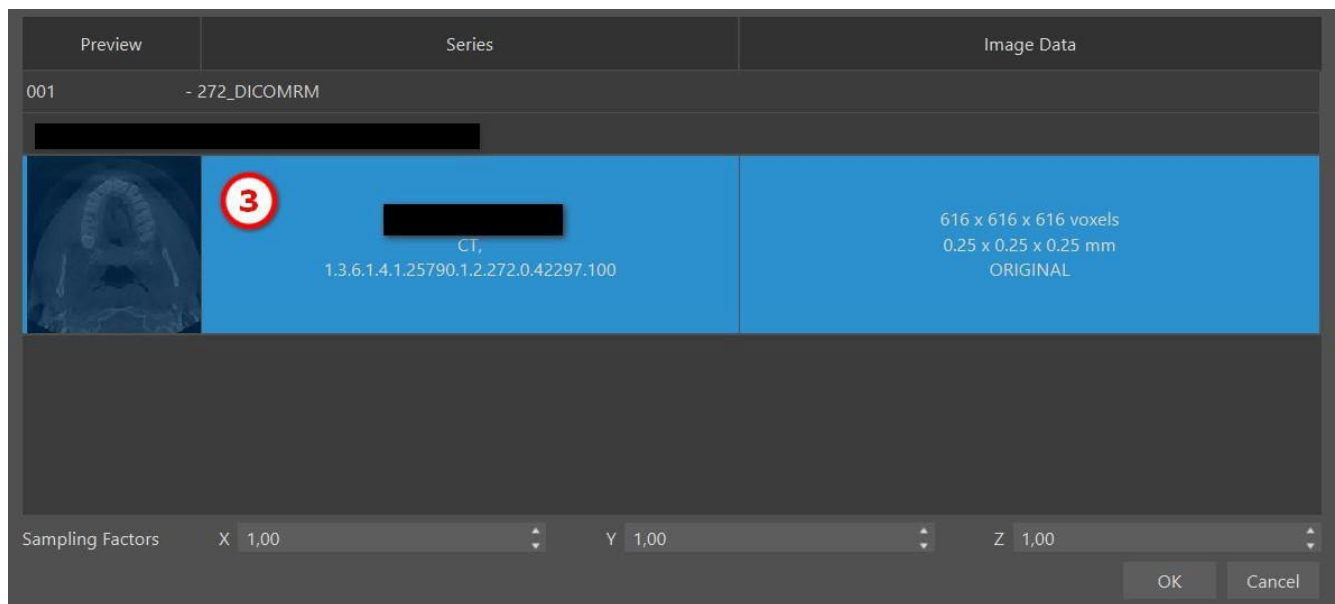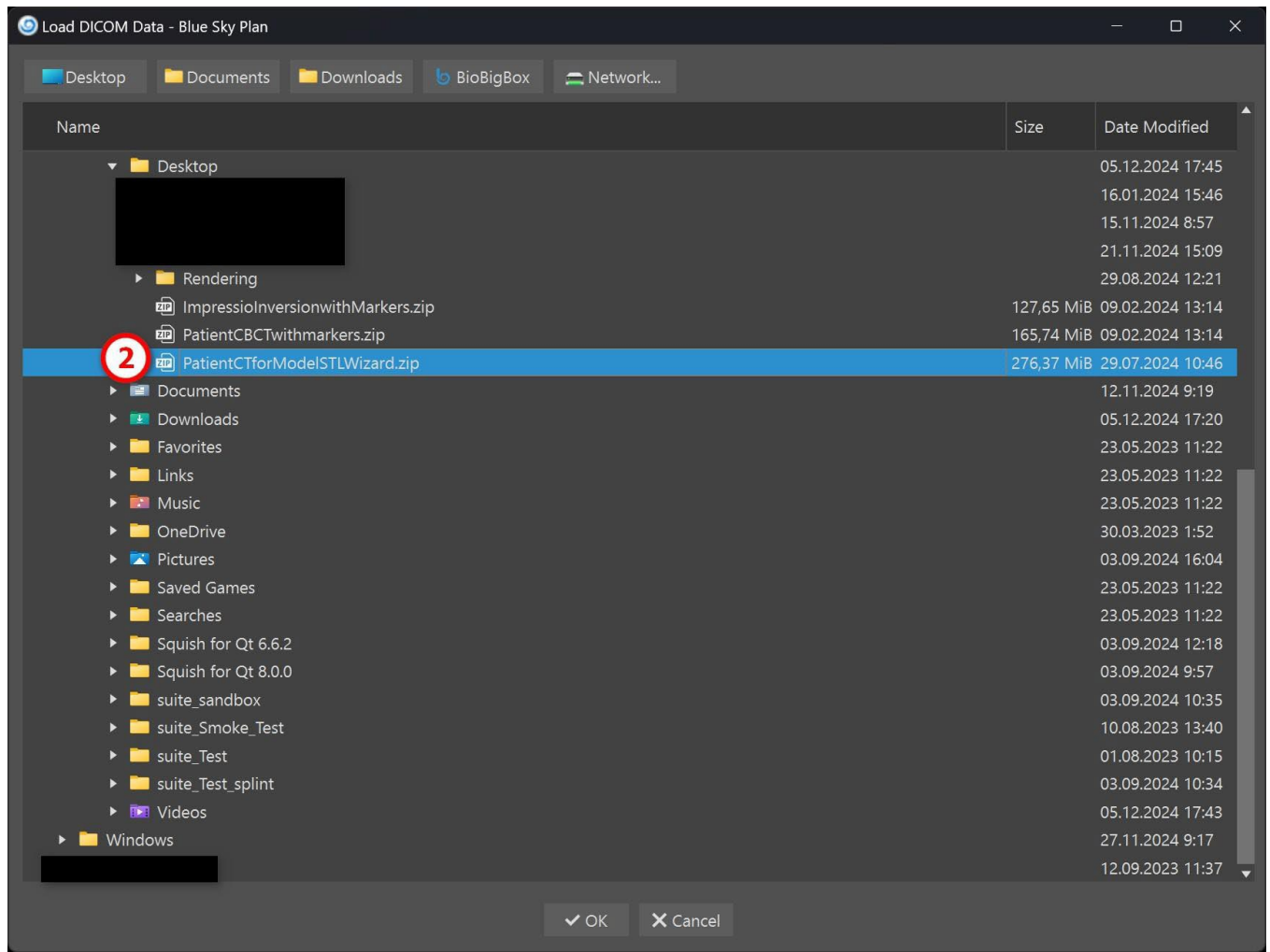

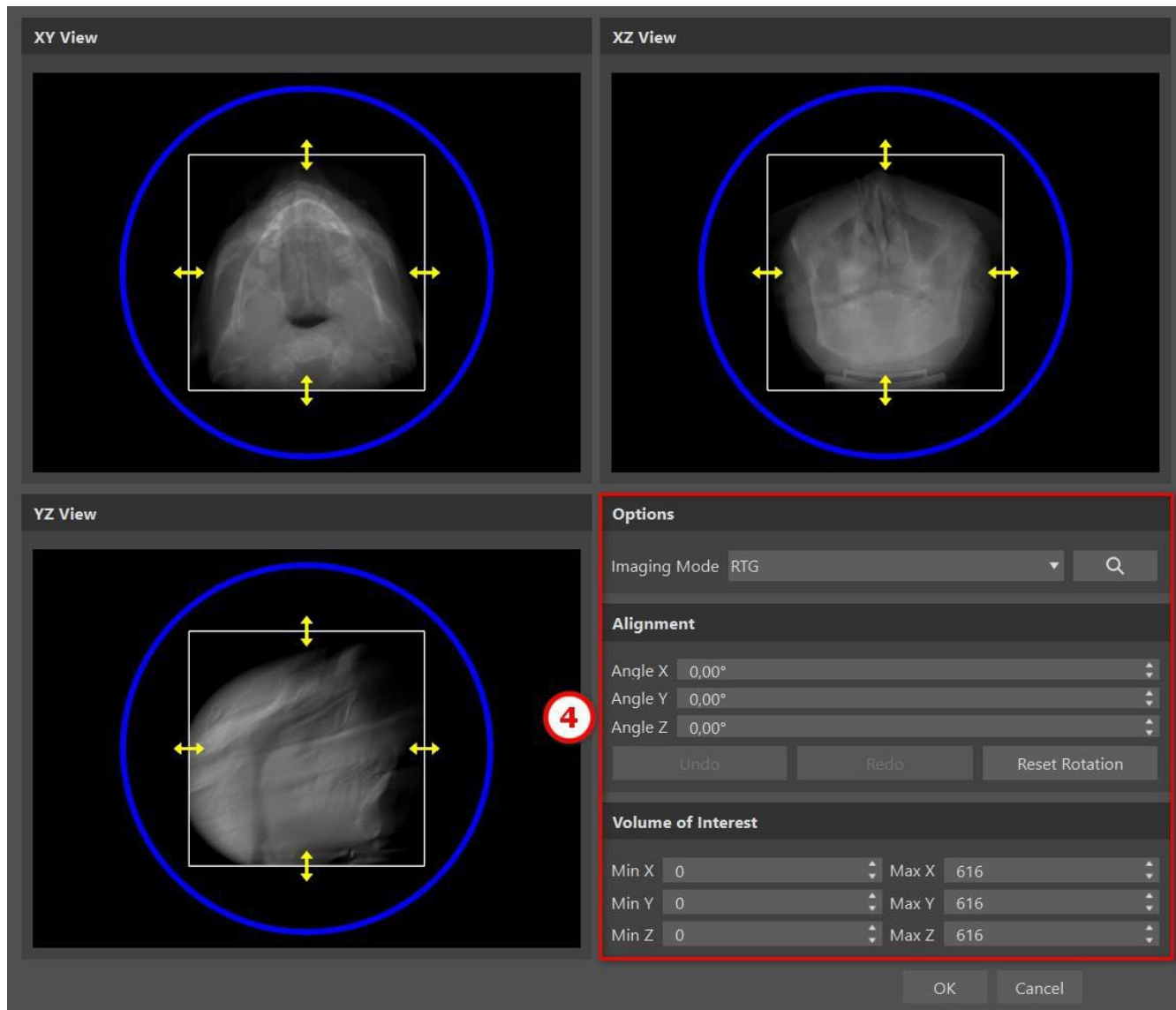


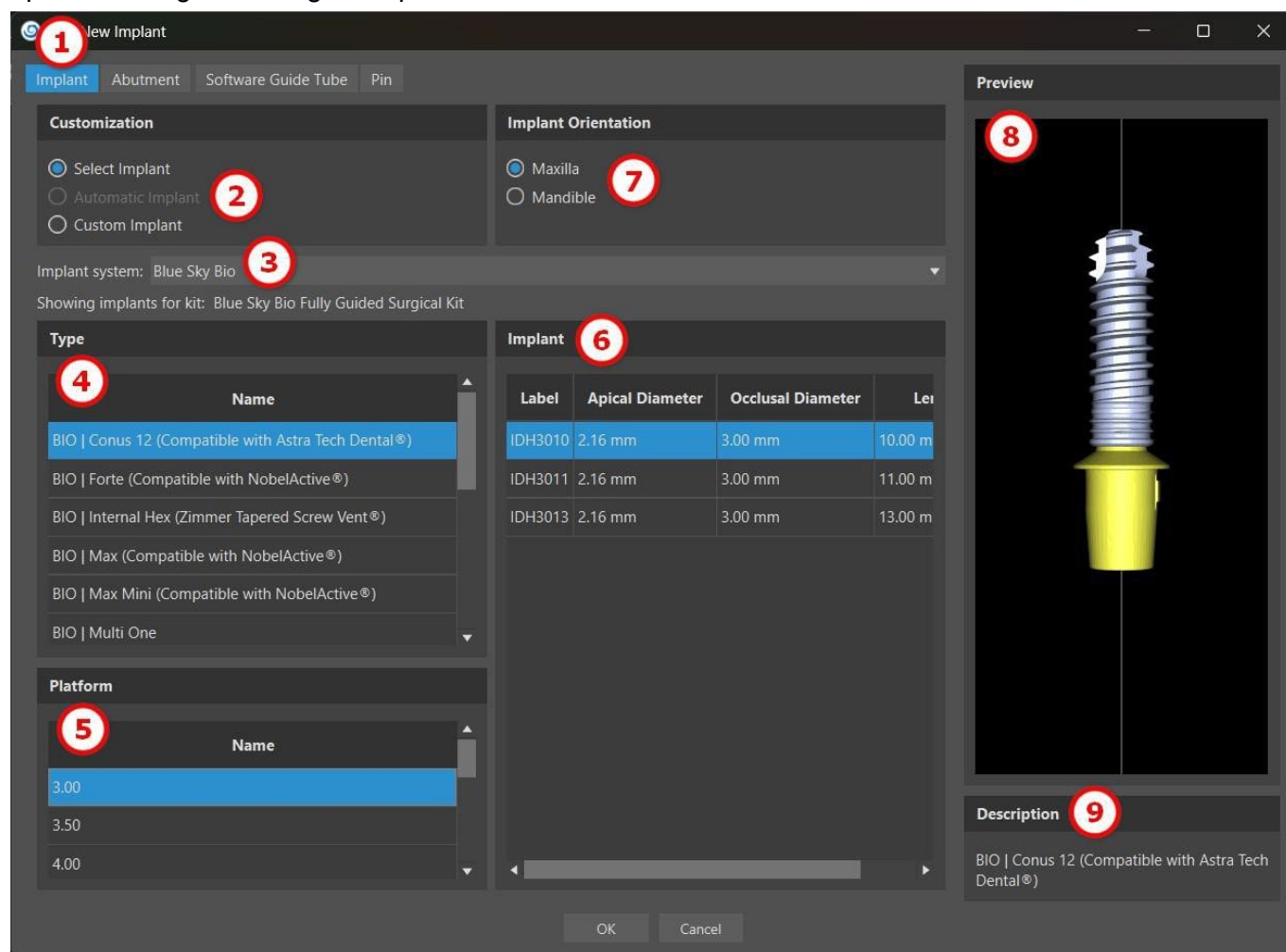
8 Placing implants and pins

Blue Sky Plan® allows virtual implant planning via the patient's CT or CBCT images. It is possible to add, exchange, move, and rotate the implant and abutment or pin in the different views.

8.1 Adding an implant or pin

8.1.1 Implant from the catalogue


Before positioning the implant, it is recommended to first measure the dimensions of the bone. In this way it is possible to estimate what dimensions the implant should have. Clicking on the  icon in the toolbar opens a dialog for adding an implant.




In this dialog it is possible to modify the implant parameters, abutment, and drill guide. The corresponding parts can be selected in the individual tabs (1). "Select Implant" (2) is selected by default, which makes it possible to select an implant from the catalog (3) on the basis of its type (4) and platform (5) and then according to its parameters (6). It is also possible to specify the orientation of the implant (7). The currently selected implant can be viewed in the preview (8). A short description of the implant (9) is also located in the lower part of the dialog.

8.1.2 Automatic implant from the catalog


The automatic implant is added via the “Add Tooth” function in the Normal or Advanced module. By default, the “Automatically create AI Tooth” (1) tab is selected. Then choose the associated tooth from the preview (2) - the text description of the crown appears in (3). Then use the “Implant” checkbox (4) to enable the field with the implant parameters. By clicking the implant field (5), the “Add New Implant” dialog appears.

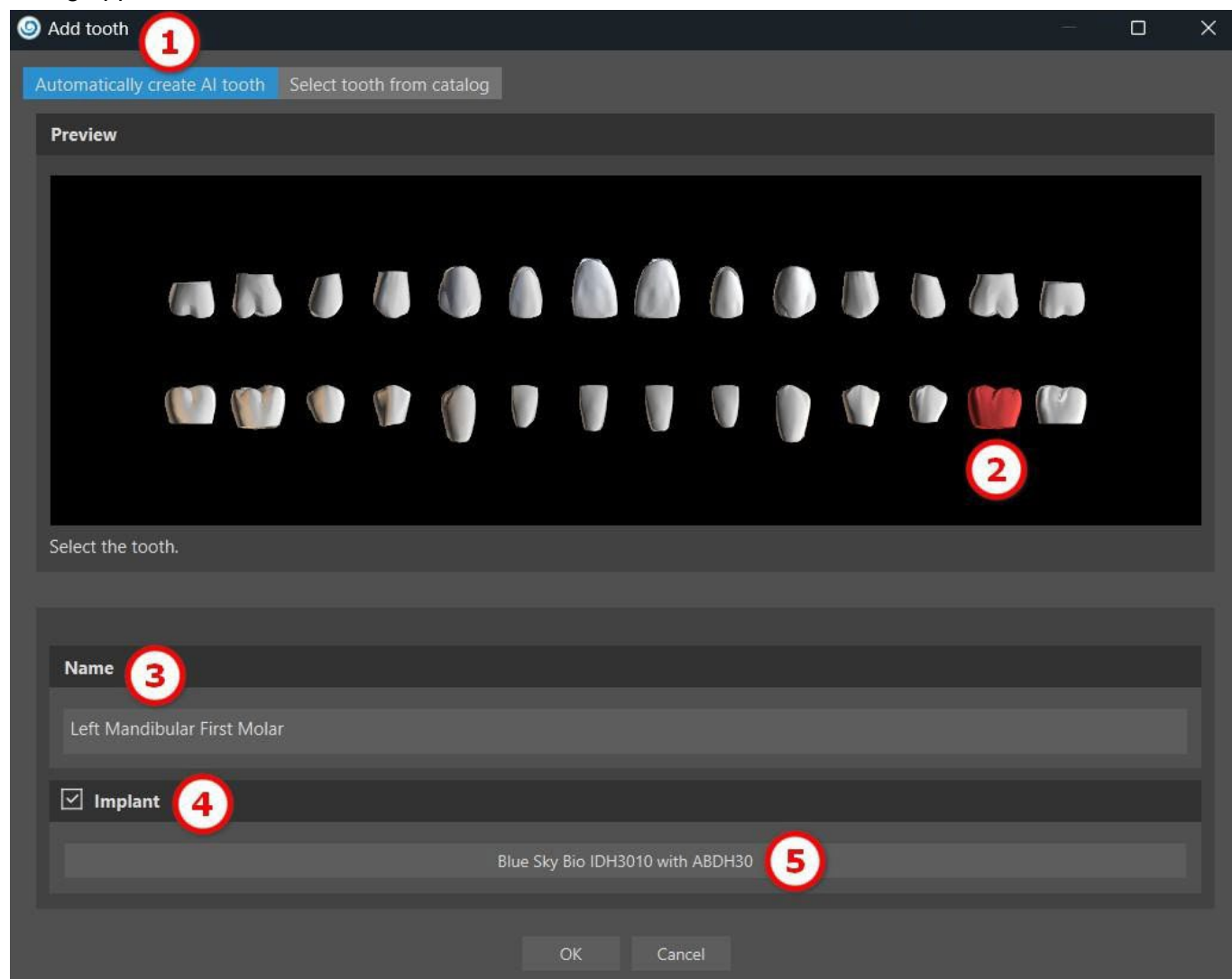


In this dialog, select the “Automatic Implant” (6) option which automatically selects a suitable implant from the selected manufacturer. It is possible to select an implant system from the catalog (7) by clicking the dropdown menu (note: it is important to have the relevant implant catalog package downloaded). The automatic implant option allows the “Implant Type” (8) to be chosen. The specific platform and implant (10) will be chosen automatically. “Implant Orientation” (9) cannot be changed because the implant is linked to the tooth that is selected in the "Add Tooth" dialog. To add an abutment to the automatic implant, switch to the "Abutment" tab (11). Automatic implant is selected by clicking the “OK” (12) button.

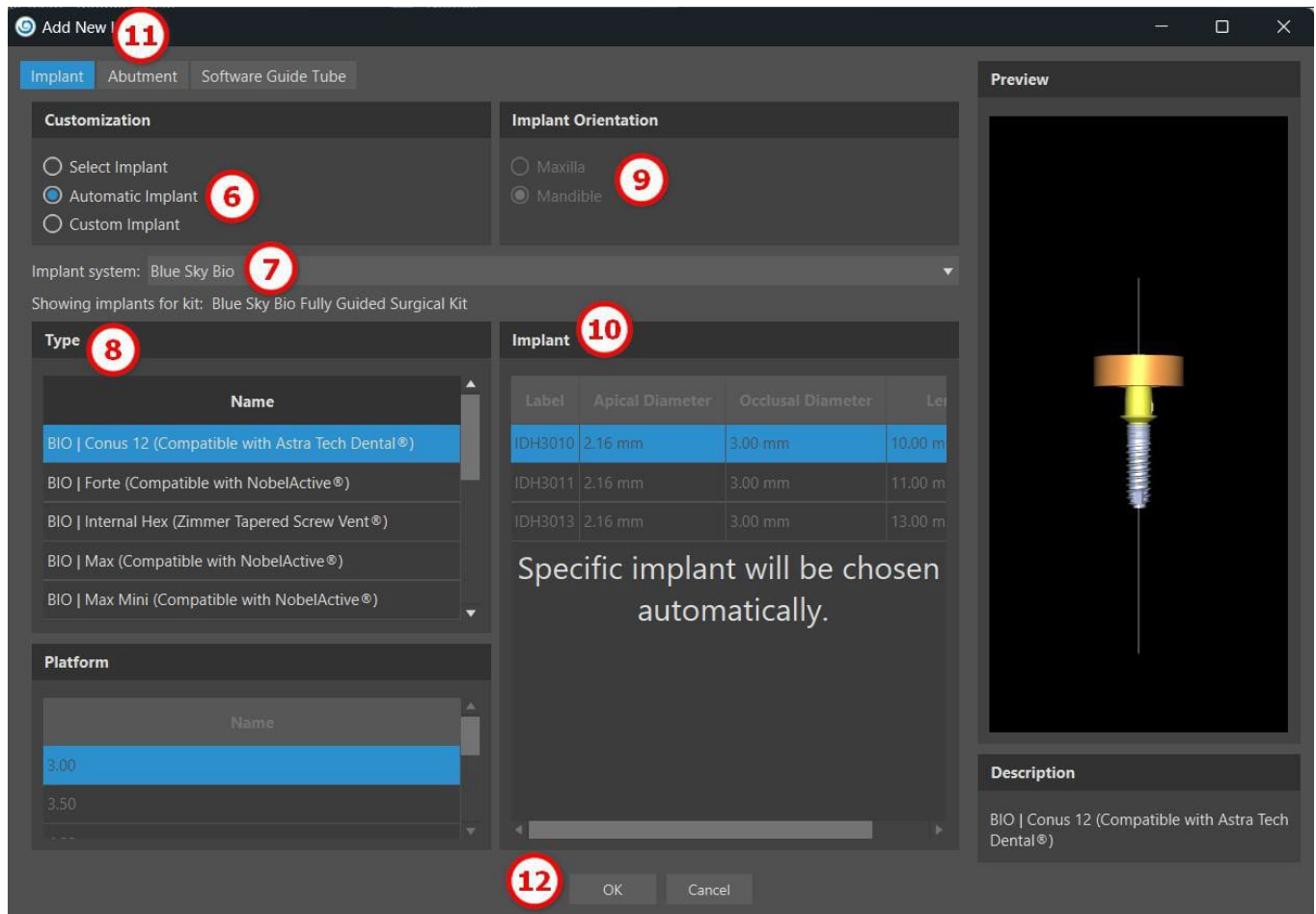


"Custom Abutment" allows the individual parameters of the abutment to be changed. The size of the last implant used is display by default. By switching to the "Implant" tab for an automatic implant, the custom length and diameter will be set to 3x20 mm.

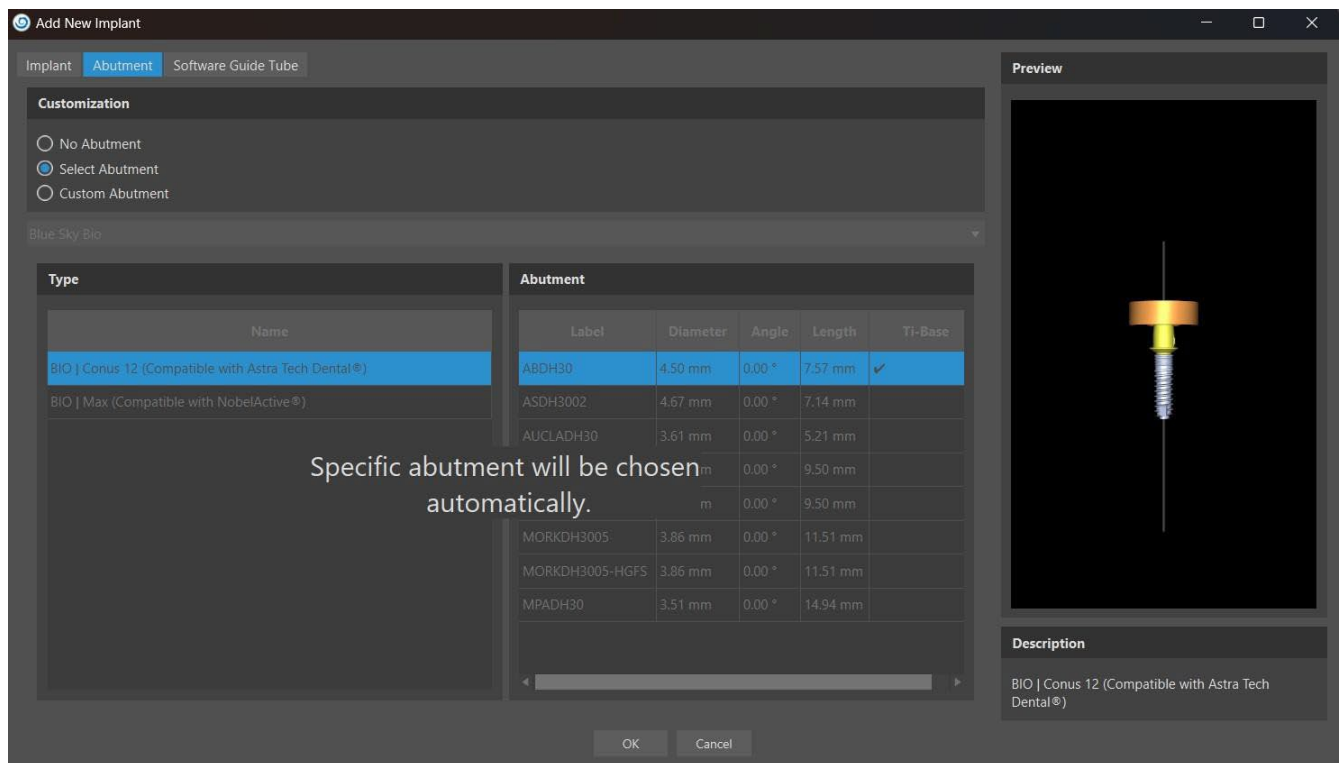

It is possible to see what kind of implant and abutment has been selected in the implant field (13). An automatic implant with an AI tooth will be added by clicking the “OK” (14) button.

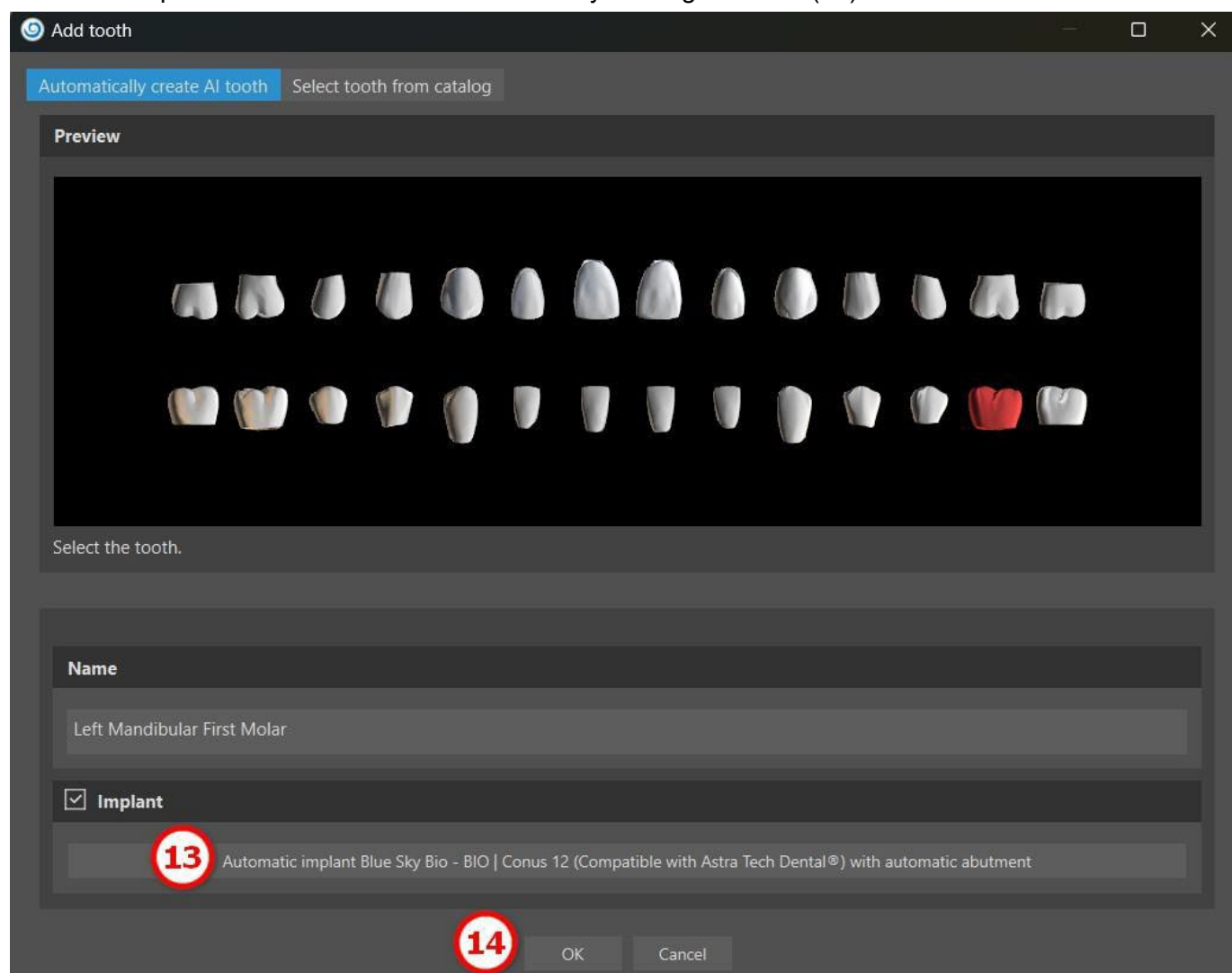


For automatic implant placement, it is necessary to introduce two control values, both found in Preferences > Surgical Guide. These values help set limitations and custom preferences for implant placement. The first control value, "Subcrestal Depth," measures the distance from the edge of the bone to the occlusal center of the implant's vertical axis. It is useful when the bone has an uneven crest shape. This allows the implant to be placed deeper into the bone. If the implant is not deep enough, it might end up too thin due to the bone crest shape restricting it. In such cases, the "Subcrestal Depth" can be increased to allow the implant to be placed deeper, thereby accommodating wider implants. The second control value, "Max Implant Width", is used to limit the width of the implant, ensuring it does not become too thick.

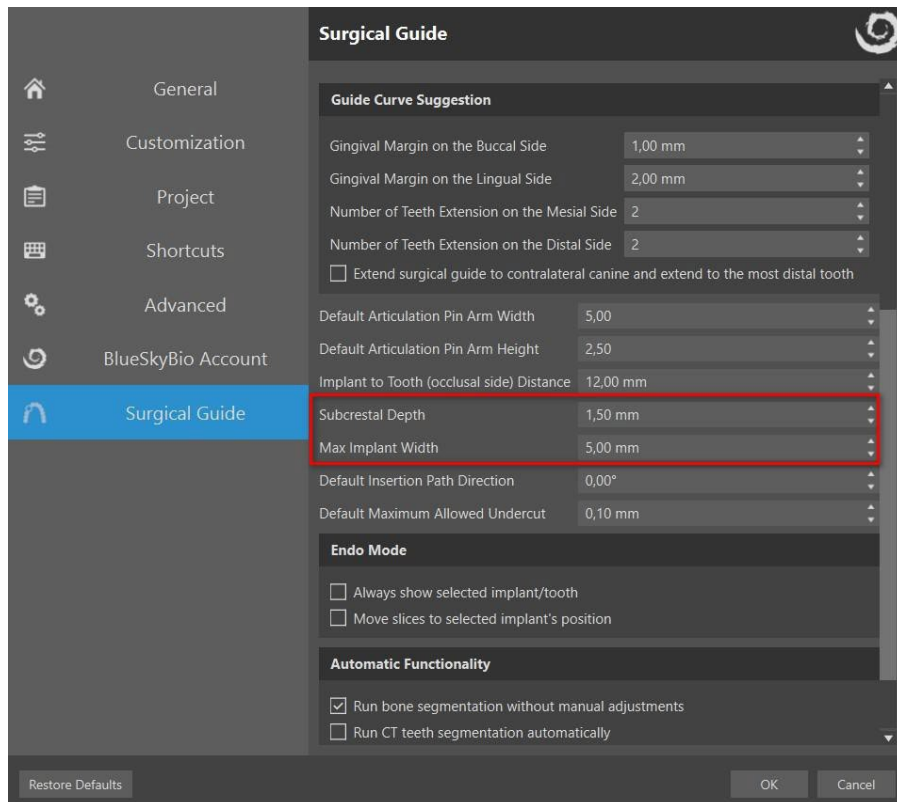

Note: Automatic implants are only available when adding an AI tooth and if CT data are imported.


8.1.3 Custom implant

If none of the implants fulfills the particular requirements, it is possible to create a simplified but fully parameterized implant. This is achieved by clicking on the "Custom Implant" option (1). The individual parameters (2) can be modified with the corresponding fields. These changes appear immediately in the preview.

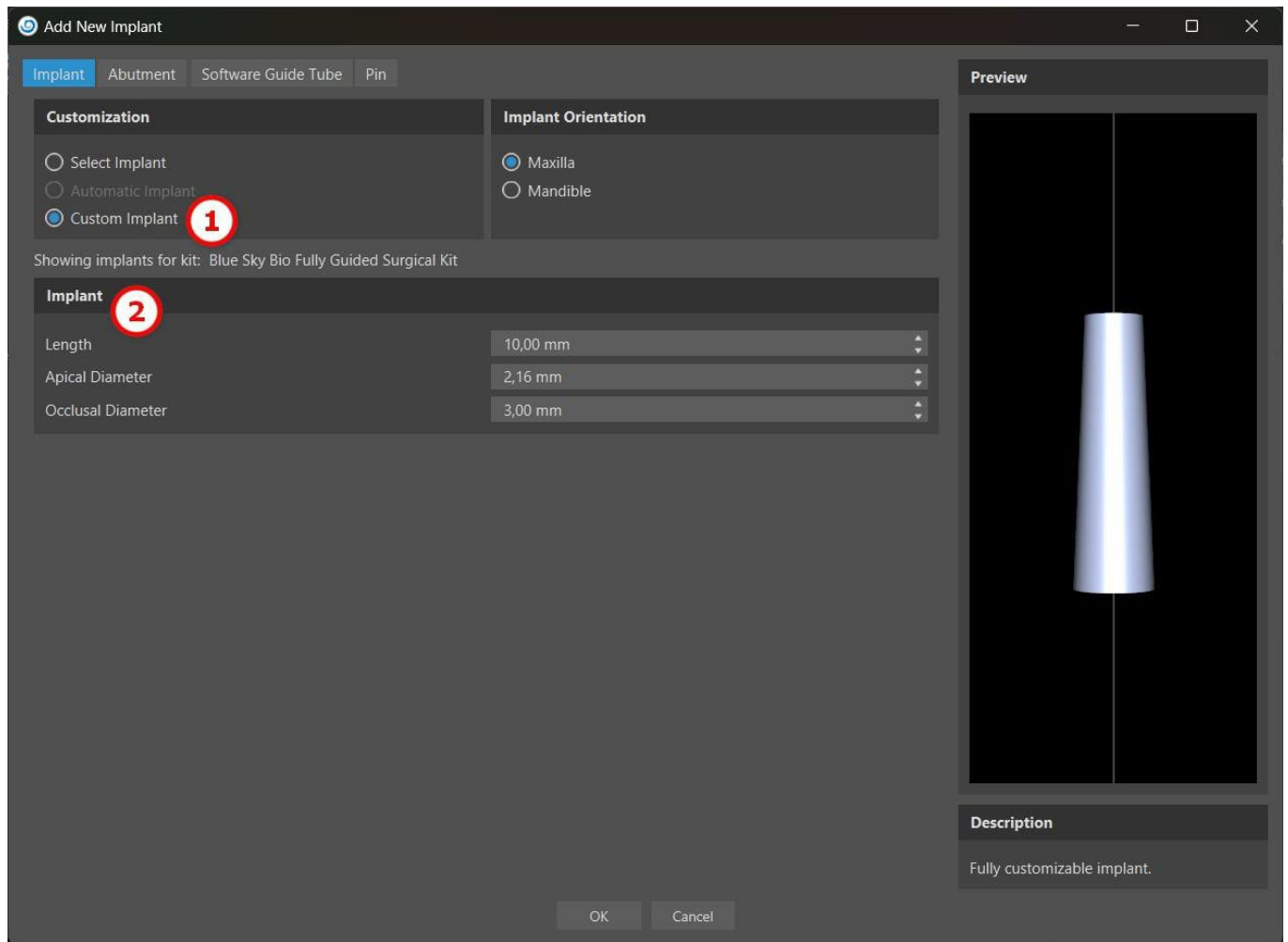


8.1.4 Abutment from the catalogue

By default, the implant is loaded without an abutment. If we want to add an abutment, it is necessary to first switch to the "Abutment" tab (1) and then select the "Select Abutment" option (2). A list of abutments that are compatible with the selected implant appears in the table (3). Once we have chosen a parameterized implant, a table for selecting the type and platform will be displayed here similarly as for the implant.

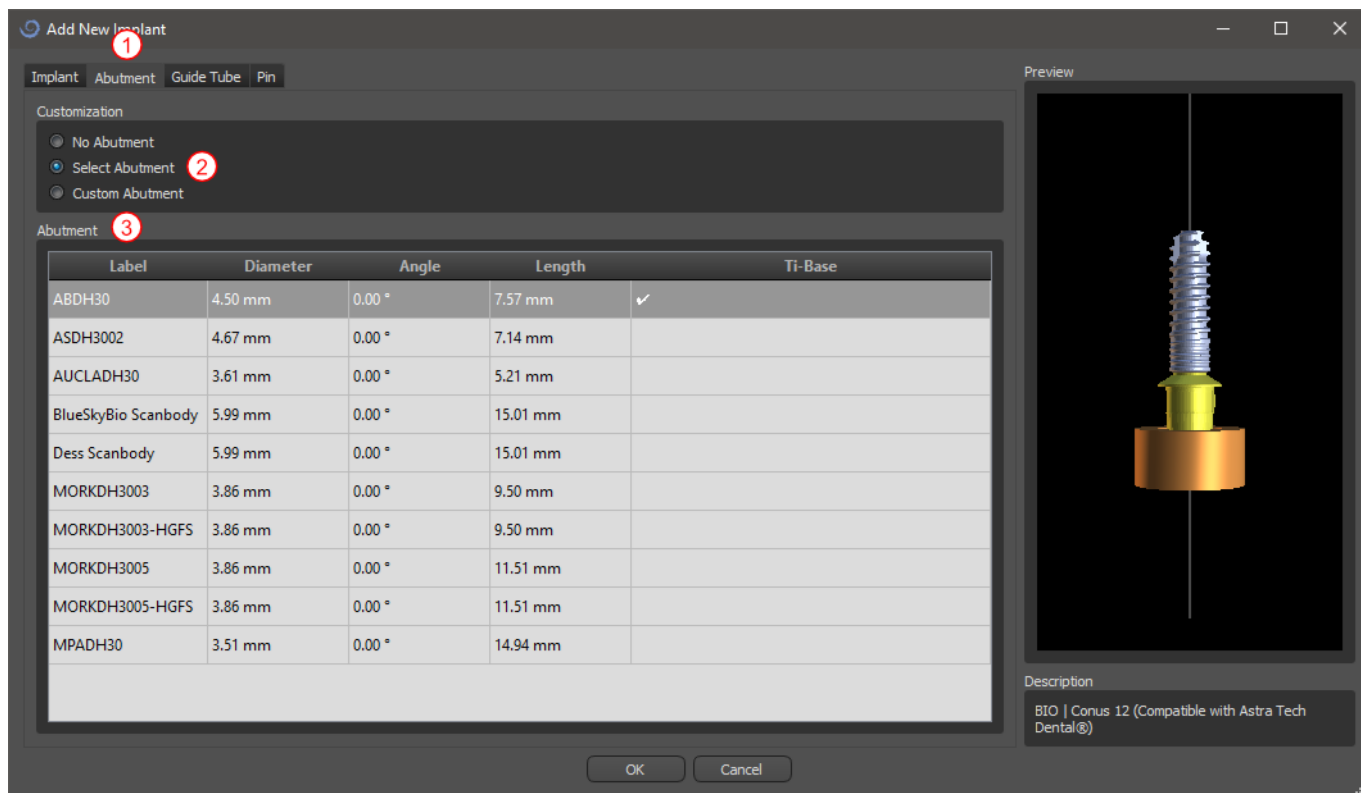


8.1.5 Custom abutment

The abutment can also be fully parameterized in the same way as the implant. We first select “Custom Abutment” (1); it is then possible to set the individual parameters (2).

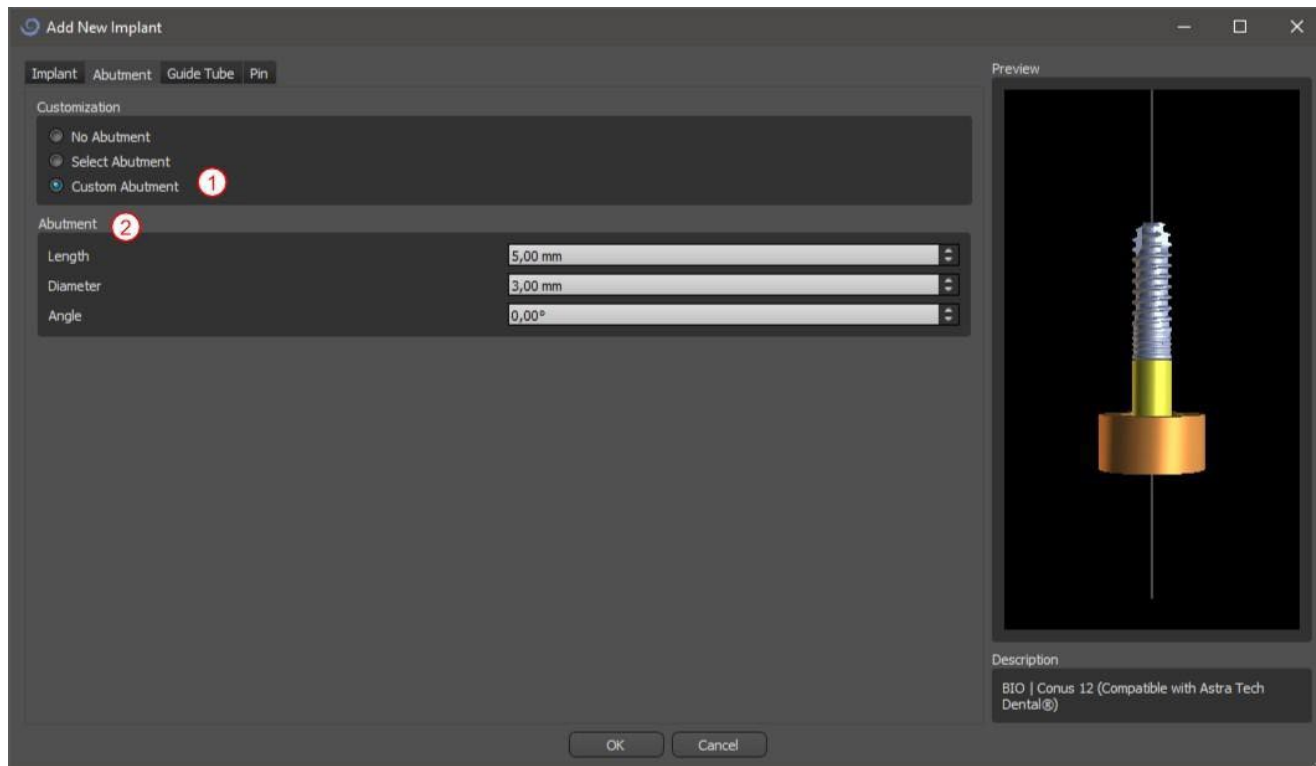


8.1.6 Software Guide Tube

The implant has an auxiliary drill guide added by default. If we want to modify it, it is necessary to switch to the “Software Guide Tube” (1) tab. Its parameters can be changed in the corresponding fields (2). If we want to display it or hide it in the preview, we can use the check box (3).

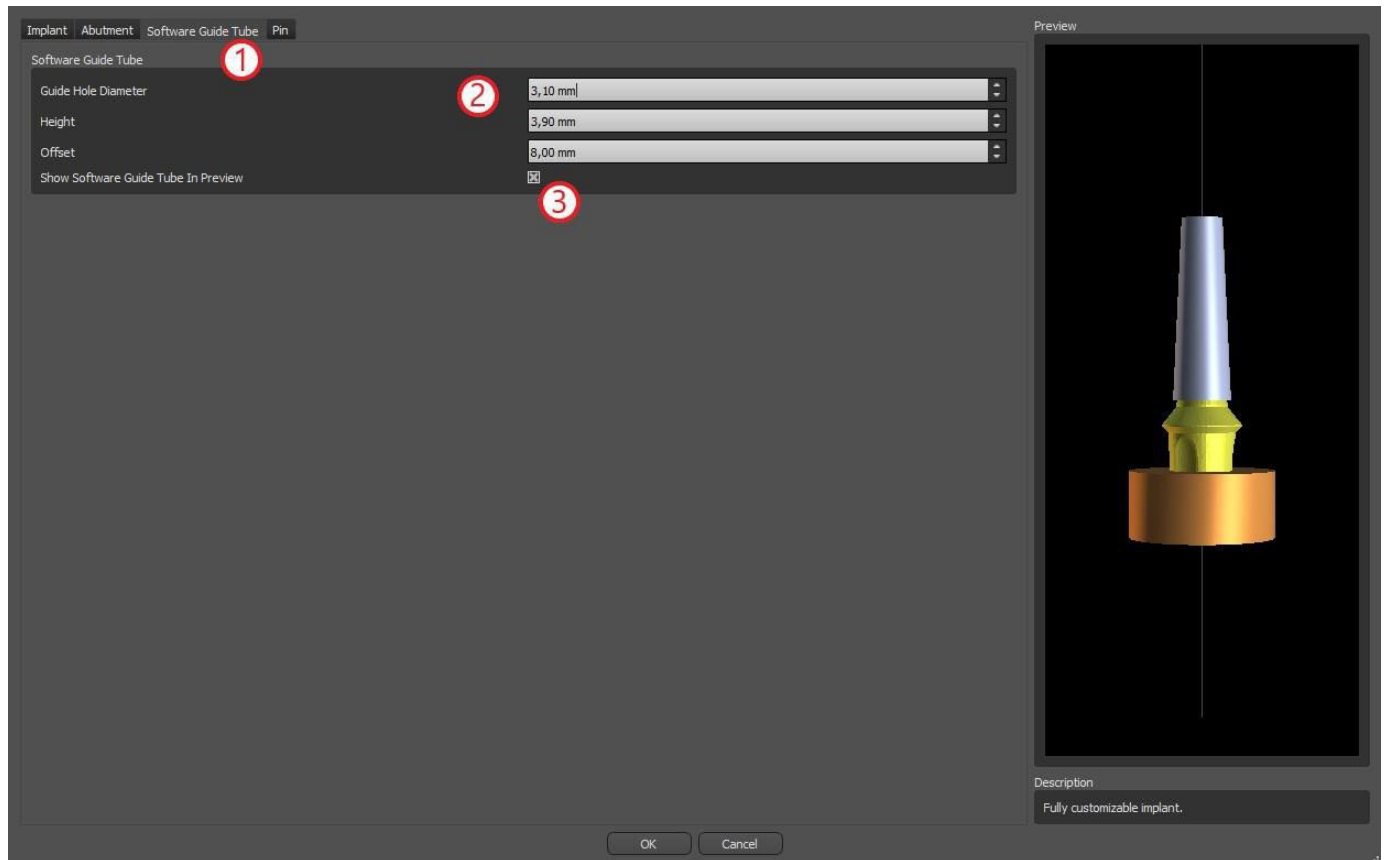


8.1.7 Pin from the catalogue

Instead of an implant, it is possible to add a pin (1). This is a special implant for positioning a template more firmly. In contrast to an implant, it cannot have an abutment but only a drill guide.

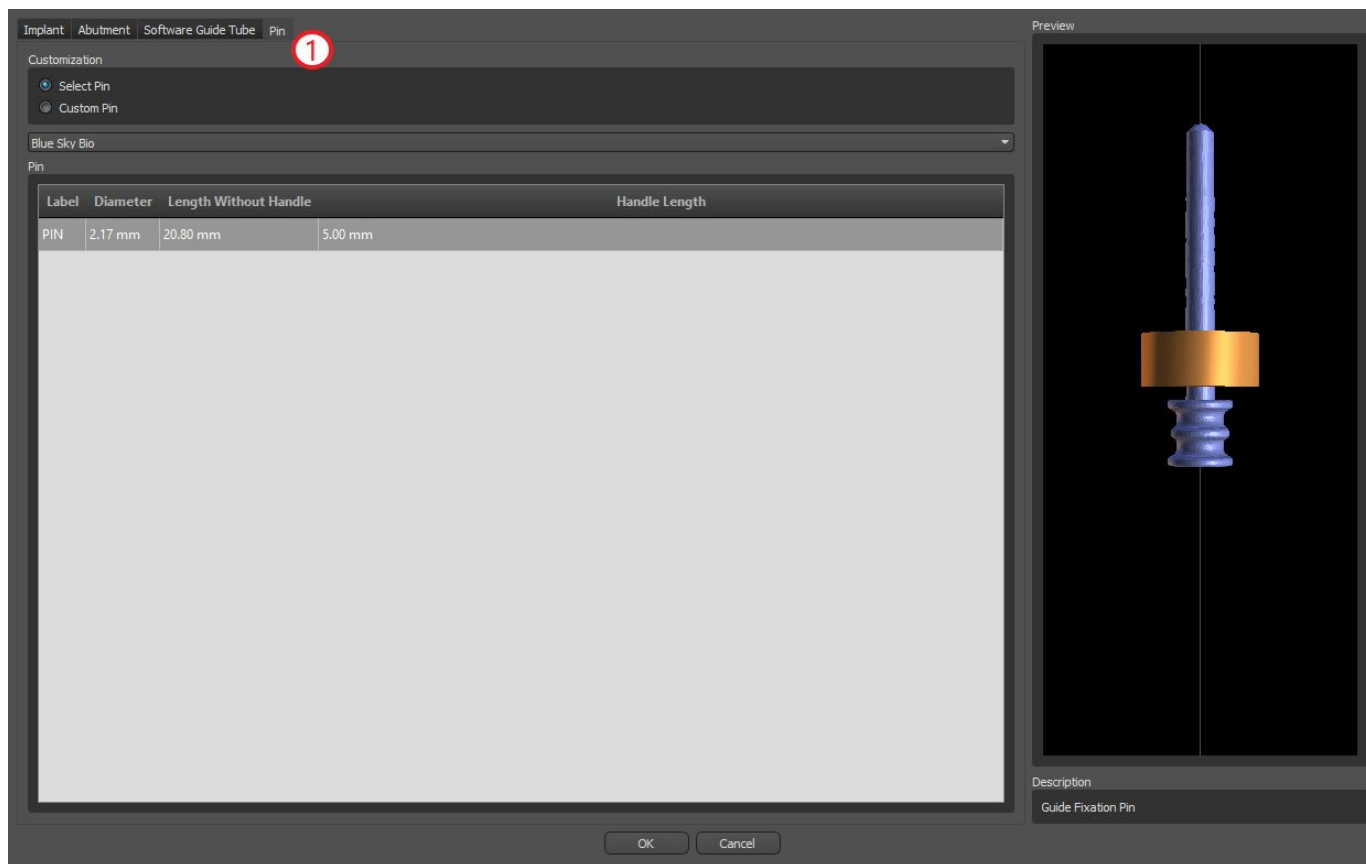


8.1.8 Your own pin


It is also possible to adjust your own pin similarly as for previous parts (1). Length 0 (2) sets the length of the blue part of the pin; length 1 (3) then sets the length of the green part.

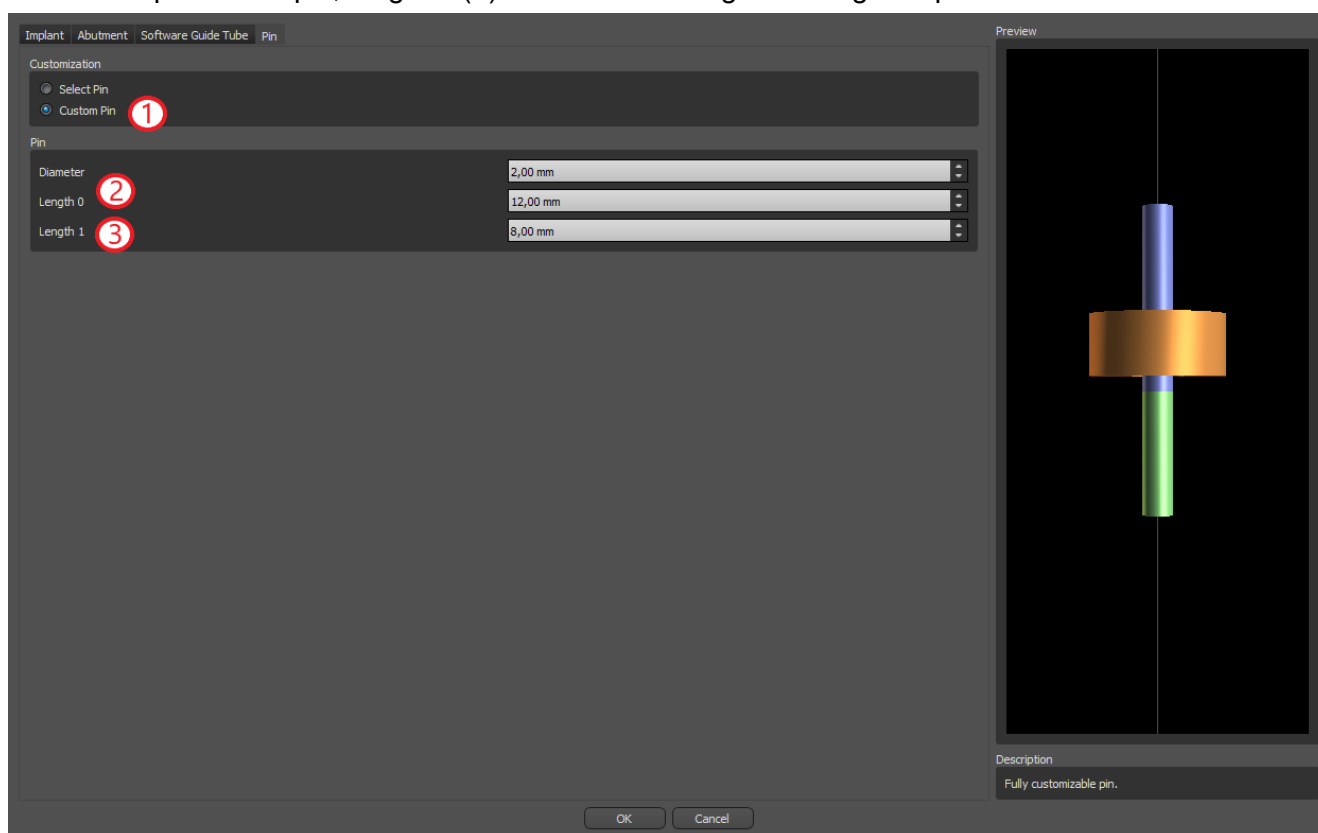

8.2 Positioning and manipulating the implant

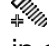We confirm the choice of implant in the dialog by pressing “OK”. The mouse cursor then changes to . We can then position the implant by clicking on the slice and, if necessary, on the 3D visualization in any window. After being added, the implant is always positioned perpendicular to the selected orientation. It is up to the user to fine-tune the position and rotation.

It is possible to move the implant in all of the windows. This is achieved by moving the cursor onto the implant, clicking the left mouse button, and dragging. The implant always moves in the plane of the view.

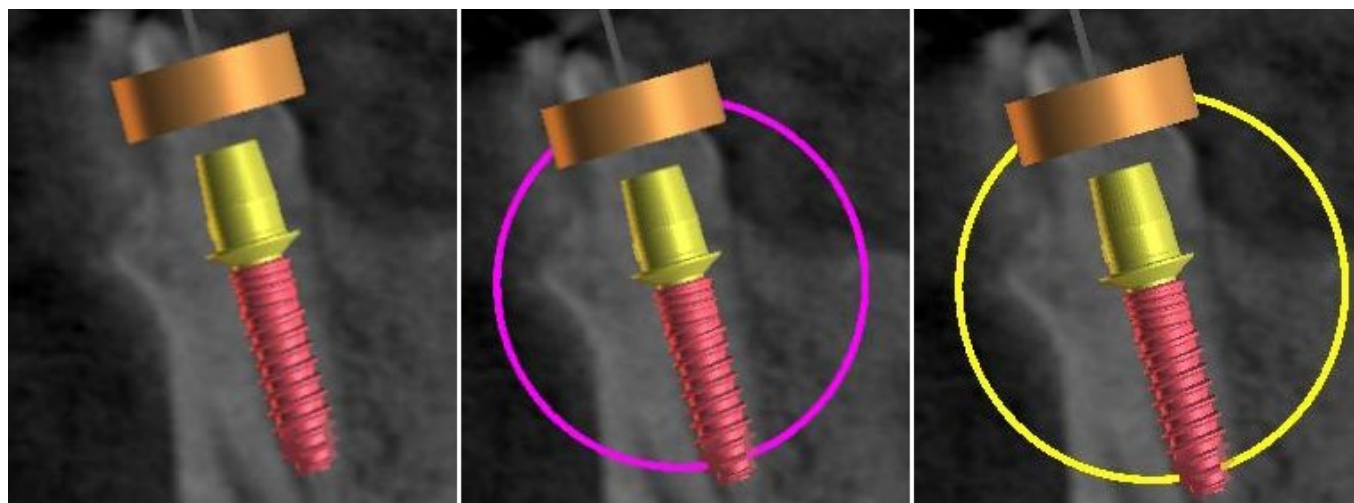

Besides moving it, it is also possible to rotate it using the so-called draggers. These are displayed when you position the cursor on the implant, its abutment, or the drill guide. By moving the mouse cursor onto the dragger and left-clicking the mouse, the dragger changes color, and by dragging the mouse it is possible to modify the implant's rotation.

8.3 Further operations on the implant


8.3.1 Toolbar

In the upper menu bar it is possible to display a toolbar of further operations on the implant by selecting “View > Toolbars > Implants Toolbar”. Here are icons for removing the implant (1), replacing the implant (2), displaying information about the implant (3) and Endo mode for placing implants by tip ((4) see chapter 17).




Operations 1 and 3 are used in the same way: click on the icon, move the cursor onto the implant, and click on it. For the option of replacing the implant (2), the same dialog appears as when adding an implant, in which we choose which implant we want to replace the selected implant with.

8.3.2 Context menu

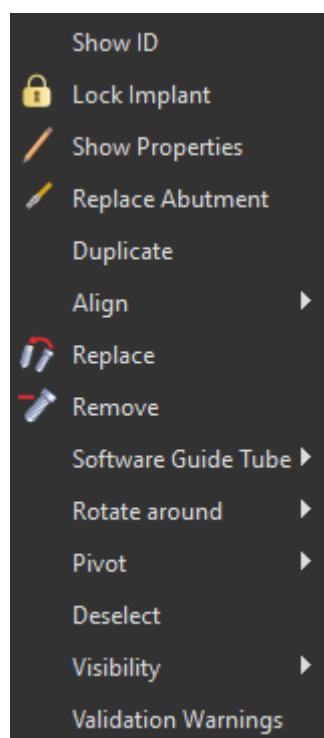


It is also possible to perform several operations on the implant via the context menu, which we can invoke by clicking the right mouse button on the implant.


It is possible to display the implant ID (a short editable description of the implant) or its properties. It is also possible to lock the implant against edits. The locked implant has a padlock icon displayed next to it. It is similarly possible to replace and, if necessary, add an abutment. The entire implant can be duplicated or aligned with another implant. That means that although the implant remains in its position, it turns in the space in such a way that its axis is parallel to the axis of another implant. A complex implant replacement can also be performed from the context menu. The implant can also be removed. Besides this it is possible to change the point around which the implant rotates during manipulation (the so-called Pivot). It is possible to cancel the choice of the active implant via the context menu. Finally, the visibility of individual parts of the implant can be changed ("Visibility > Implant/Abutment/Software Guide Tube").

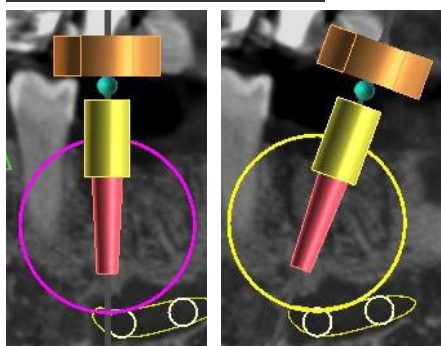

Pivot set to Center. The implant rotates around its center.

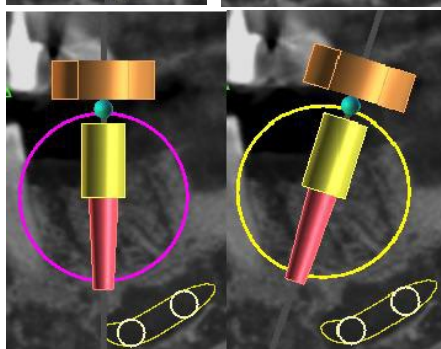


Pivot set to Head. The implant now rotates around its head.

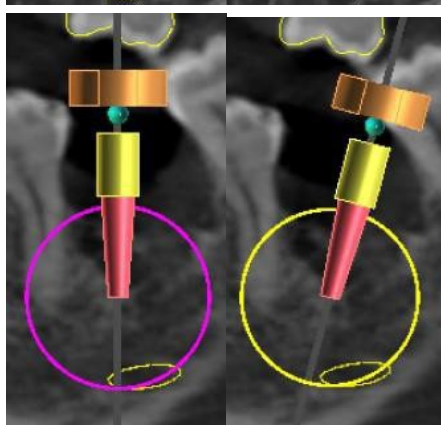

Pivot set to Tip. The implant now rotates around its tip.

8.4 Side panel

To make it easier to find our way among implants, the side panel can be displayed via “Panels > Implant List”.

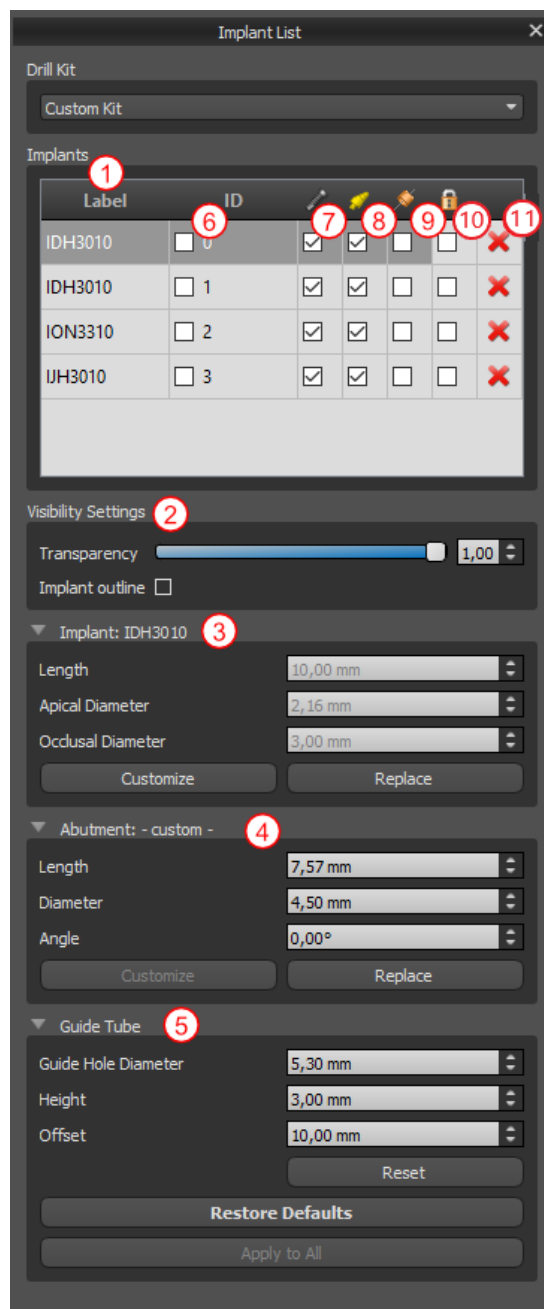

It contains a list of all the implants (1), the settings for the transparency and visibility of the outlines in the 2D view (2), the parameters of the active implant (3), its abutment (4), and the drill guide (5). For a pin, the properties of the pin are displayed instead of the properties of the implant and abutment. For the implant and abutment sections, there are also “Customize” and “Replace” buttons. The “Customize” button replaces the implant with its fully parameterized variant. “Replace” opens the same dialog as for adding an implant and makes it possible to replace the implant with the selected variant.

Several properties of the implant can be changed quickly in the list of implants. One is the ID (6), which is a short description of the implant. It is possible to change the visibility of the ID by double-clicking on it and to toggle its visibility in the windows with the check box. The check box under the implant (7) changes the visibility of the implant, the check box under the abutment icon (8) toggles the visibility of the abutment, and the check box under the drill guide (9) changes the visibility of the drill guide. All the visibility settings are independent of each other. Finally, every implant has an icon for removing it (11) and locking it (10).

8.5 Dialog with the parameters of the selected implant

The parameters of the implant can be displayed from the context menu. If the side panel is displayed, then the associated implant is selected in this panel, and its parameters appear. If this panel is hidden, a special dialog appears when the user attempts to display the parameters. It again displays the implant ID (1), the parameters of the implant (2), the parameters of the abutment (3), and the


parameters of the guide tube (4). For a pin, the parameters of the pin appear here instead of the parameters of the implant and abutment, similarly as in the side panel.

The screenshot shows a software interface with four main sections, each with a 'Customize' button and an 'OK' button at the bottom right.

- General (1):** Contains 'Implant ID' with a value of 0.
- Implant (2):** Contains 'Length' (3,50 mm), 'Apical Diameter' (2,06 mm), 'Occusal Diameter' (3,00 mm), and a checked 'Implant Visible' box.
- Abutment: - custom - (3):** Contains 'Length' (7,57 mm), 'Diameter' (4,50 mm), 'Angle' (0,00 °), and a checked 'Abutment Visible' box.
- Software Guide Tube (4):** Contains 'Hole Diameter' (5,15 mm), 'Height' (4,00 mm), 'Offset' (14,00 mm), and a checked 'SW Guide Tube Visible' box.

8.6 Validating implants

All implants are validated on the background. That means that their positions with respect to each other and to the nerves are checked. If an implant is evaluated as invalid, the icon appears next to it.

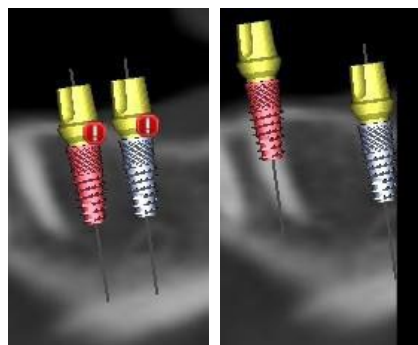

The first criterion considered is the distance between implants. If two implants are too close to each other, both are evaluated as invalid, which is represented by the appearance of icons over both implants.




The second criterion is the angle between implants. If this angle is too large, the implants are again evaluated as invalid, despite the fact that they are relatively far apart. To fulfill the validation criteria, we must reduce the angle between the implants and, if necessary, to increase the distance between the implants again.

The third criterion is the distance from the nerve. The same considerations apply here as for the distance between implants.

If you are not certain why the validation had failed, you may open a panel with the validation results via "Panels > Implant Validation". In this panel it is also possible to set the validation parameters (1). There is a list here of all invalid implants (2) and the reasons the validation of the selected implant failed (3).

Note: pins are not validated either against themselves (or implants) or against nerves.

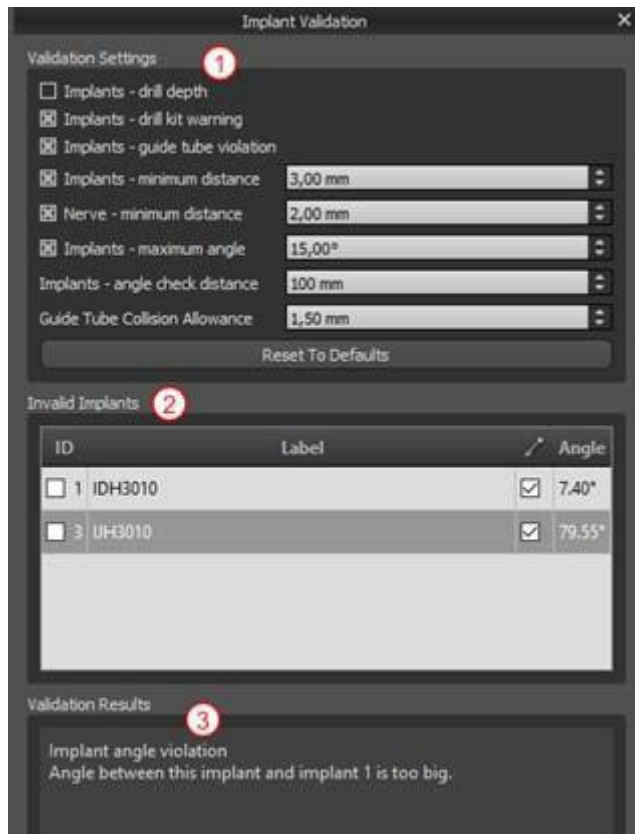

8.7 Ordering implants

Implants can be ordered directly from the application by selecting Order Parts in the toolbar.



The Blue Sky Bio website will load automatically, and the implants which were used in the treatment plan will be added automatically to the shopping cart. Additional implants and items can be added to the shopping cart by continuing to shop via the website.


If custom implants were used (implants that were not selected from the Blue Sky Bio catalog) in the treatment plan, those implants will not be added automatically to the shopping cart.

9 Tools

This chapter explains the properties of each Blue Sky Plan® tool and the toolbar that contains the most-used tools in the application.


9.1 Toolbar

- 1 – Menu bar (access to all the software tools)
- 2 – Icon bar (fast access to the main tools)
- 3 – 3D view icons (view rotation, volume rendering settings, and visibility of slices)
- 4 – Panoramic icons (offset step, X-ray mode, etc.)

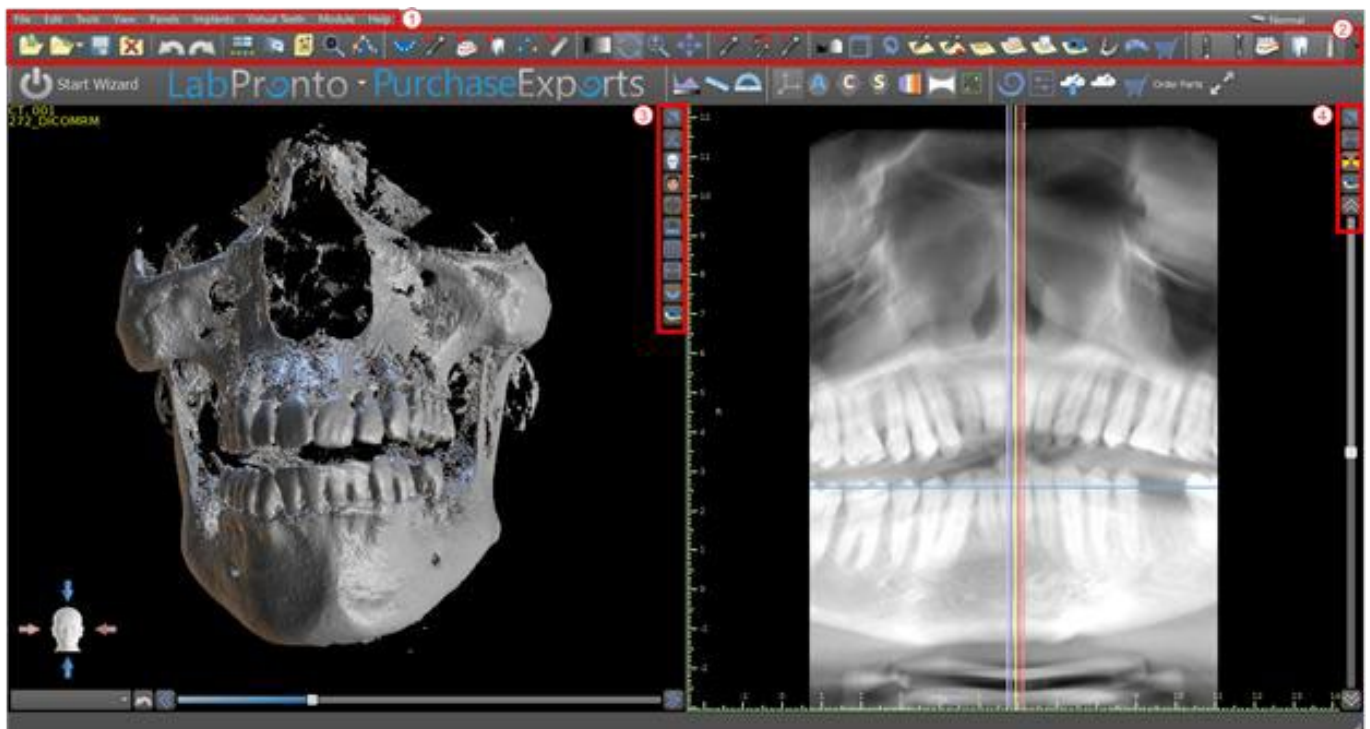

Further toolbars can be displayed by means of the context options in the main bar or via the option “View > Toolbars”.

9.2 Opening a project

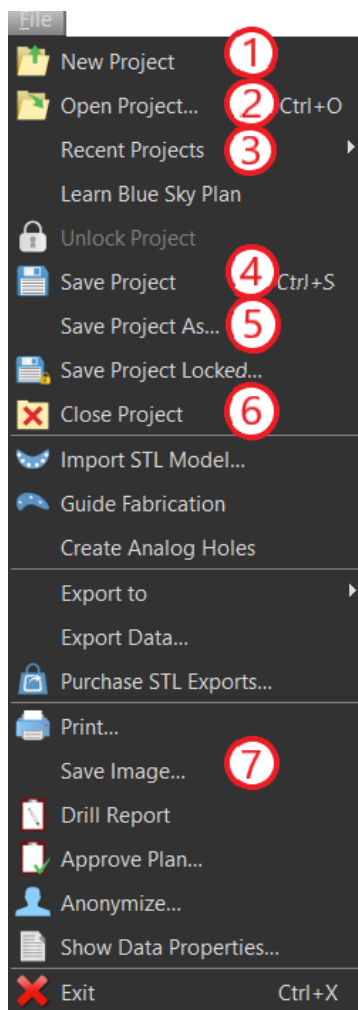

After clicking the "File > New Project" (1) menu option the application will return to the main menu. You can create new project by selection of planning type. Already existing project you can open by clicking the "File > Open Project" (2) and the most recent used project you can open through the "File > Recent Projects" (3).

9.3 Saving a project

To save a modified project, click the "File > Save Project" (4) menu option. To save a project under a new name, click the "File > Save Project As" (5) menu option.

9.4 Closing a project

To close a project file, click the "File > Close project" (6) menu option.


9.5 Saving images in JPG format

To save the currently visible screen images in .jpg format, click on the "File > Save Image" (7) menu option, then choose the folder where you want to save the images or PDF file.

9.6 Saving screenshots and generating a Drill Report

Screenshots can also be captured individually using the "Screenshot" button (1) located on the right side of each view. Captured screenshots are inserted into the list in the "Screenshots" panel (2), which can be accessed from the "Panels" menu.

For most slices this makes it possible to extract not only the current view, but also all further possible slice positions with the current settings. Snapshots are captured in the size corresponding to the current window size and the setting (3) in the "Screenshots" panel.

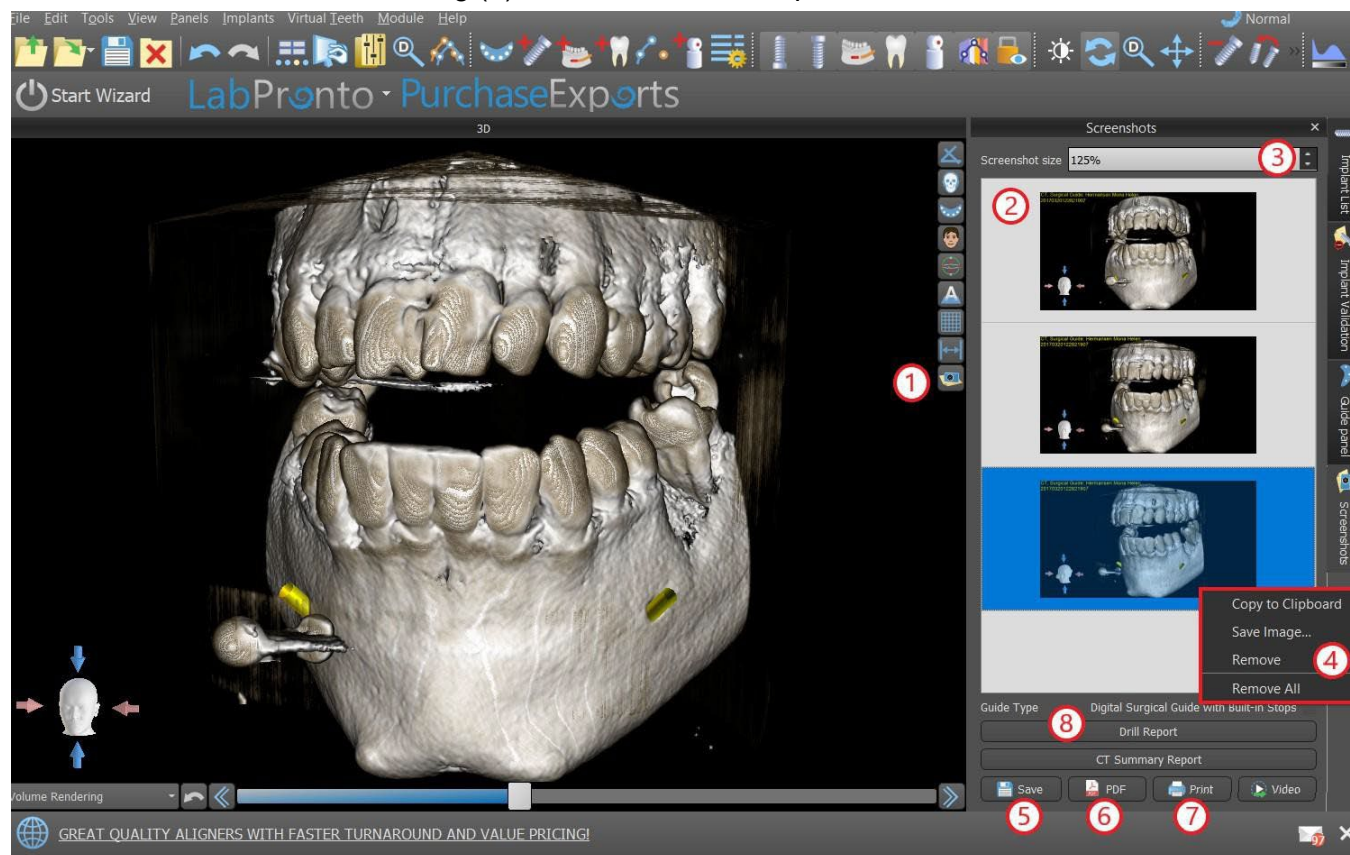

Using the context menu options, by clicking the right mouse button (4) in the list of screenshots (2), it is possible to copy the selected screenshot to the clipboard, save it to a disk, or remove it from the list. The screenshot can be displayed in its original size by double-clicking on the preview.

Snapshots can be saved individually in the selected bitmap format (5), a PDF document of all the windows can be created (6), and the document can be printed out directly (7).

9.6.1 Drill report

A set of screenshots can also be created with the "Drill Report" button (8) which creates a set of snapshots containing the 3D view, the panoramic view, and a pair of the normal and tangential view for each planned implant.

9.6.2 PDF export and print settings


When exporting to a PDF file and printing, the layout of the images on the page can be selected in the "Page layout" section (1). In "Downsize only" mode (2) the program will not enlarge the images and will attempt to maintain their actual size.

The screenshot shows the 'Page layout' settings dialog box for Blue Sky Plan. It is divided into several sections, each with a numbered callout:

- 1** **Page layout**: Contains radio buttons for 'One image per page', 'All on one page' (selected), and 'Custom'. The 'Custom' option has input fields for '3' and '2'. There are also checkboxes for 'Best fit', 'Downsize only (prefer real size)', and 'Put different view type on a new line'.
- 2** **Header**: Contains a font dropdown set to 'Arial' and a size dropdown set to '14'. Below is a text area with the text 'Message| Blue Sky Plan'.
- 3** **Footer**: Contains a font dropdown set to 'Arial' and a size dropdown set to '12'. Below is a text area with the text 'Blue Sky Plan'.
- 4** **Info**: Contains a font dropdown set to 'Arial' and a size dropdown set to '12'. Below are checkboxes for 'Patient Name' (checked), 'Patient ID', 'Scan Date', 'Date' (checked), and 'Project Name' (checked).
- 5** **Implant List**: Contains a font dropdown set to 'Arial' and a size dropdown set to '10'. Below are checkboxes for 'Guide report format' (checked), 'Hide drilling report 0.7mm message', and 'Abutment information' (checked).
- 6** **Image Title**: Contains a font dropdown set to 'Arial' and a size dropdown set to '12'. Below is a 'Colour' dropdown set to black. There are also radio buttons for 'Align to top' and 'Align to bottom' (selected), and checkboxes for 'Place title over image', 'Show slice position', and 'Show scale'.
- 7** **Notes**: Contains a font dropdown set to 'Arial' and a size dropdown set to '12'. Below is a large text area.


At the bottom right, there are 'OK' and 'Cancel' buttons.

The document created can be supplemented with a user header (3) and footer (4), basic information about the patient and project (5), a table of implants (6), and a description of each image inserted in the image (7). Once the settings are confirmed, the document created appears in the preview window.

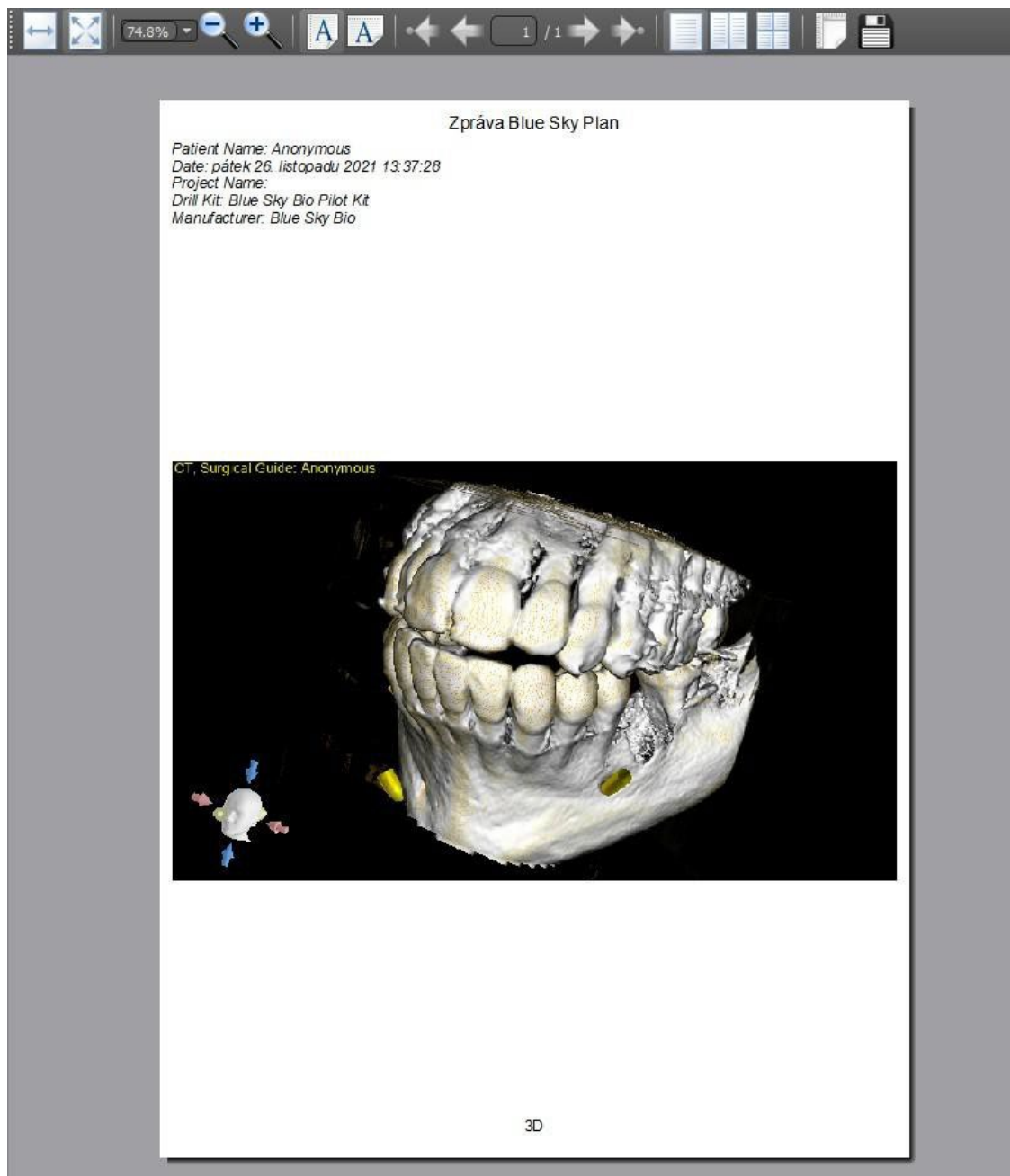


9.7 Modifying window and level

The display brightness and contrast of density data can be changed in the "Brightness/Contrast" panel (2), which can be accessed from the "Panels" menu (1). The brightness and contrast can also be changed with a special mouse mode (3) in which movement along the horizontal axis while pressing the left mouse button changes the brightness, and movement along the vertical axis changes the contrast.

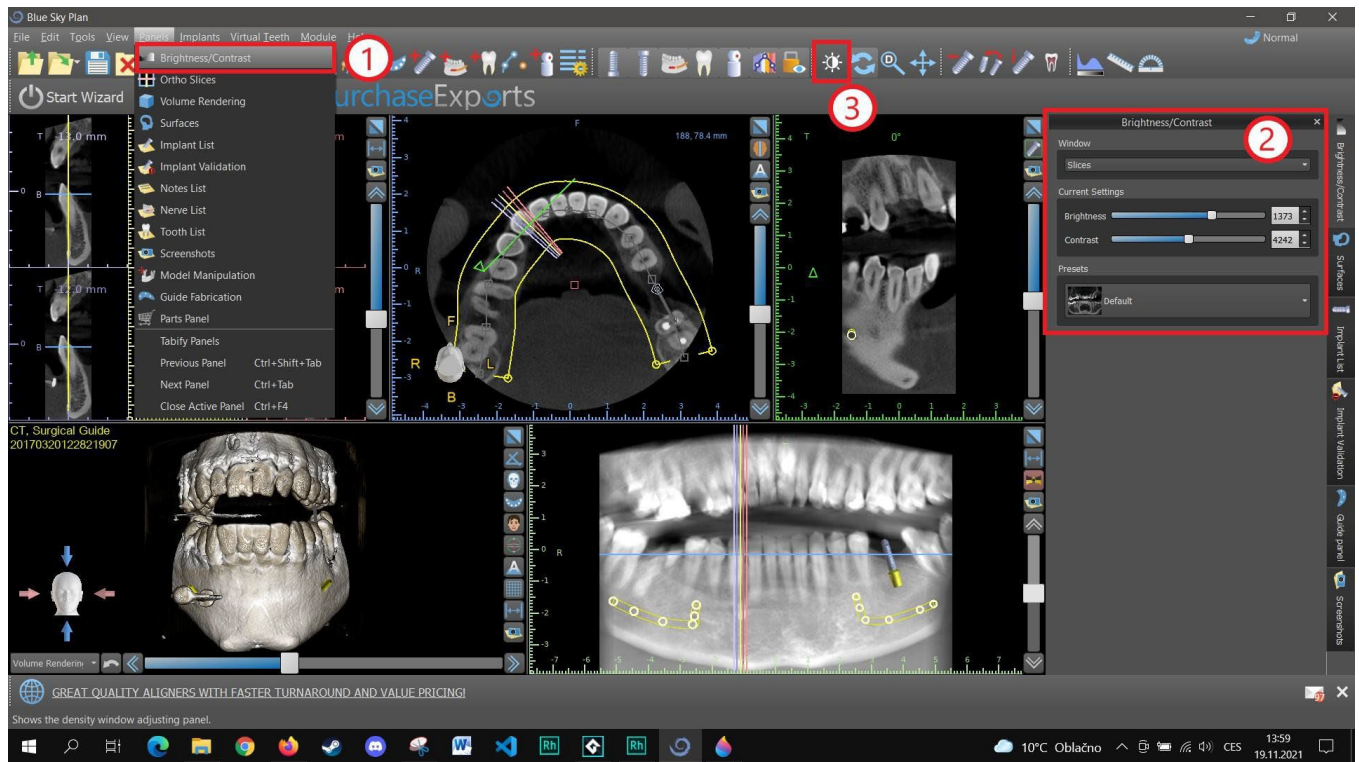


9.8 Zooming

To zoom in, press and hold down the right mouse button and drag the mouse up or down.

A second option is to select the zoom icon  (as shown below) and hold down the left mouse button while moving the mouse forward or backward.




9.9 Moving the image

Click on the Move Image icon  and drag the image as desired. An image can also be moved by holding down the mouse wheel (pressing it, not scrolling it) and dragging the image.




9.10 Moving objects


In the default mouse mode (represented by Trackball Mode icon  in the icon bar), left-clicking changes the position of the slice. If the mouse cursor is positioned over an object that can be manipulated, the mouse cursor changes, and the object can be moved, and if necessary its angle of rotation or size can be changed using the displayed dragger.



9.11 Measurements and overlays

Users can perform any measurement of density (1), length (2), or angle (3) in 2D images. They can also insert arrows (4), text descriptions (5), or rulers (6).

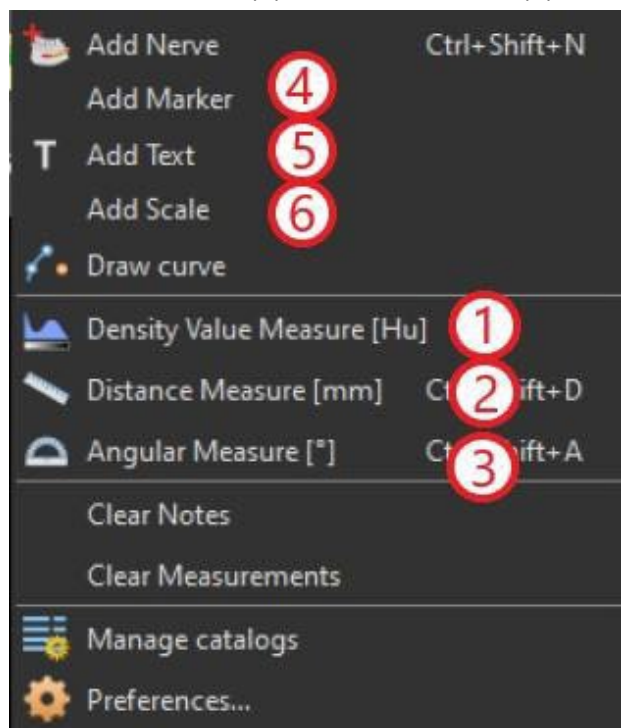

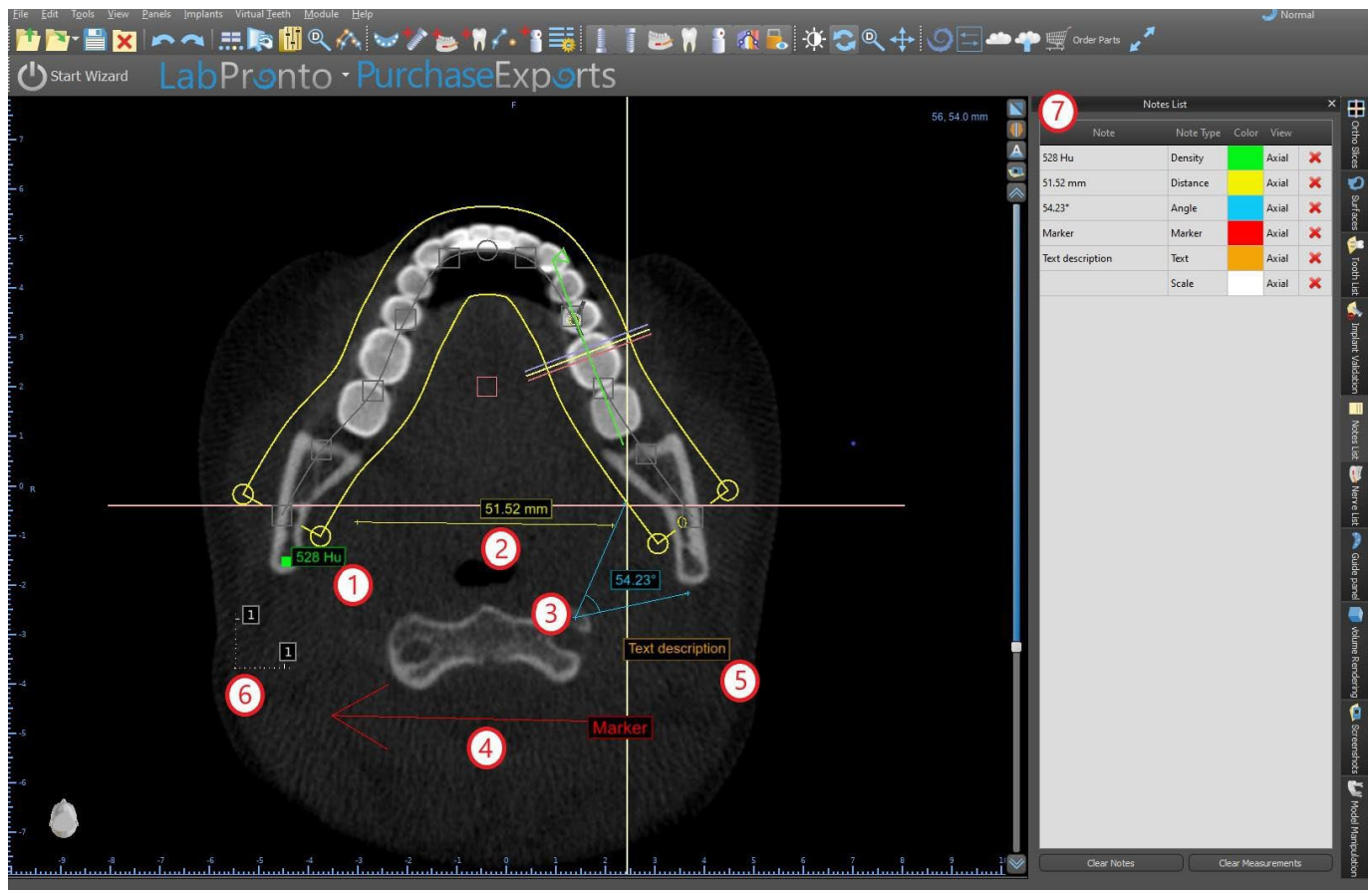

9.12 Linear measurement

Click the “Tools > Distance Measure” menu option (2) in order to perform any 2D measurements.

Click the left mouse button with the pointer on the image to mark the start point of the distance to be measured and click again on the image to mark the end point of the measured distance. To correct the measurement you can grab one of the end points and move them.

9.13 Angular measurement

Click the “Tools > Angular Measure” menu option (3). Click the left mouse button with the pointer on the image and repeat the process twice to get an angle. To correct the measurement you can grab the points specifying the angle.

9.14 Density measurement

Click the “Tools > Density Measure” menu option (1). The value of the density measurement is obtained from the information contained in the patient's image and can vary for each CT or CBCT scan. The measurements are given in Hounsfield units.

9.15 Measurement correction and number position

The user can correct the measurement and all the overlays in the default mouse mode. It is possible to correct a measurement by clicking at the edge of the measurement with the left mouse button and dragging. It is also possible to position the numerical value of the measurement outside of the image.


9.16 Deleting a measurement

It is possible to erase a measurement in two ways:

- a) Click the measurement edge or click on the number with the right mouse button. A menu will appear with the deletion option.
- b) Click the "Panels > Notes List" menu option. A panel will appear (7) with a list of all measurements, their types, values, and colors. To erase a measurement, click the X beside the measurement.

9.17 Overlays

To add text or other types of overlays, select the corresponding action from the "Tools" menu. Specify the position of the item in the selected window by clicking on it. The inserted overlay is bound to a specific position in the slice; it will not be visible for other positions. Orthogonal slices and the normal slice can be positioned in the location of an overlay by double-clicking on the "Note Type" field (1) or "View" field (2) of the associated item.

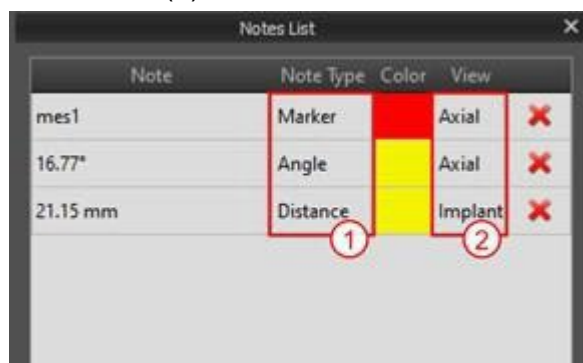

The text of an overlay can be changed by clicking on the "Note" field in the "Notes List" panel (7) or via the context menu on the item in the associated slice. The color of several types of items can also be changed in the "Notes List" panel by clicking on the "Color" field.

9.18 Panoramic

A panoramic image is a panoramic slice of the mouth and teeth. It is a perpendicular plane of the axial image.



9.18.1 Creating a panoramic line

Refer to Section 4.6.


9.18.2 Mandibular canal

The mandibular canal can be added in the panoramic image and viewed in all of the images.

9.18.3 Adding a mandibular canal

Adjust the panoramic curve to fit the patient's arch as described in section 4.6. Click on the "Add Nerve" button either located in the icon bar or under the "Tools" option in the menu bar (1). The Nerves panel (2) will appear and show the default properties for the nerve (3). Click on portions of the nerve in the panoramic window and follow the canal to the mental foramen.

When entering nerve points, the position of the slice can be changed with the scroll bar on the right side of the window, and the last-added point can be removed with the "Edit > Undo" action. You can fine tune the buccal/lingual aspect by adjusting the nerve point in the cross-sectional view later. Add another nerve by clicking the "Add Nerve" button (4) on the Nerves panel, and repeat on the other side. All images should show the nerve with a default diameter in yellow.

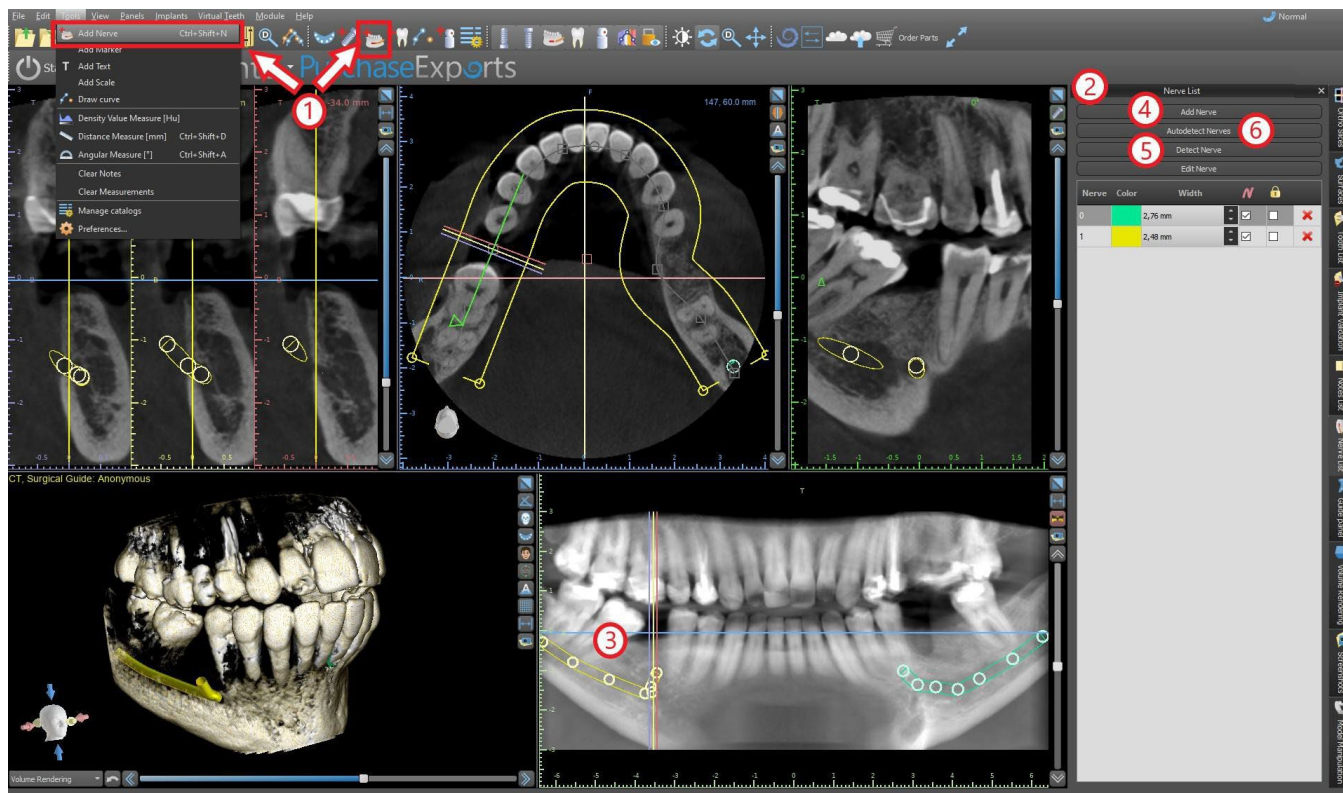

It is also possible to add a nerve by clicking the "Detect Nerve" button, where a whole nerve canal is added by clicking inside the mental foramen in the corresponding slice of the cross-sectional view. You can also use the fully automatized nerve detection by clicking the button "Autodetect Nerves".



9.18.4 Changing mandibular canal parameters


Clicking on the color of the nerve in the list (2) opens the color selection window. After choosing, confirm the selection by pressing "OK". The width of the nerve is displayed in the adjacent column. This can be changed either by clicking on the window and entering new values or by repeatedly clicking on the arrows beside it to change the value in increments of 0.1 mm.

9.18.5 Deleting the mandibular canal

Select the "Panels > Nerve List" menu option and click on the letter X beside the nerve to be deleted.


9.18.6 Canal display in different panoramic view modes

As described in Section 5.1.3, within panoramic view it is possible to switch between composite mode and slice display mode. In composite mode, the entire mandibular canal is displayed in the form of an outline, including all control points (see image). In slice mode, only the control points near the panoramic plane are visible. The mandibular canal itself is only displayed in the form of slices of the same plane

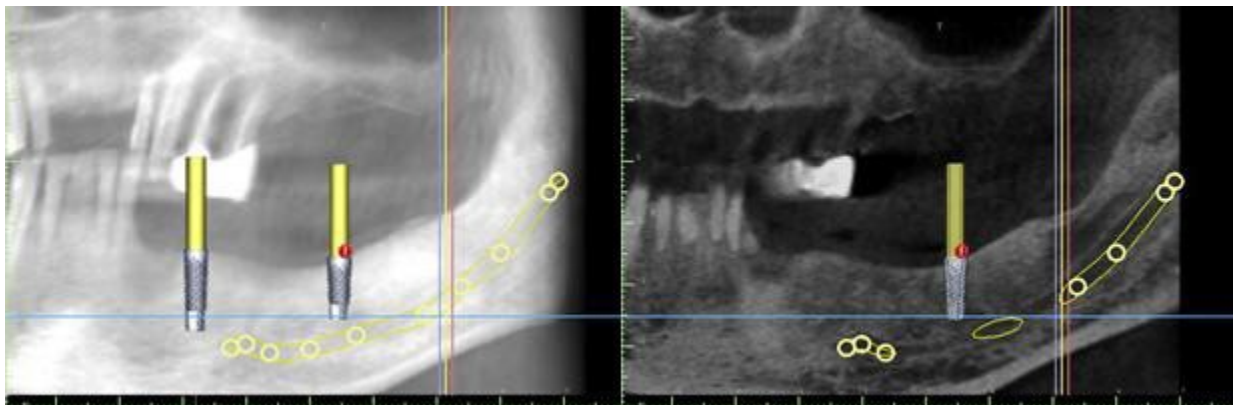

9.19 Show Dimensions

This measuring tool is available in the Model Master, Advanced and Surgical Guide (Endodontic) modules. Right-click on an object in the 3D scene and select "Show Dimensions" from the context menu. The dimensions are hidden by clicking "Hide Dimensions" in the same context menu. In Tools > Show Dimensions (1), it is possible to activate multiselection without having to click on "Show Dimensions". Multiselection is turned off by clicking outside the object. All dimensions can be hidden or redisplayed in the Surfaces panel (2).

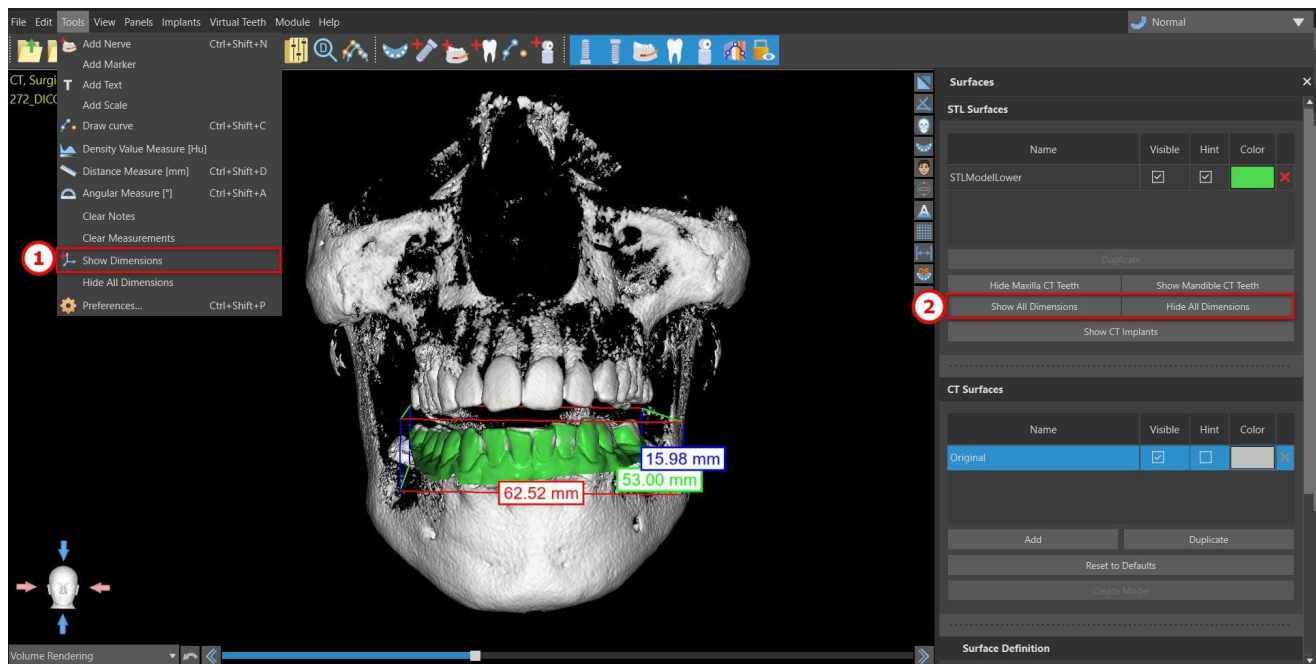


10 Importing the Scan Appliance

The Blue Sky Plan® Scanning Protocol calls for two CT scans. The first scan should be of the patient wearing a Scan Appliance, and the second scan should be of just the Scan Appliance. Since the Scan Appliance is made of mostly radiolucent material, the Scan Appliance does not appear in the patient's scan.


It does however appear when scanned alone, since the relevant density of the Scan Appliance is greater than the density of air, causing it to appear in the scan. The software allows the two scans to be merged so the Scan Appliance can appear in the patient's scan. Merging the Scan Appliance is an aid for proper treatment planning as it shows the position of the future teeth and removes any artifacts which may have previously appeared in the area of the teeth.


Follow the steps below to import the Scan Appliance:

- 1) Load the patient's DICOM set into the Blue Sky Plan application.
- 2) In the "Model Master" module, choose "File > Import DICOMS > Scan Appliance". Follow the prompts to identify the location of the Scan Appliance DICOMs and select the relevant region of interest, selecting only the range containing the Scan Appliance.

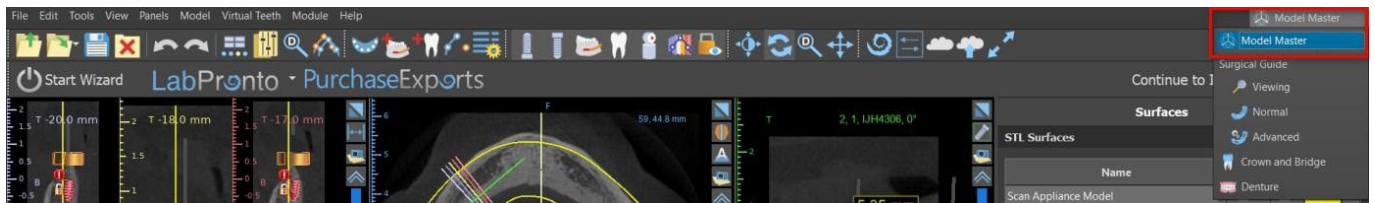

Scan Appliance can also be imported via the "Model Master" module on the main screen. After choosing "Model Master > Import Patient CT Scan", follow the prompts and select the regions of interest (as described above). After that, the window with various types of additional surfaces appears. Choose "Import CT Scan of Denture or Scan Appliance with Markers".

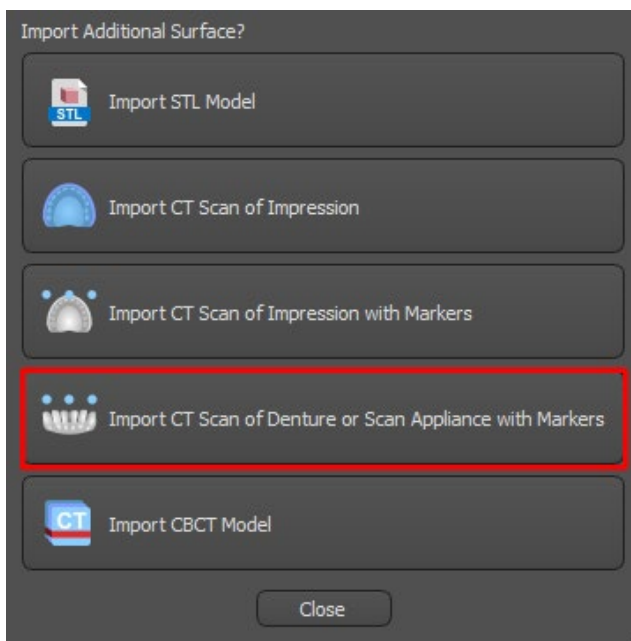

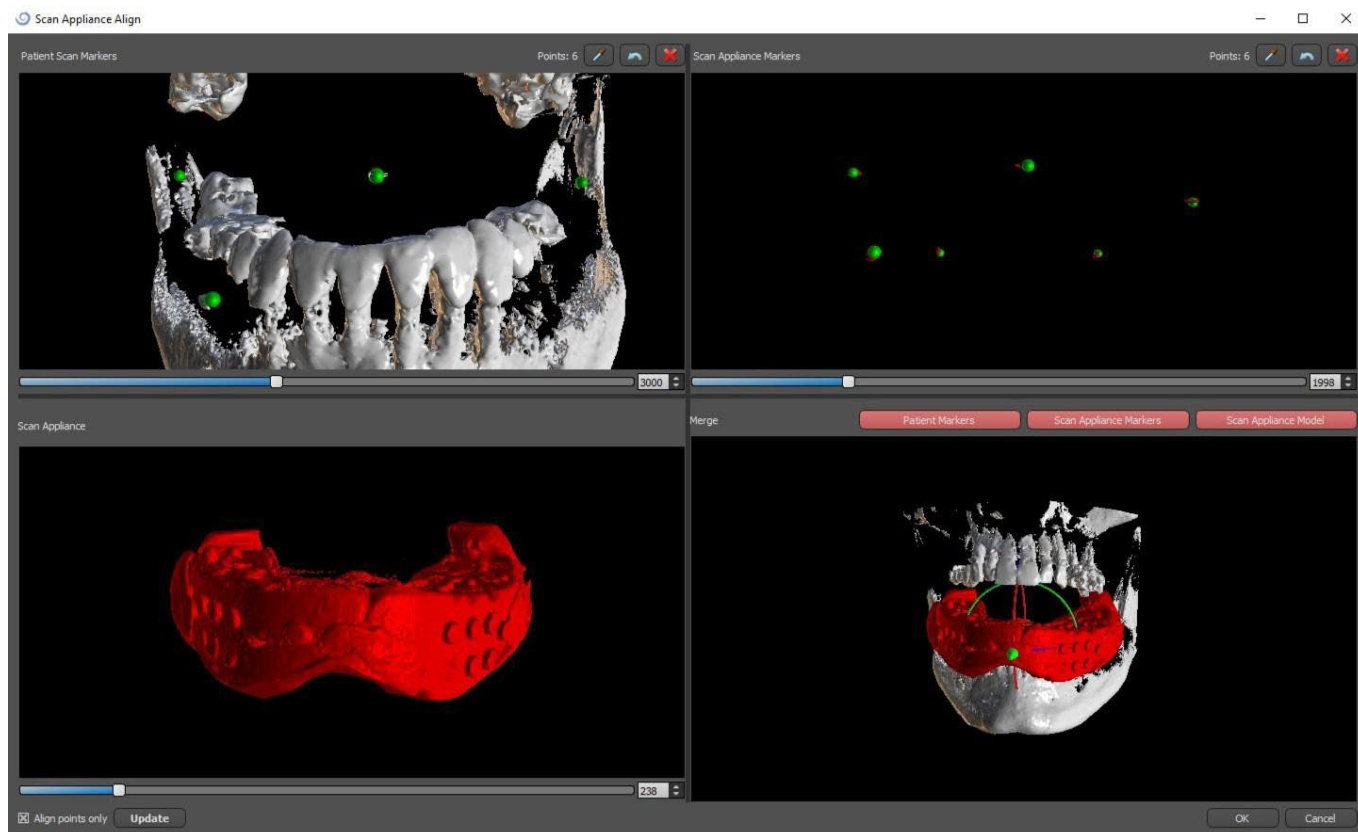


3) In the process of alignment, you can use the buttons to toggle simultaneous display of the patient data gutta markers (white) and Scan Appliance (red). You can display or hide the Scan Appliance model with the button.

The top left-hand box contains the gutta markers from the patient's scan. The top right-hand box contains the gutta markers of the Scan Appliance. The bottom left-hand box contains an image of the Scan Appliance. The bottom right-hand box displays both sets of gutta markers if they coincide.

10.1 Gutta percha markers coincide


If the two sets of markers do coincide, we continue as follows. If they do not coincide, proceed to section 9.2.

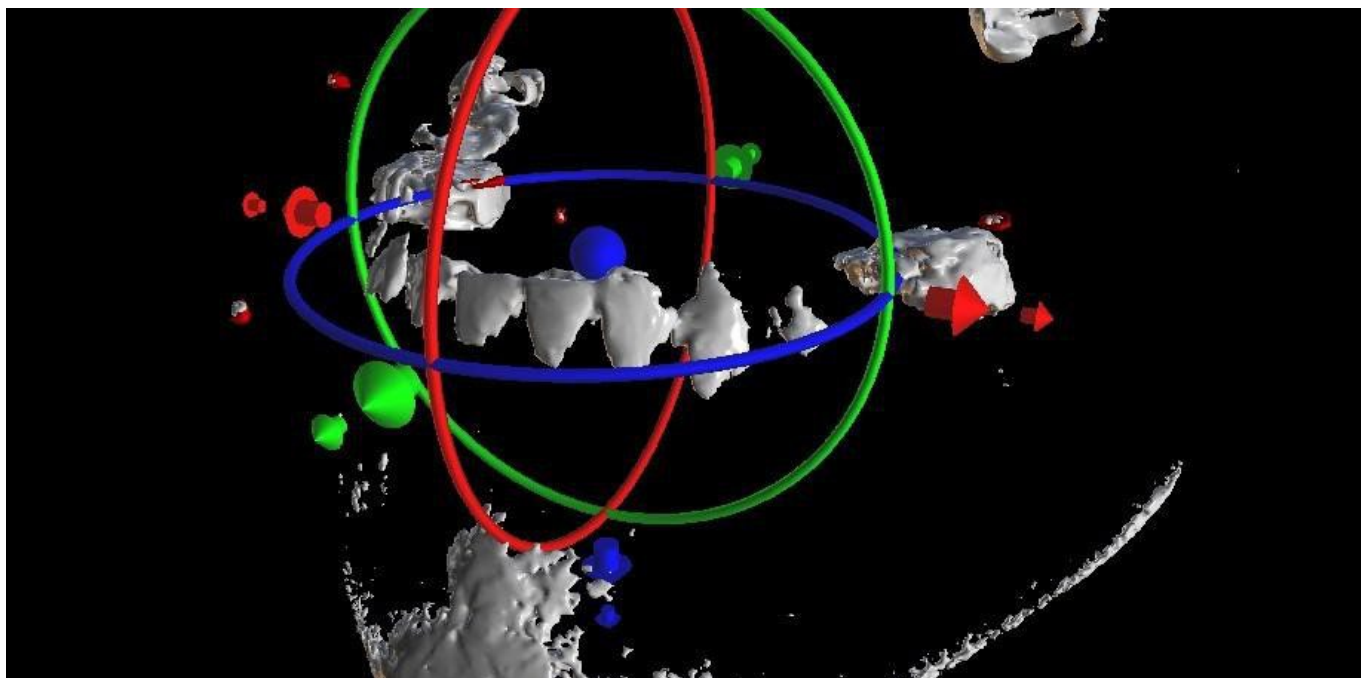


4) Confirm the data alignment by pressing “OK”.

5) The Scan Appliance will appear in the 3D image of the patient’s jaw. The properties of the Scan Appliance can be changed in the “Surfaces” panel in the “Imported Surfaces” table.

10.2 Gutta percha markers do not coincide

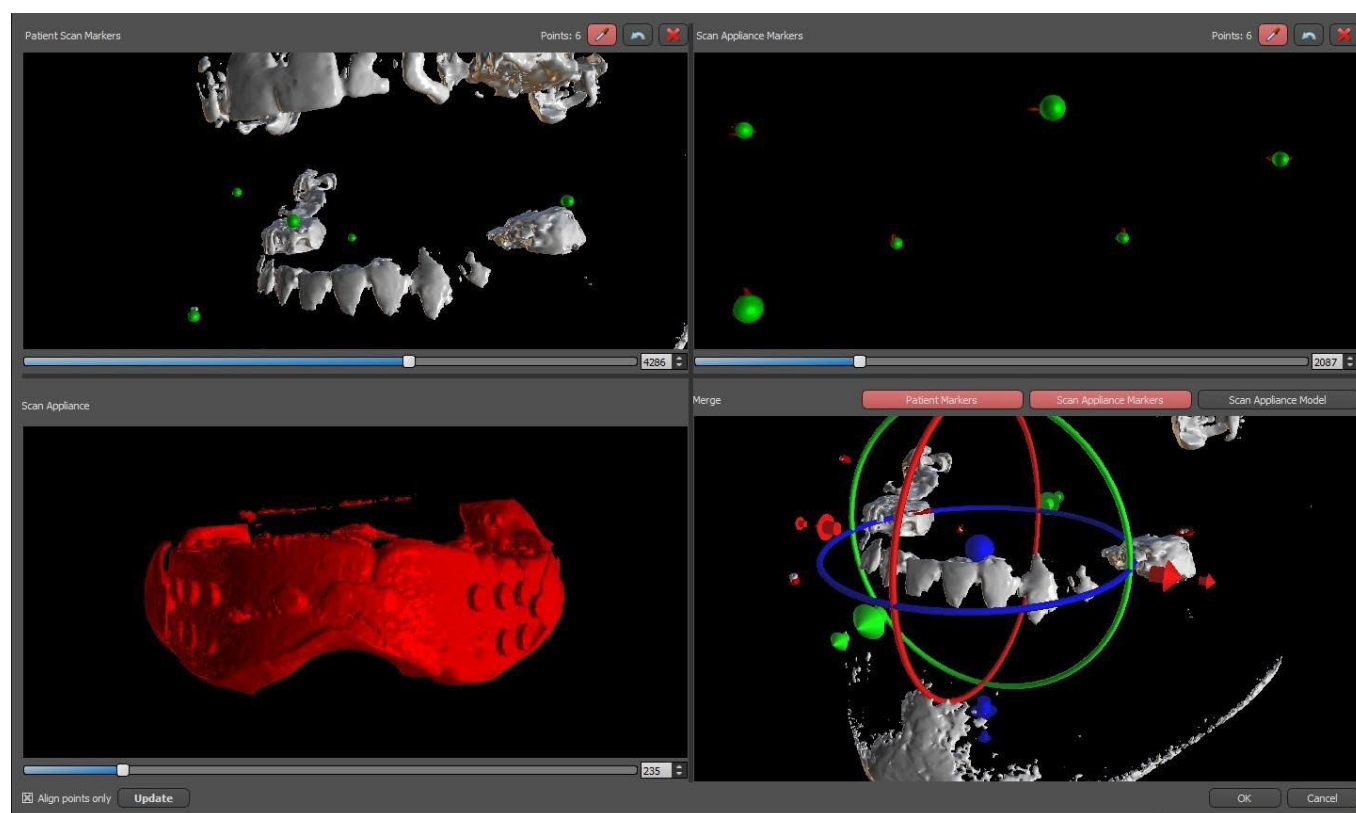

4) In the top right-hand box use the zoom tool (right mouse button), the 3D density slider, and the rotation tool (left mouse button) to clearly view at least four gutta markers. Identify the gutta markers by clicking the  button or pressing the "Shift" key followed by left mouse clicks on the gutta markers.

An erroneous identification of one marker can be reversed with button. If necessary, all identifications can be canceled with a button.


5) Repeat the previous step for the top left box to identify the patient's markers.

6) Perform the alignment on the basis of the gutta markers identified by pressing the "Update" button. If the result does not roughly correspond to what is desired, identify more points for alignment or, for data with gutta markers that are hard to discern, check "Align points only". Under this setting, the program aligns the points identified without reference to dimensional data. The alignment can also be corrected manually using the dragger in the lower right-hand window.


7) The Scan Appliance will appear in the 3D image of the patient's jaw. The properties of the Scan Appliance can be changed in the "Surfaces" panel in the "Imported Surfaces" table.


11 STL model import (planning with CT scan)

Existing STL models are imported into Blue Sky Plan® by selecting “Import STL Model” option from automatic pop-up menu after Patient CT Scan Importing. Another possibility is selecting “File > Import STL Model”.

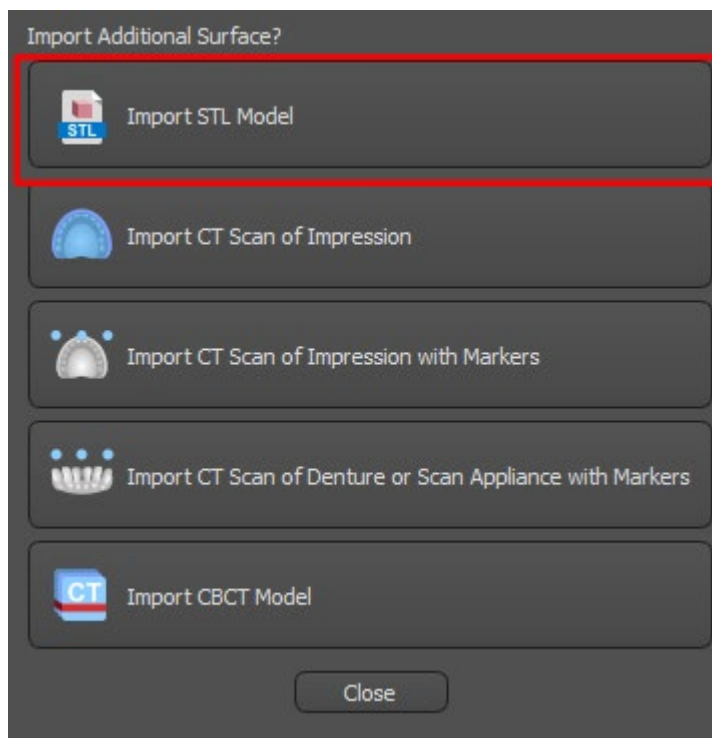


After the model has loaded, the Blue Sky Plan Automatic Alignment asks for model orientation and attempts to align the model automatically. The “Model Manipulation” panel can also be used to position the model to the desired location.

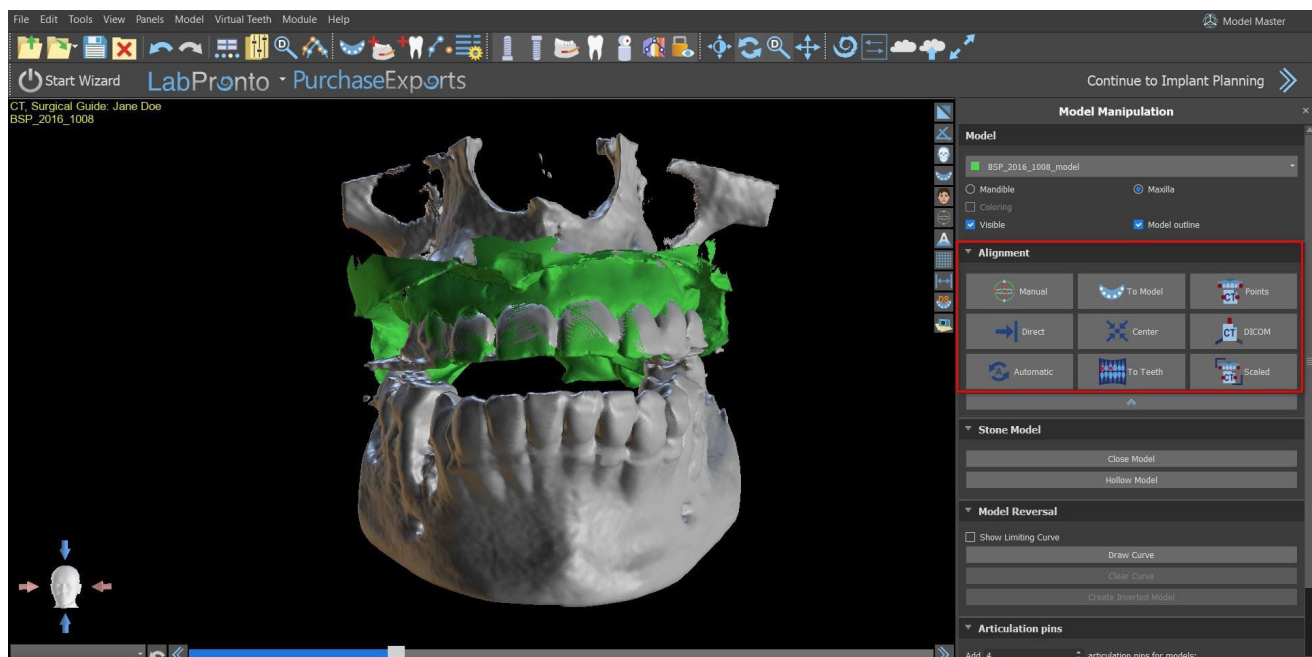

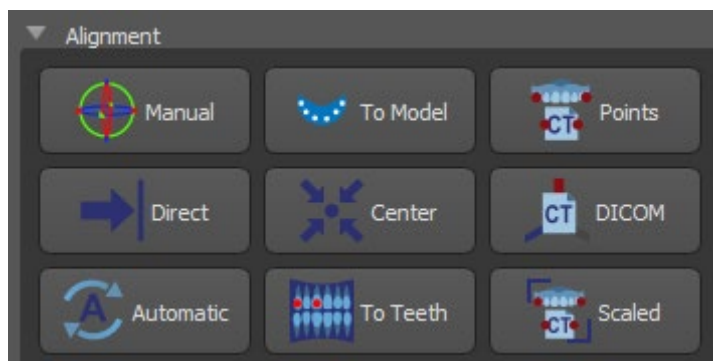


If the model is already aligned with respect to the data, the “Direct” should correspond to the correct position of the model. For STL models created with different software than Blue Sky Plan®, use the “DICOM” alignment. “Center” places the model into the middle of the scene without performing any rotation.


Because automatic alignment is usually not precise enough, Blue Sky Plan® also makes it possible to perform the alignment by entering a pair of points corresponding to each other. If you select “Points”, a dialog appears for entering points manually. The surface model displayed in the left-hand window of the dialog is created on the basis of the settings of the currently selected density surface. It is also possible to significantly influence the time it takes to create the model and the visibility of the pertinent reference points by duly setting the thresholds of the density surface.

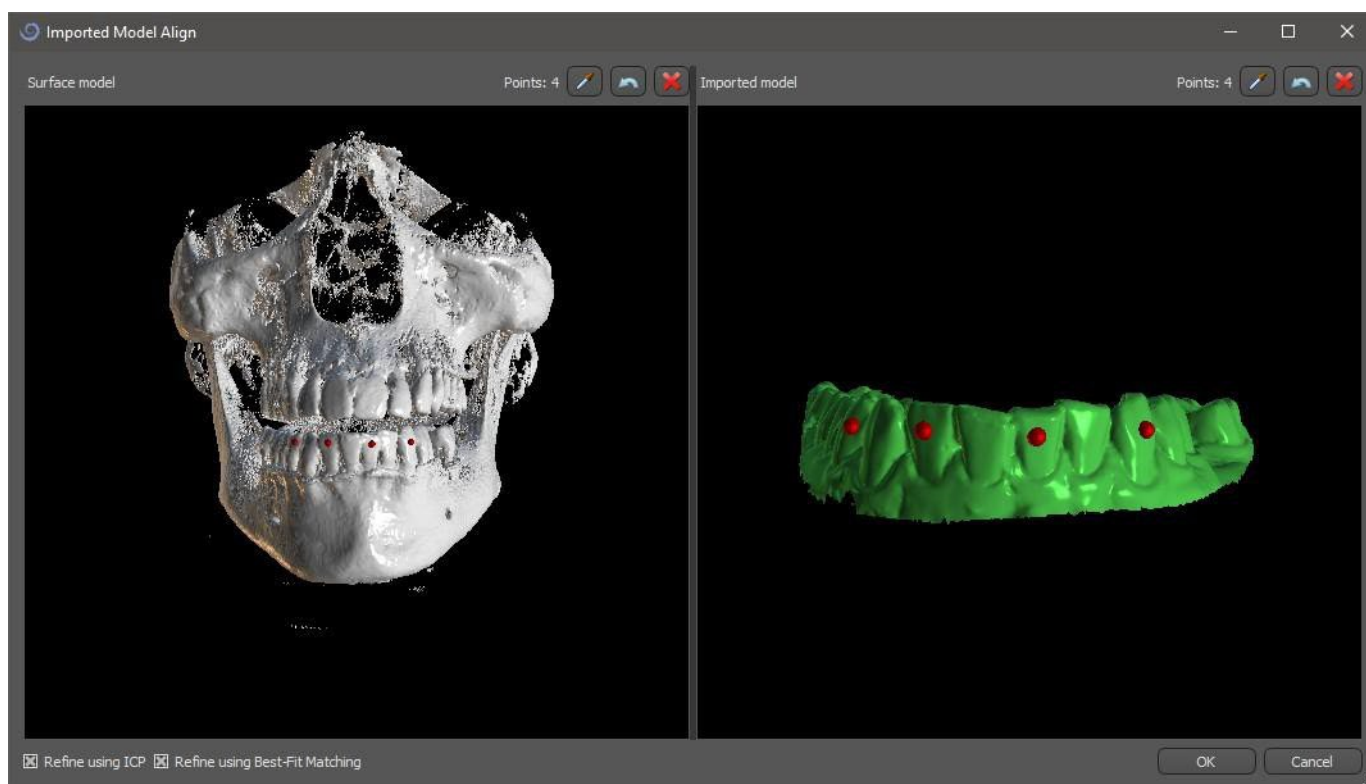

Points can be added by clicking on the button with the eyedropper  or by pressing the Shift + Left click. If “Refine using ICP” is activated, the program attempts to perform an additional automatic alignment after the alignment it made based on the points entered.

A similar point based manner of alignment is “Scaled” (works with extra scale) and “To Teeth” (works with the panoramic image instead of the surface model).

It is also possible to align the model manually in the 3D window using the dragger that appears when you press the “Manual” button. Clicking this same button closes the manual alignment mode, and the dragger disappears.

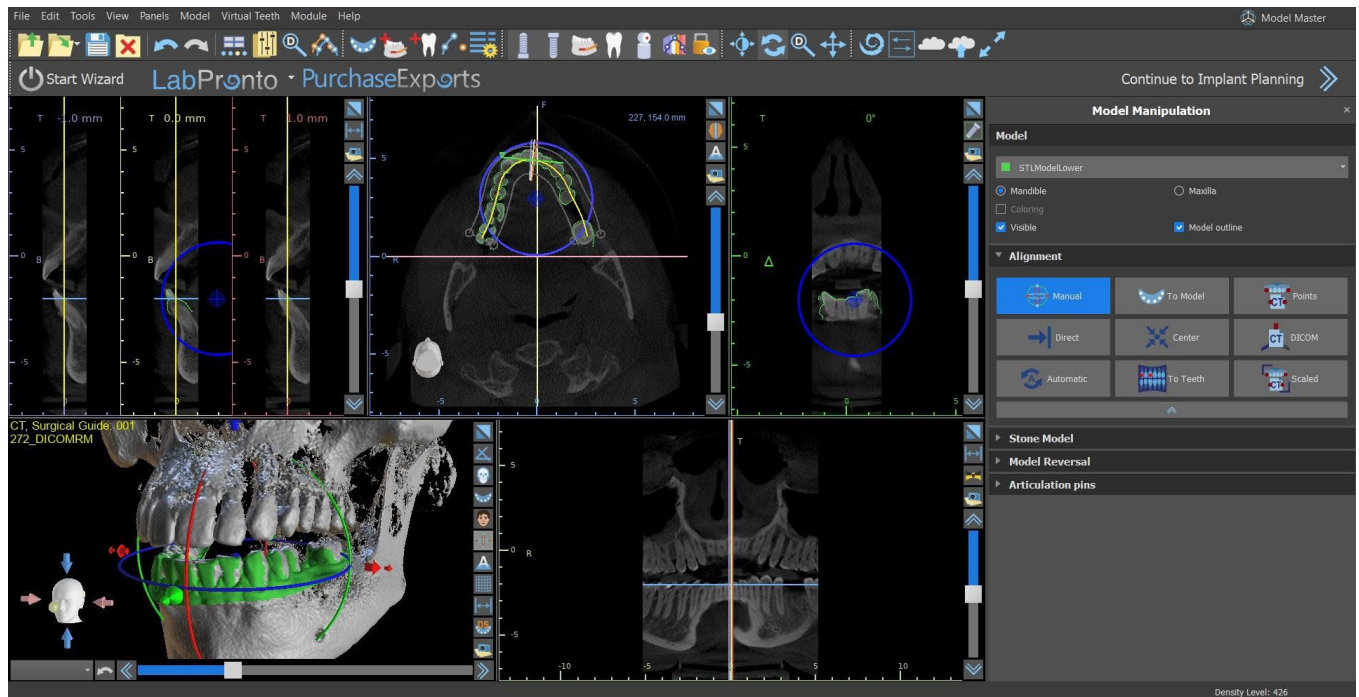


When aligning manually, it is possible to define a fixed point (a so-called pivot) around which the model rotates. In 2D views, the pivot is represented by a cross. By dragging the cross, it is possible to move the model in the plane of view. The model can be rotated using the blue circle. Hold Shift and click the left mouse button to position the pivot at a different position in 2D views. In the 3D window, the pivot can be repositioned using the small dragger arrows.


12 Planning without CT scan

Blue Sky Plan® software also allows planning for cases without a CT scan – STL models only.


If a user plans a case without a CT scan, the imported models might not be oriented properly (the orientation does not match the head in the bottom left of the 3D view). This makes planning very difficult. For these cases, Blue Sky Plan has a model alignment dialog, which opens when selecting the planning type of the “Import models”.

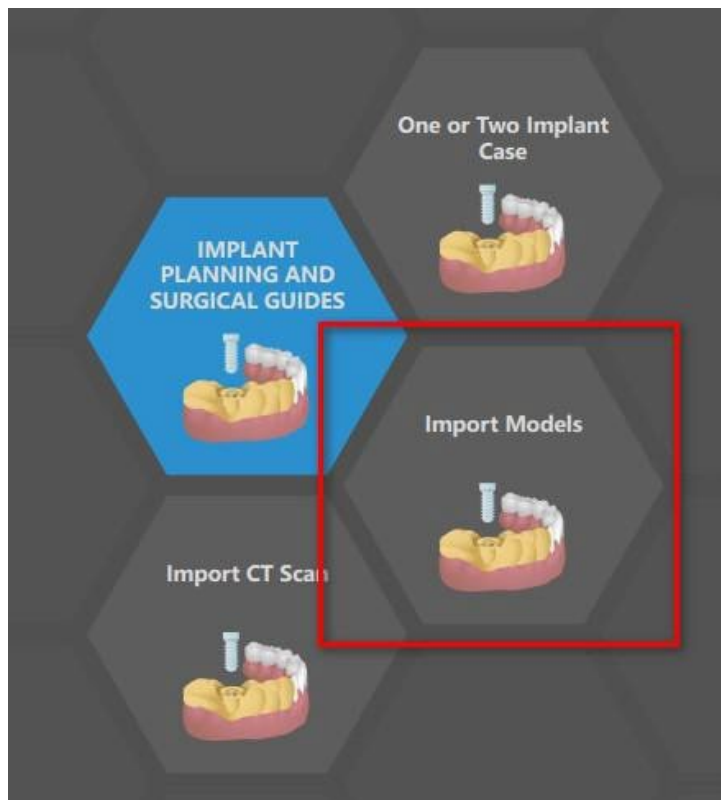


You can import one or several models. For alignment, it is necessary to select the primary model (ideally Maxilla) through which the alignment is done. The user also needs to specify if the model is a Dentated Arch, an Edentulous Arch or a Partial Arch.

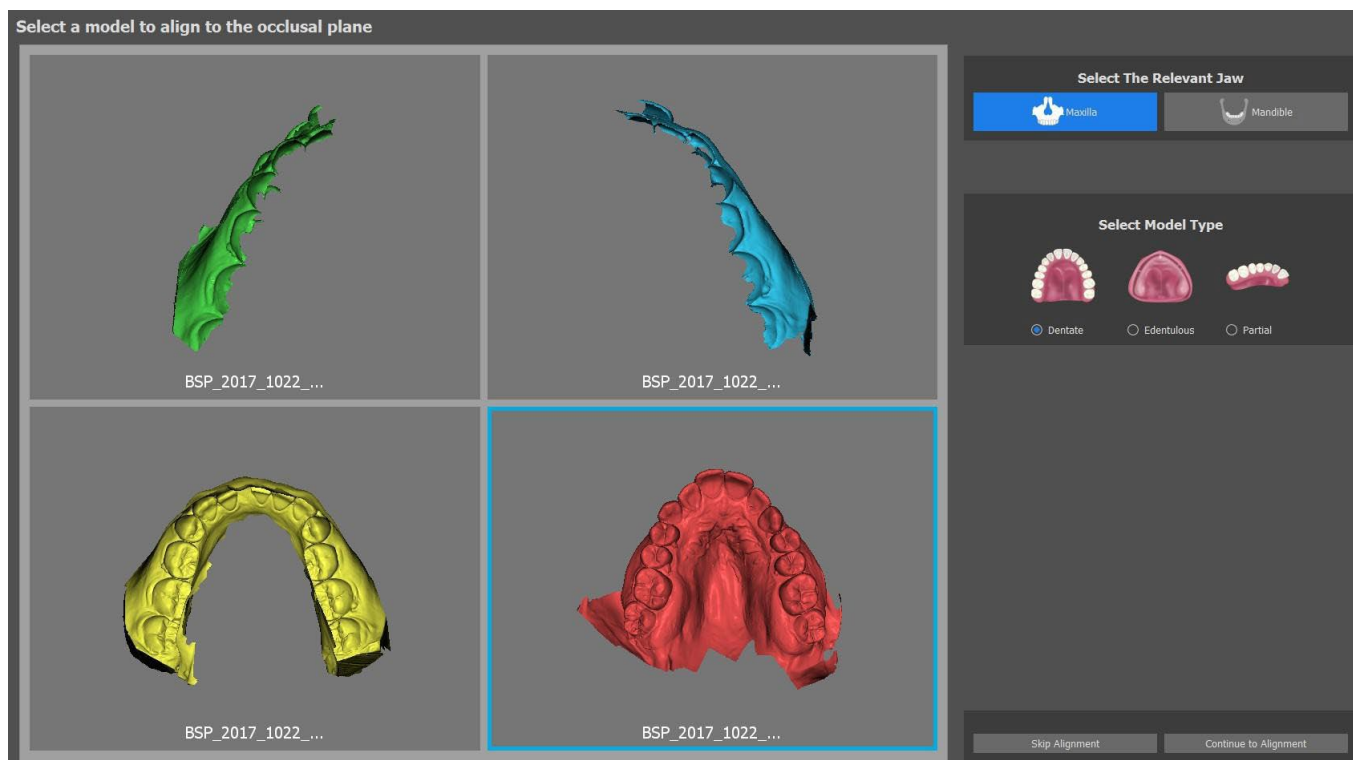


In the next step, the user marks three points on the model (by holding Shift + Left clicking the mouse) to help the software align the models properly (to match the orientation of the BSP coordinate system). The rest of the models will be aligned relative to the alignment of the primary model.

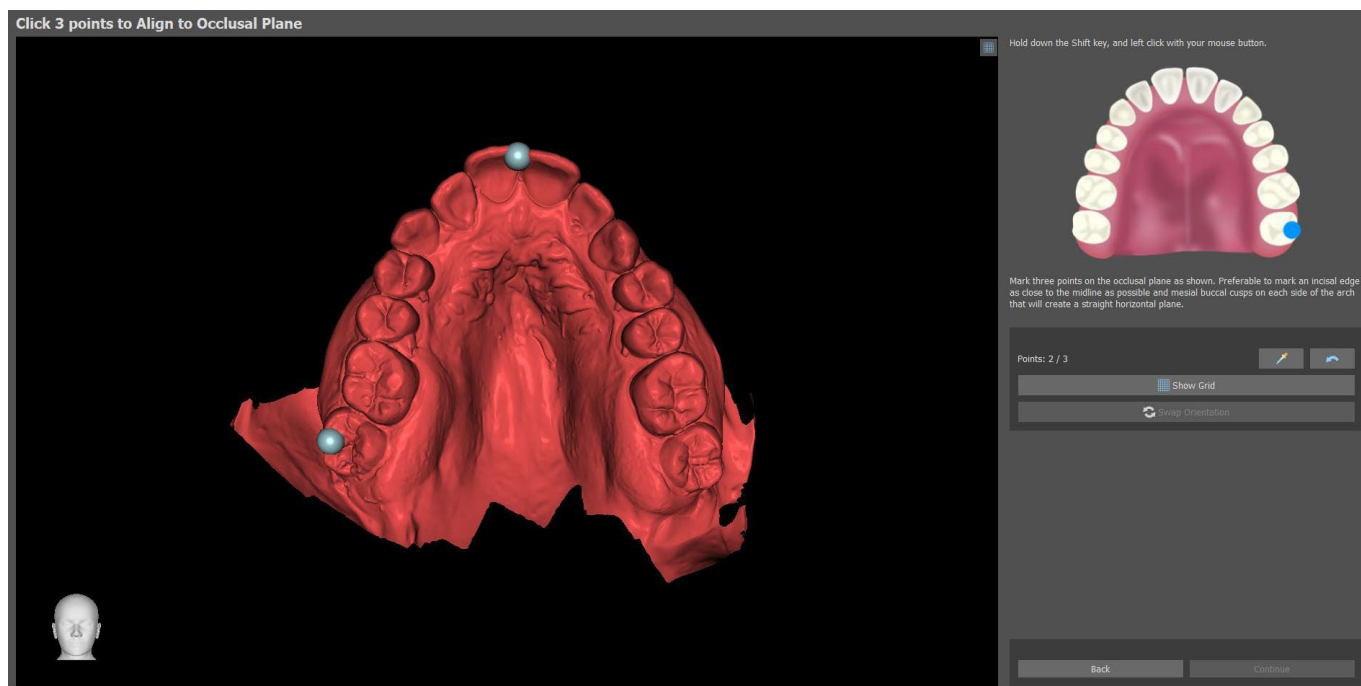


In the last step, the user can manually fine-tune the primary model using the grid. If several models were imported, they will move together as a group.

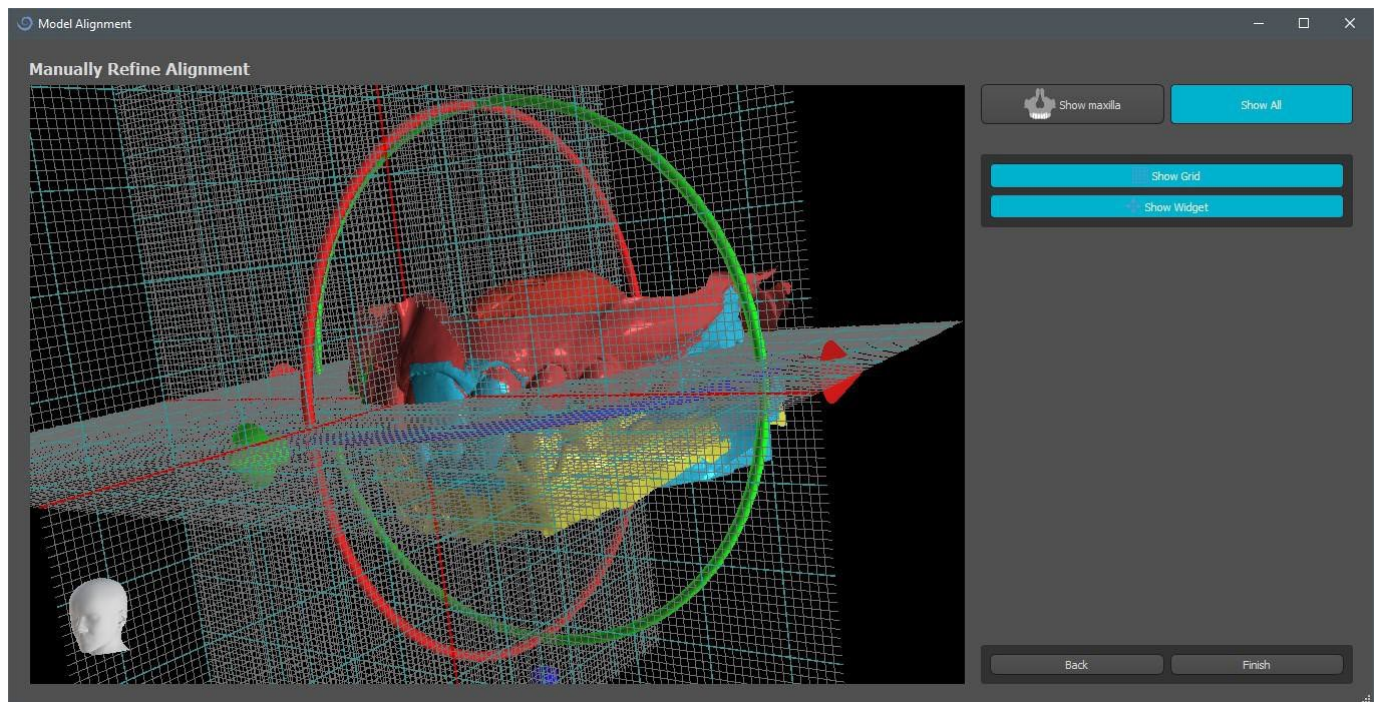

Blue Sky Plan® software also allows the user to align models automatically. This option can be modified in Preferences.

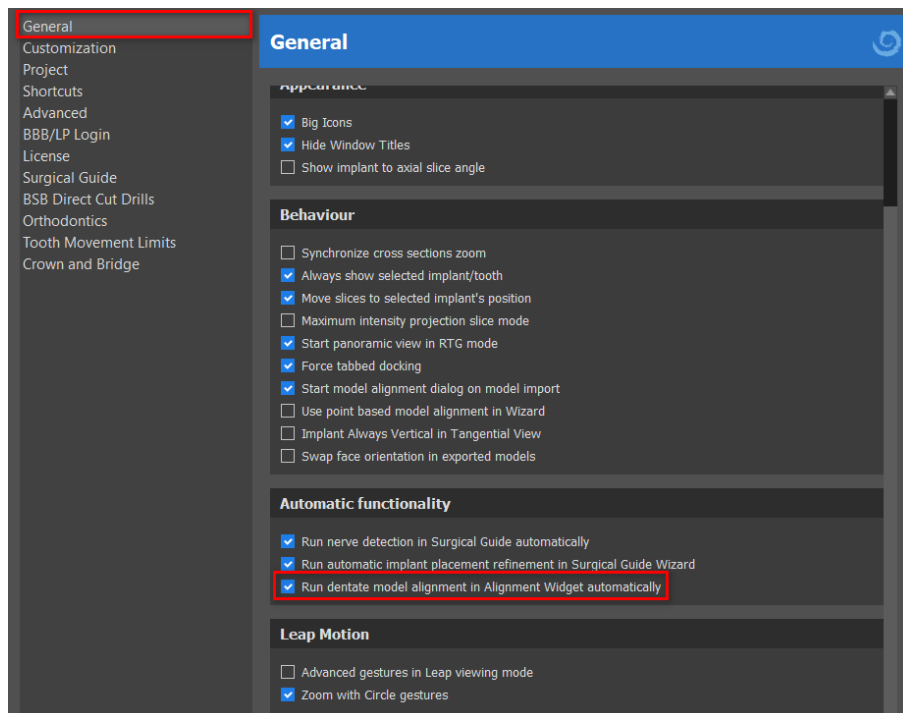


The primary model can be manually adjusted using the grid. If several models were imported, they will move together as a group. It is possible to reset the position of the fine-tuned model to its automatically aligned position. STL model alignment will be completed by clicking the “Finish” button.

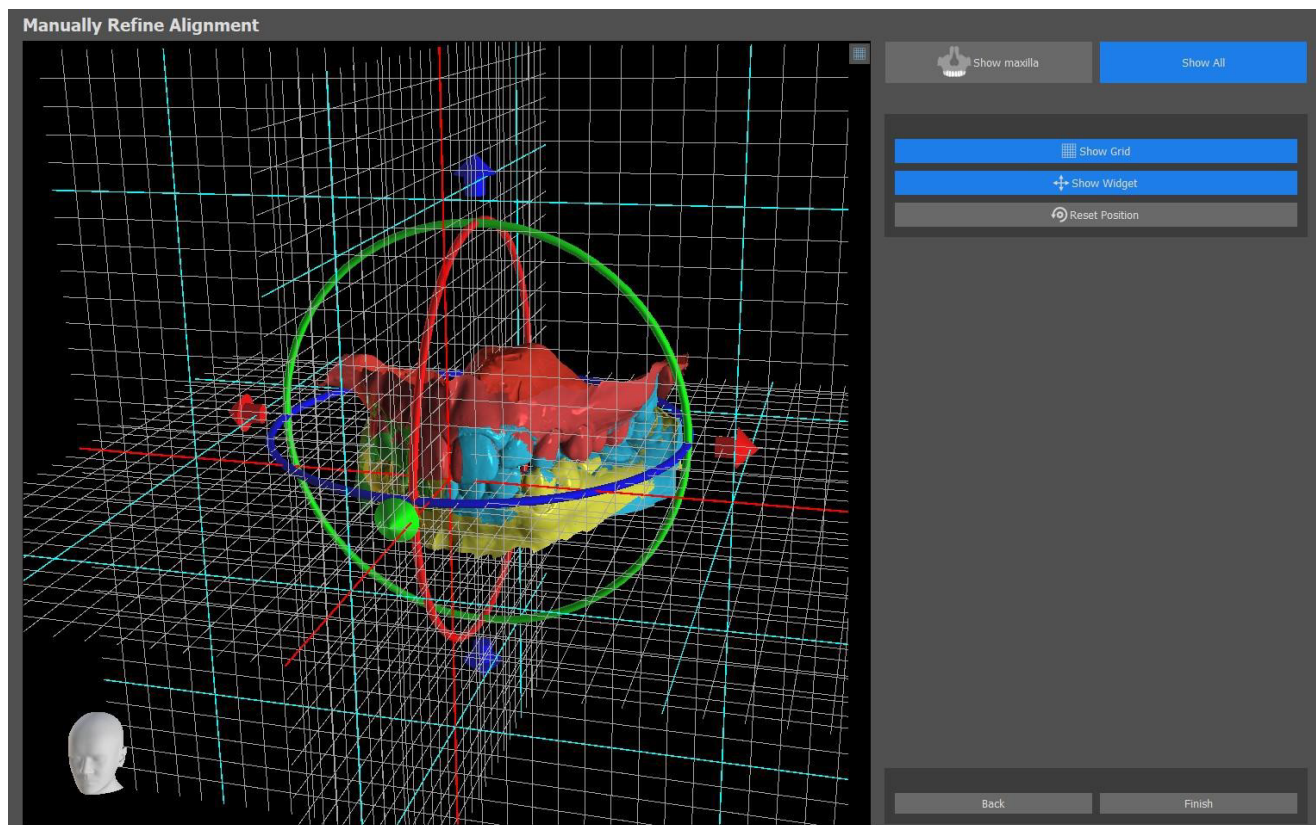

13 STL model adjustment

Blue Sky Plan software allows the user to select and remove part of the surface (model) and close any holes which have been created during the process.

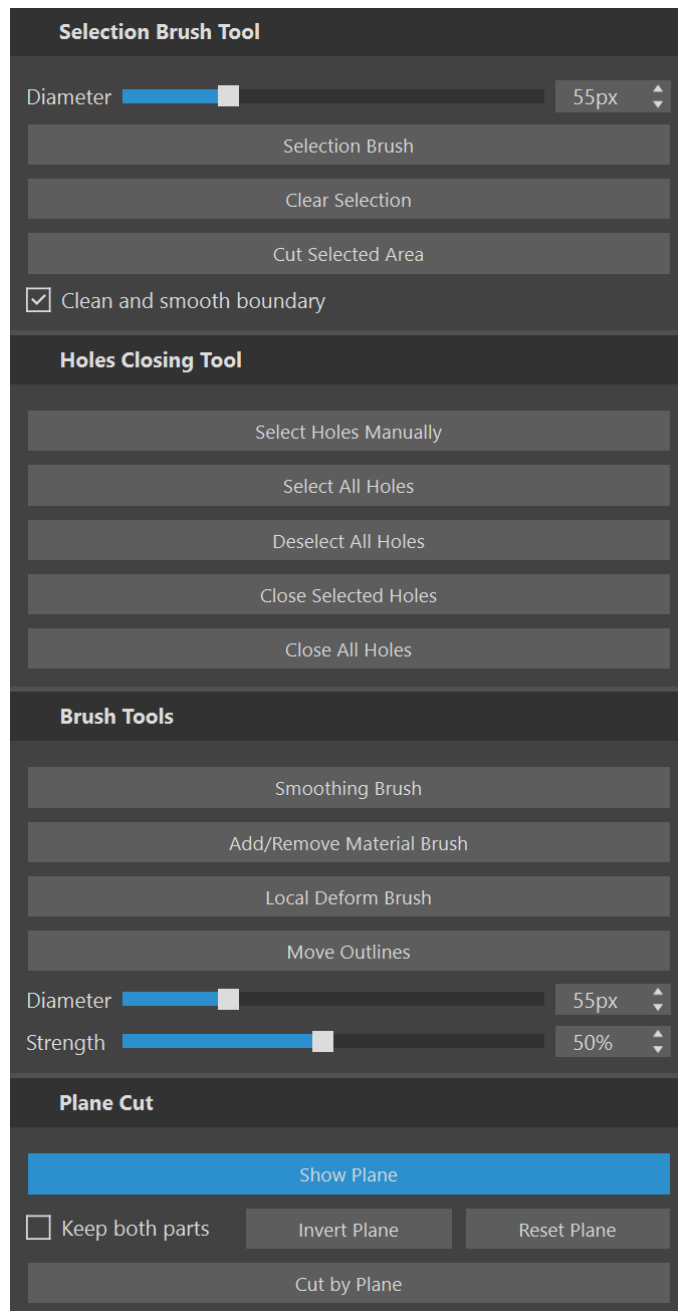

All the necessary tools are available in the Model Edit panel. Using the “Selection Brush” the user can select a part of the surface and remove it by pressing the “Cut Selected Area” button. This way the user can clean the surface from scuttered parts or “extract” a tooth, for example. The selected area can be cleared by pressing the “Clear Selection” button.

Removing parts of the model, however, will create a hole (or several). The “Holes Closing Tool” can be used to close them. The user can select and close one specific hole using, or select and close all of them at once. All selected holes can also be deselected.

Note: This tool is made for closing smaller and simpler holes. More complicated cases may take a long time. The model can be completely closed and repaired using the “Close Model” tool, which also creates a stone model with a base.


Surface irregularities can be fixed using “Move Outlines”, which can only be used in the orthogonal view (it doesn’t work in the 3D view). “Add/Remove Material Brush” is used for the addition/removal of surface material. “Local Deform Brush” deforms the surface at a local point. For final modifications and polishing the surface of the closed hole, “Smoothing Brush” can be used.



“Plane Cut” is used to cut a model by a cutting plane. The plane can be moved and rotated and the cutting direction can be inverted. The plane can be reset to the default horizontal position. Both parts of the cut model can be retained (a new Plane Cut remainder model will be created).

14 Scan Body Registration

Blue Sky Plan® has a useful functionality that allows the user to register the STL files of the scan bodies present in the BSP parts catalog into the scan bodies visible in a scanned model.

Blue Sky Plan® can convert the scan body (type of abutment that serves as a "post" at the time of the scan) into an actual abutment with an implant attached.


Once the STL model with visible scan bodies is imported, clicking on the  icon in the toolbar (while "Normal" or "Advanced" mode is active) opens a dialog for fitting the catalog scan body to the scanned model.

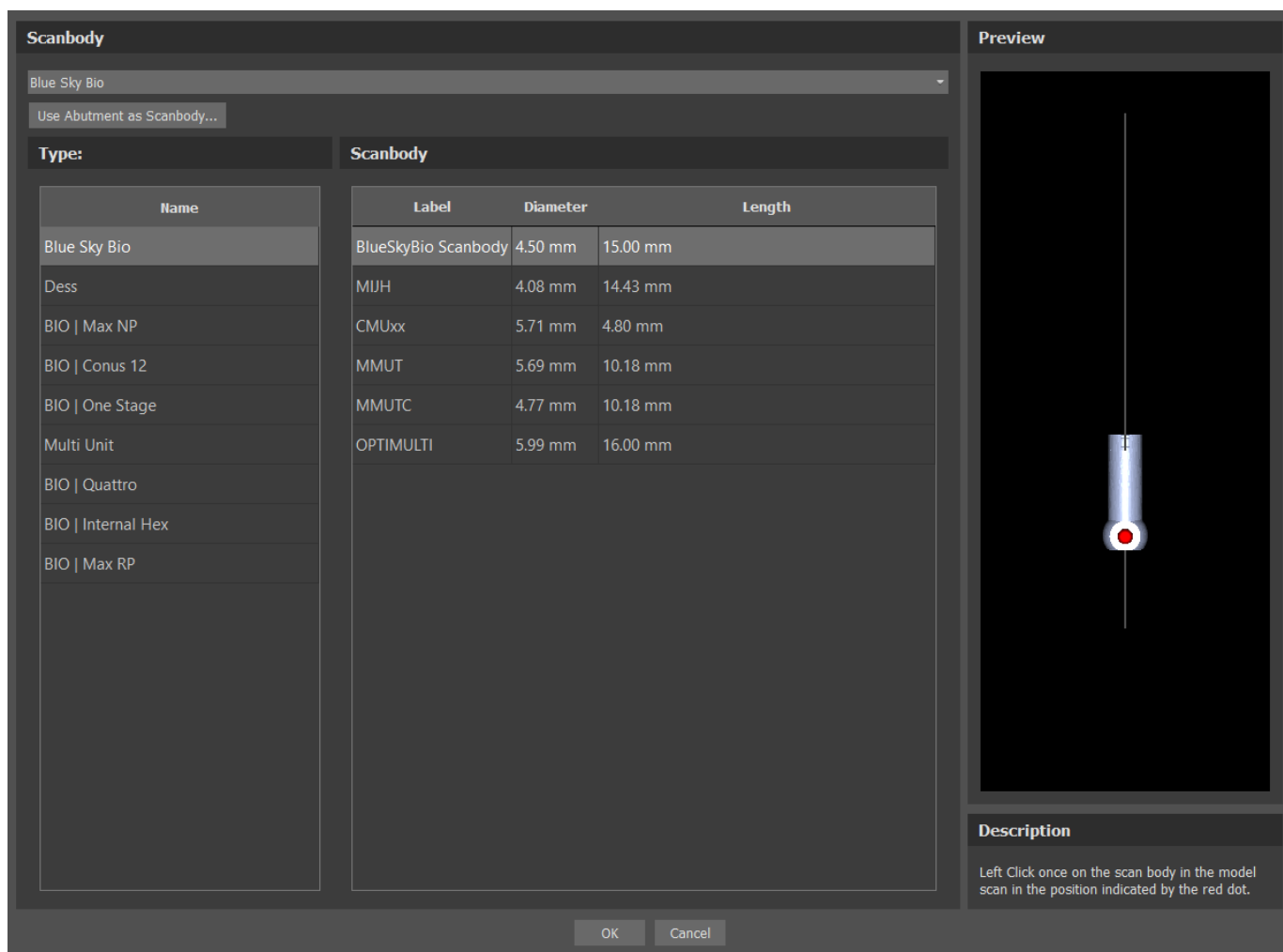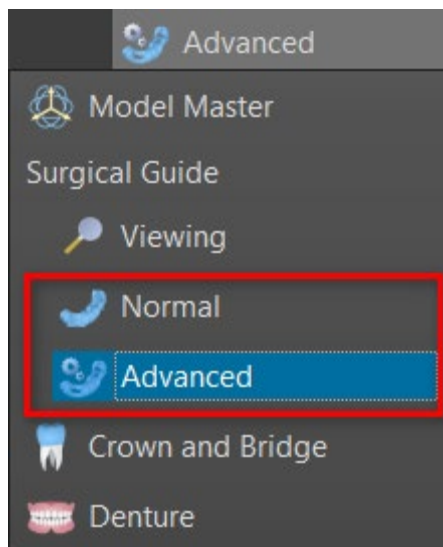

After choosing a scan body from the catalog, it can be placed by clicking on the head of the scanned abutment in the model – for the best fit position, it is good to click on the place illustrated by the red marker as shown in the catalog scan body preview.


Once the scan body position is precisely defined, the window with implants pops up. After choosing the desired implant, the scan body and implant are automatically aligned together. Tooth can be added by clicking "Add Tooth" from the toolbar or by right clicking the scan body > Add Tooth. Another option for aligning a tooth is by right clicking the implant > Align > Align New Tooth.

It is also possible to align either an existing or a new implant with the scan body by right clicking on the scan body and choose the desired option from drop-down menu.

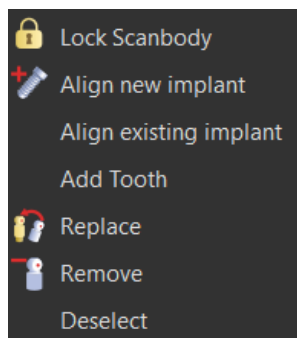

Later on, the scan body position can also be refined manually using the common manipulation widget in the 3D view.


Anonymous patient

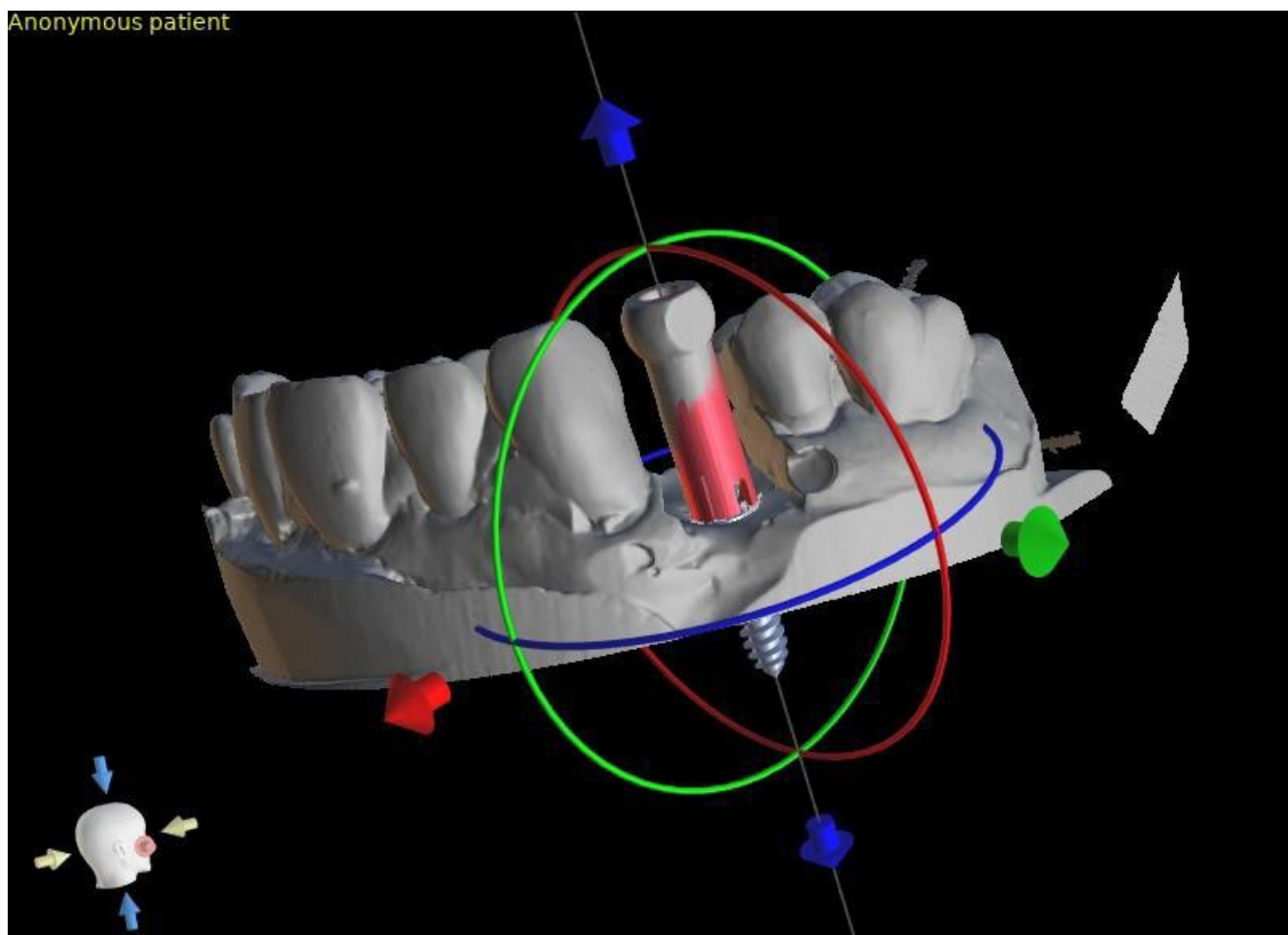

15 Virtual teeth

Blue Sky Plan® makes it possible to insert "virtual" teeth into the project. Using automatically generated crowns or pre-prepared crowns from catalogs, we can show the patient the possible result after all the planned procedures have been performed. Virtual teeth also make it possible to align either a new or an existing implant and thus easily visually inspect the respective position of the resulting crown with respect to the position of the implant and to plan any abutment.


15.1 Inserting crowns

It is highly recommended to specify dental curve, since the inserted crown will be rotated automatically with respect to it. The individual crowns are inserted either by clicking on the  icon in the main toolbar or by using the button in the "Tooth List" side panel (indicated below), or, if necessary, by choosing the "Virtual Teeth > Add Tooth" menu item.

A dialog for selecting crown will open. Firstly, the user can choose between two types of crowns: automatically generated crowns and predefined crowns from catalogs. Note that automatically generated crowns are available only in the Surgical Guide module.

For the automatic creation of a crown, select the "Automatically create AI tooth" tab (1). Then choose the associated tooth from the preview (3) - the text description of the crown appears in (4). Implants can also be placed together with the new teeth (5). For more information about placing implants, see chapter 8. Confirm the selection by clicking the "OK" button (6) (the "Cancel" button closes the dialog without making a selection (7)). Then user can place the crown on the associated spot and the crown is automatically generated. The dimensions and location of the tooth can be subsequently modified directly in the sequence.

For a predefined crown from the catalog, select the "Select tooth from catalog" tab (2). Compared to automatic creation of the crown, there are a few differences in the dialog. From the dropdown menu, it is possible to select the teeth from a specific teeth geometry set (8). In the preview, it's possible to select additional teeth by pressing and holding the Shift or Ctrl key (9). The size of the teeth can be adjusted as well (10). A tooth can be aligned to an already existing implant (11), or the teeth's position can be adjusted on the basis of the panoramic curve height (12).

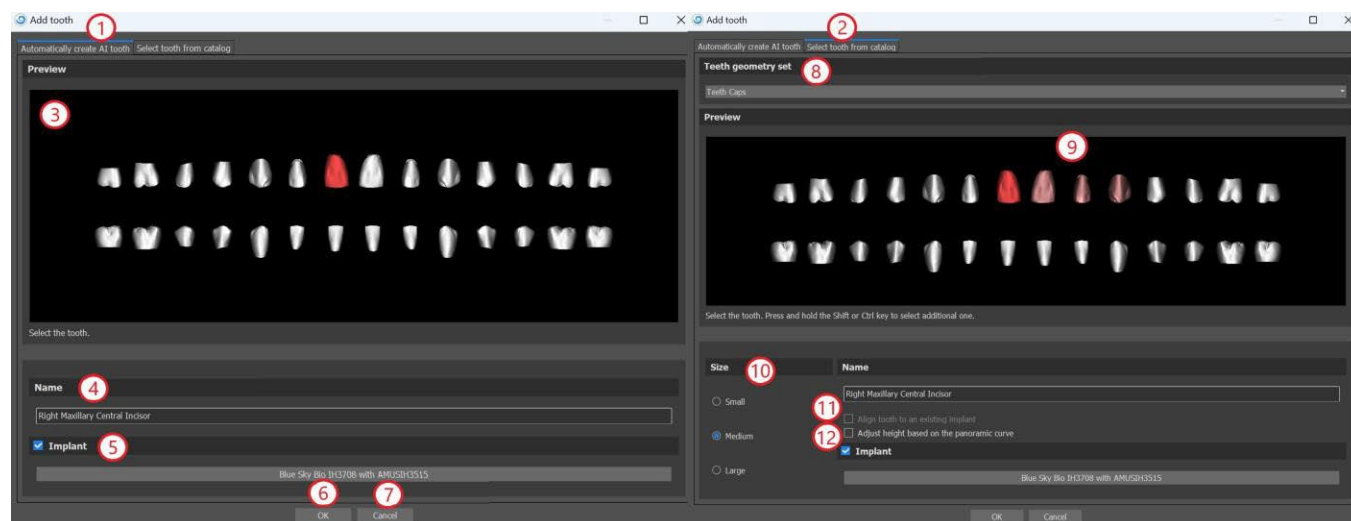


15.2 Positioning the crown and manipulating it


The mouse cursor changes to a cross when the dialog closes. The crown can then be positioned by clicking on the slice, and, if necessary, on the 3D visualization in any window.

The crown is positioned in a vertical position according to the type of the simulated tooth and rotates with respect to the dental curve. It is up to the user to fine-tune its position.

The tooth can be moved in all the windows. This is achieved by positioning the mouse on the tooth geometry, clicking the left mouse button, and dragging. The tooth always moves in the plane of the view. The tooth can also be rotated using the so-called draggers. These appear again when the mouse cursor is positioned on the tooth. The dragger can then be grabbed by clicking the left mouse button; the dragger changes color, and the tooth's rotation is modified by dragging.

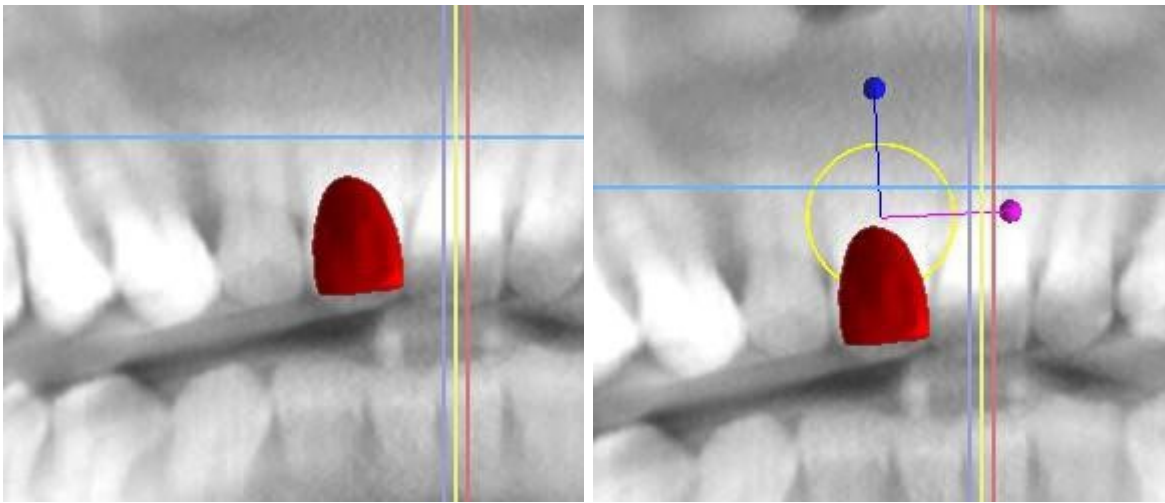


The rotation point (the so-called Pivot) can either be in the center of the tooth geometry or at the beginning of the local coordinate system. This is located in the same place as the head of the aligned implant.

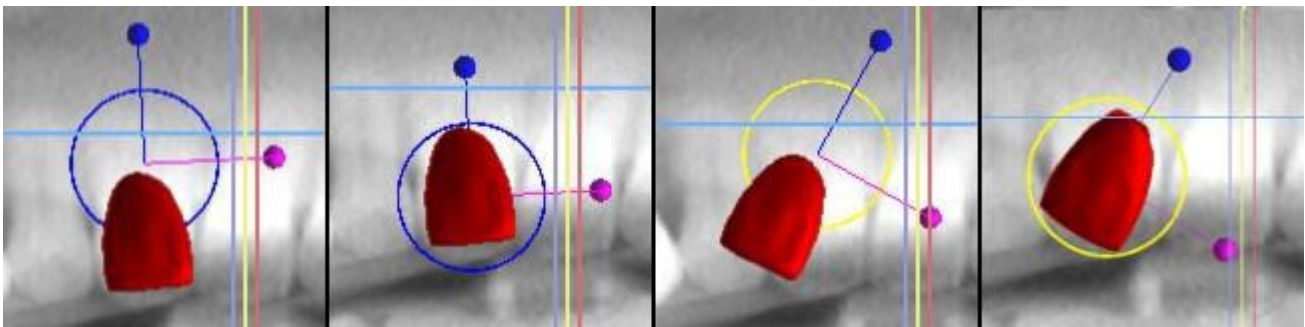

In the 3D view, the tooth can be dragged not only in the plane of the view but also along the axis of the tooth. As in the other views, it is necessary to move the cursor onto the tooth and display the draggers. The blue arrows above and below the tooth serve to shift it along the axis. We change the position of the tooth by moving the cursor to the arrow, clicking the left mouse button, and dragging. As for implants, three differently colored circles appear in the 3D view for rotating the tooth around the corresponding axes of the local coordinate system.

In addition to the position and rotation, the size of the crown can also be modified in the 3D view. Spheres displayed along the axis of the tooth serve to change the size. The size of the tooth can be changed by moving the cursor to it, clicking the left mouse button, and dragging.

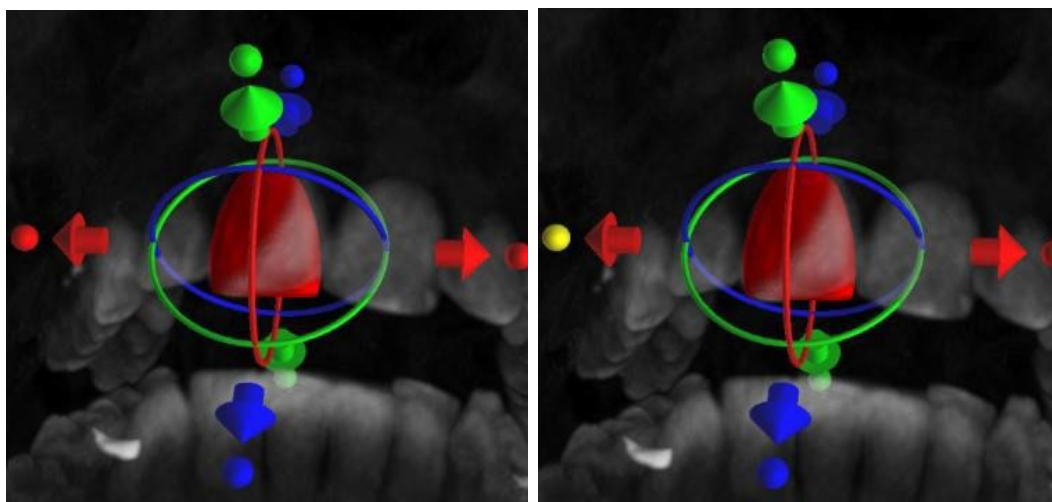


15.3 Adding the AI crown

To generate an AI tooth for a case where both jaw models are imported, first select a tooth from a Tooth list, then pick an antagonist. This antagonist model must be aligned in occlusion with the model with the missing tooth.

Antagonist selection can be done through a dropdown menu which will appear right after selecting an AI tooth in the Tooth List and pressing the "OK" button. Clicking the "Cancel" button cancels jaw selection, not tooth generation. It is necessary to click on the Cancel button again.

Regardless of whether the opposite jaw is available, it is possible to regenerate an AI tooth. Right-click on the AI tooth and select "Regenerate AI Tooth" from the context menu. The mouse cursor changes to a tooth. Left-click on the model in the desired tooth position.

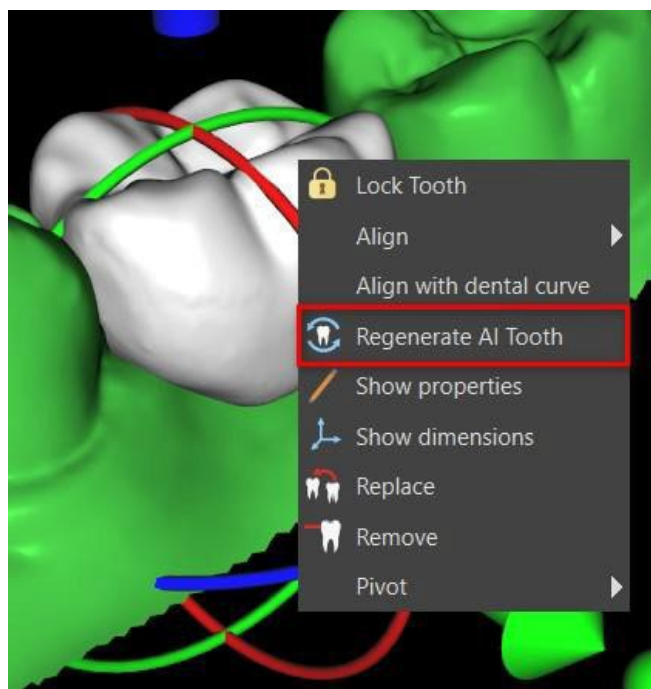


15.4 Aligning an implant and further operations

The side panel, which can be opened with the “Panels > Tooth List” menu option, provides further operations on the tooth. A list containing a description of the tooth (1), a check box (2) for turning the visibility of the individual tooth on and off, and a button to remove the given tooth (3) appear in this panel. The selected tooth is displayed in red in the sequence, and the associated row is highlighted in the list.

As for implants, it is also possible to set the transparency (4) or outline the display (5) for teeth.

There is also an "Add Tooth" button here for adding a tooth (6), which functions the same way as the toolbar button described above, optionally via a menu item.

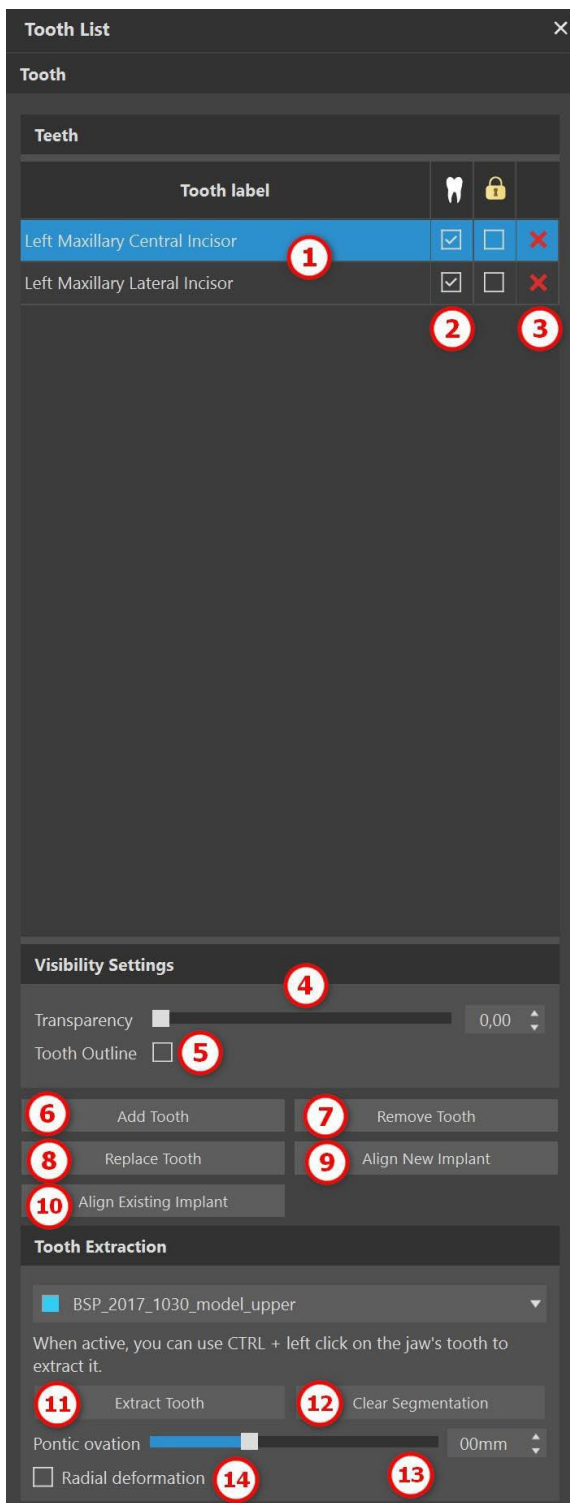

The result of clicking the “Remove Tooth” button (7) depends on whether a tooth has been selected. If so (if one of the teeth is selected), the action is performed directly on the chosen tooth. If not, the selected tooth is removed from the sequence by clicking the mouse. The button for changing the geometry of the tooth is “Replace Tooth” (8). The dialog for selecting the new geometry works analogously: if one of the teeth is selected, the new geometry is set to it; if none is selected, the mouse cursor changes to a selection cross, and the geometry changes to that of the tooth selected by clicking on it.


The "Align New Implant" button (9) first opens a dialog for selecting the implant (see chapter 7), and after the selection is confirmed with the "OK" button, the new implant is added to the project with a position setting corresponding to that of the selected tooth.

The button for aligning an existing implant (10) works analogously. When it is pressed, the mouse cursor changes to a selection cross, and when one of the existing implants is clicked on, its position with respect to the tooth is modified.

The tooth can be extracted in the "Tooth Extraction" section. By clicking the "Extract Tooth" button (11), segmentation starts on the selected scan and a segmented model is created. When segmentation is active, it is possible to use CTRL + left click on the jaw's tooth to extract it. Segmentation is deleted by clicking the "Clear Segmentation" button (12).

Radial deformation (14) affects the shape of the resulting ovate pontic. When turned on, the amount of surface deformation depends solely on the distance of the surface point from the theoretical center of the tooth, resulting in a symmetric, radius-dependent shape.


The slider (13) then affects the resulting depth of pontic ovation upon the extraction of the tooth.

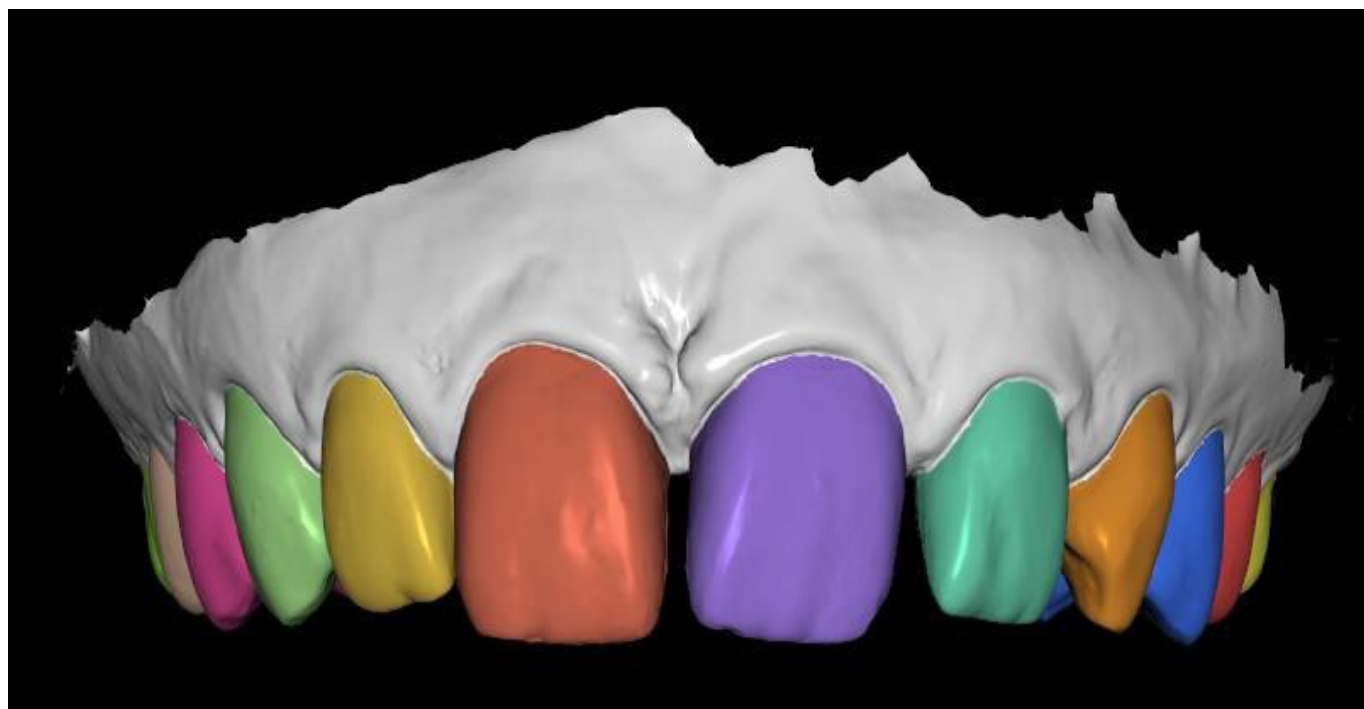

Model prepared for tooth extraction


15.5 Context menu

Some operations can also be performed via the context menu. We invoke this by clicking the right mouse button on the tooth geometry. Besides the operations described above, such as aligning the implant, removing the tooth, or replacing it with another type, it is also possible to group a tooth with an implant and move them together across the scene, and also to choose the point of rotation of the draggers (the Pivot) as described in chapter 14.2.

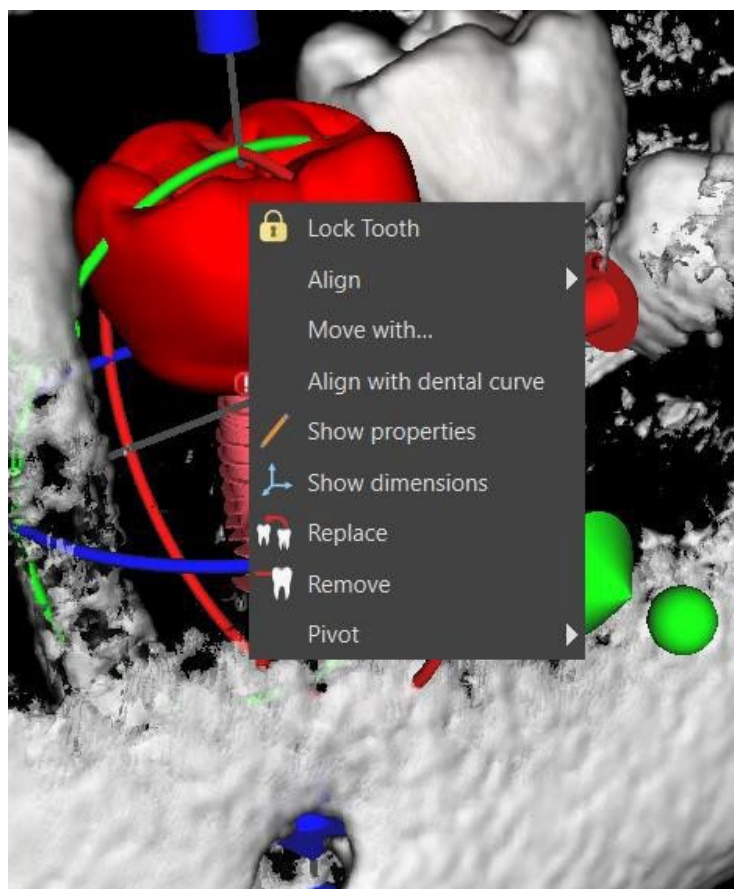

16 Tooth and jaw segmentation


Blue Sky Plan® makes it possible to segment and remove individual teeth and jaws from the data. The following segmentation methods are supported. Segmentation is available in the Model Master module > Panels > Segmentation. Automatic segmentation for both tooth and jaw is accessible via Model Master > Tools > Automatic Teeth Segmentation or Automatic Jaw Segmentation.

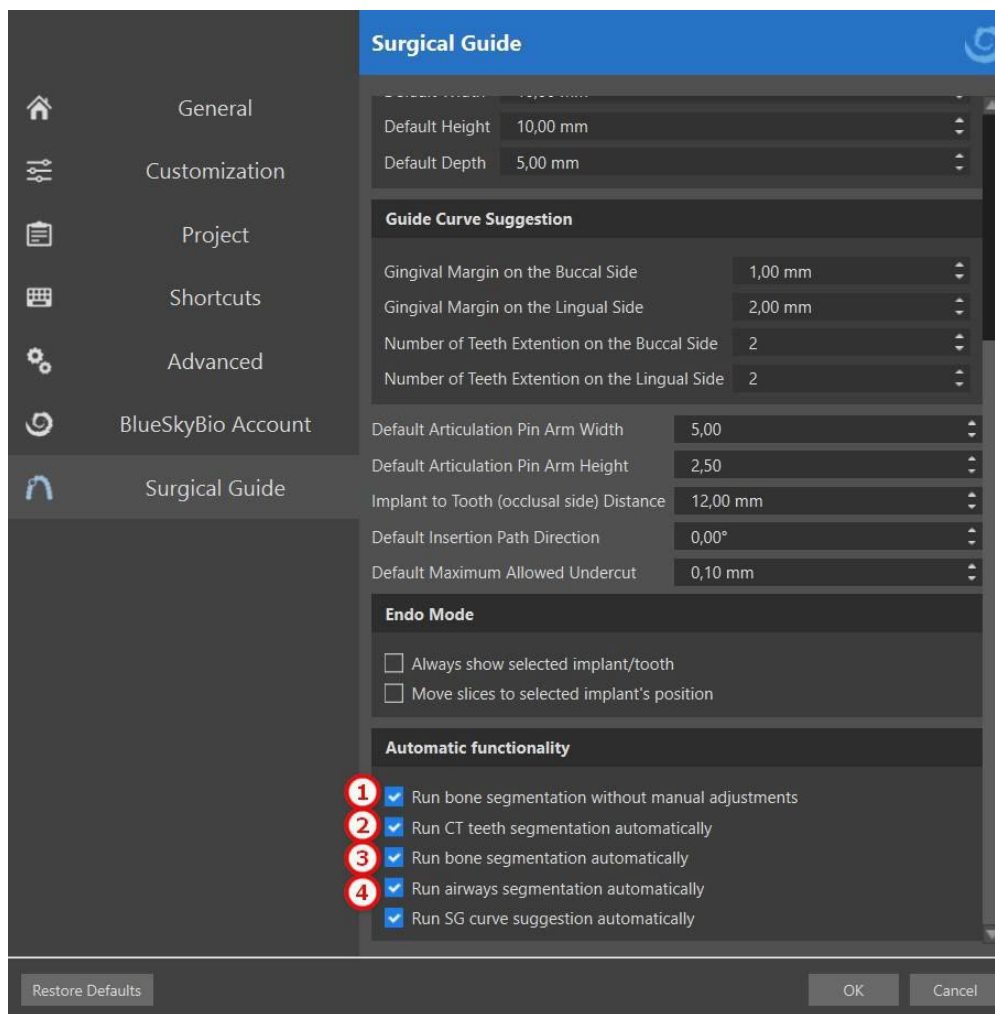


Segmentation settings can be adjusted in Preferences > Surgical Guide > Automatic Functionality. "Run Bone Segmentation without manual adjustments" (1) won't show the user manual editing tools during the jaw segmentation process. The other three checkboxes will start specific segmentation process after the CT data are imported. "Run CT Teeth Segmentation automatically when importing CT scan" (2) will only start teeth segmentation. "Run Bone Segmentation automatically when importing CT scan" (3) will start teeth and jaw segmentation (note: this will overwrite the "Run CT Teeth Segmentation automatically when importing CT scan" checkbox). "Run Airways Segmentation automatically when importing CT scan" (4) will start Airways segmentation.

16.1 Automatic Tooth Segmentation


Using the Segmentation functionality, you can also use „Automatic Tooth Segmentation“. Two options are available – Select Tooth and Create STL Surfaces for All Teeth.

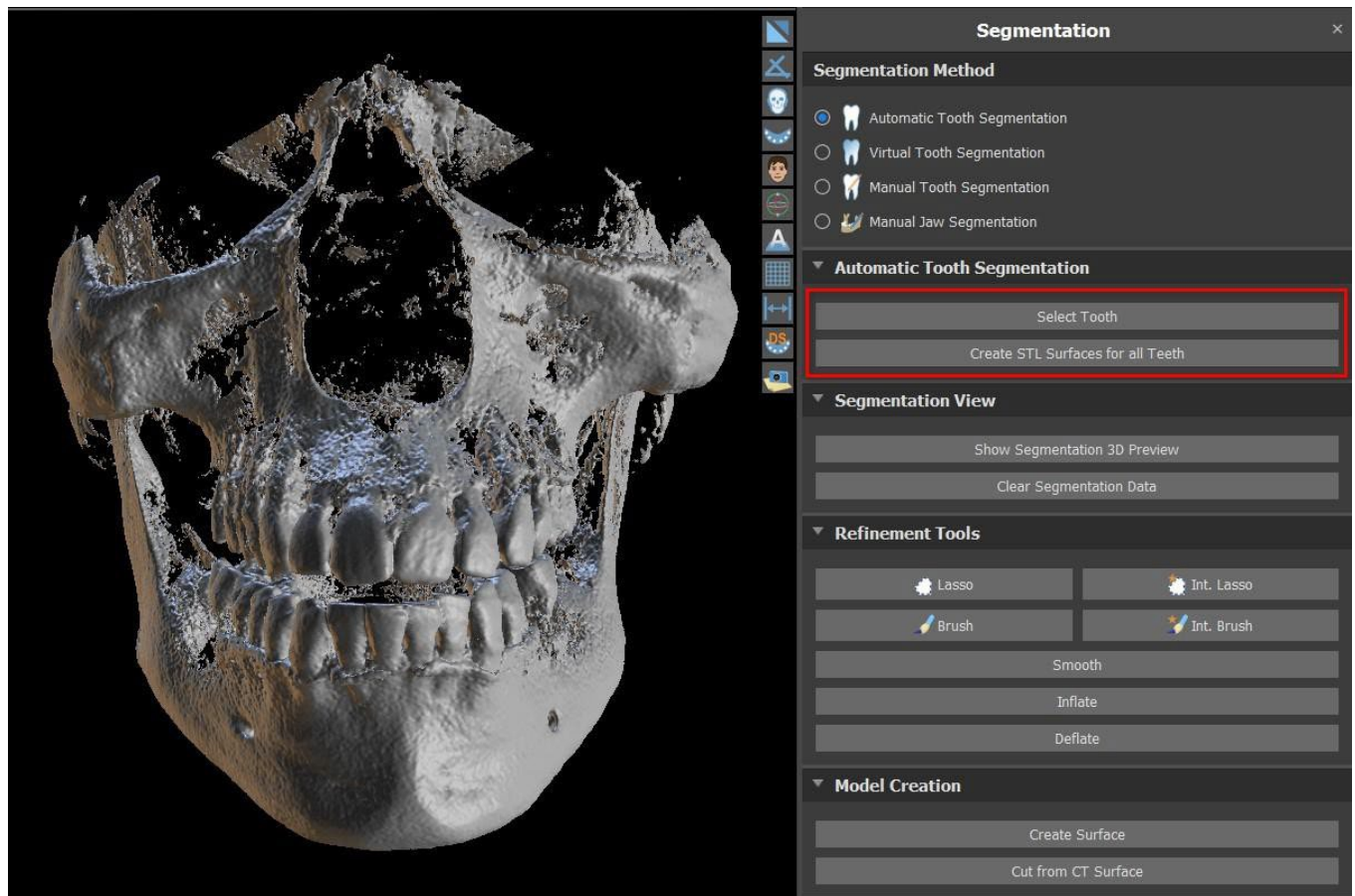


To work with one tooth segmentation at a time, click on Select Tooth button (1) and choose a tooth from one of the orthogonal slices (2) you need to select. The result of the selection will be colored in all the slices. A preview of the selected tooth can be shown in 3D by clicking on Show Segmentation 3D Preview (3). Tooth selection can be removed from the orthogonal slices by clicking on Clear Segmentation Data.

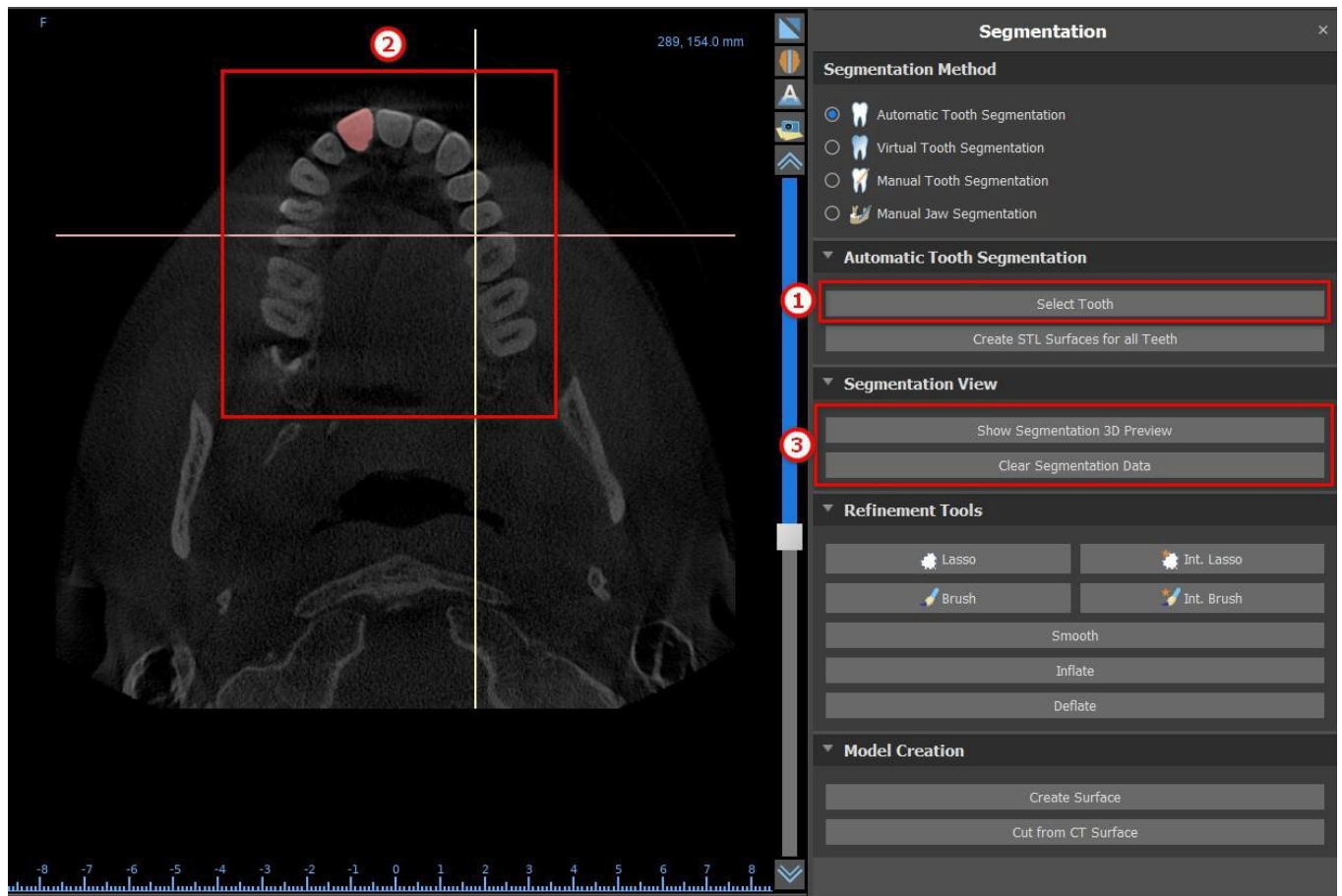


Tooth selection can be corrected with the "Refinement Tools" (4). It is also possible to apply a "Smooth" filter on the segmentation data. "Inflate" inflates data by one voxel while "Deflate" shrinks data by one voxel.

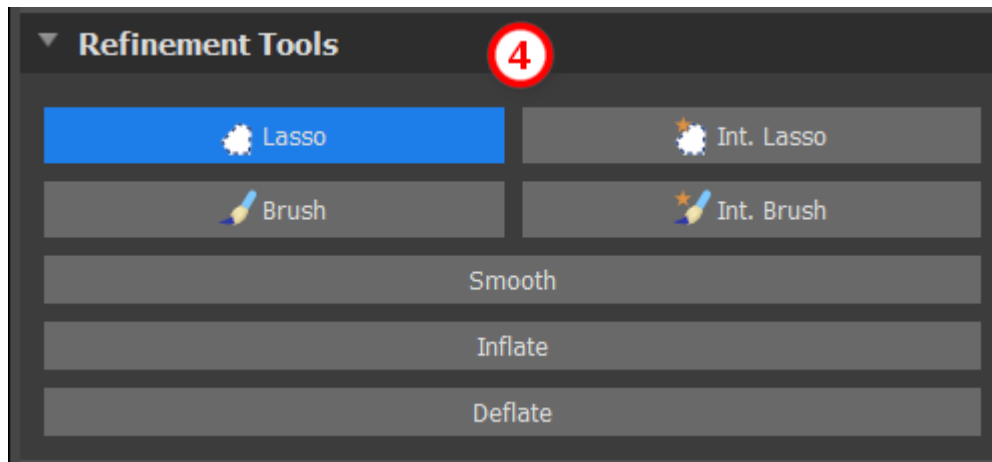


To create a 3D model from the segmentation data, click on "Create Surface" (5). To create a new CT surface with the current segment cut out, click on "Cut from CT Surface".

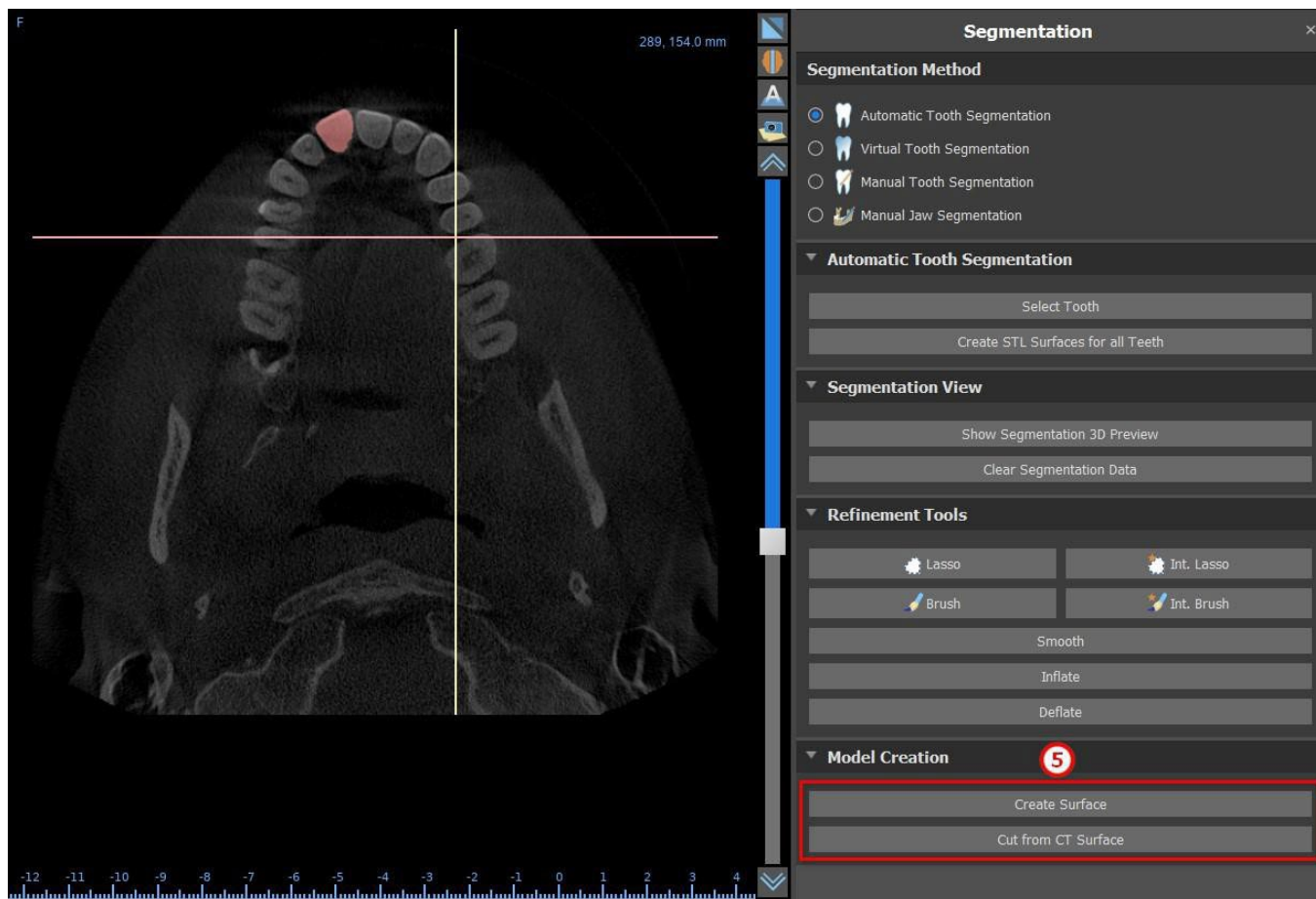

If you want to run fully automated segmentation, click on “Create STL Surfaces for All Teeth” (6).

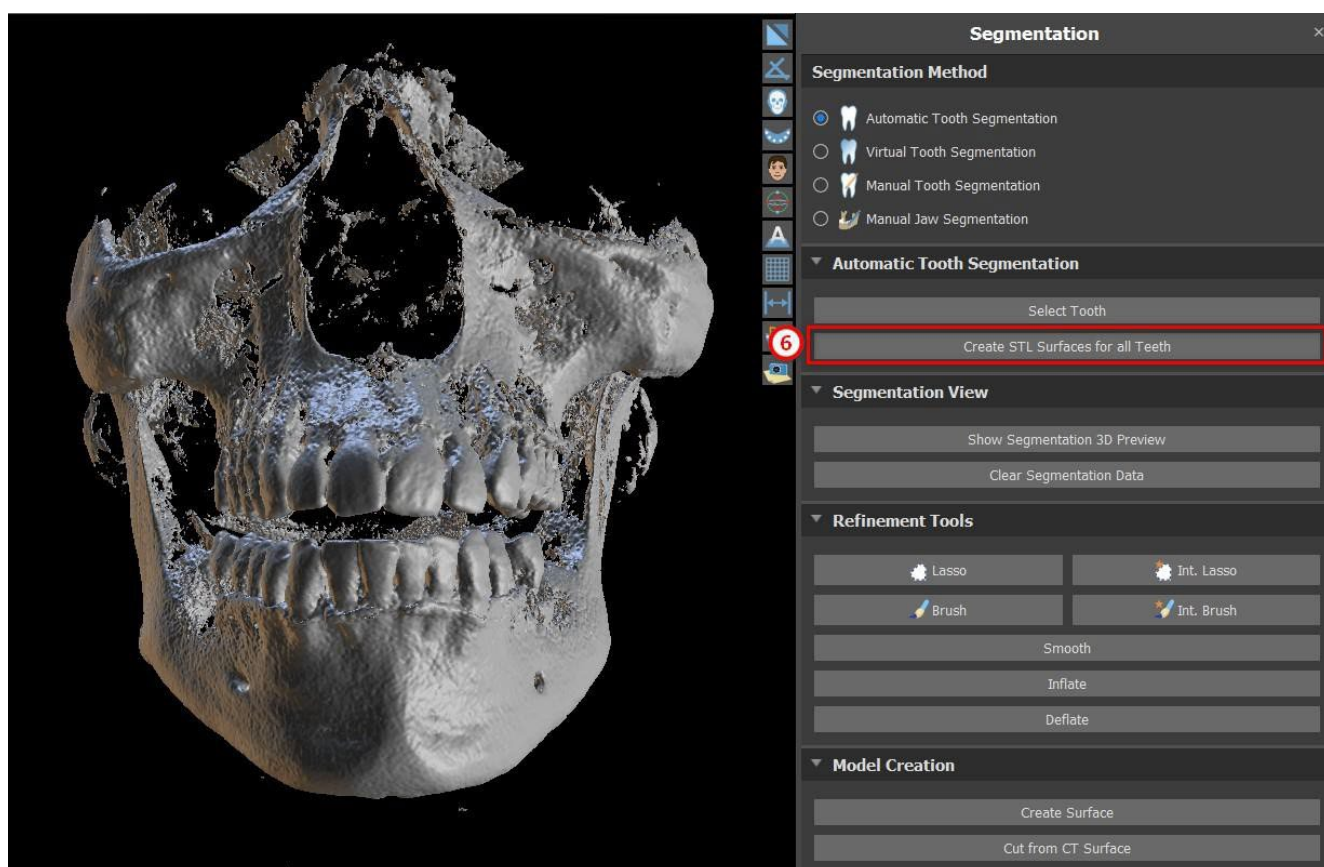


You can check fully segmented teeth in the Surfaces panel.

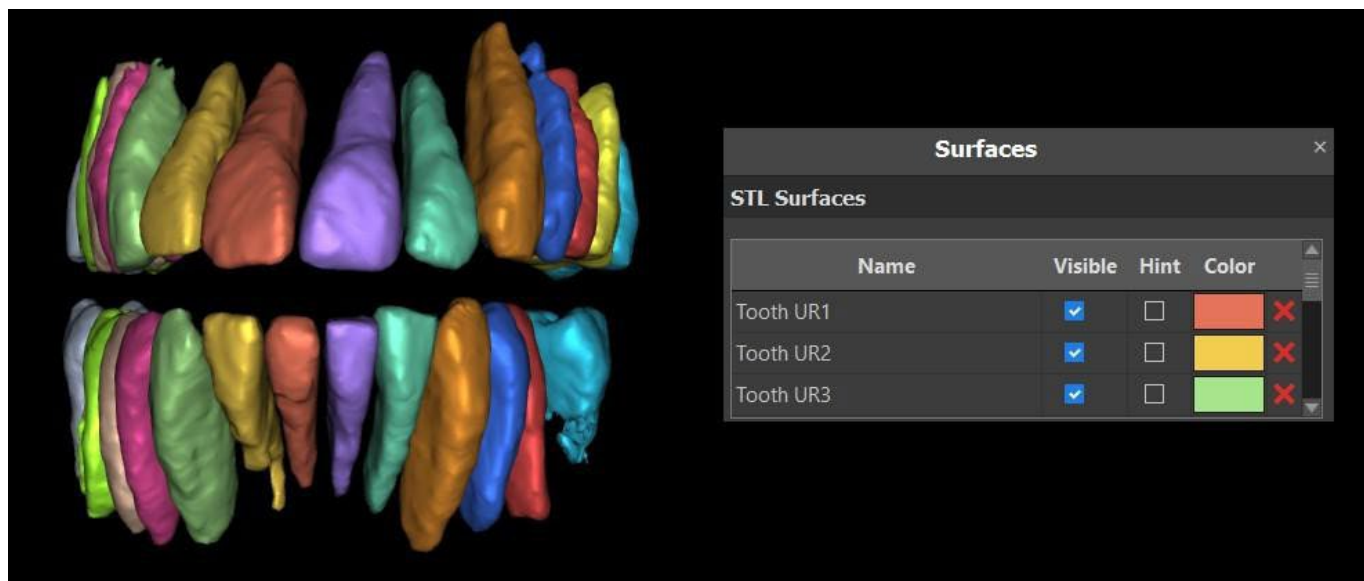

16.1.1 Implant Segmentation


To perform implant segmentation, select "Segment CT implants when automatically segmenting CT teeth" in Preferences > General > Automatic Functionality. Segmentation will occur when segmenting the jaw or a tooth. It is necessary to have CT data with implants already added in the laboratory, it is not possible to do segmentation on data where only a virtual implant has been added.

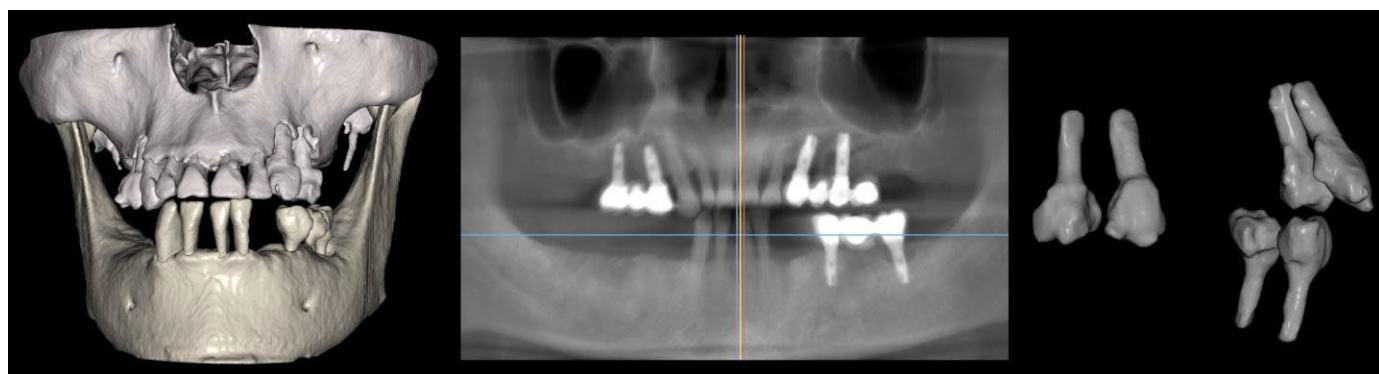

Merged teeth and implants (left) and segmented implants and teeth (right)

16.2 Virtual Tooth Segmentation

If automatic delimitation of the tooth fails, it can be selected in the data using the virtual tooth manually positioned in the data via the "Virtual Teeth > Add Tooth" option from the main menu. After positioning the model, perform the "Use Active Tooth Model" action for the designated data on the basis of the tooth model entered. Using the "Model Neighborhood Parameter" it is possible to slightly correct the area around the positioned model that has been selected.

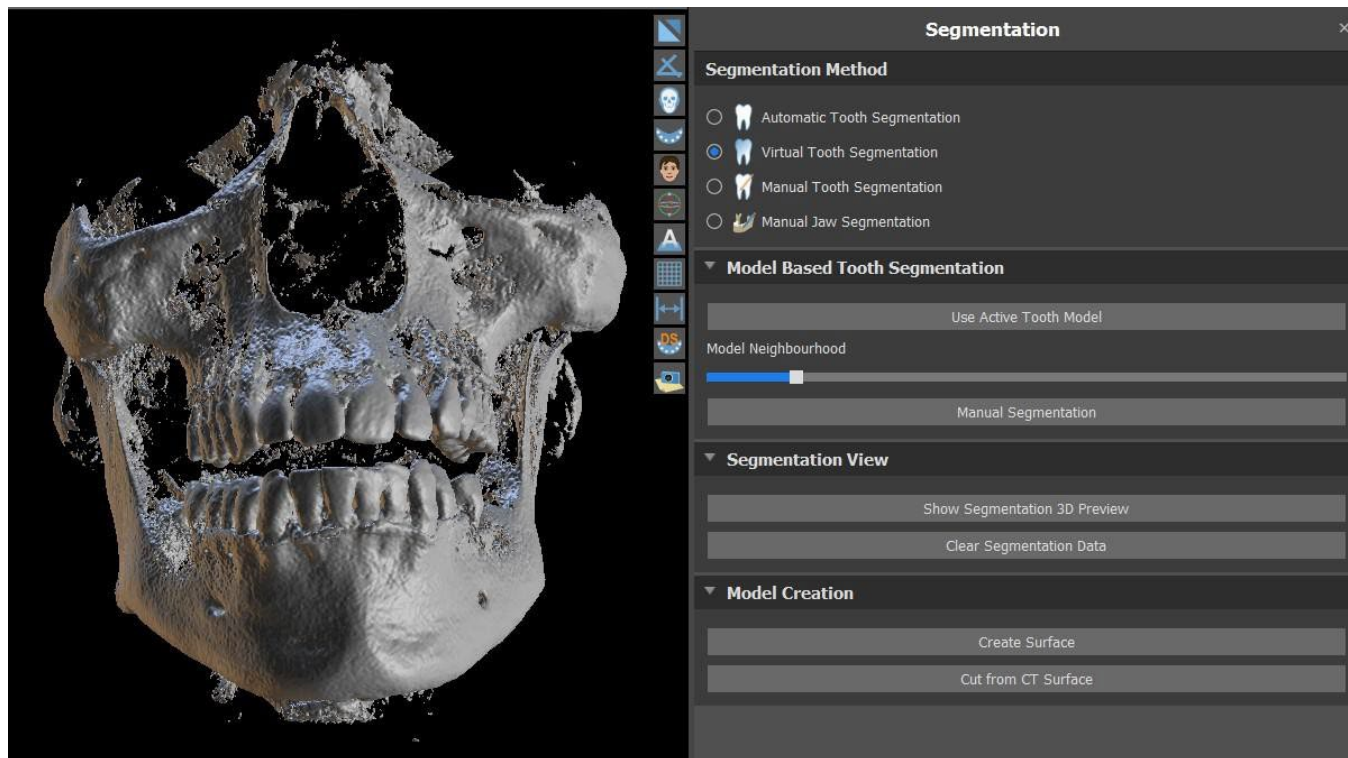


It is possible to transfer the area of the tooth marked in the data into the model by clicking "Create Surface". A model created in this way appears in the "Surfaces" panel in the "Imported Surfaces" table.

The tooth can be removed from the data by clicking "Remove Tooth". If you are not satisfied with the result, the action you carried out can be reversed with the "Undo" button, and the tooth selection parameters can be modified.

16.3 Automatic Jaw Segmentation

Automatic Jaw Segmentation is available for the user in the Segmentation method section. Simply click on the option (1) and choose between the mandible or maxilla or both jaws (2). Teeth and jaw will be separated into individual models if selected (3). Start the segmentation by clicking the "Start Automatic Segmentation". It might take a few minutes. If "Airways" is selected, Airways segmentation will also occur (see chapter 16.4).

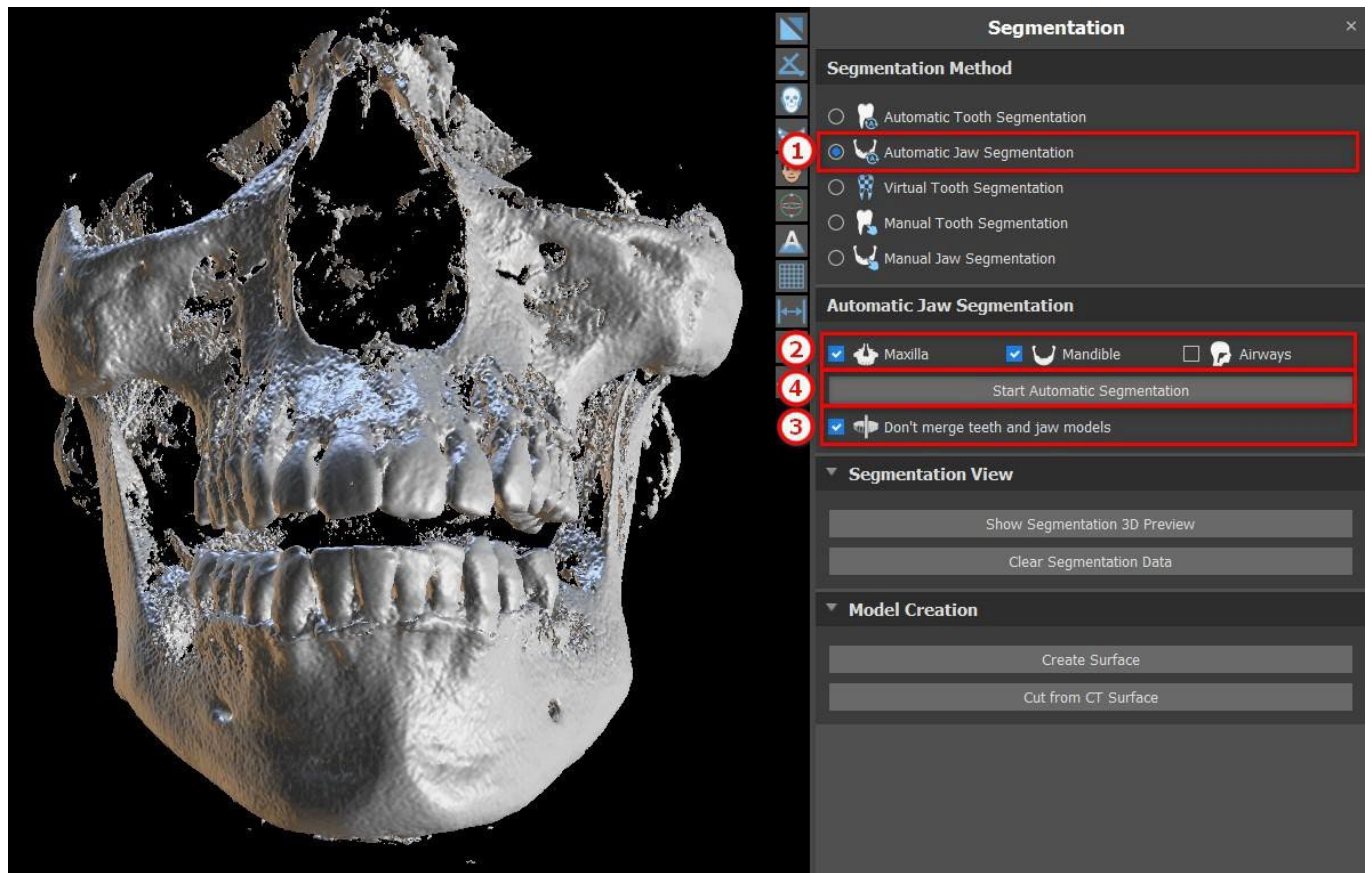


The results of the jaw segmentation can be corrected with the segmentation tools (5). The tools can only be used on a jaw preview. Segmentation will be completed by clicking the "Next" button (6).




Click on the "Create surface and continue" button (7) for surface creation. It is possible to go back and use the segmentation tools again. In this case, click on the "Previous" button (8). "Start New Segmentation" (9) starts new segmentation for the jaw.

The user is able to show or hide CT teeth from the 3D scene in the STL Surfaces table (10). The same option applies to segmented implants (11). If a new jaw bone segmentation starts, previous models from STL Surfaces will be hidden. Models have to be removed manually from the STL table.




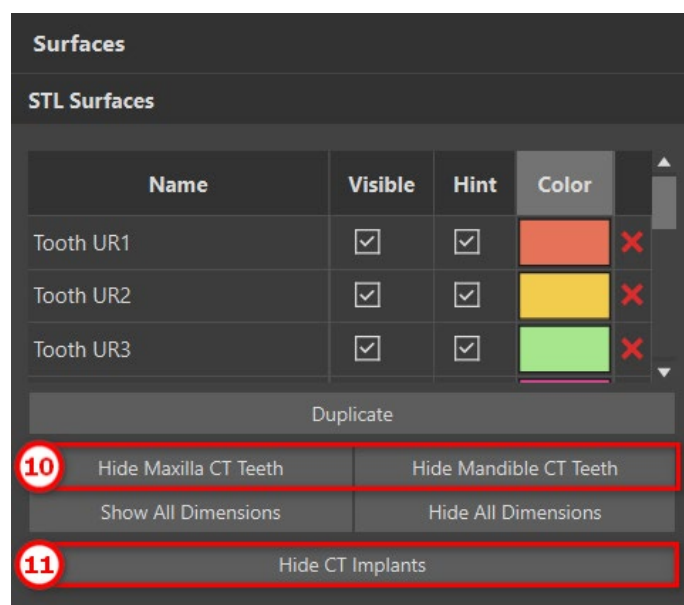


16.4 Airways

Airways segmentation is available in the "Jaw Bone Segmentation" method (1). To run this functionality, select "Airways" (2) and click on "Start Automatic Segmentation" (3).

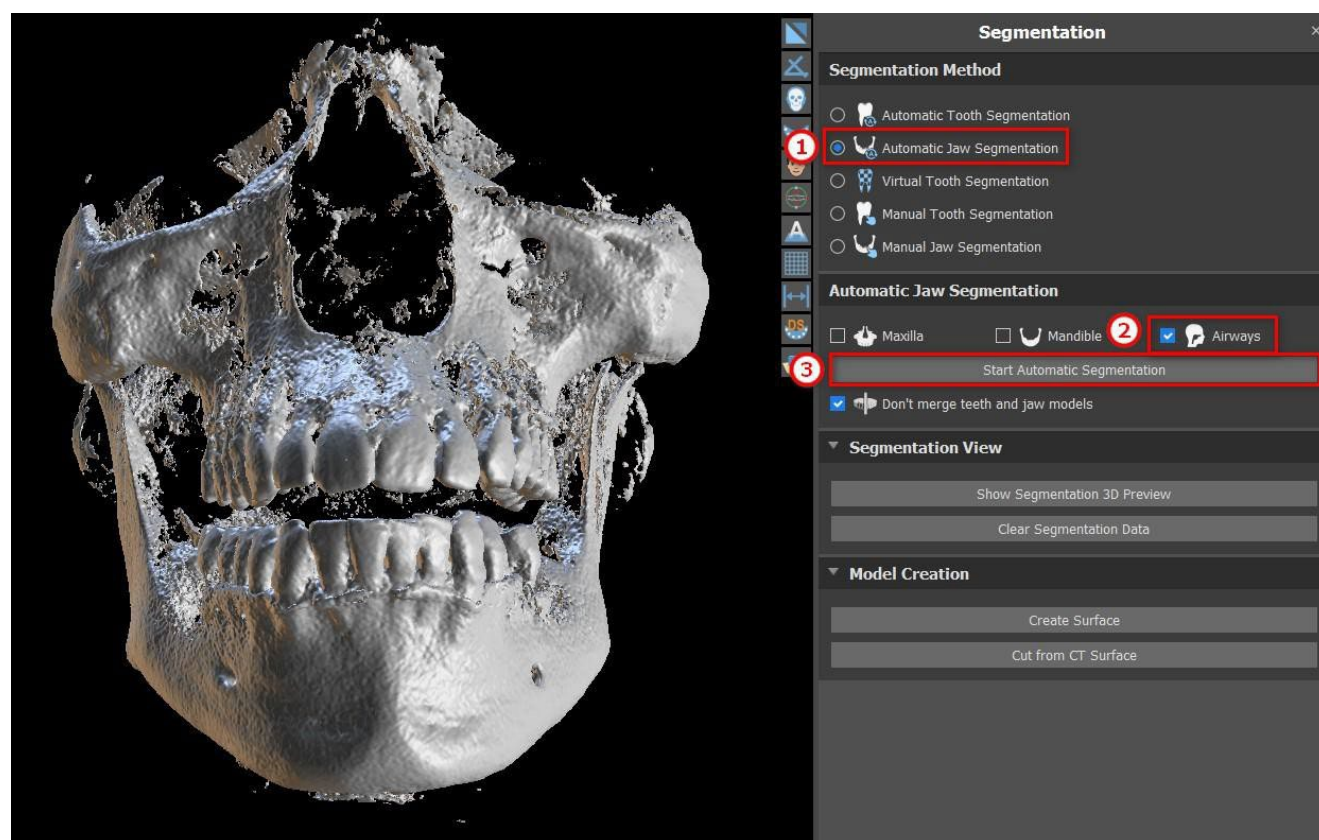


The Airways model is visible in the "Surfaces" panel and can be exported via the Export panel for a fee.

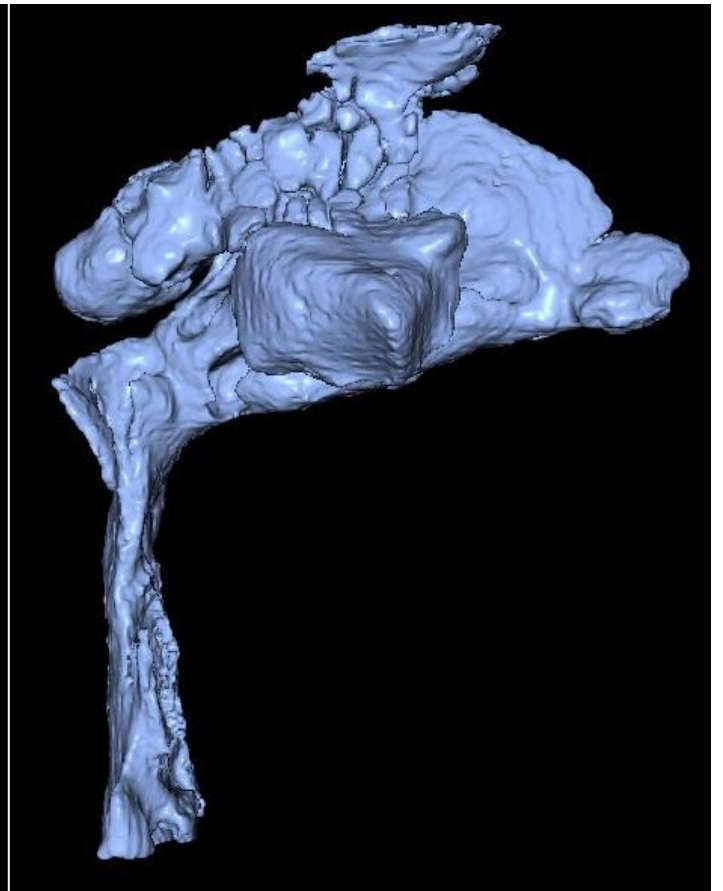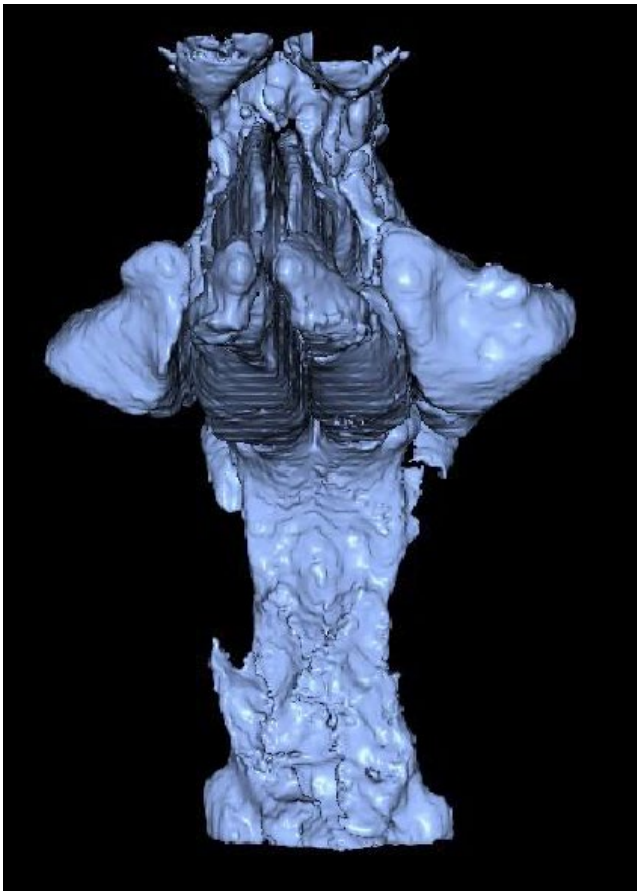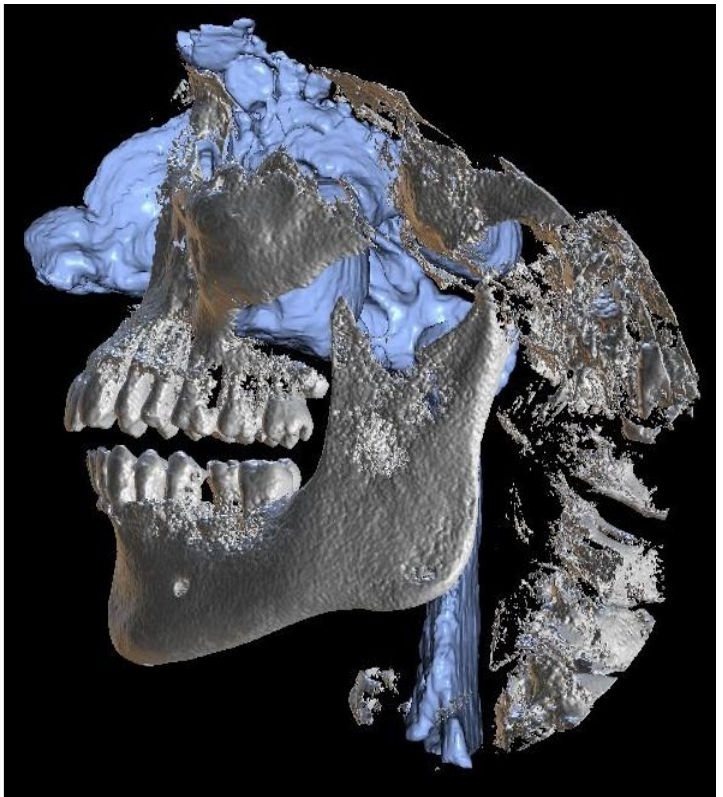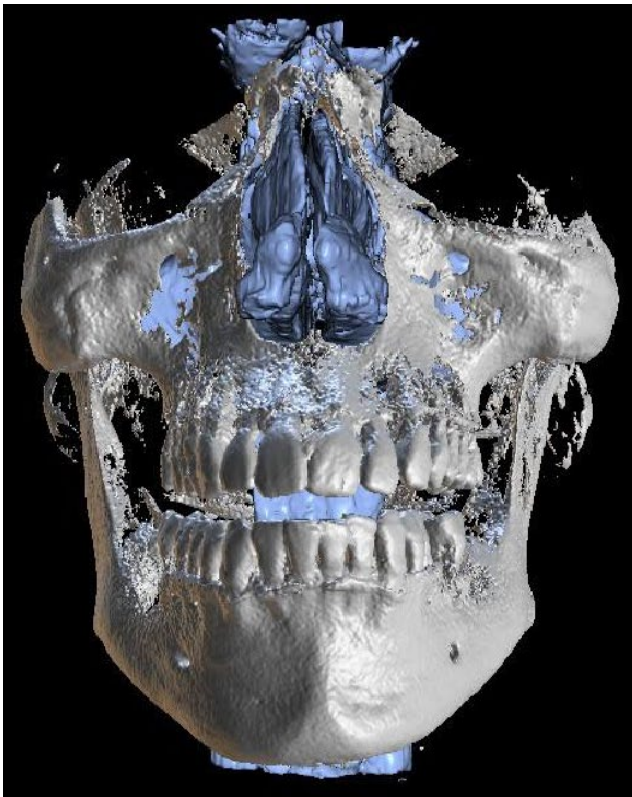

17 Surgical guides

It is possible to fabricate surgical guides in the program on the basis of imported STL models or a so-called scan appliance. It is also possible to edit the surgical guide fabricated. Surgical guides are fabricated using the functions located on the "Guide" panel (1), which can be opened by clicking on the "File (or Panels) > Guide Fabrication" (2) menu item (while "Normal" or "Advanced" mode is active (3)). The tools are divided into groups according to their purpose.

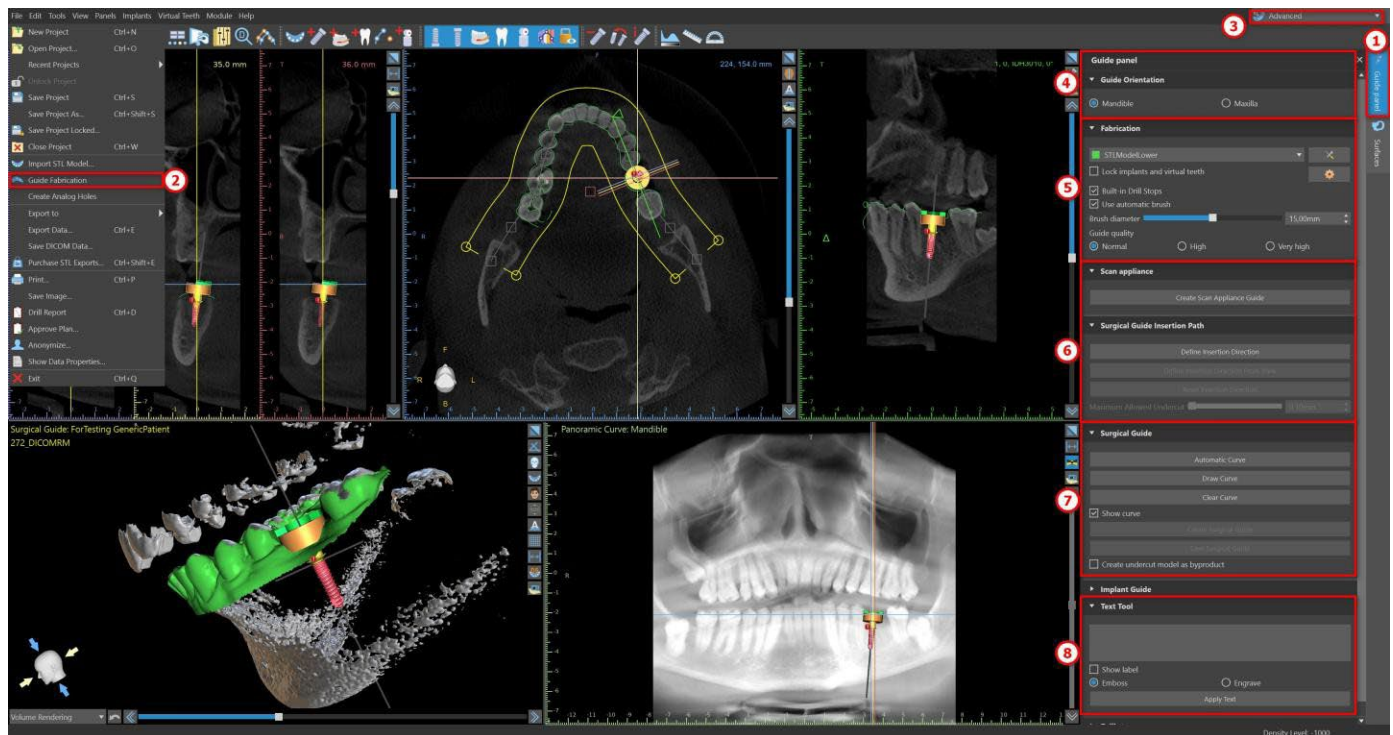


The first group (4) combines elements used jointly for both model types. The second (5) contains a button for fabricating surgical guides on the basis of a scan appliance model. The third (6) contains a function that creates the undercut model. The fourth (7) contains all elements for fabricating an offset surgical guide from a surface model. The fifth (8) contains tools for creating label text. These are followed by panels for creating labels.

The method of fabrication differs according to the type of model being processed.

17.1 Guide fabrication based on scan appliance


Open the "Guide Fabrication" panel. The first section ("Fabrication") contains all common settings for fabricating a surgical guide. The combo box (1) is used to select the input model.


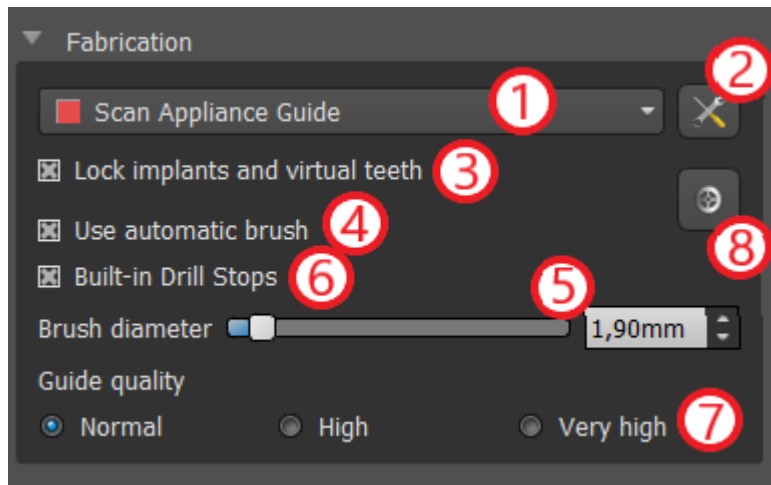


Some input models may be damaged – they are composed of several parts, and their internal structure may not be correct. Fabrication of the surgical guide may fail in these cases. If the input model is damaged, it is possible to try to correct the input geometry using the correction tool (2) and run model fabrication again. When manipulating the model, it is a good idea for the option of working with implants to be turned off so that they are not moved unintentionally, for example. The "Lock Implants" checkbox (3) locks them. Implants are also locked


automatically when using certain tools for fabricating surgical guides.

In order to be able to work conveniently with the resulting surgical guide and so that protruding parts do not prevent you from working with the tools, it is possible to select automatic alignment of the areas above and around the guide tubes. Select "Use automatic brush" (4) for this purpose and then use setting (5) to set the diameter of the free area. If these selections are not checked, only an area of the same diameter as the given guide tube is enabled above the guide tubes.

Automatic brush settings for modifying the software guide tubes can be set in the Guide settings by clicking the  button in the Fabrication section.

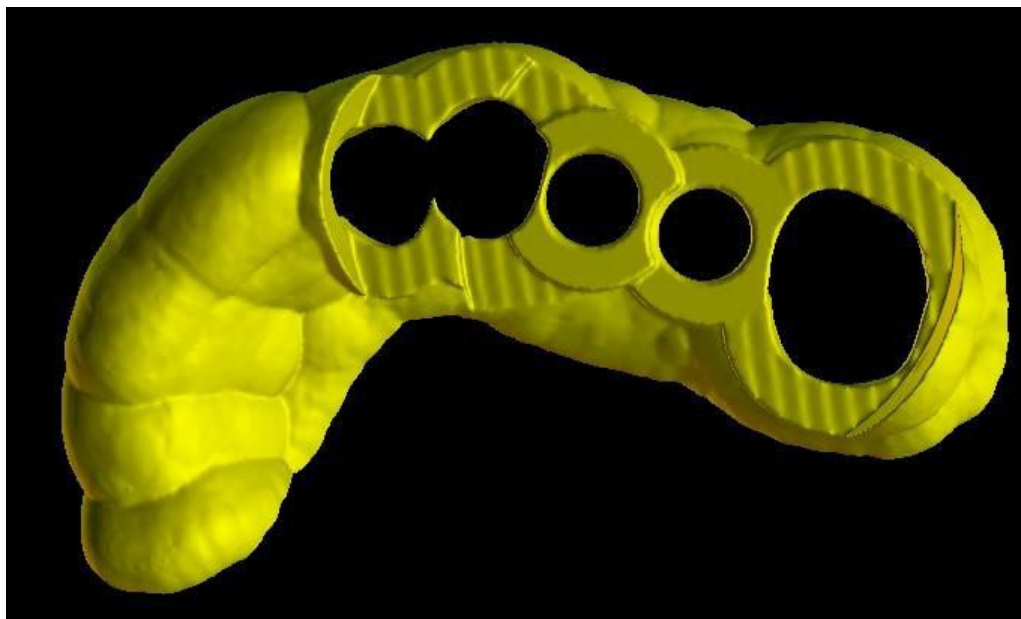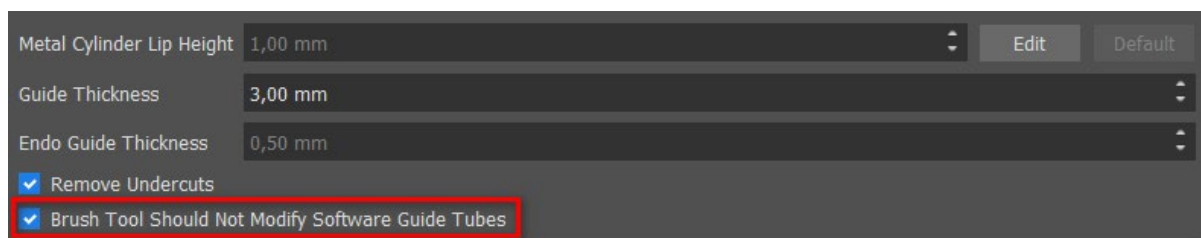

Software Guide Tubes are not modified

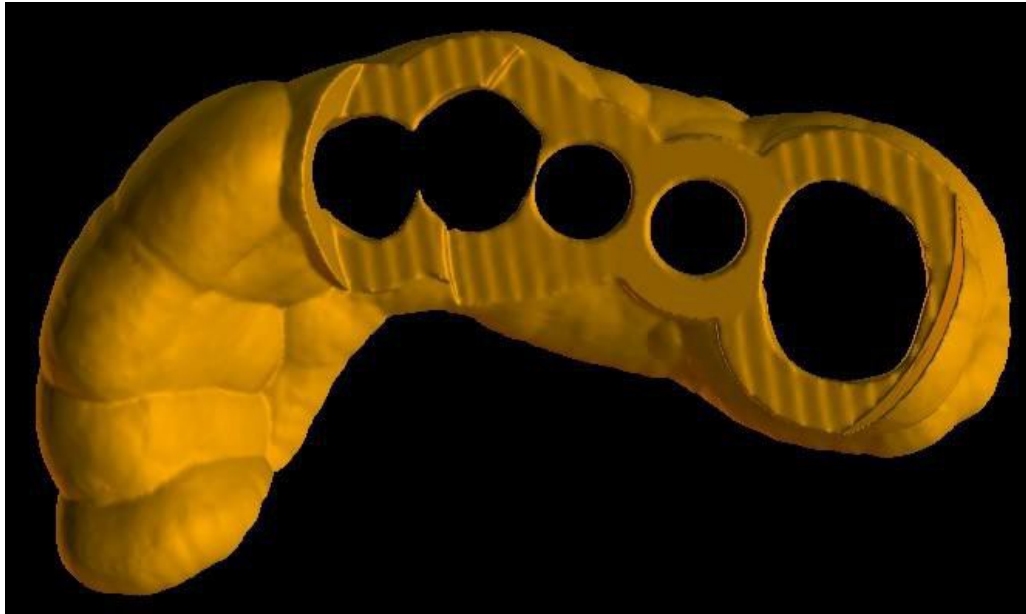


Software Guide Tubes are modified

By using "Built-in Drill Stops" checkbox, it is possible to generate in-built drill stop, which enables the handpiece head to bottom out to create the proper depth into the jaw (6).

The detail of the resulting model is adjustable (7). The general rule is that the more detailed the model is, the longer it will take to generate it, and the slower manual edits with tools such as the Brush Tool will take. A model with the "Normal" setting should be sufficiently detailed for most input data.

Finally, the additional properties of the surgical guide can always be adjusted (8).

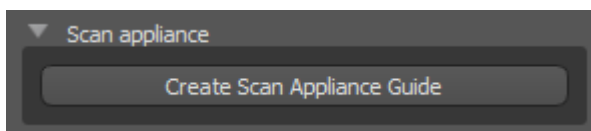


After entering and checking all settings in the "Fabrication" panel, it is possible to proceed to automatic fabrication of the surgical guide. Clicking the "Create Scan Appliance Guide" button in the "Scan Appliance" section runs this. A progress bar is then displayed, and once this runs out, the resulting surgical guide is displayed automatically.

17.2 Guide fabrication based on scanned surface models



It is also necessary to have an input STL model open in this case.

First check whether all the settings in the "Fabrication" panel are correct, i.e. whether the correct input model is selected, whether or not automatic model alignment is used above the guide tubes, and what level of detail is selected for the resulting surgical guide. Further settings specific to fabricating a surgical guide from a surface model are found in the "Guide settings" section (8).

When opening a project, the program tries to guess whether it contains maxillary or mandibular data and automatically makes the selection. The same test is then also conducted when initiating the generation of a surgical guide.

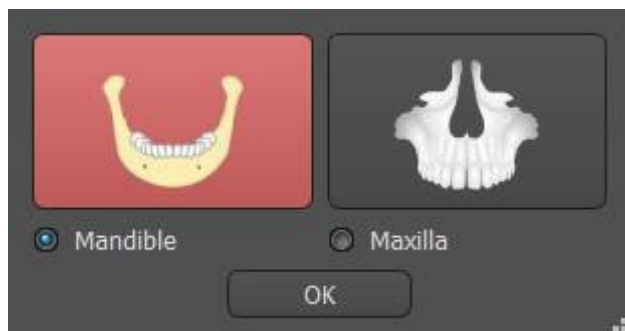


If the program's estimate does not agree with the actual situation, it is possible to change selection manually.

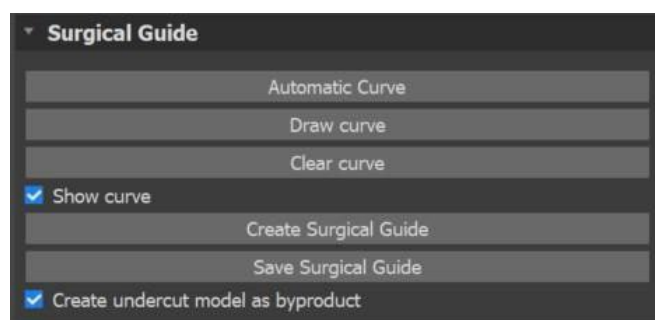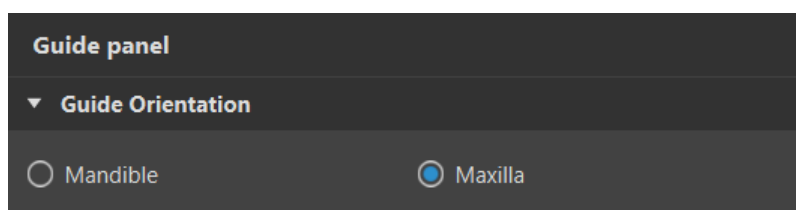


Before actually fabricating the surgical guide, it is necessary to specify the area that should be covered. This is done automatically by clicking the "Automatic Curve" button. If the automatic curve drawing fails, it is possible to draw the curve manually by clicking "Draw Curve". This activates the drawing mode. The state of the button is indicated by a change in color.

If we are in curve drawing mode, it is possible to manipulate the model without pressing any keys.

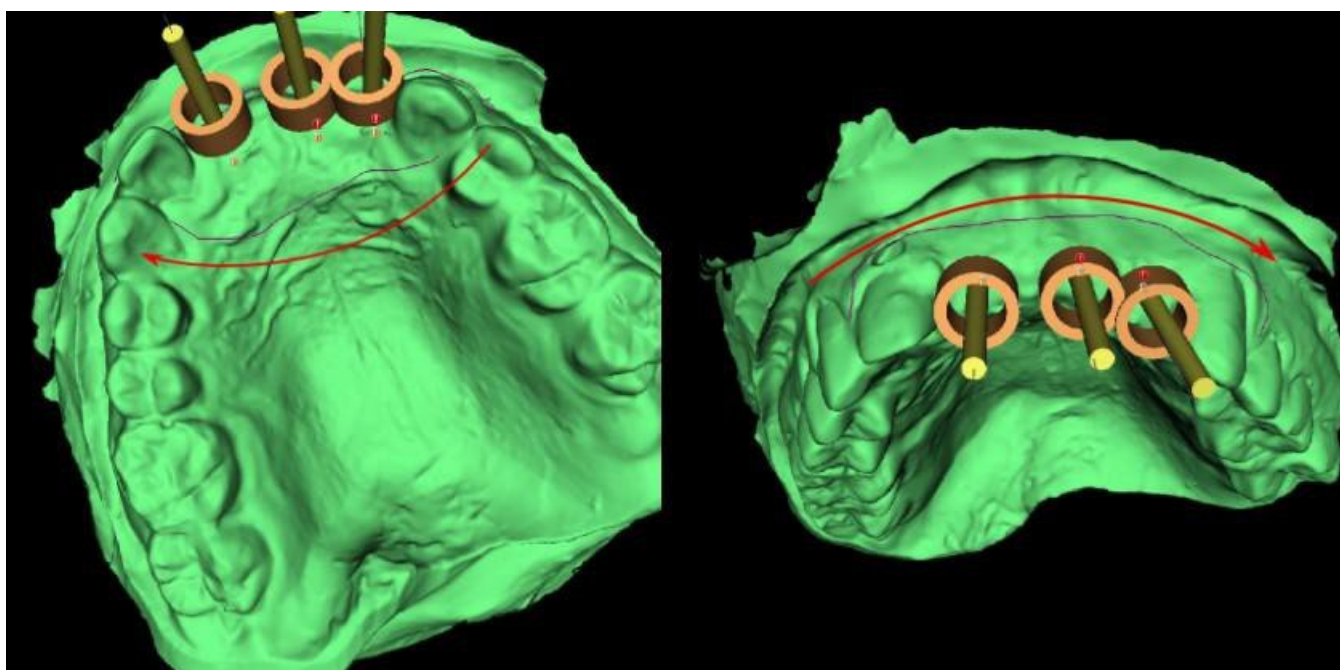


The shape of the curve can be edited using individual control points on it. Before actually generating a surgical guide, it is a good idea to check its shape in this mode. If needed, the curve can be deleted by clicking the "Clear curve" button and the curve can be automatically drawn again.

As a consequence of fragmentation, errors in some input models, and, in some cases, local unevenness in the surface, it may happen that the curve does not correspond exactly to the intended shape. In this case, it is necessary to move the nearest points of the erroneous section so that the resulting outline aligns; see the following images.

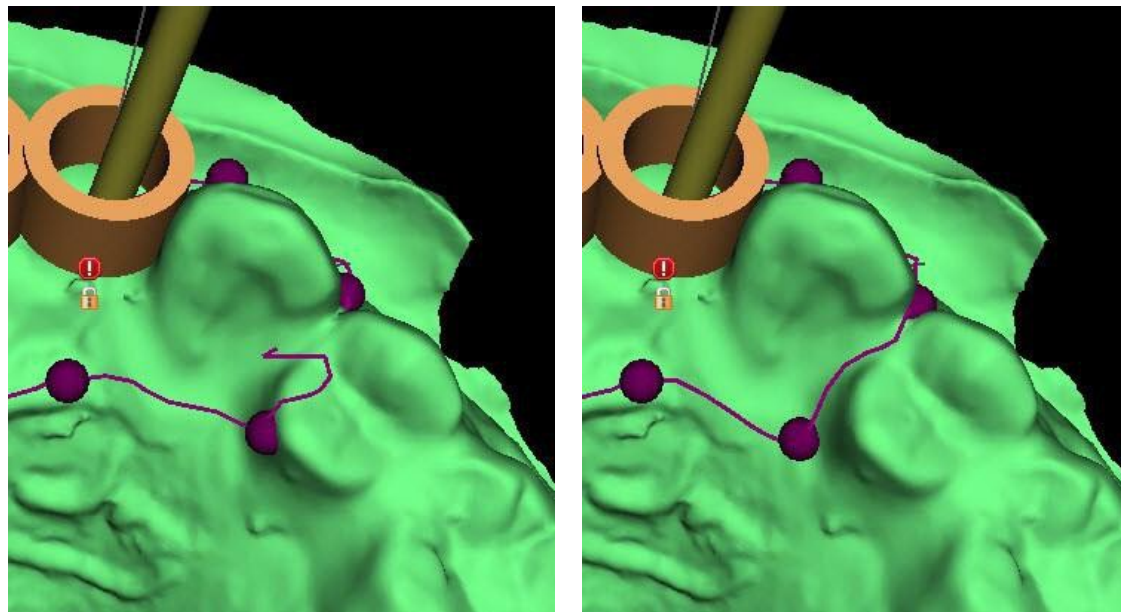

Parameters for the automatic curve can be set in Preferences > Surgical Guide > Guide Curve Suggestion. The suggested curve will be shown based on the settings.

Selecting "External Surgical Guide to contralateral canine and exceed to the most distal tooth" will extend the surgical guide curve from the most distal tooth on the side of the arch, where an implant is located, to the contralateral canine.

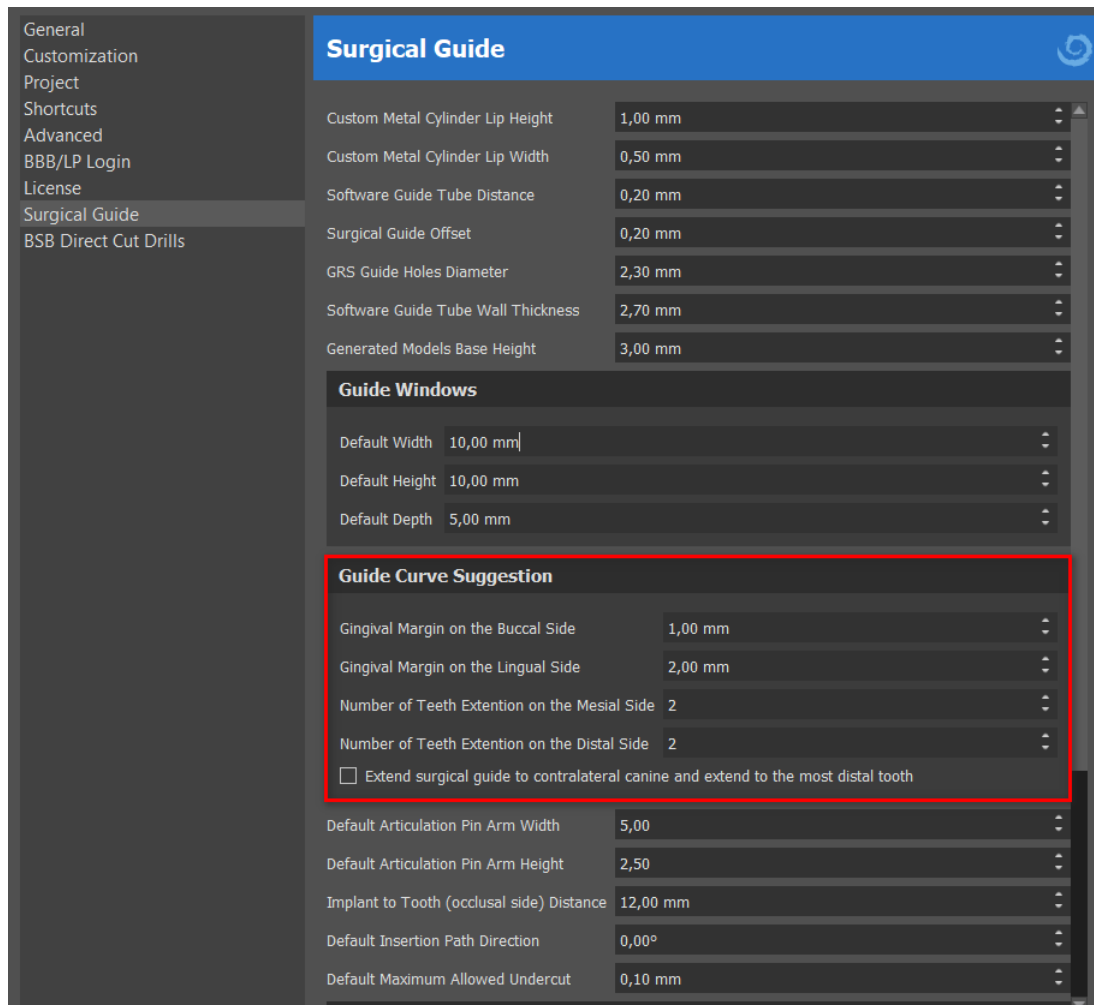


After you finish editing, you can run generation of the surgical guide by clicking the button "Create Surgical Guide". The now unnecessary cropped curve is hidden automatically at this point. This is indicated by the "Show curve" checkbox, which can be used to redisplay the curve. If the curve entered and thus the resulting surgical guide do not contain all the planned implants, the user is notified and has the option of halting generation.

Once it is completed, the fabricated surgical guide model is displayed in the "Surfaces" panel, where its parameters can be changed as standard.



The surgical guide can be regenerated with edited parameters and the same curve; the curve is merely hidden and can be displayed with selection, edited again, and used to create a modified model.

17.3 Removing undercuts


The "Surgical Guide Insertion Path" function is used to define the direction from which the surgical guide will be inserted. Using this function, you can visualise the areas that run "under" the surgical guide (undercuts) and may cause problems when inserting the surgical guide on teeth, especially in the case of using stiff materials for fabrication. These areas can be removed by clicking on the "Create undercut model as byproduct" (generates model with undercuts removed) checkbox. The surgical guide created on this model will be easy to mount.

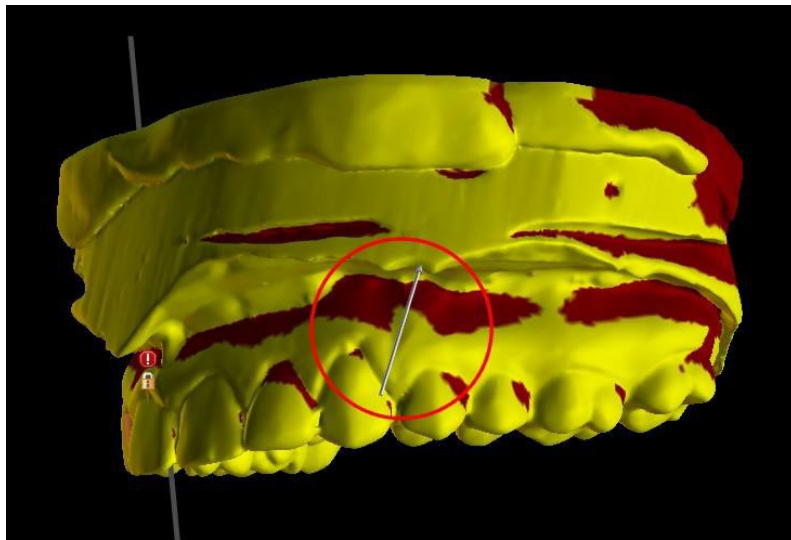

The differences between individual outputs can be compared in the following pair of images. On the left is a guide on the model with undercuts removed; on the right, all details are preserved.

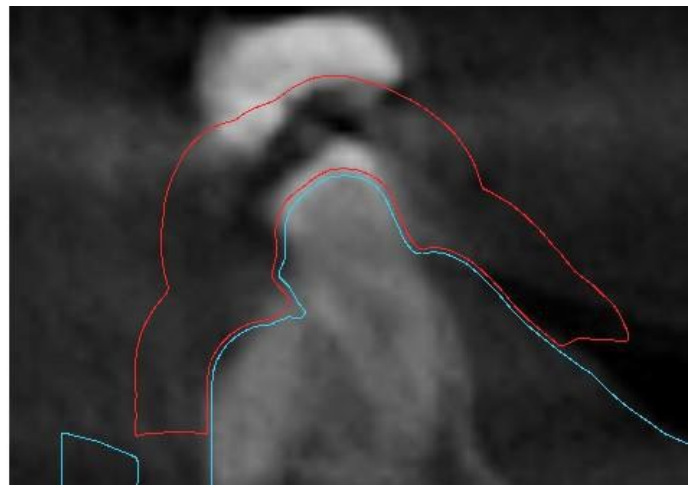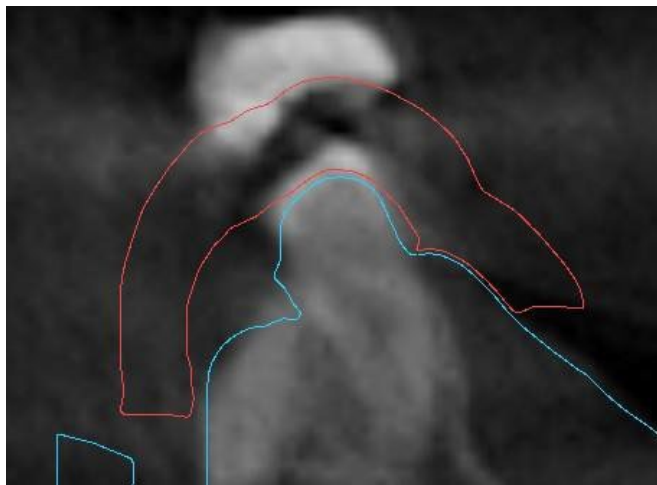


The default Insertion Path Direction angle and default Maximum Allowed Undercut are defined in Preferences, Surgical Guide tab. In addition, the Define Insertion Direction From View functionality is available from the Guide Panel. Creating an Undercut Model generates a new model that can be worked with independently.

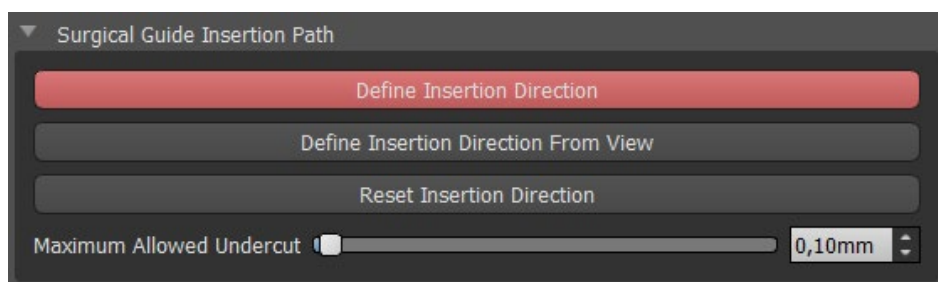

17.4 Brush tools


Except for the automatic brush tool, the application provides basic tools for manually adjusting the resulting surgical guide: "Brush Tools". These tools can smooth the surface (i.e. they lower bulging parts and even out depressions), add or remove surface material, locally deform the surface material

and edit outlines in the model (only works in 2D). The tool is found under "Model master" module, in the "Model Edit" panel. For more information about model adjustments see section 12.

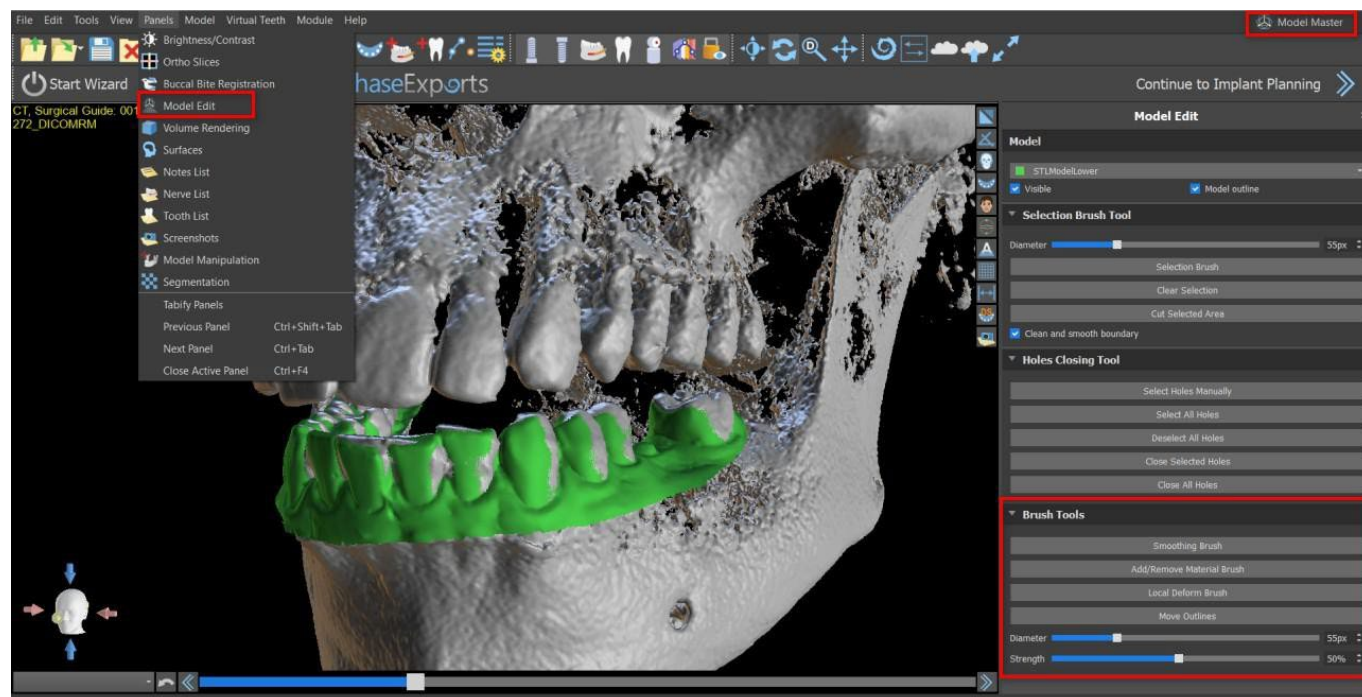

17.5 Text tool


The surgical guides fabricated can be supplemented with one or more labels with a 3D profile. This is done with the "Text tool".

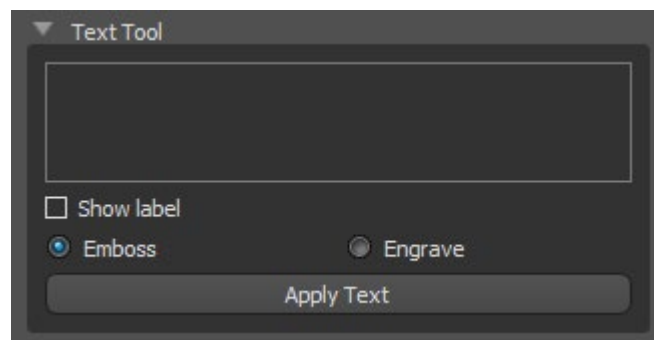

It is first necessary to check whether the correct model is selected. If the model is set as invisible at this point in time, the user is notified of this when he or she tries to insert a label on it. The text is inserted into the editing field. At this point, an auxiliary display of the text entered is also shown in the 3D display window. The text will be located on the model at the location at which it is displayed in this window. Therefore, it is necessary to position the model in the 3D window in such a way that the

text will be at the right position on its surface. It is also necessary to choose one of two methods for creating text-depth: emboss or engrave.

Then press the "Apply Text" button. The auxiliary display of the text in the 3D window is hidden, and the text is pressed into the surface of the surgical guide. The auxiliary display in the 3D window can be reactivated with the "Show label" checkbox.

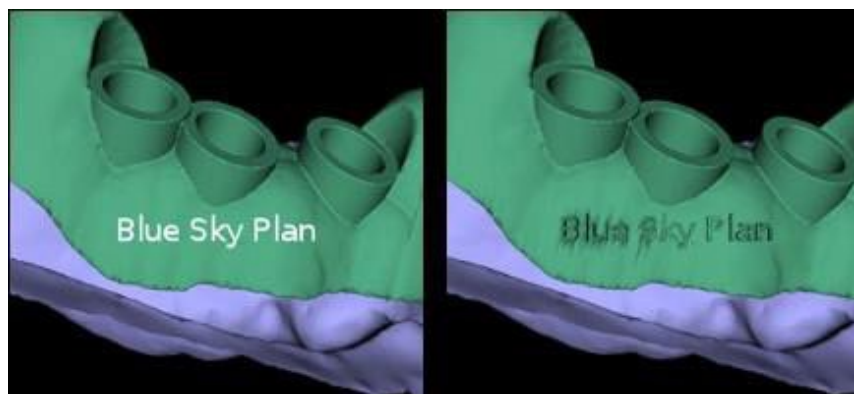

18 Endodontic Mode

Endodontic treatment (also known as root canal treatment) is a treatment sequence for the infected pulp of a tooth which is intended to result in the elimination of infection and the protection of the decontaminated tooth from future microbial invasion. It is usually done on one tooth and may include all 4 roots.

The planning begins with an intro screen titled "Endodontics", where the user selects the type of planning. For more information, see chapter 3 "Blue Sky Launcher".




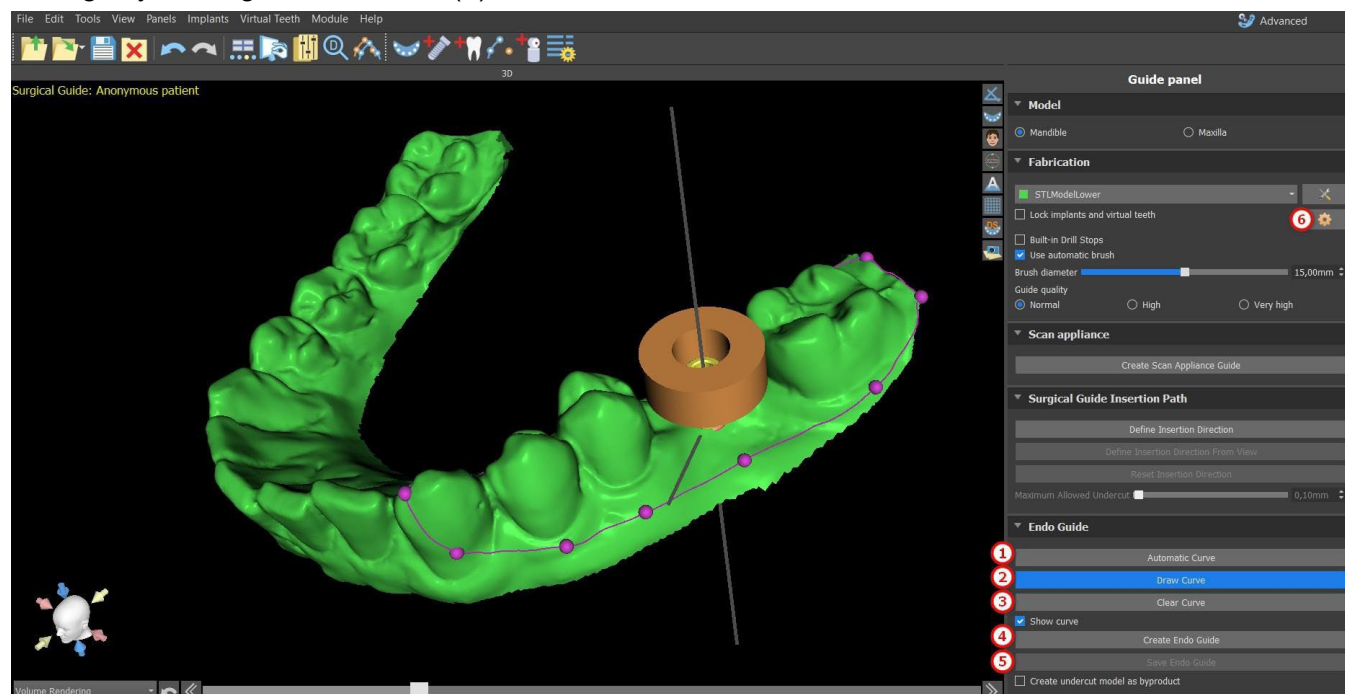
To continue to Endo Mode, click on the "Continue to Endodontic Planning" button in the upper right screen.

Before actually creating the endodontic guide, it is necessary to specify the area that should be covered. This is done automatically by clicking the "Automatic Curve" button (1). Clicking the "Draw curve" (2) button activates the drawing mode. The state of the button is indicated by a change in color. The curve border can be edited by using the individual control points. Before actually generating the endodontic guide, it is a good idea to check its shape in this mode. If needed, the curve can be deleted by clicking the "Clear curve" (3) button and the curve can be drawn again.


Once editing is over, the endodontic guide can be generated by clicking the "Create Endo Guide" button (4). If the curve entered, and thus the resulting endodontic guide, do not contain all the planned implants, the user is notified and has the option of halting generation.

Once it is completed, the Endodontic guide model that has been created is displayed in the "Surfaces" panel and can be exported and saved by clicking the "Save Endo Guide" button (5).

Endo Guide Thickness can be set in the Advanced Module in the Guide panel. Open the Guide settings by clicking on the icon  (6).




The Endo mode has to be active in order to set the Endo Guide Thickness.

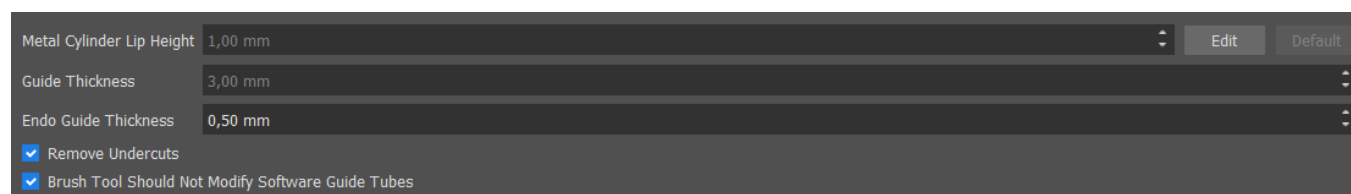

The user is able to place an implant tip exactly where clicked. The implant pivot is "Tip" by default ("Head" with Endo mode turned off).

The user can turn on "Always show selected implant/tooth" for 2D visibility (even when the transparency is set to 1.00) and "Move slices to selected implant's position" to move slices to the added implant. Those options are available in Preferences -> Surgical Guide and Endo mode has to be active in order to work with it.

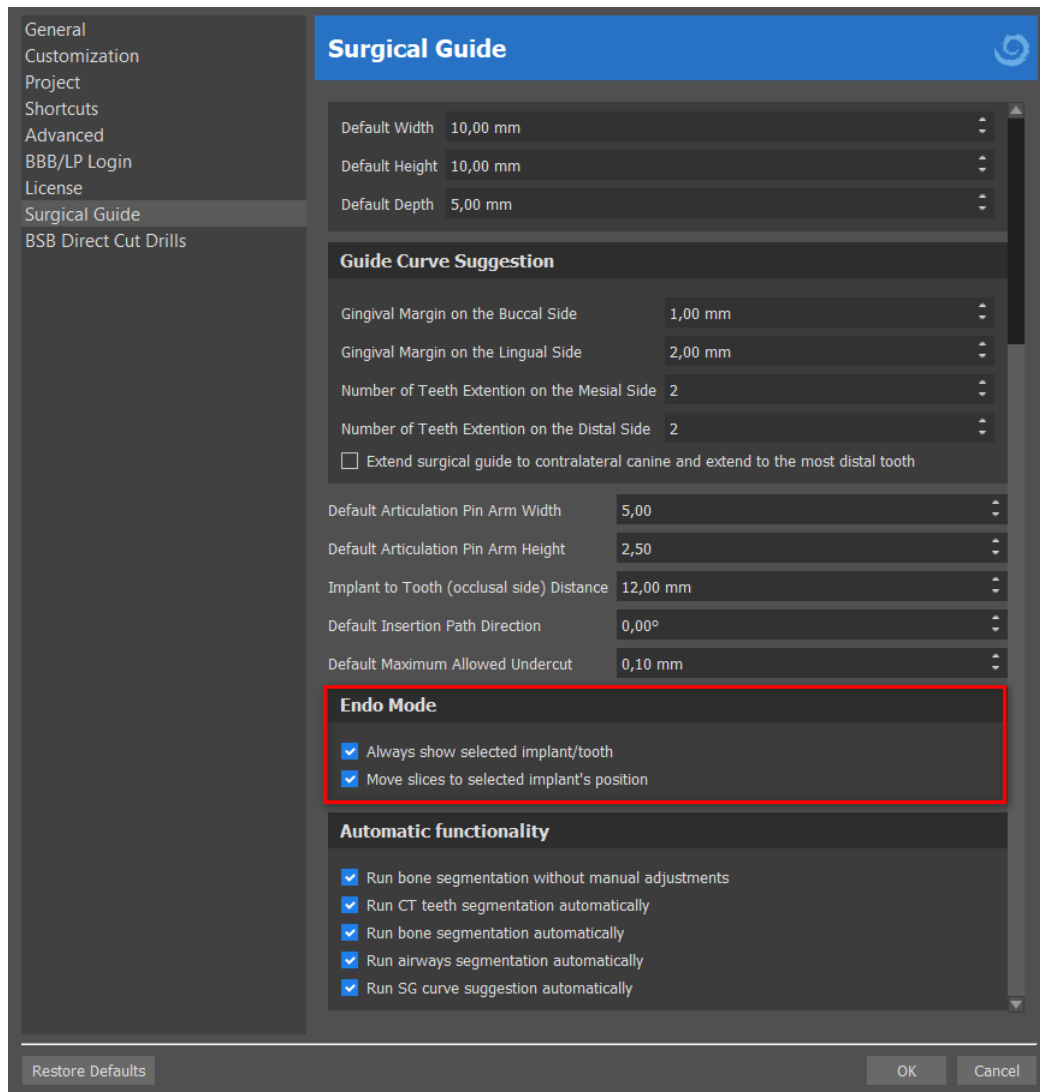

Note: Once the user is in the endodontic module, it is not possible to switch to the surgical guide module. It is necessary to start a new project in the Surgical Guide module!

19 Data export

Most of the project data can be exported from the application in the form of an STL model. The menu selection “File > Export Data” is used to export data.

The quality of the export can be set in the dialog, which influences the quality of the resulting model but also the time necessary to generate the export. The check boxes can be used to select what should be exported from the project. Selection is limited to data that are currently visible in the 3D view of the application. Nerves, imported models, implants, density surfaces, virtual teeth and scanbodies can be selected. It is possible to export either to a file or directly to the installed VRMesh application for a license with a limited number of exports. The number of remaining exports is indicated on the “Export” button and in the window’s title. The number of exports is only reduced once in a project that is currently open. By selecting the "Open in Alpha AI" or "Open in RayWare" or "Open in Uniz Dental", models can be exported to the respective external 3D printing software.

Export Quality

☒ Standard☐ High☐ Very High

Export File Format

STL

Nerves

Nerve	Selected
0	<input checked="" type="checkbox"/>
1	<input checked="" type="checkbox"/>

STL Surfaces

Name	Color	Selected
BSP_2021_1164_Lower_Model		<input checked="" type="checkbox"/>
BSP_2021_1164_Upper_Model		<input checked="" type="checkbox"/>

CT Surfaces

Name	Selected
Original	<input checked="" type="checkbox"/>

Implants

ID	Selected I.	Selected A.	Selected G. T.
0	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>
1	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>

Virtual Teeth

Tooth Name	CAM	Edited	Selected
------------	-----	--------	----------

Scanbodies

ID	Selected S.
----	-------------

☐ Show only teeth with CAM file support☐ Swap face orientation☒ Export separated files to a folder☐ Open in RayWare☐ Open in Alpha AI☐ Open in Uniz Dental

Cost: 0

Create Surface

Export (43 remaining)


Cancel

20 External Softwares

20.1 Medit import

The goal of the connection between Blue Sky Bio and Medit is to enable scans from the clinician's intraoral scanner to be used directly in Blue Sky Plan.

To import intraoral scans into Blue Sky Plan, choose the case from your scan box that has the status "Scan Completed". Then click on the Export Blue Sky Plan button to export the scan data to the Blue Sky Plan application.

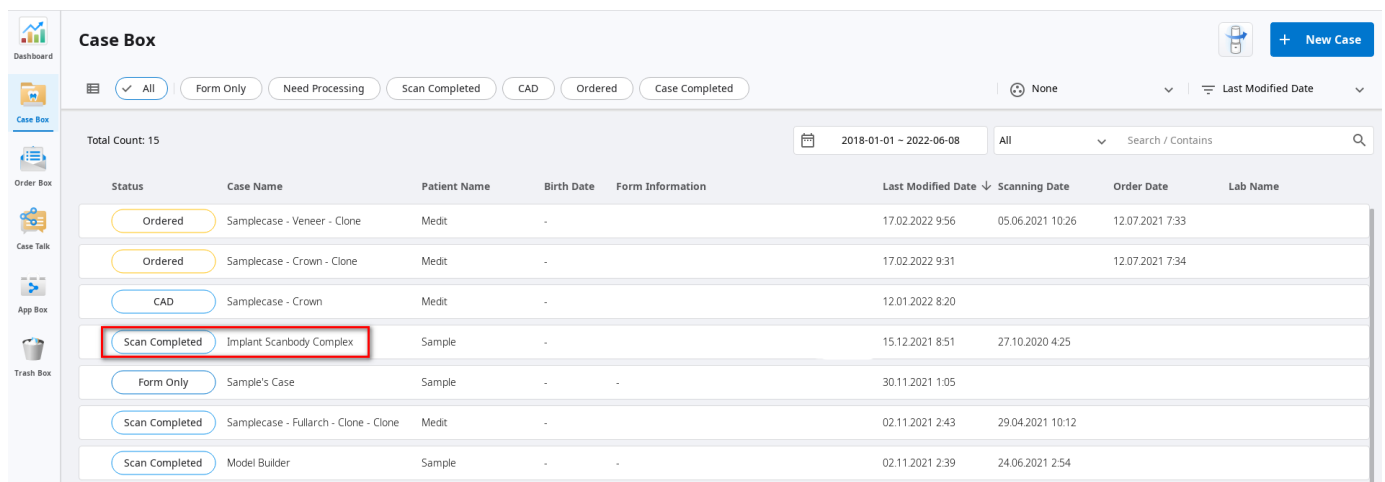


Case Box

Dashboard | Case Box | Order Box | Case Talk | App Box | Trash Box


Filters: All (selected), Form Only, Need Processing, Scan Completed, CAD, Ordered, Case Completed

Total Count: 15 | Date Range: 2018-01-01 ~ 2022-06-08 | Search / Contains

Status	Case Name	Patient Name	Birth Date	Form Information	Last Modified Date ↓	Scanning Date	Order Date	Lab Name
Ordered	Samplecase - Veneer - Clone	Medit	-		17.02.2022 9:56	05.06.2021 10:26	12.07.2021 7:33	
Ordered	Samplecase - Crown - Clone	Medit	-		17.02.2022 9:31		12.07.2021 7:34	
CAD	Samplecase - Crown	Medit	-		12.01.2022 8:20			
Scan Completed	Implant Scanbody Complex	Sample	-		15.12.2021 8:51	27.10.2020 4:25		
Form Only	Sample's Case	Sample	-	-	30.11.2021 1:05			
Scan Completed	Samplecase - Fullarch - Clone - Clone	Medit	-		02.11.2021 2:43	29.04.2021 10:12		
Scan Completed	Model Builder	Sample	-	-	02.11.2021 2:39	24.06.2021 2:54		

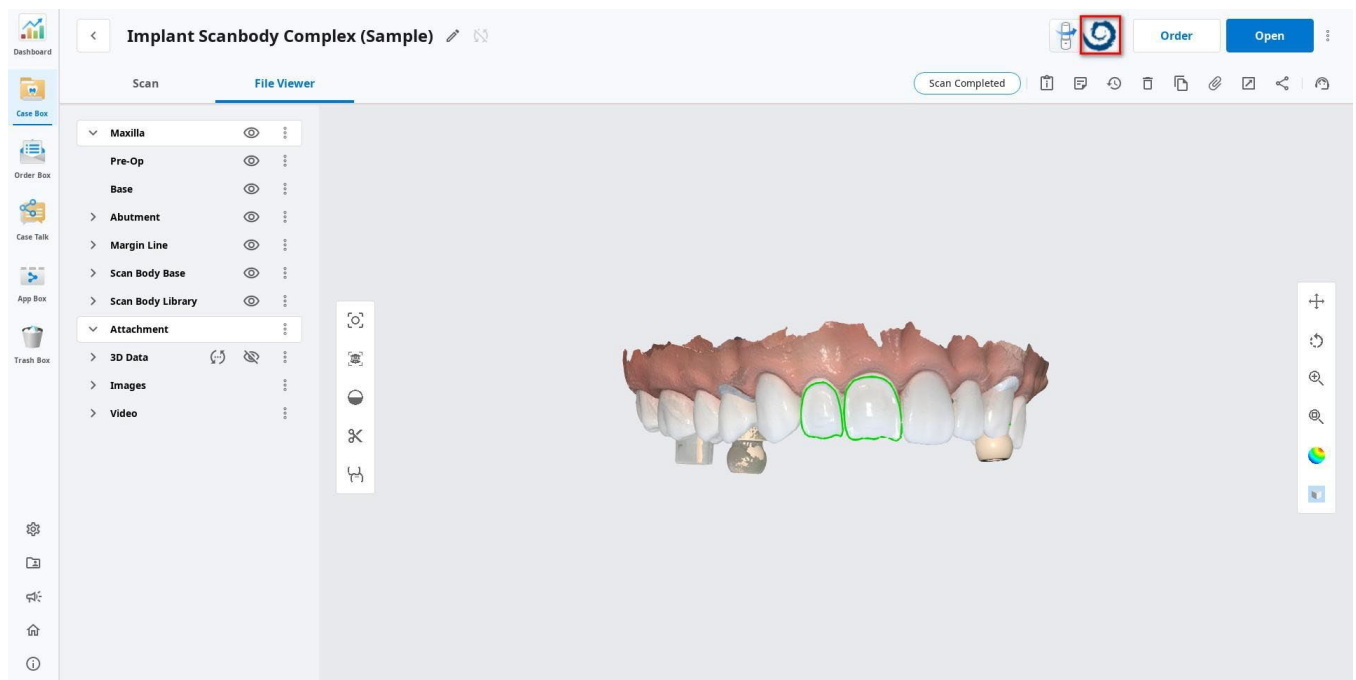


To import a scan from Medit, it is necessary to have a project opened in the Blue Sky Plan. The scan will be loaded automatically.

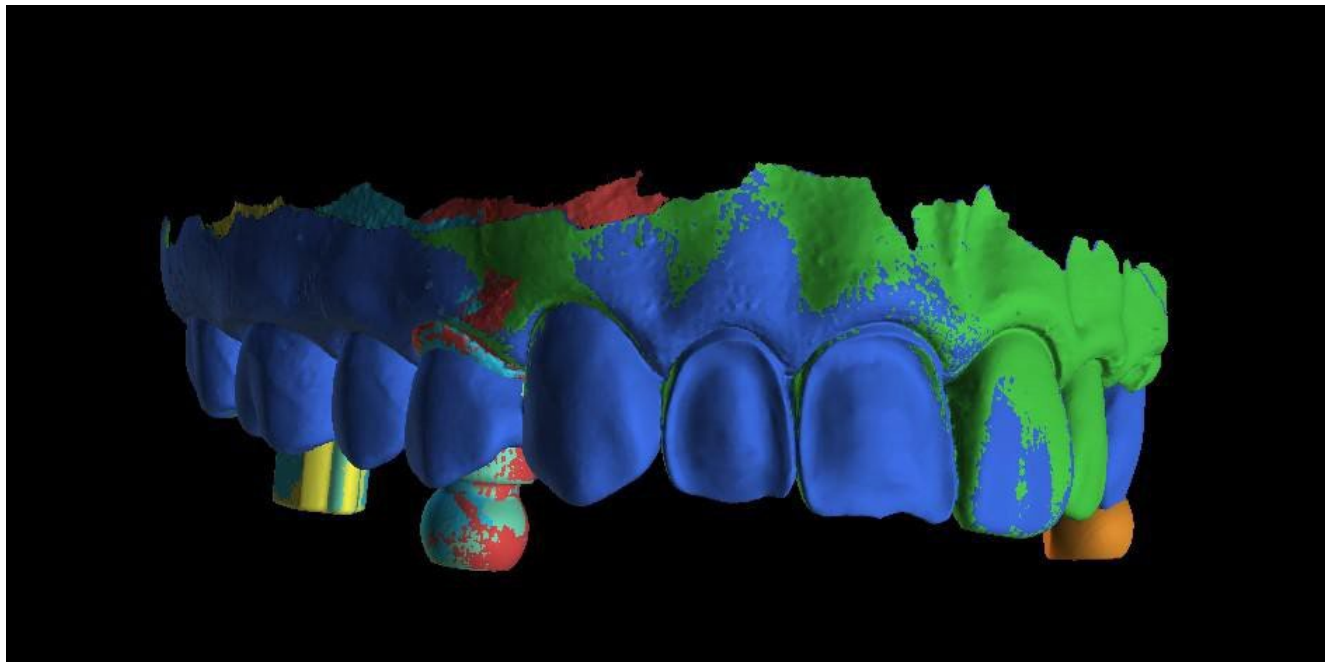

21 Articulation pins


For cases when the user needs to 3D print models of the upper and lower jaw in an exact position, the Articulation pins function should be used. Articulation pins can be attached to two arbitrary STL models (two opposing jaws), and the models with the pins can be exported.

Once the upper and lower jaws are printed (models, denture bases, etc.), the user can insert the male and female pins into each other to recreate the exact articulation. Articulation pins can be placed in the Model Manipulation Panel in the Surgical Guide Module.

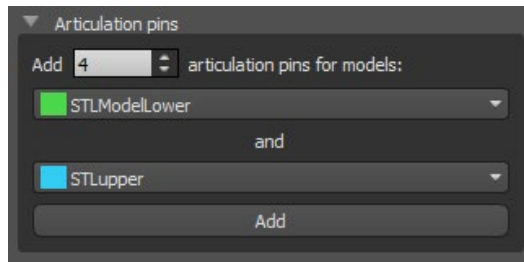

The user needs to select the upper and lower jaw models and click the “Add” button. The number of pins can also be changed.

Both of the models do not need to be closed before adding the articulation pins. Open models are closed during this process automatically. Both of the models should be aligned before adding pins.

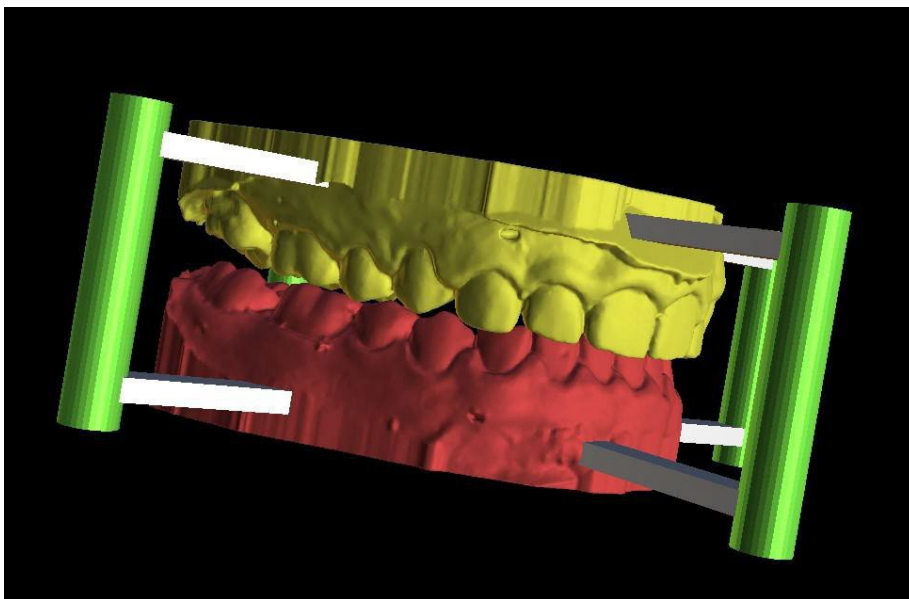

Pins and manipulation widgets will show up. The dimensions of the connecting arms as well as of the pins can be changed by dragging the manipulation widgets which appear when the user hovers over that object. The arrows allow the pins to be moved to the sides and the scale widgets change the thickness of the pins.

It is not possible to drag the pin arm outside of the pins. The default dimensions can be set in Preferences, in the Surgical Guide tab, as Default Articulation Pin Arm Width and Default Articulation Pin Arm Height.

General
Customization
Project
Shortcuts
Advanced
BBB/LP Login
License
Surgical Guide
BSB Direct Cut Drills

Surgical Guide

Custom Metal Cylinder Lip Height1,00 mm
Custom Metal Cylinder Lip Width0,50 mm
Software Guide Tube Distance0,20 mm
Surgical Guide Offset0,20 mm
GRS Guide Holes Diameter2,30 mm
Software Guide Tube Wall Thickness2,70 mm
Generated Models Base Height3,00 mm



Guide Windows
Default Width10,00 mm
Default Height10,00 mm
Default Depth5,00 mm

Default Articulation Pin Arm Width5,00
Default Articulation Pin Arm Height2,50
Implant to Tooth (occlusal side) Distance12,00 mm
Default Insertion Path Direction0,00°
Default Maximum Allowed Undercut0,10 mm

Endo Mode
☐ Always show selected implant/tooth
☐ Move slices to selected implant's position


Restore Defaults
OK
Cancel

Once the position and size of all pins and arms are tuned, by clicking on the "Create Articulation Pins" the final jaw models with the articulation pins are created. The upper and lower jaw models are separated for export and prepared with male and female articulation pins so they can be connected in the original position.

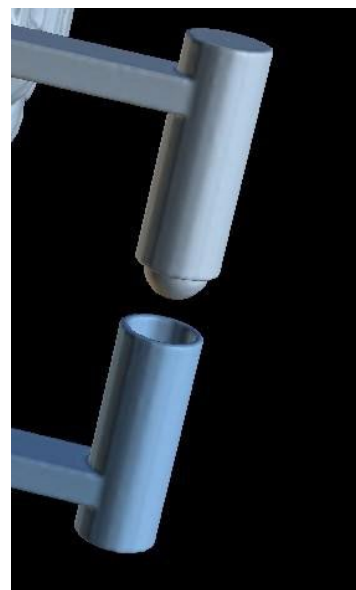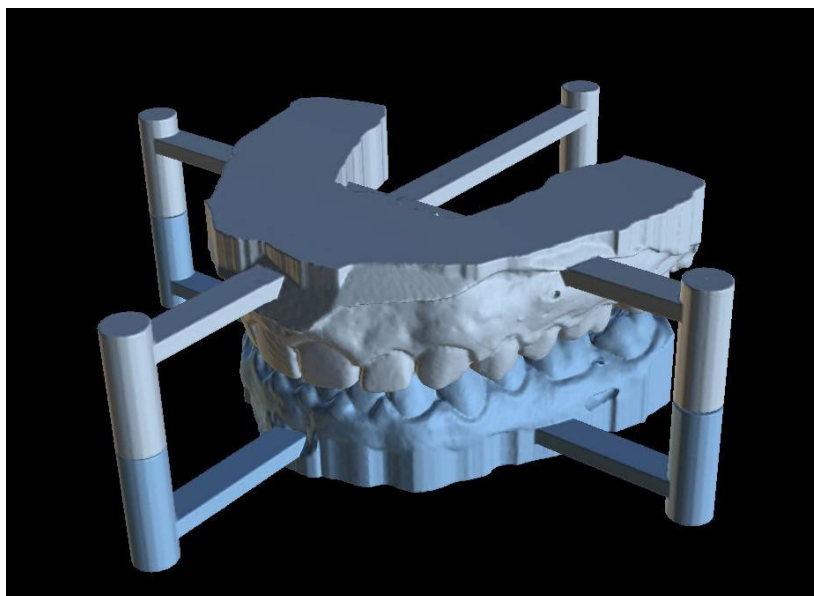


22 Coordinates export

For the purpose of producing a surgical guide, the program has a function for exporting the coordinates of implants on the basis of markers that are part of the scan appliance. It is necessary to first designate markers which are visible in the data. Open the panel with the coordinates of the export functions that is in the menu option “File > Export To > GPI Coordinates...”

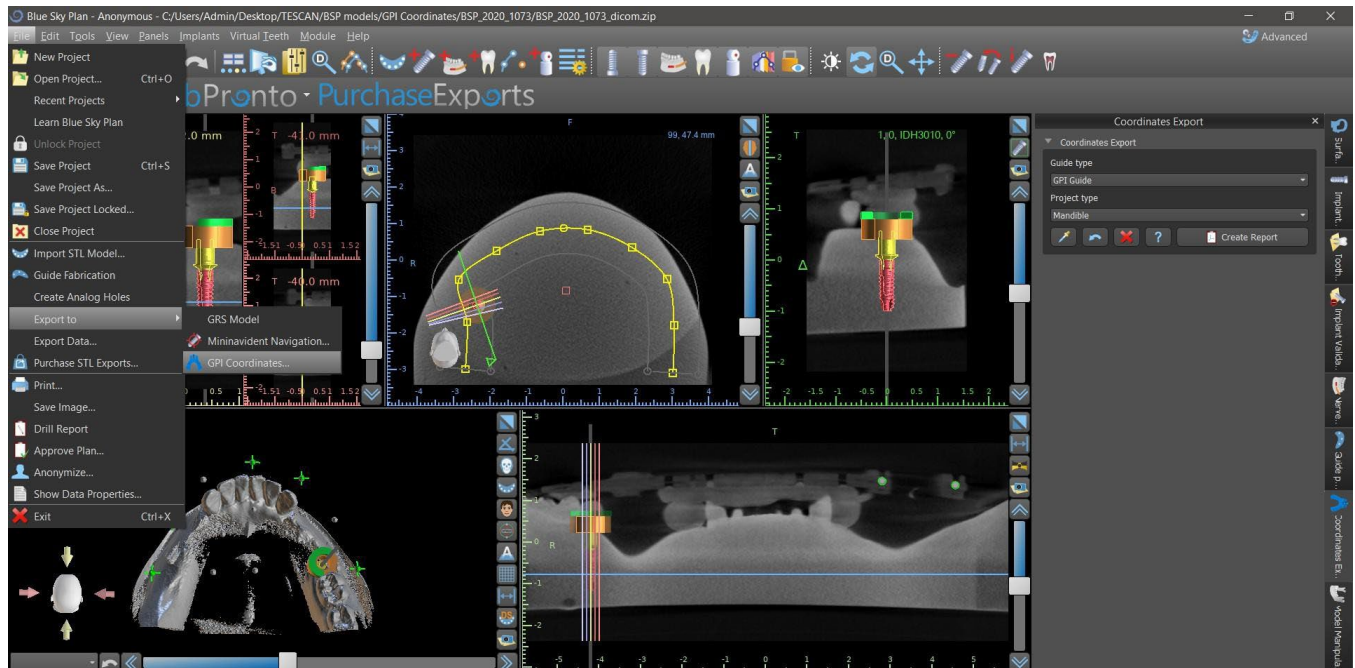

In the coordinates export settings panel, select the corresponding type of scan appliance (1) and corresponding type of jaw (2). A schematic preview of the template currently selected can be displayed by clicking on the help button (3).


GPI Guide

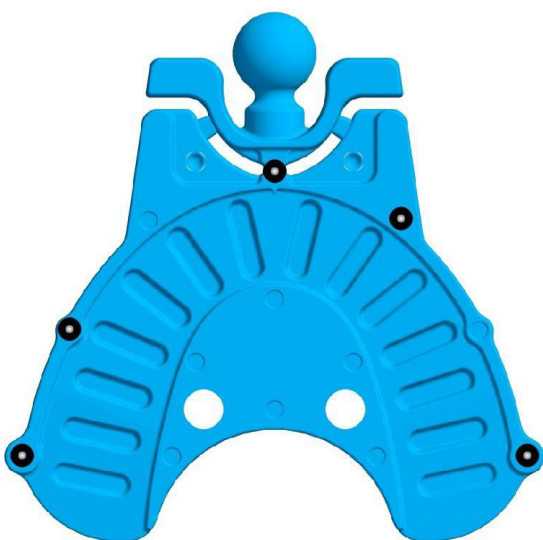

In the 3D window, view the model from above and adjust the display with the slider so the markers are clearly visible. Click button to activate marker display mode (4), and designate markers by clicking on them. The order in which the markers are designated must correspond to the order marked on the schematic template. If no order is prescribed then markers can be selected in any order. The markers can be designated in all available views and their position is automatically adjusted for the best fit.




If you make a mistake when entering a marker, the most recently entered point (5) or all points (6) can be removed by clicking the corresponding buttons or with a right click on a marker when the marker selection mode is active. The coordinates are actually exported when you click the “Create Report” button (7). For a license with a limited number of exports, the number of remaining exports is indicated on the button; the number of exports is only reduced by one each time the project is opened.

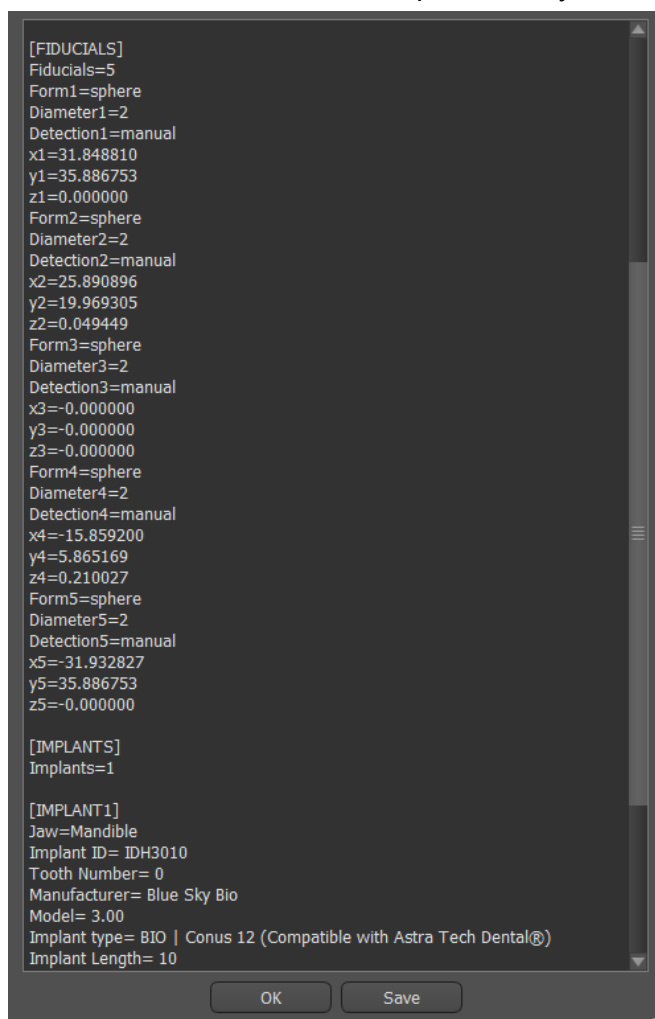

23 Boolean Operations

Boolean operations allow the user to join and subtract 3D models. This function can be activated in the Model Master or Normal/Advanced Module in the Panels section.

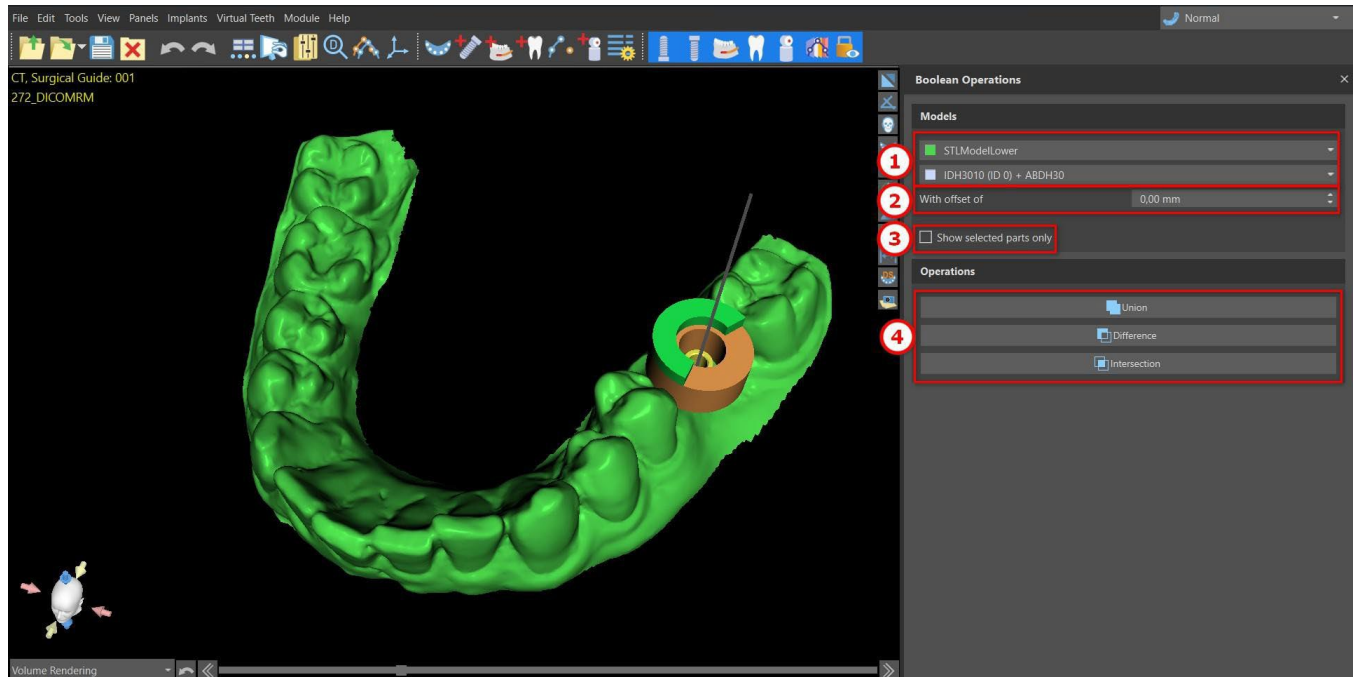


To start a boolean operation, select two models in the models section (1) with the preferred offset (2) and then click on the operations option (4). The resultant mesh of the operation will be available in the Surfaces panel.

If multiple models are available, the checkbox "Show selected parts only" will display only those models among which the boolean operation will take place.


Note: Boolean operations are applied to any STL model or implant.

23.1 Union

Objects will be joined to become a single object.

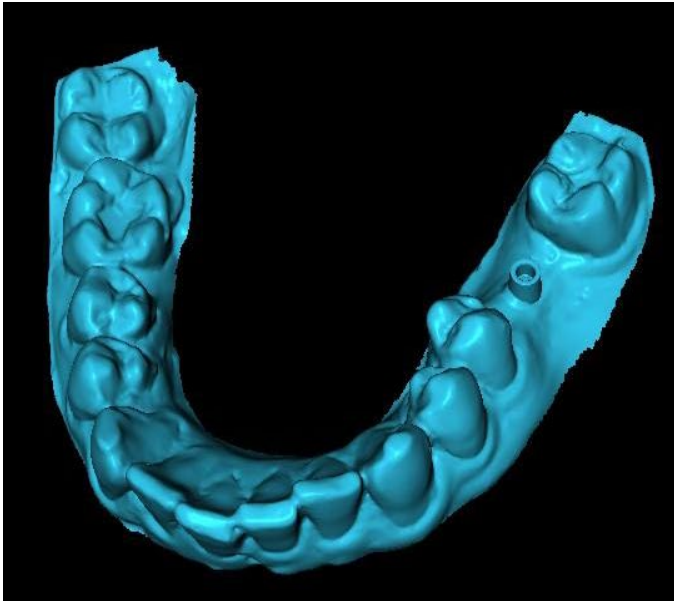


Union with zero offset

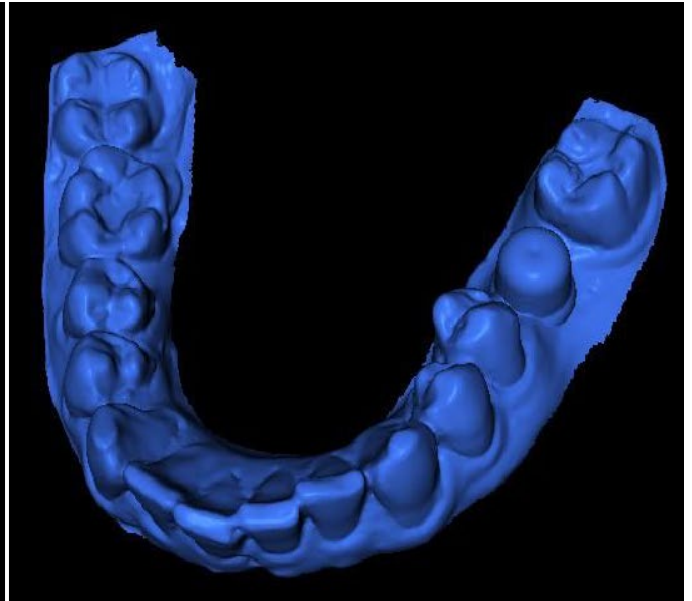


Union with 2 mm offset

23.2 Difference

The difference operation is used to remove a selected part of an object.

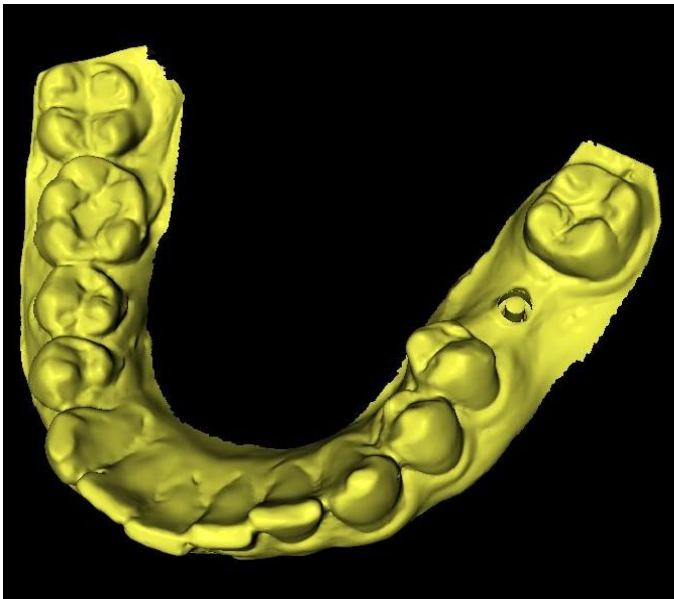


Difference with zero offset

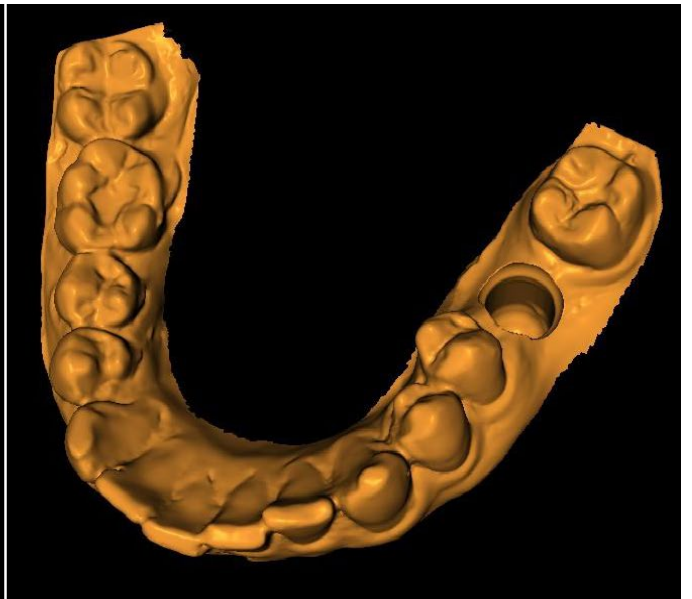


Difference with 2 mm offset

23.3 Intersection

The Intersection operator retains the parts that are common in all the objects involved.

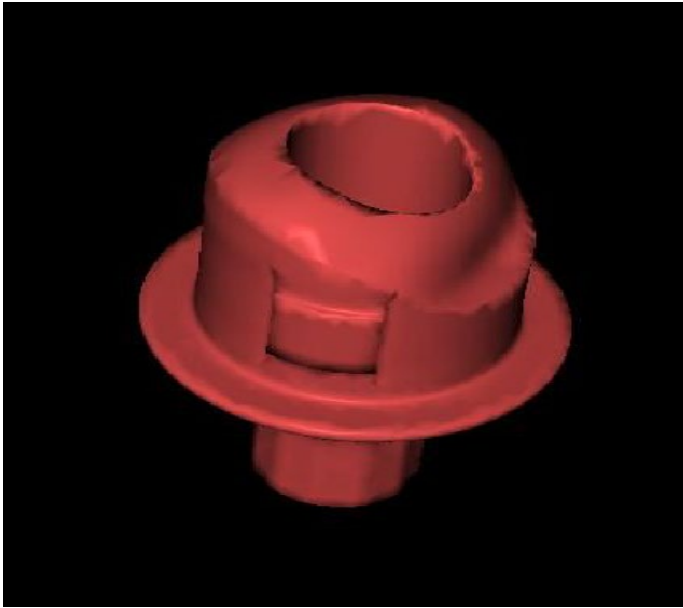


Intersection with zero offset

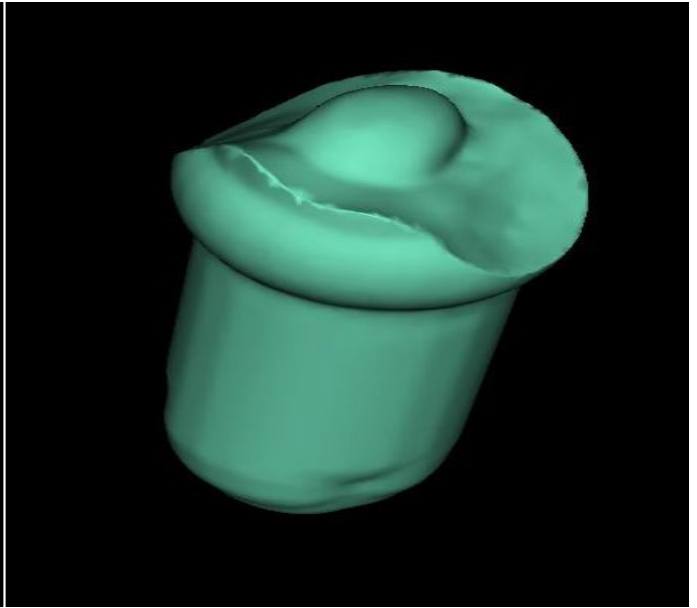

Intersection with 2 mm offset

24 Blue Sky Plan Wizard


For easier cases of implant planning in the Normal Mode there is the Blue Sky Plan Wizard tool. The tool allows case processing in the form of partly automated sequential steps with help. To start the Wizard, first select "Implant Planning and Surgical Guides" in the main screen, then choose "One or Two Implant Case". For more information, see chapter 3 "Blue Sky Launcher".

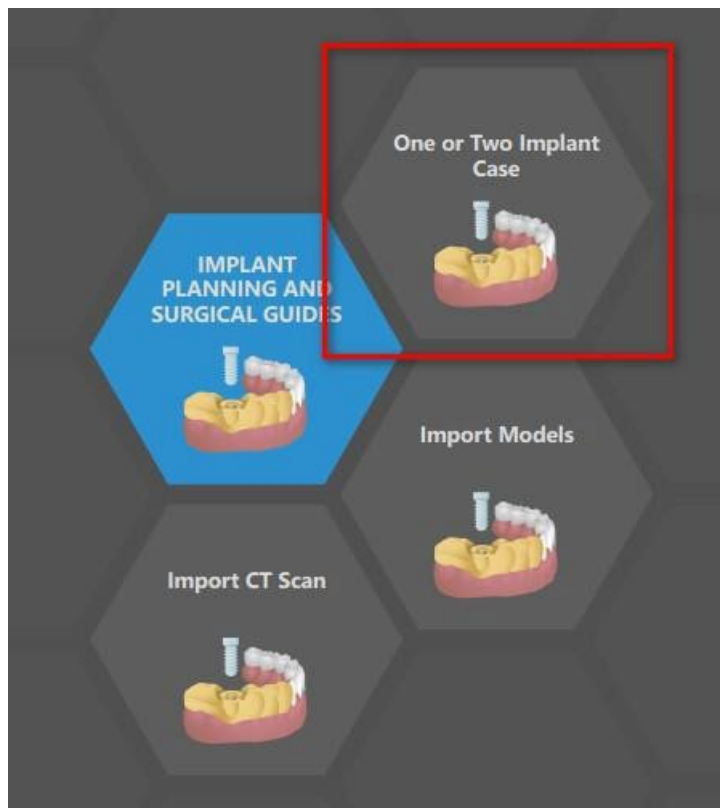

If you have already opened some project, you can also press "Start Wizard" button in the top left corner.




The notification pops up, if you wish to start the new case, or open the Wizard with existing case.

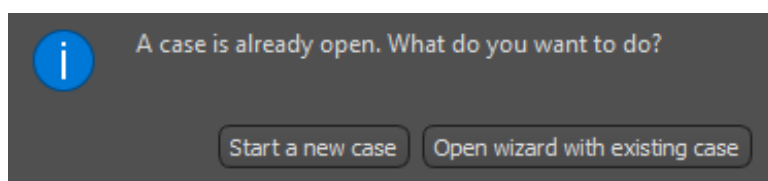

You can move between individual Wizard steps using the “Next” (1) and “Previous” (2) buttons. Every step clearly shows the Wizard stage you are in (3). There are thematic learning videos available for some of the activities (4). Whilst running the Wizard you can use the “Save Screenshot” (5) button and continue working with the screenshots via the “Screenshot Panel”. The button in the top bar is used for reactivating or restarting the currently used tool (6).

If you find that the case requires more complex processing you can switch the Wizard off at any time and continue work in the Normal Mode (7). Any nerves that have been detected, implants inserted or guides created will be saved.

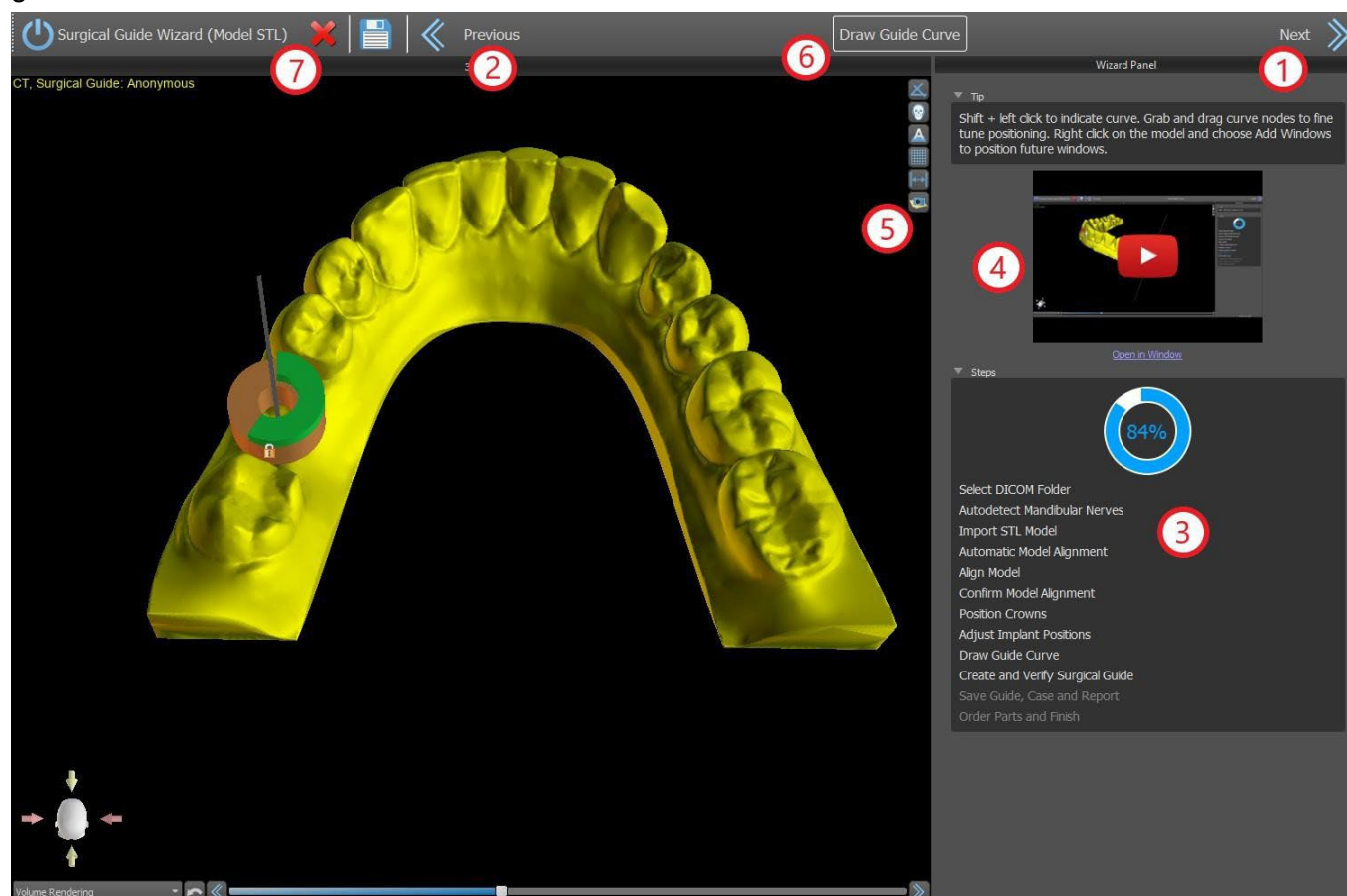


The Wizard allows various types of input data to be used in planing (Import Type):


- DICOM + STL Model (Model STL)
- DICOM + Model Scan (Model CT)
- DICOM + Impression Scan (Impression CT)
- DICOM + Impression Scan with Markers (Impression CT With Markers)

24.1 Load DICOM data

After starting the Wizard a dialog will open where you can select the import type, kit type and, after selecting some kind of kit types, the handpiece you want to use (1).

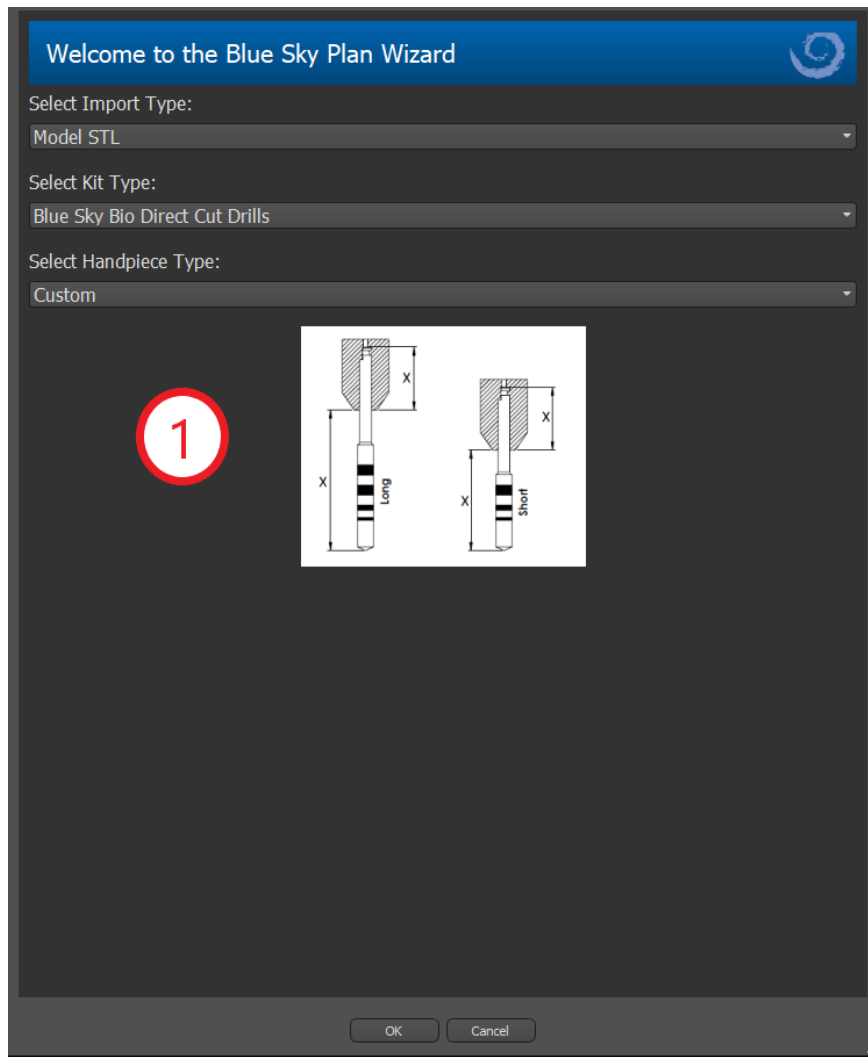


The following windows serve for DICOM data upload, potential dataset selection and to select volume of interest (2). If you have already uploaded some DICOM data when you started the Wizard, you will work with these data. The window “Load DICOM Data” will not open and the Wizard will move directly to the next step.

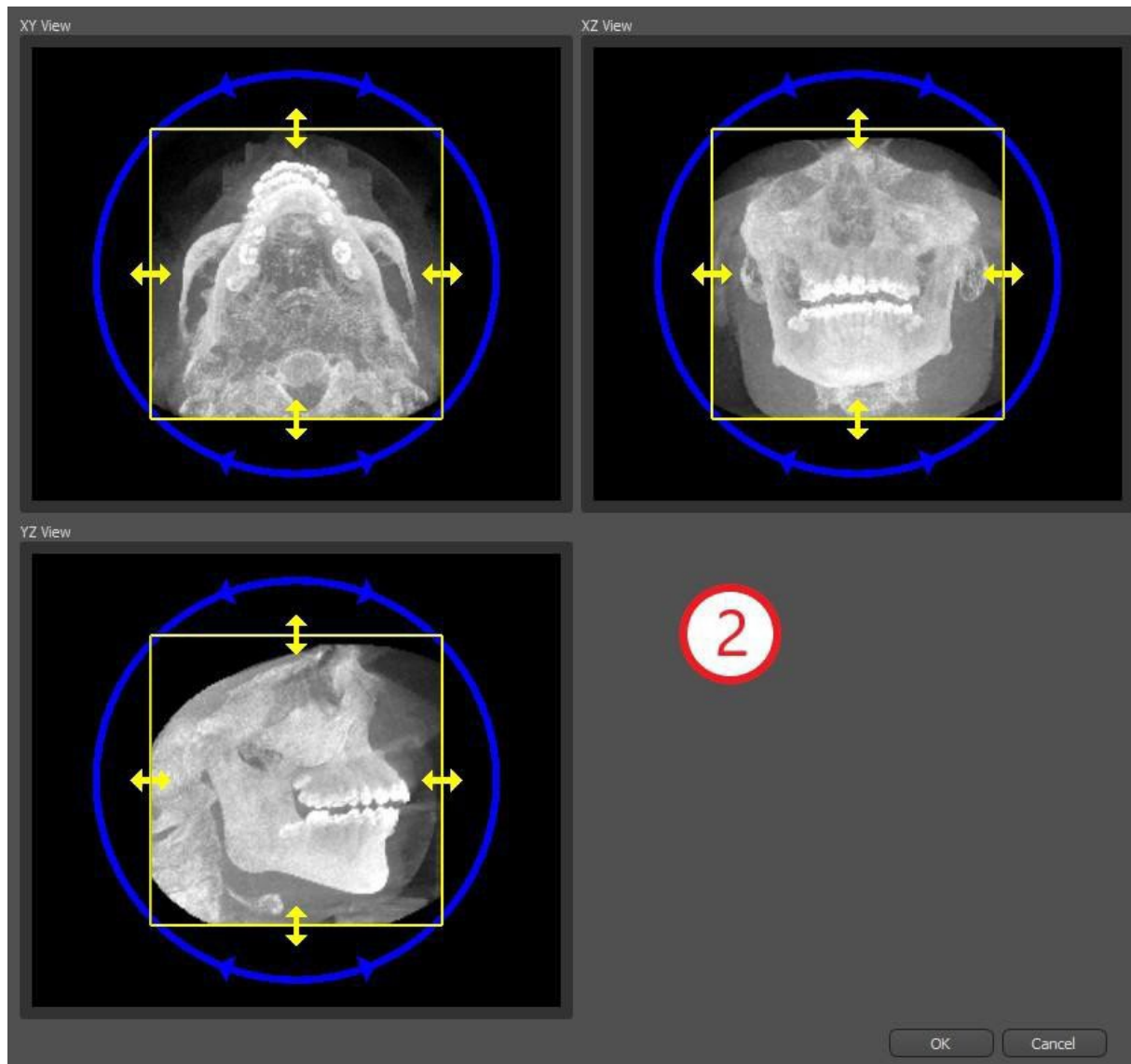


24.2 Adding a mandibular canal


In this step, automatic nerve detection is carried out for both foramens. It is possible to cancel nerve detection if the user wants to work with the maxilla; in this case, click on “Cancel, it’s maxilla”.

Automatic nerve detection for both foramens can be disabled in Preferences, Automatic functionality section.

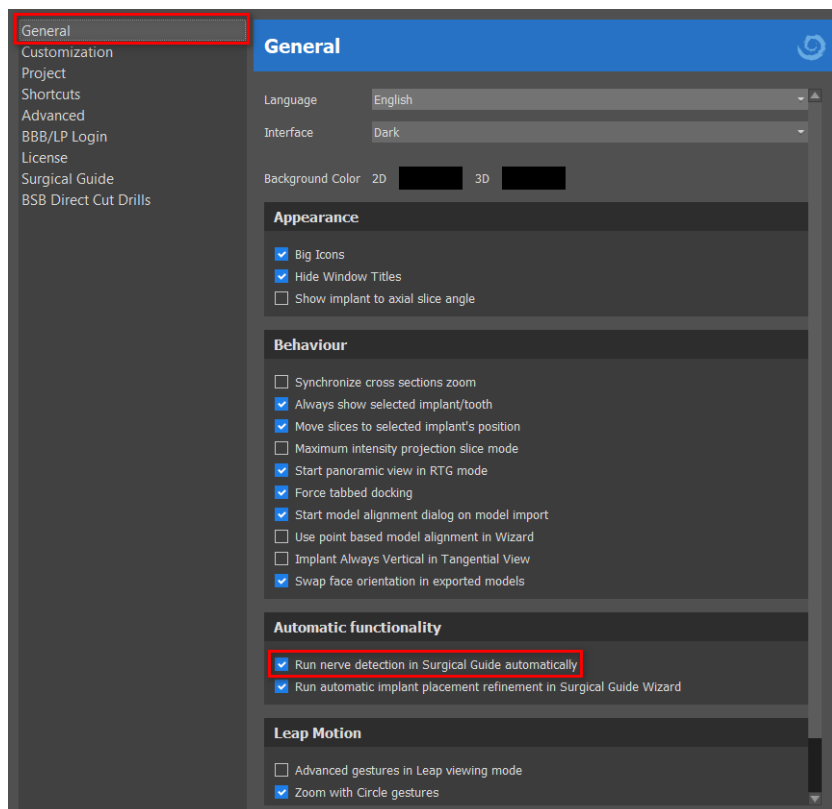

If this option is chosen, the user sets the foramen manually. Select “Set Right Foramen Manually” and in the 3D view click the cross-shaped cursor first on the right and then on the left mental foramen by selecting “Set Left Foramen Manually” (1). The mandibular canal is automatically detected (2) and its shape may be edited manually in any view except for 3D (3). If you do not need to detect the mandibular nerve, you can skip this step using the “Next” button. If you are working with the maxilla, click directly on the “Maxilla” button in the top bar.

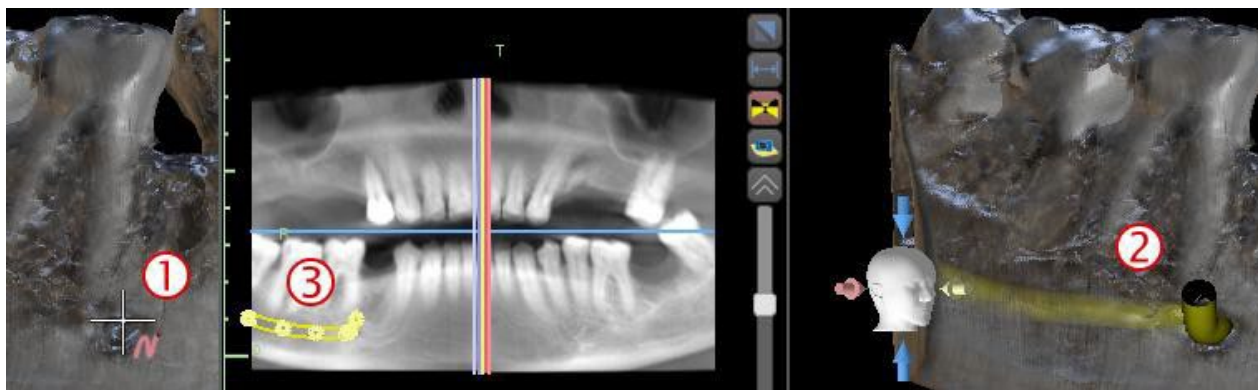

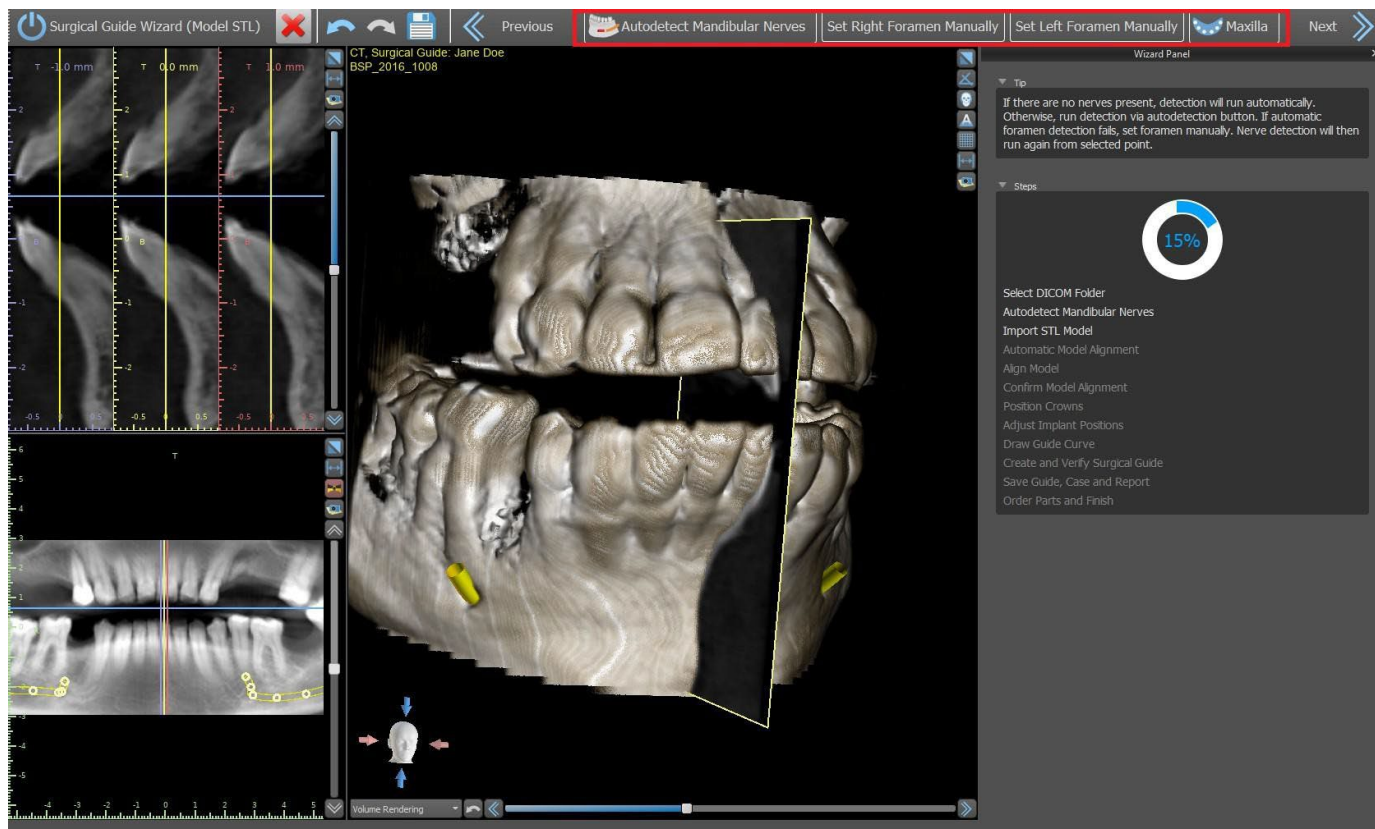

24.3 Model upload and preparation

In the next step the model will be uploaded and prepared. Depending on the surface scan import type, several preparatory steps can be run before a model is prepared for aligning with the DICOM data.


- Model STL - simply upload.
- Model CT - it is necessary to set the surface threshold and create a surface model.
- Impression CT - it is necessary to first set the surface threshold and create an impression model. It is then necessary to plot and edit a curve in the impression model to delimit the appropriate model part and create an inverted model.
- Impression CT with Markers – you need to align the scan appliance with the gutta percha markers (refer to section 7) and then, as in the case of the Impression CT, create an inverted model.

24.4 Model aligning


Except for the impression CT with Markers process, it is necessary to align STL model to CT dataset. After importing the model, select if the model is mandible or maxilla. The process of alignment will perform automatically after that. The same function of automatic alignment has the button "Automatic Model Alignment".



It is also possible to align the model manually by clicking the button "Manual Alignment". At least five identical teeth must be marked in the model and in the panoramic image. Points should be placed in exactly the same position in both scans. In the next step you must confirm the alignment or return to the previous step to improve the point setting.

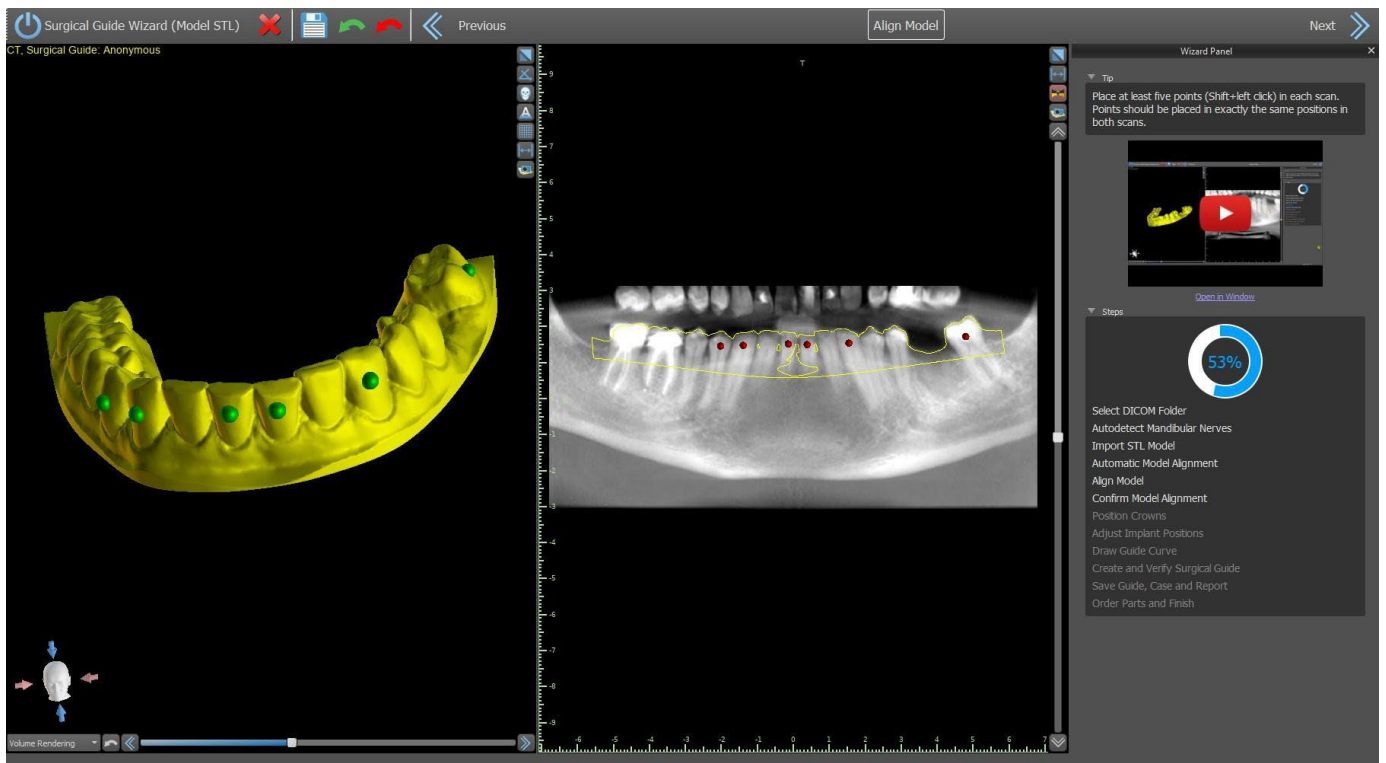



For reversing the alignment, click on the button "Undo Alignment".

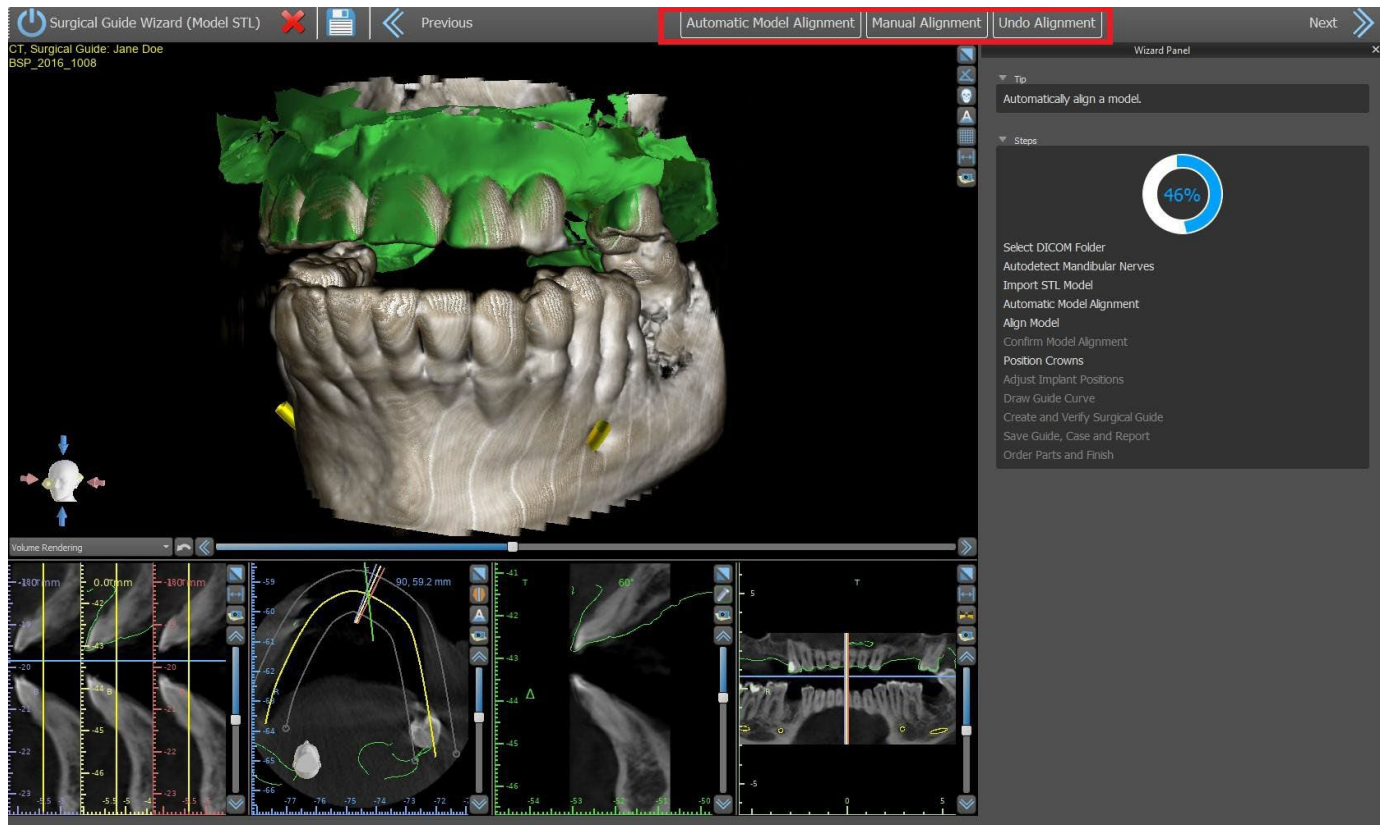

24.5 Crowns and implants


Now you can add crowns and implants. Crowns can be generated either automatically, or can be selected from predefined catalog. For more information about virtual teeth, see chapter 15.

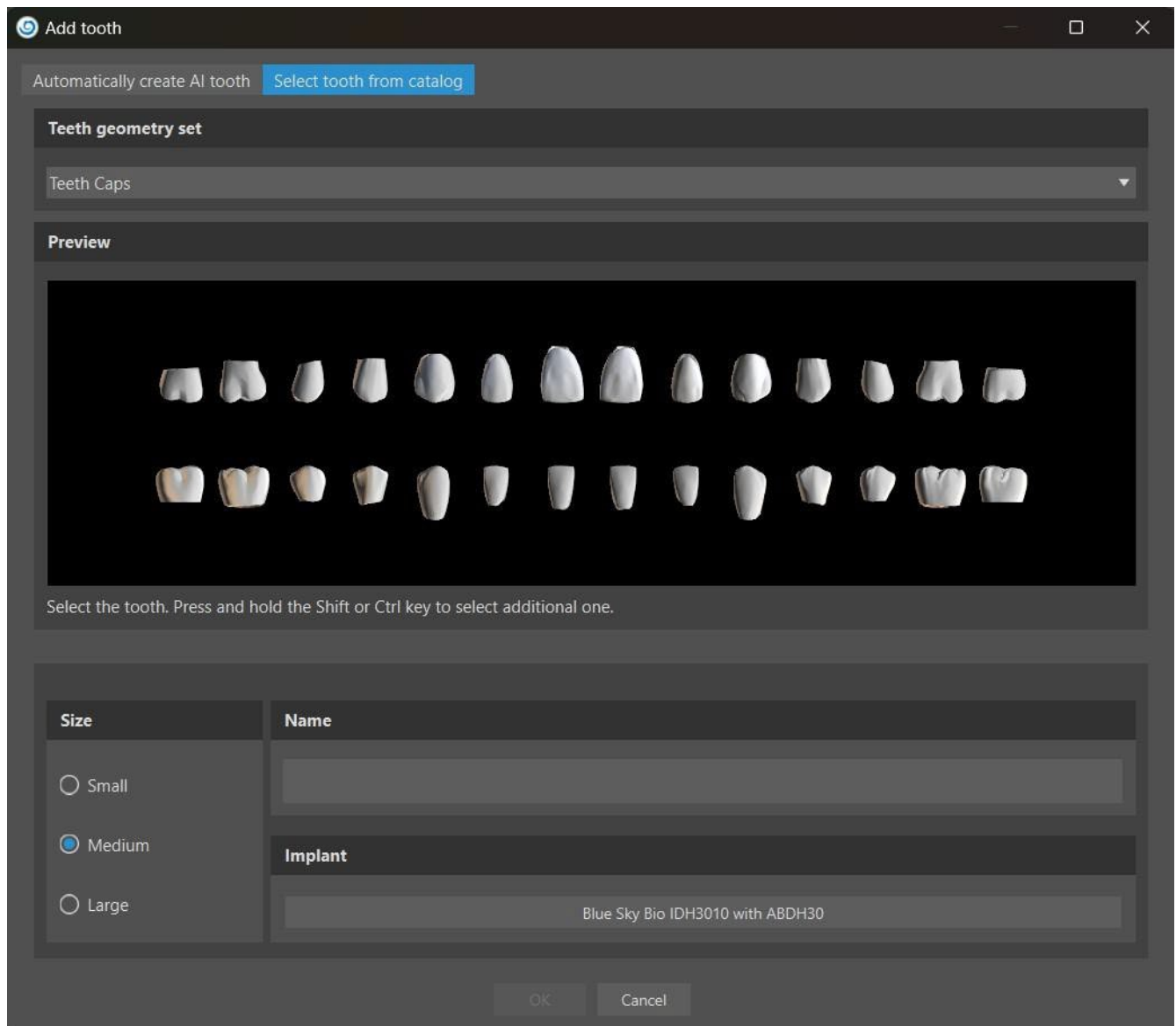

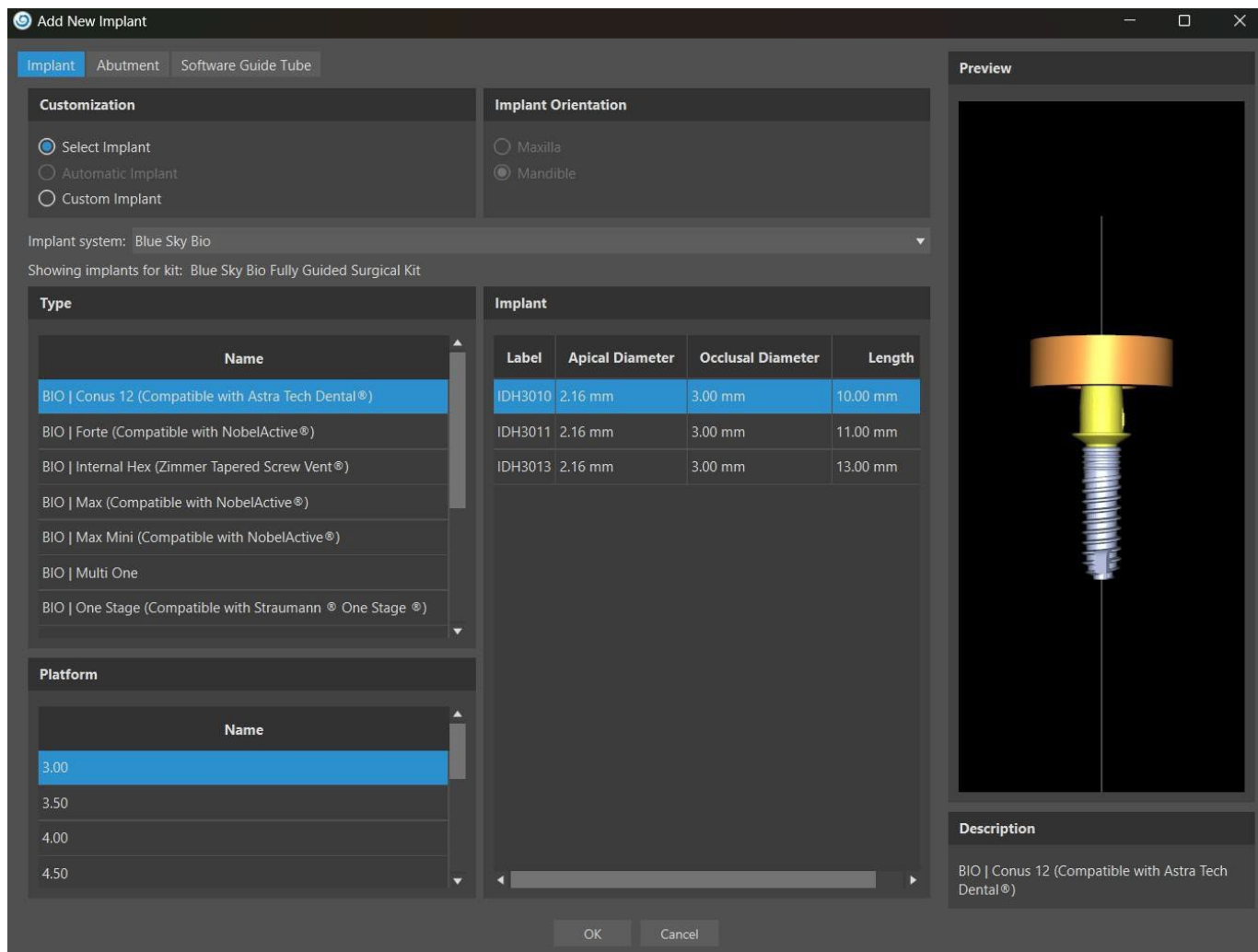


You can choose from several teeth geometry sets and implants compatible with the kit type selected at the start of the Wizard. The crowns are added together with the selected implants and, in this step, you can only move them jointly.

Blue Sky Plan® examines the available space for placing the tooth and implant. This only works for one tooth option. If the system evaluates a bigger gap for the implant in question, an info message appears and the user chooses between automatic or manual implant placement. The user manually places and positions the implant by clicking the "Place Manually" button. If the user selects "Add Automatically", the program will try to place the implant, but, most likely, it may be defective. Please note that this is only applicable to the implants associated with teeth from the catalog.

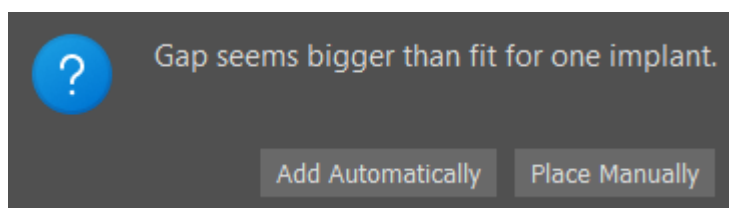



Blue Sky Plan Wizard offers the possibility to add fully automatic implants. An automatic implant is added via the "Add Tooth" dialog. Choose the associated AI tooth from the preview in the "Automatically create AI Tooth" tab (this is selected by default). By clicking the implant field, "Add New Implant" a dialog appears. Select "Automatic Implant" and implant type. The specific implant and platform will be chosen automatically. Click on "OK" to confirm implant selection and then once again on "OK" for automatic implant and adding a tooth. For more information about automatic implants, see chapter 8.1.2.


After the AI tooth with an automatic implant is added, segmentation takes place and the local bone and two surrounding teeth are segmented. Segmentation cannot be hidden in the Wizard, it can only be done outside the Wizard in the Surfaces panel.

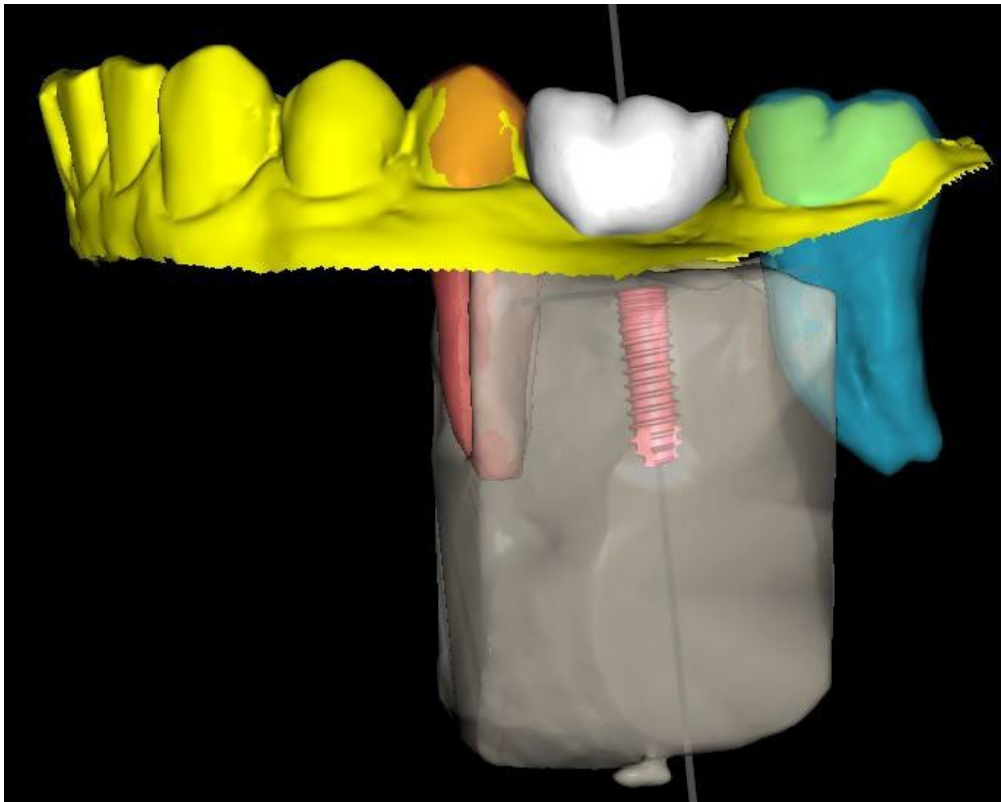

In the next step you can adjust the implant and the crown positions separately. Both steps also display validation information about the status of the individual implants.

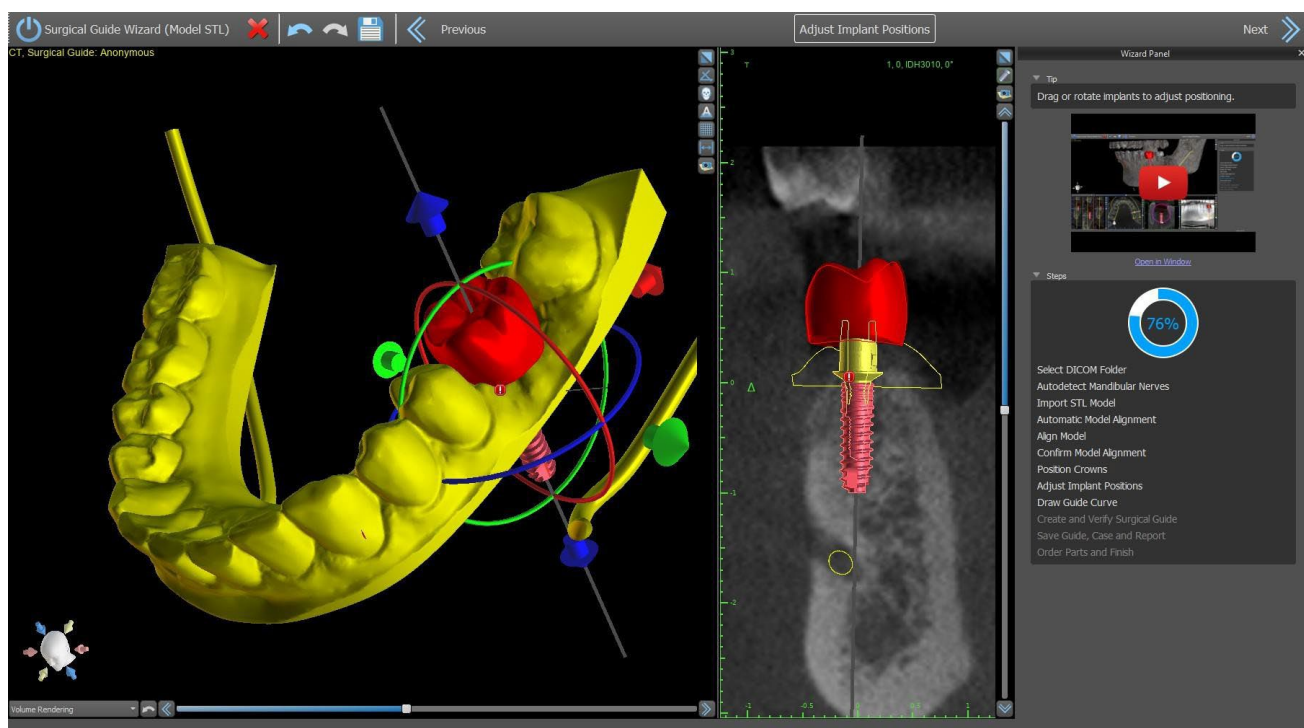

The  icon appears if an implant is evaluated as invalid. After double clicking on this icon the Implant Validation Panel will open with further information.

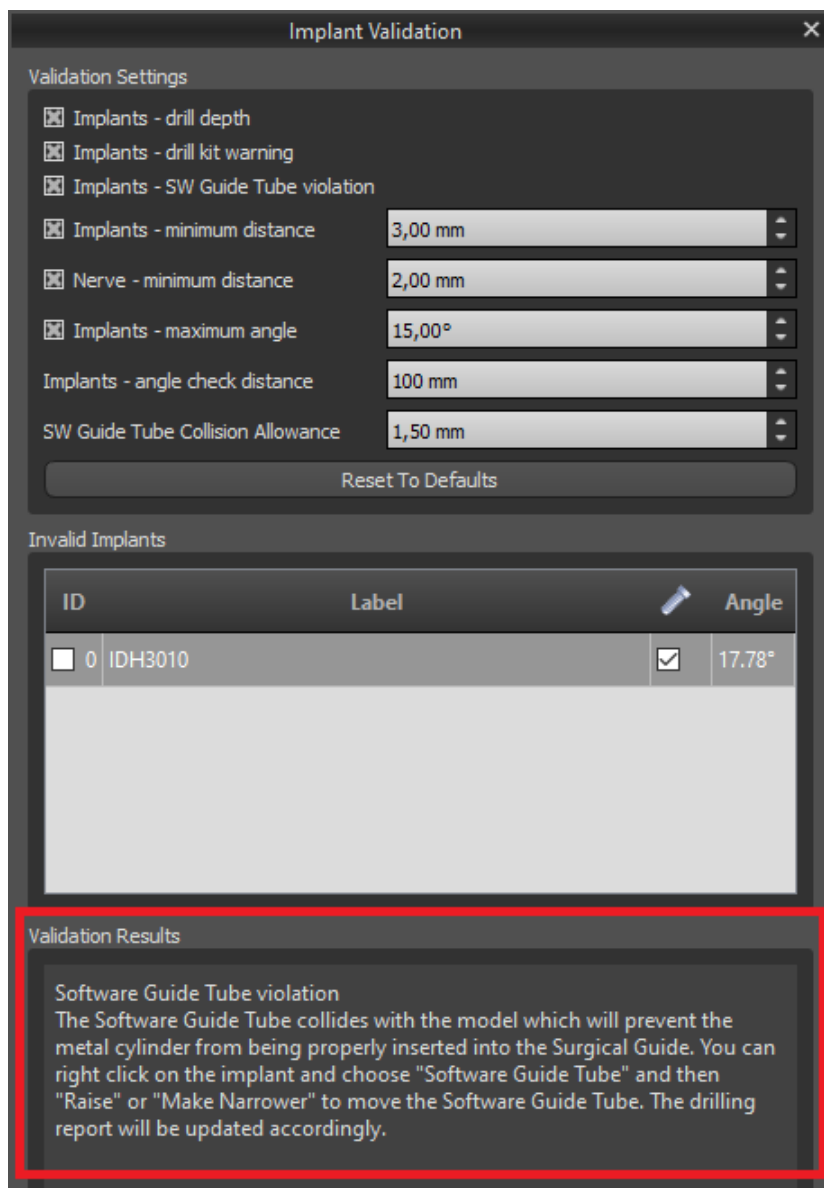


The screenshot shows the 'Implant Validation' window. It has a 'Validation Settings' section with several checked options: 'Implants - drill depth', 'Implants - drill kit warning', 'Implants - SW Guide Tube violation', 'Implants - minimum distance' (3,00 mm), 'Nerve - minimum distance' (2,00 mm), 'Implants - maximum angle' (15,00°), 'Implants - angle check distance' (100 mm), and 'SW Guide Tube Collision Allowance' (1,50 mm). There is a 'Reset To Defaults' button. Below is an 'Invalid Implants' table with one entry: ID 0, Label IDH3010, and Angle 17.78°. At the bottom, the 'Validation Results' section is highlighted with a red box, showing a 'Software Guide Tube violation' message.

ID	Label	Angle
<input type="checkbox"/> 0	IDH3010	<input checked="" type="checkbox"/> 17.78°

Validation Results

Software Guide Tube violation
The Software Guide Tube collides with the model which will prevent the metal cylinder from being properly inserted into the Surgical Guide. You can right click on the implant and choose "Software Guide Tube" and then "Raise" or "Make Narrower" to move the Software Guide Tube. The drilling report will be updated accordingly.

If you continue while the collision between Software Guide Tube and the model is detected, you can choose in this step if you resolve collisions immediately or leave it as it is.

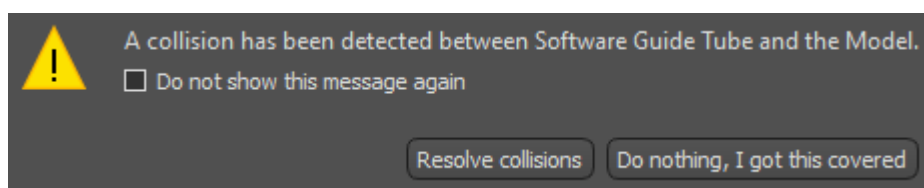


A dialog box with a yellow warning icon. The text reads: 'A collision has been detected between Software Guide Tube and the Model.' Below this is a checkbox labeled 'Do not show this message again'. At the bottom are two buttons: 'Resolve collisions' and 'Do nothing, I got this covered'.


24.6 Surgical Guide

Prior to fabrication of the surgical guide it is necessary to specify the part of the model to be covered. In the "Draw Guide Curve" step, a curve around the area of interest is drawn automatically on the base model. By generating a surgical guide, the undercuts are removed and the newly-generated model does not have them. To edit the curve, move its individual points and when moving on to the next step,

the surgical guide will be created. In both the “Draw Guide Curve” step and the “Create and Verify Surgical Guide” step, windows can be added by right clicking on the base model and they can be adjusted by manipulating the widget. By clicking the buttons "Next" (in the "Draw Guide Curve" step) or "Recreate Guide with Windows" (in the "Create and Verify Surgical Guide" step), a surgical guide will be recreated with windows.

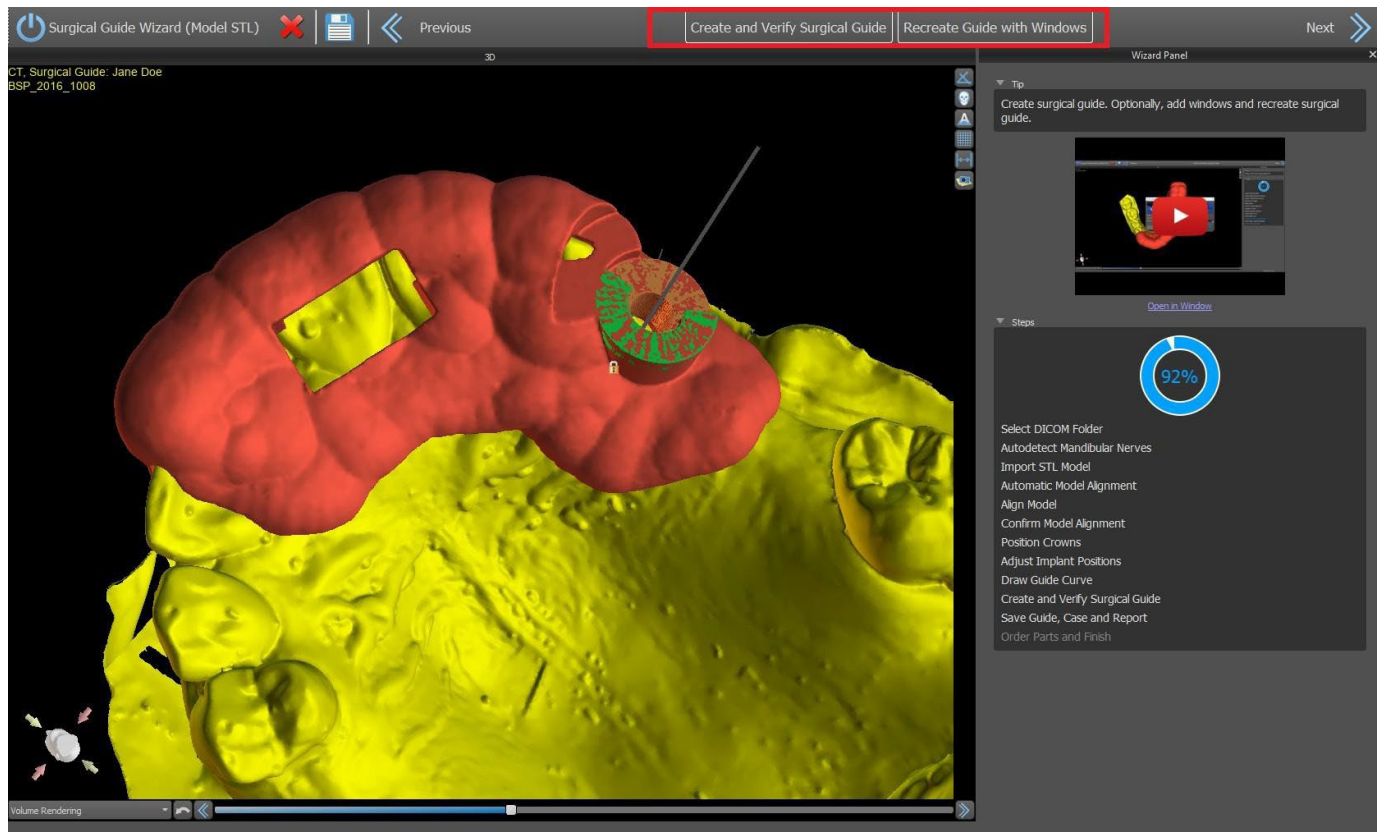

The automatic surgical guide suggestion can be enabled in the Preferences > Surgical Guide > Run SG curve suggestion automatically.

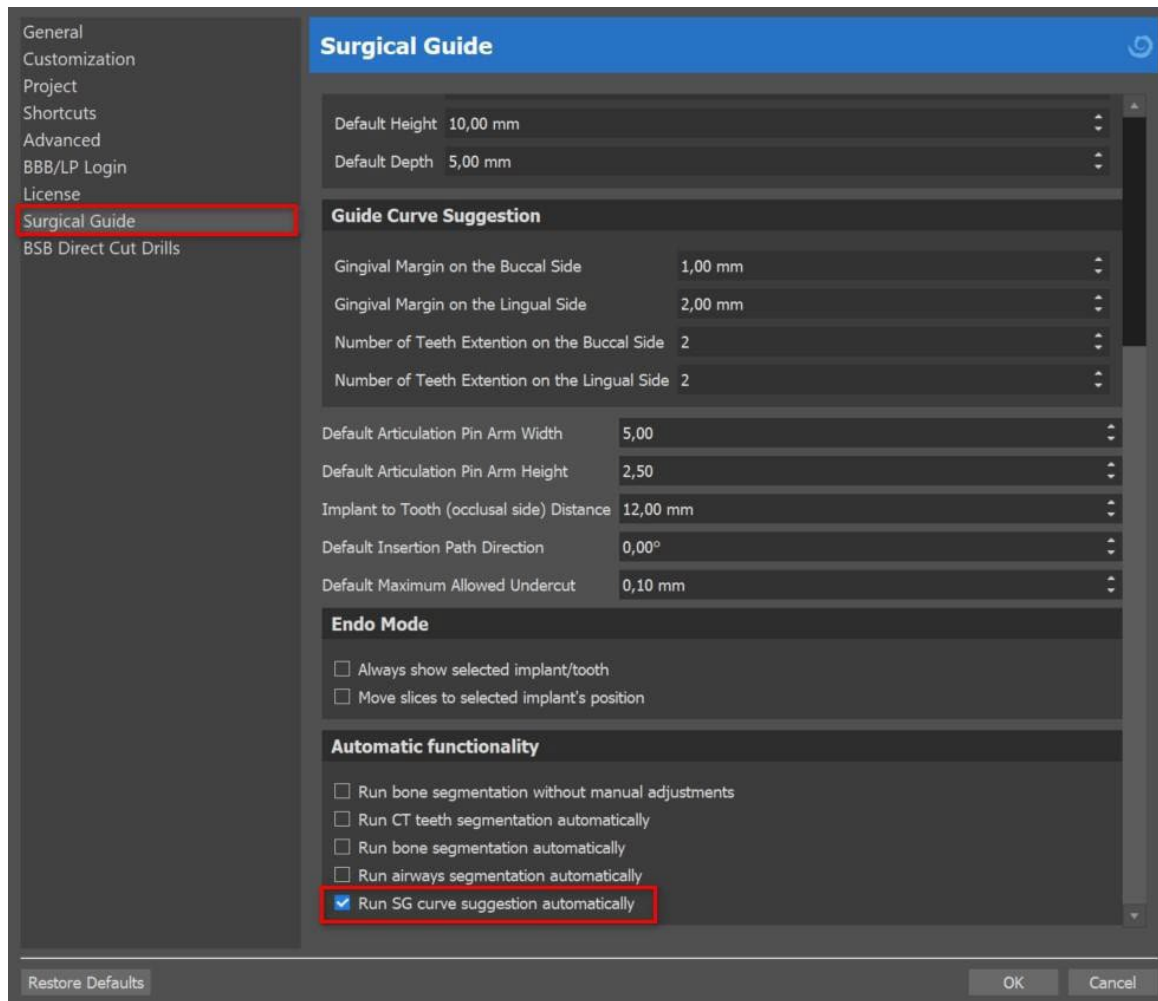


24.7 Saving and ordering

In the remaining Wizard steps, you first save the prepared project. The template created is exported and can be submitted via BioBigBox.com. Then the PDF report with basic information and screenshots is saved. The necessary parts can be ordered directly at <http://www.blueskybio.com>. Then you can continue with a new case or close the Wizard.

25 Orthodontics Module

Blue Sky Plan makes it possible to plan orthodontic treatments based on imported STL models of the jaw with teeth. All of the controls are available in the “Orthodontics” panel.

Planning begins with an intro screen entitled “Orthodontics”, where the user selects the type of planning – “Aligners”, “Indirect Bonding Tray - Align Brackets to Teeth”, or “Indirect Bonding Tray - Align Brackets to Wire”. For more information, see chapter 3 “Blue Sky Launcher”.

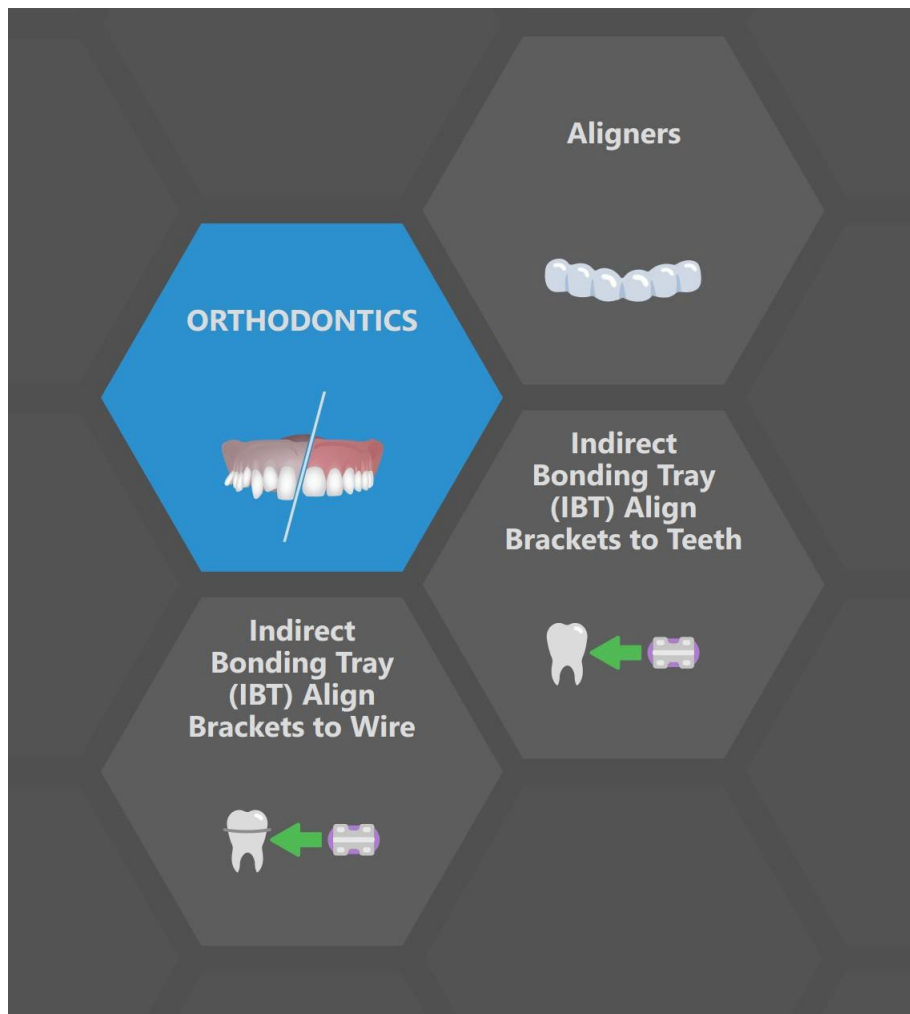

The specific type of planning can't be changed in an upcoming process. You can view the selected type of planning for your project in the top left corner of the 3D window where the patient data is shown.

The functions for model preparation are identical in all types of planning. Specifically, this includes the steps for Loading the Input Model, Jaw Alignment, Teeth Segmentation and Model Trimming.

The individual types of planning include:

Aligners: Teeth Moving – Edit Steps – Add Attachments (optional) – Design Aligners or Aligner Trim Curve (optional) – Export


IBT (Align Brackets to Teeth): Add Brackets – Prepare IBT – Export


IBT (Align Brackets to Wire): Teeth Moving – Edit Steps – Add Brackets – Prepare IBT – Export

25.1 Loading the Input STL Model

The planning starts by loading the STL jaw model using the menu selection “Import Models” and choosing the correct orientation. It is also possible to use the option “Import Patient CT Scan”, with the stepwise uploading of CT data and jaw models. The alignment of the STL model and the CT scan is done via “Matching Teeth”, and the STL model can then be further worked with.

If the other jaw (the opposing arch) is available, it can be imported in this step. After it has uploaded, the user is able to show/hide this other model together with the “Collisions” and “Closeness” functions, switching on the colour visualization of collisions as the teeth approach each other or collide.

Both jaws should be automatically aligned, or it is possible to activate the model alignment widget  to allow the user to move the jaw manually in the 3D view. Another function for this purpose is “buccal bite registration” (see chapter 25.2).

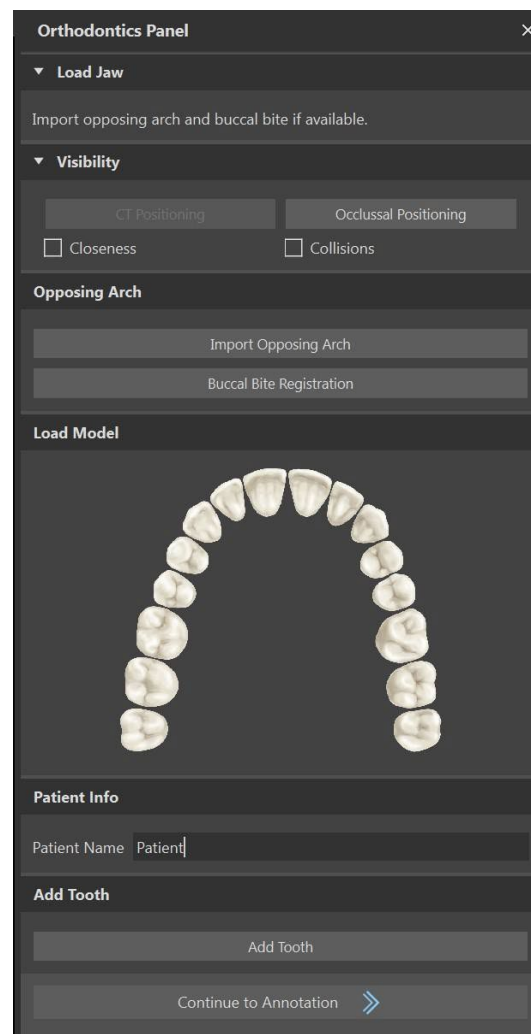

Blue Sky Plan also permits the processing and planning of both jaws in a single case. This requires uploading and aligning both models in the import stage. This step further includes entering the patient’s name.


If there is a missing tooth in the STL model, the user is able to add a tooth via the “Add Tooth” button. By clicking the option, the Add Tooth dialog appears and it is possible to choose a tooth from several teeth geometry sets.

After the tooth selection, simply click on the OK button and left click on the model with missing tooth. It is possible to adjust the tooth position or remove the tooth by right clicking the mouse > Remove.

Note: Teeth can be added/removed only in the import step. The selected tooth must match the correct jaw!




Add teeth

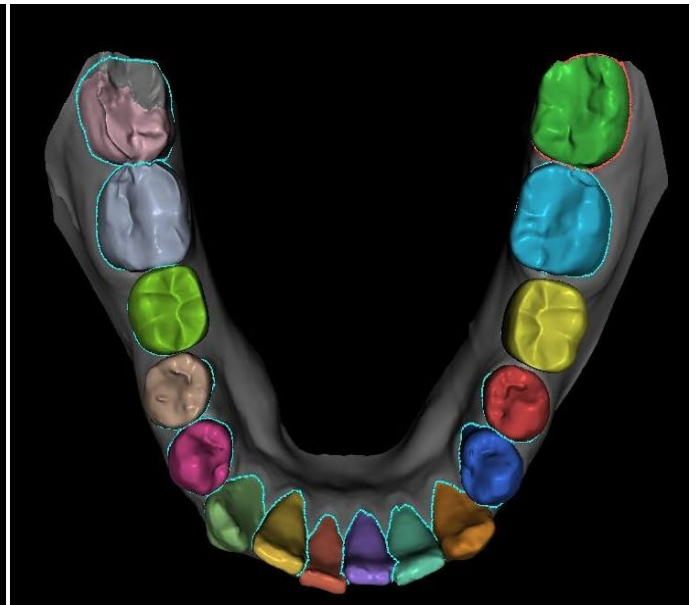

Merged teeth in the segmentation step

It is possible to automatically run the steps "Annotation", "Segmentation", "Trimming", and, if CT data are available, also to align STL models to CT data. If enabled in Preferences, all the selected steps will be skipped and the user will get to the "Teeth Moving" or "Continue to Add Brackets" steps faster.

"All skipped steps can be returned to using the "Back" button"

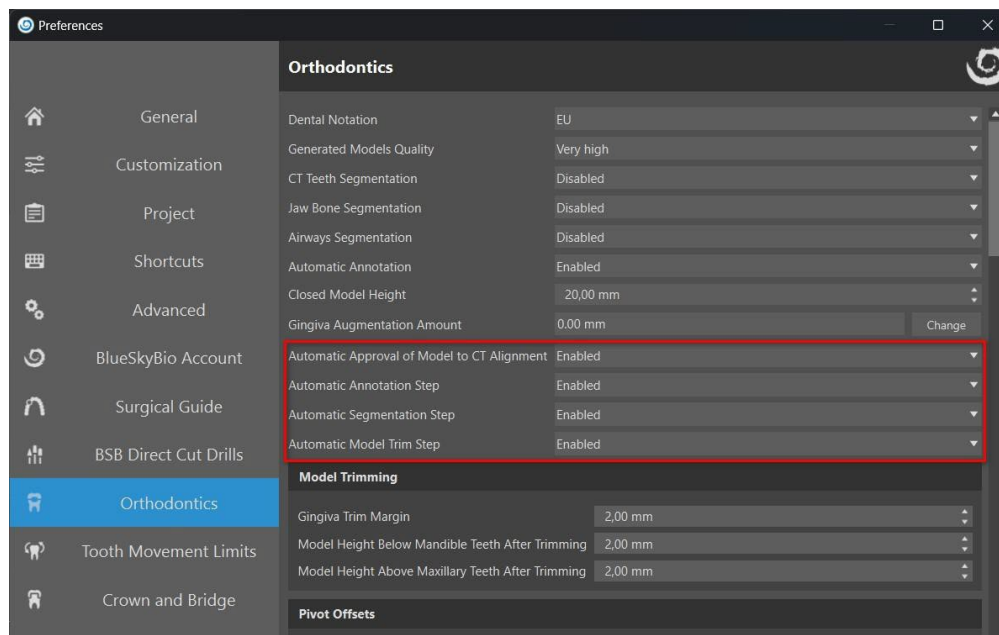


If the automatic function for the steps is disabled, proceed to the next step by clicking the "Continue to Jaw Alignment" button.

25.2 Buccal Bite Registration


Blue Sky Plan allows for upper-lower jaw alignment using a "buccal bite" surface scan. This process requires both jaw models to be imported. Then, the "Buccal Bite Registration" button can be used for transferring to the special "Buccal Bite Registration" panel. The maxilla and the mandible are already automatically uploaded to the panel and it is only required to add one or more buccal bite scans. The software then automatically aligns the maxilla and the mandible. Additional buccal bite alignments can

be created (according to the number of the buccal bite scans) and visualization is possible for the mandible moving from the normal to each of the additional buccal bite alignments.

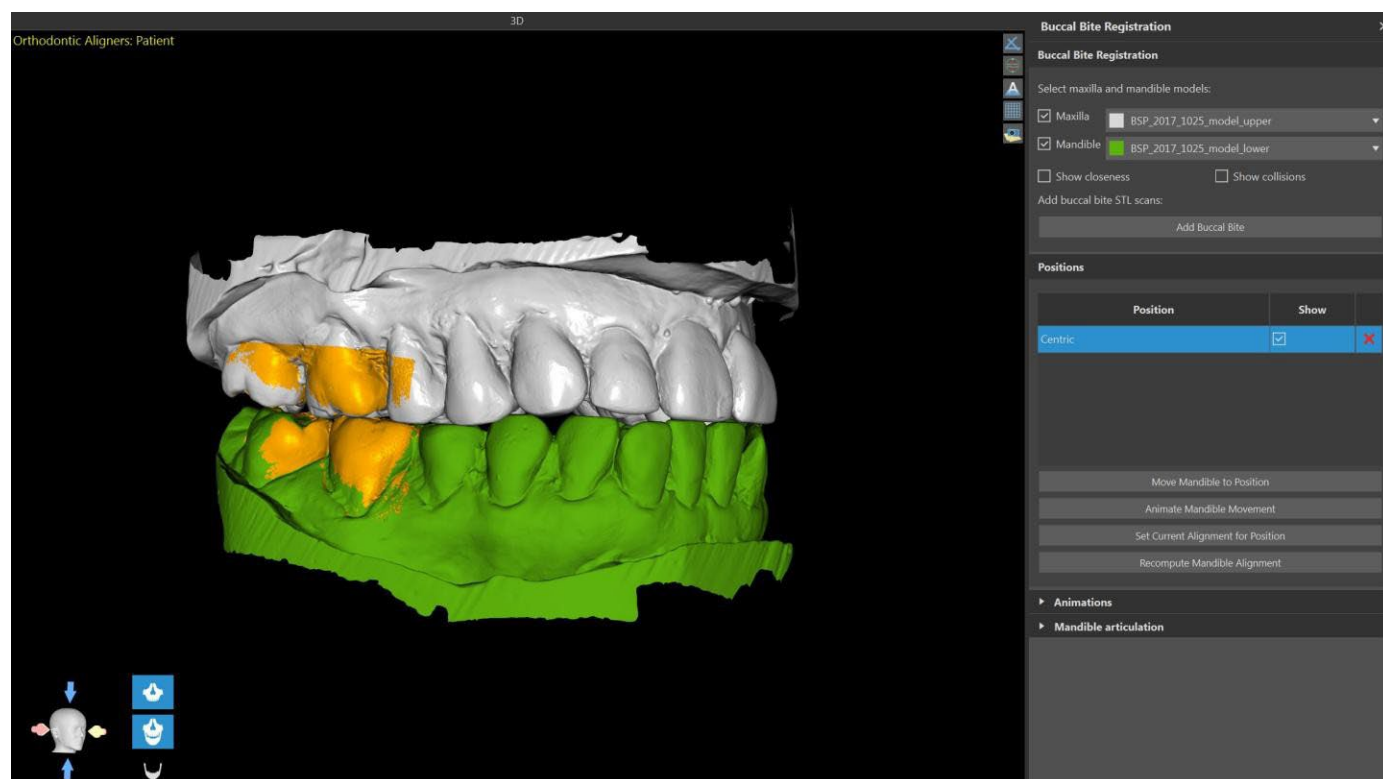

25.3 Mandible articulation


The lower jaw articulation can be visualised by adding jaw articulator. This functionality is not dependent on import of the buccal bite and is available in all types of planning and all their steps on Buccal Bite Registration panel. Collisions between the upper and the lower jaw can be shown in all positions.

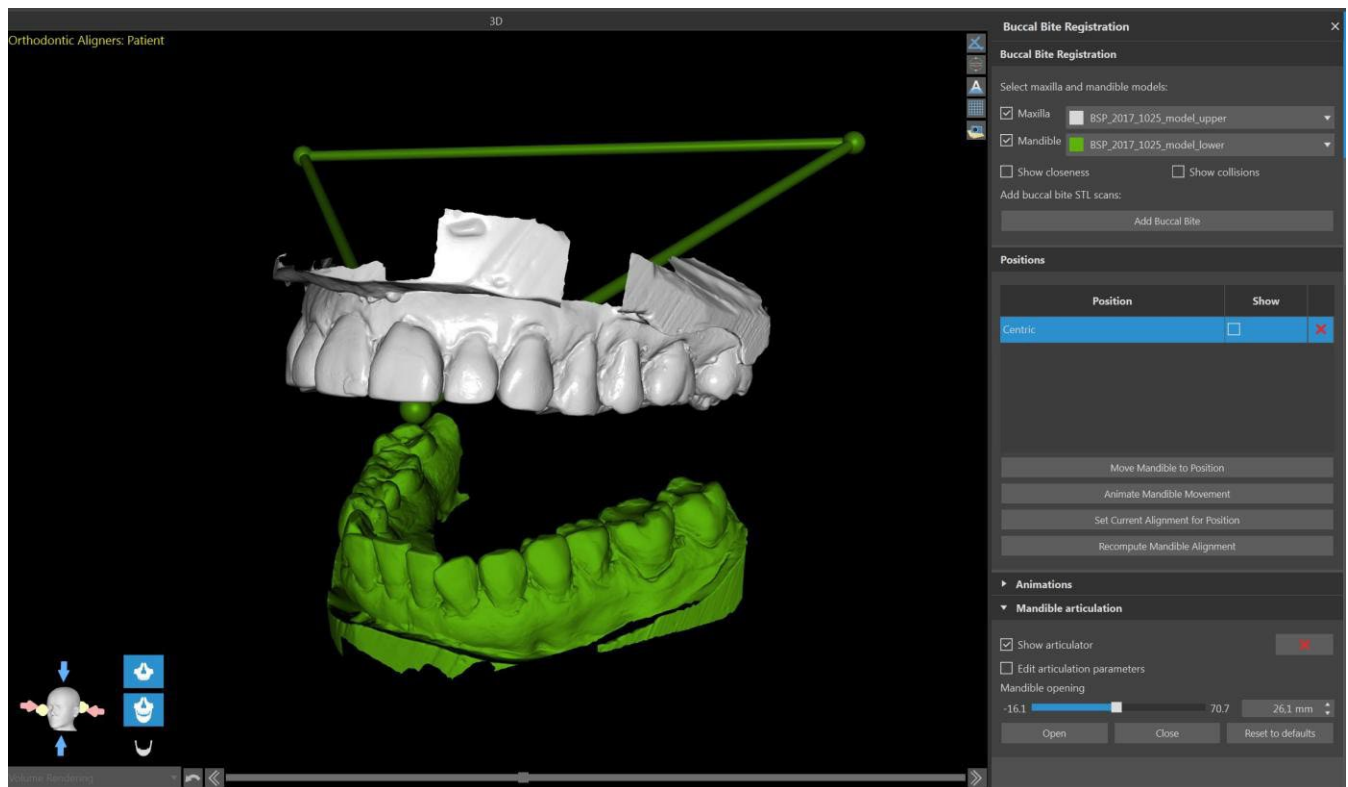

25.4 Annotating Teeth and Jaw Alignment

25.4.1 Manual Teeth Annotation

Once the jaw model is imported, it is necessary to mark all teeth, including those teeth that will not be moved or teeth that are missing. Teeth annotation is only possible when the “Mark All Teeth Manually” button is active.

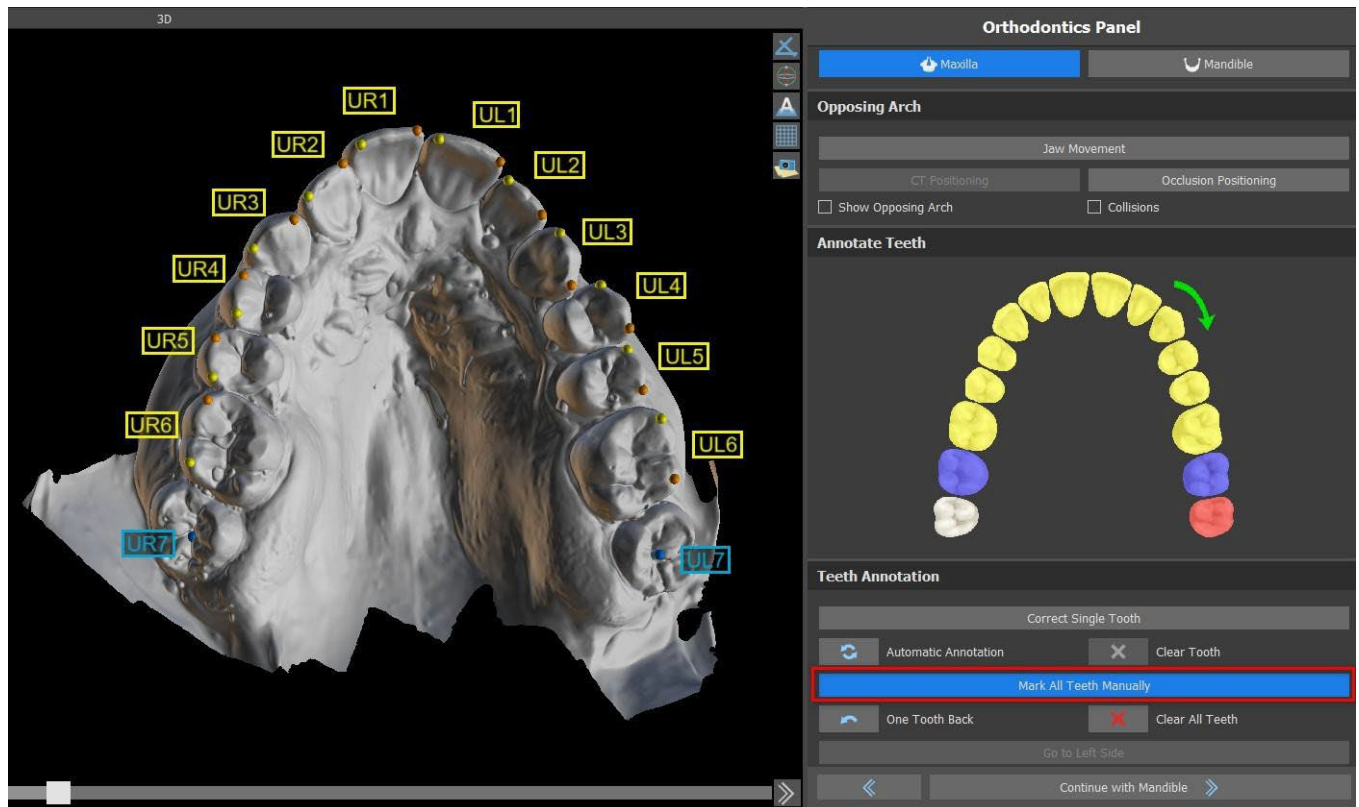


The tooth from which the teeth marking starts is highlighted in red in the schematic jaw figure in the panel. First, the teeth on the patient's right-hand side are annotated starting from the incisor and finishing with the molars. To mark a present tooth that will be moved, hold the "Shift" key and left click on the tooth. Orange and yellow dots illustrate the annotated teeth on the 3D model. A tooth that will not be moved, or is missing, should be marked by right clicking on the mouse (blue dots appears on the model).

Once all the teeth on the first side are annotated, you can proceed with the other side by clicking "Go to Left Side" starting from the incisor highlighted in red again.

It is not necessary to annotate all the teeth on each side. If the last molar, or so, is not present, you can skip marking the remaining teeth on the side by clicking either "Go to Left Side" or "Finish Teeth Marking" .

While marking the teeth, it is possible to return one step back (the "One Tooth Back" button) and repeat the marking of the previous tooth, or to start from the beginning again by pressing "Clear All Teeth" . The colored (blue and yellow) labels indicate the teeth names.

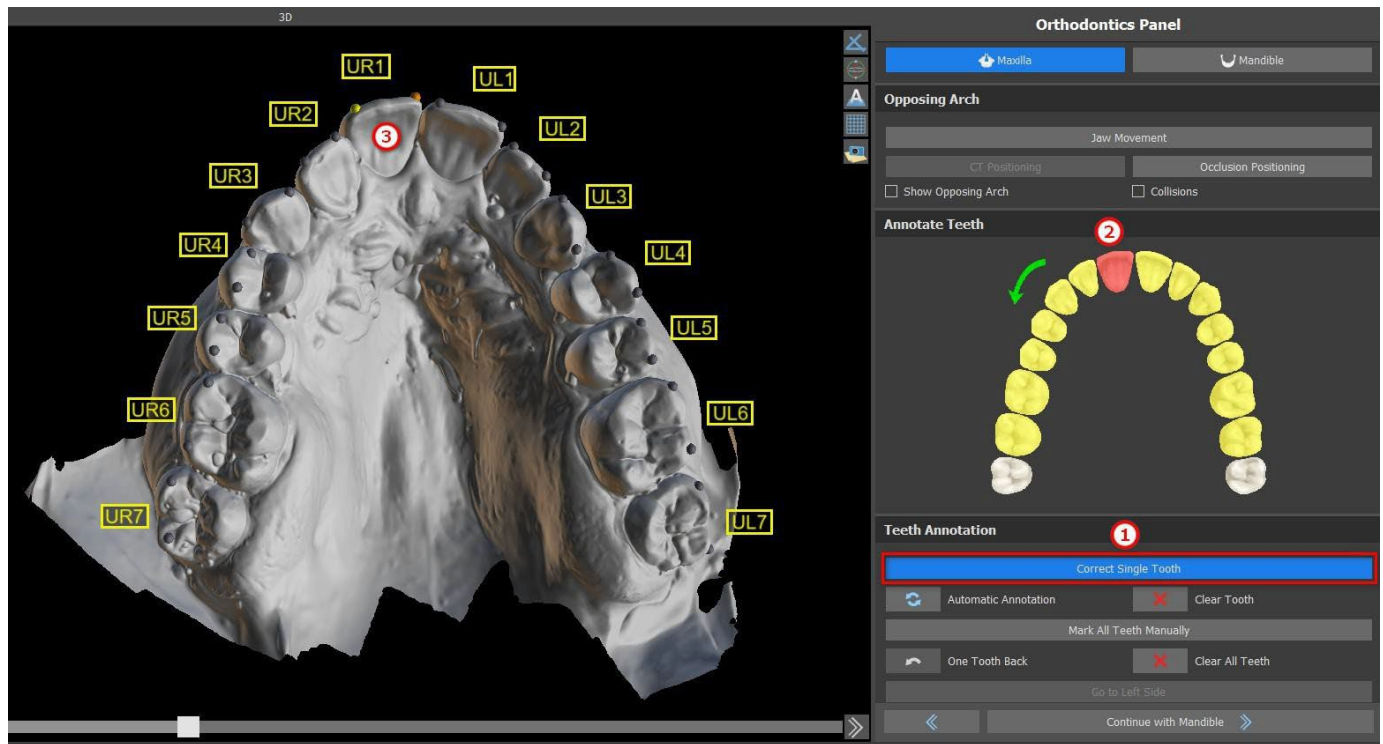


A single tooth correction is possible by clicking on the “Correct Single Tooth” button (1). Click on the scheme to select the tooth (2) and correct the annotation on the jaw model (3). To remove the annotation from the selected tooth, click on “Clear Tooth” button. It is possible to mark the tooth again by holding the “Shift” key and left clicking on the tooth.

To complete the step, click the “Continue with Mandible” button.

Note: The proper marking of the teeth is crucial for the subsequent automatic teeth segmentation step. It heavily impacts the precision of the segmentation. Incisors and canines should be marked by clicking/placing two dots on the occlusal edge of each tooth. Molars and premolars should be marked by placing the dots on the buccal side of the tooth surface as indicated in the image above.

25.4.2 Automatic Teeth Annotation

Blue Sky Plan allows the user to automatically annotate the teeth. To disable or enable this function go to Tools -> Preferences -> Orthodontics -> Automatic Annotation.

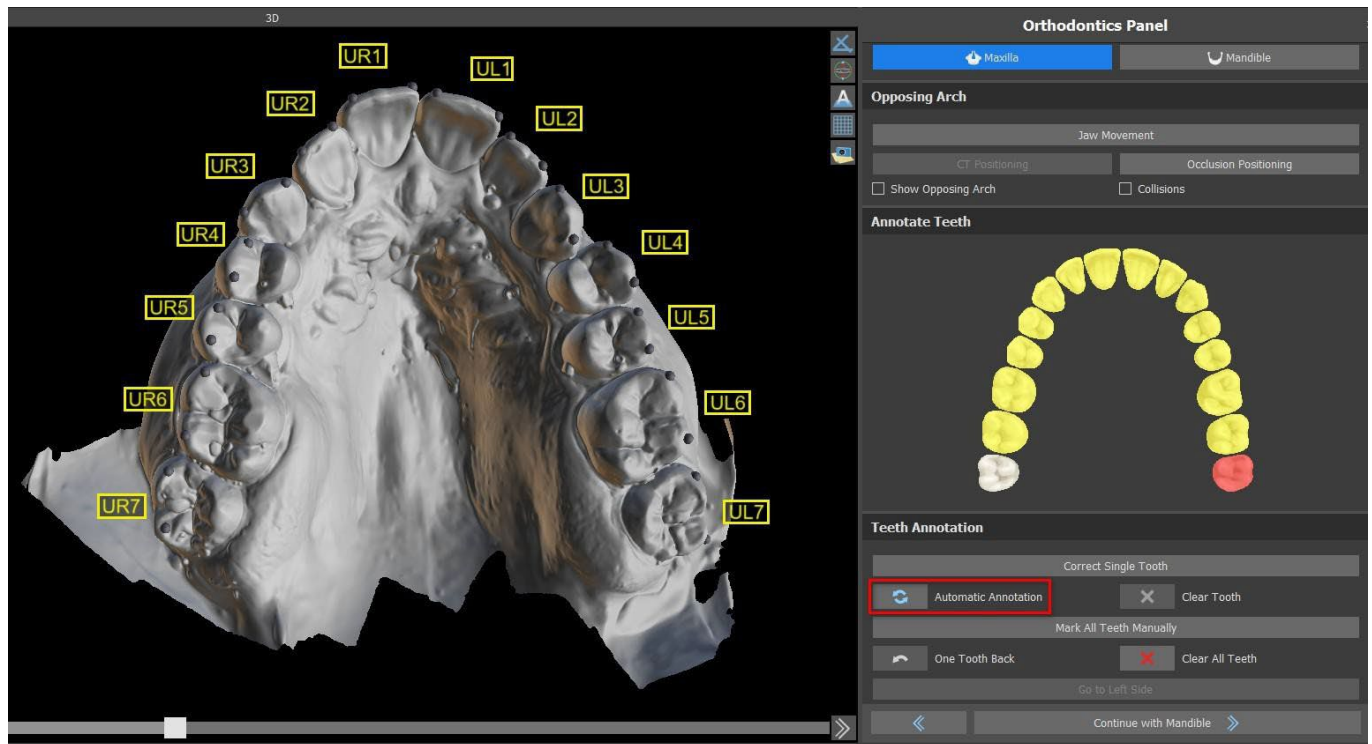

This functionality works exactly the same way as manual annotation. The user is able to switch between manual and automatic annotation for manual modifications of the dots.


The thickness of the final stone model, i.e. the minimal allowed height in millimetres, is at least 20 mm and can be increased in Preferences.

25.5 Two Jaw Functionality


Once the teeth of the first jaw (maxilla) are annotated and the model is closed, the transfer to the mode of planning both jaws is activated. By default, the workflow is set so that after clicking “Continue with Mandible” a direct transfer is performed to the second jaw annotation. To plan both jaws, both models must be annotated and aligned. In the following steps there is the option to actively work with one or the other jaw. If the dentist uses the other model (the opposing arch) for visual control in the course of the planning step and for the depiction of possible inter-mandibular collisions only, the “Skip this Jaw” button can be used for direct transfer to a single jaw segmentation.

25.6 Teeth Segmentation

The transfer to the step of the automatic segmentation of individual teeth may take a while as the program tries to identify the gingival margin automatically.

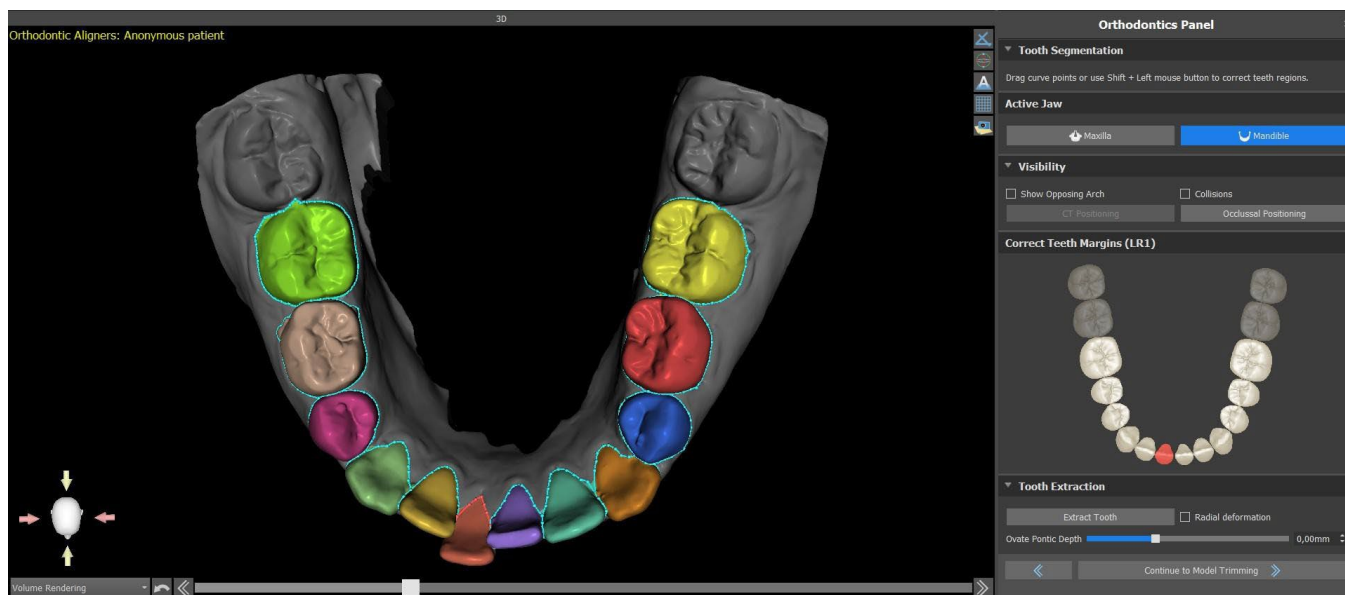


After the automatic segmentation, you can adjust the detected gingival margin of the active/selected tooth by dragging the control points of the curve, or by manually drawing a part of the curve on the model while holding down the “Shift” key and left mouse button simultaneously.

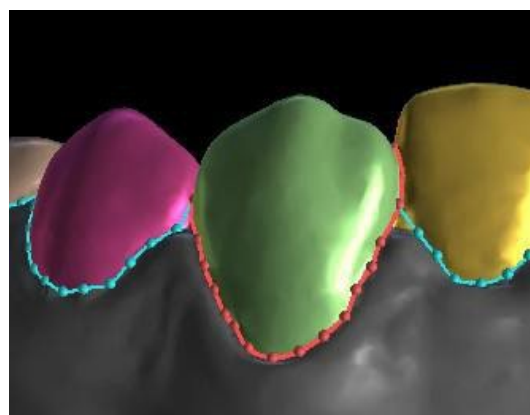

The gingival margin can be edited for the active tooth only. The active tooth can be changed throughout the schematic jaw image in the panel, or by clicking the segmented tooth directly in the 3D view.

Select the tooth you want to remove and click the Extract Tooth button. The last tooth can't be extracted.


Radial deformation affects the shape of the resulting ovate pontic. When turned on, the amount of surface deformation depends solely on the distance of the surface point from a theoretical center of the tooth, resulting in a symmetric, radius-dependent shape. When toggled off, the amount of deformation applied to the gingiva at the pontic site is calculated adaptively to the shape of the tooth's boundary, resulting in an ovate pontic that copies the boundary shape.

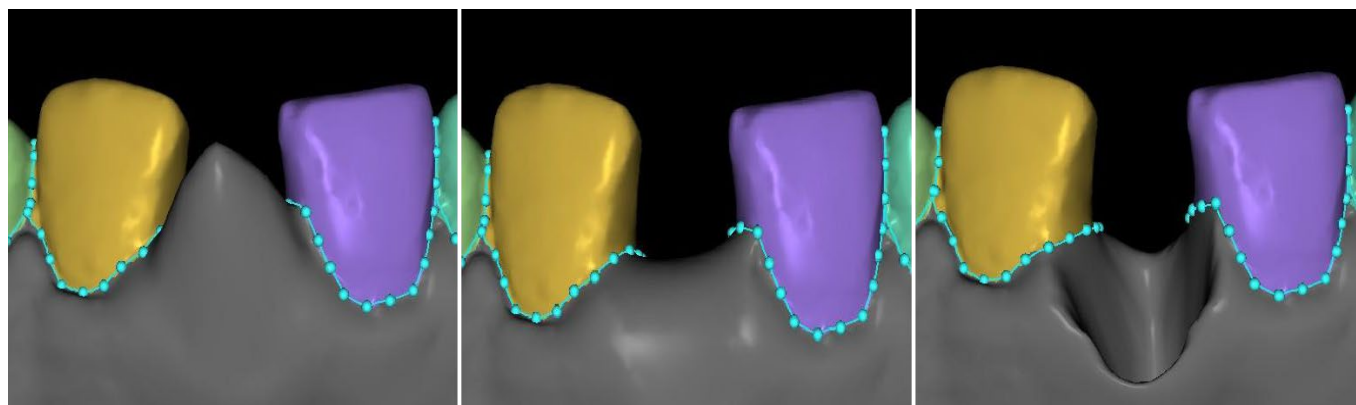

Ovate Pontic Depth: -5 mm

Ovate Pontic Depth: 0 mm

Ovate Pontic Depth: 10 mm

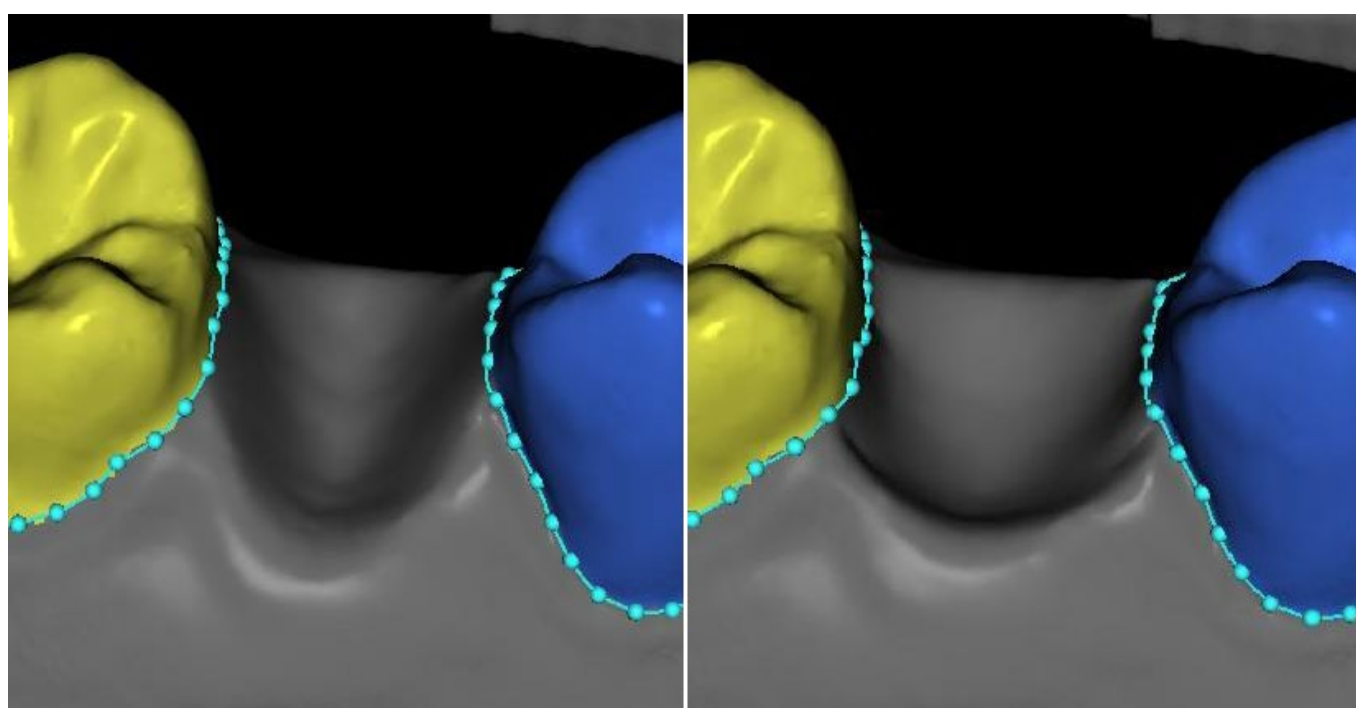

Tooth extraction without using "Radial Deformation"

Tooth extraction using "Radial Deformation"


Once the boundaries need no further refinement, you can finish the segmentation by clicking the "Continue to Model Trimming" button. This applies the changes made and separates the teeth from the 3D scan being able to move them.

25.7 Model Trimming

The STL model imported to BSP may contain excess material and printing it would mean an unnecessary waste of printing material. For these situations, BSP offers the trimming function by means of a curve. The trimming curve is created automatically at a distance from the teeth based on the "Gingiva Trim Margin" parameter (which can be modified in Preferences). The curve is further manually adjustable by dragging the control points.


Part of this step is also the closure and trimming of the model at a specific height based on the distance of the segmented teeth. The parameters "Model Height Below Mandible Teeth After Trimming" for the lower jaw and "Model Height Above Maxillary Teeth After Trimming" for the upper

jaw can be modified in Preferences. The Model Trimming step can be skipped by using the options “Don’t Trim Maxilla” or “Don’t Trim Mandible”.

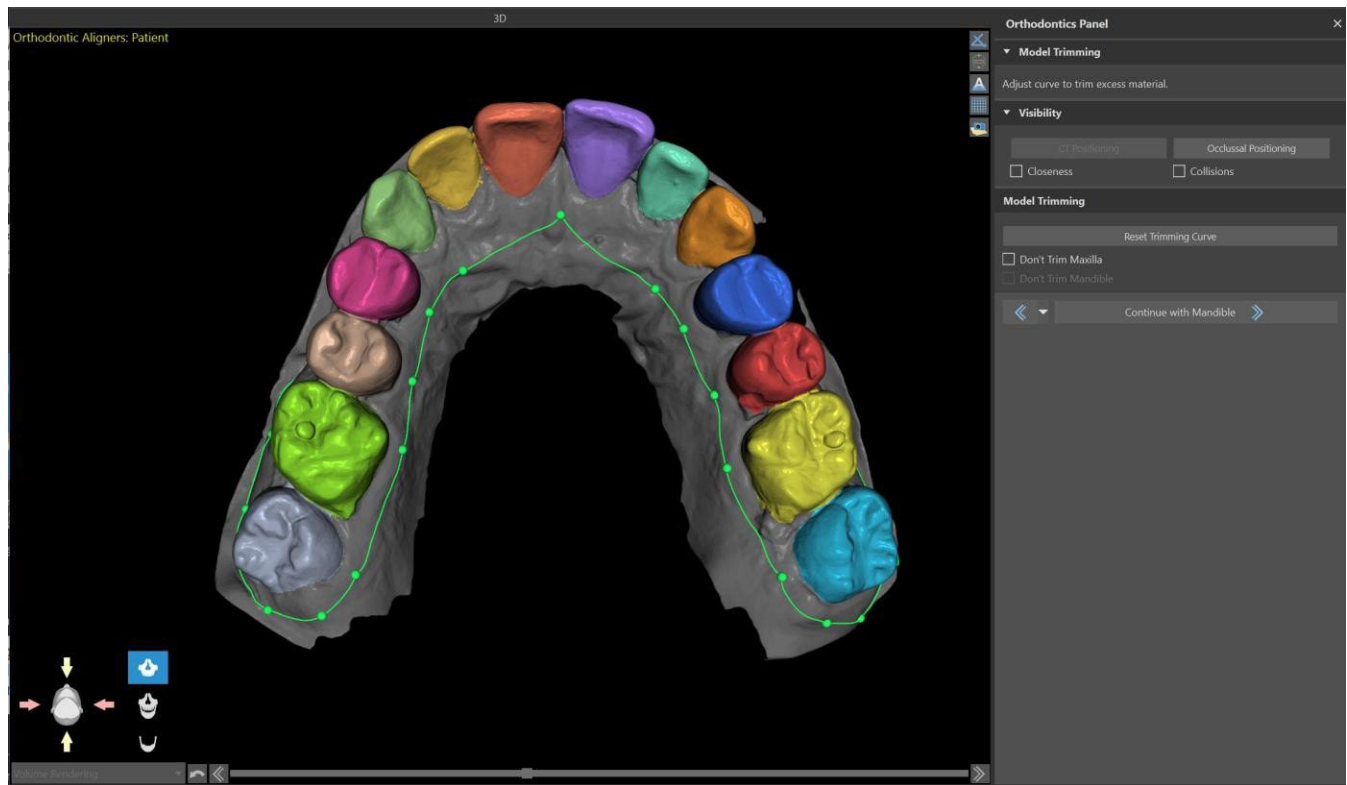

In this step, Model Trimming is only defined. It will only start after clicking on Continue to Teeth Moving, so this operation may take a while.

After the separation of the teeth, the gingiva may be augmented (i.e. inflated) to prevent interference with the aligner and to help prevent tissue impingement. Note: Due to the optimisation in the processes, the .0mm value is recommended for versions 4.7.50 and higher. The .2mm is recommended for versions 4.7.20 and older. This is the “Gingiva Augmentation Amount” parameter in Preferences.

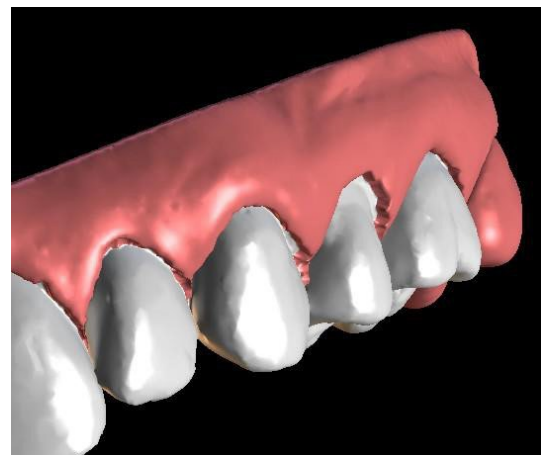


Also, the colours of the 3D model and the teeth change to a more natural pink and white, respectively. Now the software allows you to reposition the teeth and plan their final positions.

25.8 Teeth Moving




In this step, the target positions of individual teeth can be planned. You should start by selecting the active tooth – either by left clicking on a tooth in the schematic jaw image in the panel, or by clicking the tooth in the 3D view.

The active tooth has a visible tooth midline and a 3D manipulation widget shown around it. Using the widget, you can move and rotate the tooth in the following types of movements:


- Mesial / Distal tipping
- Buccal / Lingual tipping
- Rotation along longitudinal tooth axis
- Intrusion / Extrusion
- Left / Right
- Forward / Backward

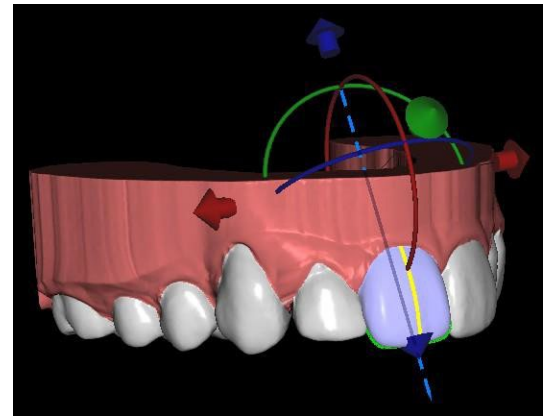

A couple of interesting functions can be activated in the orthodontic panel after this step. The "IPR" option will show a 3D view of the sizes of the over-limit Interproximal reductions, and the limits of the maximum allowed IPR can be set in Preferences. The red areas informatively visualise inter-teeth collisions. "Diastemata" shows the measurement of the space between the teeth. "Initial Teeth Positions" shows the original model before positioning the teeth. "Final Teeth Positions" shows the last model after positioning the teeth. "Teeth Midlines" allows the user to see the estimated tooth axis with its midline. Teeth widths and their sum is also displayed by checking "Teeth Widths".

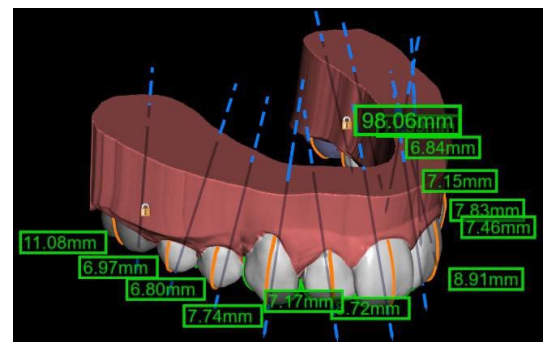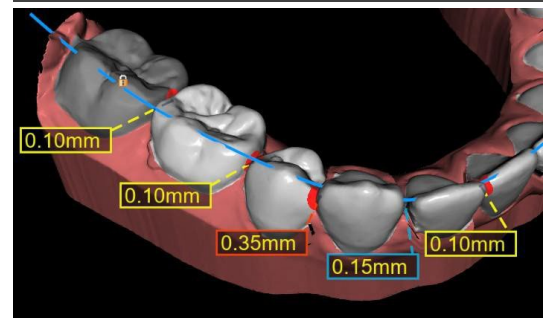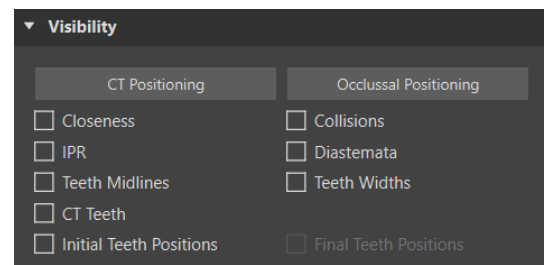


The "Automatic" section (1) allows the final positions to be computed for the aligning treatment. This requires the user to upload and annotate both jaws. "Align Both Jaws Automatically" (2) runs the alignment algorithm repeatedly, trying to improve the final positions with every step. Collisions between jaws are not resolved. IPR is automatically applied afterwards. "Align without IPR resolving" (6) only runs the algorithm once and doesn't resolve IPRs. Each run only tries to improve the positions of the teeth. "Resolve IPR" (7) tries to resolve IPRs by arch expansion and teeth shifting. The "Alignment Repetitions" (3) slider shows the number of algorithm runs,

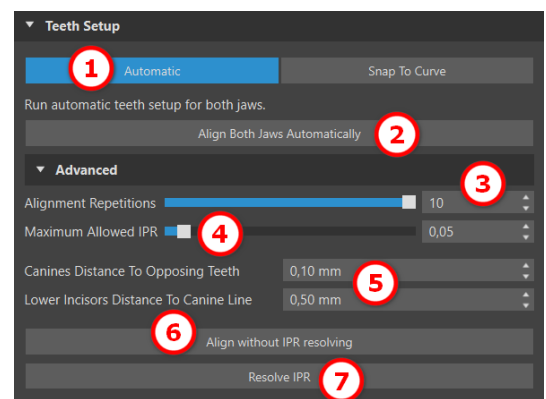

10 by default. The "Maximum Allowed IPR" (4) slider allows IPR up to a certain value, as the default is no IPR. Additional parameters can be edited in the leveling section for every jaw (5).



The "Snap to Curve" section (1) allows the user to adjust a curve representing an ideal teeth arch and snap one or all teeth to this curve.

In order to straighten the teeth to the curve while keeping the IPR data in an acceptable range, select "Level to Curve" (2) and click on the "Snap All Teeth" button. When the user works with the mandible jaw and an opposing arch is available, the software allows the user to level the teeth to the opposing ones (3). Click on "Snap All Teeth" for snapping. Additional parameters can be edited in the leveling section for every jaw. (4). The "No Levelling" (5) option snaps the teeth without extrusion or intrusion. By clicking on "Edit Curve" (7) the user edits the curve points symmetrically or asymmetrically. This button also enables "Move with Curve", where it is possible to move both the curve and the teeth at the same time. When "Edit Curve" is active, the measurements of the dental curve are visible. The "Reset Curve" (8) button resets the curve to its original position. If the user clicks on the "Edit Curve" button again, the curve can no longer be edited.

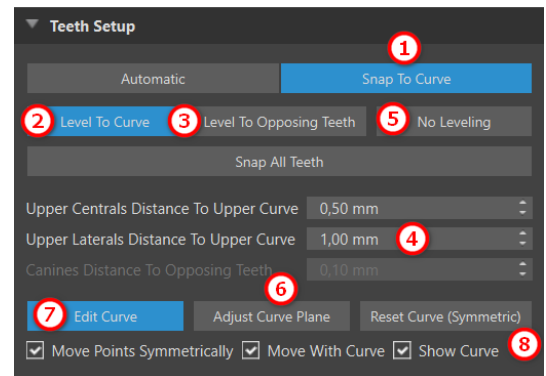


Clicking the "Adjust Curve Plane" (6) button activates the occlusal plane for adjusting the curve. It is possible to move the plane in the 3D scene based on the angle and distance of the plane. Edit

Additional parameters are available in the Preferences for Orthodontics, Tooth Movement section.

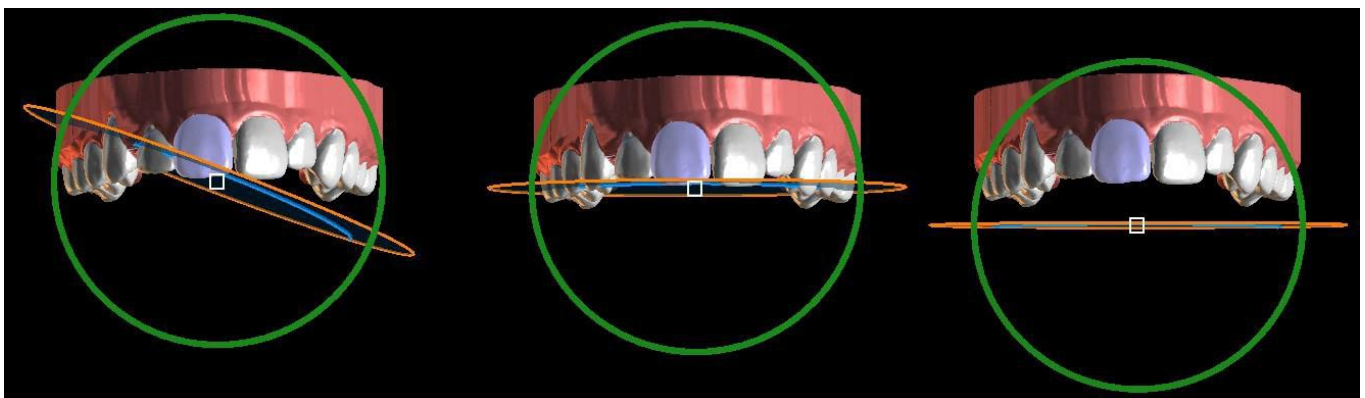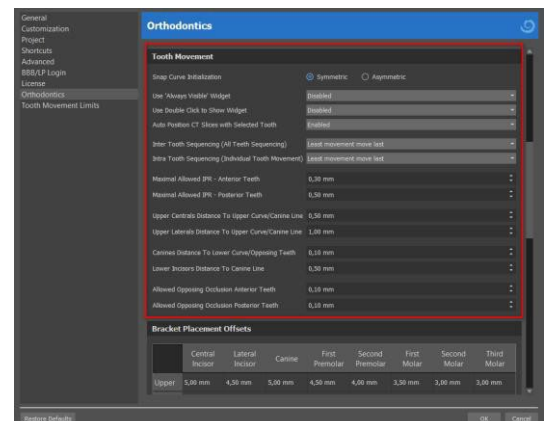

Curve Plane Adjusting

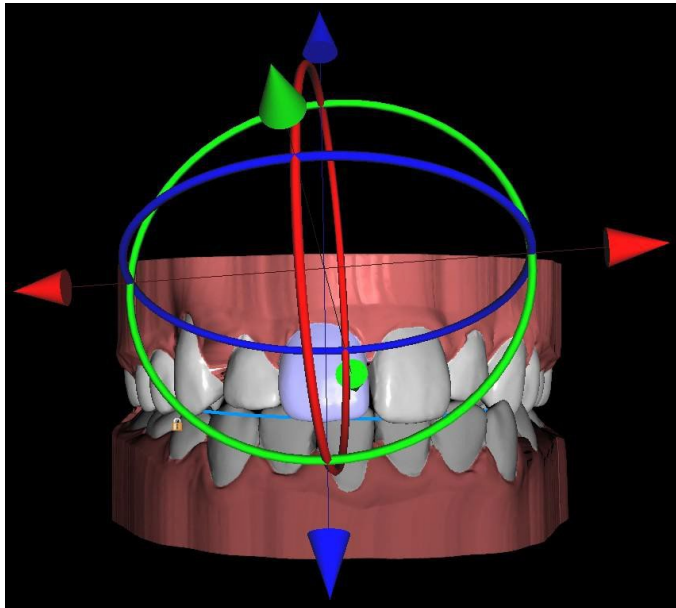

"Always visible" widget enabled



"Auto Position CT Slices With Selected Tooth" enabled

Every tooth can be locked in position by right clicking on the tooth and selecting "Lock Tooth" from the content menu. Unlocking proceeds in a similar manner, or the user can unlock all teeth at once by clicking on the "Unlock All Teeth" button.


The user can also adjust the midline when the "Adjust Midlines" button is active.

The "Reset All Teeth" button returns all the teeth to their original positions. As the active tooth is being moved, the "Orthodontics" panel shows information about the total movements in the form of distances [mm] and angles [°] from the initial position. Movement limits can be set as monthly, bi-weekly or weekly.

Tooth Deviations (LR1)	
Trans - Buccal/Lingual	-0,02 mm
Trans - Mesial/Distal	0,86 mm
Extrude/Intrude	-0,08 mm
Tip - Buccal/Lingual	-4,70 °
Tip - Mesial/Distal	1,29 °
Rotation	0,77 °
<input type="checkbox"/> Transfer Movement Locally	
Limits	Monthly

Multiselection allows multiple teeth to be manipulated at once - hold Shift key and by left clicking select multiple teeth. With multiselection, it is possible to utilize all the manipulation methods (via the manipulation widget, deviations table and arrow keys). Checking the "Transfer movement locally" checkbox, the tooth's local deviation is applied to other selected teeth.

Context menu options (locking, snapping, changing teeth movement speed, etc.) can also be used with multiselection.

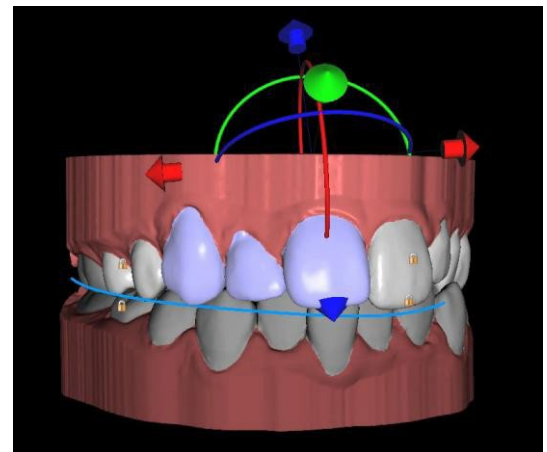

Milestone is a mandatory step across all teeth (i.e. "sync point"). Final and initial positions are implicit milestones, but it's possible to create additional milestones if necessary.

Milestones can be created in two possible ways:


- By clicking the button "Insert Milestone" (2)
- By duplicating the "Teeth movement" step

The slider (1) shows the sequence of individual milestones. Each milestone (excluding initial position) can be deleted




(3). It's also possible to clear the custom milestones and return to default ones (4).

When positioning a tooth, it is sometimes useful to see the movement of the whole tooth, including its roots. Therefore, Blue Sky Plan allows you to add a virtual tooth with roots simply via the context menu > Show Virtual Tooth or by enabling the "Virtual Teeth" checkbox (note: This option won't be available if you import CT Data). To refine the shape and position of the newly added virtual tooth so that it closely matches the underlying teeth separated from the 3D scan, click on the "Adjust Virtual Tooth" button. Now, you may adjust the position and size of the virtual tooth using the in-scene widget. To hide the virtual teeth, click on the "Virtual Teeth" checkbox again.

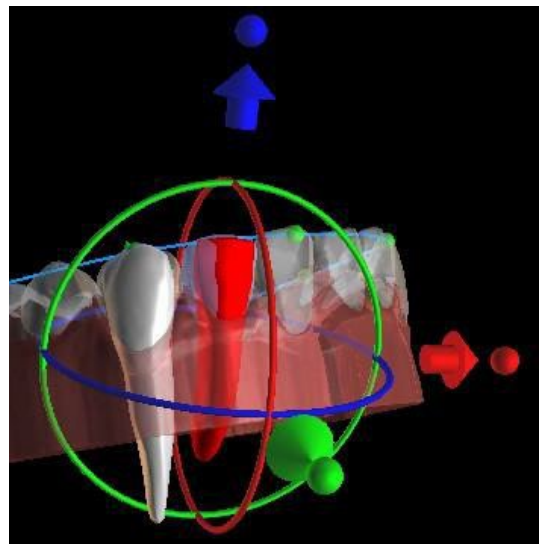


To work with a selected tooth, right click on the tooth and choose the functionality. To hide the tooth, select "Hide virtual tooth". To reset the tooth to its original position, select "Reset virtual tooth position". It is possible to change the point of rotation of 3D manipulation widget, i.e. to switch between the tipping and torque. The tipping and torque angles are tracked and reported separately. Pivot offsets can be set in Preferences.

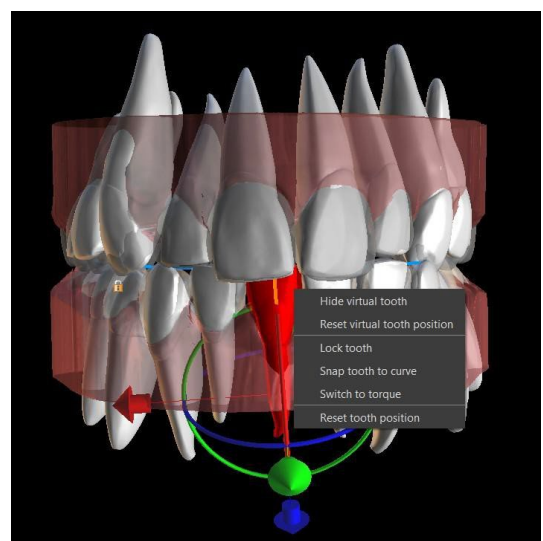

Blue Sky Plan software allows the user to work with segmented CT data.

Go to Preferences > Orthodontics > CT Teeth Segmentation to enable segmentation of CT teeth.

To hide all the visible CT teeth simply click on the "CT Teeth" checkbox. To hide one (or more) CT tooth, right click the selected tooth and choose "Hide CT tooth". For better clarity of the roots, the transparency of the jaw can be increased by moving the slider in the panel (1).

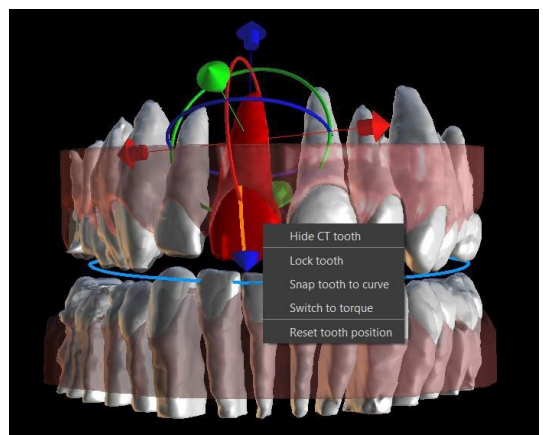


The user can enable automatic jaw bone segmentation. This option is available in the Preferences > Orthodontics > Jaw Bone Segmentation.

When entering the Teeth Moving step, the segmented jaw bone with CT teeth will automatically become visible. To hide the data simply use the "Jaw Bone" checkbox. For better clarity of the CT data you can increase the transparency by moving the slider in the panel (2).


Airways segmentation can be enabled in Preferences > Orthodontics > Airways Segmentation. To hide the data use the "Airways" checkbox. For better clarity of the CT data you can increase the transparency by moving the slider in the panel (3).



Note: The options "CT Teeth Segmentation", "Jaw Bone Segmentation" and "Airways Segmentation" must be enabled during the import step of the CT Data in order to produce segmented data!

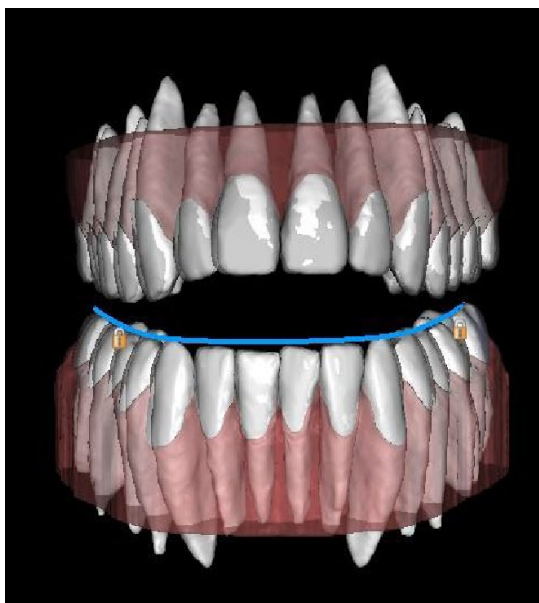

CT Teeth

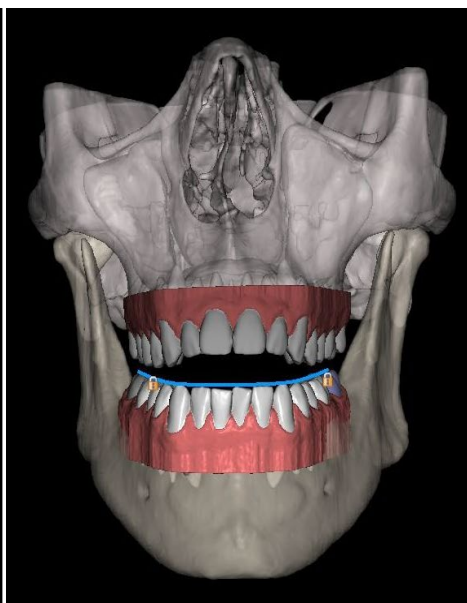


Jaw Bone



Airways

BSP allows the user to correct ongoing treatment with original final positions. The user loads the new scans and proceeds to the "Tooth Alignment" step, and selects the original project via File > Import Final Tooth Setup from Existing Case (1). When the project is loaded, the final position planning from the new project is updated and a color map showing per tooth alignment accuracy is displayed. After loading the original project, the "Confirm Import" button (2) will appear, after clicking it the color map will disappear, but the final positions will remain.

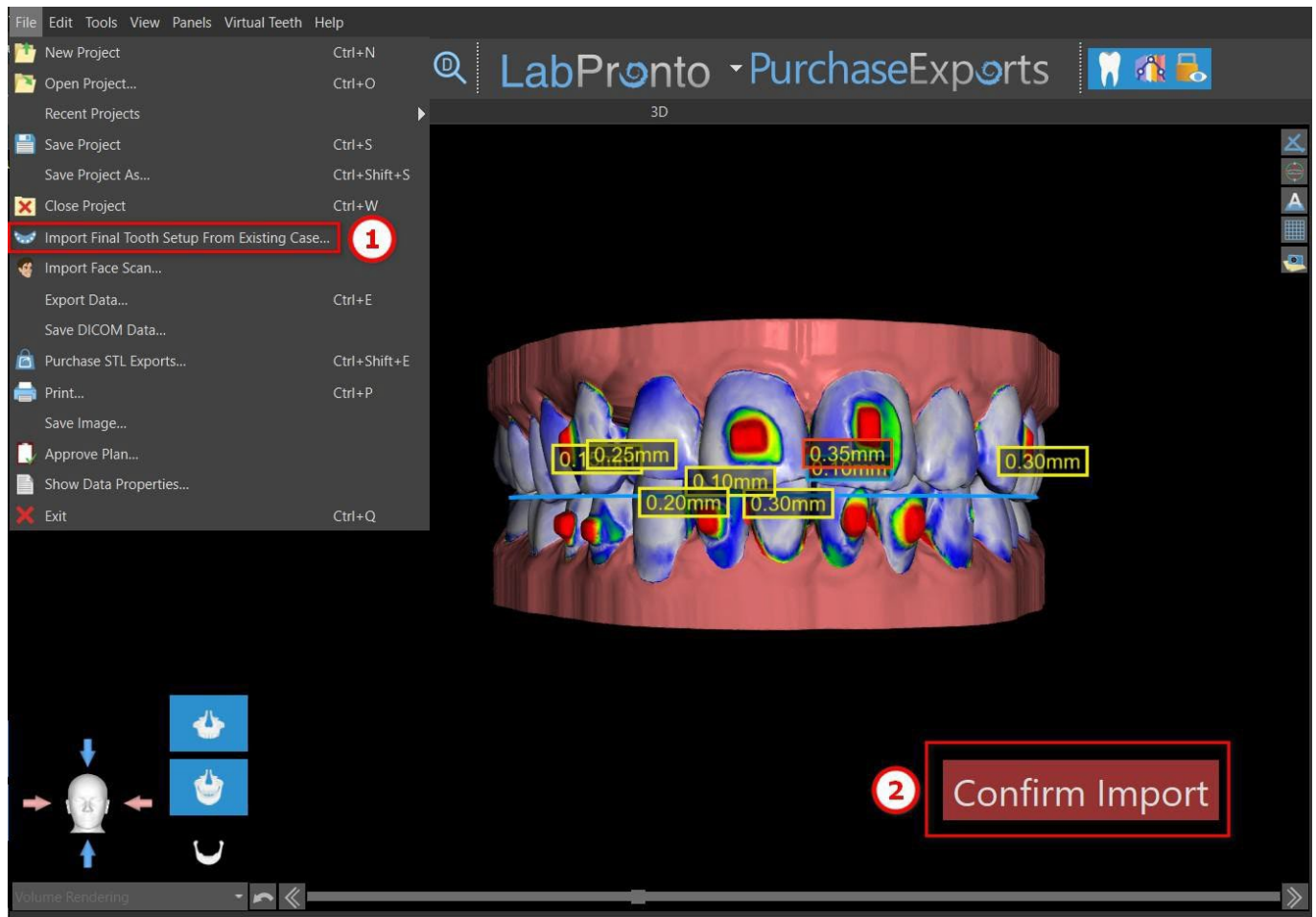

25.9 Generating Intermediate Positions

Once you are satisfied with the target positions of the teeth, use the “Continue to Edit Steps” button to generate intermediate positions and move to the next planning step.

The table in the “Orthodontics” panel lists the number of necessary intermediate steps to reach the planned target position of the active tooth. The number of steps is always the same for all the teeth, but it is possible to start the movement of teeth in later steps. The first and last row in the table represent the initial and target position of the active tooth, so these positions are locked and cannot be changed at this planning stage. The number of steps is estimated automatically based on the pre-defined settings of maximum movement limitations. The default movement limitations are stored in the application Preferences and can be adjusted manually.

The slider at the top of the “Orthodontics” panel (2) allows you to move teeth over time and in intermediate steps. If the slider is moved to the left, the teeth are visualised at the initial positions. The very right position of the slider represents the planned target positions. By dragging the slider, you can animate the movement of the teeth through intermediate steps over time. Further, you can move the slider to one of the interim steps and, if needed, adjust the position of the active tooth in this concrete step to prevent collisions with its neighbouring teeth.



The movement of an active tooth is illustrated by a green color in the table. Milestones are depicted in red. Orange rows are milestones with changes with respect to the previous step. The row selected by the user is marked blue.

In the table, there are two columns of checkboxes. It may be important to start the movement of the active tooth later in a step. The starting step can be changed by checking the checkbox in the left column (“Start Step”)(3). After any change, it is necessary to regenerate the entire table by clicking the “Refresh Steps” button (7). The checkboxes in the right column (“Mandatory”)(4) can be used to lock the intermediate positions of the tooth – the tooth always moves through the locked intermediate positions.

Three optional plans for organizing teeth movement are available – “Least movement move last”, “Least movement move first” and “Uniform movement”. “Inter Tooth Sequencing” (5) moves all teeth with respect to each other, while “Intra Tooth Sequencing” (6) is related to the rotation and translation movement for individual teeth. It is also possible to set Intra Tooth Sequencing by right clicking on the tooth.

The movement of all teeth can be postponed in one step by clicking the “Duplicate steps” button (8). The details about each tooth movement within the specific step can be found in “Step Details” (9). For an illustration of teeth movement in each individual step use “Open Movement Chart”. If necessary, each adjusted plan can be reset to the default one (10).

The movement speed of each tooth can be changed by right clicking on the tooth. Decreasing speed will slow down tooth movement by 65%, while increasing speed will accelerate tooth movement by 65%. Resetting the speed will return the tooth movement speed to the original values. Note, that any change of tooth movement speed will also affect the number of intermediary steps in the table.

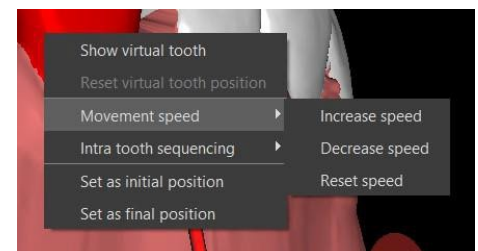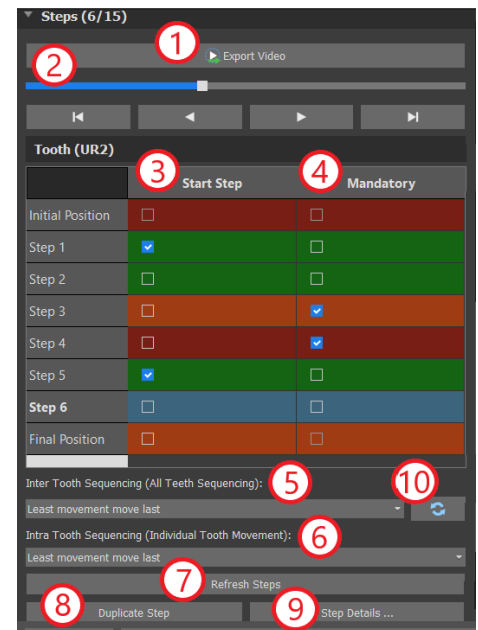


In the Aligners planning type, it is necessary, at the end of this step, to select which method of creating aligners will be used further on. For the generation and export of model intermediate positions only, the default selection “Export Models Only” is sufficient.

By selecting the “Design Printable Aligners” option, the aligners will be drawn and generated directly as models in BSP, which can later be exported. Selecting “Design Aligner Trim Curve” means that the trimline, applicable to the mining machine, will be drawn on the models and exported. It’s very helpful for automatic aligner trimming.

If attachments are meant to be added on model intermediate positions, it is necessary to check the “Add Attachments” as well. Doing that will add this step to the rest of the process.

The "Export Video" button (1) allows the user to export video content of teeth movement. A couple of interesting functions can be set in this option. Note that by default, the size of the video is given by the size of the 3D scene window. In some cases, this can produce large videos (e.g. on 4k monitors), which some video players may have issues with. To prevent these issues, you can limit the size of the video by choosing the "HD" or "Full HD" limit in the "Video size" option. Alternatively, you can install a different video player (e.g. the VLC media player) which can handle large videos.

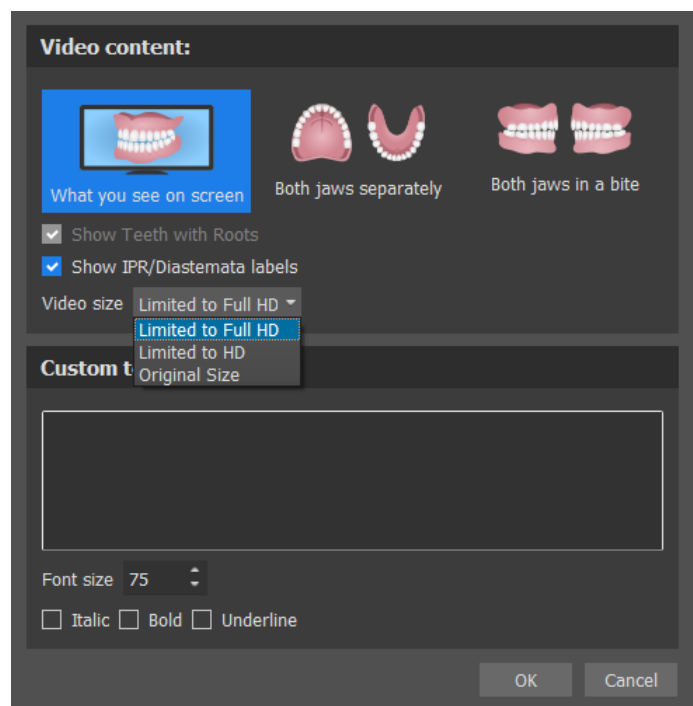


25.10 Orthodontic Aligners

The method of creating aligners can be selected at the end of the step "Generating the intermediate positions" (described in the previous chapter). The upcoming steps will be based on this selection.

25.10.1 Add Attachments



Attachments are added automatically through the predefined "Automatic Attachment Placement" in Preferences. The attachment are placed according to the clinician's instructions.

To change the placement of the attachments, click on one of the three options: "Gingival Placement", "Mid Placement", or "Incisal Placement". This will only affect attachments that are positioned vertically.

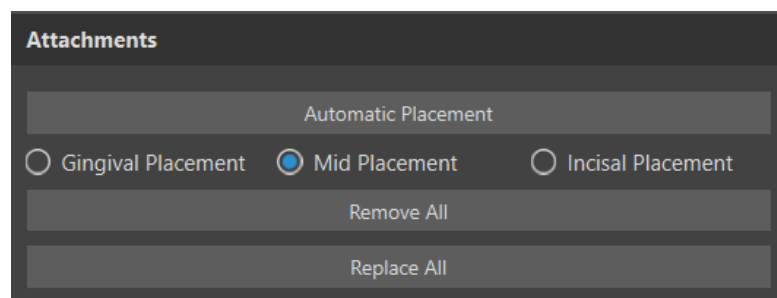


Attachments can be added manually after right clicking on the tooth in the 3D view and selecting among several sets, shapes, and sizes. The attachment position can be fine-tuned manually.

If the user decides to go to the previous step to change teeth movement, the attachments for the current placement can be kept. This also works for the brackets.

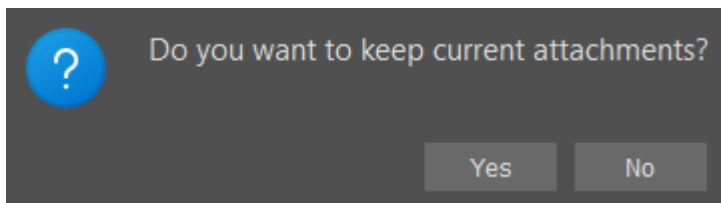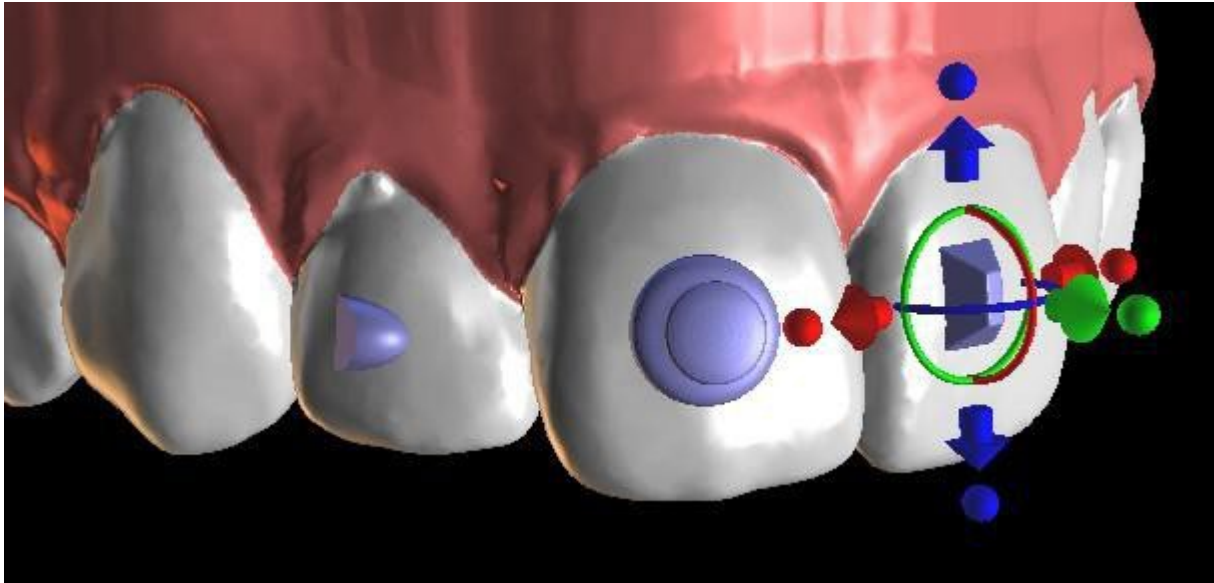

25.10.2 Design Printable Aligners


This step is intended to be used for drawing and editing the curve for the aligner fabrication. The aligner curve will be generated automatically based on the settings of the “Curve Shape” and “Trim Margin” parameters in Preferences. Both parameters can be adjusted as well and regenerate directly on the Orthodontics panel. These changes are saved in the project. If the parameters are changed in this process, it is necessary to regenerate the curve by clicking the “Reset Curve” button. The generation of aligners is a part of the transition to the STL model export step.

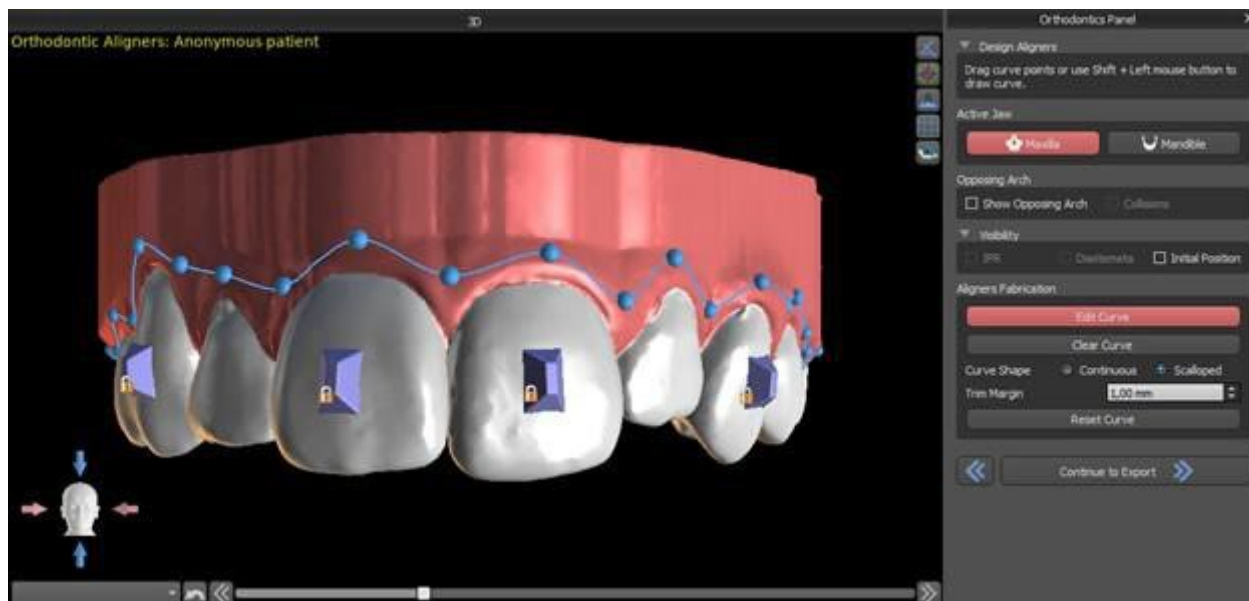


The production of aligners may be ordered at Blue Sky Bio. You must send the STL files containing the models with the 3D profile on them to identify the patient and stage number to Blue Sky Bio. Blue Sky Bio will make the aligners and ship them to you. If at any stage the patient no longer matches your model for that stage (making the aligner not fit) then edit the models and re-order the aligners. More detailed instructions for use are available in Footnotes 31.1.

25.10.3 Design Aligners Trim Curve

The Aligner Trim Curve will be (just like the Aligner Curve) generated automatically based on the settings of the parameters “Curve Shape” and “Trim Margin” in Preferences. Both parameters can be adjusted as well and regenerate directly on the Orthodontics panel. These changes are saved in the project. If the parameters are changed in this process, it is necessary to regenerate the curve by clicking the “Reset Aligner Trim Curve” button. To use the trimline information to automatically trim the aligners in the milling machines’ information in the future, it is possible to turn on “Show locator” and modify the trim curve accordingly. It will be possible to export the curve in the last step.

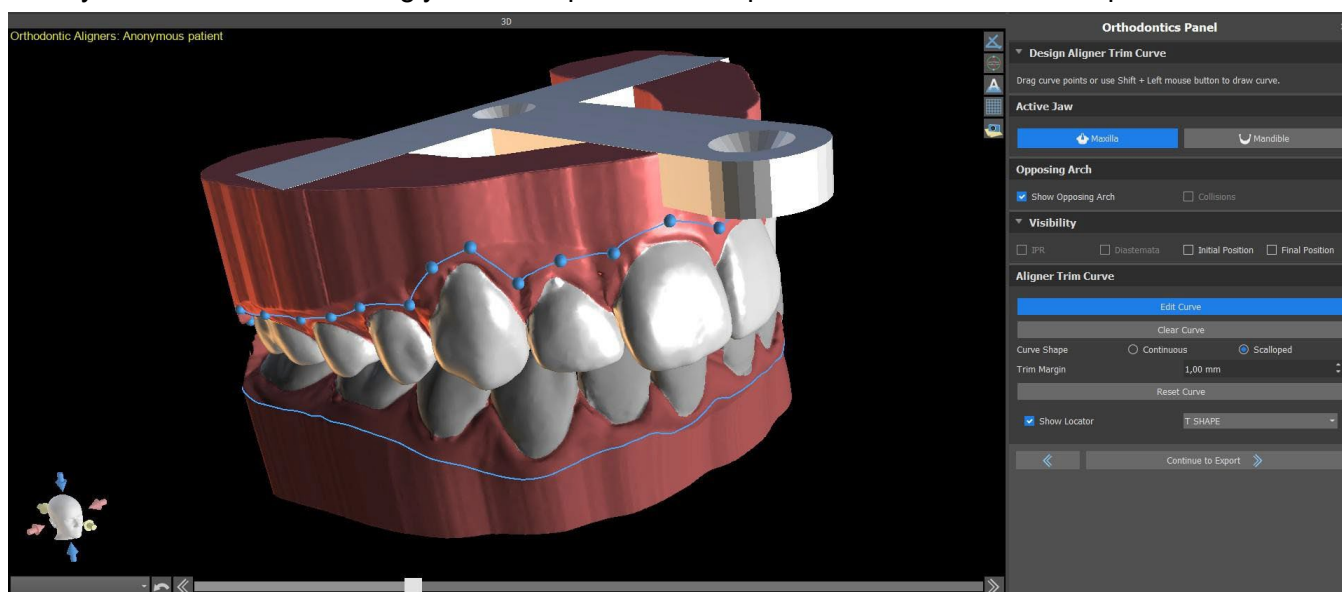

The type of the aligner locator model can be set in the Preferences -> Orthodontics -> Aligners section. The locator type can be changed in the Export step as well.

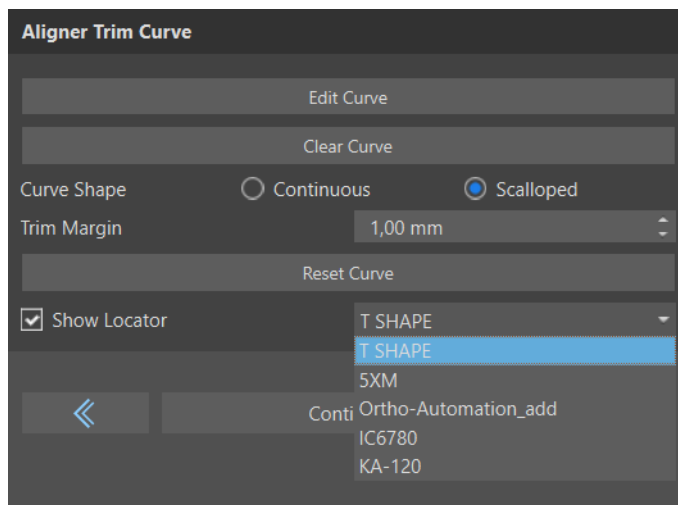

25.10.4 Send for Aligner Manufacturing

By selecting "Send for Aligner Manufacturing" (1) and clicking on "Send for Aligner Manufacturing" (2), you are able to order aligners for a case via LabPronto.



On the LabPronto page, select order information and attach files. Click "Order Now" to order your aligners.

Predictable TAD Placement with TAD Surgical Guides! Now Available!



BlueSkyBio.com

Your Partner In Implant & Digital Dentistry

Discover Now!



1. Order Details

2. Attach Files

Aligners

Aligners

SELECT SERVICE

Clear Aligner Fabrication (\$30 + \$20/ Aligner)

More information on selected service:

[Service Details](#)

ORDER INFORMATION

Number of Aligners

1

Patient Name or Identifier*

Select Existing Patient

OR

ADD NEW PATIENT

Export from BSB to STL *

Select Option

How Many Steps Should Be Included In Each Shipment (up to two years)? *

12

Expedited Turnaround (4 Business Day) ? *

Select Option

Additional information for Lab:

Include any relevant information or special considerations Above, please select the number of aligners for the case. The user that planned this case should be able to provide this information, or it can be checked in the Blue Sky Plan Software.

Select Shipping Option

USPS (\$ 0.00)

**Total Order :
\$ 50.00**

Placing an order via LabPronto indicates that you have read and agree to the [LabPronto Terms](#)

Shipping Address

Payment Method


Primary contact

Continue to Attach Data Files



Predictable TAD Placement with TAD Surgical Guides! Now Available!

Discover Now!


BlueSkyBio.com
Your Partner In Implant & Digital Dentistry

1. Order Details

2. Attach Files

Attach File(s)

File uploaded successfully. 100%

Remove

Attach Upper / Lower Arch

Previous

Select Shipping Option

USPS (\$ 0.00)

Total Order : \$ 50.00

Coupon Code

Apply

☐
IMPORTANT: Automatically submit order when file upload(s) complete.

ORDER NOW

Placing an order via LabPronto indicates that you have read and agree to the [LabPronto Terms](#)

Shipping Address

Payment Method

Primary contact

Note that this service is currently available only in the United States of America and you need to have filled the shipping address.


25.11 Indirect Bonding Tray - Align Brackets to Teeth

Blue Sky Plan also allows for controlled virtual bracket placement in the software and the fabrication of an indirect bonding tray to allow the physical brackets to be placed.


This planning type is used to add brackets to the initial position of the teeth. That is the reason why it doesn't include the Teeth Moving step and after Model Trimming goes straight to Add Brackets.

25.11.1 Add Brackets

Brackets are added directly on the teeth based on the predefined "Brackets Placement Offsets" in Preferences. The position of already added brackets can be modified by the 3D manipulation widget and the function "Snap to Tooth" by right clicking with the mouse.

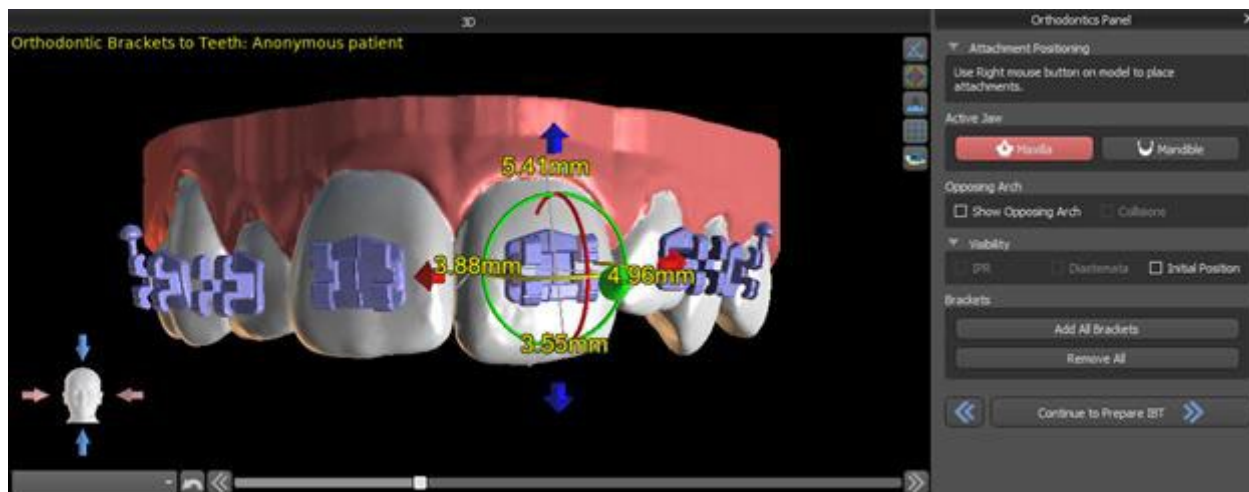



Brackets from several systems are available.

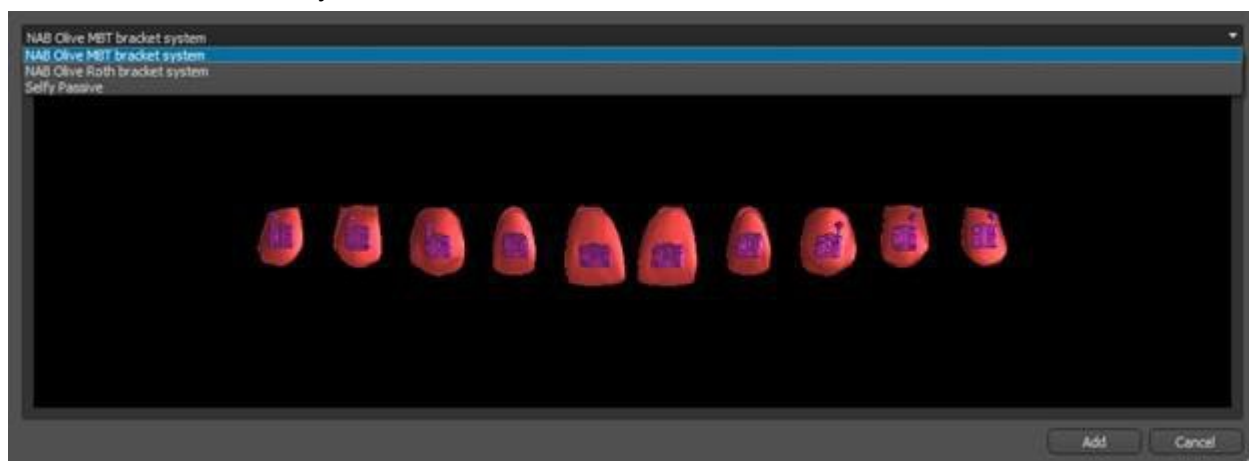

25.11.2 Prepare IBT

Once the brackets are placed, the next step will include drawing and editing the curve for indirect bonding tray fabrication. Its use allows the physical brackets to be placed accurately in real life.

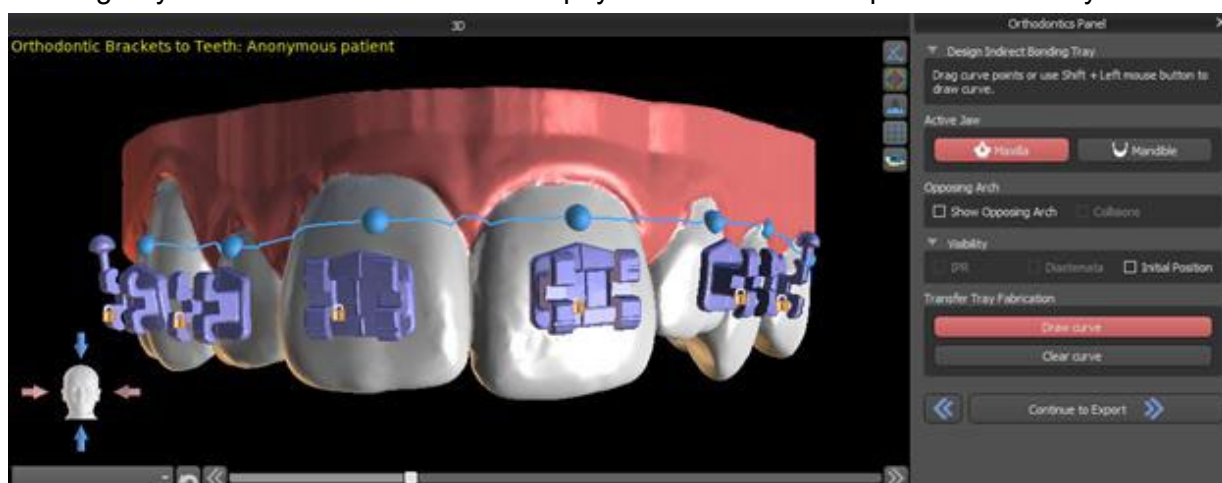


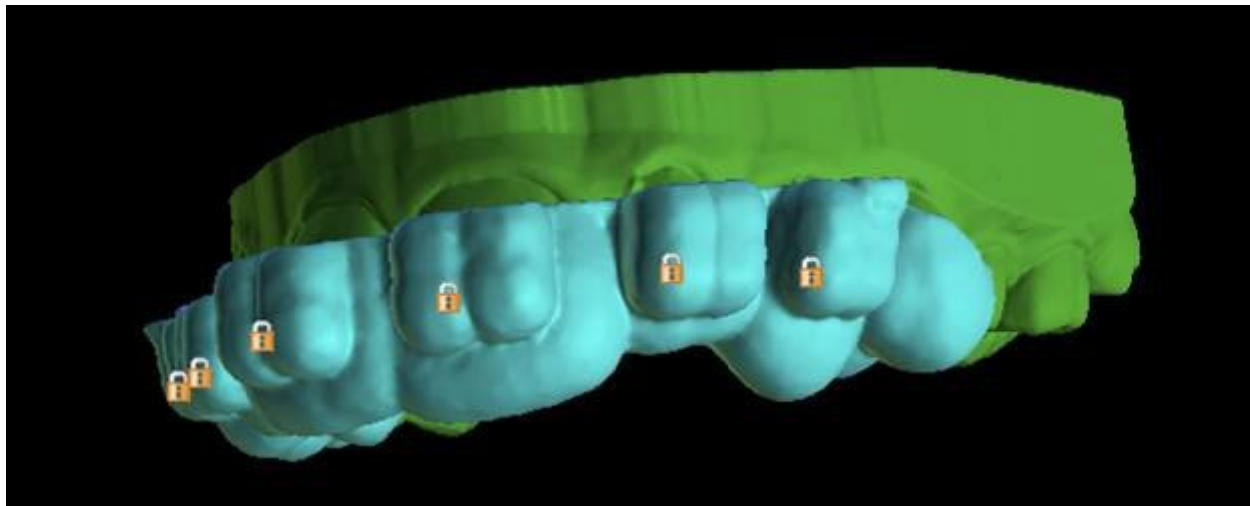

25.12 Indirect Bonding Tray - Align Brackets to Wire

25.12.1 Add Brackets

The “Align Brackets to Wire” planning type is used to add brackets to the final position of the teeth. Blue Sky Plan automatically places the brackets along the wire. The brackets fully respect the rectangular wire and do not adjust their position according to the teeth surface. Just as in the previous method, it is possible to choose from several bracket systems and the added bracket can be modified using the 3D manipulation widget and function Snap to Wire by right clicking with the mouse.

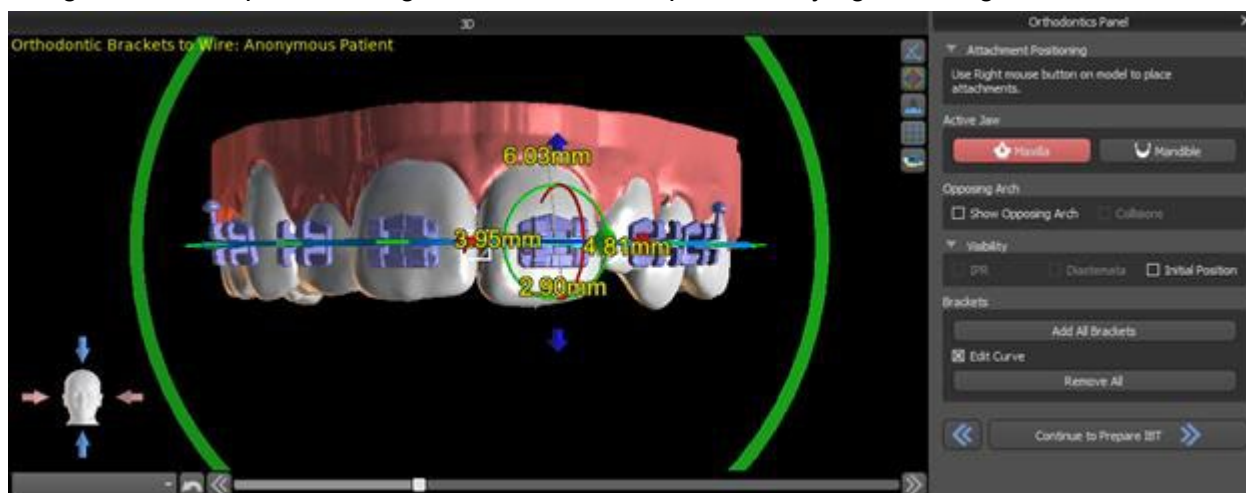


25.12.2 Prepare IBT


This steps includes drawing and editing the curve for indirect bonding tray fabrication similarly to the previous type of planning. The curve is drawn on the final position of teeth. The indirect bonding tray is generated for the first position model only. The default setting for IBT fabrication can be changed in Preferences.


25.13 Generating and Exporting STL Models

The final step, Export, is again common for all types of planning. The only difference is in what can be exported in this final step (OBJ or STL models or the parameters of the curve).

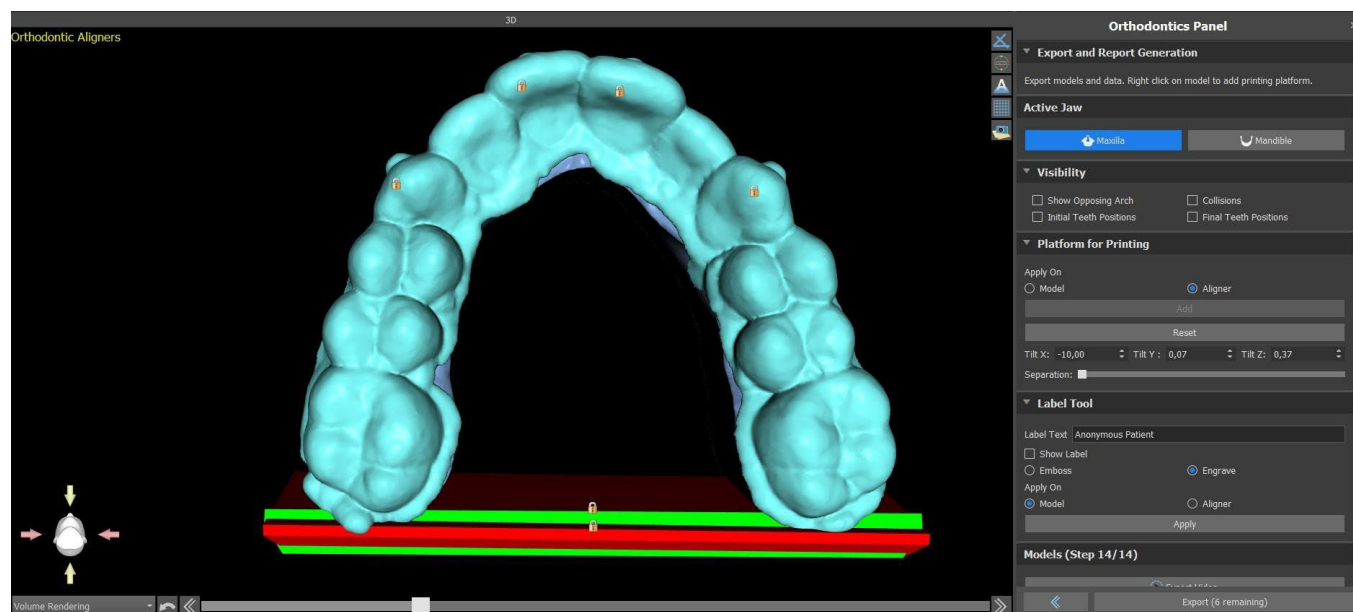


Before the export itself, it is necessary to check all of the intermediate positions of the generated models. After the check, it is possible to export models in different coordinates; regular and separated models can be sent separately. A model's export coordinates can be selected in the Orthodontics Panel.

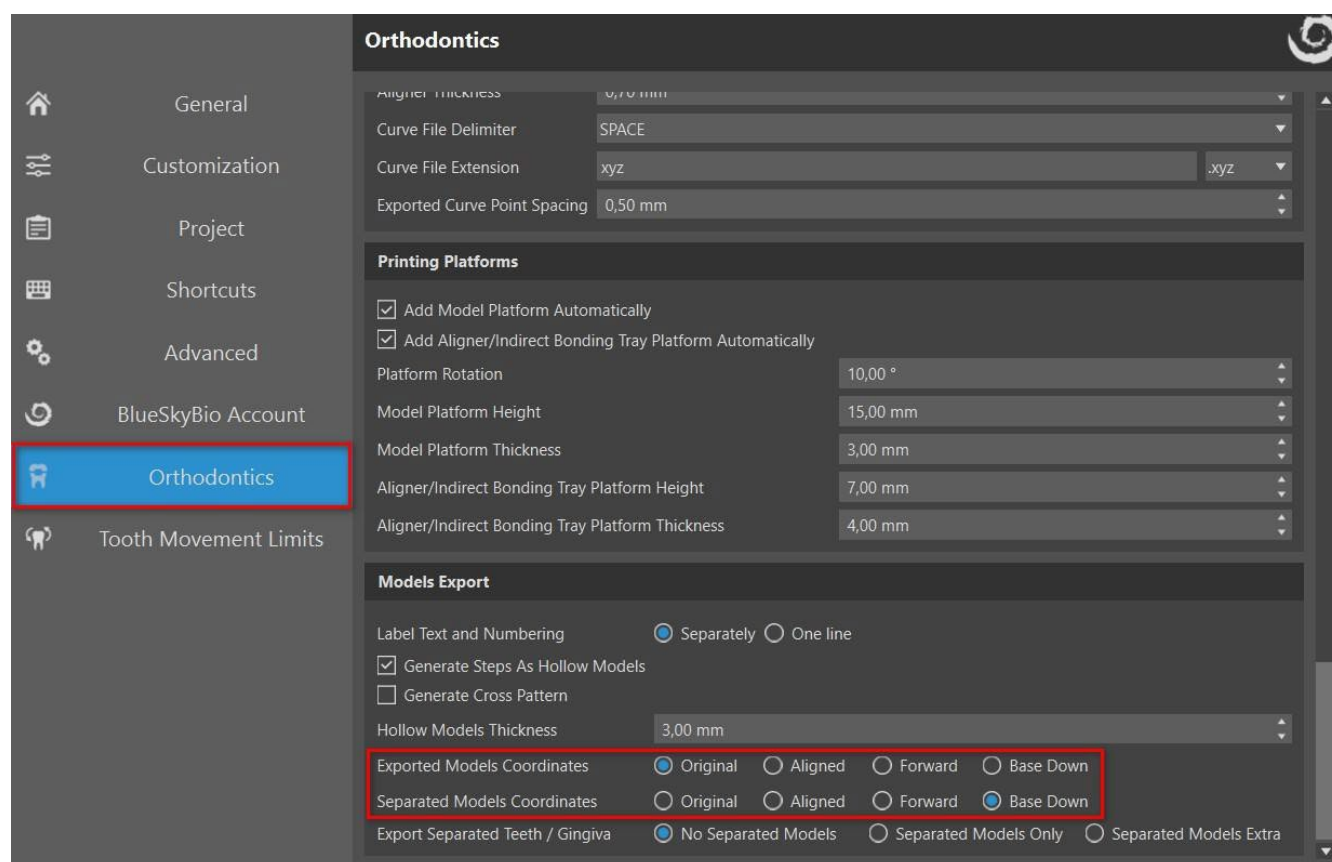


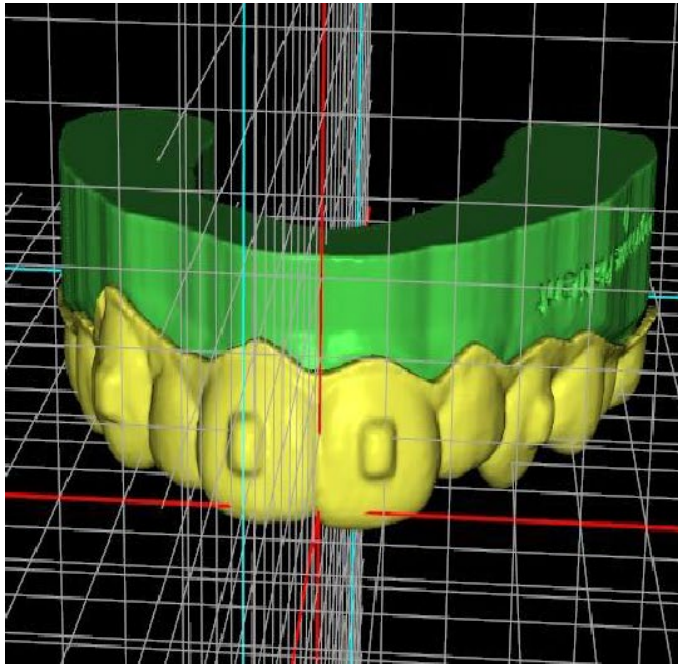


coordinates: forward

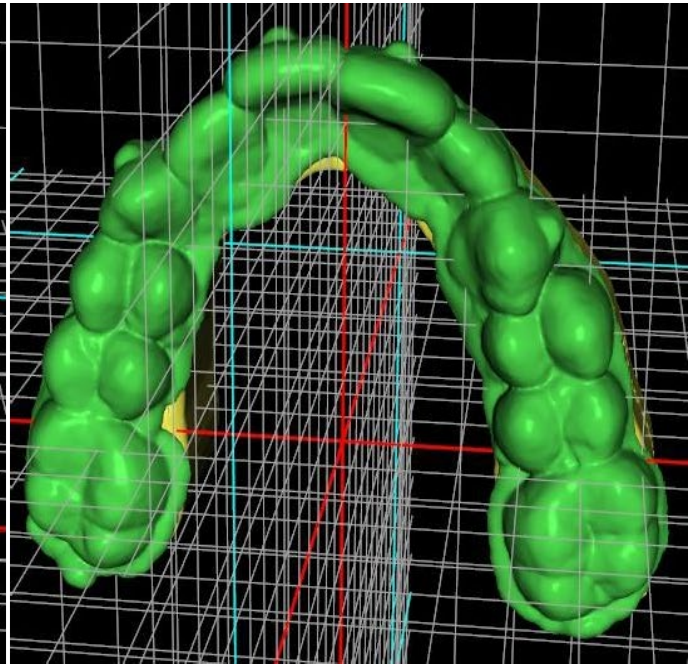

coordinates: original

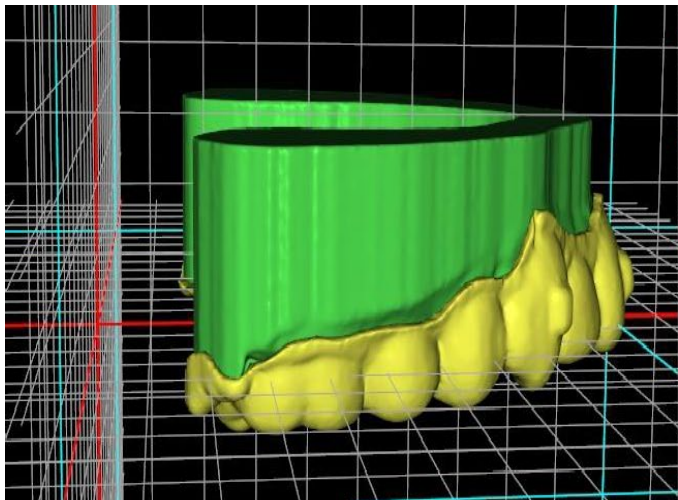


coordinates: aligned

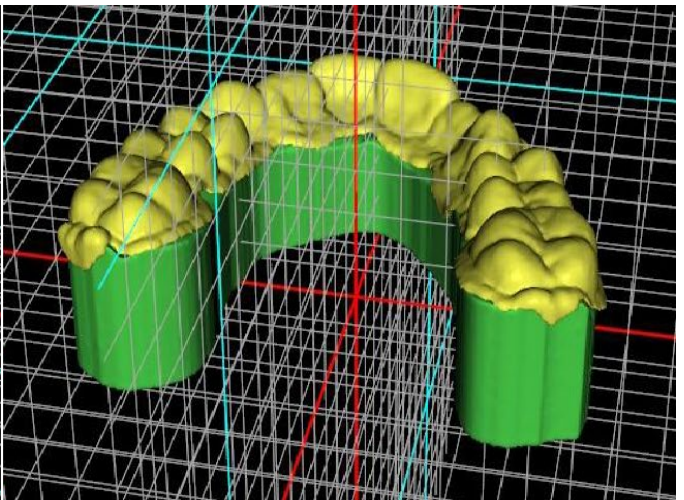

coordinates: base down

Platforms are automatically added to the generated models, allowing for the horizontal 3D printing of a larger number of models. They are added separately for models and for aligners, and for additional work with them, it is necessary to have the correct option checked on the Orthodontics Panel. The default settings for platforms (size and rotation) are found in Preferences, Orthodontics tab. These parameters can be further manually adjusted for a specific case. Using the “Separate” slider, it is possible to create two smaller parts at the end of the jaw, which are sufficient for printing. You can add the platform to a model just in one step – it will be generated in the remaining steps.

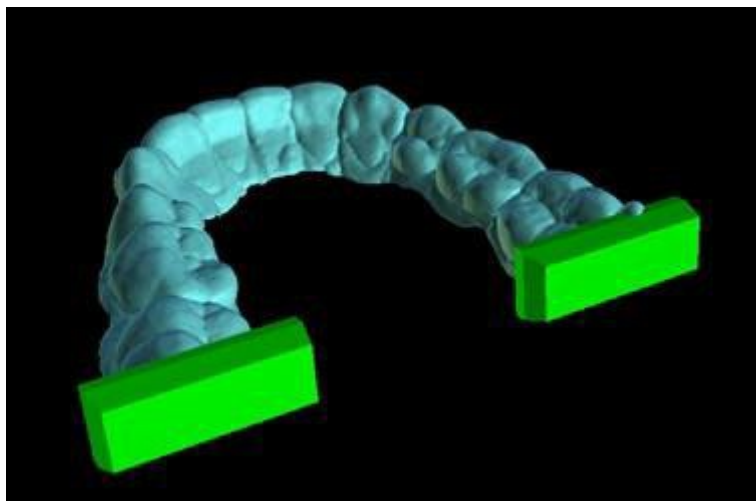

The Orthodontics panel also allows you to supplement the generated intermediate models and aligners or IBTs with text with a 3D profile emboss or engrave (typically the patient's name and the step number). This is done with the "Label Tool" which works like the one used to emboss text on surgical guides. The text needs to be attached to the first model only and it is automatically propagated onto the other intermediate models. It is necessary to choose either the emboss or engrave method in the "Label tool" and if it will be applied on the generated models or aligners (IBT). It is also possible to choose if the label text and number of the model will be generated on the same row.




If the generated STL model needs to be optimised for print, hollow with (or without) a cross pattern, the user must check the *"Generate Steps as Hollow Models"* and check the *"Generate Cross Pattern"* in Preferences, in the Orthodontics tab, before exporting.



Besides the models, the parameters of the Aligner Trim Curve can be exported as well for the Aligners planning type. The generated curves for intermediate models can be modified individually with the visualised locator to be optimally prepared for use on the milling machines. The parameters "Curve File Delimiter" and "Curve File Extension" can be modified in Preferences.

In this step, once again, it is possible to export a video. If you added attachments in the previous steps, it is now possible to export video with the available attachments.

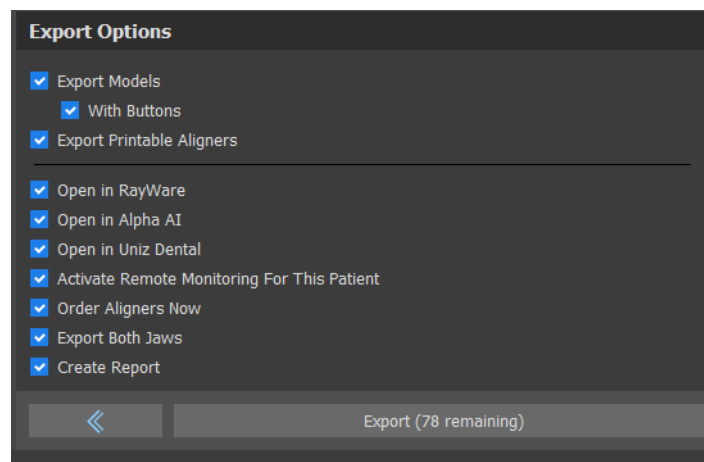

The generated models can be exported from the software in the form of STL models by pressing the "Export" button. By selecting the "Open in Alpha AI" or "Open in RayWare" or "Open in Uniz Dental", models can be exported to the respective external 3D printing software.

Note: To export the STL files, you must have a valid STL export license! To see the option "Open in Alpha AI" or "Open in RayWare" or "Open in Uniz Dental" you must have those software programs installed on your computer.


There is an option in the export step to export files to BioBigBox, where it is assigned as a new Patient and Treatment. This information will be sent to the BlueSkyMonitoring service when the "Activate Remote Monitoring For This Patient" is checked.

LabPronto (represented by the "Order Aligners Now" button) is a solution for sending the user's order directly to the most capable provider to get the user's job done properly and quickly. LabPronto analyzes the user's order, lab availability and the price the user will pay. Lab Pronto will then prioritize the user's order to the specific lab that will create the order. Lab Pronto offers various ordering options, allowing the user to order exactly and only what he needs. He can outsource the entire case, manufacturing or planning.

"Export Both Jaws" exports both the maxilla and mandible into the selected folder.

For immediate and automatic STL license activation, purchase exports directly in the Blue Sky Plan Software (File > Purchase STL Exports). You may also purchase STL case exports from <https://blueskybio.com/store/stl-case-export>.

Finally, it is also possible to generate the so-called Tooth Movement Report with the angles and displacements of the teeth moved from the original position to the final position. In addition, the report shows the movements from each intermediate position to the next stage.

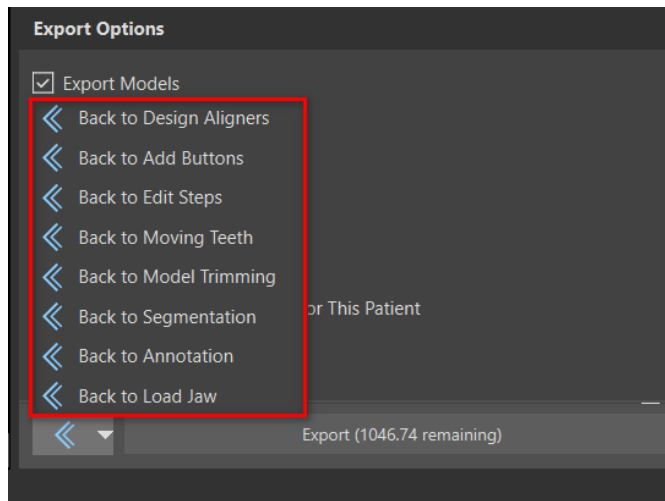


By clicking on the dropdown menu next to the "Back" button, it is possible to "fast jump" several steps back.


25.14 Show Dimensions

"Show Dimensions" in the orthodontics module is a measuring tool. It shows bounding boxes that can be used to analyze objects. This tool can be used in all steps during orthodontics.


To use this function, right-click on an object in the 3D scene and select "Show Dimensions" from the context menu. The dimensions are hidden by clicking "Hide Dimensions" in the same context menu. In Tools > Show Dimensions, it is possible to activate multiselection without having to click on "Show Dimensions". Multiselection is turned off by clicking outside the object.

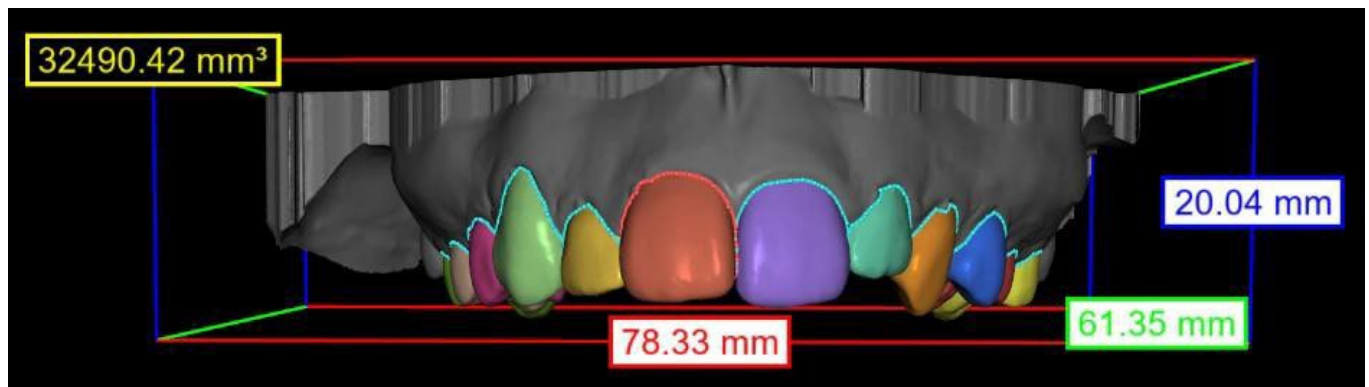

Dimensions shown in the "Segmentation" step

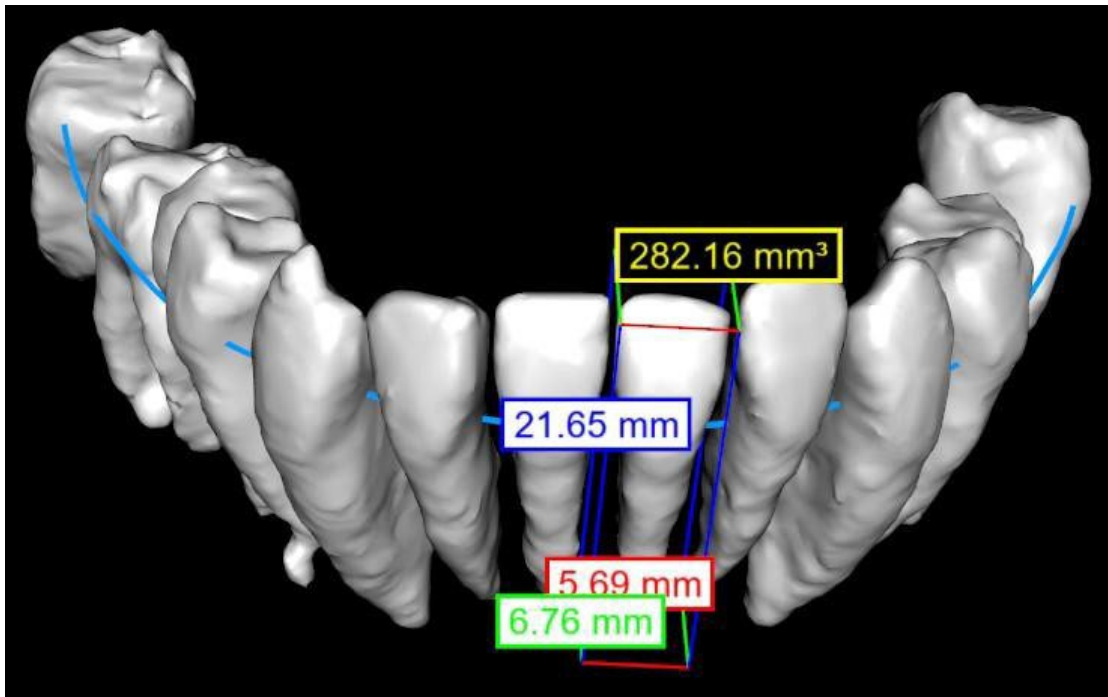

Dimensions shown on the tooth CT in the "Tooth Alignment" step

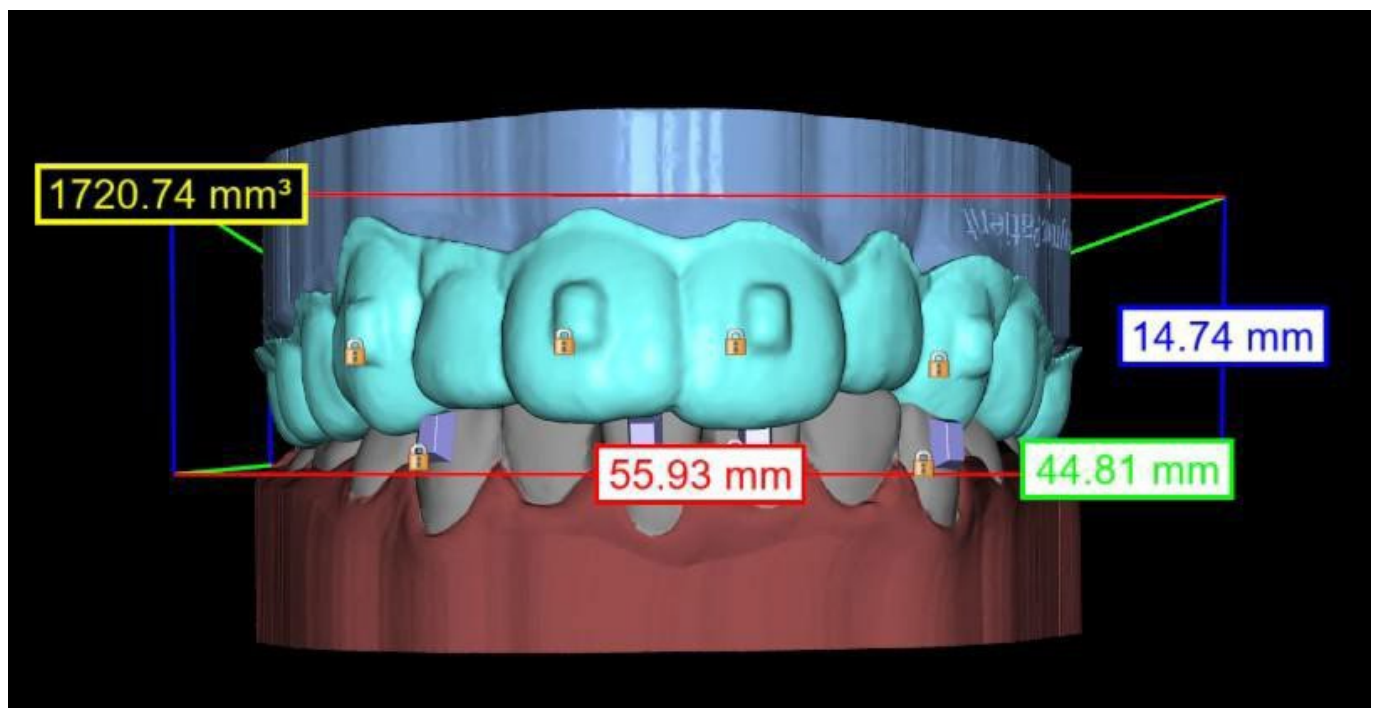

Dimensions shown on the aligner in the "Export" step

25.15 Quick Instructions for Using the Computerised Orthodontic Bracket System

More detailed instructions for use are available in Footnotes 31.2

The user during the export step should select the check boxes for "Export models" and "With Brackets" and generate a model, including the bracket location indicators, in the form of an STL model.

25.15.1 Printing the Model

The user should use a relevant 3D printer that prints at a resolution of at least 150 microns on the x/y axis and 100microns on the z axis, to print the exported model in grey or tan model material. The printer control software, printer and model material should be labelled for use together.

Select the exported STL model file in the printer control software for printing.

25.15.2 Adding Brackets

Use cement to firmly secure the desired brackets to the bracket location markers on the 3D printed model.

25.15.3 Creating the Indirect Bonding Tray by Pressure Forming

The user creates the Indirect Bonding Tray by pressure thermoforming over the 3D printed model with brackets attached to an outer carrying shell from a legally marketed orthodontic resin such as Essix Ace. The user should then trim the shell to the middle of the brackets for easy release.

The brackets should then be placed into wells created in the shell. Skip to section 1.9 using the indirect bonding tray to place the brackets.

25.15.4 Creating the Indirect Bonding Tray by Printing (this method should only utilised in an FDA registered establishment)

The user should click "Create the indirect bonding tray".

The user should check the print screen image of the tray for accuracy, and when it is correct export/print the indirect bonding tray by choosing File>Export Data and selecting the indirect bonding tray to be exported as an STL.

The user should use a relevant 3D printer that prints at a resolution of at least 150 microns on the x/y axis and 100microns on the z axis, to print the exported model in grey or tan model material. The printer control software, printer and model material should be labelled for use together. The material should be a light cured resin which is not recycled or recyclable.

Select the indirect bonding tray STL file in the printer control software to be printed in the legally marketed impression tray resin material called NextDent Ortho IBT.

Brackets should then be placed into wells created in the shell of the printed indirect bonding tray.

25.15.5 Using the Indirect Bonding Tray to Place the Brackets

The user should place the brackets onto the teeth by placing conventional cement onto the bracket pads following the manufacturer's instructions and seat the tray with the brackets into the patient's mouth. Excess cement should be removed, and a light cure should be used via the Indirect Bonding Tray.

The outer carrying tray should be removed and the inner liner tray should be peeled off. Excess cured cement should be removed.

25.15.6 Use of the Blue Sky Plan Calibration Verification STL file

The calibration of the printer should be done according to the printer manufacturer's instructions and in accordance with the quality system of the FDA registered establishment printing the trays. The frequency of calibration should be determined by the printer manufacturer's printer instructions and the

establishment's quality system. Blue Sky Bio provides a calibration verification file to confirm adequate calibration. Calibration verification should be done whenever the printer is calibrated and any time the printing material is changed.

Download the STL calibration verification file from the Blue Sky Bio website


- Import the STL calibration verification file into the printer control software
- Print the STL file
- Measure each cube in all 3 dimensions and the distances between adjacent cubes with a calliper.

The size of each cube is printed on the cube.

The distance between cubes is 3mm.

The measurements of the cubes and open spaces should be within the certified accuracy of the printer.

If the measurements fall outside that range, recalibrate the printer according to the printer manufacturer's instructions.

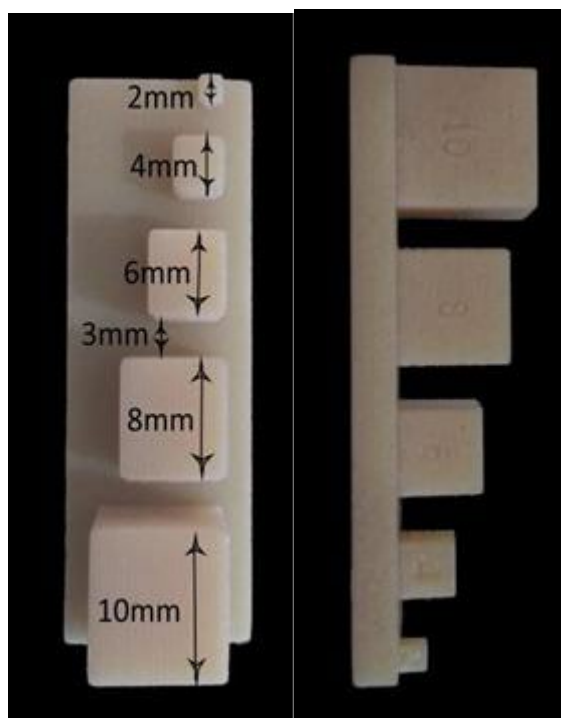

Precautions: Do not touch the bonding surfaces with your bare fingers as skin oils may diminish the adhesion. Do not allow adhesive to get under the bracket in the guide slot. The position of each bracket should be verified by the clinician and if necessary malpositioned brackets should be removed and repositioned. Ceramic brackets may abrade the opposing arch. Wait to apply the ceramic brackets until the bite has opened. If necessary, use metal brackets before the placement of ceramic brackets. Provide the patient with instructions on how to conduct thorough oral hygiene, as tartar or food debris may impair the function of the bracket. Do not apply excessive force when opening the bracket as the bracket may bend or detach from the tooth. To prevent splintering when removing the brackets, use the appropriate instrument. If patient experiences an allergic reaction, immediately remove the bracket and advise patient to seek medical care.


26 Denture module


Blue Sky Plan permits the user to model and export dentures. The functions are available in the “Denture” panel once the user switches to the “Denture” module.

26.1 Loading the patient’s STL model

New planning starts by loading the STL jaw impression model using the main module selection “Dentures > Import Models”. To start planning the denture, the user needs to switch to the “Denture” module from “Model Master”. For more information, see chapter 3 “Blue Sky Launcher”.


26.2 Adding virtual teeth

Virtual teeth can be added by clicking on the  icon in the toolbar. A dialog for selecting crown(s) then opens. Clicking on a tooth selects it, and a text description of the crown appears below the preview. Multiple teeth can be selected by holding down the Shift or Ctrl key. Another possibility is clicking on Select All Maxillary Teeth/Select All Mandibular Teeth or Select All Teeth. It is also possible to select from three sizes. Teeth are added as teeth chain. We confirm the selection by clicking the “OK” button; the “Cancel” button closes the dialog without making a selection.

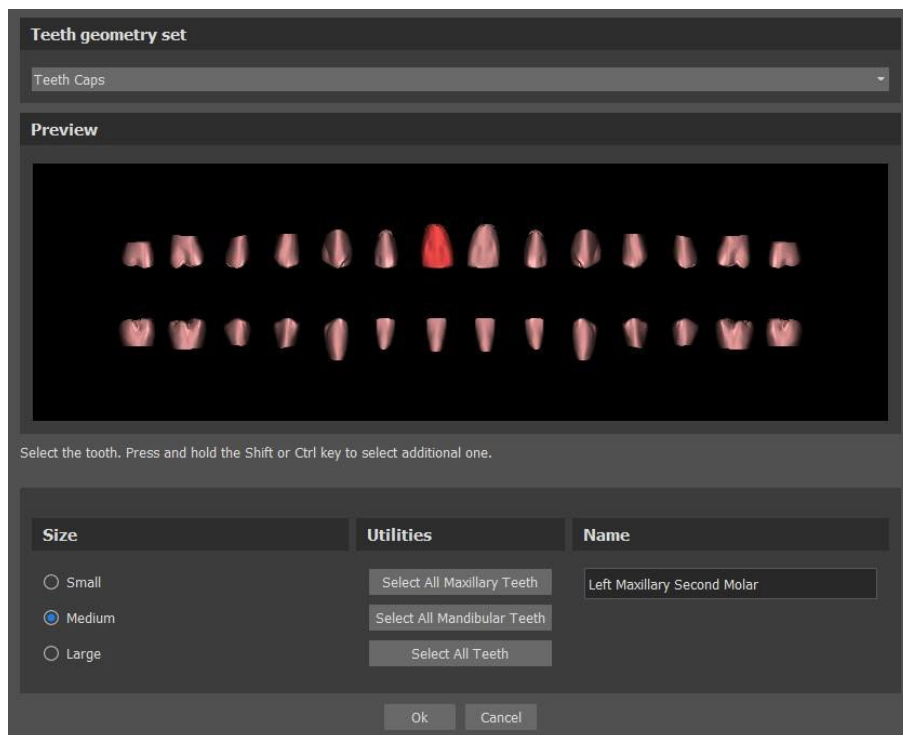


Virtual teeth are regular teeth sets, available in the Surgical Guide, Denture, and Crown and Bridge, they are modifiable and exportable.

The Denture Panel is used for denture modelling. After clicking on the “Denture Panel” tab, the below menu will appear with buttons for working with the teeth chain.

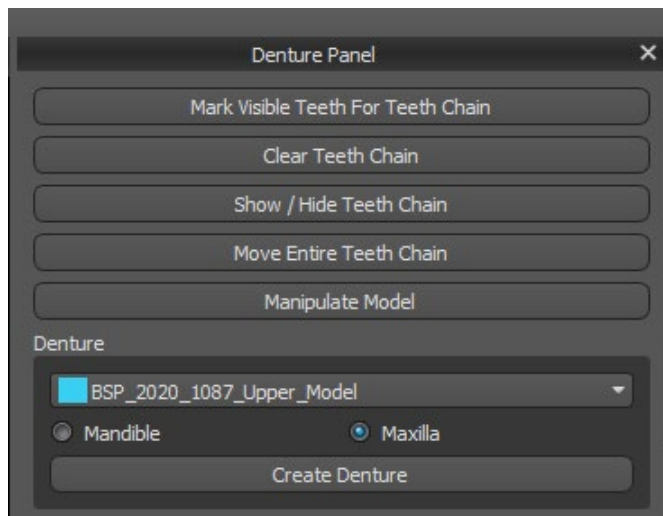

26.3 Mark visible teeth for teeth chain


Clicking on “Mark Visible Teeth for Teeth Chain” and using the control allows for the manipulation of the teeth chain. The teeth can also be moved individually with the arrows to different sides. Rotation is done by using the circles surrounding the teeth chain. Each axis is represented by one colour.

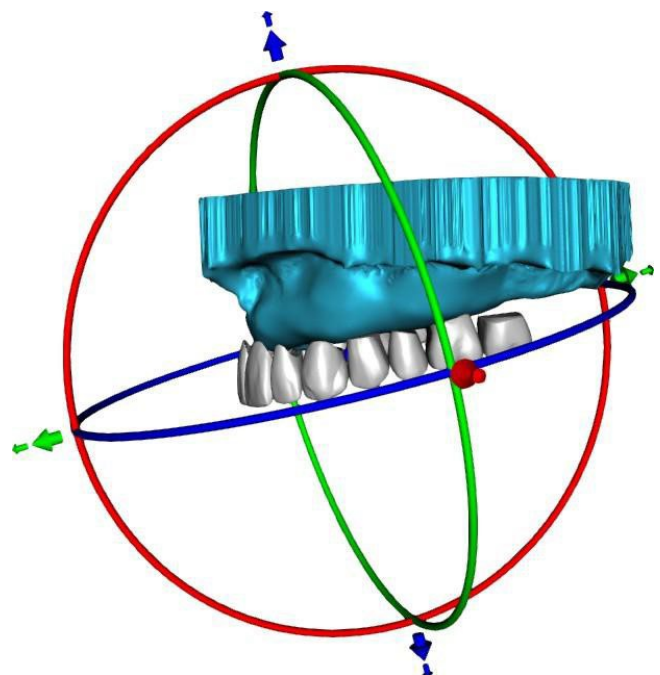

26.4 Clear Teeth Chain

Clears selected teeth chain.

26.5 Show / Hide teeth chain

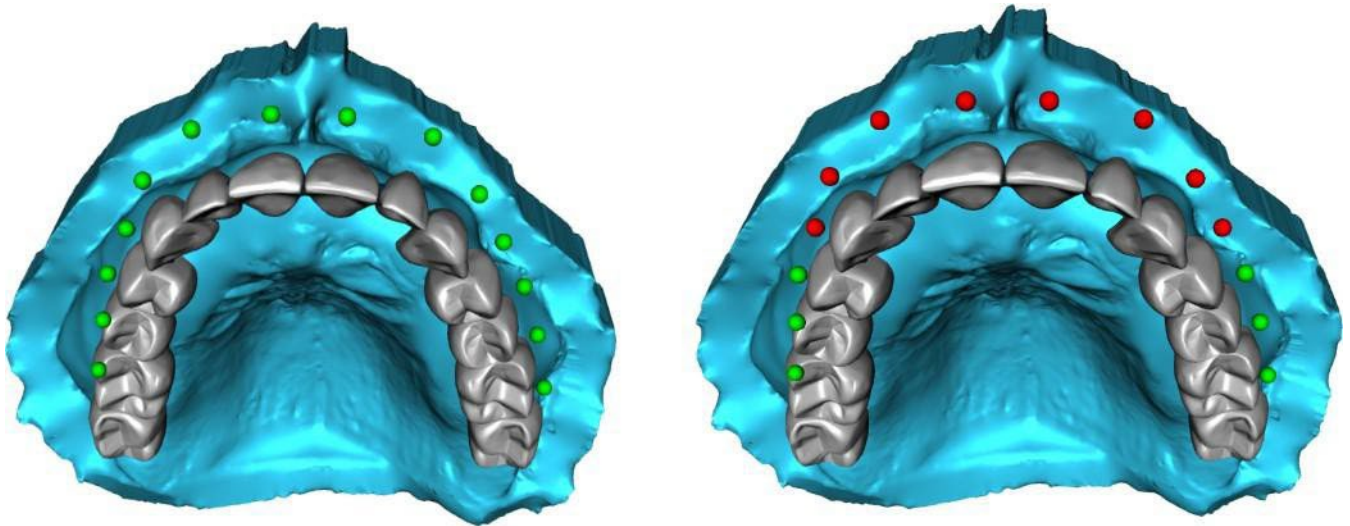


The button “Show / Hide Teeth Chain” is used for showing and hiding the green dot indicators in the teeth chain. There is one indicator per tooth.

26.6 Move Entire Teeth Chain


“Move Entire Teeth Chain” permits the user to hide teeth chains in the maxilla or the mandible by selecting the appropriate option.

The scale is used to enlarge or diminish the teeth.

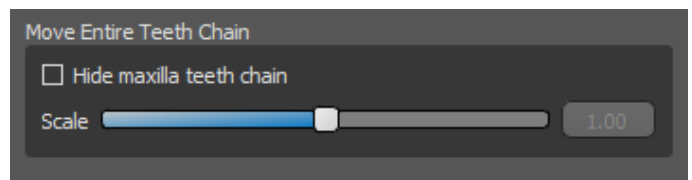


26.7 Manipulate model

The model can be unlocked by clicking the checkbox and the centre of rotation can be changed.

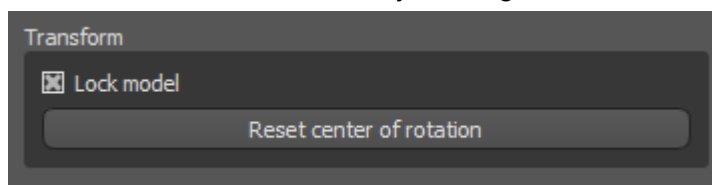


26.8 Create denture

A denture can be created in the menu in the right panel. This requires selecting a file, the correct jaw orientation, and confirming by clicking “Create Denture”.

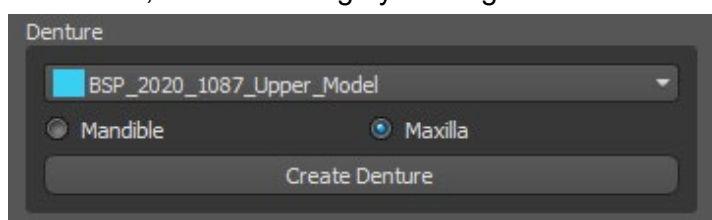

26.9 Step 1 – Undercuts and insert direction

Step 1 is used to define the initial parameters for the denture.

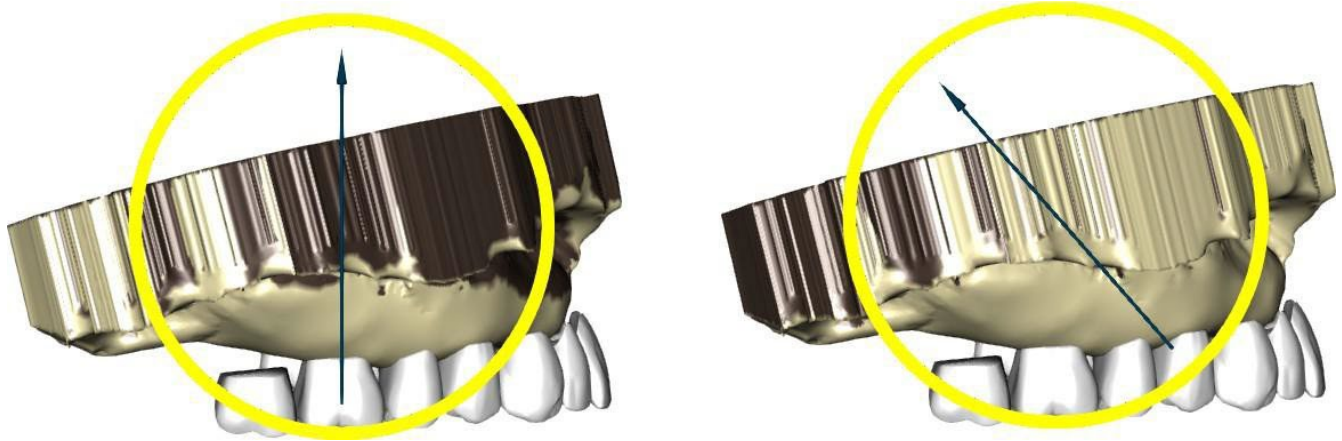


The sliders can be used for setting the values for offset and undercut removal.


The insert direction should be adjusted using the rotation ring so that the undercuts are minimised.

“Set insert direction from view” is used to set the insert direction according to the current viewing angle setting.

“Reset insert direction” will restore the original status of the arrow and the model.

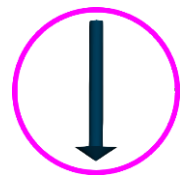


“Define posterior palatal seal” permits the modelling of this structure. The Palatal Seal is created by left clicking with the mouse on the two required points with Shift key pressed. Adjustments can be done by drawing with the mouse.

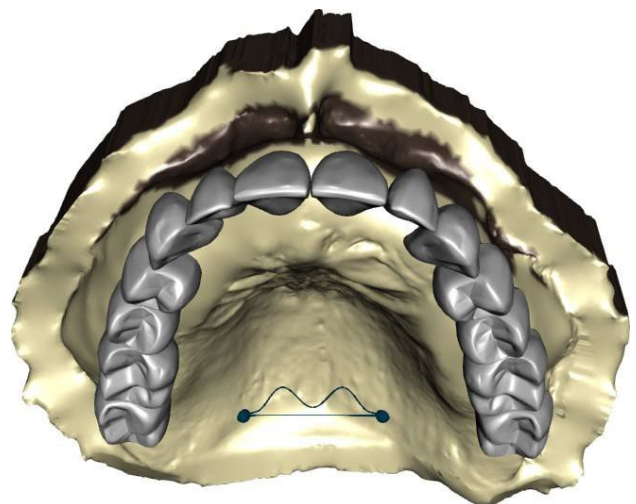

The “Back” and “Next” buttons are used for moving one step back or forwards.

26.10 Step 2 – Select denture base

Step 2 defines a contour around the teeth chain to define the borders of the denture. If a contour is not defined, user is not able to get to another step.

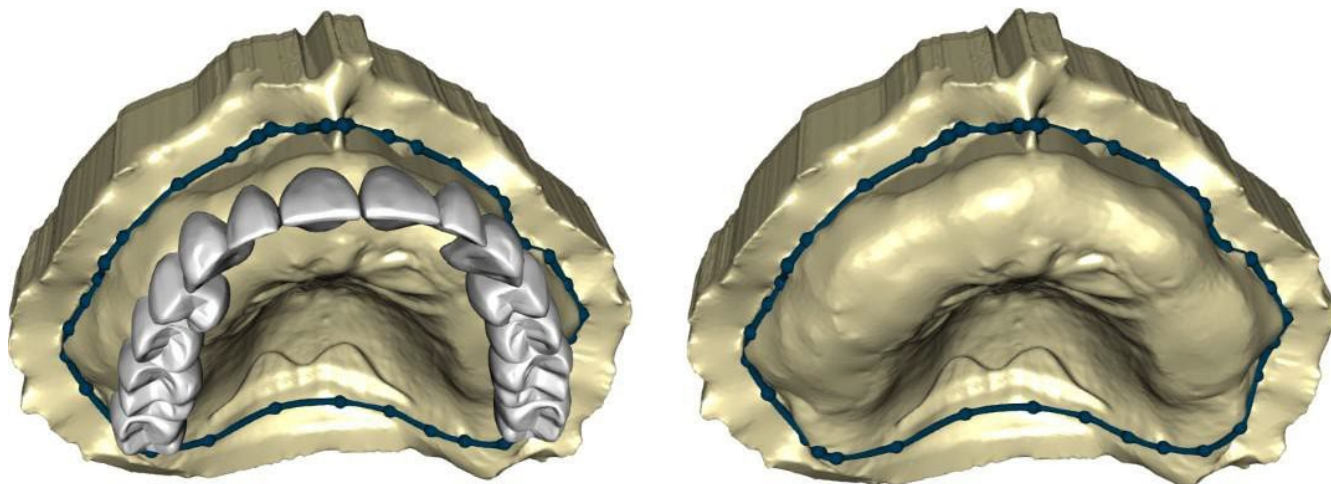

The offset from the gingiva and thickness of the denture can be set by the sliders.


Gingiva extension allows the user to extend the gingiva with the slider. The thickness and smoothness from the gingiva can be set with the sliders.

Pressing the Shift key and double clicking on the model around the teeth will add new points and create a blue contour around the area representing the denture base:

The contour can be deleted by selecting “Clear contour”.


26.11 Step 3 – Shape gingiva

Step 3 is used for adjusting the created gingiva.

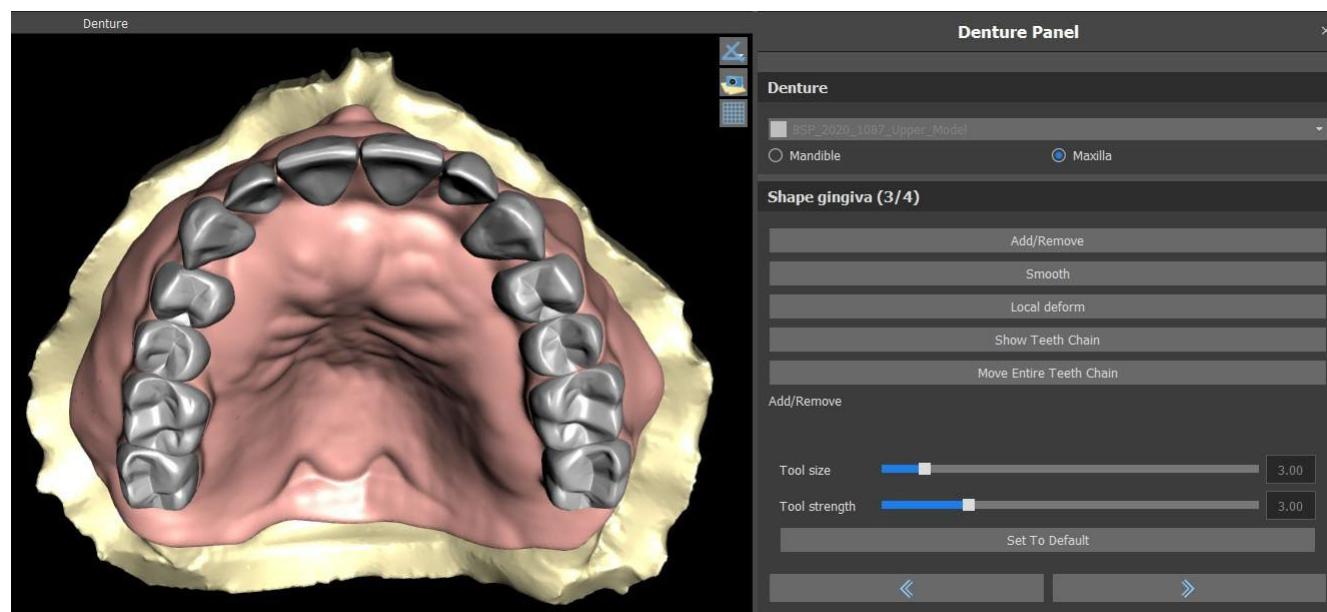

The following adjustments can be made within this step


- Add/remove – addition/removal of gingiva material
- Smooth – gingiva surface smoothing
- Local deform – creation of ups and downs
- Show Teeth Chain – display of the green dot indicators
- Move Entire Teeth Chain – manipulation with the teeth chain

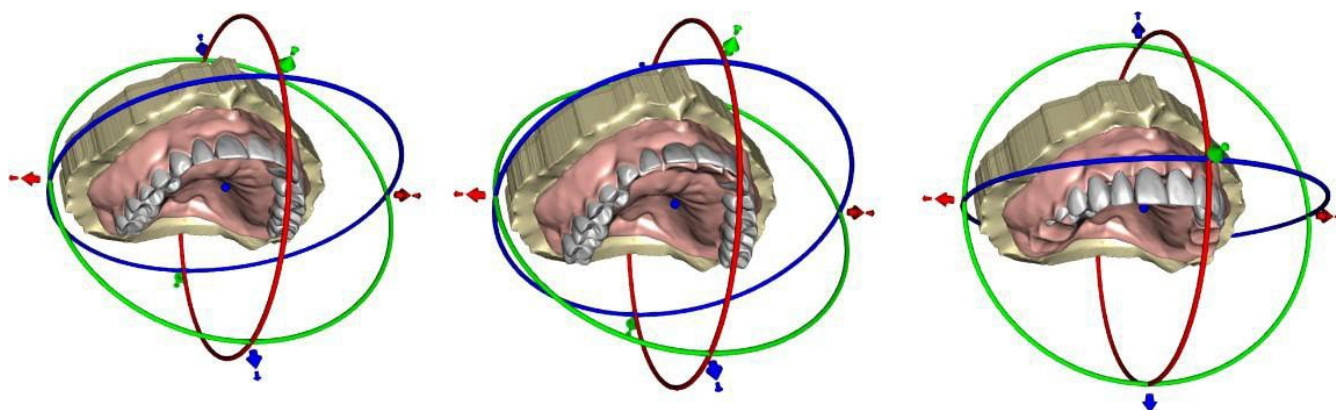

The “Tool size” slider is used for tool size selection, and “Tool strength” defines the strength of the tool. The original values can be restored by selecting the “Set to Default” button.

26.12 Step 4 – Denture

Step 4 is used for finalizing the denture creation. During this step, virtual teeth penetrating through the denture base are reduced while maintaining the minimum denture thickness.

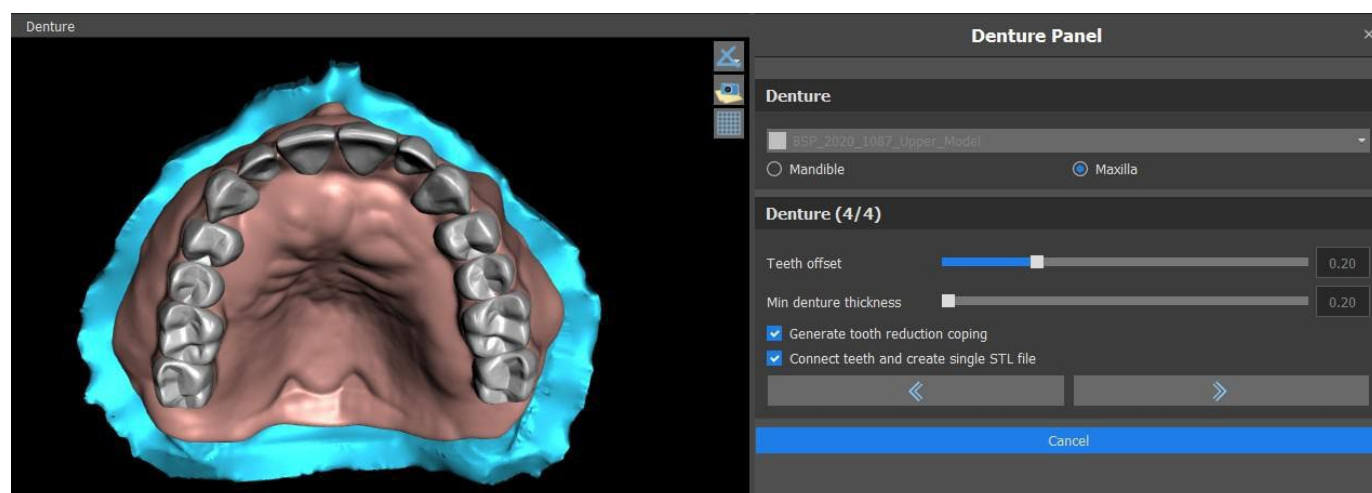

Sliders are used for selecting Teeth offset and Min denture thickness.

Tooth reduction coping generates a denture coping. Connect Teeth as single STL file makes the work easier for the export.

The model design is completed by selecting the “Next” button.

26.13 Export

The user can export the denture created in this step. Quality of the export can be set as well as swap face orientation. Export separated files exports the selected objects to a separated file or one single file created by the user. The “Create Surface” button allows the user to unite the selected models in order to create one model. The user won’t be charged again when additional surfaces are exported from the case.

Export Quality

☐ Standard ☐ High ☒ Very High

Export File Format

STL

STL Surfaces

Name	Color	Selected
BSP_2020_1087_Lower_Model	Green	<input type="checkbox"/>
BSP_2020_1087_Upper_Model	Blue	<input checked="" type="checkbox"/>
Maxillary_Undercut	Yellow	<input type="checkbox"/>

Virtual Teeth

Tooth Name	CAM	Edited	Selected
Right Maxillary Central Incisor	N/A	No	<input type="checkbox"/>
Left Maxillary Second Molar	N/A	No	<input type="checkbox"/>
Left Maxillary First Molar	N/A	No	<input type="checkbox"/>

☐ Show only teeth with CAM file support ☐ Swap face orientation ☒ Export separated files to a folder

Cost: 1

Create Surface Export (119 remaining) Cancel

26.14 Teeth Surfaces panel

Teeth Surfaces lists the imported patient’s STL models, placed virtual teeth and their reduced versions, and the newly generated denture base:

Lock	Name	Closeness	Visible	Color
<input type="checkbox"/>	BSP_2020_1087_Lower_Model	<input type="checkbox"/>	<input type="checkbox"/>	Green
<input type="checkbox"/>	BSP_2020_1087_Upper_Model	<input type="checkbox"/>	<input checked="" type="checkbox"/>	Blue
<input type="checkbox"/>	Right Maxillary Central Incisor	<input type="checkbox"/>	<input type="checkbox"/>	White
<input type="checkbox"/>	Left Maxillary Second Molar	<input type="checkbox"/>	<input type="checkbox"/>	White

The table includes the following attributes:

“Lock” – surface locking controlled by a checkbox

“Name” – surface name


“Closeness” – surface closeness controlled by a checkbox

“Visible” – surface visibility or hiding by a checkbox

“Colour” – surface colour


 - surface removal

Filters can be used to display either the maxilla or the mandible only or selected object types:

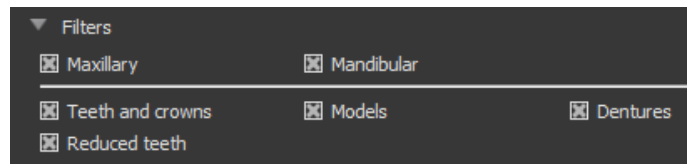


26.15 Opening saved project


A saved case is opened with “File > Open Project...” in the top left corner.

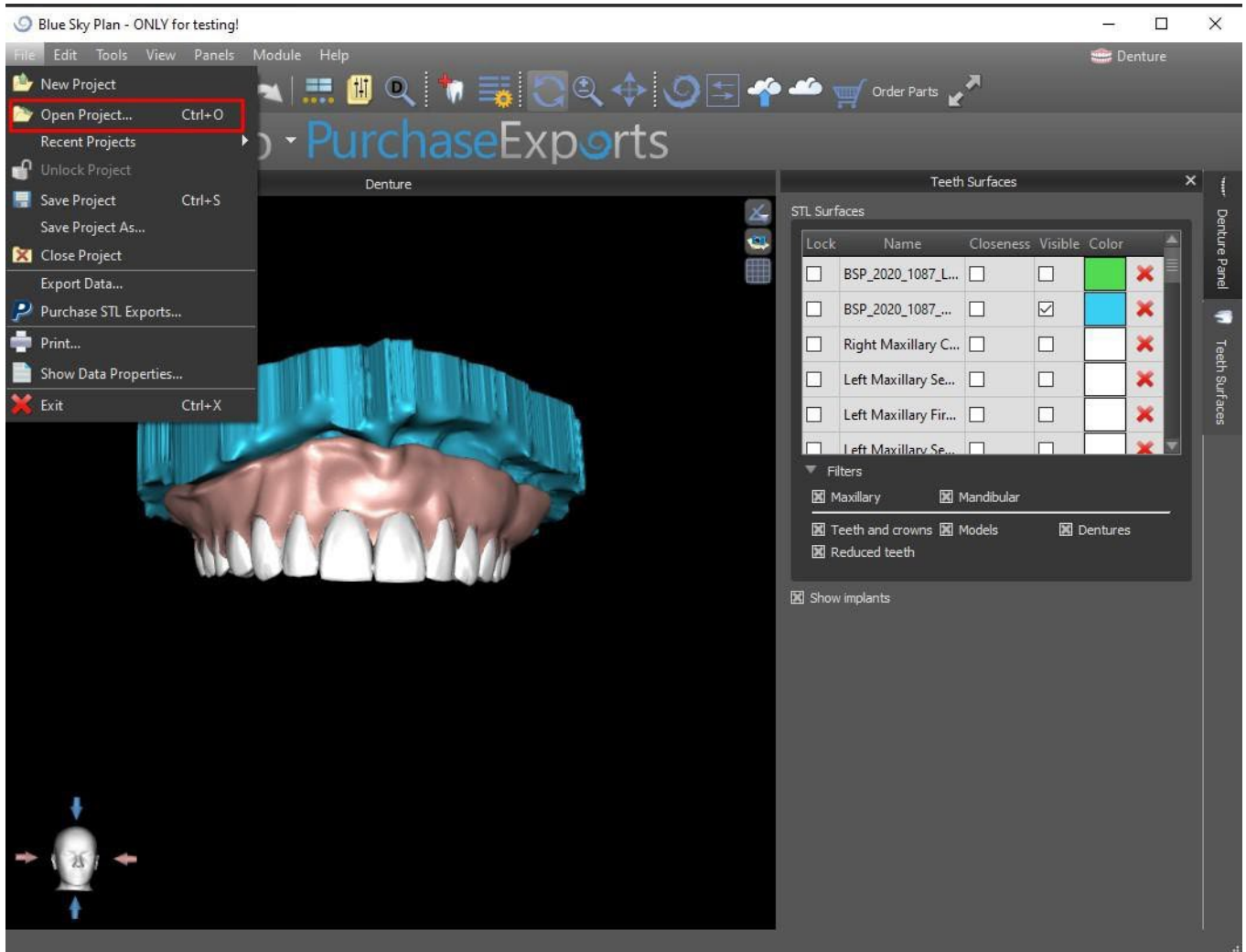


After clicking, the file to open is selected and the selection is confirmed with “OK”.

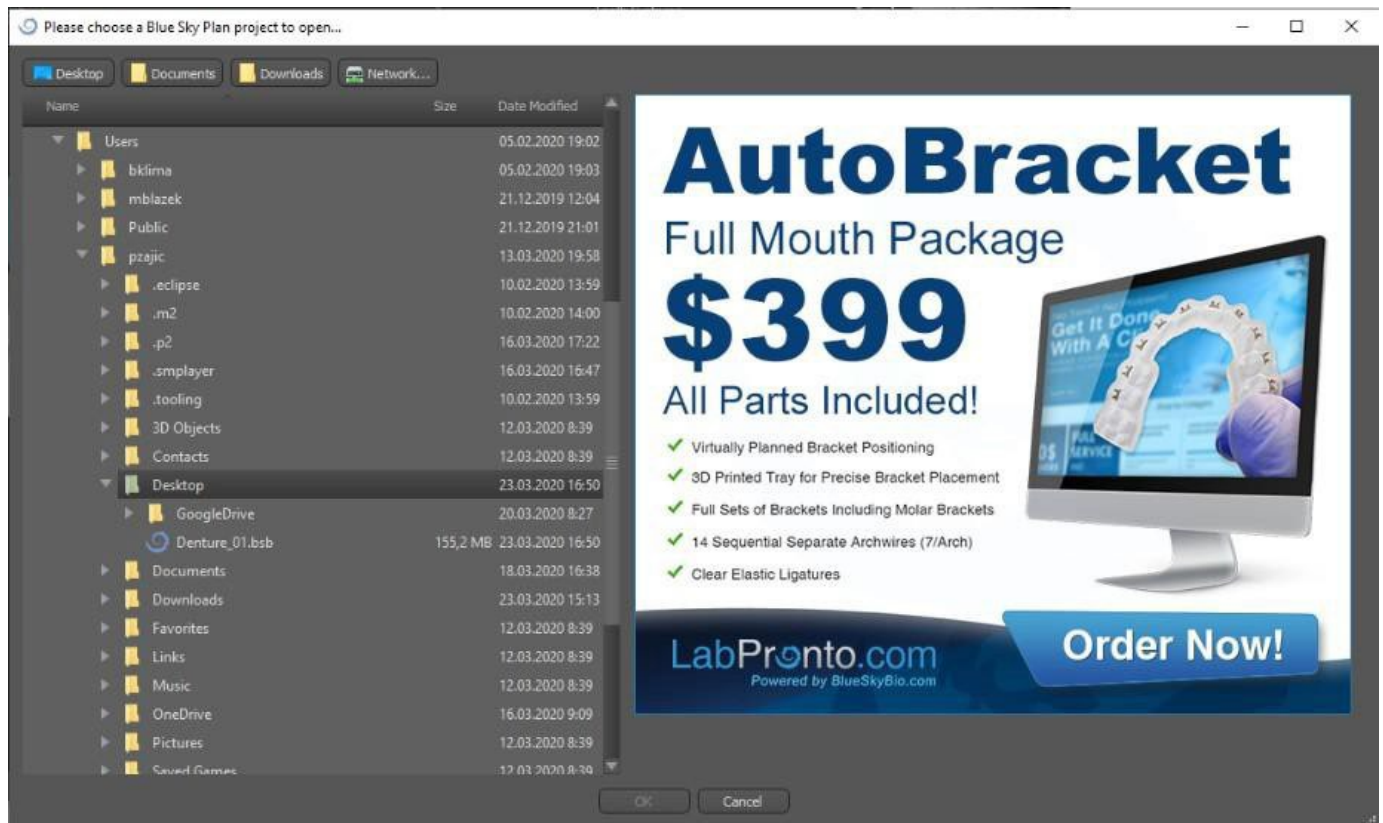


The selected project opens.

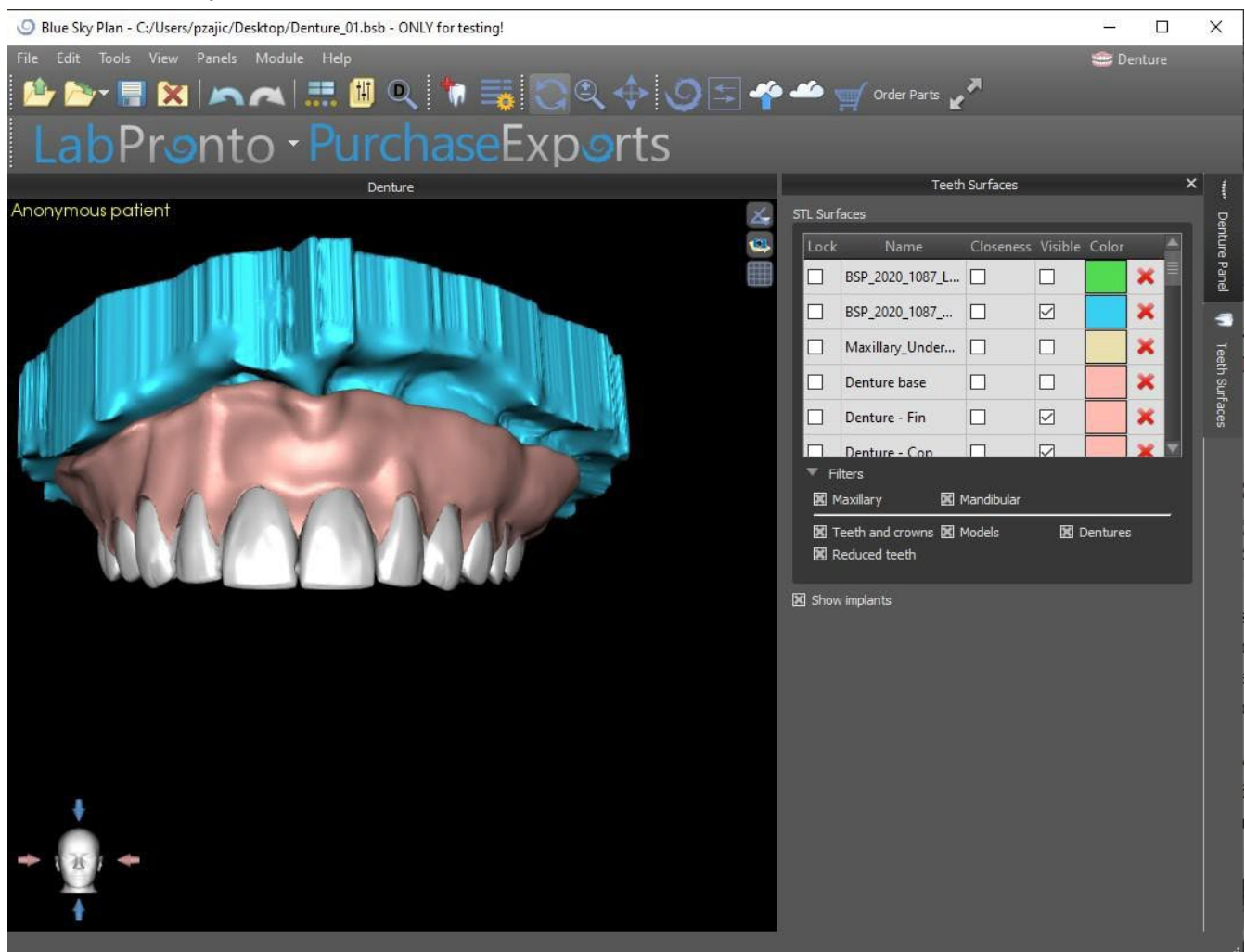


26.16 Screenshots

The Screenshot panel can be used for taking a screenshot with the  button located in the top right corner of the workspace.


After clicking the button, the “Save screenshot” message appears, and the screenshot is included in the column in the right panel.

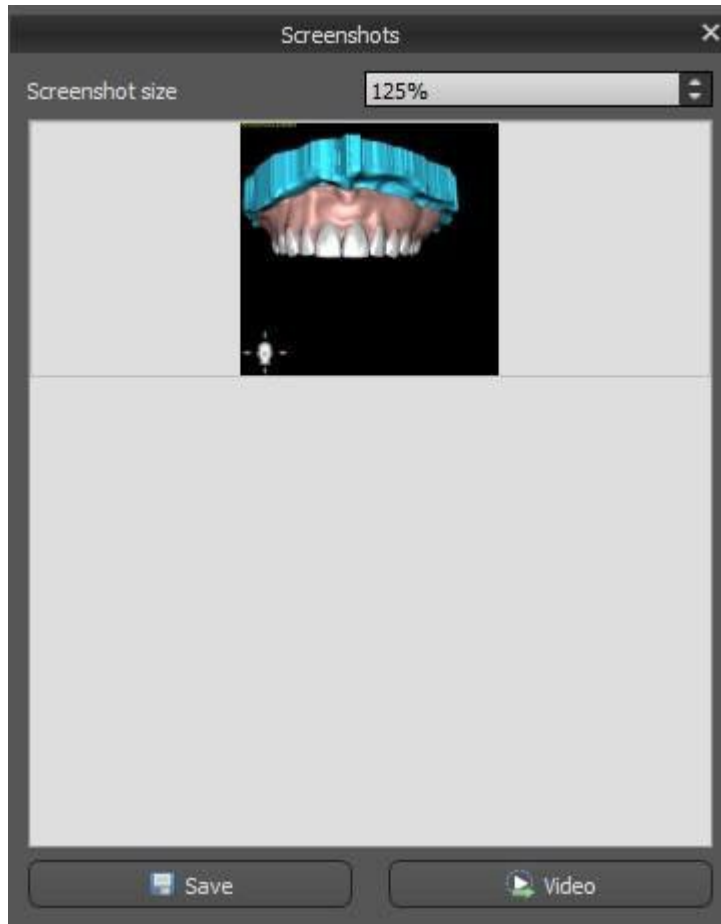

The arrows on the right are used for changing the screenshot size.

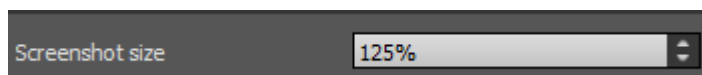

Video allows the exporting of the screenshots as AVI video or animated gif with custom text and selected font size and type.

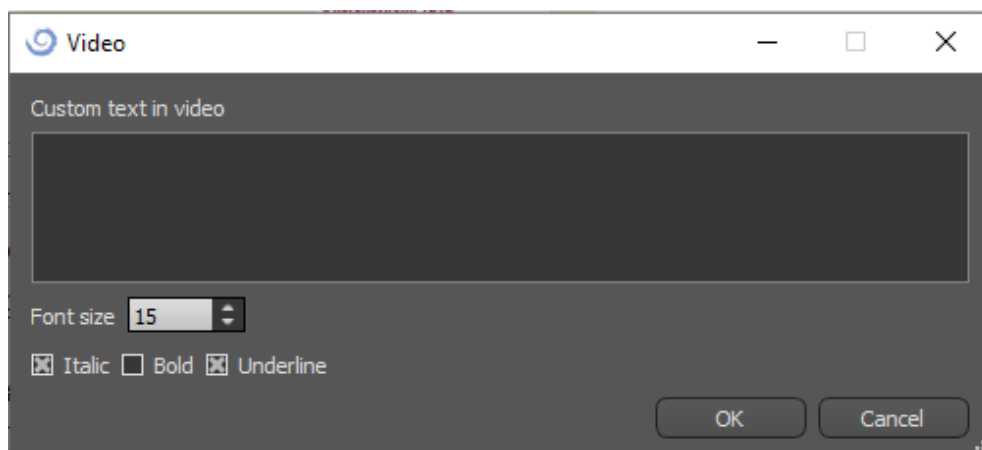


27 Crown and Bridge module

Blue Sky Plan® allows the user to create a bridge and crowns and change the geometry of the virtual teeth. These functions are available in the "Crown and Bridge" panel once the user switches to the Crown and Bridge module.


27.1 Loading the patient's STL model

The new planning starts by loading the STL jaw impression model using the main module selection "Crown and Bridge > Conventional Crown | Crown for Titanium Base | Bridge". To start planning the crown and bridge, the user needs to switch to the "Crown and Bridge" module from "Model Master". For more information, see chapter 3 "Blue Sky Launcher".

27.2 Adding virtual teeth

Virtual teeth can be added by clicking on the  icon in the panel list "Virtual Teeth > Add Tooth". A dialog for selecting the crown(s) then opens. Clicking on a tooth selects it, and a text description of the crown appears below the preview. Multiple teeth can be selected by holding down the Shift or Ctrl key. Another possibility is clicking on Select All Maxillary Teeth/Select All Mandibular Teeth or Select All Teeth. It is also possible to select from three sizes. Teeth are added as a teeth chain.

By clicking the "OK" button, the selection is confirmed; the "Cancel" button closes the dialog without making a selection.

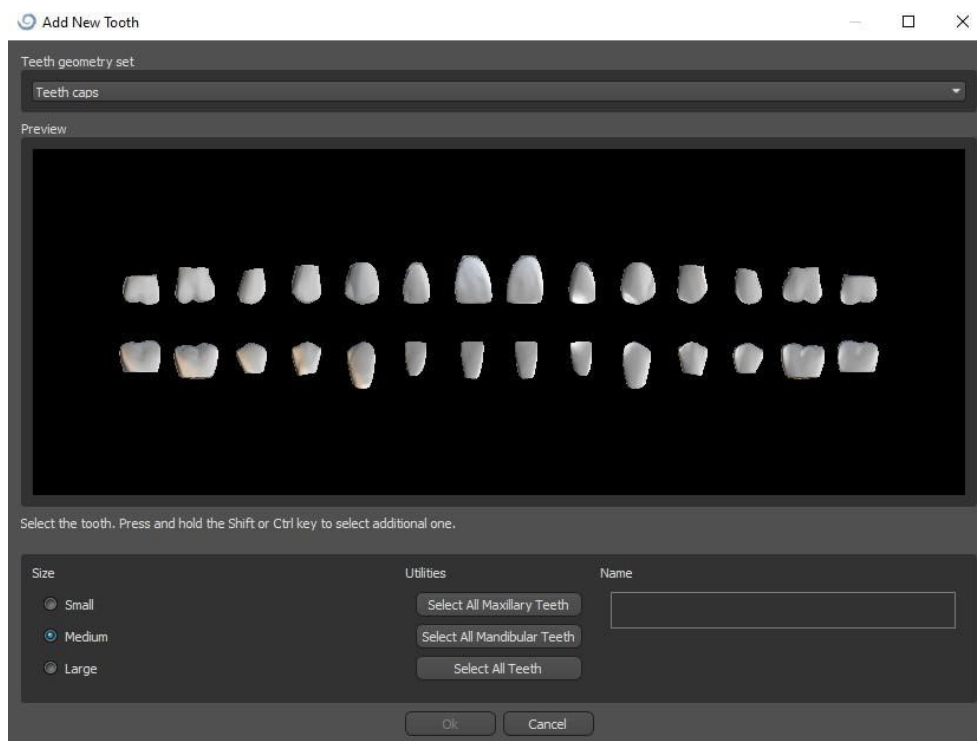

Virtual teeth are regular teeth sets, available in the Surgical Guide, Crown and Bridge and Denture module, are modifiable and exportable.

27.3 Restoration Design panel

The Restoration Design panel allows the user to choose the restoration type – crown on a tibase and conventional crown. It is also possible to select the jaw type and antagonist model as well as to create a bridge.

The antagonist model resolves and detects a teeth collision. If there is only one jaw available, the “no antagonist” selection resolves this situation.

Select models:

Model: BSP_2020_1069_Lower_Model

Crown: Left Maxillary First Molar

Tibase: ABDH30 - Left Maxillary First Molar

Antagonist: (optional)

The tibase can be chosen by clicking on the tooth or by clicking on the tibase dropdown. If a tibase is not selected, the user cannot go further.

27.4 Restoration type selection – Crown on a Tibase

The Crown on a Tibase selection can be chosen in the menu in the right panel. This requires the selection of a restoration type, jaw, model, crown, tibase, and antagonist model (optional) (if a user uses maxilla and mandible) and confirming by clicking “Start”.

Note that it is mandatory to have a scanbody scan for "Crown on Tibase" restoration.

Tip:
Select model, crown and tibase and click "Start".

Select restoration type and jaw

Restoration type: Crown on a Tibase

Select jaw: ☐ Mandible ☒ Maxilla

Select models:

Model: BSP_2020_1069_Lower_Model

Crown: Left Maxillary First Molar


Tibase: ABDH30 - Left Maxillary First Molar

Antagonist: (optional)

Start

Step 1 – Undercuts

Step 1 is used to define the initial parameters for the crown.

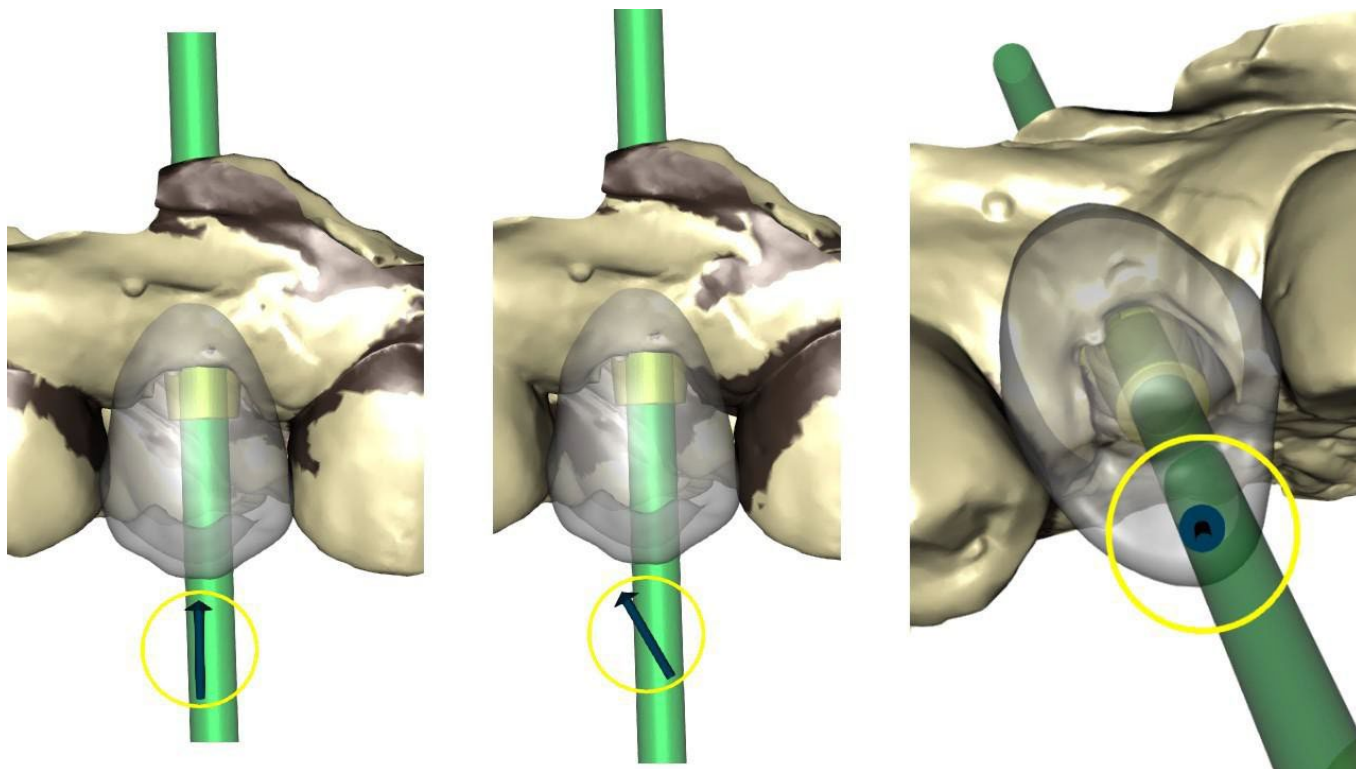


The slider can be used for setting the values for the undercut.

“Set insert direction from view” is used to set the insert direction according to the current viewing angle in the 3D view.


“Reset insert direction” restores the original setting of the insert direction.

The “Back” and “Next” buttons are used for moving one step back or forwards in the design flow.

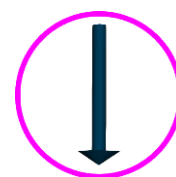

Step 2 – Identify Interproximal Areas

Step 2 is used to select/colour the existing adjacent teeth so the crown can be resized by the software to fit.

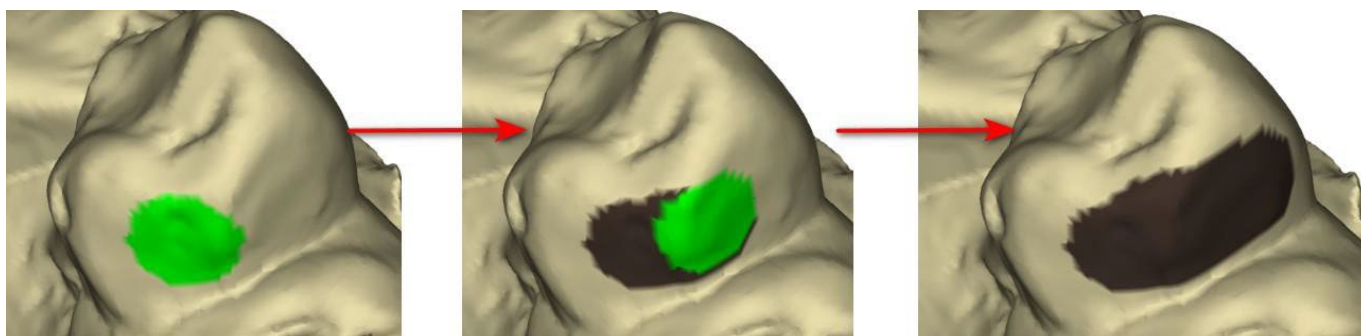


The interproximal area can be deleted by selecting “Clear Area”. Pressing the Shift key and holding the left mouse button will define the interproximal area and create a brown contour around the selection representing the crown and bridge base.


By pressing the Ctrl key and left clicking with the mouse, you can remove the selection. The “Tool size” slider is used for brush size selection.

Step 3 – Margin detection

Step 3 detects the tooth margins used for crown creation. Pressing the Shift key and holding the left mouse button allows the margin to be edited.

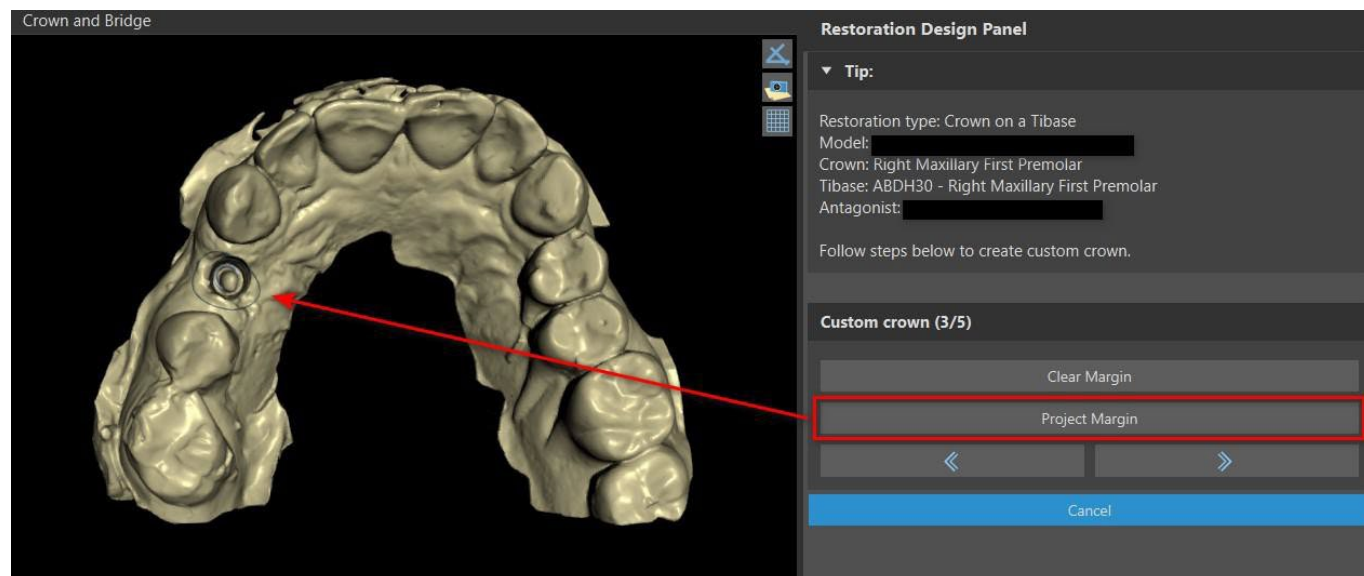


The Project Margin button projects the curve defined by the user on the deformed tooth.

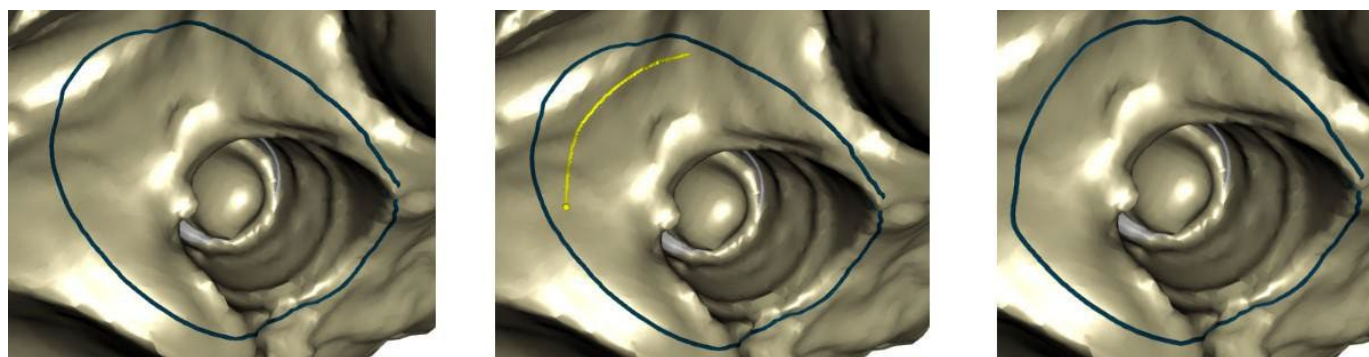

Step 4 – Tibase editing

Step 4 is used for crown upgrading and the creation of a crown screw channel.

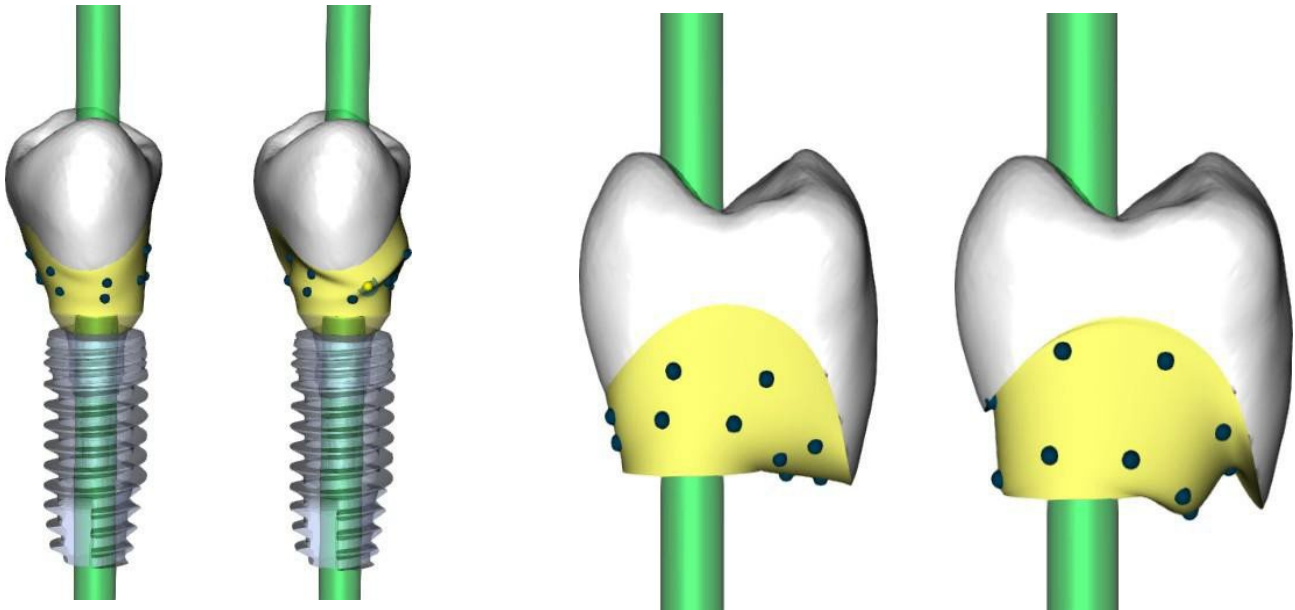

“Crown margin” can be used for setting the crown margin.


“Titanium base margin” can be used for setting the base margin.

“Crown cement spacer” is an extra space for the glue between the crown and abutment.

The create crown screw button creates a hole going through the crown so that the user can access the abutment screw and screw the abutment into the implant.

Step 5 – Crown editing


Step 5 is used for adjusting the created crown.

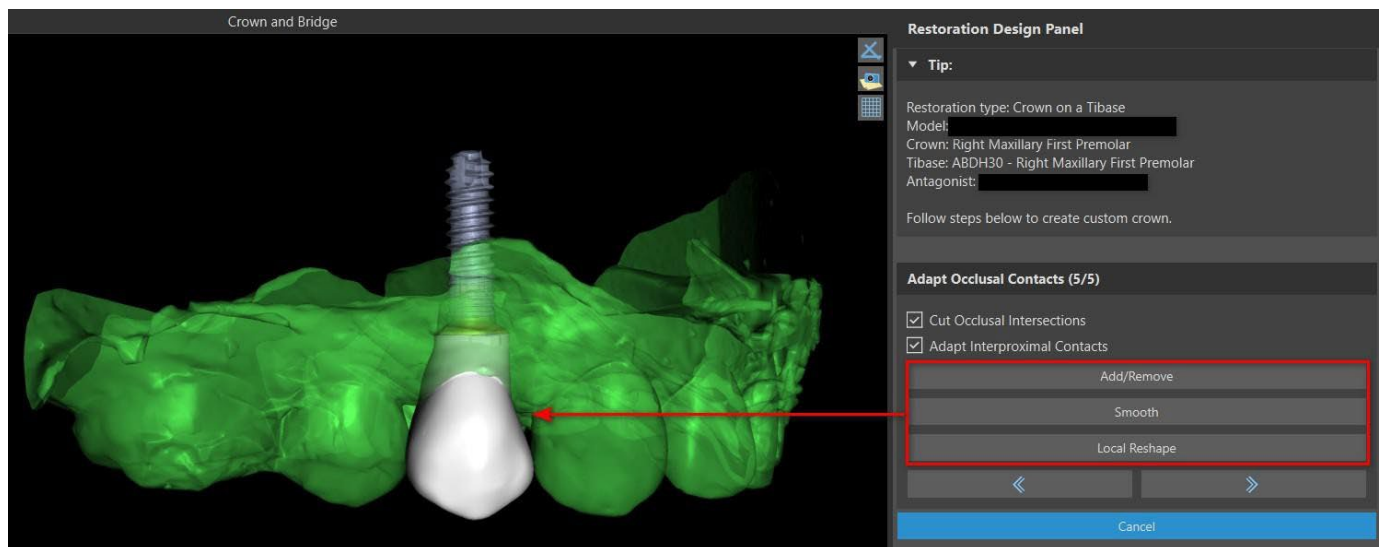

“Add/Remove” is used for the addition/removal of the crown material.

“Smooth” smooth the crown surface.

“Local reshape” Tool that deforms the tooth at a local point.

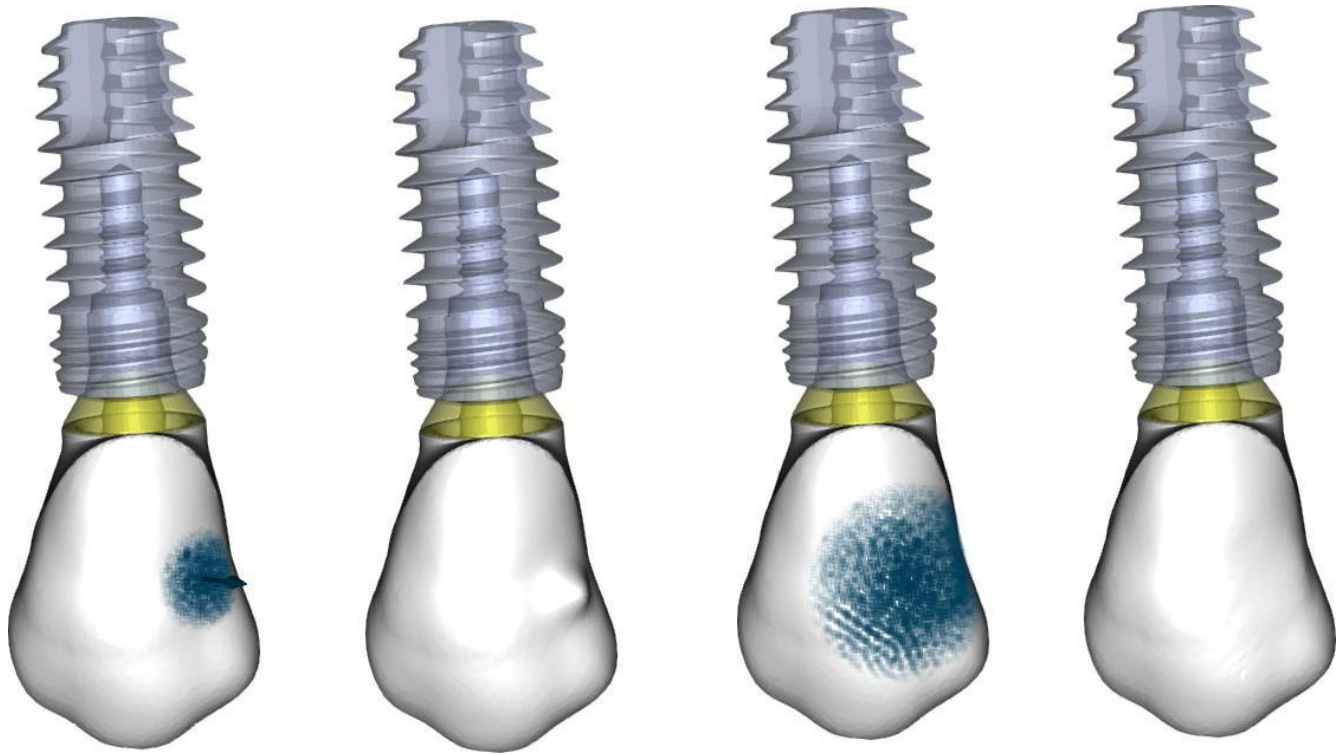

The “Tool size” slider is used for tool size selection, and “Tool strength” defines the strength of the tool. The original values can be restored by clicking on the “Set to Default” button.

The relevant checkbox can be used for hiding "Adapt Interproximal Contacts" and "Cut Occlusal Intersection".

27.5 Restoration type selection – Conventional crown

Conventional crown selection can be chosen in the menu in the right panel. This requires selecting a restoration type, jaw, model and confirming by clicking “Start”. The crown and antagonist models are optional. If a user selects the (no crown) option, the crown is added in the third step.

▼ Tip:

Select crown and model and click "Start".

▼ Select restoration type and jaw

Restoration type: Conventional crown

Select jaw:
☒ Mandible
☐ Maxilla

▼ Select models:

Model: BSP_2019_1055_Lower_Model


Crown: (optional)

Antagonist: (optional)


Start

Step 1 – Margin line definition

Step 1 is used to define a closed margin line along the remaining tooth structure. The user is able to select between semi-automatic margin line detection and manual mode. To work with semi-automatic detection, click on the “Two Point Margin Line Detection” radio button. Click 2 points (Shift + left click) , as shown in the picture below, to detect a curve. The curve should be detected automatically. It is possible to move with points or add more points to more precise detection.

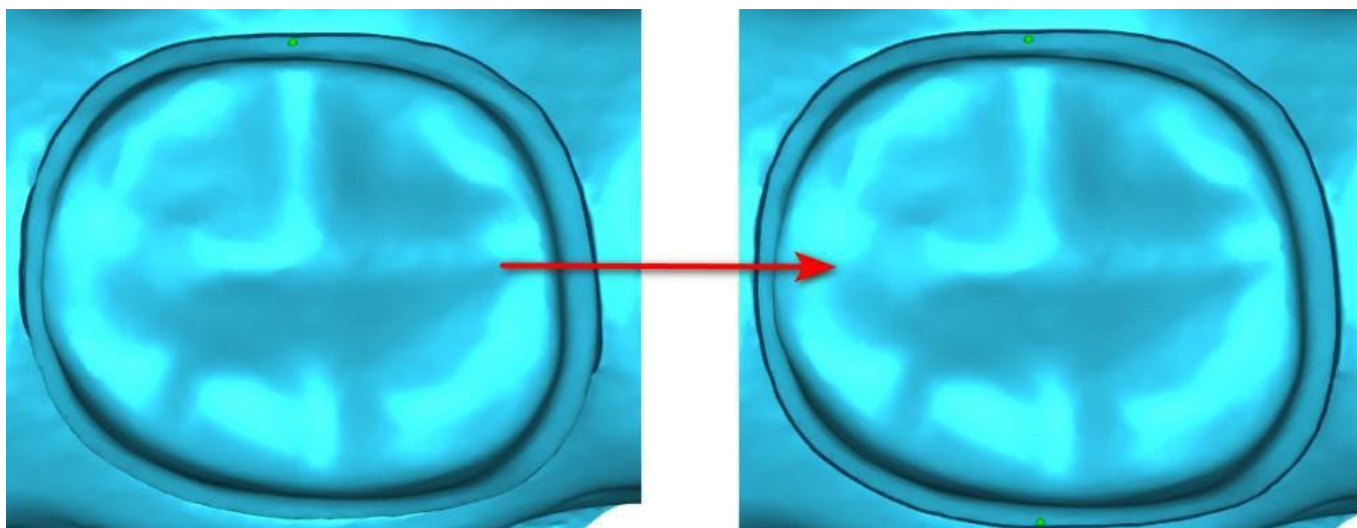

To work with the manual margin line, click on the “Manual Mode” radio button. Click the points (Shift + left click) along the remaining tooth structure. Closed curve can be modified by holding Shift and left mouse button.

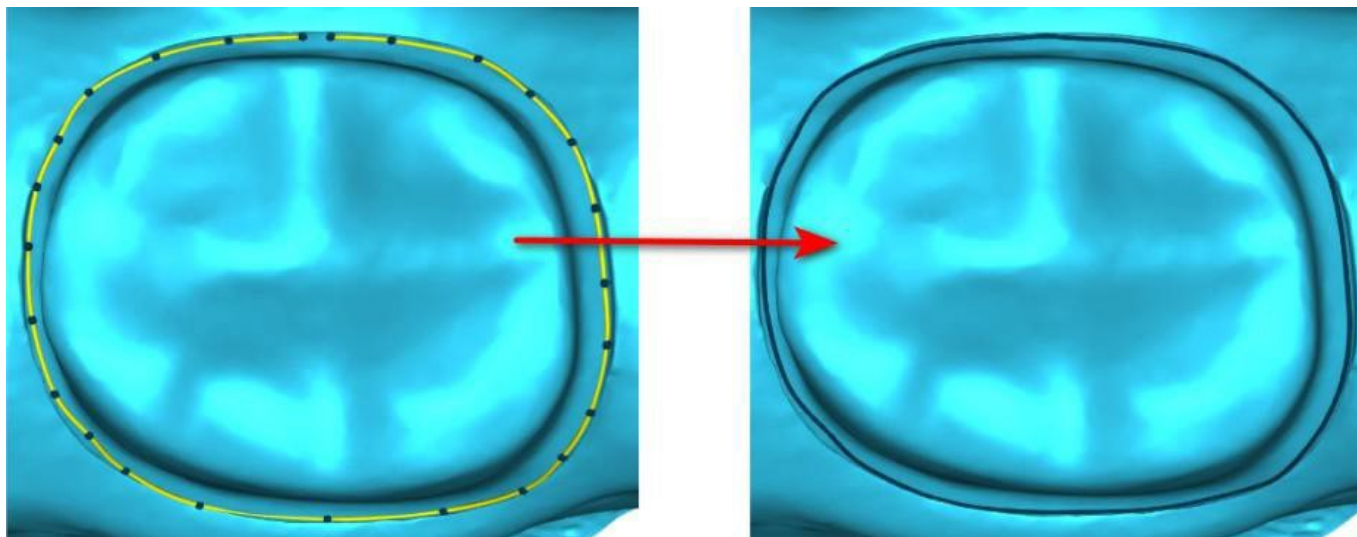


The margin can be deleted by selecting “Clear Area”. Undo and redo can be used as well for modifications

Warning: A closed margin line has to be defined first to continue to next step!

Step 2 – Undercuts Step


Step 2 is used to define the initial parameters for the crown.

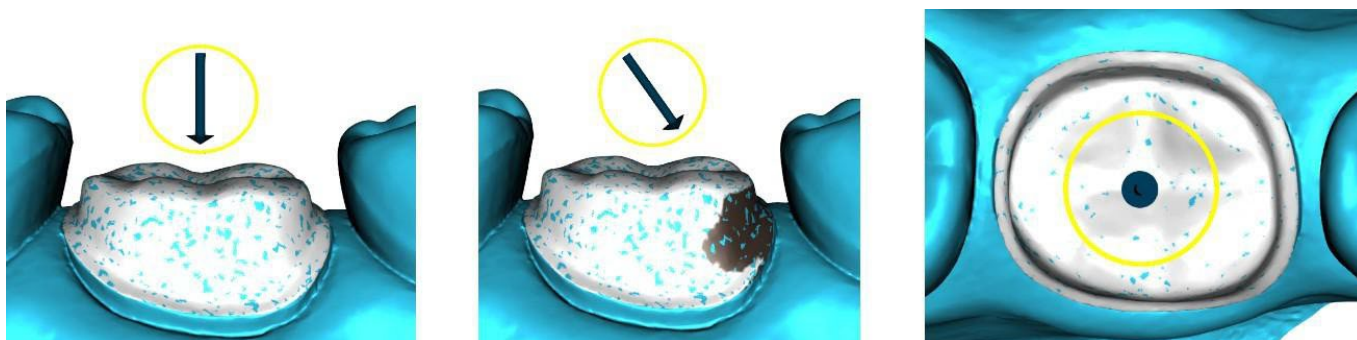

The slider can be used for setting the values for the undercut.


“Set insert direction from view” is used to set the insert direction according to the current viewing angle in the 3D view.

“Reset insert direction” restores the original insert direction.

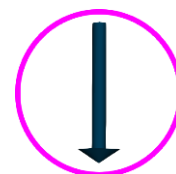

Step 3 – Identify Interproximal Areas

Step 3 is used to select/colour the existing adjacent teeth so the crown can be resized by the software to fit.

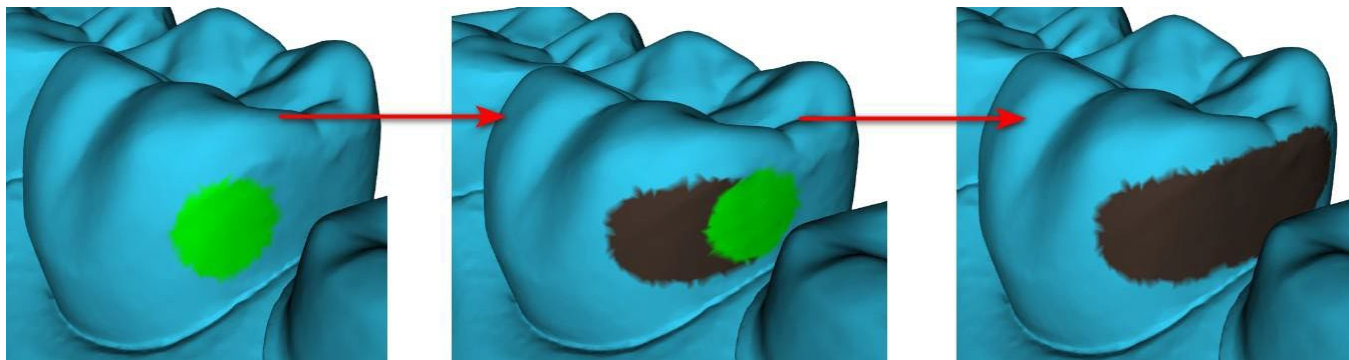


The approximal area can be deleted by selecting “Clear Area”. Pressing the Shift key and holding the left mouse button will define the interproximal area and create a brown contour around the selection representing the crown and bridge base. By pressing the Ctrl key and left clicking with the mouse, you can remove the selection.

The “Tool size” slider is used for brush size selection.

Step 4 – Crown bottom editing

Step 4 is used for editing the crown bottom.

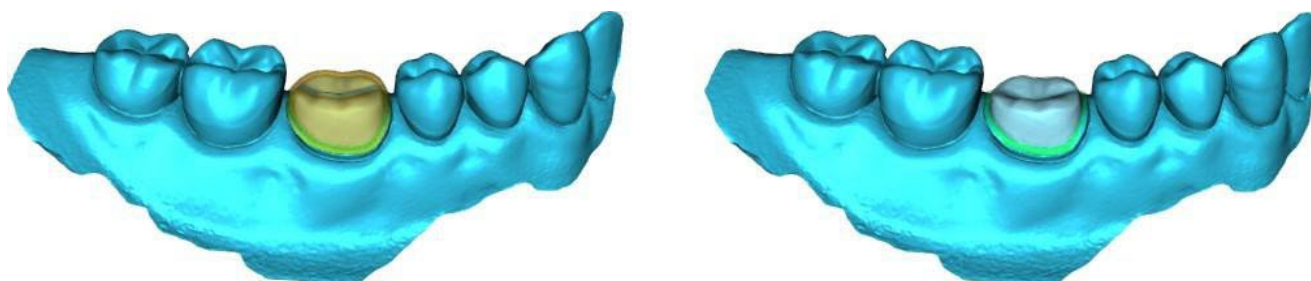

“Crown cement spacer” is an extra space for the glue between the crown and abutment.

“No cement gap” allows the user to edit the crown margin.

“Horizontal” defines the horizontal crown border width.

“Angled” defines the length of the angled part of the border.

“Angle” defines the angle of the angled part.


“Vertical” defines an additional vertical border.

“Minimal Thickness” is used to edit the minimal thickness.


The relevant checkbox can be used to hide the minimal thickness. If the user selected the (no crown) option in the restoration panel, the “Add New Tooth” dialog is available to select the tooth by clicking on the “Next” button. Only one tooth can be selected. The tooth is added by clicking on the “OK” button. Cancel closes the “Add New Tooth” dialog.

Step 5 – Tooth placement and editing

Step 5 is used for tooth placement and tooth editing.

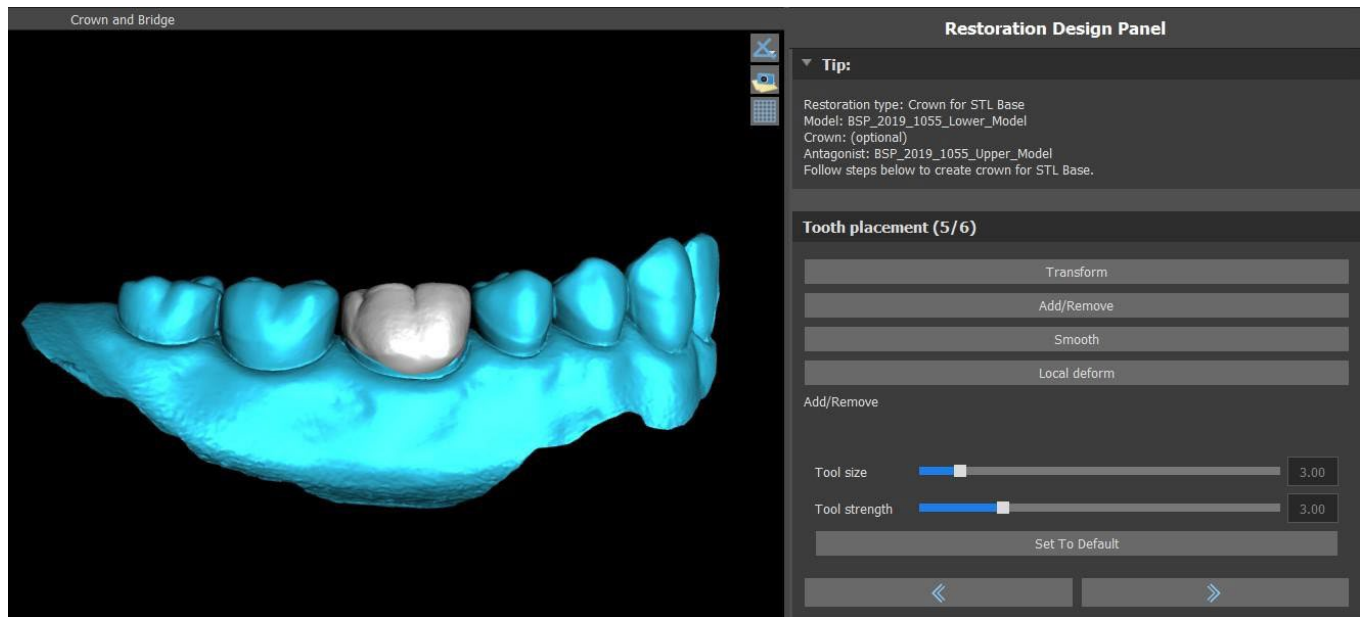

“Transform” activates the widget for tooth placement.

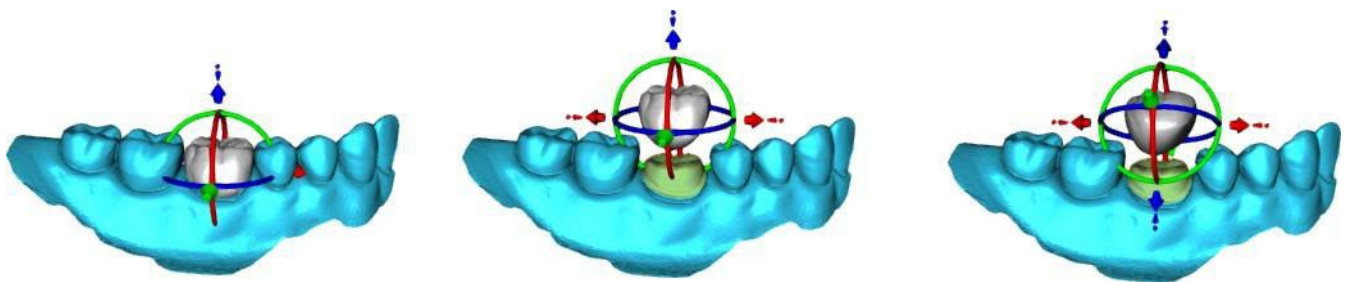

“Add/Remove” is used for the addition/removal of the crown material.


“Smooth” smooths the crown surface.

“Local deform” allows the creation of ups and downs.

The “Tool size” slider is used for tool size selection, and “Tool strength” defines the strength of the tool. The original values can be restored by clicking on the “Set to Default” button.

Step 6 – Crown editing

Step 6 is used for additional crown editing.

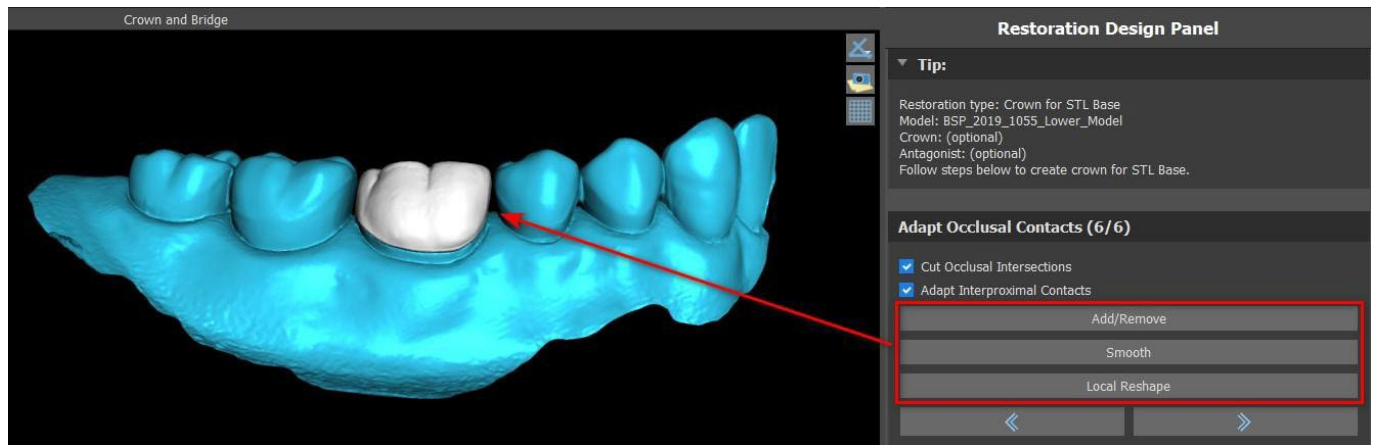

“Add/Remove” is used for the addition/removal of the crown material.

“Smooth” smooths the crown surface.

“Local Reshape” Tool that deforms the tooth at a local point.

The “Tool size” slider is used for tool size selection, and “Tool strength” defines the strength of the tool. The original values can be restored by clicking on the “Set to Default” button.


The relevant checkbox can be used for hiding "Adapt Interproximal Contacts" and "Cut Occlusal Intersection".

27.6 Create Bridge

The create bridge function is available in the Restoration Design Panel but only if you select Bridge process flow in the Main screen. It allows the user to design a bridge for missing teeth. By clicking on “Create Bridge”, the user starts a new process flow.

Step 1 – Unit Bridge


Step 1 is used for setting the points between the teeth to unit them. If the teeth are intersecting, the blue points are not created.

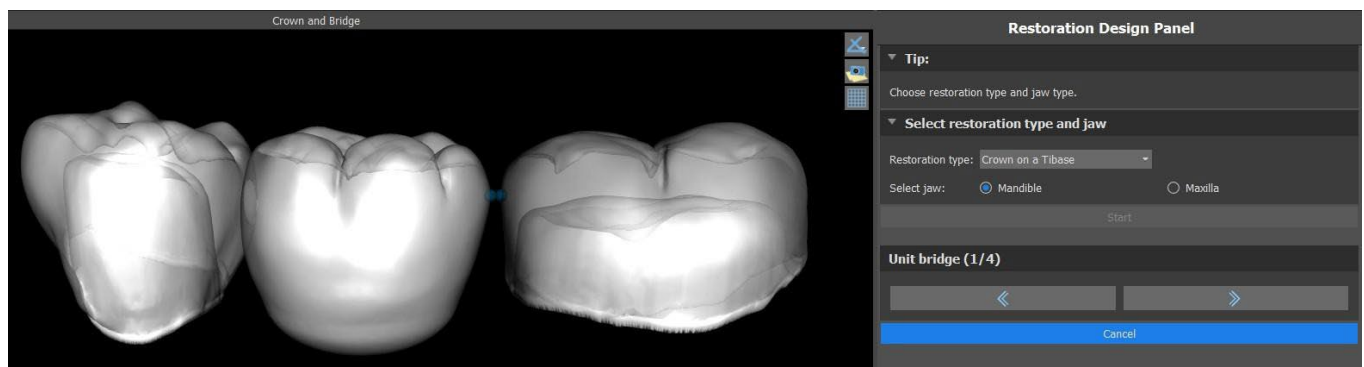

The blue points can be dragged along the teeth to set the bridge according to the user’s preferences.


Step 2 – Editing connections

The blue points allow the user to edit the connections between the crowns or teeth.

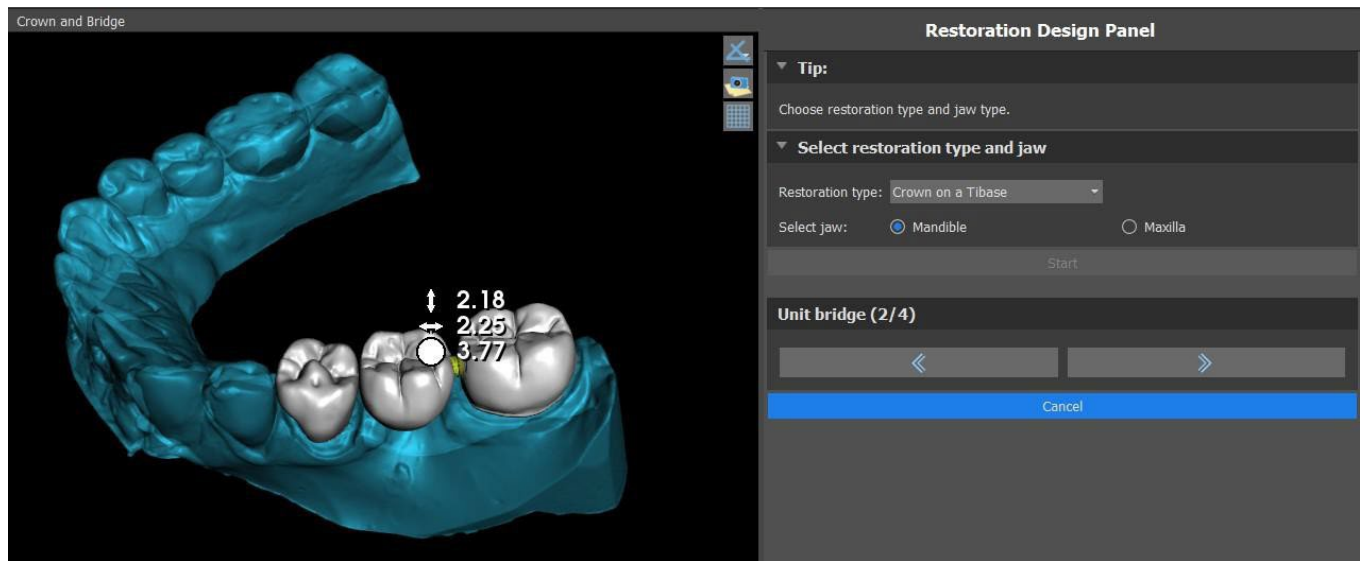

Step 3 – Bridge editing

Step 3 is used for editing the bridge.

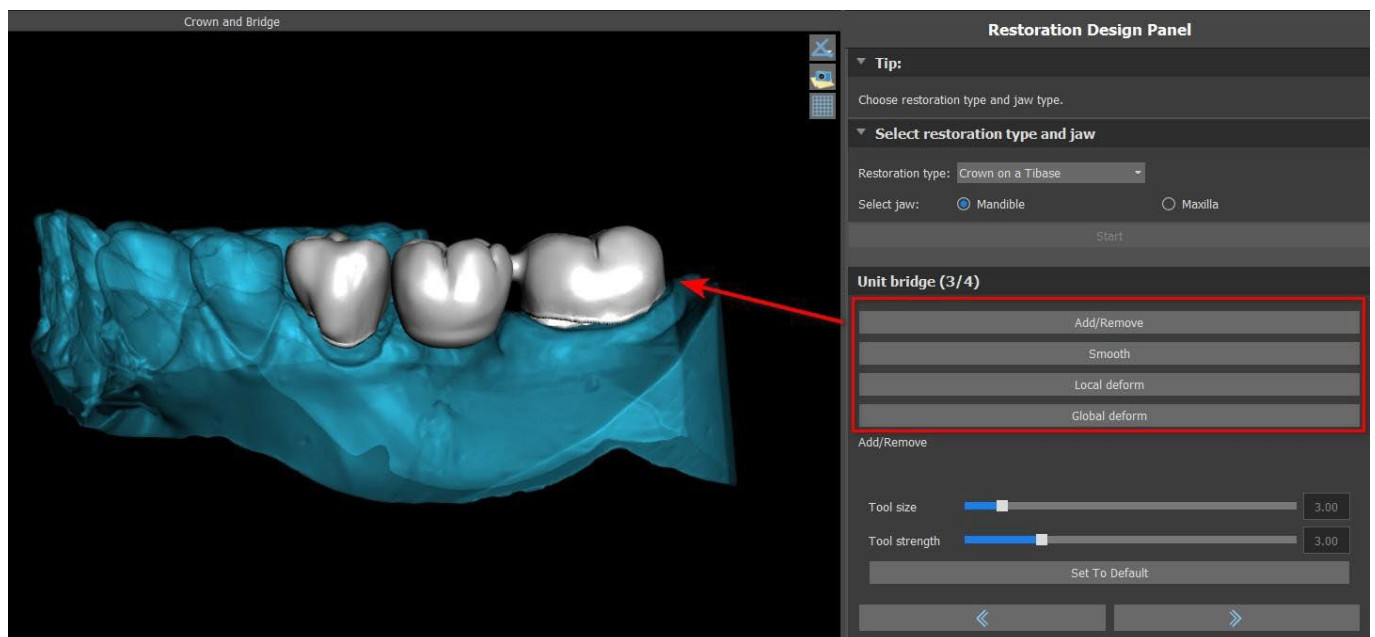

“Add/Remove” is used for the addition/removal of the material.

“Smooth” smooths the bridge surface.

“Local deform” allows the creation of ups and downs.


“Global deform” creates ups and downs on a global scale.

The “Tool size” slider can be used to change the brush tool dimension, and “Tool strength” defines the strength of the tool. The original values can be restored by clicking on the “Set to Default” button.

If the teeth are overlapping, the union operation between teeth is being performed and the first two steps are skipped.

Step 4 – Undercuts

Step 4 is used to define the initial parameters for the bridge.

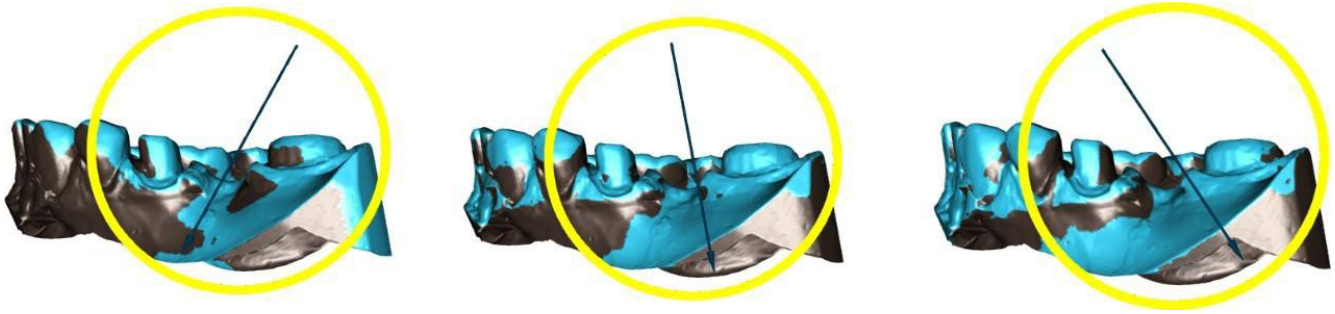


The slider can be used for setting the values for undercut.

“Set insert direction from view” is used to set the insert direction according to the current viewing angle in the 3D view.

“Reset insert direction” restores the original insert direction.

Bridge creation is finalized by clicking on the “Next” button.

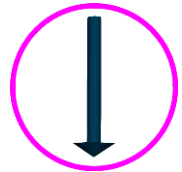

27.7 Teeth Surfaces panel

Teeth Surfaces lists the imported patient’s STL models, placed virtual teeth and their reduced versions, and the newly generated crown base:

Lock	Name	Closeness	Visible	Color	
<input type="checkbox"/>	BSP_2020_1069_Lower_Model	<input type="checkbox"/>	<input checked="" type="checkbox"/>		
<input type="checkbox"/>	BSP_2020_1069_Upper_Model	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	Left Maxillary First Molar	<input type="checkbox"/>	<input checked="" type="checkbox"/>		

The table includes the following attributes:

“Lock” – surface locking controlled with a checkbox.

“Name” – surface name.

“Closeness” – surface closeness controlled with a checkbox.


“Visible” – surface visibility or hiding with a checkbox.

“Colour” – surface colour.




– surface removal.

Filters can be used to display either the maxilla or the mandible only, or selected object types:

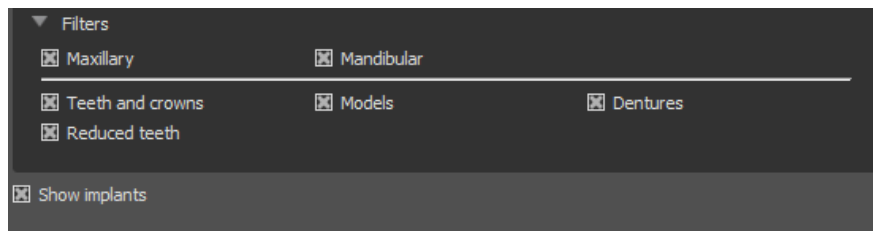

27.8 Teeth Edit panel

The Teeth Edit panel allows the user to edit the shape of a tooth by using different editing tools:

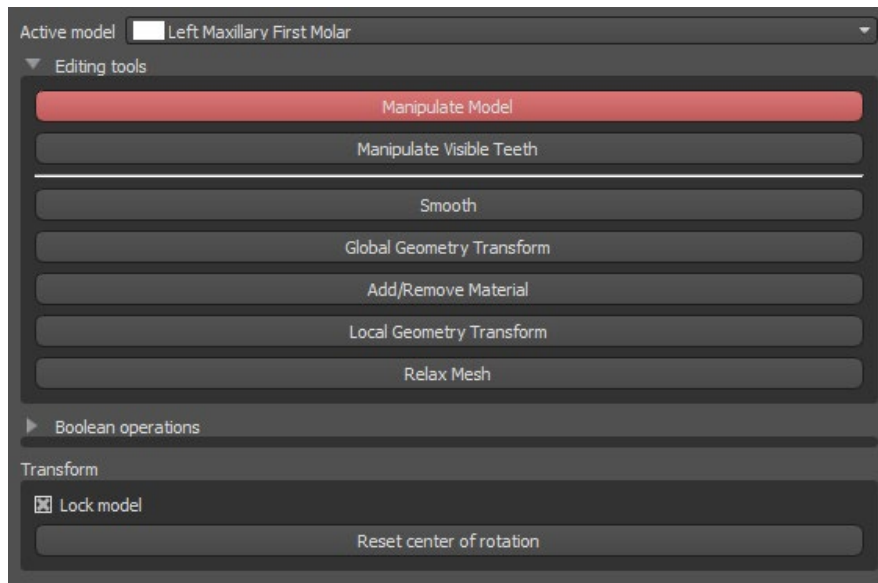

“Manipulate Model” allows the user to change the position of the tooth.

“Smooth” smooths the tooth surface.

“Global Geometry Transform” creates ups and downs on a global scale.


“Add/Remove” is used for the addition/removal of the material.

“Local Geometry Transform” creates ups and downs on a local scale.

“Relax Mesh” is similar to smoothing while it respects sharp edges to some degree.

The “Tool size” slider can be used to change the brush tool size. The original values can be restored by clicking on the “Set to Default” button.

Boolean operations allow the user to join and subtract 3D models.

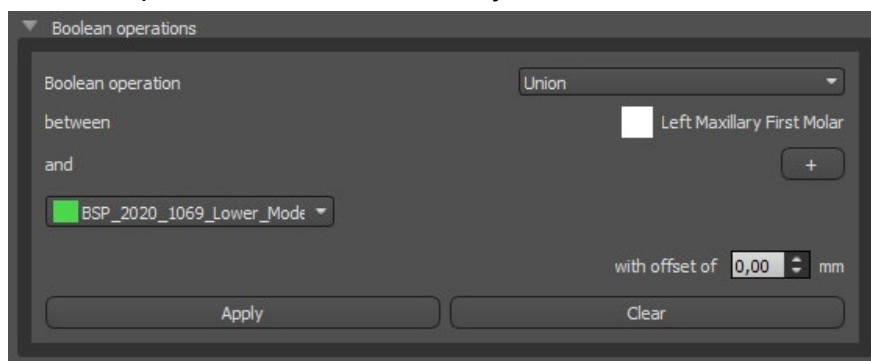


27.9 Parts panel


The Parts Panel lists the imported patient's drill kit and guide type and quantity of each part, which can be added to the shopping cart.

Parts Panel			
Drill Kit		Blue Sky Bio Direct Cut Drills	
Guide Type		Digital Surgical Guide with Built-in Stops	
	Type	ID	Quantity
<input checked="" type="checkbox"/>	Implant	IDH3010	1
<input checked="" type="checkbox"/>	Implant	IDH3011	1
<input checked="" type="checkbox"/>	Implant	ION4110	1
<input checked="" type="checkbox"/>	Metal Cylinder	TUBE44-272	2
<input checked="" type="checkbox"/>	Metal Cylinder	Tube368 (pack of 5 pcs)	1
<input checked="" type="checkbox"/>	Drill	DRILLD26L	1
<input checked="" type="checkbox"/>	Drill	DRILLD26S	1
<input checked="" type="checkbox"/>	Drill	DRILLT35S	1
<input checked="" type="checkbox"/>	Abutment	ABDH30	2
<input checked="" type="checkbox"/>	Abutment	AOA1565A	1
<input checked="" type="checkbox"/>	Drill Shaft	CDRD32L	1
<input checked="" type="checkbox"/>	Drill Shaft	CDRD32S	1
<input checked="" type="checkbox"/>	Drill Shaft	CDRT35S	1

The user is taken to the Blue Sky Plan official webpage after clicking on the “Add Parts to Cart” button. The original values of the quantity of parts can be restored by clicking on “Clear user Adjustments”.


27.10 Screenshots panel

The Screenshot panel can be used for taking a screenshot with the button  located in the top right corner of the workspace. After clicking the button, the “Save screenshot” message appears, and the screenshot is included in the column in the right panel.

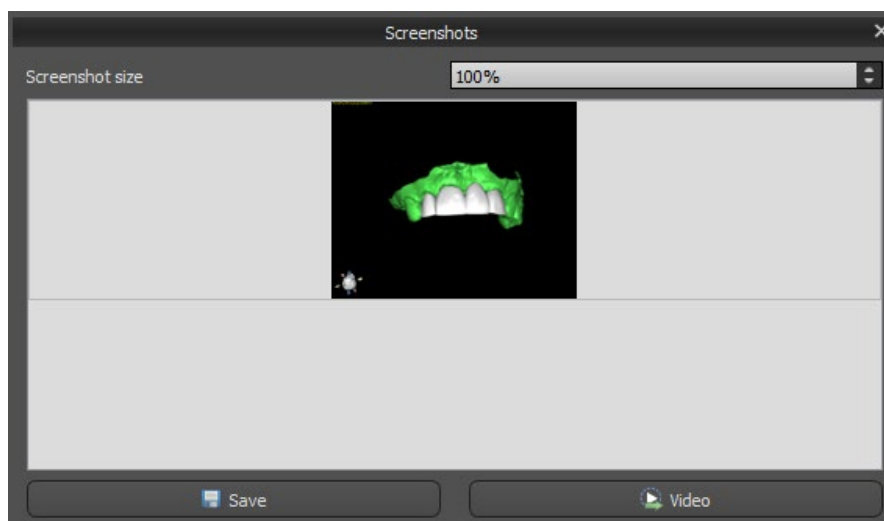

The arrows on the right are used for changing the screenshot size.

Video allows the screenshots to be exported as an AVI video or animated gif with custom text and selected font size and type.

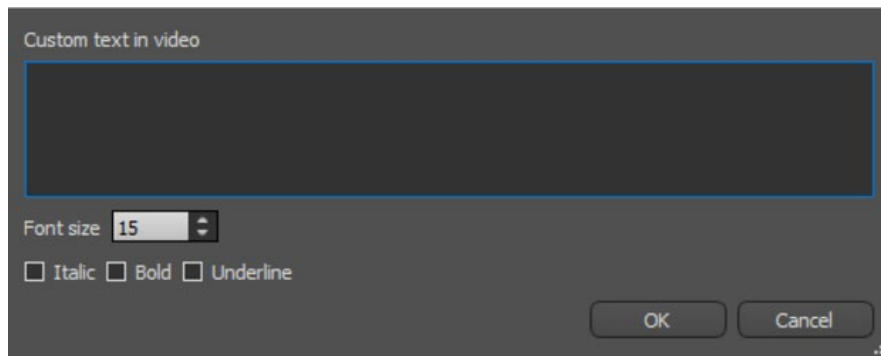


27.11 CAM Export

In addition to the normal export of models, the software allows you to create a CAM export.

In order to export, you need to create at least one crown or one bridge. Once you complete this step, you can choose which models you want to export in the CAM column (1) inside of the export dialog.


If the model is not CAM exportable the value will be "N/A" (2), otherwise there will be a checkbox for selection (3).

If you only want to see the models which can be CAM exported, use the button on the bottom left called "Show only teeth with CAM file support" (4).

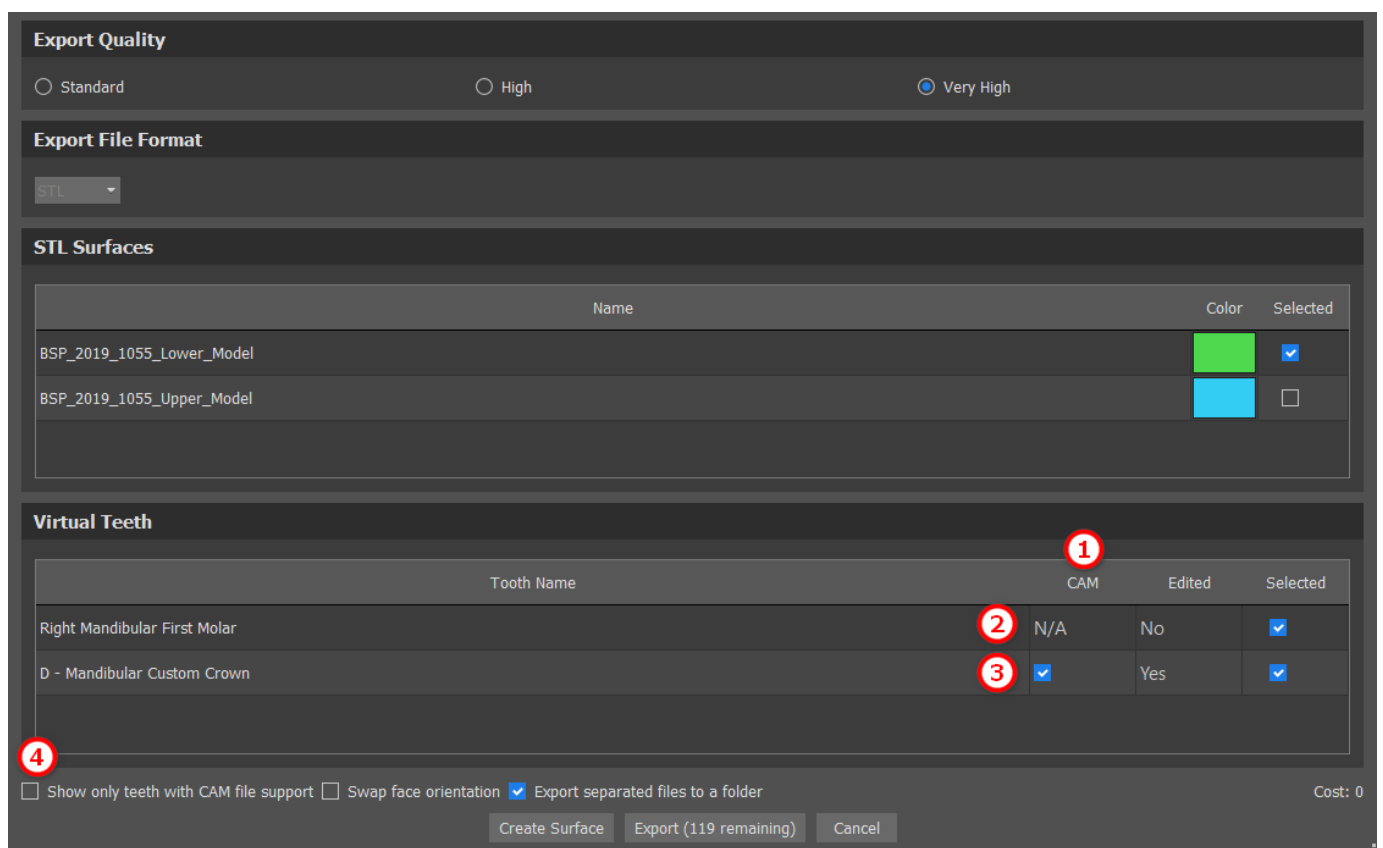

The final product of the CAM export is a file called < name > .constructionInfo where all the information about the model are stored.

27.12 Crown and Bridge Preferences

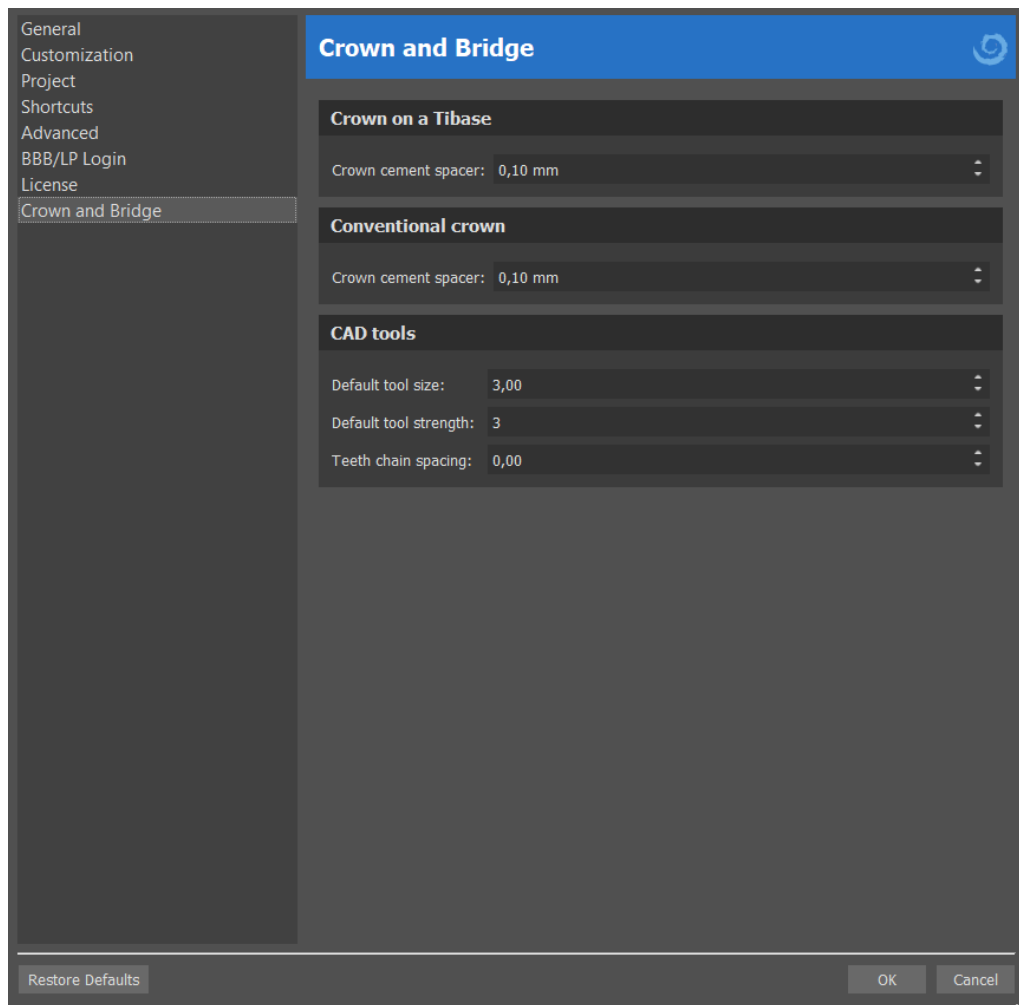


In “Preferences > Crown and Bridge” the user can set the Crown cement spacer for different types of restoration available in the Restoration Design panel. CAD tools are available for the Denture module. The “Default tool size” is used for selecting the default tool size, and “Default tool strength” defines the default strength of the tool. “Teeth chain spacing” allows the user to set the spacing between the teeth in the teeth chain.

28 Splint Module

Dental splints are the most common dental treatment for temporomandibular disorders (TMDs - tension in your jaw muscles, such as from clenching your jaw or grinding your teeth). These splints are used for short periods of time. They do not cause permanent changes in the teeth or jaw. They include bite plates and mouth guards that fit between and over the upper and lower teeth.

Dental splints can help ease muscle tension and stabilize the jaw. They do this by preventing grinding and clenching of the jaw (bruxism) that might be causing muscle tension and pain. It is an important treatment that may last for 3 months or more. The splints are also used for muscle syndromes, a new VDO muscle test, mandibular lateral displacement and anterior repositioning.

Blue Sky Plan makes it possible to plan splint treatments based on imported STL models of the jaw with teeth. The planning begins with an intro screen titled **Splint**, where the user selects the type of planning – **Incisal Splint (AMPSA)** or **Occlusal Bite Splint**.

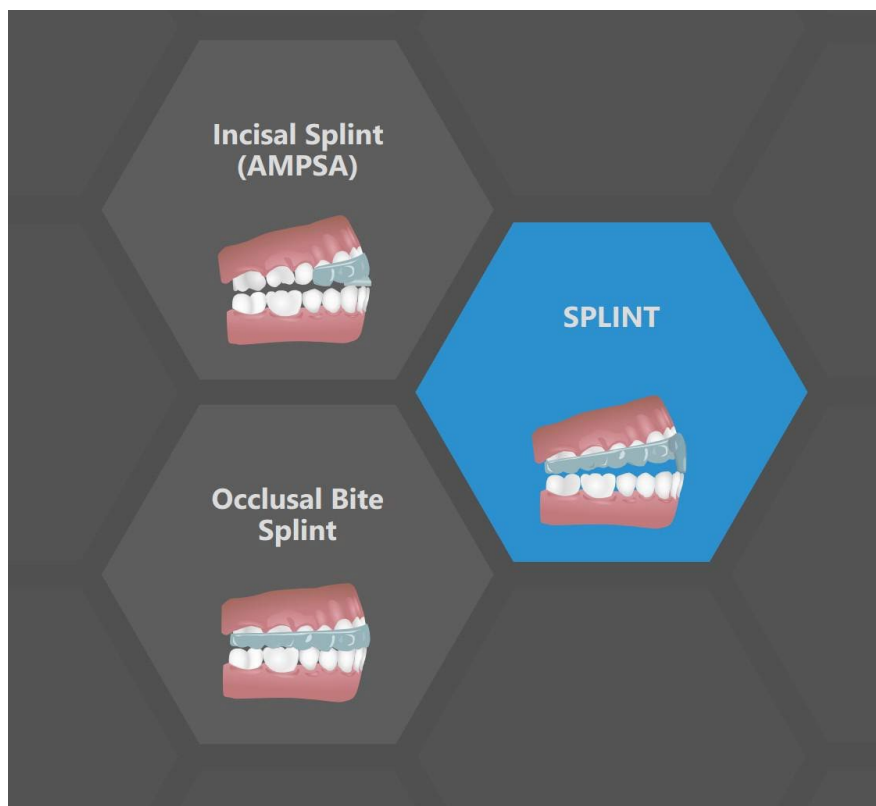

The specific type of planning can't be changed in an upcoming process.

The individual types of planning include:

- **Incisal Splint (AMPSA) -> Align to Grid - Buccal Bite Registration - Jaw Articulation - Remove Undercuts - Add Anterior Deprogrammers - Draw Splint Curve - Set Raised Areas - Edit Splint Module - Export**
- **Occlusal Bite Splint (Michigan-style splint) -> Align to Grid - Buccal Bite Registration - Jaw Articulation - Remove Undercuts - Draw Splint Curve - Set Raised Areas - Edit Splint Model - Export**

28.1 Loading the patient's STL model


To start planning the splint, the user needs to select type of splint "Incisal Splint" or "Occlusal Bite Splint". For more information, see chapter 3 "Blue Sky Launcher" (available in PDF version of the manual).

28.2 Align to Grid

The imported models might not be properly aligned (the alignment does not match the head in the bottom left of the 3D view). This makes planning very difficult. For these cases, Blue Sky Plan has a **Model Alignment** step.

For alignment, it is necessary to select the primary model through which the alignment is to be done.

To align the model to the grid, press the **SHIFT key** and add 3 points on the model by left clicking the **mouse** to align the jaw to the grid. Then fine-tune alignment with the manipulation widget.

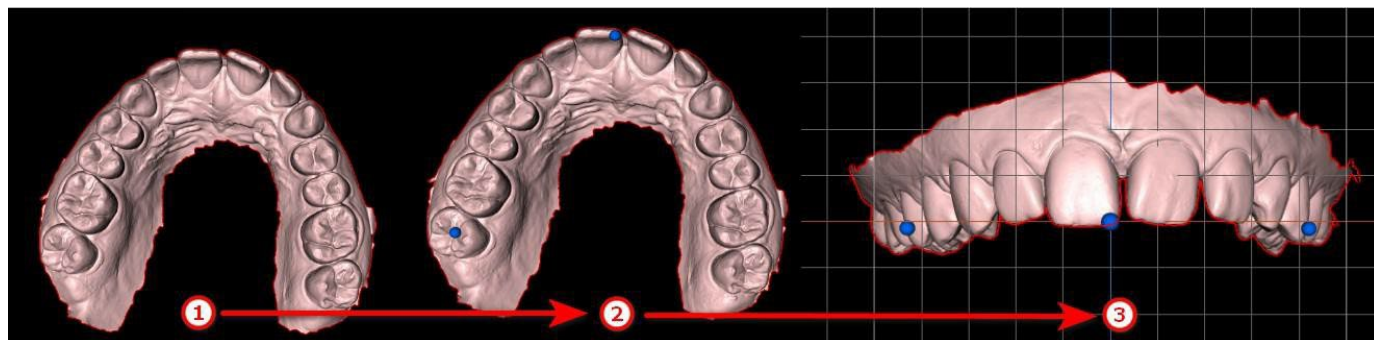

The Checkbox **Move All Models Together** can be used in situations, when the user wants to fine-tune each model separately. To change the model orientation, click on the **Swap Orientation**

28.3 Buccal Bite Registration

Note: You won't be able to work with the buccal bite registration step if you didn't import buccal bites during the data import step.

Buccal Bite Registration ?

Check loaded buccal bite alignment.

Buccal Bite(s) Positions

Positions:

Name	Visible
Bite	<input type="checkbox"/> ✖

Move Mandible to Position

Animate Mandible to Position

Stop Animation

Save Current Alignment of Mandible

Use Computed Alignment of Mandible

Animation Speed

Advanced

Here you can work only with buccal bites that do not have a position yet (if you deleted it before). You can fix annotation from the data import step from bites to dual bites etc.

Add Buccal Bite Positions

Buccal Bite Name:

Bite

Select Buccal Bite Scan:

☒ BSP_2017_1030_model_bite2

☐ Dual Buccal Bite Scan (left and right)

+

Add Buccal Bite Position

The buccal bite registration process requires both jaw models to be imported.

The positions table displays a list of the jaw positions that were aligned with the buccal bite models. Bite name displays the aligned buccal bites and can be renamed by double clicking on the bite name. Buccal bites can be deleted by clicking on the **x** button.

Move Mandible to Position - Instantly moves the mandible to the position chosen from the table.

Animate Mandible to Position: Animates the mandible from the actual position to the position chosen from the table.

Stop Animation: Stops the motion animation.

Save Current Alignment of Mandible: Sets the actual position as a starting position for the position chosen from the table.

Use Computed Alignment of Mandible: Sets the actual position for the position chosen in the table as calculated using the buccal bites models.

Animation Speed slider: Sets the speed of animation.


In the advanced section, the user can work only with buccal bites that do not have a position yet (if it was deleted before). To add a buccal bite name, click on the **Buccal Bite Name dropdown** and select a name. To rename the bite name, click onto the Bite section and rename it. By clicking on the **Add Buccal Bite Position**, the user adds a new bite position that can be seen in the **Positions** table.

28.4 Jaw Articulation

The lower jaw articulation can be visualized by adding a jaw articulator. This function is not dependent on an import of the buccal bite. Collisions between the upper and the lower jaw can be shown in all positions.

Place Jaw Articulator on mandible by left clicking on the occlusal edge between the mandibular central incisors to place the jaw articulator.


Tip: Always make sure to place the articulator and that a non zero opening is set in order to make a space for the splint or anterior deprogrammer.

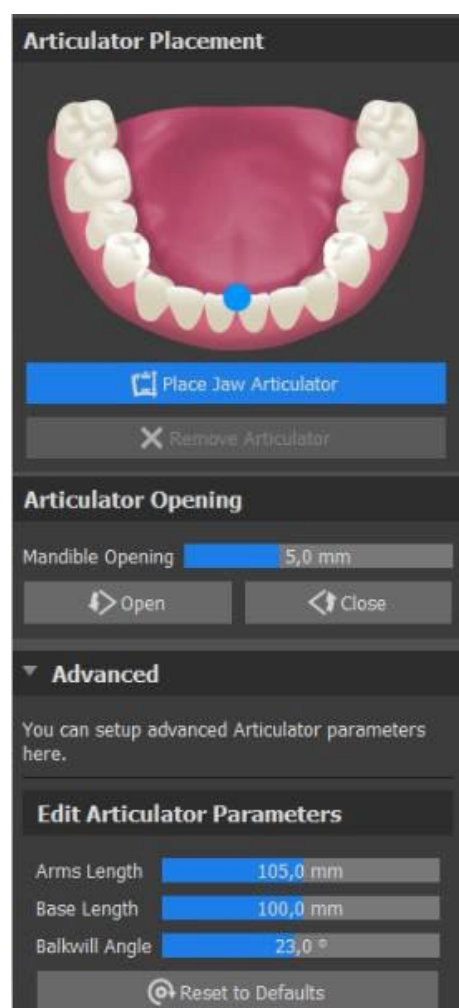


To add jaw articulator, click on the **Place Jaw Articulator** button. The **Remove Articulator** button removes the articulator from the software. The mandible opening slider sets the distance/offset between the jaws. The articulator parameters can be adjusted in the Advanced section. **Reset to Defaults** resets the parameters to defaults positions.

28.5 Remove Undercuts

Remove Undercuts is used to define the direction from which the splint will be inserted. Using this function, you can visualize the areas that run “under” the model (undercuts) and may cause problems when inserting the splint onto the teeth, especially in the case of using stiff materials for fabrication. The splint guide fabricated on this model will be easy to mount.

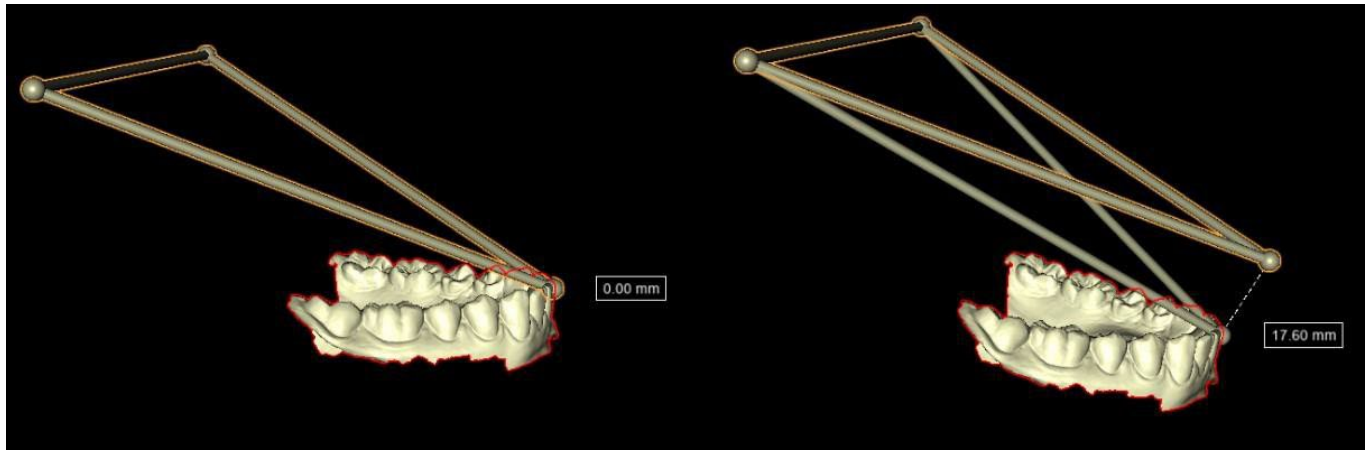




Articulator visualization shows the user how much the mandible jaw has opened. The distance is shown in the label.

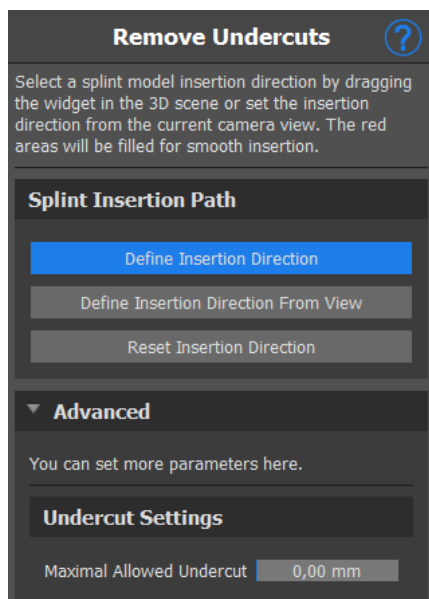

How to work with insert direction:

- Select the **model** to work with
- Click on **Define Insertion Direction**
- The insertion direction should be adjusted using the **rotation ring** so that the undercuts are minimized
- The insert direction can also be defined by clicking on **Define Insertion Direction from View**
- **Reset Insertion Direction** will **restore** the original status of the arrow
- The **Maximal Allowed Undercut** allows the user to set the values for the undercut.

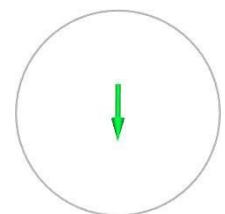

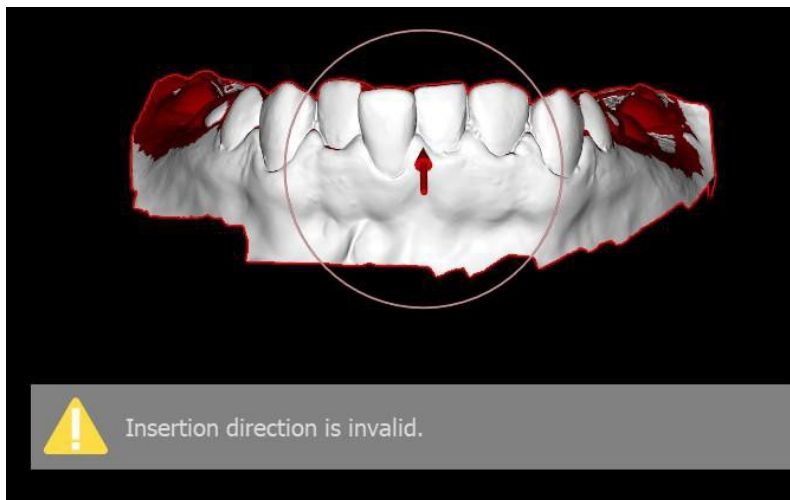
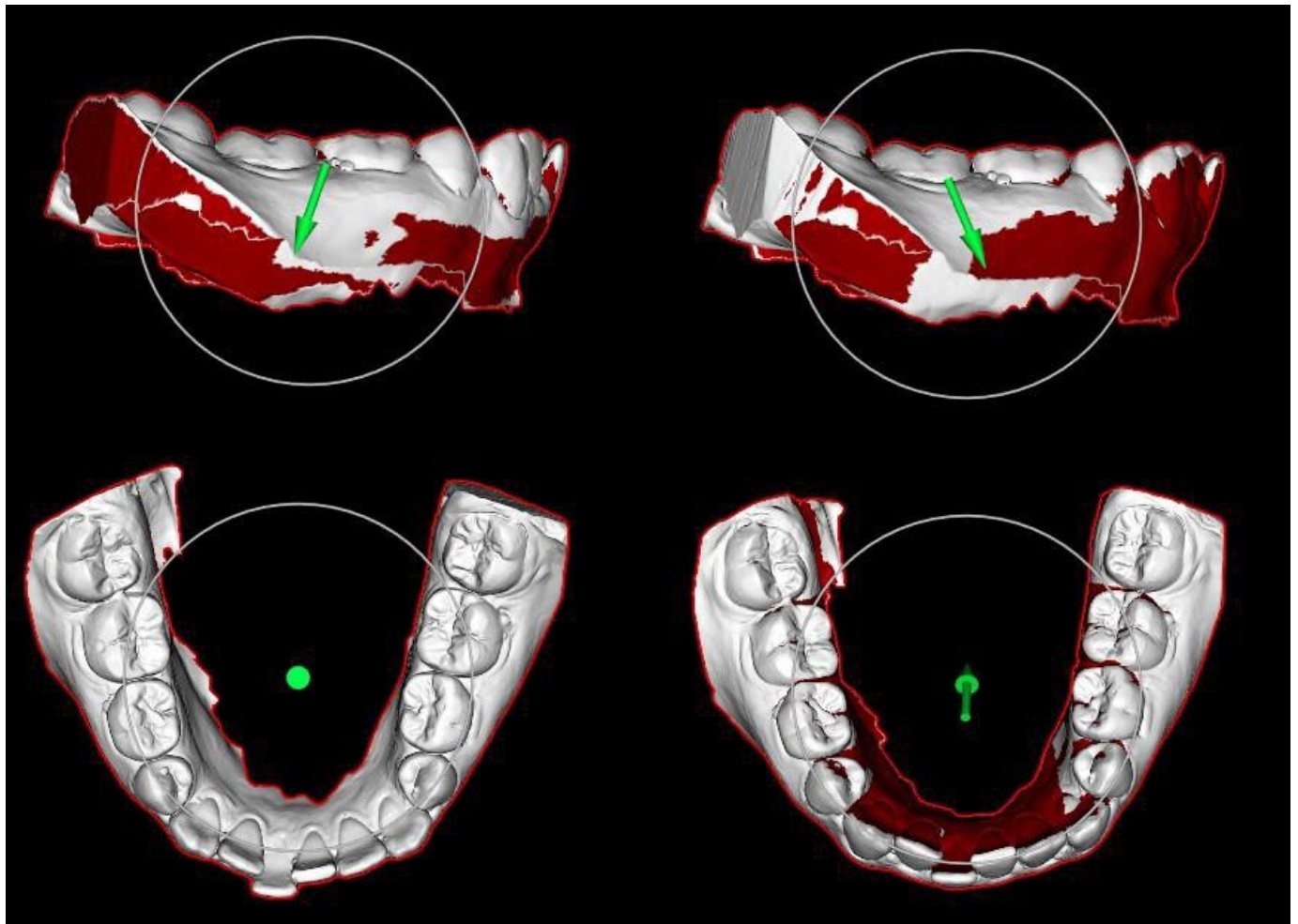


If the insertion direction is set incorrectly; **arrow turns red**, the user will receive the message **Insertion Direction is Invalid** and cannot proceed further. It is necessary to set the correct direction in order to proceed.

28.6 Add Anterior Deprogrammers

The Anterior Deprogrammer is a flat plane occlusal splint with an anterior acrylic block designed to disclude the posterior teeth. Since the body has a protective reflex to prevent added pressure to the anterior teeth, when this splint is worn, less pressure is exerted. Blue Sky Plan anterior deprogrammers are represented by STL shapes that the user can manipulate.

Note that length parameters always measure the "perpendicular" distance. For example, the anterior and posterior length only measure the distance along the Y-axis (green in the application), as is shown in the picture, even though the 3D dragger is visualized to be hugging the side of the deprogrammer. S is the center point of the support (the plane on top of the deprogrammer). The guidance angle is the angle between the sideways parallel projection of the original position of the posterior vertex B, the center of support S and the current position of the posterior vertex B. Note that changing this parameter affects all parts of the deprogrammer except the support itself.

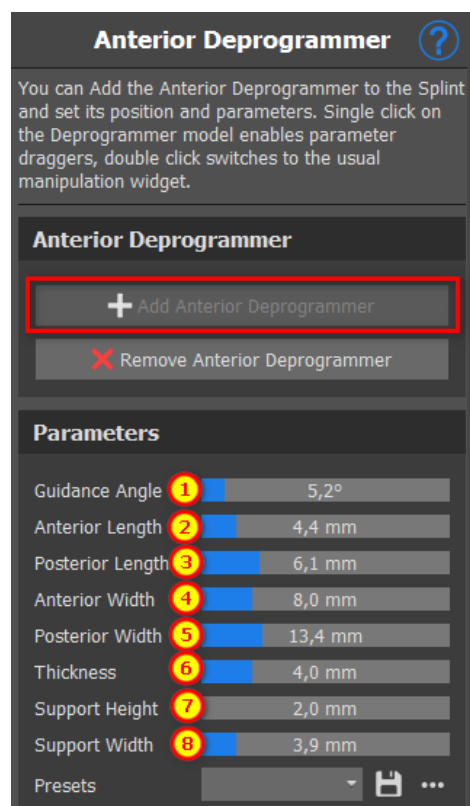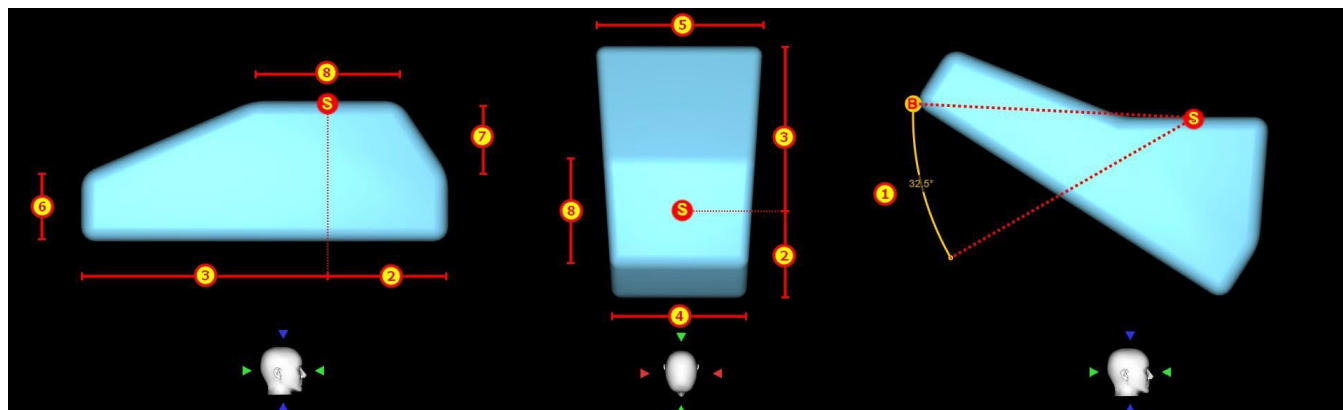


To add a deprogrammer, click on the **Add Anterior Deprogrammer** button. The user is able to choose the anterior deprogrammer shape from an existing list and adjust its location and manipulate its measurements. **Remove Anterior Deprogrammer** the button removes the anterior deprogrammer.

To create a new preset that allows you to save the current parameters of the deprogrammer, click on the **Save** button. A new dialog window appears with the **Enter preset name** option. Save your preset and see it in your presets




To show more options, click on the ... option. Here you can delete your presets as well as export/import presets.

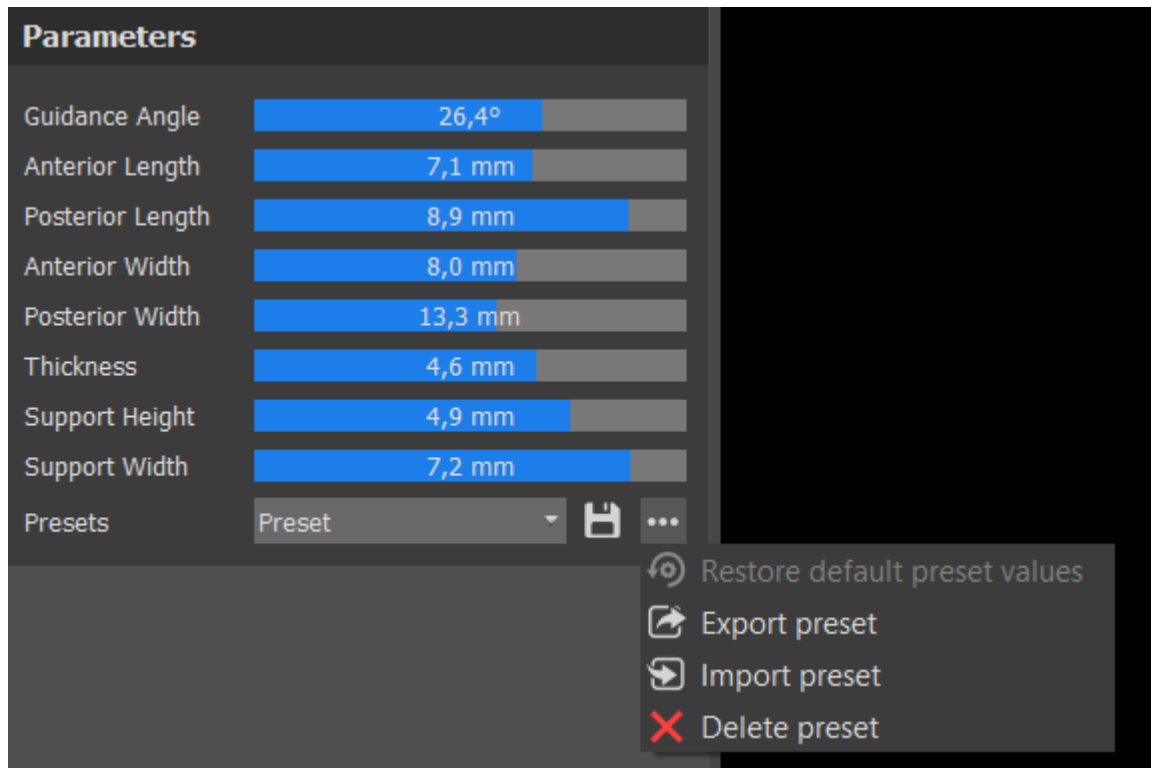

28.7 Draw Splint Curve

Press and hold **Shift** and add a curve point on the selected model by clicking the left mouse button. You can also hold **Shift** and press the **left mouse button** and drag the mouse over the surface of the model. The shape of the curve can be edited using individual control points on it. Before actually generating a splint guide, it is a good idea to check its shape in this mode. If needed, you can delete the curve by clicking the **Clear curve** button and start by drawing the curve again.

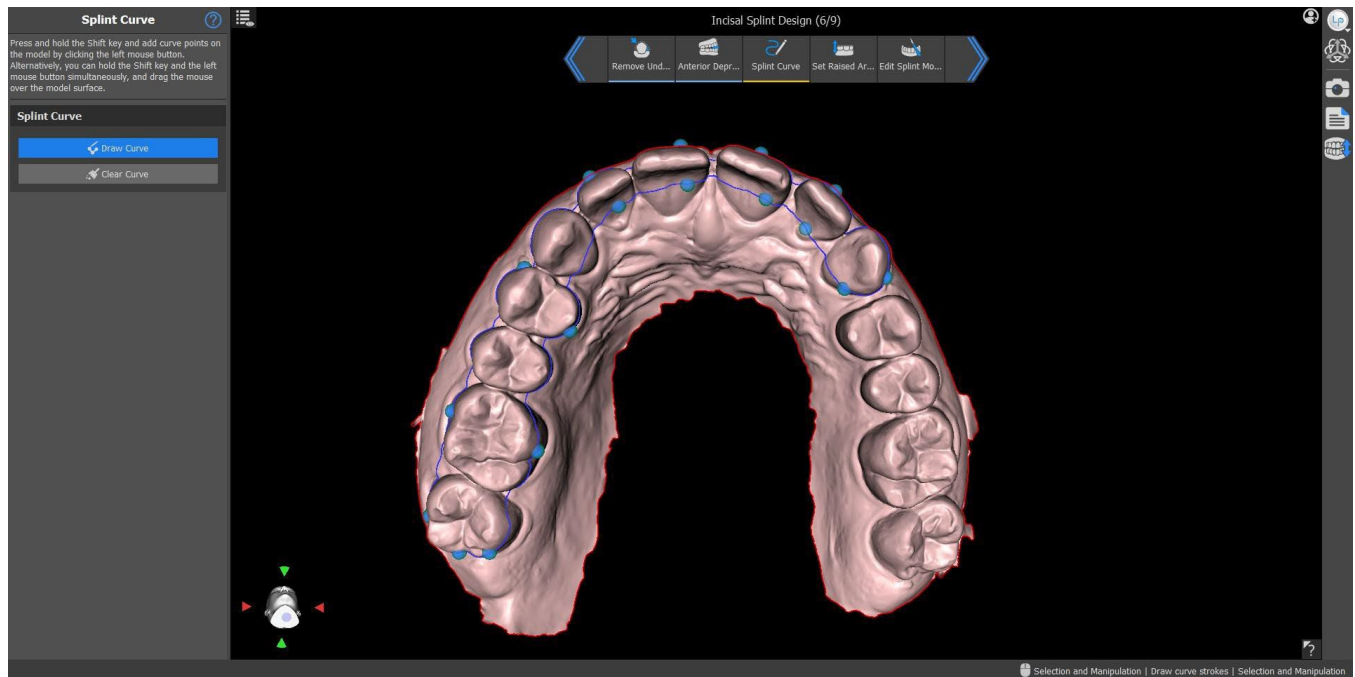

28.8 Set Raised Areas

Raise to Antagonist Cusp Tips ①	Raise to Plane ②	Raise by Ramp ③	Offset by Minimal Thickness ④
Splint Minimal Thickness <input type="text" value="1,50 mm"/>	Splint Minimal Thickness <input type="text" value="1,50 mm"/>	Splint Minimal Thickness <input type="text" value="1,50 mm"/>	Splint Minimal Thickness <input type="text" value="1,50 mm"/>
Splint Offset From the Jaw <input type="text" value="0,00 mm"/>	Splint Offset From the Jaw <input type="text" value="0,00 mm"/>	Splint Offset From the Jaw <input type="text" value="0,00 mm"/>	Splint Offset From the Jaw <input type="text" value="0,00 mm"/>
Follow Cusp Tips Radius <input type="text" value="0,10 mm"/>	<input type="button" value="Reset Plane"/>	Ramp angle <input type="text" value="15,0 °"/>	<input type="button" value="Reset Plane"/>

The **Raise to Antagonist Cusp Tips** (1) method aims to extend the region bounded by the curve to the antagonist jaw. The **Follow cusp tips radius** determines the virtual inflation of the antagonist cusp tips, that will be used for creating the resulting splint shape. A higher value will result in a flatter occlusal part of the splint, while lower values will result in a sharper indentation in the occlusal part of the splint. This value can be set by clicking on the slider.


Raise to Plane (2) raises the region bounded by the curve to the visualized plane.

Raise by Ramp (3) raises the region bounded by the curve to the visualized plane with the possibility to set the angle of the ramp.


Offset by Minimal Thickness (4) this method inflates the region bounded by the curve by the minimal thickness value. The offset and minimal thickness of the splint can be set by the sliders in the **General Splint** Settings section.

The **offset and minimal thickness** of the splint can be set by the sliders in the **Workflow Settings** in the Annotation step or in each **Set Raised Area** method.

To create a splint preview, choose the preferred **Raise Area Method**.

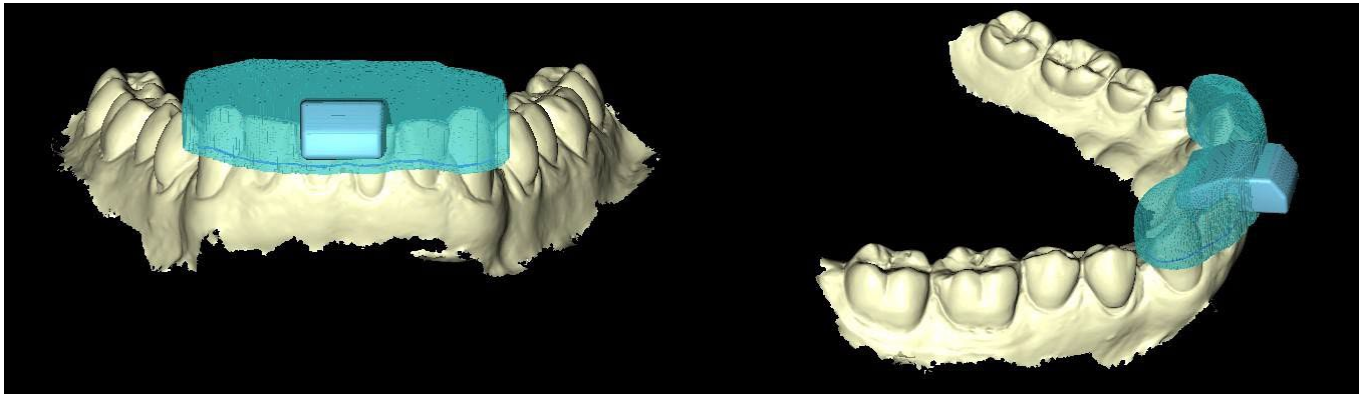

28.9 Edit Splint Model

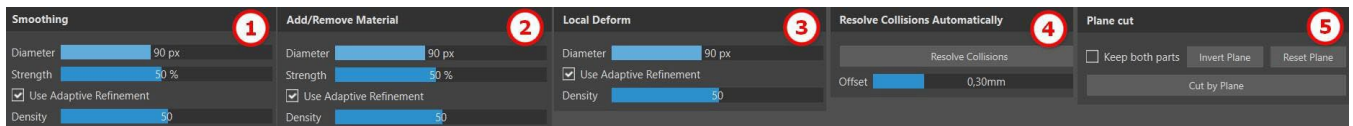


The **Density** slider defines mesh smoothness when applying a brush, if the Adaptive Refinement is on.


Blue Sky Plan software allows the user to edit the models. Click on the brush icon to select the functionality to eliminate problem areas. The **Add/Remove Brush** (2) is used for adding/removing the surface material. The **Local Deform Brush** (3) deforms the surface at a local point. The **Smoothing Brush** (1) can be used for final modifications and polishing the surface. **Resolve Collisions** (4) automatically tries to brush off current collisions. The **Offset** slider sets the offset for the selected models.

Use Adaptive Refinement enables better, smoother mesh creation when using the brush.

Plane Cut (5) allows the user to cut a model by a cutting plane. The plane can be moved and rotated and the cutting direction can be inverted. The plane can be reset to the default horizontal position. Both parts of the cut model can be retained (a new Plane Cut remainder model will be created).

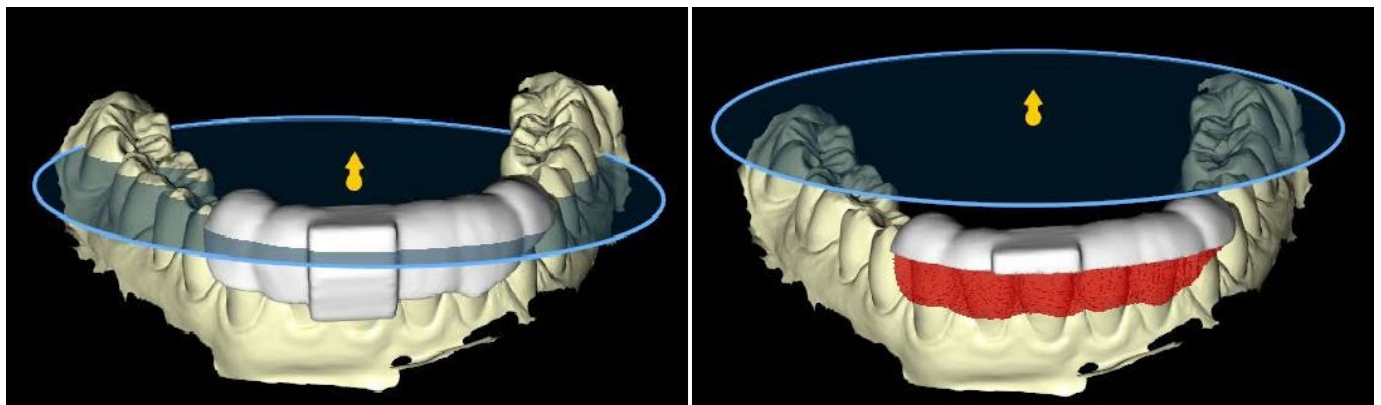

Everything below the plane cut will be removed

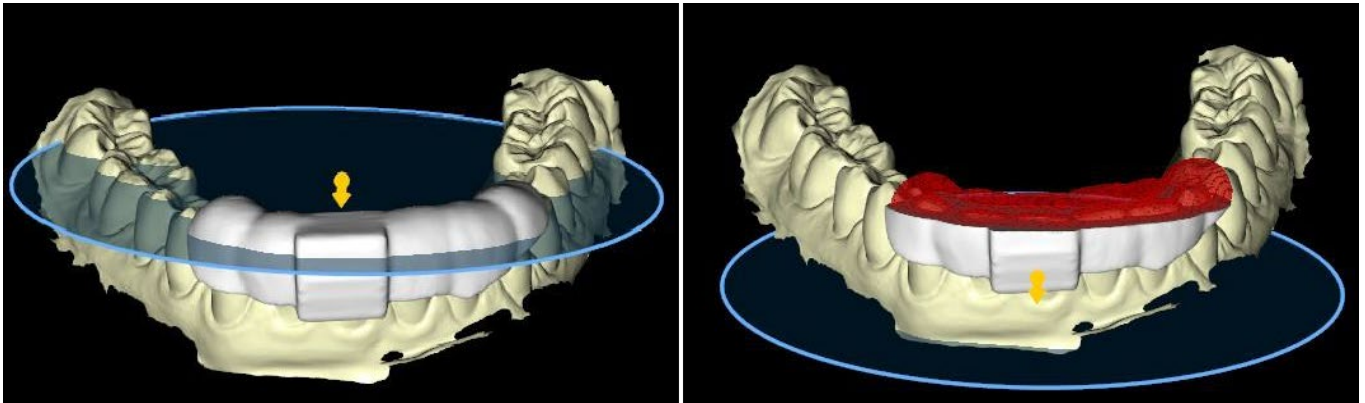

Everything above the plane cut will be removed

28.10 Export and Upload to BioBigBox

The final step, Export, is common for all types of planning. The only difference is in what can be exported in this final step, **OBJ** or **STL** models (3). **Export as Separated Files** (4) exports each selected object to a separate file. Otherwise, objects are exported as a single file. The user can choose which model will be exported (1) and also check the **Export Cost** of the models as well as the credits available on their license (2). After selection, the models will be exported by clicking on the **Export** button (5). Both project (6) and selected models (7) can be uploaded to the **BioBigBox** treatment folder.

Export (9/9) ?

Select models and set parameters for export.

Export Selection

Name	Export
BSP_2017_1030_model_bite	<input type="checkbox"/>
BSP_2017_1030_model_bite2	<input type="checkbox"/>
BSP_2017_1030_model_lower	<input type="checkbox"/>
BSP_2017_1030_model_upper	<input type="checkbox"/>
Splint	<input checked="" type="checkbox"/>

Export Cost:

0.5

Available Credit:

1044.74

Export Parameters

Format

OBJ

☒ Export as Separate Files

5

Export

6

Upload Project

7

Upload Models

28.11 Keyboard Shortcuts

The user is able to select between 2 views - **perspective** and **orthographic**. To select between those 2 views, double click on the **head orientation widget** or **number 5**. It is possible to use keyboard shortcuts **1, 3 and 7** or **Alt 1, 3 and 7** to switch between views.


Keyboard shortcuts **4, 6, 2 and 8** moves the camera 15°.

28.12 Manipulation Shortcuts




Shortcuts allow the user to manipulate the visible models in the 3D view.

Manipulation shortcuts	
Hold R + LMB	Rotate
Hold T + LMB	Translate
H	Hide Dragger

R + Left mouse button rotates the model, **T + Left mouse button** moves the model and **H** hides the dragger.

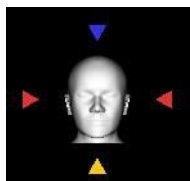

It is possible to orient the model from different perspectives using the orientation widget

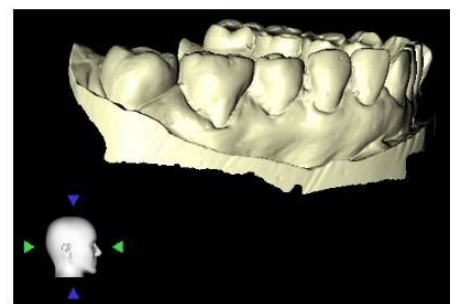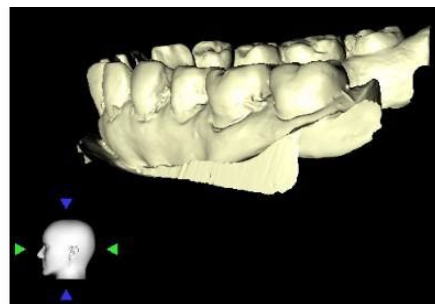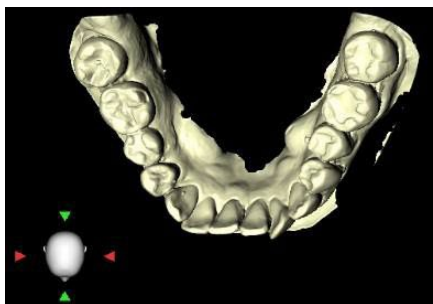



28.13 Visibility Panel

Visibility panel allows the user to work with 3D objects to set:


- visibility 
- opacity 
- renaming 
- deleting 

- changing the color
- collisions 


To rename a model, click on the **Rename** button (1), write a new name (2) and click on the **check** or **X** button (3) to confirm.

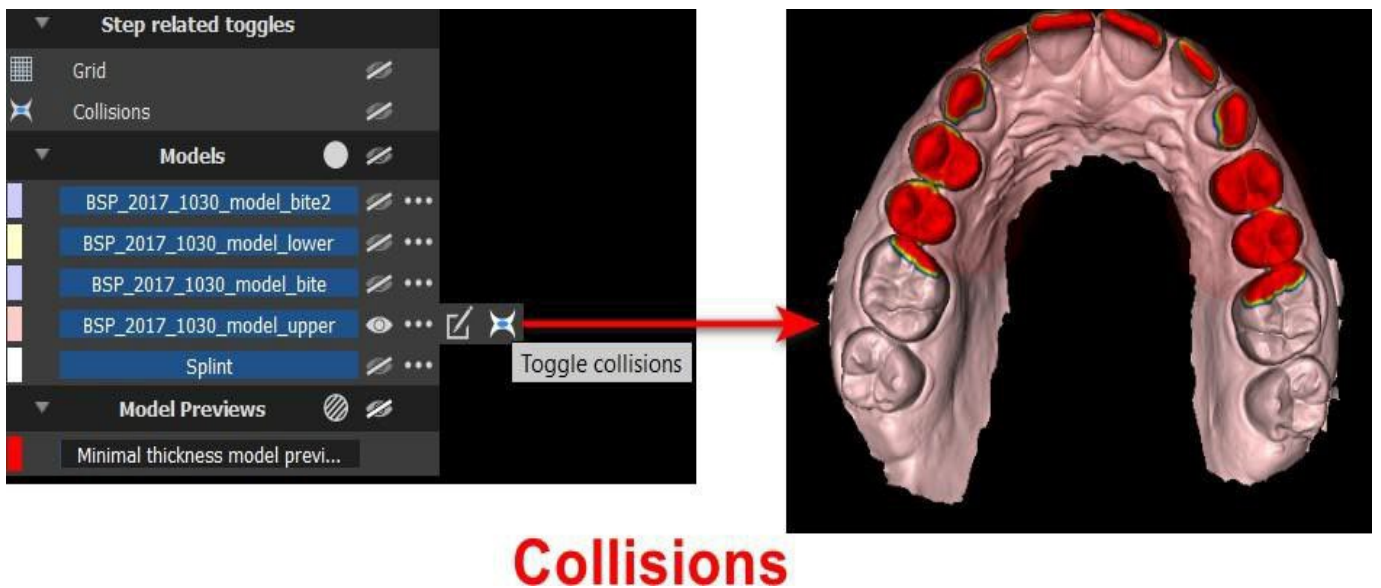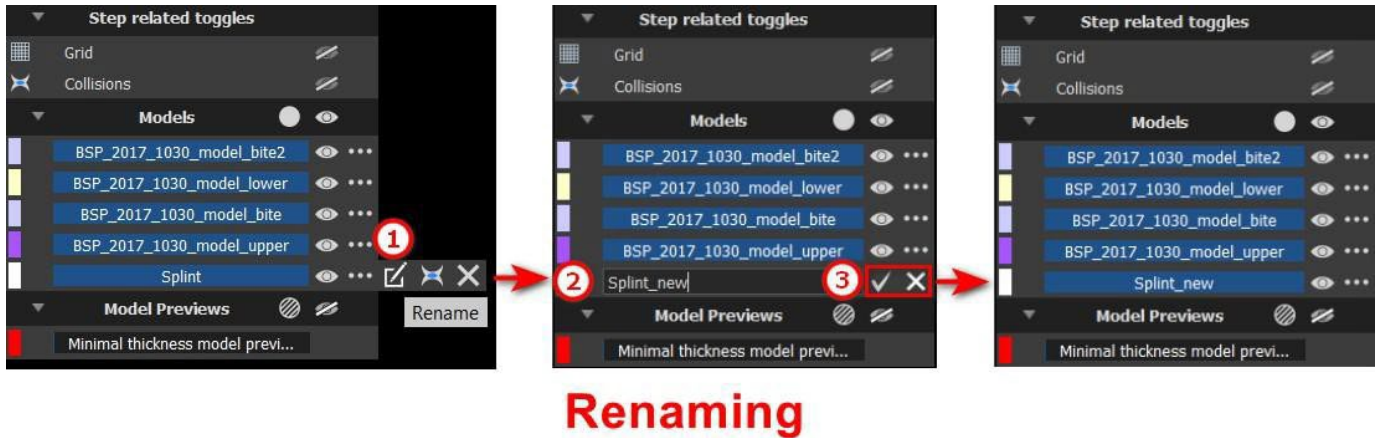

To change the model's color, select the color of the model (1) and choose any color from the "Basic colors" set (2) then click on the **OK** button (3).

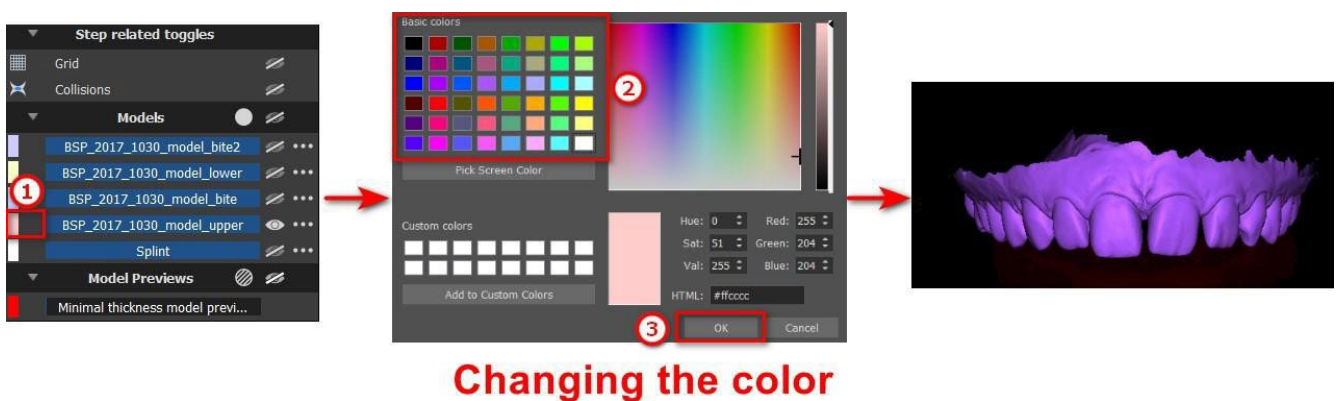

The grid scale indicator is shown in the upper right corner and can be toggled in the Visibility panel. The patient's name can be hidden/shown by clicking the icon shown in the picture.

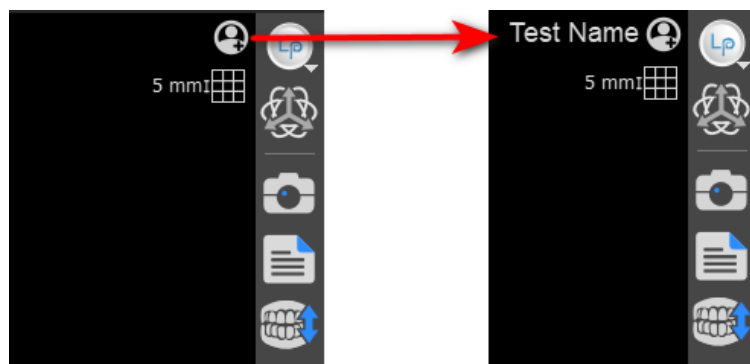


28.14 Editing Mode



The user is able to jump from Wizard to Editing Mode by right clicking **the mouse** -> **Editing Mode** or by clicking the icon in the side panel. Editing Mode is not available in the Splint Curve, Set Raised Areas and Edit Splint Model steps.

Choose one of the supported editing functions from the toolbar on the left (1).


To leave the mode, click on the **Leave Editing Mode** (2) button.

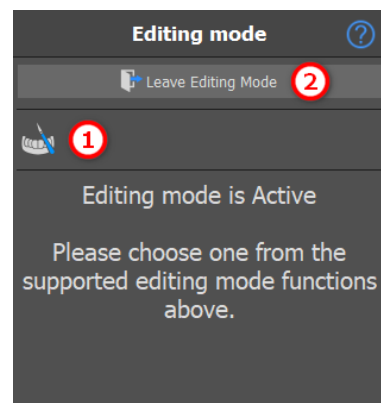

28.15 Edit Model

see section 28.10

28.16 Notes and Measurements

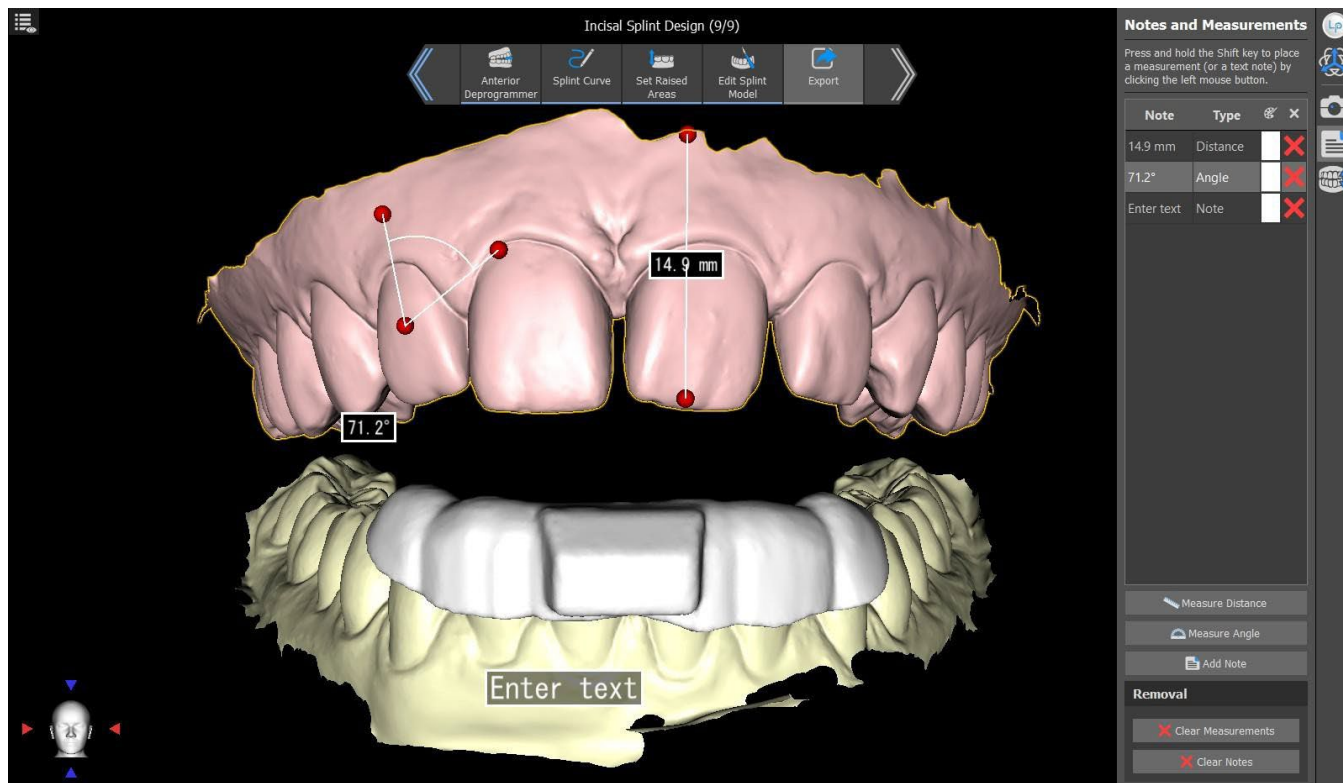


Select the **Measure Distance** button to make any 3D measurements. Click the left mouse button with the pointer on the model to mark the start point of the distance to be measured and click again on the model to mark the end point of the measured distance. To correct the measurement you can grab one of the end points and move them.

Select the **Measure Angle** button to measure angles. Click the left mouse button with the pointer on the model and repeat the process twice to get an angle. To correct the measurement you can grab the points specifying the angle.

Select the **Add Note** button to add a note on the model. Hold Shift and left click to place the text note. By default, Enter text appears, but the user can edit it by double clicking.

The text of an overlay can be changed by clicking on the **Note** field. The color of several types of items can also be changed in the panel by clicking on the **Color** field. To remove the measurements, click on ✖ the button in the **Note** field or by clicking the **Clear Measurements** or **Clear Notes** buttons.

28.17 Create a screenshot

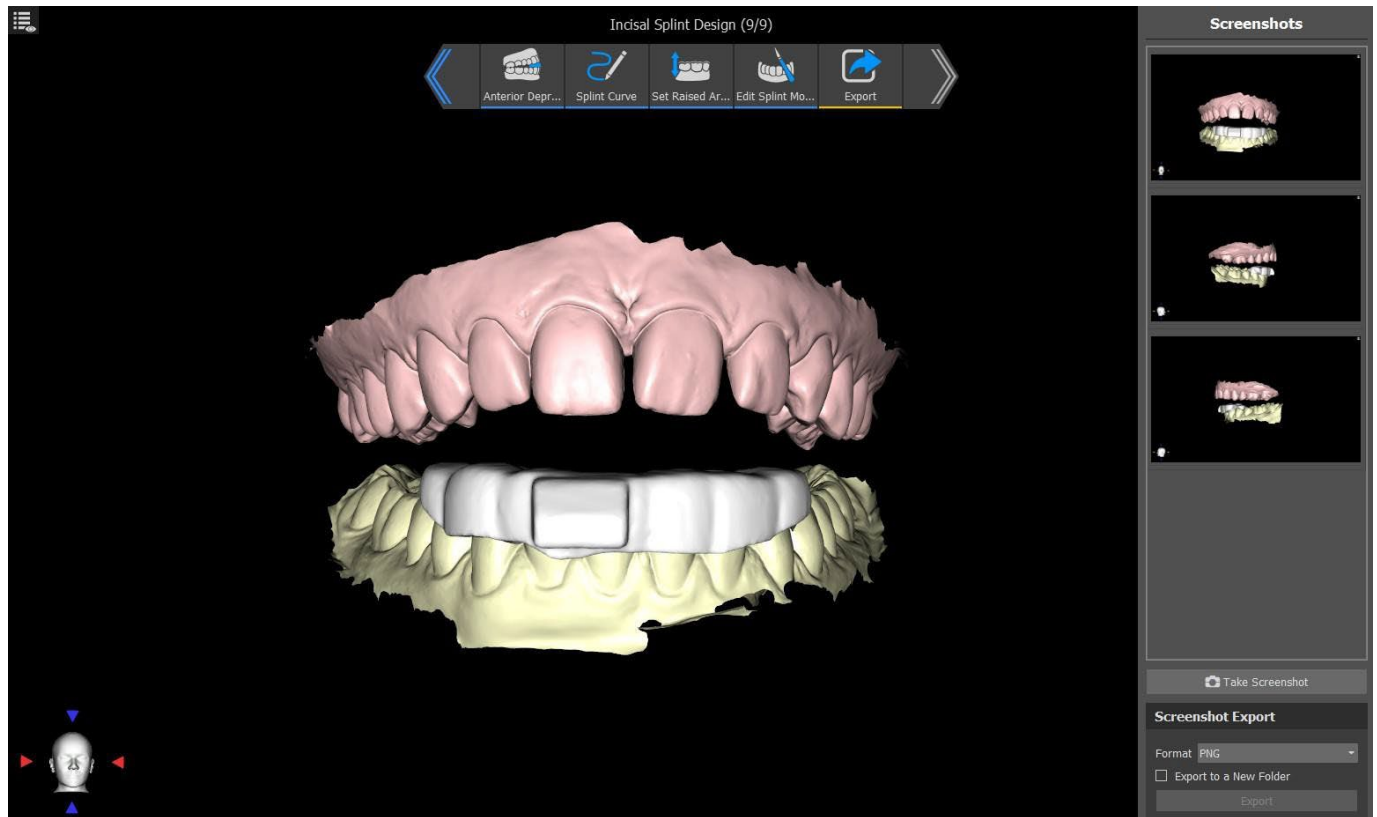


The Screenshot panel can be used for taking a screenshot with the button located in the top right corner of the toolbar. Screenshots are taken by clicking on the **Take Screenshot** button. The user can export screenshots in the corresponding format (PNG or JPG). Screenshots can be deleted using the right mouse button to select **Remove**. Clicking the **Export** button exports the selected screenshots.

28.18 Jaw Movement

See section 28.4


28.19 Show dimensions

In the splint module, it is possible to turn on the dimensions for 3D objects. Right-click on the model and select **Toggle Dimensions** from the context menu. The dimensions are hidden by clicking Toggle Dimensions again.

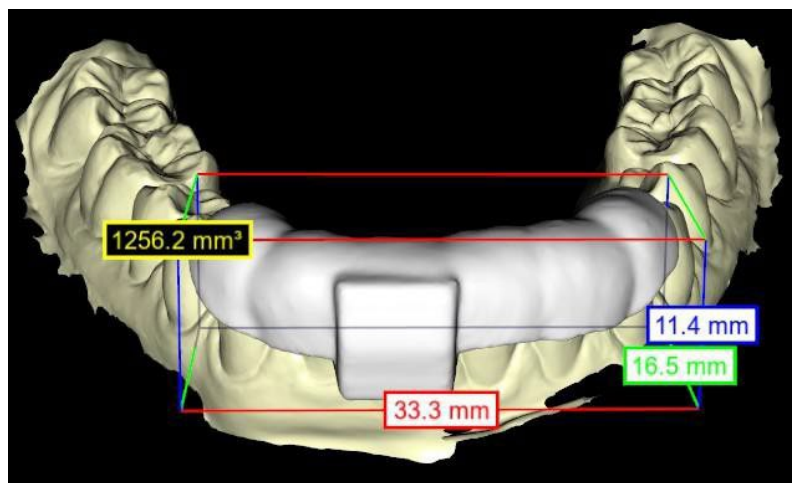

29 Preferences

Blue Sky Plan® makes it possible to save user settings for the program and its environment. You mark the settings selection with the "Tools > Preferences..." menu option.

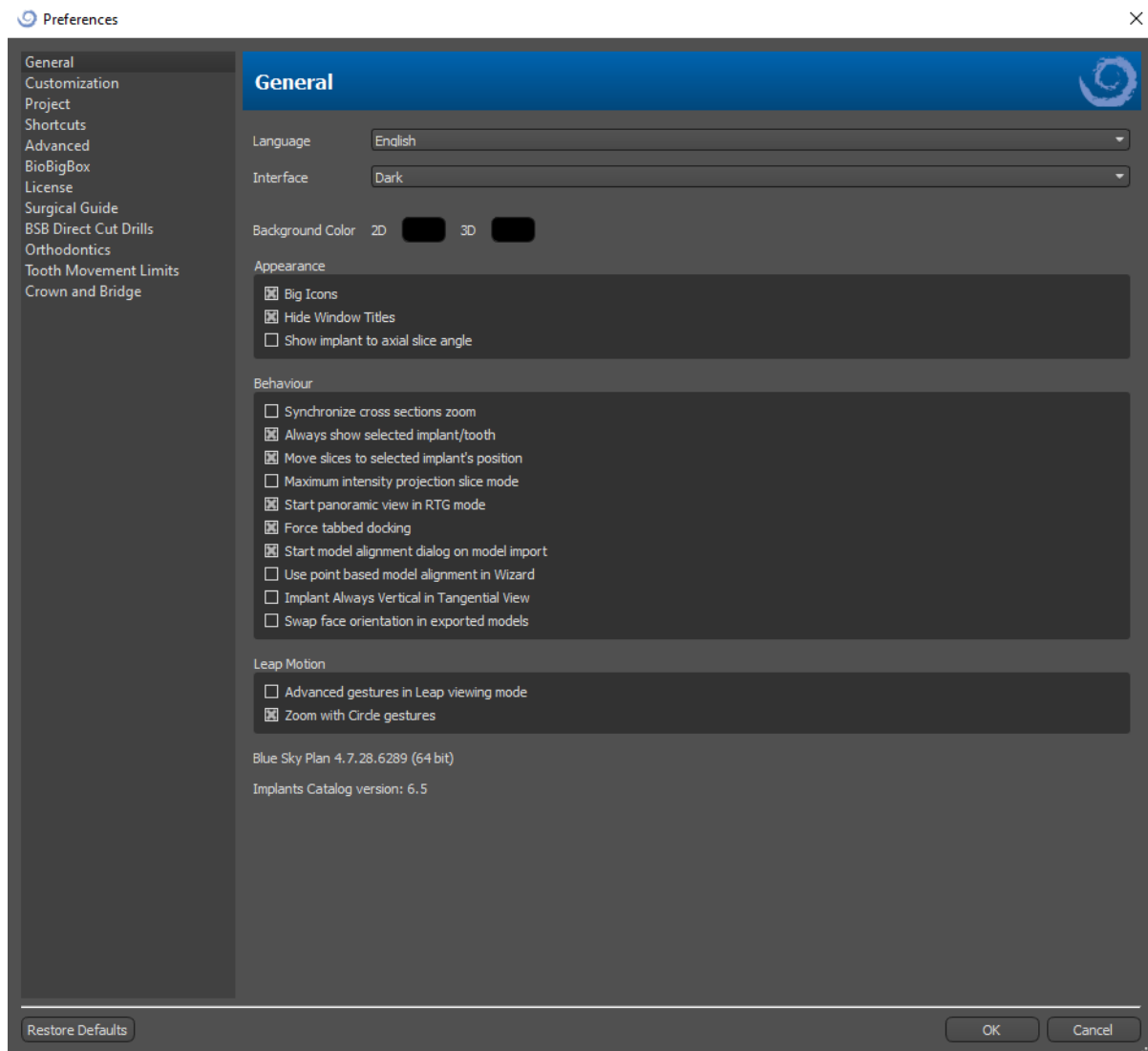

29.1 General

- Big icons – sets the size of the icons in the tool bars
- Hide window titles – sets the visibility of the titles of windows with visualization data
- Show implant to axial slice angle – displays the value of the implant to axial slice angle
- Synchronize cross-sections zoom – if this option is turned on, moving and changing the ruler in any cross-section of the slice also changes the ruler in the other cross-sections of the slice
- Always show selected implant/tooth – ensures visibility of the active implant when implants/teeth are fully transparent
- Move slices to selected implant's position – clicking on the implant sets the positions of the orthogonal and normal slices on the selected implants

- Maximum intensity projection slice mode – this button for switching the slice display mode offers two variants: MIP (maximum intensity projection) and X-ray.
- Start panoramic view in RTG mode – in panoramic window, opened data are viewed in RTG (=composite) mode
- Force tabbed docking – disables the option to cascade panels above each other
- Start model alignment dialog on model import – the process of model alignment will automatically perform, when the model is imported
- Use point based model alignment in Wizard – manual adjustment using matching points and volume in Wizard
- Implant always vertical in tangential view – any rotation of the implant displays the implant vertically in a tangential window
- Swap face orientation in exported models – the exported models have swapped face orientation of the mesh
- Advanced gestures in Leap viewing mode – enables advanced scene manipulation gestures in Leap viewing mode. A Leap controller is required.
- Zoom with circle gestures – toggle the effect of the circle gesture between slider control and zoom control.

29.2 Customization

- Dragger delay – the time until the dragger disappears after the mouse cursor moves away from the implant or tooth
- Model Import Quality – sets the quality of imported model
- Export File Format – enables to choose the default export file format
- Model Collisions Mode – enables to switch between different views of model collisions
- Video Image Delay – when exporting steps plan within Ortodontics module, it is time interval between individual images of the exported video
- Grid 3D (color, size, spacing, thickness) – the properties of 3D grid in the scene
- Axis – the thickness of implant/scanbody axis

29.3 Project

- Autosave project – automatically saves the project at the time interval entered

29.4 Shortcuts

In these sections shortcuts are defined for many different actions.

- Users can add or change the shortcut by clicking onto the window "Keyboard Shortcut" and pushing the keyboard or combinations of keyboards. To confirm the newly defined shortcut it is necessary to click the button "Set".

29.5 Advanced

- Logging – allows information about the execution of the program to be stored in a log. The log can be displayed by clicking the "Show Log" button, and its content can make it easier for the program manufacturer to solve problems that arise when running the program on specific hardware configurations.

- Log OpenGL errors – enables to log OpenGL errors
- Web Engine Settings – choose how much the default web browser uses GPU acceleration
- Module for shell open – enables to choose the default module used to open the data on double click in shell
- Automatically check for updates – every day when it first starts up, the application checks whether you are using the newest release and, if not, offers to install the update for you.
- Show Blue Sky Bio hints and messages – displays the bar containing hints and messages
- Open videos in external browser – allows to open tutorial videos in external browser
- Use system proxy – enables the proxy servers set in the system to be used
- Antialiasing – enables the edge smoothing of the models
- Enable LabPronto service – displays the LabPronto service
- Show warning when recommended Software Guide Tube settings changed – displays the warning message if the parameters of Software Guide Tube are changed
- Show warning for collisions between Software Guide Tubes and model – displays the warning message if Software Guide Tube collides with the model
- Run nerve detection in Surgical Guide automatically – the automatic nerve detection is done for both foramens in Wizard

29.6 Surgical Guide

In this section, the different parts of the surgical guide can be adjusted.

- Software Guide Tube – Distance (ie. distance from implant abutment) and Wall Thickness
- Surgical Guide – Offset (ie. distance between model and surgical guide)
- Windows in Surgical Guide – Width, Height and Depth
- Metal Cylinders – Lip Height and Lip Width (for custom Metal Cylinders)
- GRS Guide (available in “File > Export to > GRS Guide”) – Hole Diameter
- Generated Model – adjusts Base Height, if using the Close or Hollow Model function
- Articulation Pin Arms – Width and Height (see chapter 19)
- Implant to tooth (occlusal side) – when aligning a tooth to implant, Distance can be adjusted
- Undercut – Insertion Path Direction and Maximum Allowed Undercut (see chapter 16)
- Endo mode – see chapter 17

29.7 BSB Direct Cut Drills

These settings apply for BSB Direct Cut Surgical Guide Solution only.

29.8 Orthodontics and Tooth Movement Limits


See chapter 22.

29.9 Crown and Bridge

See chapter 24.

30 Automatic updates

Blue Sky Plan® supports the option of automatically checking the availability of a newer version of the program. It does this once a day when it first starts up; this can be turned off in the program settings dialog in the “Advanced” tab with Automatically check for updates. This feature can be invoked manually with the menu selection “Help > Check For Updates...”

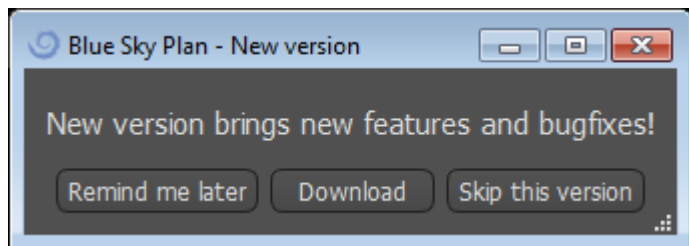

If a new version is available, the program offers the user the options of postponing the update till later, downloading and installing the update immediately, or skipping this update completely. The update download runs directly in the application environment, and to install it, Blue Sky Plan® must shut down and then start the installation, so please close open projects before checking for a new version and installing the update.



31 Footnotes

31.1 Blue Sky Bio Aligners Prescribed Instruction For Use

Caution: US federal law restricts this product for sale by or on the order of a dentist or physician.

Indication for Use

Blue Sky Bio Aligners are a series of clear, lightweight, plastic appliances indicated for the treatment of tooth malocclusions in patients with permanent dentition (i.e., all second molars). Utilising a series of incremental tooth movements, it sequentially positions the teeth by way of continuous gentle force.

Device Description

Blue Sky Bio Aligners are a series of clear plastic aligners that offer a solution for aesthetic orthodontic treatment by utilising a set of removable aligners to correct tooth malocclusions. A dental health professional (e.g. orthodontist or dentist), using a standard personal computer, prescribes the Blue Sky Bio Aligners based on an assessment of the patient's teeth, determines the course of treatment with the system, takes moulds of the patient's teeth and then imports a digital version of the mould (or model) into the ordering software, then in the ordering software aligns the teeth as they want them at different phases of treatment. The phase models are exported from the ordering software and sent to Blue Sky Bio. Blue Sky Bio then prints a series of plastic trays intended to gradually realign the patient's teeth in accordance with the physician's prescription in the ordering software. The prescribing physician creates the models used to print the aligners. Blue Sky Bio produces trays which are formed of clear, thermoformed plastic. The trays are sent back to the dental health professional who then provides them to the patient, confirming their fit and design. Over a period of time, additional trays are provided sequentially to the patient by the dental health professional to gradually move the target teeth to the designed position. The dental health professional monitors the treatment from the moment the first aligner is delivered to when the final aligner is delivered. The trays are held in place by pressure and can be removed by the patient at any time.

Storage and Handling

Store the devices in a dry, clean and dust-free environment in the original packaging at modest temperatures (5°C to 40°C / 41°F to 104°F).

Contents of the Box

The box contains the aligners for the patient identified by the stickers on the plastic bags containing the aligners. These were made from the models you sent to Blue Sky Bio. If at any office visit the aligner fits incorrectly, scan the patient, go through the construction/ordering process again and send Blue Sky Bio a new model for aligner re-fabrication.

The aligners are delivered clean but NON-STERILE in a plastic bag. The plastic bag has a sticker on it which identifies the patient and which number of aligner is in the bag. The patient should be instructed to clean the aligner with toothpaste and a toothbrush each day. The teeth should be brushed before inserting the aligner.

31.2 Computerised Orthodontic Bracket System

Instructions for Use

Caution: US federal law restricts this product for sale by or on the order of a dentist or physician.

Indication for Use

The Blue Sky Bio Computerised Orthodontic Bracket System is a software system intended for use as an aid in orthodontic treatment planning to correct malocclusions in orthodontic patients. For use by dental professionals trained in orthodontic treatment, including radiographic analyses and treatment planning. The Blue Sky Bio Computerised Orthodontic Bracket System is intended for use with commercially available brackets currently used in standard orthodontic treatment. The end product is an indirect bonding tray for use by the dental professional to place multiple brackets at the same time. The additive printing of the indirect bonding trays should only occur at a validated printing centre. Each validated printing centre is to be a registered and listed contract manufacturer with Blue Sky Bio, as documented to the FDA per the Device Establishment Registration and Listing (see CFR Part 807).

Device Description

The device consists of proprietary software that allows the placement of virtual orthodontic brackets on 3D models supplied by the clinician. The physical models are scanned and converted into virtual models in the software. An indirect bonding tray made out of resin is produced. The physical orthodontic brackets are inserted into the indirect bonding tray. The tray aids in the accurate bonding of the brackets, duplicating the position of the virtual brackets in the planning software. The tray is produced using two alternative methods.

One method utilises the virtual model of the patient model with virtual brackets. The indirect bonding tray is designed and printed as a negative of the virtual model with the brackets in place. The printing of the indirect bonding tray should only occur in an FDA registered establishment.

The second method utilises a printed patient model with markers indicating the position of the brackets. The brackets are glued to the model on the marked areas and the indirect bonding tray is pressure thermoformed over the model capturing the brackets.

Both methods result in a resin indirect bonding tray with brackets attached to the tray according to the bracket position planned in the software. The clinician inserts the tray with the brackets on the patient's dentition, allowing the accurate positioning and bonding of the brackets. Commercially available brackets are used as part of the system but are not part of the submission.

The software is available for download on the Blue Sky Bio website and it is used by Blue Sky Bio and authorised users. The download includes the instruction manual, which can also be downloaded separately on the Blue Sky Bio website. The software requires activation for authorised users.

31.2.1 User Identification

The clinician creates the patient models to be scanned into the software. The clinician either creates or reviews and approves the virtual orthodontic treatment plan, including the bracket positions. The software is intended to be used by Blue Sky Bio and authorised entities manufacturing the indirect bonding tray based on the clinician-approved positioning of the brackets.

How to use the Computerised Orthodontic Bracket System

The dental professional creates a model of the patient's dentition. The model is scanned and converted into digital model using any legally marketed scanner for dental applications which will export STL files. Scanning a physical impression to make a virtual patient model is also possible. The model data is imported into an activated copy of the Computerized Orthodontic Bracket System on a standard personal computer.

An authorised user is required to activate the software. The authorised user is either a Blue Sky Bio employee or an authorised purchaser. Authorised users follow the specification requirements guidelines determined by Blue Sky Bio for using their computer and printer with the Computerised Orthodontic Bracket System.

The software has a database of 3D models of commercially available orthodontic brackets. Based on the patient's 3D models and treatment plan the software operator identifies the ideal placement of the brackets. The bracket placement is reviewed and approved by the dental professional. The dental professional can adjust the final position of the bracket if desired.

Manufacturing of the Indirect Bonding Tray (IBT)

Two alternative methods are used to manufacture the indirect bonding trays.

1. The virtual model of the patient's dentition with the virtual brackets in place is utilised to create a negative of the model. This virtual model fits on top of the dentition of the patient and has also cavities matching the shape of the brackets. The user defines the outline of the indirect bonding tray in the software and a virtual model of the tray is created. This model is printed using a new and not recycled legally marketed material for short term use in the mouth called NextDent Ortho IBT utilising a 3D printer satisfying the specifications of Blue Sky Bio described in section 16.11.4. The tray is delivered to the clinician.

2. In an alternative method, a model of the patient's dentition with bracket positioning markers is printed utilising a 3D printer satisfying the specifications of Blue Sky Bio described in section 16.11.1.

The actual brackets are temporarily attached on the markers of the model. A legally marketed thermoplastic resin sheet such as Dentsply Essix Ace is pressure formed on top of the model capturing the patient's dentition and brackets. Any standard thermal pressure forming machine can be used following the guidelines of the manufacturer of the resin sheets. A pressure forming machine is sold by the manufacturer of Essix Ace. The sheet is trimmed to the middle of the brackets. Excessive undercuts are removed and the tray is made suitable to be used as an indirect bonding tray and delivered to the clinician.

Use of the Indirect Bonding Tray (IBT)

IBT's are patient-specific devices intended for short term use in the patient's mouth. The indirect bonding tray is a placement guide to allow a dentist to place orthodontic brackets in a precise manner. Manufacturing the tray using the two alternative methods does not change the way it is used. The key to the tray's clinical use is the fit on the patient's dentition and the location of the spaces holding the brackets during the bonding process. The clinician verifies the accurate fit of the tray on the dentition. In addition, the position of the brackets in the patient's mouth is verified to ensure that the location of spaces for the brackets matches the virtual treatment plan. Once the shape of the tray has been verified, the clinician inserts the brackets into the tray and places a legally marketed bonding agent. The final step is placing the tray back in the patient's mouth to bond the brackets onto the patient's dentition following the instructions of the manufacturer of the bonding agent. If necessary, the clinician can adjust the bracket position removing one or more brackets and using traditional manual bracket placement. The procedure is the same for a printed or thermoformed indirect bonding tray.

31.3 Blue Sky Bio Aligner G - Patient Instruction

Rx only

Federal law restricts this device to sale by or on the order of a physician.

Device Description

Blue Sky Bio Aligner G is a sequence of clear plastic, removable, orthodontic aligners that gradually move the patient's permanent teeth in small increments to correct a misalignment.

Indications for Use

Blue Sky Bio Aligner G is a series of clear, lightweight, plastic appliances indicated for the treatment of tooth malocclusions in patients with permanent dentition (i.e. all second molars). Utilizing a series of incremental tooth movements, it sequentially positions teeth by way of continuous gentle force.

Contraindications

- Do not use if allergic to plastics.

Warnings and Precautions

1. Because the aligners are worn in the mouth, accidental swallowing or aspiration of the appliance may occur and may be harmful.
2. Keep aligners away from hot water and harsh chemical solutions.
 - a) Do not use denture cleaners to clean aligners.
 - b) Do not drink hot or colored liquids with aligners in place – they may become distorted or discolored.
3. Keep aligners out of reach of young children and pets.
4. When not in use, store aligners in the container provided.
5. Aligners worn out of sequence can result in delayed treatment and complications including patient discomfort. Always follow the order of wearing the aligners as specified by a doctor.
6. Sensitivity and tenderness to teeth may occur during treatment, especially when moving forward from one aligner step to the next.
7. Wearing the aligner may temporarily affect speech. Patients typically adapt quickly to the aligners within a week or two.
8. Irritation of the soft tissue in the mouth (gums, cheeks, tongue and lips) may occur. Consult your doctor if this persists.
9. Avoid chewing gum when using the aligners.
10. Do not use sharp objects to remove aligners

Directions for Use

Hygiene

1. Rinse new aligners with cold water prior to first use.
2. Clean aligners prior to each use. Use a soft-bristle toothbrush and a small amount of toothpaste. Do not expose aligners to hot water.
3. Wash hands thoroughly with soap and water before handling aligners.
4. Take care to maintain proper oral hygiene during aligner use and treatment.
 - a) Brush and floss after each meal, prior to inserting aligners.
 - b) Maintain regular dental checkups and cleaning appointments during the aligner treatment period.

Aligner Placement

5. Aligners are to be worn according to the doctor's instructions, usually for 20-22 hours per day, for the number of days recommended by the doctor for each aligner in sequence.
6. Handle only one aligner at a time.
7. Make sure to insert the aligners as marked – for upper or lower use. Either the upper or lower aligner may be inserted first.

8. When inserting the aligner, gently push over the front teeth, then apply pressure, using your fingertips, to the tops of the left and right molars (back teeth) until the aligner snaps into place.

Aligner Removal

9. To remove the aligners, use your fingertip on the inside of the back molar on one side of your mouth to slowly pull the aligner from the molars. Repeat the process on the other side before trying to completely remove the aligner. Once it is disengaged from the molars, gentle prying away from the teeth with the fingertips should allow removal without damaging the aligner.

10. After removal, rinse aligner with water, shaking off excess water, and store in the case provided.

11. If you have attachments in place on your teeth during the aligner treatment, take care not to dislodge them when removing the aligners

31.4 Blue Sky Bio Aligner G - User Instruction

Indications for Use

Blue Sky Bio Aligner G is a series of clear, lightweight, plastic appliances indicated for the treatment of tooth malocclusions in patients with permanent dentition (i.e. all second molars). Utilizing a series of incremental tooth movements, it sequentially positions teeth by way of continuous gentle force.

Contraindications

- Blue Sky Bio Aligner G is contraindicated for patients who are allergic to plastics.

Device Description

Blue Sky Bio Aligner G is a series of clear plastic aligners that offer a solution for aesthetic orthodontic treatment by utilizing a set of removable aligners to correct tooth malocclusions.

Blue Sky Bio Aligner G is 3D printed using Tera Harz Clear, P/N TC-85DAC, photo-curable resin, by Graphy Inc.

The dental software used when planning and manufacturing the aligners is the Blue Sky Plan software module for Blue Sky Bio Aligner G.

A dental clinician (e.g. orthodontist or dentist), using a standard personal computer prescribes the Blue Sky Bio Aligner G based on an assessment of the patient's teeth and determines the course of treatment via the system. The clinician takes molds of the patient's teeth and completes a prescription form and sends the molds to the dental lab, which in turn scans the molds and uploads the .STL files of the molds into the Blue Sky Plan software. This digital file is a series of CAD files (.STL) for building models that can be used to fabricate aligners. Alternatively, the dental clinician may generate the digital files by scanning the patient's mouth directly, using an intraoral scanner, and then sends the files to Blue Sky Bio or its authorized lab.

The dental lab or the clinician designs a series of digital models and plastic trays intended to gradually realign the patient's teeth, this is done using Blue Sky Plan software. The prescribing physician reviews and approves the model scheme before the aligners are produced. Once approved by the clinician, Blue Sky Bio or its authorized dental lab produces the Blue Sky Bio Aligner G sequential aligners, which are made of clear 3D printed Tera Harz Clear resin. The sequential aligners are 3D printed and cured in a post-curing unit. The dental lab then cleans and polishes the Blue Sky Bio Aligner G series to remove rough edges, as needed. They are then delivered to the dental clinician.

Blue Sky Bio Aligner G are sent to the dental clinician who then provides them to the patient, confirming fit and design. Over a period of time, the clinician gives the patient additional trays sequentially to gradually move the target teeth to the designed position. The dental health professional

monitors treatment from the moment the first aligner is delivered to when the final aligner is delivered. The trays are held in place by pressure and can be removed by the patient at any time. Each aligner in the series is worn for approximately 1 to 2 weeks, for 20 to 22 hours per day, (however, they should be removed for eating and for cleaning), after which they are replaced by the next stage of aligners. This is repeated for the duration prescribed by the dental clinician.

Requisite Equipment and Design Software

The following material, equipment and software are required to produce Blue Sky Bio Aligner G:

	Make	Type/Model
Material	Graphy Inc.	Tera Harz Clear resin, P/N TC-85DAC
Design software	Blue Sky Bio, LLC	Blue Sky Plan Software for Blue Sky Bio Aligner G
3D Printer	SprintRay, Inc.	SprintRay Pro 95 printer
UV Light Curing Unit	CureM	U102H UV Cure Oven

Clinician's Instructions for Blue Sky Bio Aligner G

Scanning

The first step is to obtain the dimensions and details of the patient's baseline dentition. The clinician takes molds of the patient's teeth and completes a prescription form and sends the molds to the dental lab, which in turn scans the molds and uploads the .STL files of the molds into the Blue Sky Plan software. This digital file is a series of CAD files (.STL) for building models that can be used to fabricate aligners.

Alternatively, the dental clinician may generate the digital files by scanning the patient's mouth directly, using an intraoral scanner.

Design processing

The second step is to create sequential digital aligners. The dentist then uses the Blue Sky Plan Software module for Blue Sky Bio Aligner G to develop the treatment plan and to produce standard STL 3D files to facilitate the manufacturing of each sequential aligner in the treatment plan. Alternatively, a dental technician, develops the treatment plan and sends it to the licensed dentist for review and approval of the treatment plan prior to 3D printing.

CAUTION – The dentist should only provide the patient with a few sets of aligners at a time. The dentist must monitor the patient's progress periodically prior to delivering subsequent aligners.

➤ For detailed instructions on how to use the Blue Sky Plan software, please refer to **User Manual – Blue Sky Plan Software – Blue Sky Bio Aligner G**.

Instructions for Printing & Post Processing of the Blue Sky Bio Aligner G

Blue Sky Bio Aligner G Printing process

The third step is for making the aligners using a 3D printer. Use Blue Sky Plan Software for Blue Sky Bio Aligner G to plan the treatment by creating a series of sequential aligners that gradually position the teeth into their final desired position. Use a SprintRay Pro 95 3D printer to print the aligners for each treatment step.

- This step requires a digital aligners file: STL format files
- STL File size: less than 500 MB in each file.

Parametric Considerations for Printing Blue Sky Bio Aligner G on SprintRay Pro 95

- **Material Mode:** SPRINTRAY SPLINT Layer
- **Resin Curing Wavelength:** 405 nm LED
- **Layer Thickness:** 100 µm
- **Exposure Time:** 2 Seconds
- **Recommended orientation angle (degree):** 30° from platform/print bed
- **Support function:** Auto
- **Printing Position:** All position Offset: 5 mm
- **Environmental Conditions**
 - o Temperature: 15 ~ 25 °C
 - o Relative humidity: 30 – 75%

Features of Printer Software

- a) STL file import
- b) Automatic rotation and placement
- c) Layer slicer for path inspection
- d) Auto and manual generation of supports

Material Reuse (uncured resin, leftover photopolymers)

- A. The validated and accepted criteria for material reuse is 6 times or less. Discard any uncured resin (leftover resin) after being used 7 times.
- B. Required volume of resin for fabricating aligners up to 7 times for SprintRay Pro95 = 310 g.
- C. Required volume of resin for making one aligner set = 35 g. o An aligner set means an upper (maxillary) aligner and a lower (mandibular) aligner.
- D. Minimum replenishment volume (amount) of resin for SPRINTRAY Pro 95: 100 g.
- E. **Note** – Any uncured resin (leftover resin) in the resin tank should not be poured back into the original resin bottle. Please discard uncured resin (leftover resin).
- F. **Caution** – Do not mix new resin and leftover resin (uncured resin). There is no reaction between the chemical, oxygen and moisture. Therefore, there is no need to monitor the state of the resin.
- G. **Note** – Graphy Inc. only guarantees the quality of aligners that have been printed up to 6 times or less.

Printing Preparation

- A. Shake the Tera Harz Clear vigorously whilst in its original packaging before use.
- B. If different types of resin or alcohol, used during cleaning, remain in the vat, it can weaken the material properties of the printed aligner or lead to printing failure. Always clean the resin tank to remove any impurity after printing.
- C. Be sure to wear protective glasses and protective gloves when handling resins and printed parts.
- D. Open the 3D printer cover and make sure there is no impurity left on the plate or resin tank and pour the Tera Harz Clear into the printer's resin tank up to the required filling line indicated by the

manufacturer. Caution – When filling the resin into the resin tank, nitrile gloves, protective glasses and mask should be used.

E. Close the printer cover.

➤ For detailed instructions on how to prepare the 3D printer, please refer to the OEM User Manual, **SprintRay PRO 95 User Manual**.

3D Printing Process

A. Load digital aligners file (STL) for starting.

B. For printing the aligner model on the build platform, use the printer's software tool to set the optimal position at 30 degrees from the platform.

C. Generate supports on the lingual using the recommended setting by the printer manufacturer. If the support is not enough, add supports on the aligner model.

D. Start printing.

Cleaning Residual Resin from the Printed Aligner

A. Detach the printed aligner model from the build platform.

B. Remove supports from the aligner.

C. Place the aligner on a paper towel and let the aligner stand for 1 hour until the residual resin is removed or cleaned from the aligner. (Turn the inner surface of the aligner to face the bottom to facilitate removal of residual resin.)

D. Check if there is any residual resin remaining inside the aligner. If residual resin still remains, wipe it off with a cotton swab. (Note – If residual resin remains inside, it can cause incongruity.)

➤ Caution – After printing is completed, the resin tank typically contains some leftover resin (uncured resin). This resin should not be reused. Safely dispose of residual resin. Do not mix the new resin with uncured resin (leftover resin).

Post-processing

The fourth step is post-processing. In this step, the aligner becomes fully hardened. A CureM U102H UV Cure Oven is required for post UV curing and the nitrogen process.

Requisite post-processing equipment (UV-light curing & Nitrogen units)

- **Brand Name:** Cure M U102H
- **Curing Chamber:** Required
- **Supply Voltage:** 100-240, 50/60
- **Lamp Power:** 80 W
- **Light Intensity:** 400 mW/cm²
- **Lamp Wavelength:** 395~405 nm
- **Curing Time (min):** 60 minutes – Turn object between the curing time

A. The printed aligner must undergo post UV curing and the nitrogen process using a CureM machine. Do not use any other post curing unit.

B. Place the aligner on the CureM plate.

C. Connect the nitrogen generator tube to the plate's connector.

D. Place the plate inside the curing unit.

E. After that, add nitrogen at 10 L/m for 1 minute.

F. Start the first post-curing for 30 minutes, turn the aligner on the opposite side and cure for an additional 30 minutes under the same conditions, by filling the chamber with nitrogen first.

G. Take out the aligner from the CureM using tongs once post-curing is done.

H. If necessary, polish the aligner with wet sandpaper using the general dental method

➤ For detailed instructions on how to use the CureM U102H, please refer to the OEM User Manual, **CureM U102H User Manual**.

Cleaning and Finishing

After the post-curing process, clean the aligners and finish the fabricating process.

A. Wash the aligner in warm water.

B. Place the aligner on a tissue (paper towel) and let it stand for 30 minutes until it has dried.

C. After the aligner has completely dried, check the aligner's surface and finish the process.

Dental Lab Instructions for Blue Sky Bio Aligner G Labeling and Packaging

Prior to sending aligners to the clinician, place each aligner in a plastic bag, labeled with the following:

- o Patient Name and Identifier number
- o Doctor's Name
- o Dental Lab address and contact information
- o Rx – Prescription Use Only Symbol
- o Aligner Step #
- o Patient Instruction for Use – Blue Sky Bio Aligner G

Notification

The Tera Harz Clear specifications have been validated with the required equipment and the validated manufacturing process and software. Any other equipment and manufacturing process will be beyond the device's specifications.

Warnings

- Tera Harz Clear contains polymerizable monomers which may cause skin irritation (allergic contact dermatitis) or other allergic reactions in susceptible persons. If in contact with skin, wash thoroughly with soap and water. If skin sensitization occurs, discontinue use. If dermatitis or other symptoms persist, seek medical advice.
- Avoid inhalation or ingestion. High vapor concentration can cause headache, irritation of eyes or respiratory system. Direct contact with eyes may cause possible corneal damage. Long-term excessive exposure to the material may cause serious health effects. Monitor air quality in line with OSHA standards.
- Tera Harz Clear should not be used for any other purpose apart from orthodontic appliances. Any deviation from this instruction for use may have a negative effect on the chemical and physical properties of the Tera Harz Clear.

Inhalation

Tera Harz Clear is an irritant for the respiratory system. High atmospheric concentrations can cause irritation of the respiratory system, dizziness, headaches and anesthetic effects.

Skin contact

Tera Harz Clear may cause sensitization if in contact with skin. Irritation to skin, repeated or prolonged contact may cause dermatitis.

Eye contact

A high vapor concentration of Tera Harz Clear may cause irritation.

Ingestion

Tera Harz Clear has low oral toxicity, but ingestion may cause irritation of the gastrointestinal tract.

Protection

Wear protection when handling Tera Harz Clear. Protective glasses and nitrile gloves are advised.

- H317 May cause an allergic skin reaction.
- H413 May cause long lasting harmful effect to aquatic life.

Precautions

- Read all safety and operating instructions beforehand.
- Wear protective glasses and nitrile gloves whenever you handle Tera Harz Clear resin.
- Work in a well-ventilated area.
- Cure any resin you plan to dispose of.

Adverse Reactions

Tera Harz Clear may cause allergic reactions. In case of an allergic reaction, please contact an orthodontist or dentist.

Disposal















- Fully cure any liquid resin you plan to dispose of.
- Tera Harz Clear in its polymerized form is not environmentally harmful. Residual waste material in its liquid state should be delivered to a collection point for waste material.

Storage Conditions, Expiry Date and Transport

Store Tera Harz Clear in its original packaging at room temperature in a dry and dark area, preferably in 15~25 °C. Close the packaging after each use.

The expiry date is on the product label. Do not use Tera Harz Clear past its expiry date.

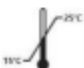Explanation of Symbols

Symbol	Definition	Symbol	Definition
	Batch number of product		Date of manufacture
	Use-by date		Consult instructions for use
	Caution		Non sterile
	Keep away from sunlight		Recycle
	Do not use if package is damaged		Fragile, handle carefully
	Upper and lower temperature limitation		Manufacturer
	Prescription Use Only		Catalog number

Blue Sky Bio, LLC
 800 Liberty Drive, Libertyville, IL 60048
 info@blueskybio.com
 (888) 446-6724