

RE-TREATY

Endodontics Files

Powered by:



Retreatment therapies are increasing compared to initial treatment. That's why Dental Perfect has launched Re-Treaty a specific kit to perform endodontic retreatments in a minimal invasive way, following the Style Italiano Endodontics philosophy. Re-Treaty includes five instruments, three different alloys, two rotary motions and one cross section.

The first step is to remove existing obturation material with the Bull and Skinny files in a crown-down manner. Thereafter, the canal will be shaped respecting the anatomy with the conservative tapered Shape 1-3 files.

BULL

#25/.07

Has been specifically designed to remove the coronal obturation material. The heat treatment of the alloy (gold colored), the tip design, and the shorter length of the file provide the instrument with the right features to remove the obturation material in a short time without any solvent.



SKINNY

#25/.04

This thin non-heat treated file is very effective in removing debris and obturation material in the middle apical third of the root canal system preserving the coronal region of the canal and creating a glide-path close to the apical terminus.



SHAPE-1

#20/.05

It's the first shaping file after patency has been established. The file geometry combined with a special heat treatment that creates a gold-colored file, provides the perfect flexibility and great cutting efficacy that easily follows anatomy of the canal.



SHAPE-2

#25/.05

This is the second shaping files. This file has the same characteristics as Shape-1 for additional removal of obturation material and shaping.



SHAPE-3

#30/.05

This Blue file is more flexible than the gold-colored files to facilitate further apical shape enlargement without alteration of the original anatomy.



www.dental-perfect.com