


Insert the Hex Driver within the 30Ncm torque ratchet




Insert driver into abutment and turn torque ratchet clockwise until torque of 30 Ncm is reached and the head of the ratchet releases

30 Ncm

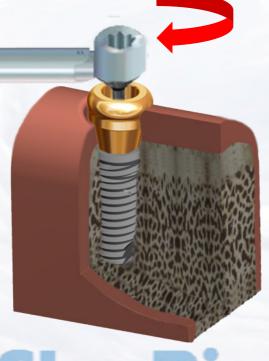




Put silicon blockout ring on the abutment

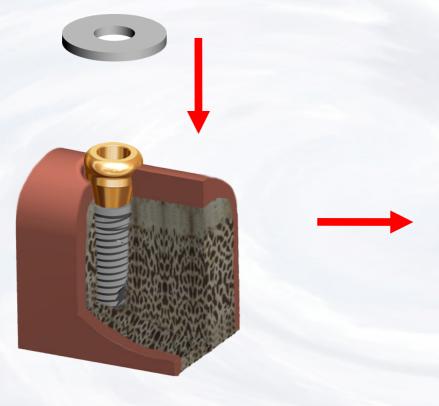





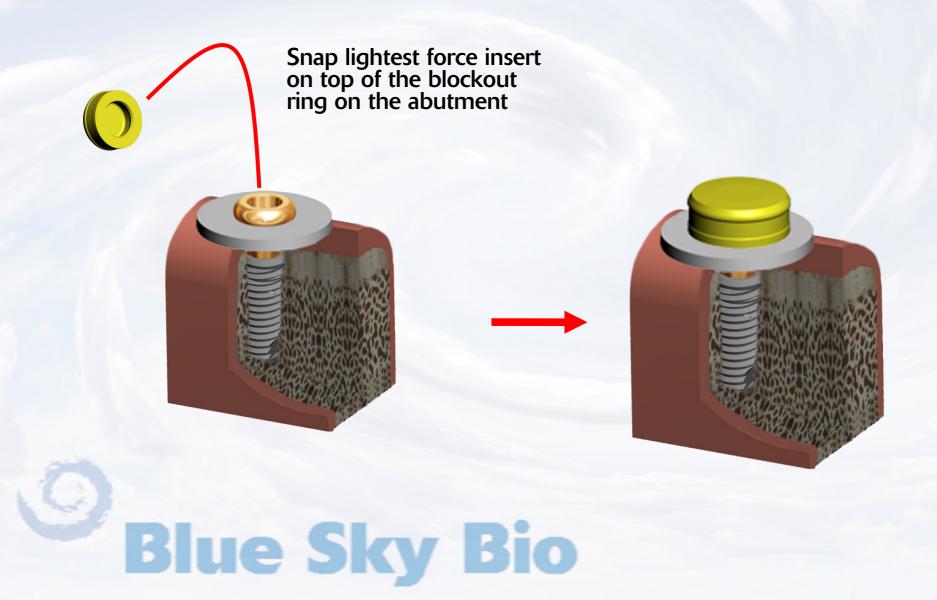



Remove enough material form the tissue side of the denture to ensure that it does not interfere with the housings.

Typically it is necessary to make the cavity in the denture 2-3mm wider and 1-2mm taller than the housing



Insert denture and ensure that when it is fully seated it does not touch the housings





Insert resin of choice into the cavities in the denture




Insert denture again over the housings and with the patient occluding on the denture





Let resin cure with denture in appropriate occlustion

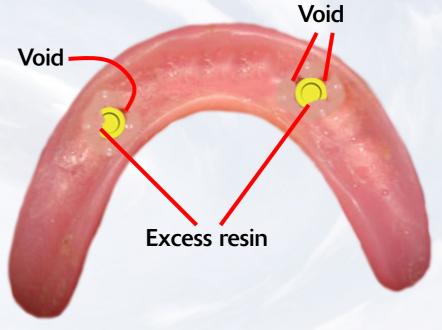

Remove denture from mouth and fill any voids with resin.
Trim any excess resin around the housing and on the ridge area






Finished prosthesis with inserts in place



Overdenture Chairside Technique with Super Snap (Optional)

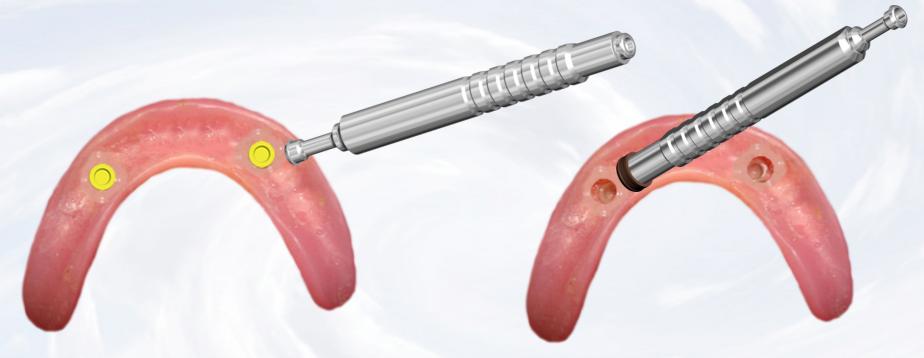

Remove the insert you want to change with the sharp point of the removal instrument

Place the insert with the desired retention into the denture with the opposite side of the removal instrument

