




Sandblast the area of the metal abutment to be covered by whiteMetal. Do not sandblast areas that will not be covered by the final restoration. Apply one coat of liquid whiteMetal bonding agent to the surface that will be coated with whiteMetal Opaquer.

**Blue Sky Bio** 

Gently air dry the coating of liquid whiteMetal bonding agent. After air drying apply a second coat of whiteMetal liquid bonding agent and air dry again.

**Blue Sky Bio** 

Paint a single thin coat of whiteMetal Opaquer on the portion of the abutment that will be covered by the restoration.

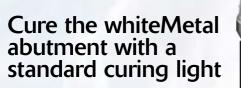

Opaqued abutment on model ready for crown fabrication



**Blue Sky Bio** 

**Blue Sky Bio** 

Fabricate

Fabricate ceramic restoration and use resin bonding material for final cementation