

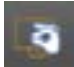


Touchless Blue Sky Plan

HARDWARE CONFIGURATION

- ❖ Connect the Leap Motion sensor to the computer's USB port.
- ❖ Place the sensor on a flat surface in front of the computer.
- ❖ Additional controller information available at LeapMotion.com

OPERATING TOUCHLESS BLUE SKY PLAN

- ❖ Touchless Blue Sky Plan is designed to be used to view an existing treatment plan. Ideal for sterile conditions.
- ❖ Open the relevant Blue Sky Plan Treatment Plan and click on the "Viewing Mode" button in the horizontal toolbar. 
- ❖ The relevant on screen buttons for viewing the Treatment Plan will appear enlarged for easier selection. All functionality used for planning the case will be hidden from the screen.
- ❖ Extend a single finger, and keep the other fingers curled into a fist. Start with your hand close to your body and move it towards the screen until a green circle representing your finger appears on the screen.
- ❖ Control Blue Sky Plan using the following motions.
 - Move your hand in any direction in order to control and move the green circle representing your pointed finger.
 - Extend your thumb (in addition to the single finger) when you want to "click" (green circle will temporarily become yellow) and then immediately return your thumb to your fist.
 - Keep your thumb extended in situations when you would want to hold down you left mouse button (green control circle will remain yellow) for continual selection.
 - Move your extended finger in a fast circular / spinning motion (with the finger still pointing forward) to zoom in and zoom out.
- ❖ If the sensor loses track of the finger, temporarily remove your hand from the sensors field of view and then once again start moving your finger closer to the screen.
- ❖ Press the "Viewing Mode" button again to leave the TOUCHLESS mode and return to normal Blue Sky Plan functionality.

View the Blue Sky Bio website for additional information, to order a Leap controller and to view relevant videos.